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### Operational documentation

Lubricant for intimate use with silicone Contex Silk

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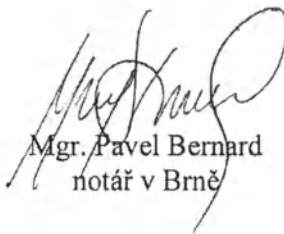
V Brně dne 04.10.2018-----

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Instruction for use <sup>1</sup>  
of medical device «Lubricant for intimate use with silicone Contex Silk», legal manufacturer  
«Altermed Corporation s.r.o»

MEDICAL DEVICE NAME

“Lubricant for intimate use with silicone Contex Silk”

Registration certificate number № \_\_\_\_\_

LEGAL MANUFACTURER

Altermed Corporation s.r.o. Pekarska 2997/36A, 74601 Opava, Czech Republic

Manufacturing site:

Cosmonde a.s. Dusikova 795/7, Brno, Czech Republic

Device description

Lubricant for intimate use with silicone Contex Silk is a medical device with intended purpose to ease the unpleasant and painful sensations during intercourse with insufficient vaginal humectation, which leads for additional pleasure and comfort.

These lubricants are applied to the intimate areas prior to sexual intercourse or as required. The products are suitable for vaginal and anal sex, as indicated on instructions for use.

Lubricant ease the discomfort that may occur when natural lubrication is missing.

Lubricant for intimate use with silicone Contex Silk has long sliding properties to improve the sensation during sexual intercourse with insufficient moistening of the vaginal mucosa (longer-acting compared with «Lubricant for intimate use Contex Wave», based on laboratory report «J20171215-RB Hull – Lubricants»). Produced on water-basis. Does not contain alcohol. Intended for adults. Does not destroy latex. Helps to prevent friction.

<sup>1</sup> The corresponding information included on the bottle of medical device

## Lubricant composition

| Device name                                           | Chemical compound/raw material name            |
|-------------------------------------------------------|------------------------------------------------|
| Lubricant for intimate use with silicone Context Silk | Purified water                                 |
|                                                       | Glycerin                                       |
|                                                       | Cyclomethicone                                 |
|                                                       | Dimethicone                                    |
|                                                       | Cellulose Gum                                  |
|                                                       | Carbomer                                       |
|                                                       | Sodium Hydroxide                               |
|                                                       | Aroma Forest fruit/Vanilla                     |
|                                                       | Propylene Glycol, Methylparaben, Propylparaben |

## MEDICAL DEVICE APPLICATION

Lubricant for intimate use with silicone Context Silk is a medical device with intended purpose to ease the unpleasant and painful sensations during intercourse with insufficient vaginal mucosal humectation

## PRECAUTIONS FOR USE AND POSSIBLE SIDE EFFECTS

Avoid contact with eyes. Do not apply on damaged skin. Do not use in case of container failure. Do not use in cases of hypersensitivity to some of the product ingredients. Use only as indicated. In case of accidental ingestion, seek medical advice. Do not use the product after the expiry date indicated on the bottom of packaging. Keep out of the reach of children. Do not expose to direct sunlight. In case of occurrence of adverse effects please contact immediately doctor.

## Description and application.

To open for the first time: use a twisting motion along the perforation following the direction of the arrows - for 100 ml bottle; open cap – 30 ml pack. Open the bottle or tuba and squeeze out by pressing on the dispenser or the tuba. Apply the gel before and during sexual intercourse on intimate areas, or externally on a fitted condom\* always in amounts as needed. After sexual intercourse, it is desirable to wash off the gel with warm water.

\* Lubricant for intimate use with silicone Context Silk compatible with NRL condoms

## STORAGE AND SHELF LIFE

### Storage

Store at the following temperature 5°C – 25°C, out of reach of children and away from direct sunlight.

### Shelf- life:

1. Lubricant for intimate use with silicone Contex Silk - 3 years

### Labelling on the bottle

| Symbol                                                                              | Interpretation                              |
|-------------------------------------------------------------------------------------|---------------------------------------------|
|    | <b>Name:</b> Batch number                   |
|   | <b>Name:</b> EXP - Use by                   |
|  | <b>Name:</b> Do not litter!                 |
|  | <b>Name:</b> PP (Polypropylene)             |
|  | <b>Name:</b> before using read instructions |

Additional information can be printed on the individual package, such as trade name, importer, authorized representative, advertising information, including reference to international standards

## CONSUMER COMPLAINTS

In the event of consumer complaints regarding the device, contact with authorized representative in Russia Federation «Reckitt Benckiser Healthcare LLC»

Legal address 115114 Moscow, Shluzovaya emb.,4, 3rd floor.

Phone: +7 (495) 961 25 65; fax: +7 (495) 961 25 66.

Telephone Hotline (free of charge in Russia): 8-800-200-82-20

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