Chokhani Pharma

Lalit Chokhani
Managing Partner

INSTRUCTION FOR USE

Condoms Casanova Premium
natural latex not flavored

production: Chokhani Pharma, India

306 Kakad Chambers, 132 Dr. A.B. Road, Worli, Mumbai - 400018, Maharashtra, India

2013

Description

ontoured condom with a ribbed , dotted and smooth texture in a silicone lubricant with the
ore sperm , transparent color ;

Casanova Premium Triple sensation Dotted & Ribbed (Textured) /

"Casanova Premium " Triple feel " point- ribbed (textured)

- Ribbed condom with dotted texture in a silicone lubricant , with the store sperm , transparent
colors .

Terms of use

The condom is a single means of contraception , so each Just use a new condom .

Tear off a strip along one side of the foil and carefully remove the condom .

Condoms are strong enough , but it can be torn by sharp fingernails or jewelry. You should only
wear a condom on the penis is erect and before contact with the body of a partner, it helps to
prevent disease , sexually transmitted diseases , and pregnancy.

Hold the tip of the condom , squeeze two fingers tank condom to force out the air. This will
create space for the sperm . Do this carefully , as the remaining air in the tank can cause the
condom breaking .

If the penis is uncircumcised (a foreskin) , you need to pull the foreskin back before putting the
condom .

Then place it over the head of the penis beaded cuff out. Unroll the condom the other hand ,
spreading it down to the base of the penis, so that it covered the entire penis in a state of
excitement .

If you feel that the condom is very small for you , use a condom increased size. If on the
contrary , put on a condom is kept too loose , use a condom with a locking ring, which does not
allow the condom to slip .

After ejaculation , remove the penis from the vagina , pinning him to the ring of the condom to prevent it from sliding . The condom should be removed only after the complete removal of the penis. Do not touch the penis and vagina condom to prevent any contact with perm.

After using a condom should be immediately discarded. Wrap it in a tissue and throw in the garbage container (bucket) .

DO NOT EMPTY condom down the toilet !

To reduce the risk the integrity of the condom :

- Store in a dry place away from direct sunlight .
- Protect from mechanical damage.
- Do not open the package with his teeth.
- Do not unroll the condom before you put it on the penis.
- If the condom is unwound in easily, by choosing the wrong side of it .
- Condoms have a shelf life , which is always indicated on the package.
- Do not wear a condom in your back pocket is much better to keep it in a shirt pocket or a protected case.
- At the end of the condom leave the place . If the "reservoir" No, just squeeze the top of the condom.
- Use only water-based lubricants , and the rest will damage the latex .
- If you feel that the condom broke , you should immediately stop the sexual act and replace the condom.

GENERAL SPECIFICATION

condoms "Casanova Premium"
Standard: ISO 4074

Test parameters	specification
DIMENSIONS	
Length (mm)	at least 180
Width (mm)	53 ± 2
Thickness (mm)	$0,065 \pm 0,015$
Fracture volume (L) Prior to the expiration date	at least 18.0
BURST PRESSURE (kPa) before the expiration date	Minimum 1,0
Leak test	No leakage
visual inspection	No visible punctures and tears
The content of lubricant (mg)	Минимум 250
Integrity of the package (kPa)	Нет утечки при $(20 \pm 5 \text{ кПа})$ в течение 1 минуты
Packaging and labeling	

Shelf life

Store in a dry place at a temperature below 0 ° C.
Keep container away from direct sunlight.
Keep away from packing heat.
Before use, perform a visual inspection and check the integrity of the packaging.
Shelf life - 5 years from date of manufacture.

The procedure for disposal

Condoms are disposed of with household waste in compliance with local regulations for disposal in the country of use.

REPRESENTATIVE IN RUSSIA

When a defective product claims or discrepancies are accepted
Representation of manufacturer in the Russian Federation:

Address: LLC "Caravaggio", Russia, 129337, Moscow, ul. Yaroslavl highway,
116, page 3, tel. / Fax: 8 (499) 188-25-93, 183-90-10 dob.131; e-mail: gurova_l@bk.ru
TIN 7716664661 7716664661 FREE , 1107746412839 PSRN

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The Signature of Shri Prakash
 is hereby certified
 Secy. Office
 Consulate General
 Government of India
 Mumbai, India

ATTESTED BY MR
S. R. LOKHANDE
 NOTARY
 GOVT. OF INDIA

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