BẢN SAO

Vice General Director
VRG Khai Hoan Joint Stock Company
_____ Mr. Trinh Hoang Nguyen

Binh Duong, date 28 month 7 year 2017.

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Instructions for Use

Trinh Hoàng Nguyễn

**BI-SAFE EXAM LATEX MEDICAL
TEXTURIZED NON-STERILE POWDER-FREE
DISPOSABLE GLOVES**

manufactured by VRG Khai Hoan Joint Stock Company, Vietnam

2017

Table of contents:

1. Purpose	3
2. Area of application	3
3. Indications for use	3
4. Counterindications	3
5. Possible side effects	3
6. Use of gloves and precautions	3
7. Technical characteristics	4
8. Completeness	10
9. Operating information	11
10. Operating conditions	11
11. Marking	11
12. Packing	12
13. Transportation and storage	12
14. Environmental protection requirements	13
15. Manufacturer's warranty	13
16. Recycling information	13
17. Warranty card	14

1. PURPOSE

Bi-Safe exam latex medical texturized powder-free non-sterile disposable gloves (hereinafter referred to as the Gloves) are intended to protect medical personnel from exposure to harmful factors of external occupational environment, to reduce the risk of personnel hands contamination by transient pathogens and their subsequent transmission to patients, to reduce the risk of infection of patients with microbes that are a part of the resident flora of the hands of medical professionals.

2. AREA OF APPLICATION

Bi-Safe latex medical exam texturized powder-free non-sterile disposable gloves are applied in medical organizations and at medical and treatment and prevention facilities.

3. INDICATIONS FOR USE

Bi-Safe exam latex medical texturized powder-free non-sterile disposable gloves shall be used to reduce the risk of contamination of external environment objects by potentially pathogenic microorganisms and their transfer from medical personnel to a patient and vice versa, as well as from one patient to another.

4. COUNTERINDICATIONS

Allergic reactions to the material from which the gloves are made.

5. POSSIBLE SIDE EFFECTS

The Gloves can cause an allergic reaction.

6. USE OF GLOVES AND PRECAUTIONS

Open the package before using gloves. Scrub your hands before taking gloves out of the package and using them as intended.

It is necessary to conduct hygienic antiseptic hands scrubbing, depending on the intended manipulations or procedures.

Put on gloves, avoid making the outside surface dirty.

It is necessary to replace gloves in the following cases:

- If you move to another patient.
- If you found a visible defect.
- If there is or may be a perforation.
- If a glove begins to swell up, stretch, stick, or sits loosely on the arm as a result of liquids or lubricants absorption by the material.
- In accordance with the rules and regulations adopted by the health facility.

Gloves are not treated during manipulations. If necessary, replace them.

Removal of gloves: it is recommended to remove gloves according to the following algorithm: • Take the glove in the heel of the hand area with the fingers of the other hand. • Pull the glove and hold it with the other hand. • Then place the fingers of the free hand in the cuff of the glove on the other hand and pull it like you did with the first one. • Having turned the glove inside out, use it as a bag for the other glove. • Put the gloves to a container for gloves. Then scrub your hands.

7. TECHNICAL CHARACTERISTICS

The Gloves are disposable and texturized.

The Gloves are either colorless (natural color of the latex used) or of the following colors: green, brown, ivory and dark blue according to standard samples or an approved color catalog.

Gloves size and tolerances:

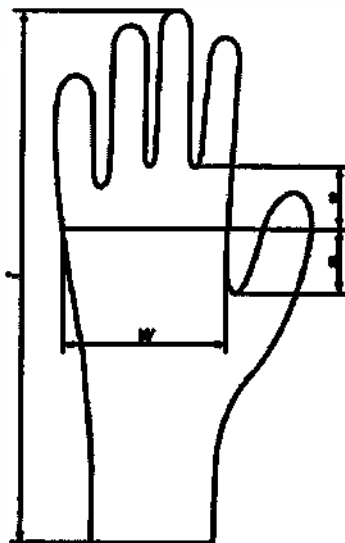
Figure 1 - Points for measuring the width and length of a glove

Table 1.

Size	Color	Cuff	Length, mm	Width, mm	Distance to thickness measurement points, mm (Figure 2)	Thickness, mm	
						texturized surface	Smooth surface
		Cut	238 ± 2	83 ± 5	A = 12 ± 1 B = 30 ± 2	0.15 ± 0.02	0.14±0.02

Natural

C = 44 ± 3

Extra Small (XS)	Natural color, ivory, dark blue, green, brown	Rolled	240 ± 2				
Small (S)	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	89 ± 5	A = 13 ± 1 B = 32 ± 2 C = 47 ± 3	0.15 ± 0.02	0.14±0.02
		Rolled	240 ± 2				
Medium (M)	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	95 ± 5	A = 13 ± 1 B = 33 ± 2 C = 48 ± 3	0.15 ± 0.02	0.14±0.02
		Rolled	240 ± 2				
Large (L)	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	102 ± 6	A = 14 ± 1 B = 34 ± 2 C = 50 ± 3	0.15 ± 0.02	0.14±0.02
		Rolled	240 ± 2				

Extra Large (XL)	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	109 ± 6	A = 15 ± 1 B = 36 ± 2 C = 53 ± 3	0.15 ± 0.02	0.14 ± 0.02
		Rolled	240 ± 2				
Extra Small (XS), reinforced	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	83 ± 5	A = 12 ± 1 B = 30 ± 2 C = 44 ± 3	Min. 0.22	Min. 0.2
		Rolled	240 ± 2				
Small (S), reinforced	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	89 ± 5	A = 13 ± 1 B = 32 ± 2 C = 47 ± 3	Min. 0.22	Min. 0.2
		Rolled	240 ± 2				
Medium (M), reinforced	Natural color, ivory, dark blue,	Cut	238 ± 2	95 ± 5	A = 13 ± 1 B = 33 ± 2 C = 48 ± 3	Min. 0.22	Min. 0.2

	green, brown	Rolled	240 ± 2				
Large (L), reinfor ced	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	102 ± 6	A = 14 ± 1 B = 34 ± 2 C = 50 ± 3	Min. 0.22	Min. 0.2
		Rolled	240 ± 2				
Extra large (XL), reinfor ced	Natural color, ivory, dark blue, green, brown	Cut	238 ± 2	109 ± 6	A = 15 ± 1 B = 36 ± 2 C = 53 ± 3	Min. 0.22	Min. 0.2
		Rolled	240 ± 2				

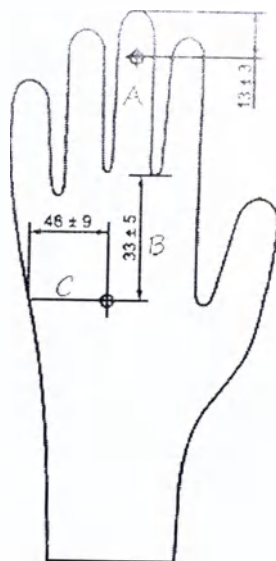
Figure 2 - Points for measuring the thickness of a glove wall

Note - The distance (48 ± 9) mm shows the position of the palm center for gloves of different sizes.

Depending on mechanical strength characteristics, gloves can be classified as Type I.

The gloves must be five-fingered, seamless, with rolled or cut cuffs; their shape must comply with the three-dimensional hand model and be made for the right and left hands.

The surface shall be powder-free.

The texturized pattern shall be applied to the entire surface of gloves fingers and the palm.

The glove cuff can be cut or rolled.

The gloves shall be tight.

No defects exceeding those ones specified in Table 2 can be on the surface of a glove .

Table 2

Defect name	Checked part of a glove	
	Fingers (working surface), webs of fingers, the palm	Rear part, cuff
Foreign impurities	not allowed	allowed on cuffs
Bubbles	not allowed	not allowed when more than 1+ mm
Coagulum inclusions	not allowed	not allowed
Folds	not allowed	allowed
Streaks	not allowed	allowed
Perforations, holes	not allowed	not allowed

Strength characteristics of gloves shall comply with Table 3.

Physical and mechanical characteristics of gloves before and after accelerated aging should comply with the standards specified in Table 4.

Table 3

Feature	Value	
	Gloves with normal strength	Reinforced gloves
Force at break before accelerated aging, N, min.	7.0	9.0
Strech at break before accelerated aging, %, min.	650	650

Feature	Value	
	Gloves with normal strength	Reinforced gloves
Force at break before accelerated aging, N, min.	6.0	8.0
Strech at break before accelerated aging, %, min.	500	500

Table 4

Indicator name	Normal	
	before accelerated aging	after accelerated aging
Nominal tensile strength, MPa, min.	18.0	14.0
Elongation at break, %, min.	650	500

The weight of one glove should correspond to Table 5. Tolerance: +3%.

Table 5

Size	Weight, g
Extra Small (XS)	5
Small (S)	5
Medium (M)	6
Large (L)	7
Extra Large (XL)	7
Extra Small (XS), reinforced	26
Small (S), reinforced	27
Medium (M), reinforced	28
Large (L), reinforced	29
Extra large (XL), reinforced	30

The amount of residual powder substance should not exceed 2.0 mg/dm^2 .

The content of water-soluble protein should not exceed 200 µg/g .

The gloves are made of SVR 3L natural hevea latex.

Water-based dyes by Farben Technique (M) Sdn Bhd, Malaysia shall be used for glove manufacturing: FARSPERSE ivory color, series IVORY, article number PI204; FARSPERSE blue color, series BLUE, article number PL1508; FARSPERSE green color, series GREEN, article number PG707; FARSPERSE brown color, series BROWN, article number PB1408.

8. COMPLETENESS

Extra Small (XS)	0-100 pcs.
Small (S)	0-100 pcs.
Medium (M)	0-100 pcs.
Large (L)	0-100 pcs.
Extra Large (XL)	0-100 pcs.
Ultra-small (XS) reinforced	0-100 pcs.
Small (S) reinforced	0-100 pcs.
Medium (M) reinforced	0-100 pcs.
Large (L) reinforced	0-100 pcs.
Extra large (XL) reinforced	0-100 pcs.
Instruction for use	1 pc.

9. OPERATING INFORMATION.

Every medical professional, knowing the specifics of their work, shall formulate requirements for gloves independently, taking into account the following:

- Actual production risks (assessment of biological risks, assessment of mechanical risks, assessment of chemical risks);
- The state of the skin of the hands of a medical professional, allergological anamnesis of a patient, and a number of other factors.

It is necessary to choose gloves of the right size.

It is necessary to choose gloves depending on the nature of manipulations performed - they should be easily put on and should not impede movements or squeeze the hands.

When putting on gloves, no great effort should be exerted.

After removing the gloves, wash hands with soap and treat them with an alcohol-containing skin antiseptic.

The gloves cannot be reused after performing a manipulation.

Do not use gloves after the expiry date.

After transportation under sub-zero temperatures, the gloves shall be kept at a temperature of $+ (20+5)^{\circ}\text{C}$ for at least 24 hours before using them.

10. OPERATING CONDITIONS

The gloves should be suitable for operation at an ambient temperature of $+10^{\circ}\text{C}$ to $+35^{\circ}\text{C}$ and relative humidity up to 98% at a temperature of 25°C .

11. MARKING

A glove pack shall bear:

- The name or trademark of the manufacturer or supplier.
- The name of the material used.
- Words "TEXTURIZED", "POWDER-FREE".
- Size.
- Lot number.
- Word "DISPOSABLE".
- Word "NON-STERILE".
- Words "EXAM GLOVES".
- Words "DATE OF MANUFACTURE", the year by four digits and the month of manufacture.
- Words "The product is made from natural rubber latex, which can cause an allergic reaction".
- Net weight;
- Guaranteed shelf life;

Transport marking is applied directly to containers or labels made from plywood, metal, paper, or wood fiber board. At the same time, handling instructions applied to containers shall be as follows: "Fragile. Caution", "Keep dry".

12. PACKING IN THE RUSSIAN FEDERATION IN ACCORDANCE WITH THE STANDARDS OF THE RUSSIAN FEDERATION.

Gloves shall be packed in a way that ensures their safety during transportation and storage and prevents contamination.

Packing Instructions - GOST R 52239, GOST R 50444/GOST 20790 and Technical Regulations of the Customs Union TR TC 005/2011 "On Packing Safety" (approved by Decision of the Commission of the Customs Union No. 769 dated 16.08.2011).

Gloves shall be packed in retail packs in accordance with GOST 12302 made from polyethylene according to GOST 16337, GOST 16338.

Gloves in retail packs can be placed to boxes made from corrugated cardboard in accordance with GOST 13514 and fixed with an adhesive tape.

The gross weight of a box shall not exceed 10 kg.

It is allowed to use other types of packs, including those manufactured according to the drawings of the manufacturer of gloves ensuring their safety.

Delivery of gloves shall be accompanied by trade documentation and a packing sheet wrapped in a polyethylene package.

When shipping gloves to regions of the Far North and equated localities, marking and packaging shall be carried out taking into account the instructions of GOST 15846.

13. TRANSPORTATION AND STORAGE IN THE RUSSIAN FEDERATION IN ACCORDANCE WITH THE STANDARDS OF THE RUSSIAN FEDERATION.

Transportation of gloves is carried out by any kind of roofed transport provided they are protected from pollution and mechanical damage in accordance with the shipping rules applicable for this means of transport.

Products shall be loaded and unloaded in accordance with GOST 12.3.009.

The method of loading and unloading shall prevent gloves damaging.

Transportation conditions - according to group 2 of GOST 15150, at temperatures from -50°C to +40°C.

When transporting gloves at temperatures below -30°C, impact loads or deformations are not allowed.

Gloves shall be stored in closed dry warehouses on racks or pallets at temperatures from 0°C to +25°C and relative humidity not exceeding 85%, protected from direct sunlight, in the absence of vapors of salts, acids and other harmful Impurities, at a distance of at least 1m from the heating system and heat-emitting devices.

It is allowed to store gloves at a temperature not exceeding +40°C for no more than 2 months.

It is prohibited to store gloves with oils, gasoline, kerosene and other substances that destroy rubber.

14. ENVIRONMENTAL PROTECTION REQUIREMENTS

The main type of possible hazardous effect on the environment is pollution of air, soil, and water due to:

- Accidental leaks (spills) of production materials;

- Uncontrolled waste disposal on the territory of the manufacturer's plant or outside it

- Improvised waste depositing in areas unintended for this purpose.

Materials used to manufacture gloves shall not pose hazard to human life, health, and environment both during the operation and after expiry of their useful life.

15. MANUFACTURER'S WARRANTY

The guaranteed shelf life for gloves is 3 years from the date of manufacture. Gloves are not intended for secondary use. Do not use after the expiry date. It is guaranteed that the product has no defects related to production and the quality of production materials.

Manufacturer's authorized representative is Limited Liability Company (OOO) Bisans, registered office and official address: 192102, St. Petersburg, Bucharestskaya Str., 1, Building 3, Letter A, Office 508, Tel./Fax: (812) 449-91-61; 449-91-62

16. INFORMATION ON RECYCLING IN THE RUSSIAN FEDERATION

Disposal of gloves shall be carried out according to SanPiN 2.1.7.2790-10.

According to SanPiN 2.1.7.2790, gloves belong to class B - epidemiologically hazardous waste.

All packaging including transport packaging is subject to recycling. Paper, wood, polyethylene and plastic shall be disposed of separately in accordance with the rules for packaging collection, accounting and recycling.

Regarding recycling please contact Limited Liability Company (OOO) Bisans, registered office and official address: 192102, St. Petersburg, Bucharestskaya Str., 1, Building 3, Letter A, Office 508, Tel./Fax: (812) 449-91-61; 449-91-62

17. WARRANTY CARD

Note of sale

Name Trinh Hoàng Nguyễn

Manufacturer

Seller's
stamp

Date of sale

Seller

Seller's signature

The product was inspected, it has no damage.

I have read and agree with the operation rules.

Buyer's signature

Warranty storage period – 3 years.

Regarding recycling please contact Limited Liability Company (OOO) Bisans, registered office and official address: 192102, St. Petersburg, Bucharestskaya Str., 1, Building 3, Letter A, Office 508, Tel./Fax: (812) 449-91-61; 449-91-62

CAUTION! Fill out the warranty card correctly. All lines in the warranty card (except for the producer's/supplier's lines) shall be filled out.

In case the card is filled out inaccurately or incompletely, warranty for the product is made void.

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