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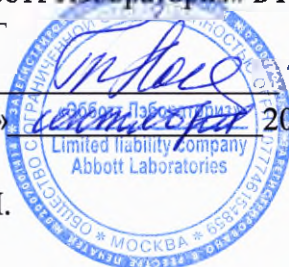
Abbott

Abbott Ireland Diagnostics
Division,
Lisnamuck,
Longford
Co. Longford
Ireland

Tel. +353 43 3331000
Fax +353 43 3331001

I, Aoibheann Gill, Regulatory Affairs Specialist, Abbott Ireland Diagnostics Division, Longford, Ireland, hereby certify that the attached documents that keep the number of pages indicated below are the Instruction for Use. These documents are in a full accordance with actual manuals of manufacturer.

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Signed: Aoibheann Gill

Date: 25th Sept 2018

Aoibheann Gill
Regulatory Affairs Specialist
Abbott Ireland Diagnostics Division
Lisnamuck, Longford
Co Longford
Ireland

ABBOTT DIAGNOSTICS DIVISION LONGFORD

SLIGO CHAMBER OF COMMERCE (INC)
CERTIFIED CORRECT IN ALL GOOD FAITH;
Bereadine Courtenay 26/09/18 Authorised Official

SLIGO CHAMBER OF COMMERCE (INC)
CERTIFIED CORRECT IN ALL GOOD FAITH.

Authorised Official

Alinity i

TSH Controls

 **en**
TSH
REF 07P4810
G72193R02
C7P480

Read Highlighted Changes: Revised December 2017.

NAME

Alinity i TSH Controls (also referred to as TSH Ctrl's)

INTENDED USE

The Alinity i TSH Controls are for the verification of the accuracy and precision of the Alinity i analyzer when used for the quantitative determination of human Thyroid Stimulating Hormone (TSH) in human serum and plasma.

For additional information, refer to the Alinity i TSH reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

CONTROL L, **CONTROL M**, and **CONTROL H** contain TSH (recombinant) prepared in TRIS buffer with protein (bovine) stabilizers. Preservative: sodium azide.

The controls are at the following ranges and target concentrations:

Control	Quantity	TSH		TSH	
		CONC (μ IU/mL)	RANGE (μ IU/mL)	CONC (mIU/L)	RANGE (mIU/L)
CONTROL L	1 x 8.0 mL	0.1	0.065 - 0.135	0.1	0.065 - 0.135
CONTROL M	1 x 8.0 mL	6	3.9 - 8.1	6	3.9 - 8.1
CONTROL H	1 x 8.0 mL	30	19.5 - 40.5	30	19.5 - 40.5

NOTE: The insert ranges for the controls are not lot specific and represent the total range of values which may be generated throughout the life of the product. It is recommended that each laboratory establish its own means and acceptable ranges which should fall within the package insert ranges. Sources of variation that can be expected include:

- Calibration
 - Control lot
 - Reagent lot
 - Calibrator lot
 - Instrument

STANDARDIZATION

The controls are manufactured by addition of Recombinant Human TSH of known concentration to obtain a target concentration. This internally assigned concentration is established using calibration values referenced against WHO TSH 80/558.

PRECAUTIONS

- IVD**
 - For In Vitro Diagnostic Use
 - Rx ONLY**

Safety Precautions

The following warnings and precautions apply to: CONTROL L , CONTROL M and CONTROL H	
Contains sodium azide.	
EUH032	Contact with acids liberates very toxic gas.
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped. Return to refrigerated storage after use.

INSTRUMENT PROCEDURE

- To obtain the recommended volume requirements for the controls, hold the bottle vertically, and dispense 4 drops of the low control, 4 drops of the medium control, and 4 drops of the high control into each sample cup in the assigned position.
- For instructions on ordering and loading controls on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, or if controls do not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

ISO 15223 Symbols	
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	In Vitro Diagnostic Medical Device
	Lot Number
	List Number
Other Symbols	
	Control Number
	Concentration
	Contains Sodium Azide. Contact with acids liberates very toxic gas.
	Control Low, Medium, High (L,M,H)
	Distributed in the USA by
	Information needed for United States of America only
	Product of Ireland
	Range
	For use by or on the order of a physician only (applicable to USA classification only).

Alinity is a trademark of Abbott Laboratories in various jurisdictions.
All other trademarks are property of their respective owners.

 Abbott Ireland
Diagnostics Division
Lisnamuck, Longford
Co. Longford
Ireland
+353-43-3331000

DISTRIBUTED IN THE USA BY

Abbott Laboratories
Abbott Park, IL 60064 USA

**Customer Service: Contact your local representative
or find country-specific contact information on
www.abbottdiagnostics.com**

Revised December 2017.
©2016, 2017 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity i TSH Controls (for Russia)

SHIPPING

Recommended shipping conditions for device is ambient. Device is stable until the expiration date indicated on the package. Ship in an enclosed carrier vehicle of any kind in accordance with shipping rules, applicable to the respective carrier.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. The maximum shelf life of the medical device is 13 months from date of manufacturing. Device must be discarded if stored with deviations in manufacturer's requirements (GOST R 51088-2013).

MAINTENANCE

Not required.

REPAIRS

Not required.

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste".

Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

98/79/EC, 1272/2008/EC (CLP) as amended, 1907/2006/EC (REACH) as amended, 80/181/EEC, 2009/3/EC, Regulation (EC) 1069/2009, Regulation (EC) 142/2011, CLASS 2013 (as amended), GHS (as amended), WHSR Part 7.1 (as amended), EN ISO 9001, ISO 13485, EN ISO 14971, EN 13641, EN 13612, EN ISO 15223-1, EN 980, EN ISO 18113-1, EN ISO 18113-2, EN ISO 18113-3, EN ISO 23640, EN ISO 20776-1, EN 13975, EN ISO 17511, ISO 80000-1, EN ISO 15193, EN ISO 15194

POTENTIAL USER

Professional user

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device cannot be re-applied

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

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