Gebruiksaanweisung

DDBioZ

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
Annex 1

Disc
DD Bio ZW iso (цветной)
for Wieland Zeno* Tec system or other open systems
($\pm$ Disc with a diameter of 98 mm)

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
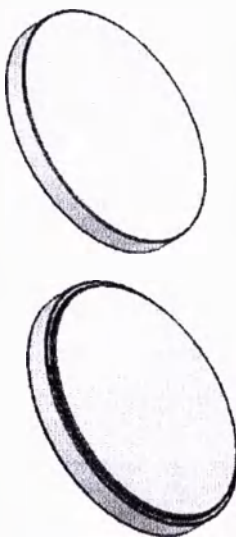
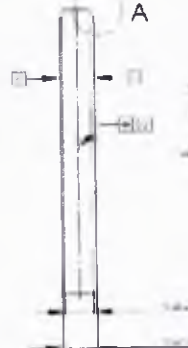
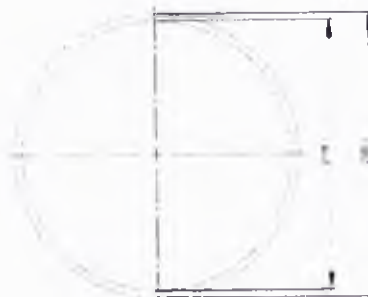
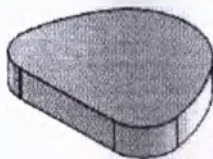
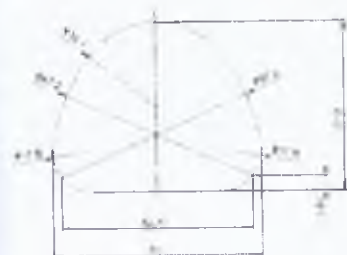
Jaw shaped blank
DD Bio ZA
for Amann Girrbach ceramill* motion system
($\pm$ Jaw shaped blank with a diameter of 71 mm)

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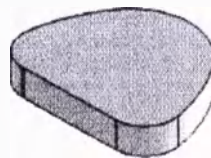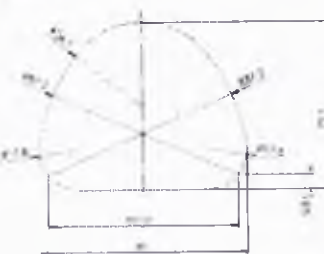DD Bio ZA-blank holder (only for fixation in milling machine, no patient contact):

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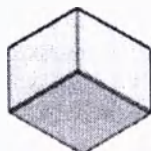
Cuboid blanks

DD Bio ZS
for Sirona inLab* and inLab* MC XL system


Example for DD Bio ZS geometry (further cuboid geometries available):

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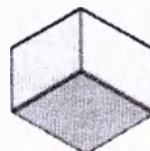Example for DD Bio ZS blank holder (only for fixation in milling machine, no patient contact):

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8. Utilization

Used products can be recycled as secondary raw materials or disposed of as Class A waste (epidemiologically safe waste).

At home, products are disposed of as household waste.

9. Guarantee

The manufacturer guarantees the conformity of the product to regulatory requirements, provided that the rules of operation, transportation and storage established in the present document are observed.

The warranty period of products storage is 120 months from the date of manufacture.

Manufacturer: Dental Direkt Gmb.

(Industriezentrum 106 – 108, D-32139 Spenge, Germany)

+49 5225 863190

info@dentaldirekt.de

10. Authorised representative

An organization authorized to accept claims from consumers:

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

Annex 1

Disc

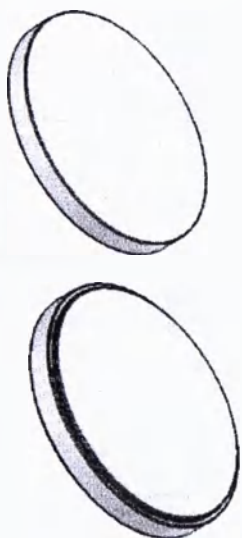
DD Bio ZX² 98 (color)
for Wieland Zeno*Tec system or other open systems
($\pm$ Disc with a diameter of 98 mm)

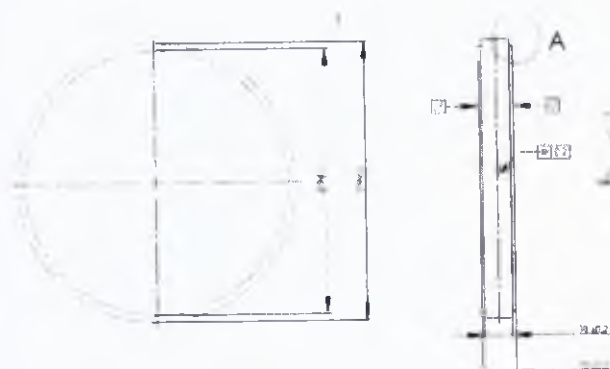Example for disc geometry (98 mm diameter, 10 mm high, with step):

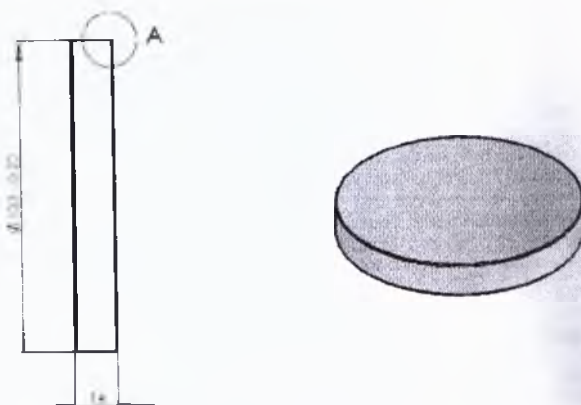
DD Bio ZX² 100
for Digital Dental Mill* system
($\pm$ Disc with a diameter of 100 mm)

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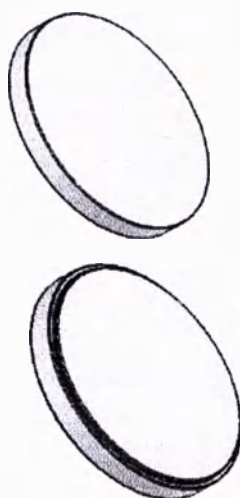
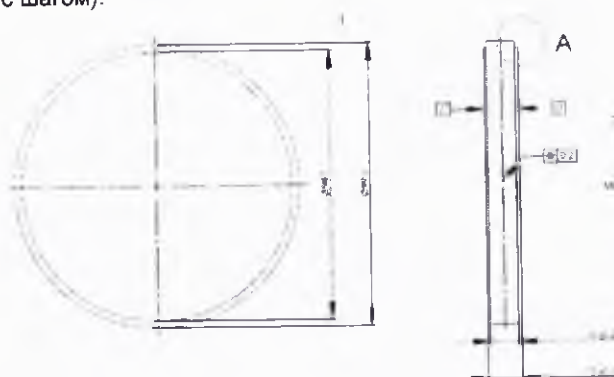
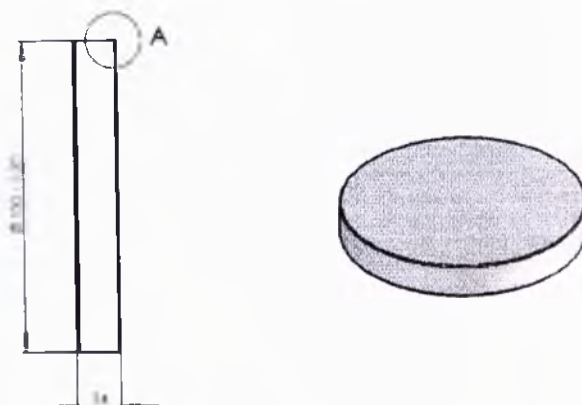
DD Bio ZX² 100-blank holder
(only for fixation in milling machine, no patient contact):

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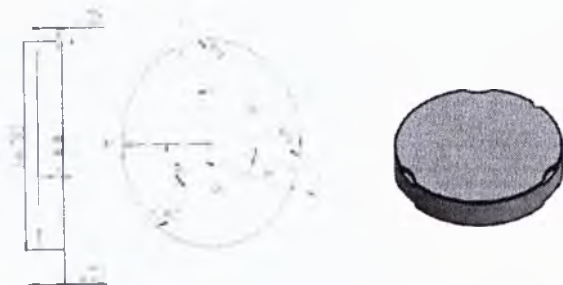DD Bio ZX² 95
for Zirkonzahn* system
(≙ Disc with a diameter of 95 mm)

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

Jaw shaped blank

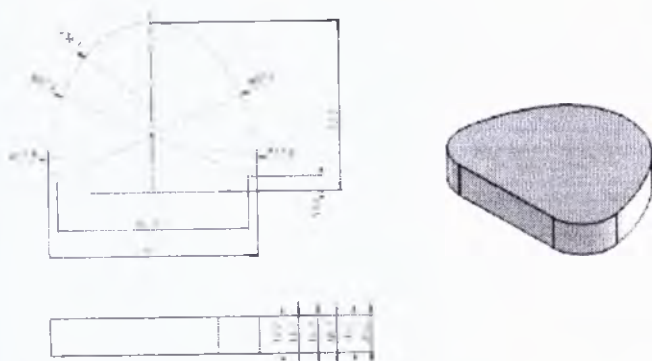DD Bio ZX² 71
for Amann Girrbach ceramill* motion system
(≙ Jaw shaped blank with a diameter of 71 mm)

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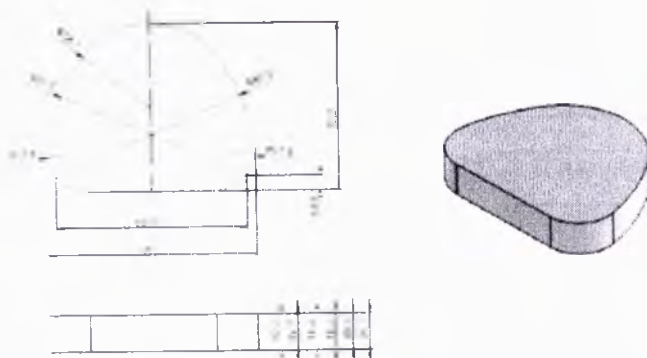Cuboid blanks

DD Bio ZX² 19
for Sirona inLab* and inLab* MC XL system
Example for DD Bio ZX² 19 geometry (further cuboid geometries available):

Example for DD Bio ZX² 19 blank holder (only for fixation in milling machine, no patient contact):

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Utilization

Used products can be recycled as secondary raw materials or disposed of as class A waste (epidemiologically safe waste).
At home, products are disposed of as household waste.

Guarantee

The manufacturer guarantees the conformity of the product to regulatory requirements, provided that the rules of operation, transportation and storage published in the present document are observed.

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info@dentaldirekt.de

11. Authorised representative

Organization authorized to accept claims from consumers:

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
The firing curves listed are recommendations. As measurement of the actual temperature in each furnace can lead to different results, adjustment of individual furnace parameters through trial firing may be necessary in individual cases. We will be more than happy to advise you in this respect.

Dental frames and crowns made of zirconium oxide should be fired in a furnace which is only used for these products. Firing restorations made of another ceramic material or baking of liquid ceramics in the same furnace can lead to impairment of sintering behaviour or local discolouring.

Schematic diagram of sintering processes

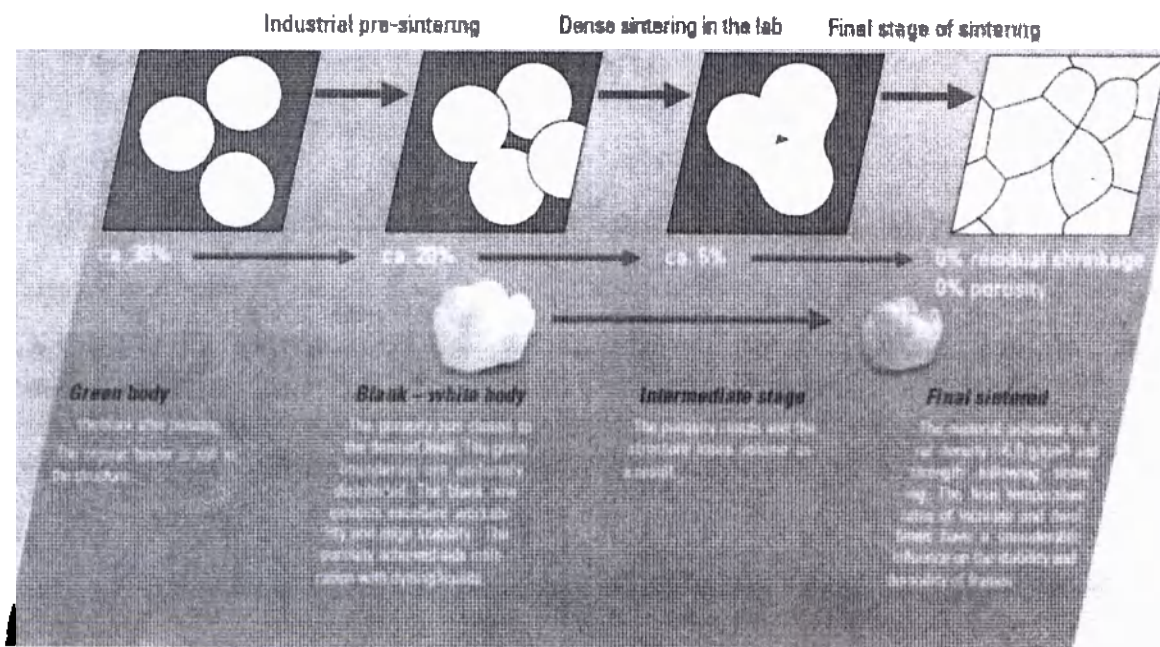26. OKT. 2018

Dental Direkt GmbH
 Industriezentrum 106-108
 D-32139 Spenge
 Tel.: +49(0)5225-8 63 19-0
 Fax: +49(0)5225-8 63 19-99
 www.dentaldirekt.de

G. de Boer
 (CEU)

Please observe the detailed Dental Direkt instruction for use for each product in our [Download-Center](#) on our homepage: www.dentaldirekt.de

CERTIFIED

MED CERT

ISO 13485

L Recommendation prior to sintering

1. „Sinterdrops“

If you have the option to design "drops" to be put on the occlusal surface, this is definitely recommended.

At least four drops must be placed to ensure the support of frameworks.
Single copings do not need any sintering drops.

- Anterior bridges with three units don't need drops. Posterior bridges should always be designed with drops.
- End units should always have drops.
- Pontics and molar copings must have always a drop.
- Premolar and anterior copings can be left out, however at least every second unit should have a drop.
- Drops must be attached alternating in oral and vestibular region. The drops should not be placed directly on the edge of the occlusal surface.

2. Cutting out (when using DD Bio Z, DD Bio ZX² and DD cube ONE)

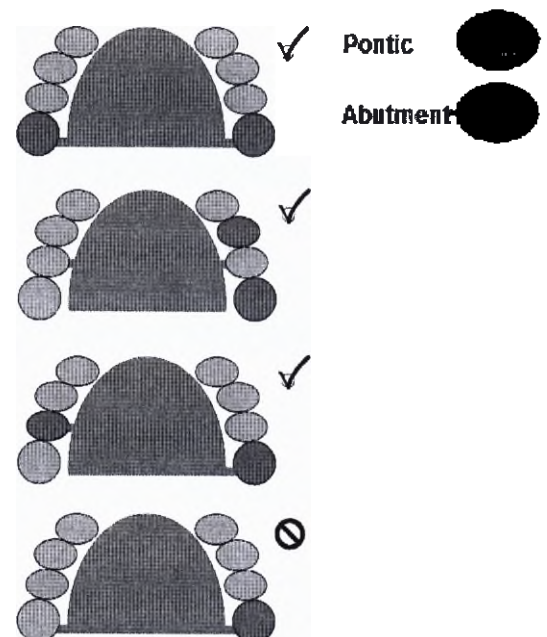
Bridges up to 7 units:

Smaller objects (1 to 7 units) can be completely removed from the blank before sintering. It is sufficient to use a turbine without cooling water, but you should work with low speed and sharp cutting tools. To prevent tension and premature damage of the zirconium construction we recommend to cut every bar half way through. Subsequently cut bars until predetermined breaking point before separating the construction.


Bridges up to 8 units:

To achieve optimum sinter-results - large spanned bridges of 8 and more units must not be completely removed from the blank. With large spanned bridges the amount of bars remaining depends upon the curve of the bridge (arc).

- The bars should only remain attached to matching units. bars should either remain attached to copings or to pontics.
- If possible the bars should remain to the end units. If the end units are not the same, then the Bar should be attached to the next matching unit.
- It is possible to leave two bars attached if the bridge is roughly symmetrical, i. e. the same number of units in each quadrant. These bars remain attached according to the first two rules, to the end units.
Three bars remain attached if the bridge is assymetrical, for example if the bridge spans from 33 to 46. Two of the bars remain attached according to the first two rules, to the end units. The third Bar remains attached to another unit.

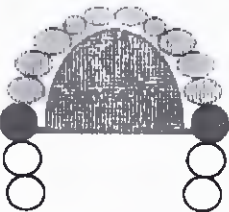If you have a solid piece from the zirconia blank, you should thin this out ideally.

situation	schematic drawing	regulation
12 units, symmetrical		2 bars end position, oral
11 units, roughly symmetrical		2 bars end position, oral
10 units, asymmetrical		2 bars end position, oral 1 bar within the arc, oral Distance from bridge to the blank surface 2 mm
9 units, asymmetrical		1 bar end position, oral 1 bar orally attached to the coping next to the cantilever bridge 1 bar within the arc, oral Distance from bridge to the blank surface 2 mm
8 units, asymmetrical		2 bars end position, oral 1 bar within the arc, oral
unilateral		2 bars end position, oral

3. Drying process when wet milling or grinding has been realised

In case of wet processing, drying process should be realized in a (ceramic) furnace to remove coolant or lubricant from the porous structure.

Please note: User information „DD Bio ZS“ and „DD Bio ZX² 19“

DDBioZS

User information

After milling and before further processing the dental restorations should be cleaned from residue. Therefore you can carefully use water or steam. Afterwards it is necessary to dry the framework, to remove water and milling additives from the porous structure. Drying process is required for an homogenous coloration of the framework by coloring liquid.

 We recommend: Drying of restorations at 700 °C for 5 min.

 Consult instructions for use Rev. 01.2017/03 

DDBioZX²19

User information

After milling and before further processing the dental restorations should be cleaned from residue. Therefore you can carefully use water or steam. Afterwards it is necessary to dry the framework, to remove water and milling additives from the porous structure. Drying process is required for an homogenous coloration of the framework by coloring liquid.

 We recommend: Drying of restorations at 800 °C for 30 min. or 150 °C for 10 min.

 Consult instructions for use 

4. Dyeing with Colouring liquid for zirconium oxide DentalDirekt (DD)

Important:

Do not color wet frames. Only completely dry frames should be colored.

Please adhere to the specified drying times.

Please note: Instruction for use „Colouring liquid for zirconium oxide DentalDirekt (DD)“

5. Inspection of milling work

- No material pitting
- No identifiable cracks
- No impurities
- No surface discolouring
- No glossy patches on the surface

The construction should not be used for the manufacture of dental prostheses if one of the deficiencies listed is detected.

II. Sintering process

1. General information

In addition to use of the correct sintering program, incl. final temperature, dwell time and heating and cooling rate, the quality of sintered frames can depend on:

- the firing capacity and chamber volume of the furnace used.
- the size and volume of individual constructions.
- the mass in the firing chamber (auxiliary sintering media and quantity of constructions).

In order to achieve ideal results, a sintering program should be selected that heats up all the constructions and parts in the furnace uniformly. All bridges require a somewhat longer time to heat up uniformly and for temperature equalisation, due to the differing material thickness of (abutment) crowns and bridge elements. Local differences in sintering caused by heating too rapidly can lead to warping and the formation of cracks. This effect is particularly observed in the case of long-span and solid bridges. Where your furnace is loaded with numerous parts and additional elements such as hoods or similar, the thermal energy may, depending on the furnace type involved, prove inadequate to ensure flawless sintering of all the parts. Solid covers can absorb large quantities of energy which is then no longer available for dental work. In addition the optical characteristics might be enhanced.

Therefore, we recommend to sinter the zirconium constructions without cover!

General rule:

Selecting a slower sintering cycle is favorable to improve the quality of the zirconia structure. Furthermore the translucency can be optimized.

2. Final firing temperatures

All mentioned temperatures are recommendations.

The actually reached temperature varies from sintering furnace to sintering furnace.

An opaque result after sintering can be an indication of a wrong oven temperature.

DDBioZ & DDcubeONE & DDcubeX² → 1450°C

DDBioZX² → 1450°C – 1530°C

Important:

Do not place milled work in the furnace where the maximum residual temperature is greater than 70°C (temperature shock).

Never open the furnace door at a temperature exceeding 200°C.

(possible damage to heating elements and the objects being fired - temperature shock).

Manufacturer:

Dental Direkt GmbH
Industriezentrum 106-108
32139 Spenge, Germany
Tel.: +49 (0)5225 8 63 19 – 0
Fax: +49 (0)5225 8 63 19 – 99
info@dentaldirekt.de

We reserve the right to make changes as a result of the continuous development of our products. Please find the current version of the instruction manual on our website: www.dentaldirekt.de

This version replaces all previous versions.

You will find the current DD instructions for use in our [Download-Center](#) on our homepage: www.dentaldirekt.de

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A. de Boer

As we are continually developing our products, we reserve the right to change them at any time. The latest digital version of these instructions for use can be found on our homepage at www.dentaldirekt.de
This version supersedes all previous versions.

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I) GENERAL INFORMATION

1. Indications

DD Basic Shades are universally applicable liquids for coloring pre-sintered white zirconia constructions made from DD Bio Z, DD Bio ZX², DD cube ONE® or DD cubeX² zirconia milling blanks for the manufacture of dental restorations.

DD Art Elements are universally applicable liquids for coloring and individualizing pre-sintered white zirconia constructions made from DD Bio Z, DD Bio ZX², DD cube ONE or DD cubeX² zirconia milling blanks for the manufacture of dental restorations.

DD Pro Shade Z are liquids especially for coloring pre-sintered white zirconia constructions made from DD Bio Z, DD Bio ZX² or DD cube ONE zirconia milling blanks for the manufacture of dental restorations.

DD Pro Shade C are liquids especially for coloring pre-sintered white zirconia constructions made from DD cubeX² zirconia milling blanks for the manufacture of dental restorations.

2. Contraindications

Do not use if there is an intolerance to the constituents that are contained in these products. Please always observe the contraindications before using any Dental Direkt zirconia products.

The coloring liquids are not compatible with zirconia blanks produced by other manufacturers.

3. Safety instructions

Please observe the information contained in the most recent version of the safety data sheet.

Please wear gloves, safety glasses and a face mask to avoid any irritations. Please do not eat, drink and smoke when handling the liquid.

The materials may only be processed by trained personnel who must use tried and tested techniques.

4. Handling and storage

Ensure that the bottle is still intact before processing the liquid for the first time. The liquids must be stored in the original container, which must be kept tightly closed at a max. temperature of **25°C**, or ideally in a refrigerator (6-8°C). After using the DD Color Box, the liquid must be poured back into the original container for storage.

To avoid contamination, liquid that has already been used should not be mixed with unused liquid. Failure to comply with this instruction may result in color changes.

If stored correctly in its original container, the shelf life of this product is 2 years.

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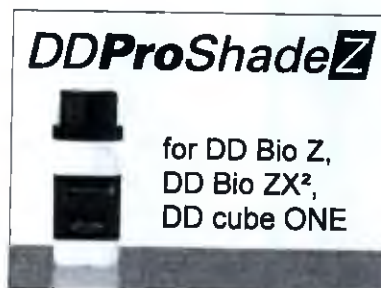
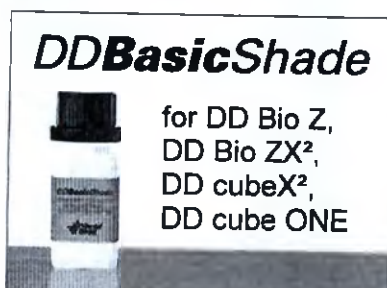
II) COLORING LIQUIDS - COLOR VARIANTS

1. Dentine liquids: DD Basic Shade, DD Pro Shade Z and DD Pro Shade C

The coloring liquids available from Dental Direkt offer a wide range of color options and deliver reproducible and highly aesthetic results. You can choose which system suits your individual requirements the most.

There are 16 dentine liquid shades and 3 bleach shades available for base colorations. The incisal liquids from the "DD Art Elements" range can be used with all three types of liquids. The same applies to the effect colors from the "DD Art Elements" range, which can be used to individualise the colorations (see the following page).

The color combination table below is intended to serve as a guide for the selection of dentine and incisal liquids in accordance with the chosen coloration technique.

Color combination table for dentine and incisal liquids:

Target color in accordance with VITA® shade guide	Dentine liquid	Incisal liquid	
	DDBasicShade DDProShadeC DDProShadeZ	DDArtElements	
		Immersion	Painted
0M1	0M1	SO	SO
0M2	0M2	SO	SO
0M3	0M3	SO	SA1
A1	A1	SO	SA1
A2	A2	SO	SA1
A3	A3	SO	SA1
A3,5	A3,5	SO	SA2
A4	A4	SO	SA2
B1	B1	SO	SA1
B2	B2	SO	SA1
B3	B3	SO	SA2
B4	B4	SO	SA2
C1	C1	SO	SC1
C2	C2	SO	SC1
C3	C3	SO	SC2
C4	C4	SO	SC2
D2	D2	SO	SC1
D3	D3	SO	SC2
D4	D4	SO	SC2

*VITA® is the manufacturer's registered trademark

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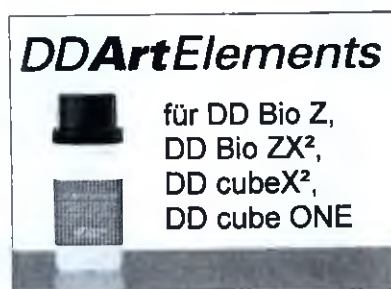
2. Incisal liquids and effect colors: DD Art Elements

Incisal liquids

There are 5 different incisal liquids available for occlusal and incisal individualisation:

SA1	(reddish / brown shade)	:	for the painting technique (0M3, A1, A2, B1, B2)
SA2	(reddish / brown shade)	:	for the painting technique (A3, A3.5, A4, B3, B4)
SC1	(grey shade)	:	for the painting technique (C1, C2, D2)
SC2	(grey shade)	:	for the painting technique (C3, C4, D3, D4)
SO	(neutral shade)	:	for the painting technique (0M1, 0M2) for the immersion technique (all dentine liquids)

Effect colors

There are ten highly concentrated effect colors available that can be used to tint the dentine liquids or to neatly characterise fissures, incisal areas or gingival areas.

WARNING!

If you intend to mix the DD Art Elements effect colors with the dentine liquids, you should always add the specified amount on the full packaged unit (30 ml or 100 ml) to the dentine liquid. This will ensure that you are able to achieve color results that can be easily replicated.

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Vario: Brightening liquid

The color intensity of the liquid can be easily reduced to suit individual requirements by mixing it with DD Art Elements Vario. To achieve reproducible color results, we recommend the use of the dosing syringe that is included in the DD Art Elements Vario set, which dilutes the 30 ml or 100 ml Dental Direkt dentine liquid.

Examples of mixing ratios needed to achieve the dentine liquid shades in %						
DD Art Elements Vario	0%	5%	10%	15%	20%	25%
Dental Direkt Dentine	100%	95%	90%	85%	80%	75%

Examples of mixing ratios for the 30 ml dentine liquid						
DD Art Elements Vario	0.0 ml	1.6 ml	3.3 ml	5.3 ml	7.5 ml	10.0 ml
Dental Direkt Dentine	30 ml	30 ml	30 ml	30 ml	30 ml	30 ml

Examples of mixing ratios for the 100 ml dentine liquid						
DD Art Elements Vario	0.0 ml	5.3 ml	11.1 ml	17.6 ml	25.0 ml	33.3 ml
Dental Direkt Dentine	100 ml	100 ml	100 ml	100 ml	100 ml	100 ml

This information should serve as a basic guide when diluting the dentine liquids with DD Art Elements Vario. It is advisable to prepare a color sample before use.

III) PIGMENTATION OF THE LIQUIDS

All colors have a special pigmentation to make the coloration process easier for the user. This pigmentation does not affect the color result after sintering.

DD Basic Shade, DD Pro Shade Z & DD Pro Shade C (all dentine liquids)
→ **Red pigmentation.**

DD Art Elements incisal liquids (SA1, SA2, SC1, SC2, SO)
→ **Yellow pigmentation.**

DD Art Elements effect colors: light pink, purple, yellow, orange
→ **Orange pigmentation.**

DD Art Elements effect colors: light brown, dark brown
→ **Green pigmentation.**

DD Art Elements effect colors: light grey, dark grey, graphite
→ **Grey-violet pigmentation.**

DD Art Elements effect colors: Blue
→ **Blue pigmentation.**

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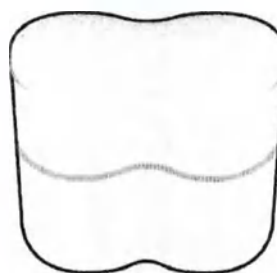
Color Box

WARNING!

It is advisable to use instruments made from plastics in order to avoid metal traces on the dental constructions.

WARNING!

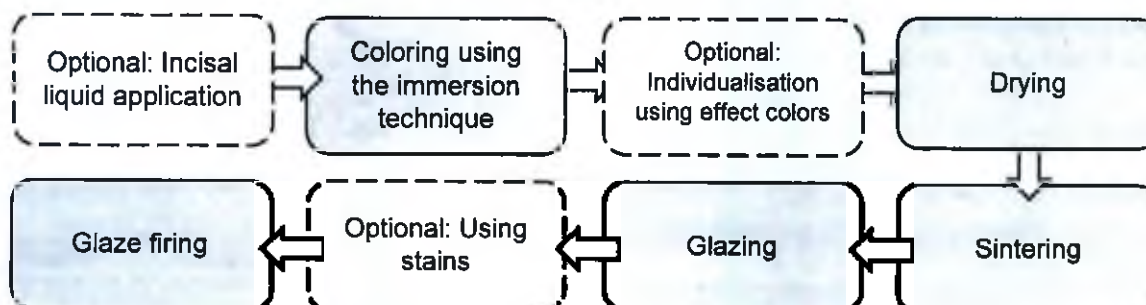
Please keep the DD Color Box closed when not in use to ensure that the liquid does not become contaminated. We recommend that the coloring liquid is stored in the original container, which is designed to be kept firmly closed.

V) COLORING TECHNIQUES - OVERVIEW

The coloring liquids available from Dental Direkt offer a wide range of color options and deliver reproducible and highly aesthetic results.

You can choose from two coloring options: immersion or paint.

1. **Immersion technique (quick, easy and reproducible)**

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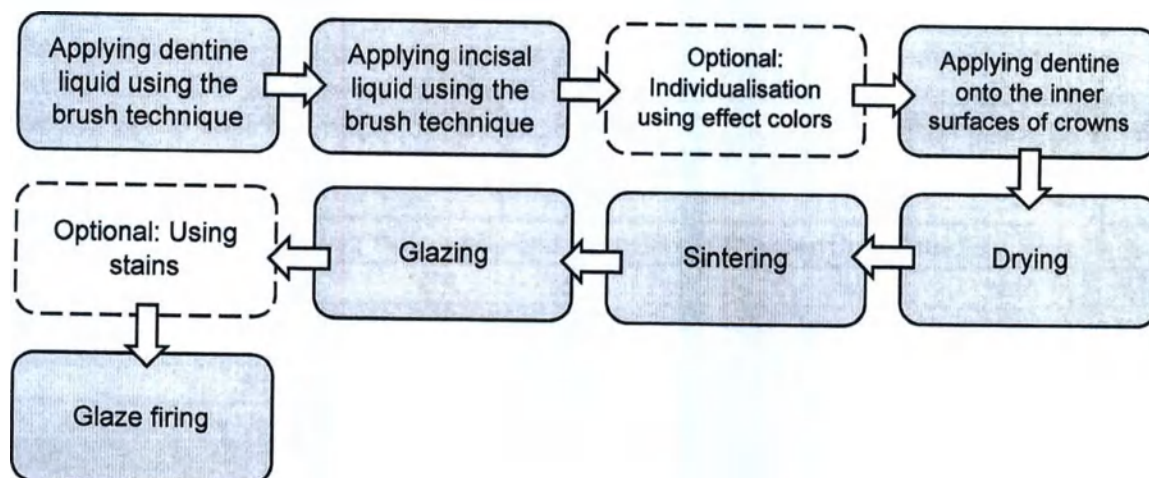
2. Paint technique (individual & artistic)

VI) IMMERSION TECHNIQUE - STEP BY STEP

WARNING!

In case of wet processing or wet grinding (e.g. by using DD Bio ZS, DD Bio ZX² 19), drying process should be realized in a (ceramic) furnace to remove coolant or lubricant from the porous structure.

Drying process is required for a homogenous coloring result when using coloring liquid!

WARNING!

The liquids must be shaken well before use!

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1.

1. Applying the SO Incisal liquid:

- The incisal liquid can be applied with a suitable applicator (e.g. snowflake) or a brush
- It is important to ensure that the applicator is moderately wet to avoid large quantities of liquid from dispersing over the construction.
- Apply the incisal liquid to the upper cusps and to the edges of the incisal area 1 to 2 times.

WARNING!

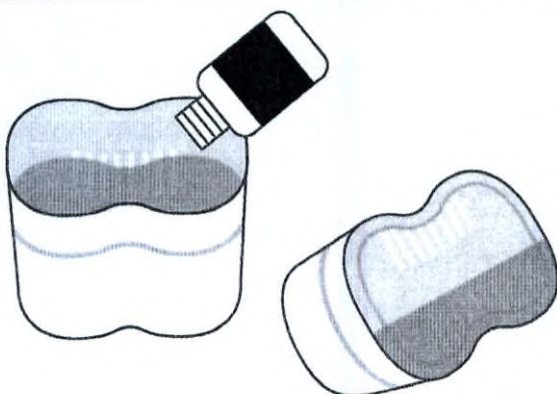
Applying too much incisal liquid will cause all the dentine color to become brighter. The incisal liquid should not be applied beyond the dashed blue line shown on the crown.

IMMERSION TECHNIQUE - STEP BY STEP

WARNING!


Immersion should take place quickly after the incisal liquid has been applied!

2. Preparation for the immersion process:

- We recommend that you use the DD Color Box for the immersion process.
- Tip:
When using less than 100 ml of the liquid, you can create the necessary volume of liquid for immersion by tilting the DD Color Box.

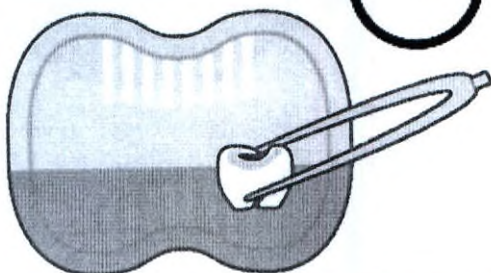Immersion time:
15 - 60 Secs.


3. Immersion process

- Use the plastic tweezers to carefully place the construction into the sieve insert inside the DD Color Box.

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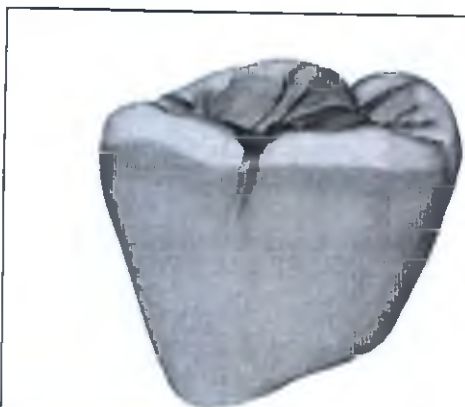Incisal liquid
Incisal / dentine overlapping area
Dentine liquid

- During the immersion process the dentine liquid and the incisal liquids blend together, which creates an overlapping area between both areas.
- The construction can be easily lifted out of the DD Color Box using the sieve insert once the immersion time has been completed.
- Alternatively, this can be carried out using plastic tongs.

IMMERSION TECHNIQUE - STEP BY STEP

- Carefully dab away any surplus liquid (especially on the inner sides of the crown and in the occlusal area) with a paper towel or blow this away using compressed air.

WARNING!

The colored construction can be drained for a few moments on a clean, absorbent pad (e.g. kitchen towel), however it cannot be stored on this type of surface.

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
TIP 1

A combination with the brush technique enables individual effects to be achieved also when using the immersion technique.

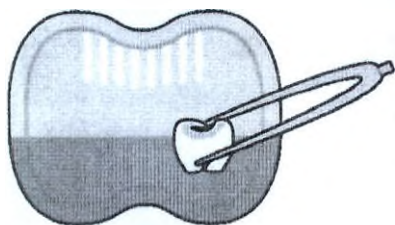- Apply the SO incisal liquid as previously described.
- Once the SO incisal liquid has been applied to the upper cusps and to the edges of the incisal area, the tooth neck can also still be coloured.
- Depending on the current colour of the tooth, it is possible to select one of the following color options to color the tooth neck: DD Art Elements light brown / dark brown, DD Art Elements orange, dentine liquid (one shade darker)

**Pre-drying:
10 mins.**

- Carry out the immersion process in the desired dentine liquid.
- Then pre-dry for 10 minutes

TIP 1 (continued)

- The pre-dried crown can now be individualised further with effect colors in the "blocked incisal area" and is then pre-dried before sintering as described under the heading entitled "Drying".
- Tip: Use the DD Art Elements effect colors to individualise as described on page 14.

Instruction for use

Colouring liquid for zirconium oxide DentalDirekt (DD):
DD Basic Shade, DD Art Elements, DD Pro Shade Z & DD Pro Shade C

 TIP 2  1-2 drops	Blocking of bulky bridge elements with incisal liquid using the immersion technique to prevent the dentine liquid from permeating too deeply. • Particularly, in the case of highly translucent zirconia, blocking can prevent bridge elements from appearing darker following longer immersion times than would occur with thinly walled crowns or caps. • Before the immersion process, apply 1-2 generous drops of SO DD Art Elements basally and allow to slowly permeate.
--	--

VII) BRUSH TECHNIQUE - STEP BY STEP

WARNING!

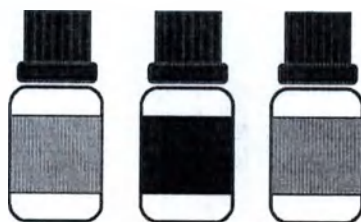
In case of wet processing or wet grinding (e.g. by using DD Bio ZS, DD Bio ZX² 19), drying process should be realized in a (ceramic) furnace to remove coolant or lubricant from the porous structure.

Drying process is required for a homogenous coloring result when using coloring liquid!

WARNING!

The liquids must be shaken well before use!

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2. Apply incisal liquid (SA/SC - colors)

3. Customisation using effect colors

- 1-2 times dark brown / light brown (select according to the color of the tooth)

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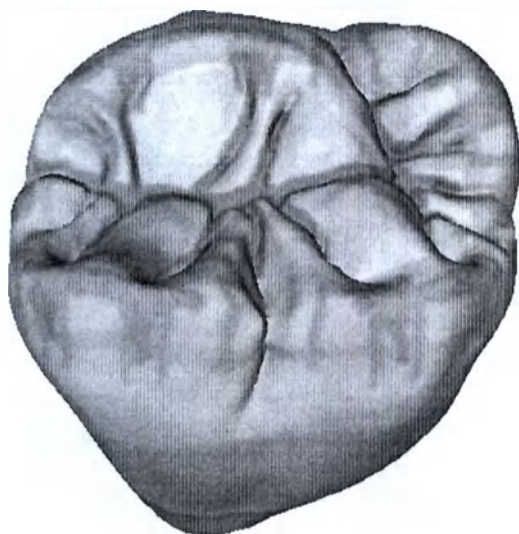
BRUSH TECHNIQUE - STEP BY STEP

Effect colors application sequence:

1. dark brown, light brown
2. blue
3. light grey, dark grey, graphite
4. purple

Accentuation using effect colors:

Shaping of (discolored) fissures, crown margins

→ Yellow, orange, light brown, dark brown effect

Even deeper effects for tooth cusps and incisal edges

→ Purple, blue, light grey, dark grey, graphite effect

Shaping of gingival areas

→ Light pink effect

WARNING!

Always carefully clean and dry the brush before using a new color.

4. Applying dentine in the inner surfaces of the crown

To achieve optimal color results, it is finally necessary to color the inner surfaces of the crown with dentine liquid. The coloring liquid should be applied liberally a number of times to ensure that the material can completely permeate through.

WARNING!

Ensure that colored constructions are stored on non-absorbant pads!

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Painting bulky components

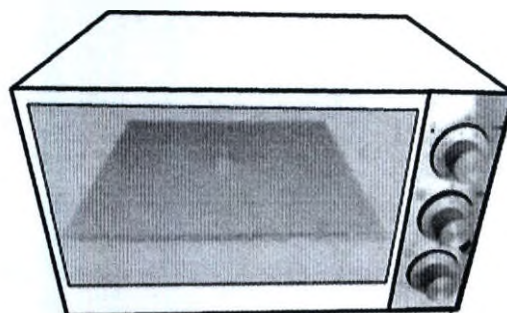
WARNING!

When coloring bulky bridge elements the number of color applications must be increased. This also applies to crowns, partial crowns and caps with thicker walls.

VIII) DRYING

We recommend that you use a small fan-assisted domestic oven with a timer switch to pre-dry the constructions because, unlike other drying methods, this will virtually guarantee an even distribution of heat.

Drying time: 10 - 30 mins.
Temperature: 80 - 100°C

Place the dental construction on a ceramic base in the oven ensuring that the air can circulate freely around it.

WARNING!

To achieve a uniform color result, the dental construction should be dried immediately after the color application. We recommend that you position crowns and bridges on their occlusal surfaces.

WARNING!

You should only ever use this oven to dry your zirconia constructions and never use it to cook food!

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IX) SINTERING

For **optimal color results** with all Dental Direkt zirconia materials (DD Bio Z, DD Bio ZX², DD cube ONE and DD cubeX²) we recommend:

Final temperature:	1450°C
Dwell:	2 hrs

To achieve a higher level of light transmittance when using DD Bio ZX², the final temperature can be raised: Final temperature: 1530°C / Dwell: 2 hrs

WARNING!

Brighter color results are achieved at higher sintering temperatures!

Sintering cycle*:

Normal furnace filling without a cover

- ↑ Heat up to 900°C (8°C/min),
- 30 mins dwell at 900°C,
- ↑ Heat to the final temp. 1450°C (3°C/min),
- 120 mins dwell at 1450°C,
- ↓ Cool to a min. of 200°C (10°C/min)

***IMPORTANT!**

It is very important that you observe our separate sintering instructions for DD Bio Z, DD Bio ZX², DD cube ONE and DD cubeX² when selecting the optimum oven programme.

X) GLAZING

The application of an additional glaze does not only ensure that the desired shade is matched precisely, but it also ensures that the restoration appears natural and is aesthetically pleasing.

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7

Firing chart for DD Nature Zr Glaze:

Firing	Base temp. [°C]	Drying [min]	Heating rate [°C/min]	Final temp. [°C]	Dwell [h]	Vacuum	Cooling / lowering [min]
Glaze firing	440	2	45	815	1:00	off	3
Glaze firing full anatomical	440	2	45	830	1:00	off	3

We recommend the use of glazes, which can be fired below 850°C. Higher temperatures and repeated glazes result in color changes.

TIP 3

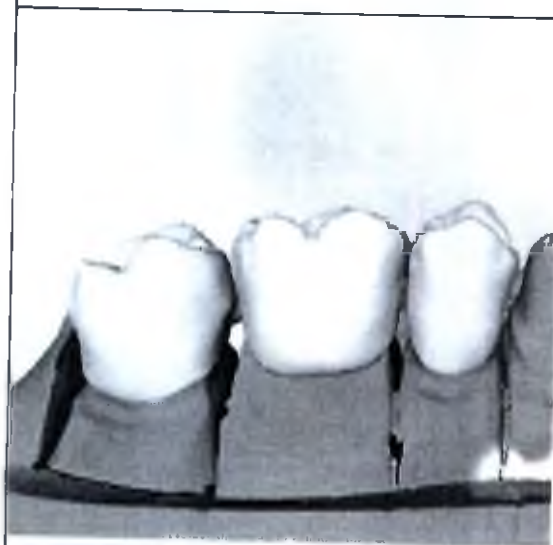
Light irradiation

Bulky bridge elements

Color assessment on the model

- Light radiation influences color perception.
- Caps are more translucent than bulky bridge elements, which means that the bulky bridge element can appear darker

Always place the bridge on the model to assess the color!

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CERTIFICATE

Biological Safety / Toxicology

Dental Direkt's coloring liquids for coloring dental frameworks made of Dental Direkt's zirconia products are medical devices and subject to strict regulatory guidelines. The DD coloring liquids are monitored and controlled regularly. The CE-sign on our products show the conformity to all regulations.

Scope:

Biological safety – toxicology, EN ISO 10993-1, Directive 93/42/EEC

Device/material:

DDBasicShade

DDProShade 

DDProShade 

DDArtElements

Manufacturer:

Dental Direkt GmbH
Industriezentrum 106-108
32139 Spenge
Germany

Tests performed:

We hereby confirm that all required tests for the evaluation of the biocompatibility have been performed according to the international standard ISO 10993-1 "Biological evaluation of medical devices".

Tests and evaluations were carried out by the accredited test laboratory „Medical Device Services – Dr. Rossberger GmbH“ in Germany.

Remark:

Possible changes in material and impurities due to the finalization at the customer were not considered in the evaluation.

Results:

The material has been tested and evaluated according to EN ISO 10993-1 for the intended use. The insolubility of the material is in compliance with the requirements of this standard. There is no evidence that hazardous effects will arise by release of leachable ingredients.

We hereby confirm the biocompatibility of the material.

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