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## CERTIFICATE

中国国际贸易促进委员会暨中国国际商会  
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade  
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of CHANGSHU KANGBAO  
MEDICAL APPLIANCE FACTORY on the annexed DOCUMENT is  
genuine.

China Council for the Promotion  
of International Trade

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CLAIM

Position: Director

Changshu Kangbao Medical Appliance Factory

常熟市康宝医疗器械厂  
Zhou Hongbao  
CHANGSHU KANGBAO MEDICAL APPLIANCE FACTORY  
Zhou Hongbao

"7<sup>th</sup> "November 2018.

## INSTRUCTION

### ON APPLICATION

Urinal disposable sterile

of 100 ml, 160 ml, 200 ml, 500 ml,

600 ml, 750 ml, 800 ml, 900 ml, 1000 ml,

1500 ml, 2000 ml, 2000+(200, 500) ml, 2500 ml,

2600+(200, 400, 500) ml, 4000 ml

with the device for fastening or without it,

with the direct or cross crane for discharge of contents

or without it, with a driving tube or without it.

1. ASSIGNMENT
2. MAIN TECHNICAL CHARACTERISTICS
3. CONTRAINDICATIONS TO APPLICATION
4. PREPARATION OF THE PRODUCT FOR WORK
5. RULES OF STORAGE AND USE

### signment.

Urinal disposable sterile of 100 ml, 160 ml, 200 ml, 500 ml, 600 ml, 750 ml, 800 ml, 900 ml, 1000 ml, 1500 ml, 2000 ml, 2000+(200, 500) ml, 2500 ml, 2600+(200, 400, 500) ml, 4000 ml with the device for fastening or without it, with the direct or cross crane for discharge of contents or without it, with a driving tube or without it productions "Chansha Kanbao Medikal of Aplayens of Fektori" are intended (Changshu Kangbao Medical Appliance Factory) for collecting urine in stationary and out-patient conditions.

Estimated class of risk of application in accordance with GOST 31508-2012: 1.

## 2. Technical characteristics.

Urinal disposable sterile of 100 ml, 160 ml, 200 ml, 500 ml, 600 ml, 750 ml, 800 ml, 900 ml, 1000 ml, 1500 ml, 2000 ml, 2000+(200, 500) ml, 2500 ml, 2600+(200, 400, 500) ml, 4000 ml with the device for fastening or without it, with the direct or cross crane for discharge of contents or without it, with a driving tube or without it are intended for use in stationary and out-patient conditions.

Urinal of small volume for direct collecting urine of 100 ml, 160 ml, 200 ml is used in offices of pathology of newborns, in intensive care units and also in house conditions for control of level of a diuresis and for an urine intake for analyses. Urinal is made of transparent colourless polyethylene, brand PE-300. On a Urinal graduation from 0 to 200 ml, with the price of division of 10 ml is put. Urinal is universal for boys and girls. It is supplied with a special hypoallergenic sticky clamp which provides a reliable prileganiye.

Urinals 500 ml, 600 ml, 750 ml, 800 ml, 900 ml, 1000 ml, 1500 ml, 2000 ml, 2000+(200, 500) ml, 2500 ml, 2600+(200, 400, 500) ml, 4000 ml are made of PVC, brand SG-7, manufactured by XINJIANG TIANYE FOREIGN TRADE CO LTD transparent colorless, tips made of ABS granules (acrylonitrile butadiene styrene) brand KunLun (manufactured by Zhejiang Kehao Plastic Machinery Co., Ltd., China), regulatory document (passport) No. SQ 5643150, white color, KunLun dye (manufactured by Fushun Dongke Fine Chemical Co., Ltd., China), regulatory document (passport) No. KS 9345171; tip caps are made from ABS granules (acrylonitrile butadiene styrene) of the brand KunLun (manufactured by Zhejiang Kehao Plastic Machinery Co., Ltd., China), regulatory document (passport) No. SQ 5643150, blue, the dye of the brand KunLun (manufactured by Fushun Dongke Fine Chemical Co., Ltd., China), regulatory document (passport) No. KS 9345174, blue, dye brand KunLun (manufactured by Fushun Dongke Fine Chemical Co., Ltd., China), regulatory document (passport) No. KS 9345178. On a Urinal graduation from 0 to 4000 ml, with the price of division of 100 ml is put. Urinal of 2000+(200, 500) ml, 2600+(200, 400, 500) ml have the 200, 400 or 500 ml additional

ulation chamber for monitoring of an hourly diuresis. The measuring scale of the accumulation chamber has large figures and a convenient arrangement, thereby, facilitating urination monitoring. Urinal have various form: rectangular, oval, pear-shaped. The bag of a Urinal is equipped with the special non-return valve which prevents a reverse current of urine, considerably reducing risk of development of the ascending infection. The volume of collected urine is easily read out from the scale located on the forward transparent party of a Urinal.

Urinal are issued with a driving tube or without it. The driving tube with the connector, steady against excesses, from 70 to 130 cm long allows to attach a Urinal to any size and type of a catheter.

Urinal are issued with the device for fastening or without it. When the patient is in a lying status, the Urinal if necessary is attached to the patient's bed. For this purpose on a bag of a Urinal special loops, hooks or belts are provided.

When the patient independently moves, the Urinal if necessary is attached to his leg. For this purpose on a Urinal there are special tapes for fastening.

Urinal is made with the direct or cross crane for discharge of contents or without it. The drain crane of a Urinal allows to make sampling of urine on the analysis and to empty it as required that the Urinal throughout the entire period of treatment allows to use. The cross crane for discharge of contents gives the chance to work with a Urinal one hand that provides higher level of hygiene in comparison with the direct drain crane.

Urinal are sterilized by ethylene oxide. Packed in polyethylene, brand PE-300.

### **3. Contraindications to application.**

Use of a Urinal in the presence of the damaged packing is inadmissible.

It is inadmissible to reuse a Urinal for collecting urine.

Use of a Urinal at an allergy to the materials used at production of Urinal is inadmissible.

Use of a Urinal on expiry date is inadmissible.

### **4. Preparation of a product for work.**

- Open packing of a Urinal
- Get a Urinal.
- Close the crane for discharge of contents.
- If necessary to fix a Urinal on a bed or the patient's leg.
- Attach the connector of a driving tube to a catheter.
- When filling a Urinal to open the crane for urine discharge.
- After depletion of a Urinal to close the crane for urine discharge.

### **5. Rules of storage and use.**

store in the dry place at the room temperature. Open packing of a Urinal before use to avoid pollution. It is necessary to pay attention to the expiry date of the validity specified on packing and integrity of packing.

Expiration date of Urinal of 5 years from the date of sterilization.

**Details of the official representative of the manufacturer:**

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常熟市康宝医疗器械厂  
Zhou Hongbao  
CHANGSHU KANGBAO MEDICAL APPLIANCE FACTORY

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