Abbott Ireland Diagnostics Division,
Finisklin Business Park,
Sligo, Ireland

Tel. +353 43 3331000
Fax +353 43 3331001

I, Lisa Murphy, Regulatory Affairs Specialist, Abbott Ireland Diagnostics Division, Finisklin Business Park, Sligo, Ireland, hereby certify that the attached documents that keep the number of pages indicated below are the Instruction for Use. These documents are in a full accordance with actual manuals of manufacturer.

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SLIGO CHAMBER OF COMMERCE (INC)
CERTIFIED CORRECT IN ALL GOOD FAITH

27/06/2019
Authorized Official

Abbott

Abbott Ireland Diagnostics Division,
Finisklin Business Park,
Sligo, Ireland

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NAME
Alinity i CEA

INTENDED

The Alinity i analyzer
Carcinoem
For addition
insert and

CONTENTS

The CAL
The CAL
protein
Preser
The ca

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Date:

24 Jun 2015

Lisa Murphy,
Regulatory Affairs Specialist
Abbott Ireland Diagnostics Division
Finisklin Business Park, Sligo, Ireland

el. +353 43 333
ax +353 43 333

Alinity i

CEA Calibrators

en

CEA

REF 07P6201

G71250R03

S7P620

Read Highlighted Changes: Revised September 2018.

NAME

Alinity i CEA Calibrators (also referred to as CEA Cals)

INTENDED USE

The Alinity i CEA Calibrators are for the calibration of the Alinity i analyzer when used for the quantitative determination of Carcinoembryonic Antigen (CEA) in human serum and plasma.

For additional information, refer to the Alinity i CEA reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

The **CAL 1** contains phosphate buffer and protein (bovine) stabilizer. The **CAL 2** contains CEA (human) prepared in phosphate buffer with protein (bovine) stabilizer.

Preservatives: Antimicrobial agents.

The calibrators are at the following concentrations:

Calibrator	Quantity	CEA CONC (ng/mL)
CAL 1	1 x 3.0 mL	0
CAL 2	1 x 3.0 mL	10

MATERIALS REQUIRED BUT NOT PROVIDED

- 04R1001 Alinity ci-series Calibrator/Control Replacement Caps

STANDARDIZATION

Abbott manufactures internal reference standards for Alinity i CEA. These internal standards are referenced to the World Health Organization (W.H.O.) First International Standard 73/601 for CEA at each concentration level. Alinity i CEA Calibrators are manufactured by dilution and tested against these internal reference standards.

PRECAUTIONS

- **IVD**
- For In Vitro Diagnostic Use
- **Rx ONLY**

Safety Precautions

- **CAUTION:** This product contains human-sourced and/or potentially infectious components. Calibrator 2 contains antigen derived from a human cell line. Antigen is purified by chromatography prior to use. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴

The following warnings and precautions apply to these components*: **CAL 1** and **CAL 2**

WARNING

H316	Causes mild skin irritation.
P332+P313	If skin irritation occurs: Get medical advice / attention.

* Not applicable where regulation EC 1272/2008 (CLP) or OSHA Hazard Communication 29 CFR 1910.1200 (HCS) 2012 have been implemented.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped with new replacement cap. Return to refrigerated storage after use.

The analyzer will track In-use Stability, which is the time the calibrator is outside of refrigerated storage while on the analyzer. The analyzer will not allow the use of the calibrator if the In-use Stability has been exceeded. Maximum In-use Stability can be found in the Assay Parameter Report. For additional information on calibrator In-use Stability, refer to the Alinity ci-series Operations Manual, Section 5.

For additional information on printing assay parameters, refer to the Alinity ci-series Operations Manual, Section 5.

INSTRUMENT PROCEDURE

- Calibrator lots may be configured using the bar code label on the calibrator carton.
- For information on configuring calibrator data, refer to the Alinity ci-series Operations Manual, Section 2.
- For instructions on ordering and loading calibrators on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

QUALITY CONTROL PROCEDURES

A single sample of each control level must be tested to evaluate the assay calibration. Ensure that assay control values are within the ranges specified in the respective control package insert.

For information on ordering controls, refer to the Alinity ci-series Operations Manual, Section 5.

Once a calibration is accepted and stored, all subsequent samples may be tested without further calibration unless:

- A reagent kit with a new lot number is used.
- Daily quality control results are outside of statistically-based quality control limits used to monitor and control system performance, as described in the Quality Control Procedures section of the associated reagent package insert.
- If statistically-based quality control limits are not available, then the calibration should not exceed a 30-day limit for recalibration frequency.

This assay may require recalibration after maintenance to critical parts or subsystems or after service procedures have been performed.

For additional information, refer to the assay reagent package insert and the Alinity ci-series Operations Manual.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, if calibration does not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria, or if controls do not meet the appropriate criteria.

BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

ISO 15223 Symbols	
	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	In Vitro Diagnostic Medical Device
	Lot Number
	List Number
	Serial number
Other Symbols	
	Calibrator (1,2)
	Control Number
	Concentration
	Distributed in the USA by
	Information needed for United States of America only
	Product of Ireland
	For use by or on the order of a physician only (applicable to USA classification only).

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

Abbott Ireland
Diagnostics Division
Finisklin Business Park
Sligo
Ireland
+353-71-9171712

DISTRIBUTED IN THE USA BY

Abbott Laboratories
Abbott Park, IL 60064 USA

Customer Service: Contact your local representative or find country-specific contact information on www.abbottdiagnostics.com

Revised September 2018.

©2016, 2018 Abbott Laboratories

ADDENDU
to the Instruction
SHIPPING
Recommended
until the expira
if shipping tem
of any kind
WARRANTY
Manufacture
stored and
months from
in manufac
MAINTENANCE
Not requir
REPAIR
Not requir
DESTRUCTION
Destructio
SanPin
Requir
Transf
with lo
LISTING
98/7
PO
Pro
IN
C
D
S
M
I

ADDENDUM

to the Instruction for Use Alinity i CEA Calibrators (for Russia)

SHIPPING

Recommended shipping conditions for device is 2-8°C. Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an enclosed carrier vehicle of any kind in accordance with shipping rules, applicable to the respective carrier.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. The maximum shelf life of the medical device is 26 months from date of manufacturing. Device must be discarded if stored with deviations in manufacturer's requirements.

MAINTENANCE

Not required.

REPAIRS

Not required.

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste".

Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

98/79/EC, EN ISO 9001, ISO 13485, EN ISO 18113-1, EN ISO 18113-2

POTENTIAL USER

Professional user

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device cannot be re-applied

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

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- US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
- US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
- World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
- Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

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