THAI NIPPON RUBBER

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INSTRUCTION FOR USE

1. Always consult a doctor or a pharmacist about the compatibility of lubricated natural latex condoms with prescribed or over-the-counter treatments for application to the vagina or penis, as well as regarding the risk of an allergic reaction.
2. Always use a condoms before any contact with your partner.
3. Check the expiry date. Open the packet carefully and handle the condom gently (take care with nails, jewellery).
4. Check the direction of unrolling and gently pinch the tip of the condom to eliminate air. Use the condom correctly. If it unrolls badly, it has probably been put on inside out: throw it away and use another one.
5. To prevent the risk of sexually transmitted diseases and HIV/AIDS or pregnancy, unroll the condom along the length of the erect penis before and physical contact between partners.
6. If necessary, use an additional lubricant compatible with latex (water-based).
7. Withdraw the penis after ejaculation whilst holding the condom firmly as its base.
8. After using, tie it up and dispose of the condom (not on the toilet.)

IMPORTANT INFORMATION

1. Specs: These condoms are manufactured in accordance with GOST 4645-81 and EN ISO 4074, requirements to the construction of this product. The manufacturer guarantees the conformity of the condoms to the mentioned standards while packing the product is not damaged or opened.
2. These condoms are made of natural rubber latex which may cause allergic reactions in individuals. If you have an allergy to natural rubber latex should stop use and consult a doctor.
3. Always use a new condom for each sexual encounter (single use only).
4. Never use oil-based lubricants: vaseline, oils (massage or baby oils), body lotions, petrol-based products, fats or any oily substances (butter, margarine) as that damages natural-latex.
5. Use correctly (in accordance with this instructions). Condoms are the best way of reducing the risk of contracting sexually transmitted diseases and pregnancy. No method of prevention can guarantee absolute protection.
6. Storage: Store condoms in a cool, dry place, away from direct sunlight, and at least 1 meter from heaters. The recommended storage temperature of 0 to 25 degrees centigrade, relative humidity less than 80%.
7. Disposal: After use, remove the condom and tie it up to prevent the discharge of seminal fluid. Place the used condom in a waste container (do not throw it down the toilet).
8. Warranty period: 5 years from manufacture date.
9. Maintenance and repair: Condoms are disposable products, maintenance and repair are not allowed.
10. Transportation: transportation of condoms is permitted without any special temperature conditions in covered vehicles.

No. 269083 13 JUN 2016
BKK100

Certified Genuine Signature(s) of

AMORN DARARATTANAROJ

SURAPON DARARATTANAROJ

(MRS. CHUNEJIT SOMBOONSUK)

General Service Officer
Experienced Level

Ministry of Foreign Affairs of Thailand

(The Ministry of Foreign Affairs assumes no responsibility for the contents of the document)

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