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UK

Control Set QC

Simulated Whole Blood Controls
For use as controls of blood grouping reagents
FOR IN VITRO DIAGNOSTIC USE

PACKAGE SIZE REF 816034 VOL 4 x 5 mL Control Set QC

SUMMARY AND EXPLANATION
The purpose of daily quality assurance in the blood bank is to confirm the reliability of the test system. The test system includes reagents, test procedures and equipment. Testing known samples is an accepted method of quality control. If expected test results are observed, procedures are being performed accurately and reagents and equipment are performing properly. If unexpected results are observed, the problem may be due to improper test performance, faulty equipment, contamination or deterioration of reagents. The source of the problem should be determined and resolved before patient test results are reported.

Control Set QC provides a means of confirming the reactivity of routinely used reagents and are to be tested on each day of use. Observation of expected test results with Control Set QC will confirm the reactivity of anti-A, anti-B, anti-A,B.

For use as controls of blood grouping reagents
Zur Kontrolle von Blutgruppenreagenzien
Pour le contrôle de réactifs pour la détermination du groupage sanguin
Para el control de reactivos de determinación de grupos sanguíneos
Per il controllo dei reagenti per la determinazione dei gruppi sanguigni

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Alba Bioscience Limited