
«CERTIFY»

TRB CHEMEDICA A.G.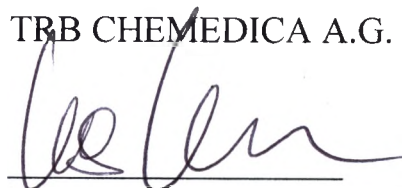

Mr. Hans Kalkbrenner
General Manager

«22» March 2016

stamp
TRB CHEMEDICA AG
Richard-Reitzner-Allee 1
D - 85540 Haar / München
Tel.: + 49 / 89 / 46 14 83-0
Fax: + 49 / 89 / 46 14 83-83

INSTRUCTIONS FOR USE

**«OSTENIL PLUS viscoelastic solution
for injection into the joint cavity»
manufactured by**

TRB Chemedica A.G., Germany/ TRB Chemedica SA, Switzerland

1. Product name:

«OSTENIL PLUS viscoelastic solution for injection into the joint cavity» (sodium hyaluronate 2.0%)

manufactured by TRB Chemedica A.G., Germany/ TRB Chemedica SA, Switzerland

2. Composition:

2 ml isotonic solution (pH 7.3) contains:

Active substance - sodium hyaluronate 40,0 mg;

excipients:

sodium chloride 12,8 mg,

disodium phosphate 1,2 mg,

sodium dihydrogen phosphate 0,1 mg,

mannitol 10,0 mg,

water for injections up to 2 ml.

3. Indications:

Pain and restricted mobility in degenerative and traumatic changes of the knee joint and other synovial joints.

4. Contra-indications:

OSTENIL PLUS should not be used in patients with ascertained hypersensitivity to one of the constituents.

5. Precautions:

Caution should be exercised in patients with known hypersensitivity to drugs. The general precautions for intra-articular injections should be observed, including measures to avoid joint infections. OSTENIL PLUS should be injected accurately into the joint cavity, if necessary under imaging control. Avoid injections into blood vessels or surrounding tissues! As no clinical evidence is available on the use of hyaluronic acid in children, pregnant and lactating women or in inflammatory joint

diseases such as rheumatoid arthritis or Bechterew disease, treatment with OSTENIL PLUS is not recommended in these cases. Do not use if the pre-filled syringe or sterile pack are damaged.

6. Undesirable effects:

Local secondary phenomena such as pain, feeling of heat, redness and swelling may occur in the joint treated with OSTENIL PLUS. Application of an ice pack for five to ten minutes onto the treated joint will reduce the incidence of these events.

7. Interactions with other products:

No information on the incompatibility of OSTENIL PLUS with other solutions for intra-articular use is available to date. The concomitant use of an oral analgesic or anti-inflammatory drug during the first few days of treatment may be helpful for the patient.

8. Dosage and administration:

Inject OSTENIL PLUS into the affected joint once a week for a total of 1-3 injections. Several joints may be treated at the same time. Repeat treatment cycles may be administered as required. In case of joint effusion it is advisable to reduce the effusion by aspiration, rest, application of an ice pack and/or intra-articular corticosteroid injection. Treatment with OSTENIL PLUS can be started two to three days later.

The content and the outer surface of the OSTENIL PLUS pre-filled syringe are sterile as long as the sterile pack remains unbroken. Take the pre-filled syringe out of the sterile pack, unscrew the Luer lock cap from the syringe, attach a suitable needle (for example 18 to 25 G) and secure it by turning slightly. Remove any air bubble, if present, before injection.

9. Characteristics and directions for use

Synovial fluid, which is viscoelastic due to the presence of hyaluronic acid, is found in all synovial joints, particularly the large weight bearing joints, where it ensures normal, painless movement due to its lubricating and shockabsorbing properties. It is also responsible for the nutrition of the cartilage. In degenerative joint disorders such as osteoarthritis, the viscoelasticity of the synovial fluid is markedly reduced thereby decreasing its lubricating and shock-absorbing functions. This increases mechanical loading of the joint and cartilage destruction which ultimately results in pain and restricted mobility of the affected joint. Supplementing this synovial fluid with intra-articular injections of highly purified hyaluronic acid can ameliorate the viscoelastic properties of synovial fluid. This improves its lubricating and shock-absorbing functions and reduces mechanical overload of the joint. As a rule this results in a decrease in pain and an improvement in joint mobility which may last for several months after a treatment cycle of 1-3 injections.

OSTENIL PLUS is a transparent solution of natural and highly purified sodium hyaluronate obtained by fermentation and is devoid of animal protein.

OSTENIL PLUS also contains mannitol, a free radical scavenger, which helps to stabilise the chains of sodium hyaluronate. In biocompatibility studies, OSTENIL PLUS was found to be particularly safe.

10. Presentation and package size:

One pre-filled syringe of 40 mg/2.0 ml OSTENIL PLUS in a sterile pack.

To be used by a physician only.

11. Transportation and storage:

Store between 2 °C and 25 °C. Do not use after the expiry date indicated on the box. Keep out of the reach of children.

Shelf life is 3 years in the original packing.

12. Manufacturer:

TRB Chemedica AG, Germany

TRB Chemedica SA, Switzerland

Address of the manufacturer:

Richard-Reitzner-Allee 1, 85540 Haar/Munchen, Germany

Chemin St.-Marc 3, CH-1896 Vouvry, Switzerland.

Authorised company representative in Russia:

AO «ORFE», 115477, Moscow, Kantemirovskaya street, 58

Telephone: +7(495)234-04-01, 7(495)234-04-02, +7(495)234-04-03

Fax: +7(495)234-97-98, +7(495)234-01-07

Email: orfeao@orfe.ru, website: www.orfe.ru

URNr. 548/2016 B

I hereby certify that the before-standing is the true signature acknowledged
in my presence of

Mr. Hans **Kalkbrenner**,

born February 14th 1969,

business address: Richard-Reitzner-Allee 1, 85540 Haar, Germany,

identified by his German Identity Card No. LF3NMM37F.

Upon examination of the commercial register of the local court of Munich of
March 18th, 2016 I furthermore certify that TRB Chemedica AG with its registered
office in 85540 Haar, Germany, mentioned in the foregoing document is a legal entity
organized and existing under the laws of Germany, registered at the local court of
Munich, HRB 114049 and Mr. Hans Kalkbrenner is authorized to sign documents on
behalf of said company as director and to individually represent the company.

Ebersberg, March 22th, 2016

A handwritten signature in blue ink, which appears to read "Dr. Baumhof".

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