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Intended use

For in vitro diagnostic use.

For use with the AQT90 FLEX analyzer as an assayed liquid quality control (LQC) to monitor product performance of the parameter listed in the Specifications insert.

Contents:

- 6 tubes with red caps
- Specifications insert - the slip of paper in a plastic sleeve on the outside slip cover

Components:

Each tube contains 0.55 mL of control solution, enough for 2 tests.

Composition:

Human serum-based matrix containing a recombinant form of human PCT peptide.

Storage:

Store below -18 °C (-1 °F).

Shelf life:

When stored correctly, the product is stable until the end of the month stated in the expiry date shown on the label.

Shelf life of the thawed product:

- When kept unused at an ambient temperature of maximum 32 °C (90 °F) after thawing, the product is stable for 1 hour (maximum).
- When kept unused at 2-8 °C (36-46 °F) after thawing, the product is stable for 8 hours (maximum).

NOTE: Do not refreeze.

Limitation of use:

Product stability is dependent on its storage temperature. The "Storage" and "Shelf life" of this test product specifications above provide details. Do not use the product if it has not been kept according to these specifications, or if it has been damaged.

Data in barcodes on the Specifications insert:

- Checksum
- Barcode type
- Area code
- Lower limit of the assigned control range
- Level name
- Lot no.
- Expiry date
- Upper limit of the assigned control range

Assigned value

The assigned value is the center value of the assigned control range. The value is lot-specific.

Quality control

NOTE: LQC material must be registered on the analyzer before use. The barcodes on the Specifications insert are used during this process.

Liquid quality control (LQC) measurements must be done:

- after a calibration adjustment
- in accordance with local, national and/or international regulations or accreditation requirements.

See the AQT90 FLEX instructions for use document.

Preparation before use

1. Remove the LQC solution from the freezer.
2. Let it thaw completely at 18-25 °C (65-77 °F). Thawing takes up to 30 minutes (maximum).
3. Analyze it within the time period specified in the "Shelf life of the thawed product" specification above.

NOTE: Do not refreeze.

WARNING - Risk of infection

Handle and discard used LQC tubes as biohazardous waste. Failure to do so may result in exposure to potentially infected material. Disposal of all waste should be in accordance with local regulations.

WARNING - Potentially infectious material

Each human donor unit used to manufacture this control was tested using FDA-accepted methods and found non-reactive for Hepatitis B Surface Antigen (HBsAg), antibody to Hepatitis C (HCV) and antibody to HIV-1/HIV-2. This product may also contain other human source material for which there are no approved tests. In accordance with good laboratory practice, all human source material should be considered potentially infectious and handled with the same precautions used with patient specimens.

Explanation of graphical symbols

See the AQT90 FLEX instructions for use document.

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