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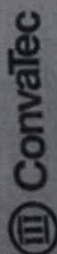
Order No./Nº de commande/Código No. 650078

Instructions

See package insert for complete instructions for use.

- ◆ Cleanse and dry perianal area.
- ◆ Trim hair with scissors.
- ◆ If necessary, enlarge opening to slightly larger than the anus.
- ◆ If required, apply Stomahesive® Paste to barrier.
- ◆ Place patient in a side-lying position with the upper knee to the chest.

- 1) Remove the release paper from the skin barrier. Fold barrier in half lengthwise, minimizing finger contact with adhesive.

- 2) Spread open the perianal area by lifting the buttock. Secure skin barrier around anus first, and then the area between the anus and perianal area. Press firmly, mold barrier.

- 3) Next, press and fit the skin barrier to the area between the anus and coccyx.

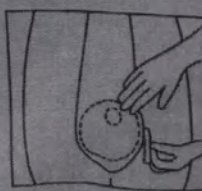
- 4) Remove release paper from the tape border, press border into place. Trim border if it impinges on vagina or scrotum. Hold barrier firmly in place for one minute to assure a secure fit.

Flexi-Seal Fecal Collector can remain in place up to 48 hours, provided it is secure.

For the collection of semi-solid stool, the drainage tap can be cut off, and then attach the enclosed tail closure [see package insert].

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© 2015 ConvaTec Inc.
®/™ indicates a trademark of ConvaTec Inc.