Abbott

Abbott Ireland Diagnostics
Division,
Lisnamuck,
Longford
Co. Longford
Ireland

Tel. +353 43 3331000
Fax +353 43 3331001

I, Nora Fitzgerald, Regulatory Affairs Specialist, Abbott Ireland Diagnostics Division, Longford, Ireland, hereby certify that the attached documents that keep the number of pages indicated below are the Insert for Use. These documents are in a full accordance with actual manuals of manufacturer.

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Date:

10 Apr - 2019

Nora Fitzgerald
Regulatory Affairs Specialist
Abbott Ireland Diagnostics Division
Lisnamuck, Longford
Co Longford
Ireland

ABBOTT DIAGNOSTICS DIVISION LONGFORD

SLIGO CHAMBER OF COMMERCE (INC)
CERTIFIED CORRECT IN ALL GOOD FAITH

Geraldine Courtney 11/04/19

Authorised Official

Alinity i

Prolactin Controls

 **en**

Prolactin

REF 07P6610

G71141R03

C7P660

Read Highlighted Changes: Revised February 2018.

NAME

Alinity i Prolactin Controls (also referred to as Prolactin Ctrl)s

INTENDED USE

The Alinity i Prolactin Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer (reagents, calibrators, and instruments), when used for the quantitative determination of prolactin in human serum and plasma. For additional information, refer to the Alinity i Prolactin reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

The **CONTROL L**, **CONTROL M**, and **CONTROL H** contain prolactin (human) in TRIS buffer with protein (bovine) stabilizers.

Preservative: Sodium azide.

The controls are at the following ranges and target concentrations:

Control	Quantity	Prolactin CONC	RANGE	Prolactin CONC	RANGE
		(ng/mL)	(ng/mL)	(mIU/L)	(mIU/L)
CONTROL L	1 x 8.0 mL	8.0	6.2 - 9.8	168	130 - 206
CONTROL M	1 x 8.0 mL	20.0	15.4 - 24.6	420	323 - 517
CONTROL H	1 x 8.0 mL	40.0	30.8 - 49.2	840	647 - 1033

NOTE: The insert ranges for the controls are not lot specific and represent the total range of values which may be generated throughout the life of the product. It is recommended that each laboratory establish its own means and acceptable ranges which should fall within the package insert ranges. Sources of variation that can be expected include:

- Calibration
- Control lot
- Reagent lot
- Calibrator lot
- Instrument

STANDARDIZATION

The controls are manufactured by dilution and referenced to the World Health Organization (WHO) 3rd International Standard 84/500 for Prolactin at each concentration level.

PRECAUTIONS

- **IVD**
- For *In Vitro* Diagnostic Use
- **Rx ONLY**

Safety Precautions

-  **CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens, Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴

The following warnings and precautions apply to: CONTROL L , CONTROL M , and CONTROL H	
Contains sodium azide.	
EUH032	Contact with acids liberates very toxic gas.
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- Thaw completely at room temperature (15 to 30°C) for 1 to 2 hours, or overnight at 2 to 8°C. Do not thaw at elevated temperatures.
- Prior to each use, mix by gentle inversion (3 to 4 times).
- Record the expiration date of the thawed control on the bottle.

STORAGE

- This product is shipped on dry ice.
- Do not refreeze controls after thaw.
- Do not use past expiration date.

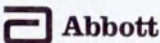
	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	-10°C or colder	Until expiration date	
Opened	2 to 8°C	Up to 60 days after thaw, not to exceed the expiration date printed on the bottle	Store tightly capped. After each use, immediately return the thawed controls to refrigerated storage (2 to 8°C).

INSTRUMENT PROCEDURE

- To obtain the recommended volume requirements for the controls, hold the bottle vertically, and dispense 3 drops of the low control, 3 drops of the medium control, and 3 drops of the high control into each sample cup in the assigned position.
- For instructions on ordering and loading controls on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, or if controls do not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria.

BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.

2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.

3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.

4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

ISO 15223 Symbols	
	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Upper limit of temperature
	Use by/Expiration date
IVD	In Vitro Diagnostic Medical Device
LOT	Lot Number
REF	List Number

Other Symbols	
AFTER THAW	After thaw store at
CN	Control Number
CONC	Concentration
CONTAINS: AZIDE	Contains Sodium Azide. Contact with acids liberates very toxic gas.
CONTROL L	Control Low, Medium, High (L,M,H)
DISTRIBUTED IN THE USA BY	Distributed in the USA by
INFORMATION FOR USA ONLY	Information needed for United States of America only
PRODUCT OF IRELAND	Product of Ireland
RANGE	Range
Rx ONLY	For use by or on the order of a physician only (applicable to USA classification only).
UNTIL FIRST USE	Until first use store at

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

Abbott Ireland
Diagnostics Division
Lisnamuck, Longford
Co. Longford
Ireland
+353-43-3331000

DISTRIBUTED IN THE USA BY

Abbott Laboratories
Abbott Park, IL 60064 USA

Customer Service: Contact your local representative or find country-specific contact information on www.abbottdiagnostics.com

Revised February 2018.

©2016, 2018 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity i Protectin Controls (for Russia)

SHIPPING AND STORAGE CONDITIONS

Recommended shipping conditions for device is - 10°C or colder in single-use thermo containers containing coolant elements, or in reusable thermo containers with automatically maintained temperature or in autorefrigerators using thermal indicators. Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an enclosed carrier vehicle of any kind in accordance with shipping rules, applicable to the respective carrier.

Medical device must be stored at a temperature - 10°C or colder in refrigerators/refrigerator chambers to ensure regulated temperature conditions with a daily temperature record.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. The maximum shelf life of the medical device is 12 months from date of manufacturing. Device must be discarded if stored with deviations in manufacturer's requirements.

MAINTENANCE

Not required.

REPAIRS

Not required

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste".

Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

ISO 9001, ISO 13485, ISO 13485 (including CMDCAS)

POTENTIAL USER

Alinity i Protectin Controls are intended to be used in a variety of clinical and laboratory conditions, including, without limitation, hospitals, diagnostic centers, reference laboratories and blood transfusion facilities around the world, whose personnel have been trained in the use of the Alinity i analyzer.

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device cannot be re-applied.

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

INDICATIONS AND CONTRA-INDICATIONS FOR USE

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