Abbott Ireland Diagnostics Division,
Finisklin Business Park,
Sligo, Ireland

SLIGO CHAMBER OF COMMERCE (INC)
CERTIFIED CORRECT IN ALL GOOD FAITH

Tel. +353 43 3331000
Fax +353 43 3331001

Abbott

May Harty 24/07/2019

Authorised Official

I, Celine Curran, Regulatory Affairs Specialist, Abbott Ireland Diagnostics Division, Finisklin Business Park, Sligo, Ireland, hereby certify that the attached documents that keep the number of pages indicated below are the Instruction for Use. These documents are in a full accordance with actual manuals of manufacturer.

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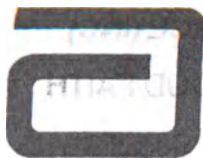
Signed: Celine Curran

Date: 24 July 2019

Celine Curran,
Regulatory Affairs Specialist
Abbott Ireland Diagnostics Division
Finisklin Business Park, Sligo, Ireland

Alinity i

AFP Calibrators

en
AFP

REF 07P9001
G71262R02
S7P900

Created August 2016.

NAME

Alinity i AFP Calibrators (also referred to as AFP Cals)

INTENDED USE

The Alinity i AFP Calibrators are for the calibration of the Alinity i analyzer when used for the quantitative determination of alpha-fetoprotein (AFP) in human serum, plasma and amniotic fluid. For additional information, refer to the Alinity i AFP reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

CAL A contains phosphate buffer solution with protein (bovine) stabilizer. **CAL B** - **CAL F** contain purified AFP (from human cord serum) prepared in phosphate buffer solution with protein (bovine) stabilizer.

Preservatives: ProClin 300 and ProClin 950.

The calibrators are at the following concentrations:

Calibrator	Quantity	AFP CONC	
		(ng/mL)	(IU/mL)
CAL A	1 x 3.0 mL	0	0
CAL B	1 x 3.0 mL	15	12.45
CAL C	1 x 3.0 mL	45	37.35
CAL D	1 x 3.0 mL	300	249
CAL E	1 x 3.0 mL	1500	1245
CAL F	1 x 3.0 mL	2000	1660

MATERIALS REQUIRED BUT NOT PROVIDED

- 04R1001 Alinity ci-series Calibrator/Control Replacement Caps

STANDARDIZATION

The Alinity i AFP calibrators are manufactured gravimetrically and are referenced to the World Health Organization (WHO) First International Standard 72/225 for Alpha-fetoprotein at each concentration level. The conversion factor is 0.83 International Units per nanogram of AFP.

PRECAUTIONS

- IVD**
- For *In Vitro* Diagnostic Use

Safety Precautions

-  **CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that these reagents and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens, Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴
- The human-sourced material used in the Calibrators B-F is nonreactive for HIV-1, HIV-2, HCV, and HBV.

The following warnings and precautions apply to: **CAL A** - **CAL F**

WARNING	Contains methylisothiazolones.
H317	May cause an allergic skin reaction.
Prevention	
P261	Avoid breathing mist / vapors / spray.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves / protective clothing / eye protection.
Response	
P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash it before reuse.
Disposal	
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped with new replacement cap. Return to refrigerated storage after use.

The analyzer will track In-use Stability, which is the time the calibrator is outside of refrigerated storage while on the analyzer. The analyzer will not allow the use of the calibrator if the In-use Stability has been exceeded. Maximum In-use Stability can be found in the Assay Parameter Report. For additional information on calibrator In-use Stability, refer to the Alinity ci-series Operations Manual, Section 5.

For additional information on printing assay parameters, refer to the Alinity ci-series Operations Manual, Section 5.

INSTRUMENT PROCEDURE

- Calibrator lots may be configured using the bar code label on the calibrator carton.
- For information on configuring calibrator data, refer to the Alinity ci-series Operations Manual, Section 2.
- For instructions on ordering and loading calibrators on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

QUALITY CONTROL PROCEDURES

A single sample of each control level must be tested to evaluate the assay calibration. Ensure that assay control values are within the ranges specified in the respective control package insert.

For information on ordering controls, refer to the Alinity ci-series Operations Manual, Section 5.

Once a calibration is accepted and stored, all subsequent samples may be tested without further calibration unless:

- A reagent kit with a new lot number is used.
- Daily quality control results are outside of statistically-based quality control limits used to monitor and control system performance, as described in the Quality Control Procedures section of the associated reagent package insert.
- If statistically-based quality control limits are not available then the calibration should not exceed a 30-day limit for recalibration frequency.

This assay may require recalibration after maintenance to critical parts or subsystems or after service procedures have been performed.

For additional information, refer to the assay reagent package insert and the Alinity ci-series Operations Manual.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, if calibration does not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria, or if controls do not meet the appropriate criteria.

BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	Calibrator (A, B, C, D, E, F)
	Control Number
	Concentration
	In Vitro Diagnostic Medical Device
	Lot Number
	Product of Ireland
	List Number
	Serial Number

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

Abbott Ireland
Diagnostics Division
Finisklin Business Park
Sligo
Ireland
+353-71-9171712

Customer Service: Contact your local representative
or find country-specific contact information on
www.abbottdiagnostics.com

Created August 2016.

©2016 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity i AFP Calibrators (for Russia)

SHIPPING

Recommended shipping conditions for device is 2-8°C. Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an enclosed carrier vehicle of any kind in accordance with shipping rules, applicable to the respective carrier.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. The maximum shelf life of the medical device is 12 months from date of manufacturing. Device must be discarded if stored with deviations in manufacturer's requirements.

MAINTENANCE

Not required.

REPAIRS

Not required.

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste".

Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

98/79/EC, EN ISO 9001, ISO 13485, EN ISO 18113-1, EN ISO 18113-2

POTENTIAL USER

Professional user

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device cannot be re-applied

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

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