«FLEXOMATIC SDN. BHD.» Technical File (Appendix 2) Section: Instruction for Use	Doc No.: TF-0-02 Revision: 0	Page: 1/8 Date: 15 th November, 2018
---	---------------------------------	--

Appendix 2 to the TECHNICAL FILE FOR MEDICAL DEVICE

Instruction for use of the medical device for the Russian Federation market:

“Dental device rubber dam «Dispodent Dental Dam», disposable, non-sterile,
using to isolate the mouth cavity”

produced by the FLEXOMATIC SDN. BHD.

Siah Guat Choon

President,

FLEXOMATIC SDN. BHD.

NOTARISED BY

Choung Youn Sin
CHOUNG YOUN SIN
ADVOCATE & SOLICITOR
NOTARY PUBLIC
1A, LINTANG GELUGOR,
41050 KLANG,
SELANGOR D.E.

31 MAY 2019

«FLEXOMATIC SDN. BHD.»	Doc No.:	TF-0-02	Page:	2/8
Technical File (Appendix 2)	Revision:	0	Date:	15 th November, 2018
Section: Instruction for Use				

Instruction for Use for Russian Federation market

Dental device rubber dam «Dispodent Dental Dam», disposable, non-sterile, using to isolate the mouth cavity

«FLEXOMATIC SDN. BHD.»	Doc No.: TF-0-02	Page: 3/8
Technical File (Appendix 2)	Revision: 0	Date: 15 th November, 2018
Section: Instruction for Use		

1. NAME OF THE MEDICAL PRODUCT

Dental device rubber dam «Dispodent Dental Dam», disposable, non-sterile, using to isolate the mouth cavity, available in the following variants:

1. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.14 mm thick (Thin);
 2. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.18 mm thick (Medium);
 3. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.22 mm thick (Heavy);
 4. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.14 mm thick (Thin);
 5. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.18 mm thick (Medium);
 6. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.22 mm thick (Heavy);
 7. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.14 mm thick (Thin);
 8. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.18 mm thick (Medium);
 9. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.22 mm thick (Heavy);
 10. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.14 mm thick (Thin);
 11. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.18 mm thick (Medium);
 12. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.22 mm thick (Heavy);
- (hereinafter referred to as Rubber Dam)

2. SCOPE OF APPLICATION OF THE MEDICAL DEVICE

The area of use for the device is stomatology.

Information for potential users of the medical device: only for professional dental use. This product is intended to be used by highly qualified medical personnel in outpatient settings only.

3. INDICATIONS FOR THE USE OF THE MEDICAL DEVICE

Used during dental treatment to separate the treatment area from the rest of the oral cavity, keep it free of saliva, prevent the patient from swallowing liquids or materials, protect soft tissues from harmful liquids, and minimize the risk of infection.

4. CONTRAINDICATIONS FOR THE USE OF THE MEDICAL DEVICE

The product is contraindicated for use in patients who are:

Unable to observe the required procedure of use.

- Claustrophobic.
- Suffering from severe asthma.
- Having difficulties breathing through the nose.

- With teeth in the process of erupting.

- Allergic to latex

5. PRECAUTIONS

Do not use expired dental dams. Keep in closed packages.

6. POTENTIAL SIDE EFFECTS WHILE USING THE MEDICAL PRODUCT

Potential side effects: Patients can experience talking difficulties, when the dental dam is in use.

7. INTERACTION WITH OTHER MEDICAL PRODUCTS

The Rubber Dam can be used with other medical devices used in stomatology.

PRODUCT UTILIZATION PROCESS

The product is intended for use with special dental tools only, including: clamps, dental dam punch, forceps for clamp holding, dental dam frame. It is to be used only by personnel with all necessary permits to conduct such procedures.

1). Choose the Rubber Dam variant you need (size and thickness).

0,14 mm thick (Thin): mostly intended for use when treating front teeth (incisors and canines), partially erupted molars and premolars.

0,18 mm thick (Medium): universal variant, suitable for use in treatment of any tooth.

0,22 mm thick (Heavy): Used when particular tissue protection is required. The advantage of this variant is its wide customizing around teeth. However, it exerts greater pressure on lips and cheeks.

2). Place the Rubber Dam on using one of the described techniques (See Dental Rubber Dam Placement Technique).

There are many techniques of Rubber Dam placement. The most common ones are these 4 techniques:

- 1 - Arch Technique (Clamps and dam are introduced simultaneously)
- 2 - Wing-Stretching Technique (Clamps and dam are introduced simultaneously)
- 3 - Dam First Technique
- 4 - Clamps First Technique

3). After the procedure is finished, the dental dam from the oral cavity needs to be removed. First remove the wedges and cords, then clamps. If floss was used for better fixation, cut and remove it from the oral cavity as well. Next, remove the frame. Finally, retract the latex sheet. If the procedure included splinting or placing orthopedic (orthodontic) structures, the dental dam is to be cut, as well.

4). Dispose of the Rubber Dam.

Safety requirements: Gloves and protective goggles should be worn when using the Rubber Dam. When not in use, the dental dam should be stored in a closed package.

Risks associated with the use of the medical device: If the materials used to manufacture the device are inappropriate, the resulting device may be unfit for its intended use. The risks were identified at the stage of design and development (inadequate materials) and utilization (infection due to negligence). At the stage of design and development, the risks were managed (by careful choice of materials). The utilization risks are minimized by informing the user. Thus, all risks identified are mitigated, and the remaining risk is acceptable.

«FLEXOMATIC SDN. BHD.»	Doc No.: TF-0-02	Page: 7/8
Technical File (Appendix 2)	Revision: 0	Date: 15 th November, 2018
Section: Instruction for Use		

Physical and mechanical performance:

Item No.	Title	Tensile strength NLT (MPa)	Elongation at breaking point, NLT (%)
1	<i>Dental device Rubber Dam «Dispodent Dental Dam», disposable, non-sterile, using to isolate the mouth cavity, available in the following variants:</i> 1. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.14 mm thick (Thin); 2. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.18 mm thick (Medium); 3. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 127mm x 127mm, 0.22 mm thick (Heavy); 4. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.14 mm thick (Thin); 5. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.18 mm thick (Medium); 6. Latex rubber dam "Dispodent Dental Dam" blue, dimensions: 152mm x 152mm, 0.22 mm thick (Heavy); 7. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.14 mm thick (Thin); 8. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.18 mm thick (Medium); 9. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 127mm x 127mm, 0.22 mm thick (Heavy); 10. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.14 mm thick (Thin); 11. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.18 mm thick (Medium); 12. Latex rubber dam "Dispodent Dental Dam" green, dimensions: 152mm x 152mm, 0.22 mm thick (Heavy);	24.0	700.0

The Rubber Dam is a non-sterile, single-use product.

Contact with the human body:

- Contact type – Short-term (under 24 hours) contact with intact mucosal membranes;
- Interaction with biological media – saliva, blood. The Rubber Dam is resistant to the effects of biological fluids (saliva, blood).

This medical device is not subject to disinfection and pre-sterilizing cleaning.

«FLEXOMATIC SDN. BHD.»		
Technical File (Appendix 2)	Doc No.: TF-0-02	Page: 8/8
Section: Instruction for Use	Revision: 0	Date: 15 th November, 2018

13. USE AND HANDLING REQUIREMENTS

The Rubber Dam is intended to be used by highly qualified medical personnel in outpatient settings only.

14. REQUIREMENTS TO THE MAINTENANCE AND REPAIR OF THE MEDICAL DEVICE.

The Rubber Dam is provided in its final form and requires no maintenance or repair, other than disposal.

15. DISPOSAL AND DESTRUCTION

Used remnants are considered Class B Waste and are disposed of as required by Sanitary Rules and Regulations SanPiN 2.1.7.2790-10 *Public Health Requirements to Medical Waste Handling* (approved by Russian Chief Sanitary Inspector Provision No. 163 dated 9 December 2010). Expired or unmarketable products are disposed of with daily waste.

16. INFORMATION ABOUT THE MEDICAL DEVICE MANUFACTURER

Registration certificate: No. RZN _____ dd. _____ 20__.

No expiry date.

Manufacturer: FLEXOMATIC SDN. BHD., Malaysia

FLEXOMATIC SDN. BHD. 38, Jalan Meru Indah 20, Taman Perindustrian Meru Indah, Off Persiaran Hamzah Alang, 42200 Selangor, Malaysia Tel: +6012-3085215 Fax: +603-3392 4049

E - mail : sales@flexomatik.com Website : www.flexomatik.com

Authorized Representative in Russia: DISPOTRADE LLC

Room 226, bld. 1, 1 Vvedenskogo Street , 117342, Moscow

INN 7728585728, KPP 772801001 Tel.: 8 (495) 589-13-30, E-mail: dispodent@mail.ru

Disclaimer:

Dental device rubber dam «Dispodent Dental Dam», disposable, non-sterile, using to isolate the mouth cavity was designed and manufactured for professional use only and is to be utilized in strict compliance with this information leaflet.

The manufacturer shall not be held liable if *Dental device rubber dam «Dispodent Dental Dam», disposable, non-sterile, using to isolate the mouth cavity*, is used not for its intended purpose and/or in breach of instructions provided herein.

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