
Supplement to the Instructions for Use of the medical device

“SG Calibrator kit for UC-3500 fully automated urine chemistry analyzer
for in-vitro diagnostics”

**manufactured by
SYSMEX CORPORATION, Japan**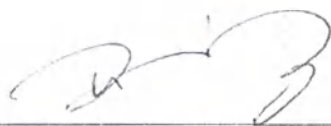

Hitoshi Nishiumi

Vice President

Regulatory Affairs

Regulatory Affairs & Quality Assurance

Instructions for Use

**SG Calibrator kit for UC-3500 fully automated urine chemistry analyzer
for in-vitro diagnostics**

**manufactured by
SYSMEX CORPORATION, Japan**

**Version 1 dated Jan 21st 2020
for the Russian Federation only**

1. Indications

“SG Calibrator kit for UC-3500 fully automated urine chemistry analyzer for in-vitro diagnostics” is intended for calibration of a refractometer when determining specific gravity of urine on UC-3500 fully automated urine chemistry analyzer for in-vitro diagnostics.

SG Calibrator must be used by qualified and properly trained medical personnel. For applicable analyzers, please refer to the IFU of the analyzer.

2. Contraindications

No contraindications.

3. Possible adverse effects

When used by specially trained personnel, and taking into account the intended use, no side effects have been identified.

4. Intended users

“SG Calibrator kit for UC-3500 fully automated urine chemistry analyzer for in-vitro diagnostics” should be used by qualified and properly trained medical personnel.

5. Physical characteristics of the device

Designation	Appearance	Color
SG Calibrator, low level (L)	Clear liquid	Yellow
SG Calibrator, medium level (M)	Clear liquid	Orange
SG Calibrator, high level (H)	Clear liquid	Crimson

6. Maintenance and repair

There is no need for maintenance and repair.

7. Storage and shelf life of unopened reagent

1. SG Calibrator should be stored at 2-10°C. Keep away from sunlight.
2. The expiration date of the unopened product is printed on the box and labels and is 24 months.

8. Storage and shelf life after first opening

1. Once opened use SG Calibrator within the same day (24 hours).

Register Number: 22


NOTARIAL CERTIFICATE

This is to certify that Kaho Uetani an agent of Hitoshi Nishiumi, Vice President, Regulatory Affairs, Regulatory Affairs & Quality Assurance of Sysmex Corporation has stated in my very presence that the said Hitoshi Nishiumi acknowledged himself to have signed to the foregoing document.

Dated this 23rd day of January, 2020



Makoto Okada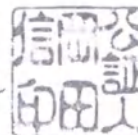

NOTARY OKADA Makoto
Kobe District Legal Affairs Bureau
44, Akashimachi, Chuo-ku,
Kobe, Japan

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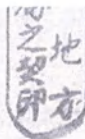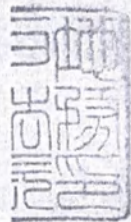Registration № 0067

CERTIFICATE

This is to certify that the annexed Notarial Certificate has been executed by Notary, duly authorized and practising in Kobe, Japan, and that the Official seal appearing on the same is genuine.

Date January 23, 2020

ISHIUCHI Masami

Director of the Kobe District Legal Affairs Bureau

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