Instruction for Use Ureteral Catheters

Device Description

- The Ureteral Catheters are provided for use in Urology procedures. The configurations available include:

Family	Size (Fr.)	Length (cm)	Type	Material
UC	3	70	Close Tip (CLT)	*PU
	4		Open Tip (OT)	
	5		Angle Tip (AT)	
	6		Cone Tip (CT)	
	7		Whistle Tip (W)	

Indications

- Ureteral Catheters are Used for drainage and navigation of a tortuous ureter.
 - Cone Tip: Used for retrograde pyelogram. The cone tip occludes the ureteral orifice.
 - Angle Tip: Used for directing a flexible Guide Wire in the ureter. The angle tip allows passage of the catheter past a ureteral stone in preparation for E.S.W.L.
 - Whistle Tip: Used for drainage and retrograde pyelogram.
 - Close Tip: Used for drainage and navigation of a tortuous ureter. Closed tip catheter will have a stainless steel stylet in which helps the catheter to straighten. No need to use guide wire.
 - Open Tip: Used for drainage and navigation of a tortuous ureter. Catheter placement should be over the guide wire only.

Contraindications

- No Specific Contraindications.

Precautions, Warnings & Adverse effects

- Carefully read all instructions for use and product labeling. The device shall only be applied for its intended use and in accordance with these instructions. Observe all cautions and warnings throughout these instructions. Failure to do so may result in complications.
- Caution: All medical staff is responsible for using the appropriate technique and deciding on the indication for use of this device based on own experience, training and medical judgment. The doctor must be trained in the proper use of the device.
- Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.
- Warning: All components of the Ureteral Catheters are for single use only.
- Caution: Do not use the device if there is any indication that the sterility of the device has been compromised.
- Adverse effects: Possible adverse effects of Ureteral Catheters placement are documented. Use of this device should be based upon consideration of risk-benefit factors as they apply to your patient. Informed consent should be obtained to maximize patient compliance. Follow up procedures.

Condition of Use

- This device should be administered to humans only by physicians authorized by the Competent Authority of the country in which the physician is practicing.
- This device is a one-time use only. Re-use may result in clinical complications.

Shelf-life of the Product

- The shelf life of the Ureteral Catheters is 3 years (Indicated on product label with the following use-by symbol and date). Store the Ureteral Catheters at a dark, dry, cool place. Avoid extended exposure to light.

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