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President D. S. Park

D. S. Park

**INSTRUCTIONS FOR USE
OF THE MEDICAL DEVICE**

**Device for the diagnostics of skin (dermatoscopy)
and hair (trihoskopiya) under magnification
Aramo Smart Lite (ASL)
with accessories in the following design styles:
see Appendix 1**

2017

TABLE OF CONTENTS

1. Name of the Medical Device.....	3
2. Intended Use.....	6
3. Area of Use.....	6
4. Indications	6
5. Contraindications.....	6
6. Possible side Effects and Adverse Events	6
7. Special Precautions.....	6
8. Conditions of Use (Temperature, Relative Humidity, Atmospheric Pressure)	7
9. Design Styles of the Medical Device	7
10. Packaging and Package Content.....	9
11. ASL Design	16
12. Software Solutionist Details	17
13. Directions of Use.....	18
14. Solutionist Software Troubleshooting & FAQ.....	70
15. Conditions of Safe Use.....	70
16. Storage Instructions.....	70
17. Technical and Performance Characteristics of the Medical Device	71
18. Device care, Maintenance and Repair	72
19. Description of the Materials Content of the Device Including Materials Contacting with Human Body.....	73
20. Device Labels Information	76
21. Safe Disposal and Utilization Instructions	86
22. Shelf Life of the Medical Device	86
23. Manufacturer's Warranty	86
24. Manufacturer	87
25. Manufacturer's Representative and Customers Service in Russian Federation	87
Appendix 1	88

1. Name of the Medical Device

Device for the diagnostics of skin (dermatoscopy) and hair (trihoskopiya) under magnification Aramo Smart Lite (ASL) with accessories in the following design styles:

1. Design style ASL-100

- 1.1 Lens ASL-100 with built-in moisture sensor – 1 pce.
- 1.2 Lithium-Ion Battery – no more than 3 pcs.
- 1.3 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 1.4 Software Solutionist (electronic version) – 1 pce.

2. Design style ASL-102

- 2.1 Lens ASL-100 with built-in moisture sensor – 1 pce.
- 2.2 Lens ASL-202 – 1 pce.
- 2.3 Lithium-Ion Battery – no more than 3 pcs.
- 2.4 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 2.5 Software Solutionist (electronic version) – 1 pce.

3. Design style ASL-300

- 3.1 Lens ASL-100 with built-in moisture sensor – 1 pce.
- 3.2 Lens ASL-202 – 1 pce.
- 3.3 Lens ASL-203 – 1 pce.
- 3.4 Lithium-Ion Battery – no more than 3 pcs.
- 3.5 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 3.6 Software Solutionist (electronic version) – 1 pce.

4. Design style ASL-103

- 4.1 Lens ASL-100 with built-in moisture sensor – 1 pce.
- 4.2 Lens ASL-203 – 1 pce.
- 4.3 Lithium-Ion Battery – no more than 3 pcs.
- 4.4 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 4.5 Software Solutionist (electronic version) – 1 pce.

5. Design style ASL-200

- 5.1 Lens ASL-202 – 1 pce.
- 5.2 Lens ASL-203 – 1 pce.
- 5.3 Lithium-Ion Battery – no more than 3 pcs.
- 5.4 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 5.5 Software Solutionist (electronic version) – 1 pce.

6. Design style ASL-202

- 6.1 Lens ASL-202 – 1 pce.
- 6.2 Lithium-Ion Battery – no more than 3 pcs.
- 6.3 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 6.4 Software Solutionist (electronic version) – 1 pce.

7. Design style ASL-203

7.1 Lens ASL-203 – 1 pce.

7.2 Lithium-Ion Battery – no more than 3 pcs.

7.3 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.

7.4 Software Solutionist (electronic version) – 1 pce.

Accessories:

1. Cradle for device Aramo Smart Lite (ASL) – 1 pce.
2. Protective cap for lens – no more than 3 pcs.
3. Sebum paper (50 pcs. per package) - no more than 10 boxes.
4. Microfiber cloth – 1 pce.
5. Hand strip – 1 pce.
6. Manual – 1 pce.

Spare parts

1. Lens ASL-100 with built-in moisture sensor – 1 pce.
2. Lens ASL-202 – 1 pce.
3. Lens ASL-203 – 1 pce.
4. Lithium-Ion Battery – no more than 3 pcs.
5. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
6. Software* Solutionist – 1 pce.

*A remark regarding the software: the software Solutionist can be delivered on removable media or can be downloaded from the manufacturer's web-site www.aramhuvis.com.

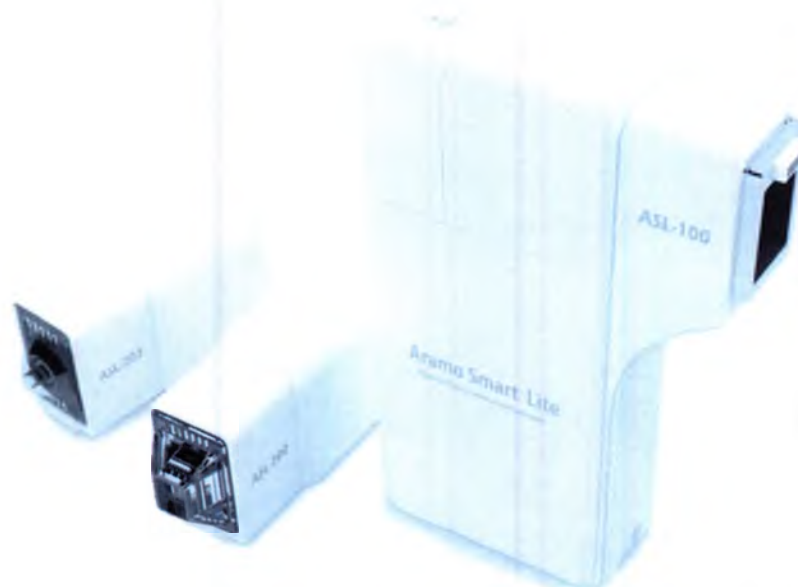
Fig. 1. a – The external view of the Device for the diagnostics of skin (dermatoscopy) and hair (trichoskopiya) under magnification Aramo Smart Lite (ASL) with lenses (from left to the right): lens ASL-202, lens ASL-203, lens ASL-100.

Fig. 1. b. *The external view of the Device for the diagnostics of skin (dermatoscopy) and hair (trichoskopiya) under magnification Aramo Smart Lite (ASL) with accessories.*

2. Intended Use

The Device for the diagnostics of skin (dermatoscopy) and hair (trihoskopiya) under magnification Aramo Smart Lite (ASL) with accessories (hereinafter – the Device for the diagnostics of skin ASL) is intended for the visual diagnostics of the patients' skin (dermatoscopy) and hair (trihoskopiya) status. Features:

Skin Analysis – skin hydration, sebum (type of the skin according the quantity of sebum), skin pores size, quantity of brown spot, acne (under UV-light), wrinkles status, skin sensitivity.

Hair Analysis – the degree of hair loss, scalp status (the type of the scalp), hair thickness, hair density, scalp redness, scalp keratin, hair pore status, hair cuticle status.

3. Area of Use

Medical and health care institutions (dermatovenerology, aesthetic medicine) and in a domestic setting.

4. Indications

Hyperkeratinization, acne, couperosis (rosacea), seborrhea, pigmentation (brown spots), hair loss, dandruff, hair dullness, hair weakness, fragility of hair.

5. Contraindications

As far as the dermatoscopy and trihoskopiya are the visual methods of analysis there are no contraindications for them known. However these methods are better to be applied at the early beginnings of diseases.

6. Possible Side Effects and Adverse Events

There are no known side effects and adverse events related to the use of the device.

7. Special Precautions

1. Since this medical device is intended for the repeated use and for the skin analysis a short-term contact with a patient's skin is necessary, the sensors surfaces (lenses) must be kept clean and protected from contamination. It is recommended to disinfect the lenses surfaces before and after each use (for details please see Section 26. Device Care, Maintenance and Repair).
2. Please read the Instructions for Use before using the device.
3. Do not use the devices at the sites with high humidity or where raining is possible. Moisture ingress can cause damage or inflammation of the device.
4. Do not disassemble or change the device construction since it can cause the shocking hazard to the user. Please do not try to repair the device if any trouble occurs – instead contact with the manufacturer's representative.
5. Do not use the power supply and current different from the recommended ones.
6. Do not leave the device under direct sun light. It can cause camera damage.
7. Avoid device falling, shaking, and knocking.

8. Conditions of Use (temperature, relative humidity, atmospheric pressure)

The medical device must be used in accordance with the good industrial hygiene and safety practice.

Conditions of use:

- Temperature: $(+15^{\circ} - +35) \pm 1^{\circ}\text{C}$;
- Relative humidity: (not more than 80%) $\pm 2\%$;
- Atmospheric pressure: $760 \pm 10 \text{ mm Hg}$.

9. Design Styles of the Medical Device

1. Design style ASL-100

- 1.1 Lens ASL-100 with built-in moisture sensor – 1 pce.
- 1.2 Lithium-Ion Battery – no more than 3 pcs.
- 1.3 DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 1.4 Software Solutionist (electronic version) – 1 pce.

2. Design style ASL-102

- 2.1. Lens ASL-100 with built-in moisture sensor – 1 pce.
- 2.2. Lens ASL-202 – 1 pce.
- 2.3. Lithium-Ion Battery – no more than 3 pcs.
- 2.4. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 2.5. Software Solutionist (electronic version) – 1 pce.

3. Design style ASL-300

- 3.1. Lens ASL-100 with built-in moisture sensor – 1 pce.
- 3.2. Lens ASL-202 – 1 pce.
- 3.3. Lens ASL-203 – 1 pce.
- 3.4. Lithium-Ion Battery – no more than 3 pcs.
- 3.5. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 3.6. Software Solutionist (electronic version) – 1 pce.

4. Design style ASL-103

- 4.1. Lens ASL-100 with built-in moisture sensor – 1 pce.
- 4.2. Lens ASL-203 – 1 pce.
- 4.3. Lithium-Ion Battery – no more than 3 pcs.
- 4.4. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 4.5. Software Solutionist (electronic version) – 1 pce.

5. Design style ASL-200

- 5.1. Lens ASL-202 – 1 pce.
- 5.2. Lens ASL-203 – 1 pce.
- 5.3. Lithium-Ion Battery – no more than 3 pcs.
- 5.4. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 5.5. Software Solutionist (electronic version) – 1 pce.

6. Design style ASL-202

- 6.1. Lens ASL-202 – 1 pce.
- 6.2. Lithium-Ion Battery – no more than 3 pcs.
- 6.3. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 6.4. Software Solutionist (electronic version) – 1 pce.

7. Design style ASL-203

- 7.1. Lens ASL-203 – 1 pce.
- 7.2. Lithium-Ion Battery – no more than 3 pcs.
- 7.3. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.
- 7.4. Software Solutionist (electronic version) – 1 pce.

Accessories (supplied along with each design style*):

- 1. Cradle for device Aramo Smart Lite (ASL) – 1 pce.
- 2. Protective cap for lens – no more than 3 pcs.
- 3. Sebum paper (50 pcs. per package)* - no more than 10 boxes.
- 4. Microfiber cloth – 1 pce.
- 5. Hand strip – 1 pce.
- 6. Manual – 1 pce.


*Except for Sebum paper – it is supplied only with the design styles including the Lens ASL-100 with built-in moisture sensor.

10. Packaging and package content

Table 1. Package content includes the following items (fig. 2)

Item	Design style						
	ASL-100	ASL-102	ASL-300	ASL-103	ASL-200	ASL-202	ASL-203
Lens ASL-100	1 pce	1 pce	1 pce	1 pce	-	-	-
Lens ASL-202	-	1 pce	1 pce	-	1 pce	1 pce	-
Lens ASL-203	-	-	1 pce	1 pce	1 pce	-	1 pce
Lithium-Ion Battery	1 pce						
DC IN Adaptor 5V (cable, power supply, adapter)	1 pce						
Software Solutionist*	1 pce						
Cradle	1 pce						
Protective cap for lens	Not more than 3 pcs.						
Sebum paper (50 pcs. per package)	Not more than 10 boxes	Not more than 10 boxes	Not more than 10 boxes	Not more than 10 boxes			
Microfiber cloth	1 pce						
Hand strip	1 pce						
Manual	1 pce						

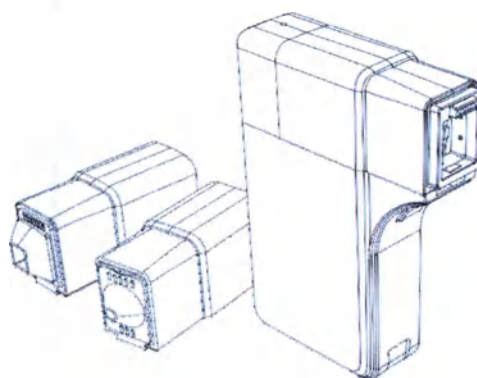*The software application Solutionist can be delivered on removable media or can be downloaded from the manufacturer's web-site www.aramhuvis.com.

Lens ASL-202 | Lens ASL-203 | The device with the lens ASL-100

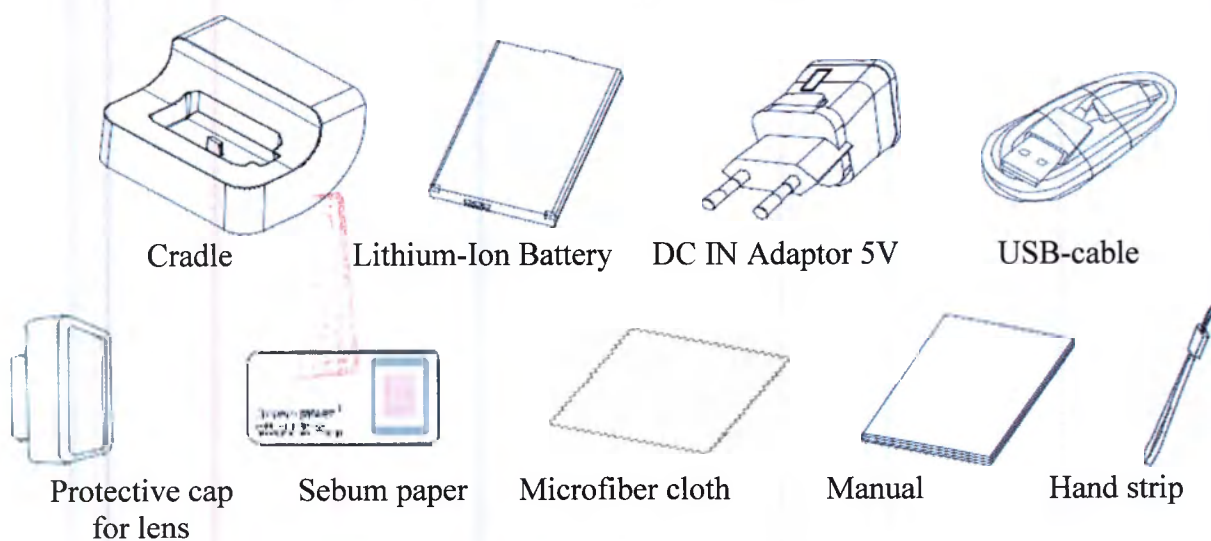
Fig. 2. *Aramo Smart Lite (ASL) package content.*

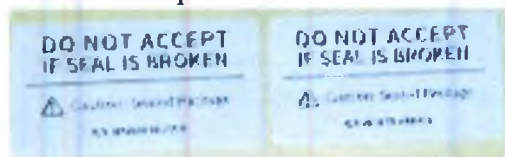
Medical device package

Device for the diagnostics of skin (dermatoscopy) and hair (trichoskopiya) under magnification Aramo Smart Lite (ASL) with accessories is packed as follows:

Customer package (gift box)


The device is packed into a cardboard box (gift box) made from white cardboard (see fig. 3). Each accessory besides the manual is packed into a PE-pouch or into a box from grey-side cardboard (as for sebum paper, DC IN adaptor) or plastic box (lenses)¹ (see fig.3, c). The device with accessories are placed inside the box at two levels: the 1st level (top – see fig. 3,a) – the device, and the 2nd level (bottom – see fig. 3,b) – for the accessories,. The device and the accessories are placed at the PS-blisters with respective cradles.

The gift box with the device and accessories are sealed with 2 sealing labels made from art paper: “Do not accept if seal is broken. Caution! Sealed Package. Aram Huvis”.

The gift box information design, symbols and signs at the package – please see the Section 28. The Device Labels Information.

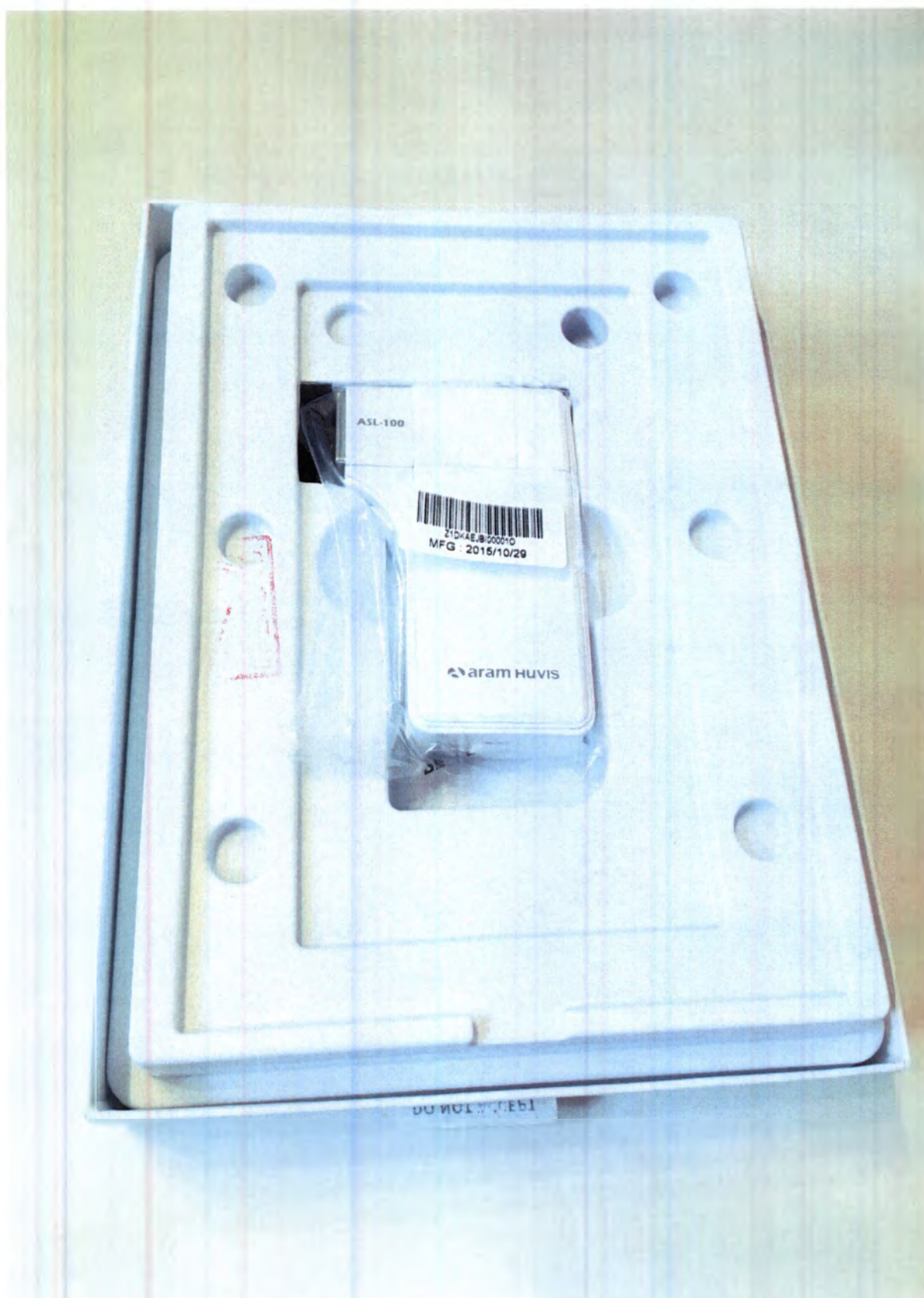¹ Lenses are packed into plastic boxes for those design styles which includes more than 1 lens (see table 1). If the medical device has more than 1 lens than it is supplied mounted onto the apparatus (see fig.. 3, b).

a

Fig. 3. ASL gift box with accessories: *a* – top; *b* – bottom (sebum paper is placed under the hand strip and cable); *c* – transparent plastic box for the lens.


b

Fig. 3. ASL gift box with accessories: *a – top; b – bottom (sebum paper is placed under the hand strip and cable); c – transparent plastic box for the lens.*

b

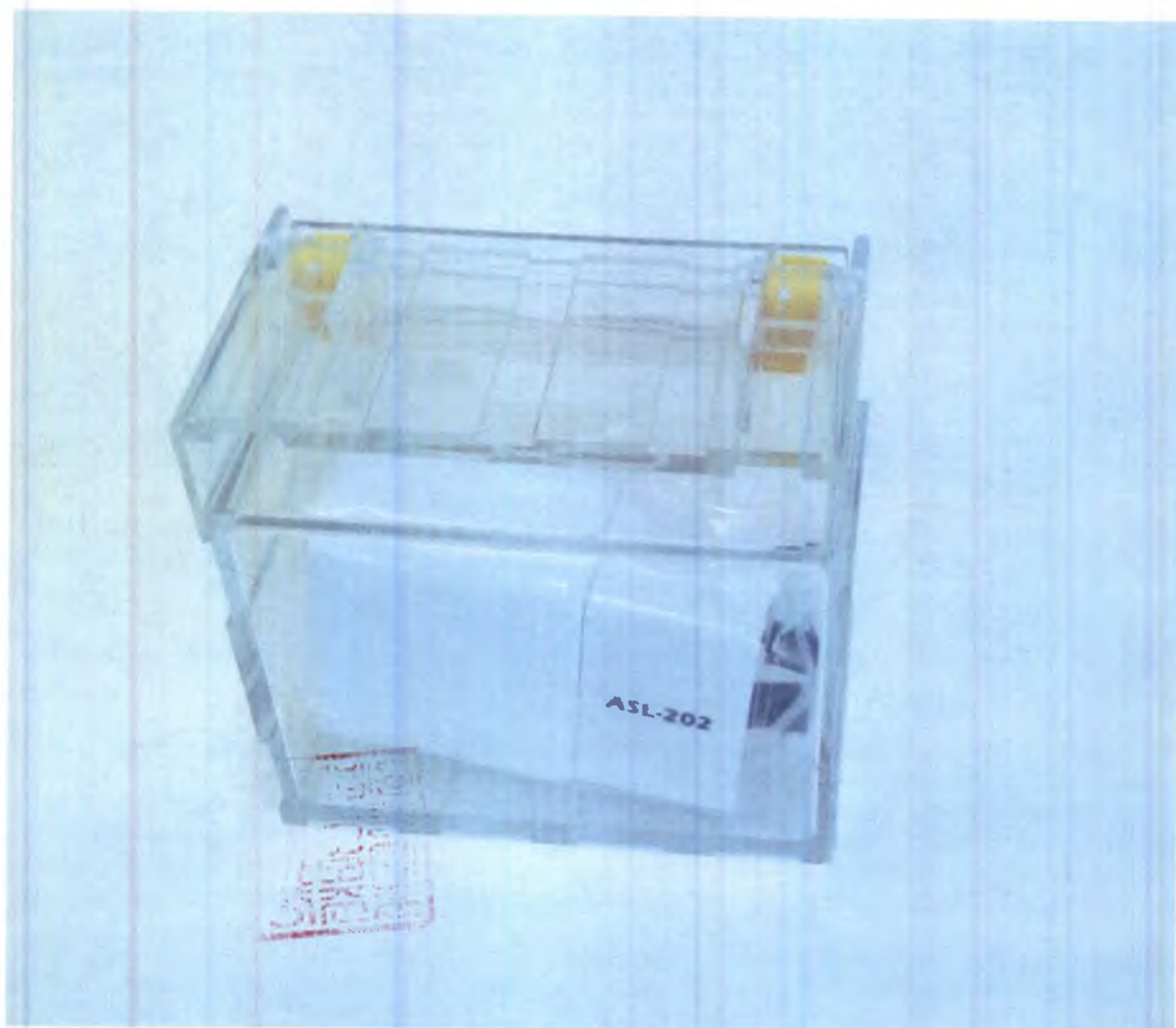
Fig. 3. ASL gift box with accessories: *a – top; b – bottom (sebum paper is placed under the hand strip and cable); c – transparent plastic box for the lens.*

c

Fig. 3. ASL gift box with accessories: *a* – top; *b* – bottom (sebum paper is placed under the hand strip and cable); *c* – transparent plastic box for the lens.

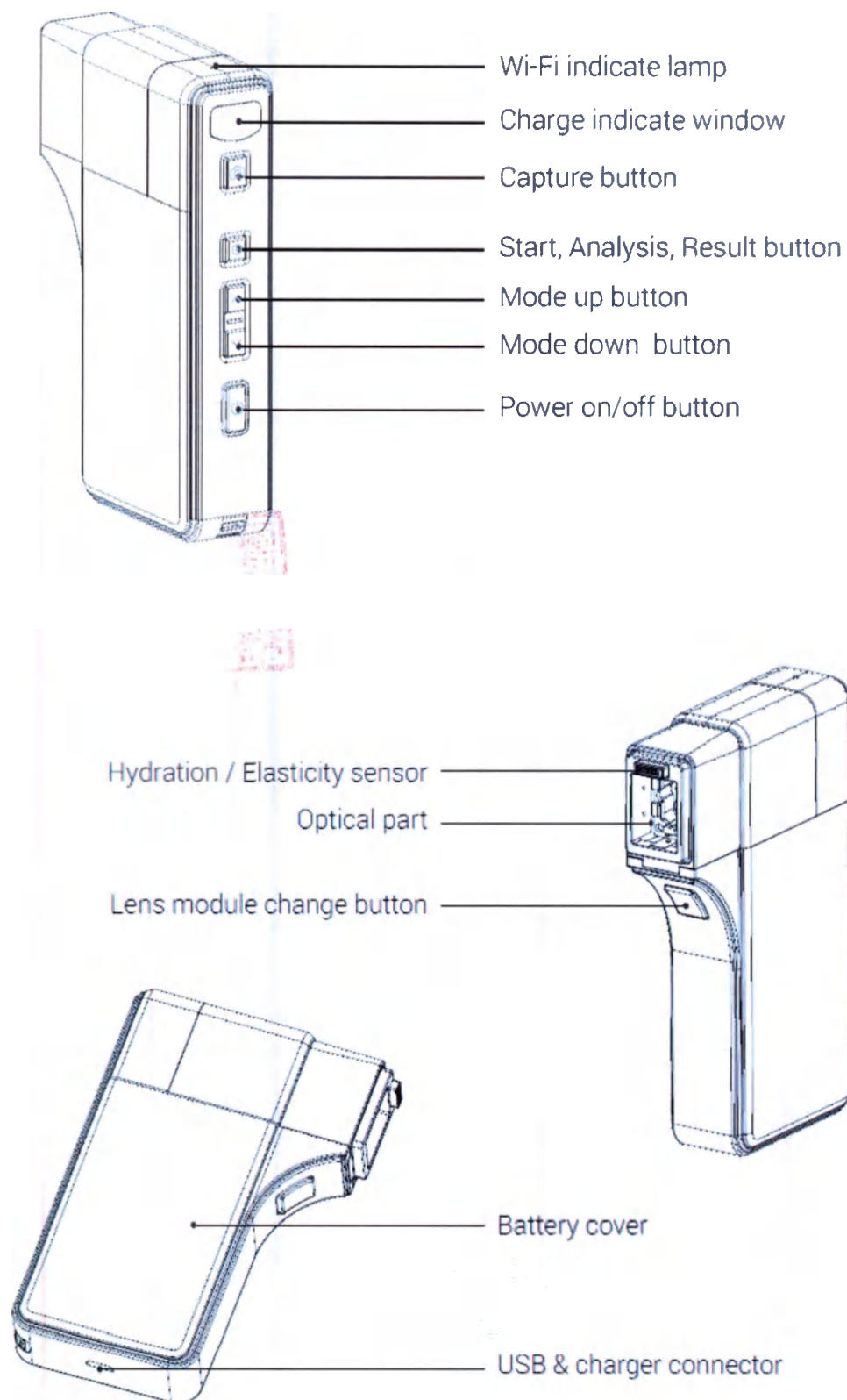11. ASL Design

12. Software Solutionist Details

Version : V1.2.35 Release date: 02.02.2017

Software version is updated when new drivers for the new target devices or operational systems (smartphones, tablets, PCs) are created.

Software size and system requirements

[Solutionist / Wizard] Windows Application is developed based on the JavaFX platform. Installing JRE(Java Runtime Environment) is essential In order to run JavaFx platform.

[Windows]

- Minimum specification

OS : Windows 8/Windows 8.1/Windows 10

Processor: Intel Core i3 or further version

Memory : More than 1GB

Disk space: Need space more than 512MB

- Recommended specification

OS : Windows 8/Windows 8.1/Windows 10

Processor: Intel Core i5 or further version

Memory: More than 2 GB (32-bit) 4 GB (64-bit)

Disk space: Need space more than 1.5 GB

[Android]

- Android 3.2 version and later required
- App size : 39.94MB

[Mac OS]

- Minimum specification

- Processor : OS X 10.8 (Mountain Lion) version or Intel Base computer that later version can be operated.

- Memory : More than 2GB

- Disk space: More than 650MB

- Recommended specification

- Processor : OS X 10.8 (Mountain Lion) version or Intel Base computer that later version can be operated.

- Memory : More than 4GB

- Disk space: More than 1.5GB

[iOS]

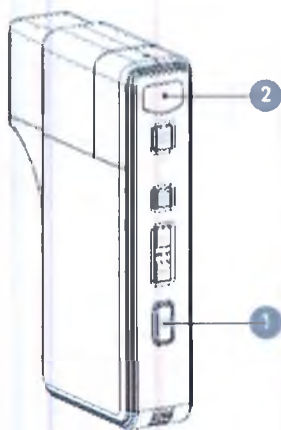
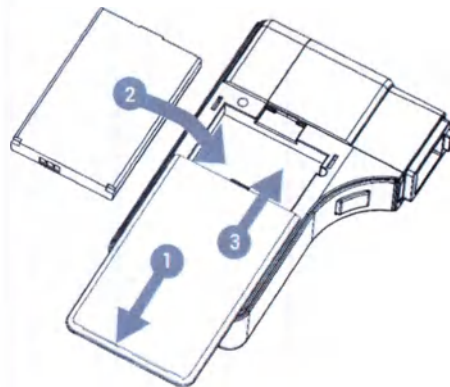
- iOS 6.0 version and later required
- Compatible with iPhone, iPad and iPod touch
- App size : 79.3MB

13. Directions of Use

System Settings

Insert the battery

1. Open the battery cover by pushing down it with finger.
2. Insert the battery and gently press the battery into place.
3. Close the cover.

How to turn on

- 1 Press and hold "power" button.
- Then "charger window" will be turned blue (2)

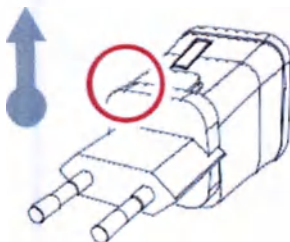
How to turn off

- 1 Press and hold "power" button.
- Then the light will be turned off (2)


DC IN Adaptor Setup

Charger can be used for American type & European type. The default is European type..


1. Push the plug up.

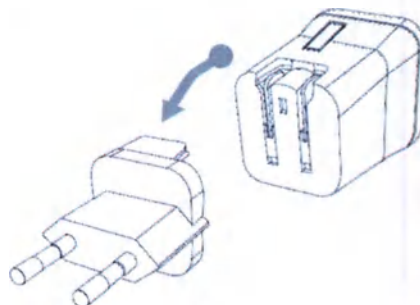2. Then 220V European type is disassembled and you can see the 110V-220V American type inside of the charger.

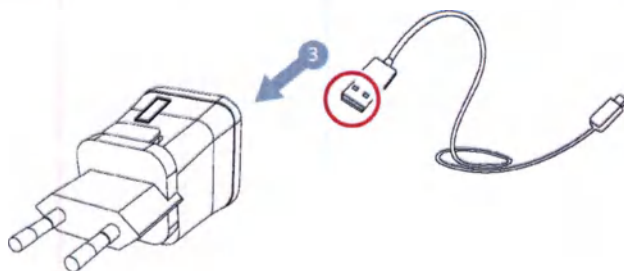
3. Choose the one you want to use and connect USB cable in adaptor.


Download and Install

PC-Users

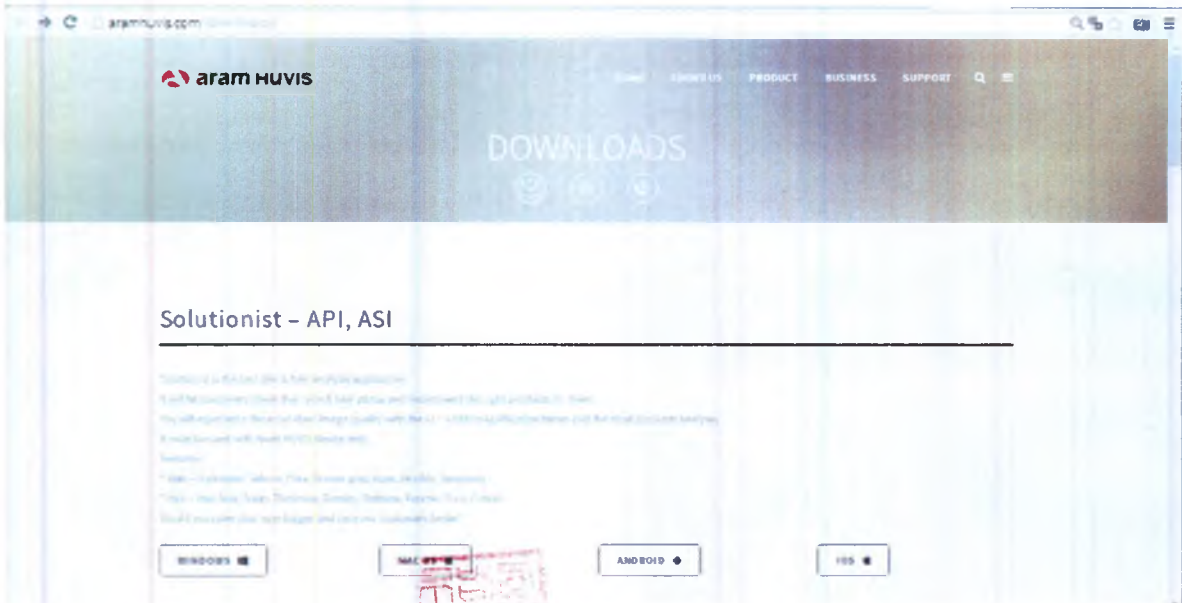1. Go to link on the browser as below. <http://www.aramhuvis.com/download>
2. Click the button “Windows” at the page. Software will be downloaded automatically.

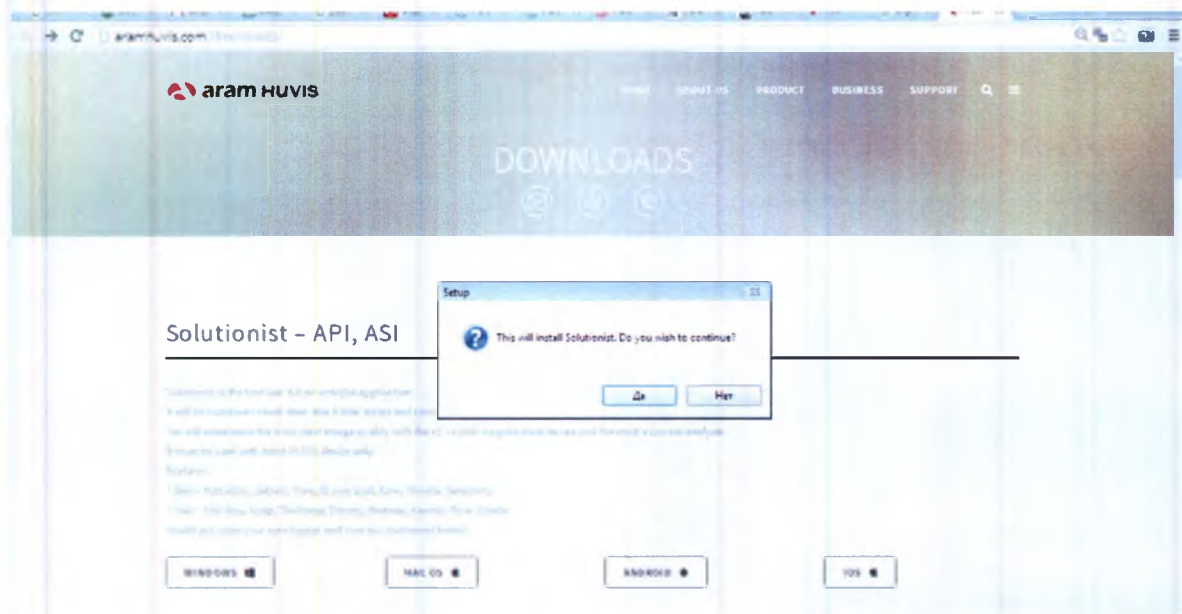3. Double click the downloaded file Solutionist_V1.2.35.exe. The message will appear: “this will install the software Solutionist on your computer. Do you want to continue?” Click the button “Yes”.

4. Choose the installation path and click “Next”.

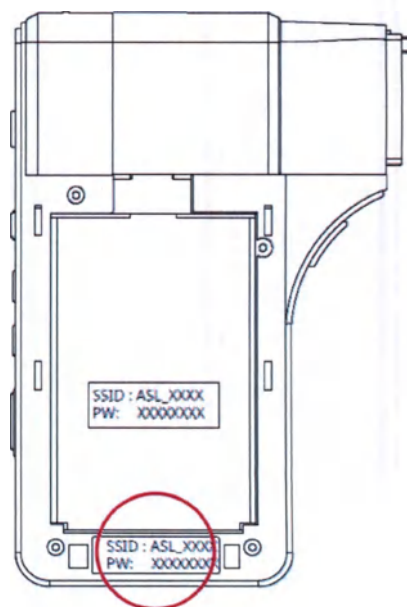
5. Install the program. The installation will start. After the installation completed the message will appear that the program has been installed successfully, and the program icon will appear at the desktop.

Tablet device or smartphone users



1. **iOS** Go to “App Store” and search “Solutionist” to download it.
2. **Android** Go to “Google Play Store” and search “Solutionist” to download it.

How to Connect with the Program

1. First of all, check the Wi-Fi name and password which is described at back side of battery cover.
2. Wi-Fi name & password must be same and the format is “ASL_XXXX”.
3. Turn on the device.
4. While turning on the device, prepare PC, Tablet, Smartphone to run the S/W.
5. Once “ASL” device is turned on, please go to network setting to set up the Wi-Fi connection.
6. Among Wi-Fi lists, find “ASL_XXXX” and select it.
7. When above code is connected, run the program “Solutionist”.
8. To check working properly, please check the program and see the left below side.

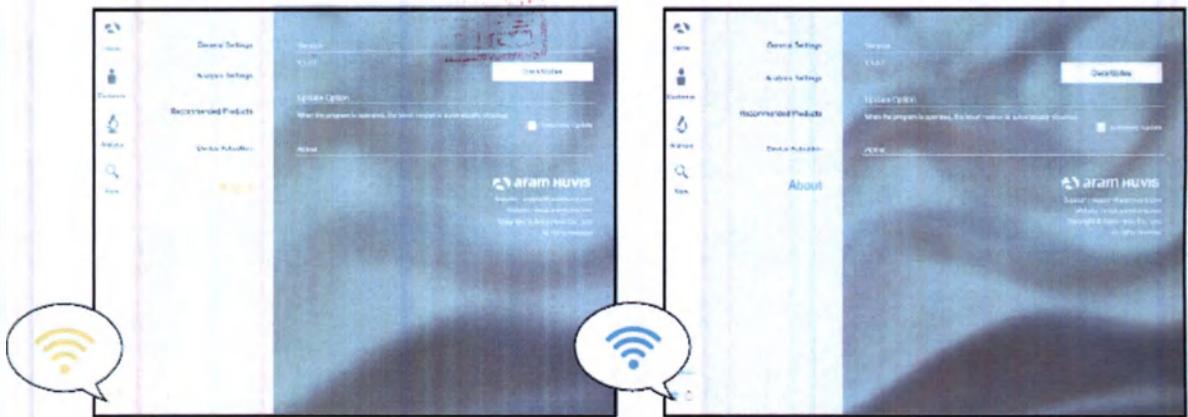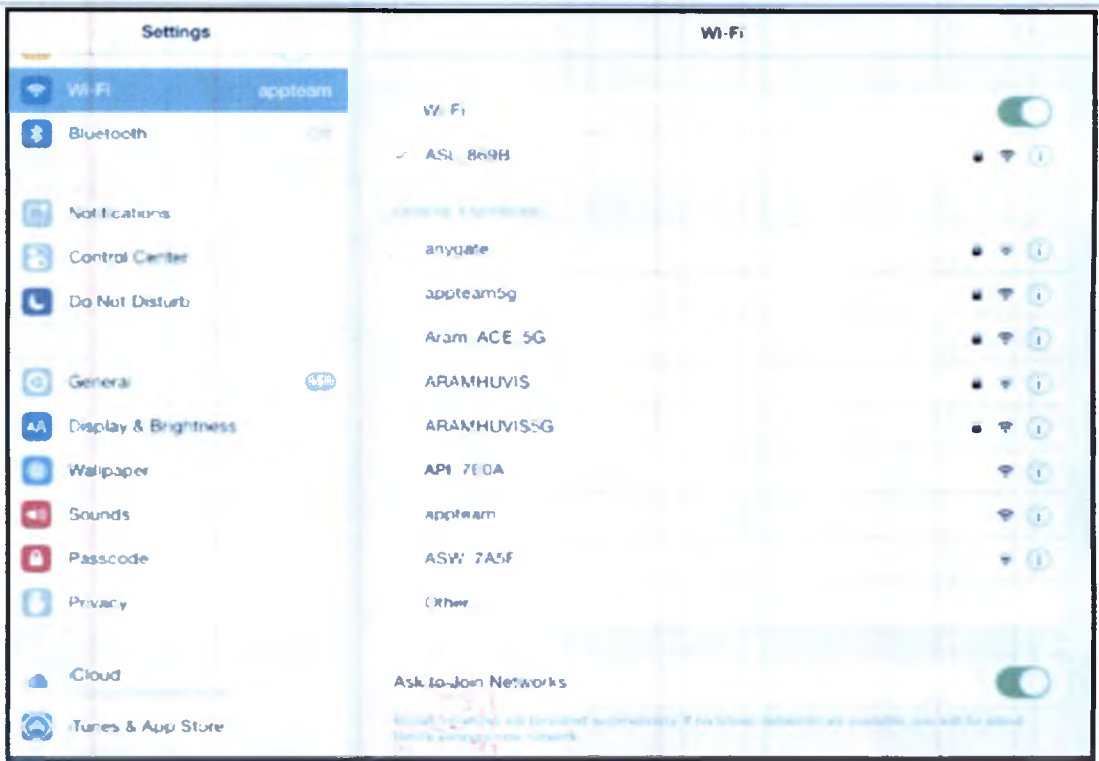9. If it is correctly set up, you can see the orange light of Wi-Fi signal


Wi-Fi Combo

It means user could use device and internet at the same time.


With this function, user can send an email after analysis immediately.

How to use?

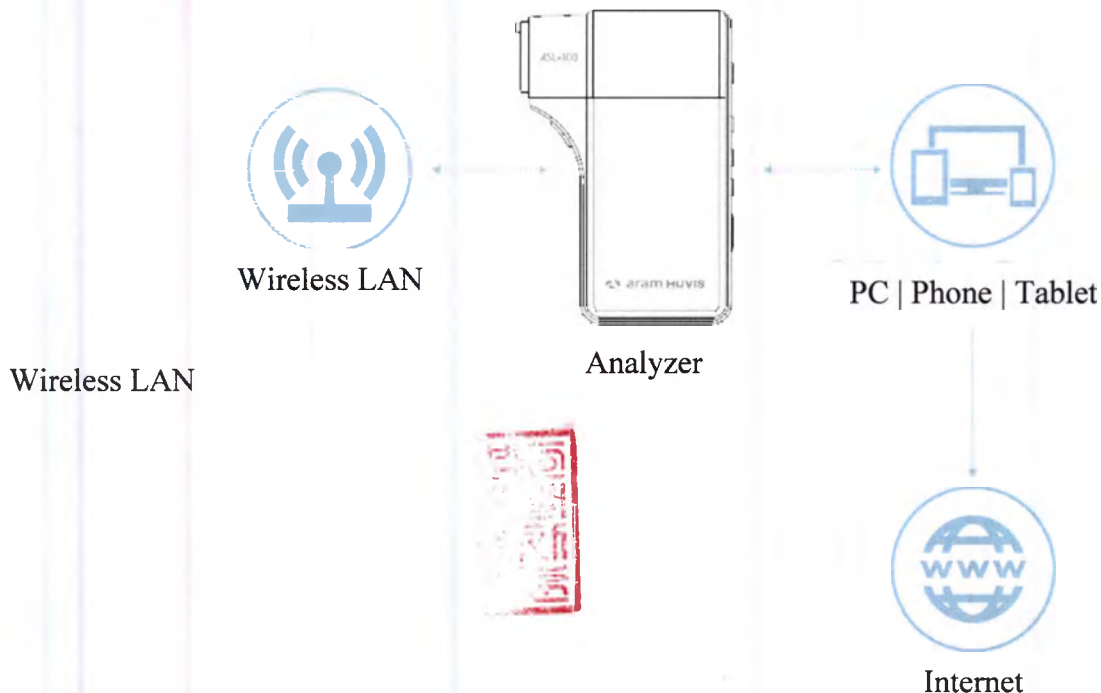1. The device has AP (ASL_XXXX) to be connected with the program “Solutionist” by Wi-Fi.
2. First of all, connect the device to AP to use it with “Solutionist”.
3. Run “Solutionist”

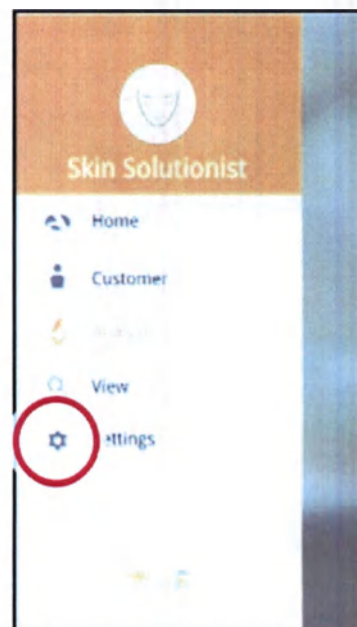
1. Select Settings


2. In “General Settings”, go to “Wi-Fi” and select “Scan” button

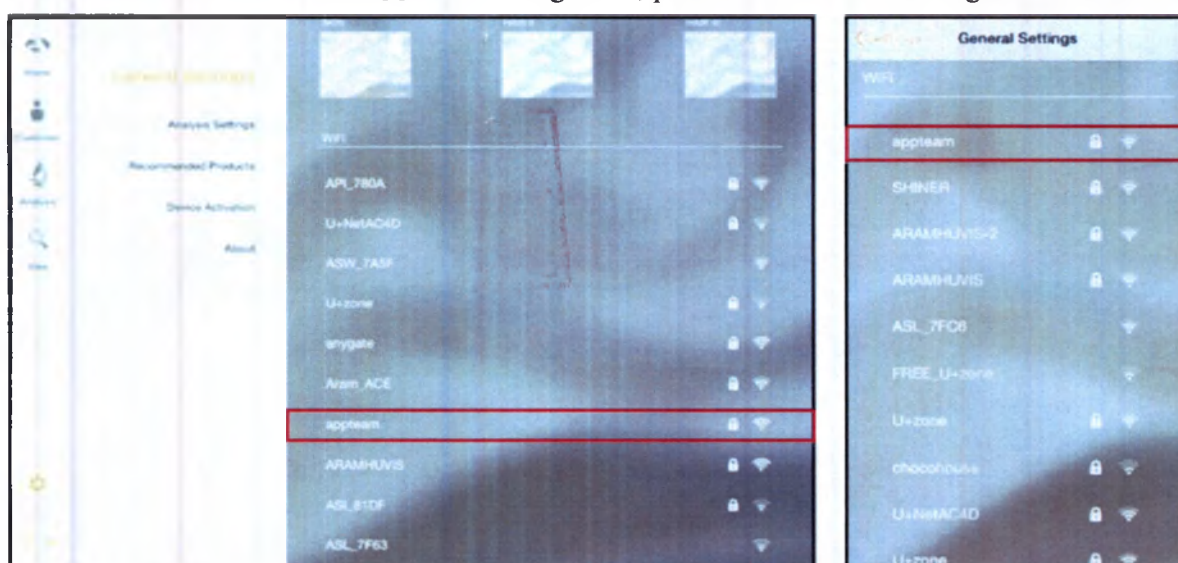3. The available Wi-Fi lists appear. Among them, please choose the best signal Wi-Fi to connect it

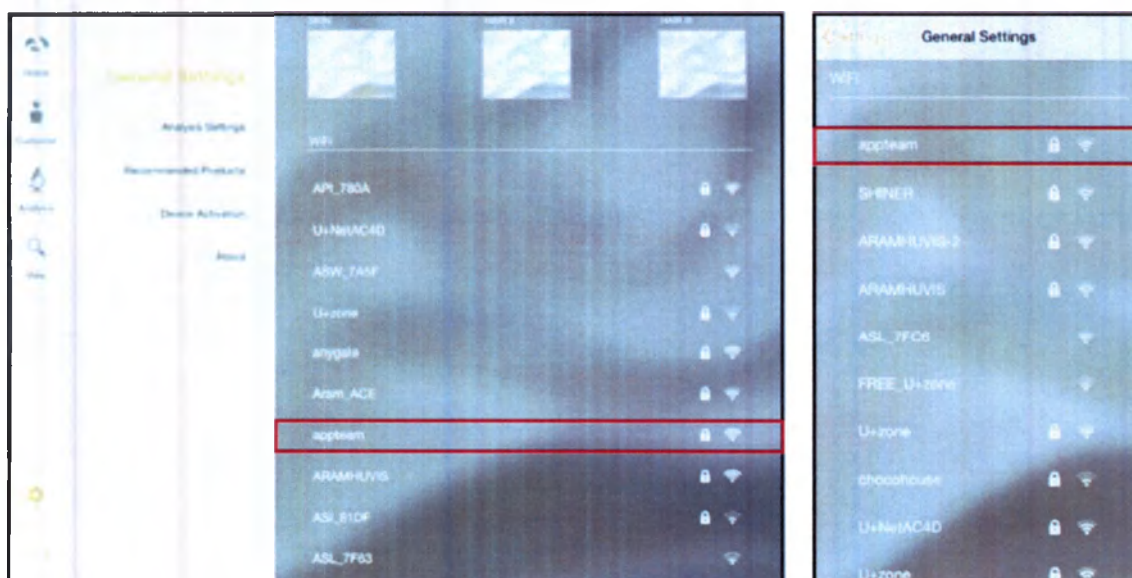
4. During connecting it, you can see the message saying that ASL device will be automatically disconnected and reboot. Please select “OK” button

5. When Wi-Fi is connected, go to “Settings” page to set up the Wi-Fi connection. You can see the Wi-Fi is connected. Please turn it off and connect with device AP code (ASL_XXXX).

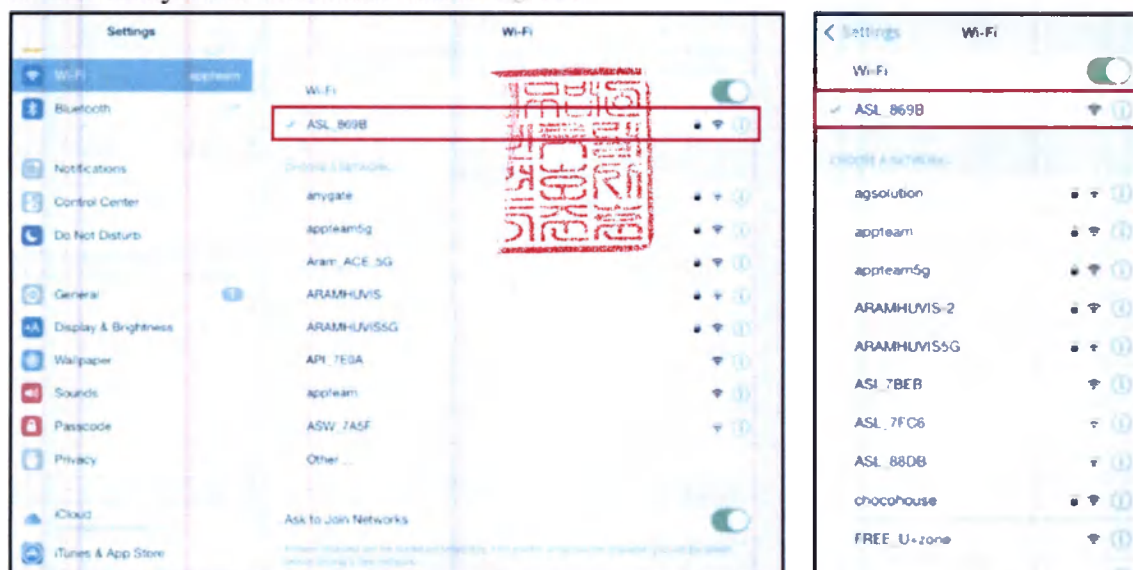6. The analyzer is connected with AP code

7. Then, run “Solutionist” program

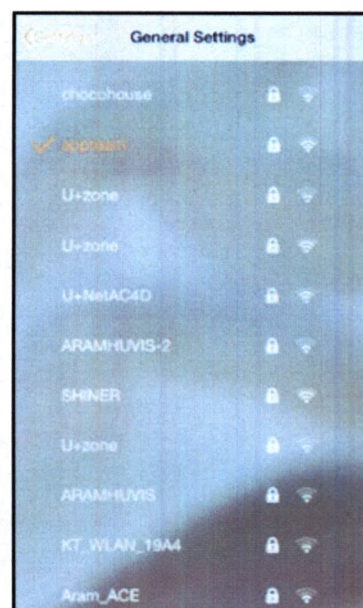
8. Then go to the settings again. The selected AP will be checked as left. The Wi-Fi combo is done.

Program Settings

1. General Settings

Change the language, analysis mode and etc.


2. Analysis Settings


Set up the skin type and analysis parameter

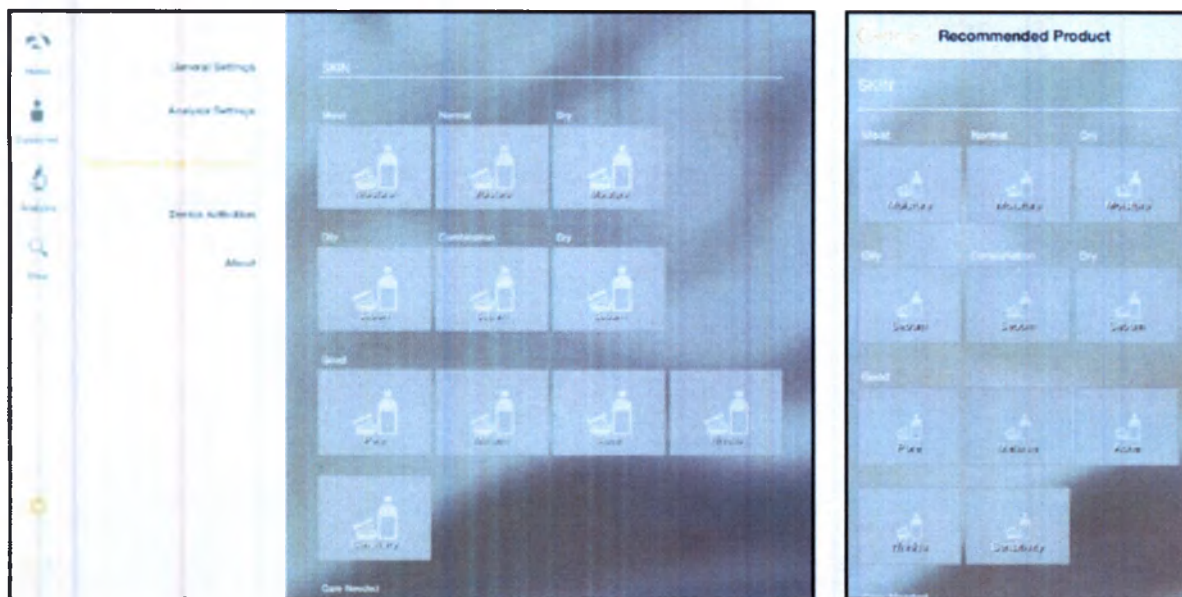Available for each analysis result

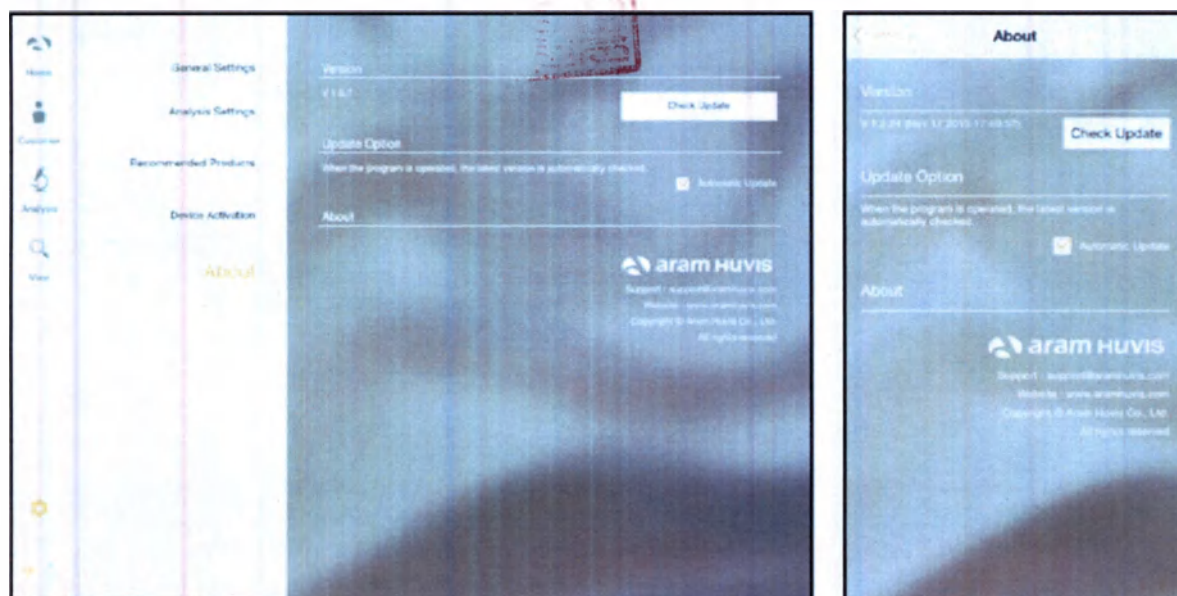Check the current version and update is available in Wi-Fi connection


Device Certification

1. You need to get a certification first before you use Solutionist program.

※ Device Activation” is only available in Wi-Fi connection.

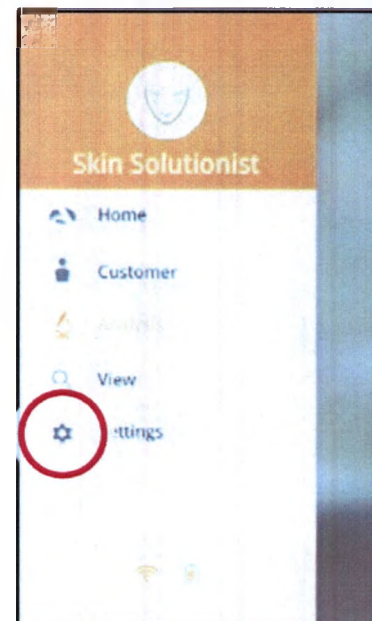
2. Start “Solutionist” program with ASL-300 device and select “Settings” on the main menu.


3. Among the menu, please select “Device Activation

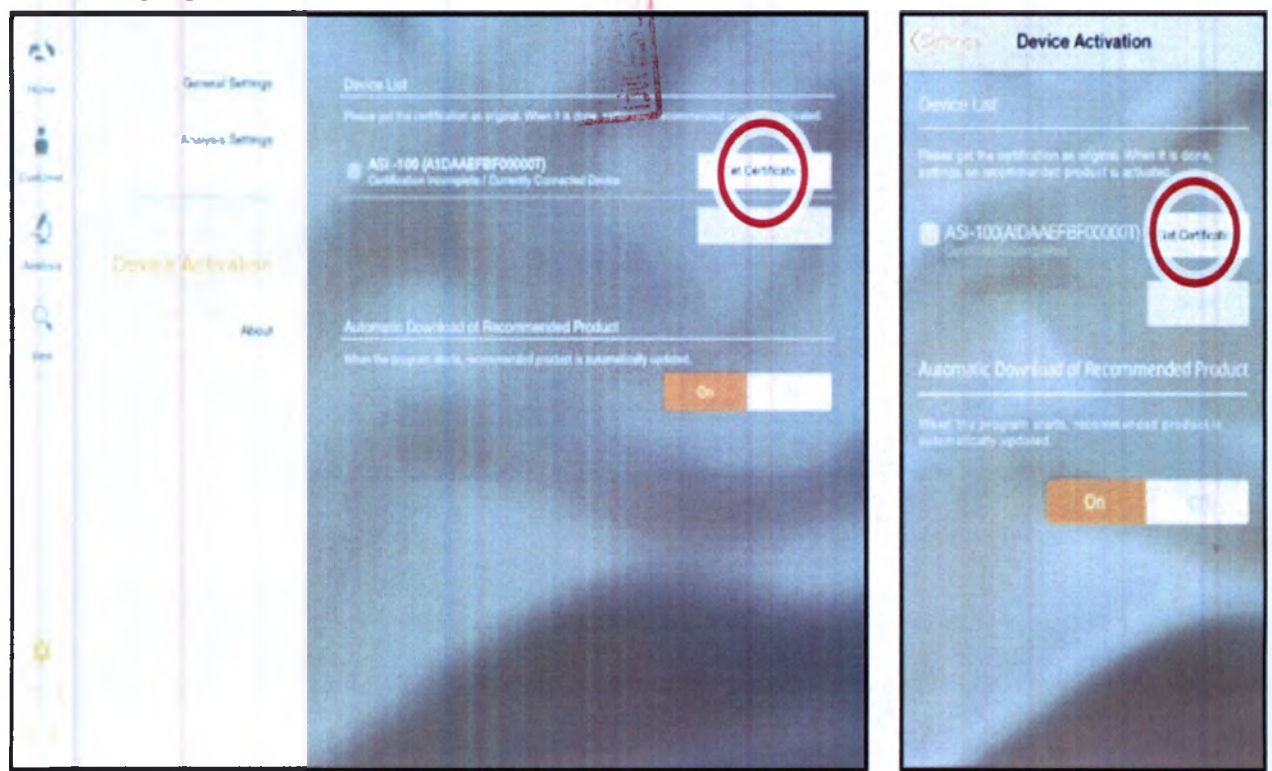
4. There is a device model name with serial code. Please select “Get Certification” to register the device in program

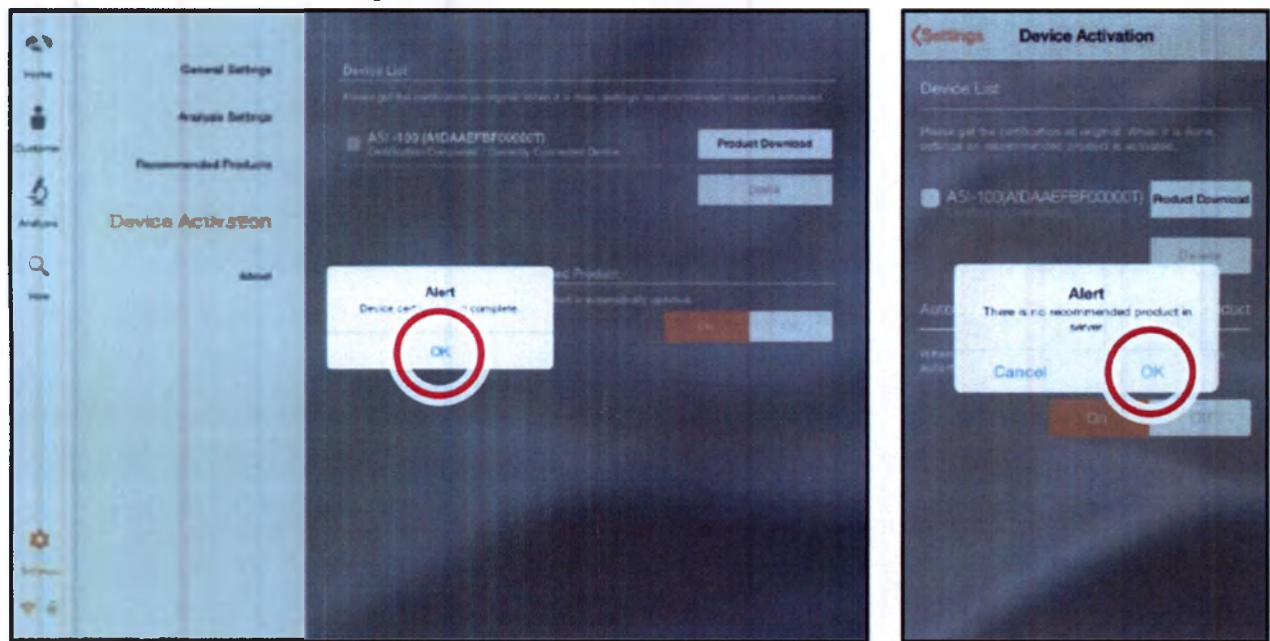
5. Device certification is complete. Please select “OK” button.

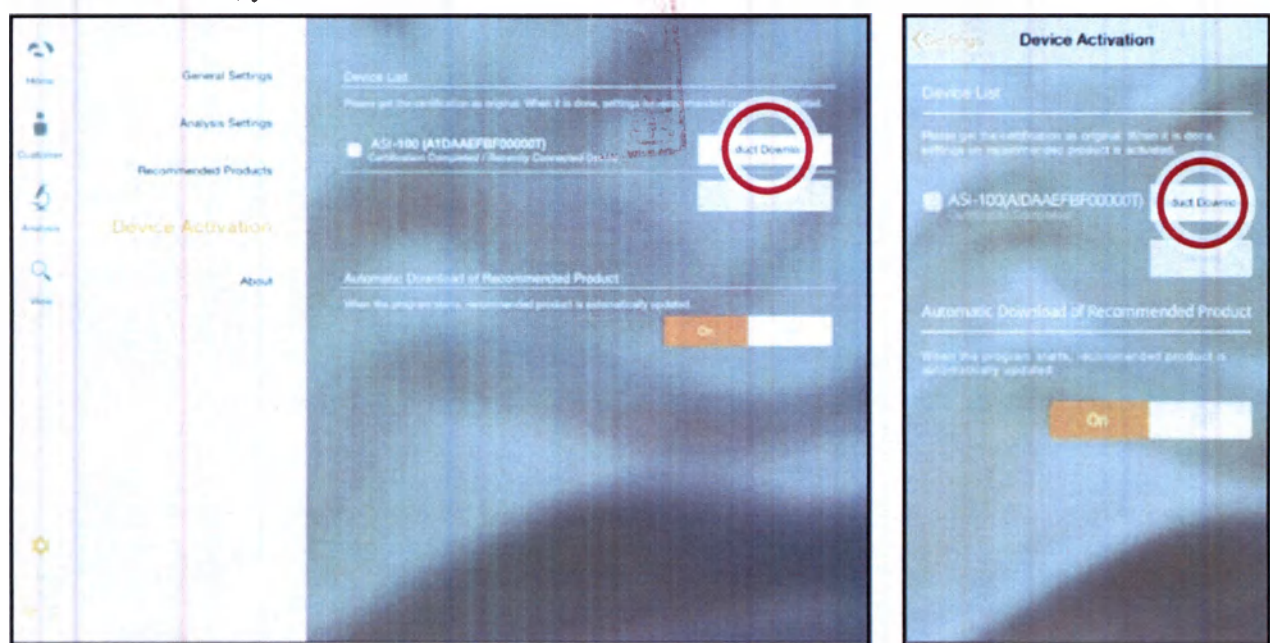6. When it is done, you could select “Product Download” button to load in server

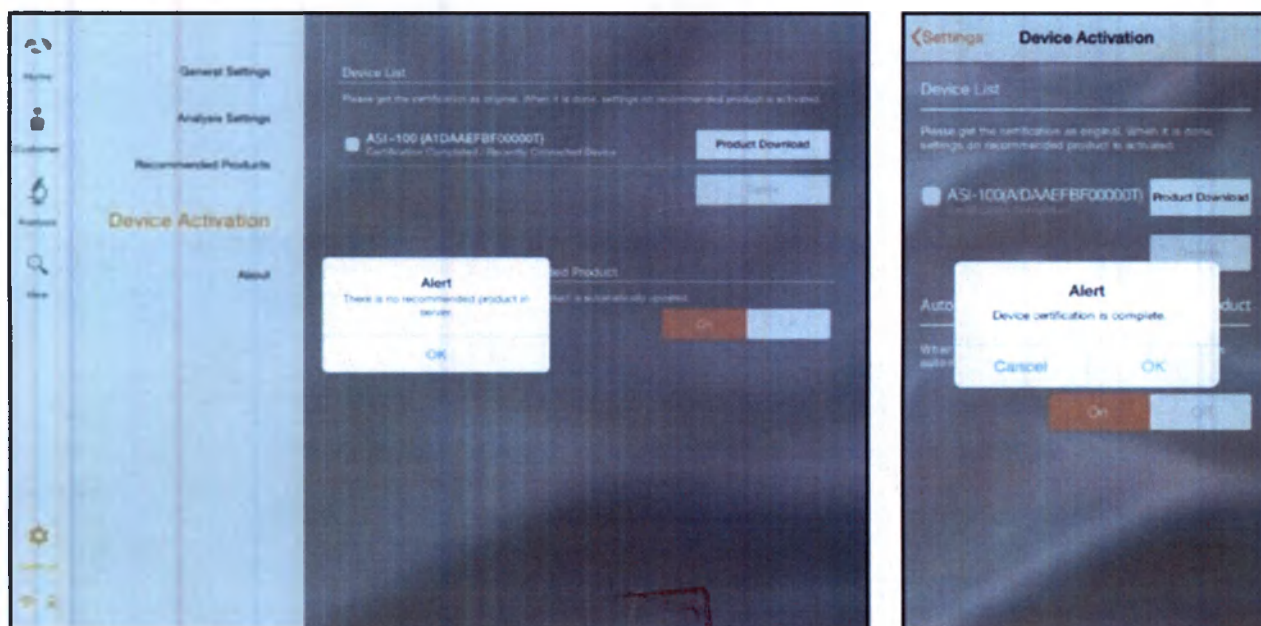
7. If you have the recommended product data in server, the product will be downloaded by clicking “Product download”.
- If you do not have the recommended product data, you can see the message as left image

Application


Start Analysis

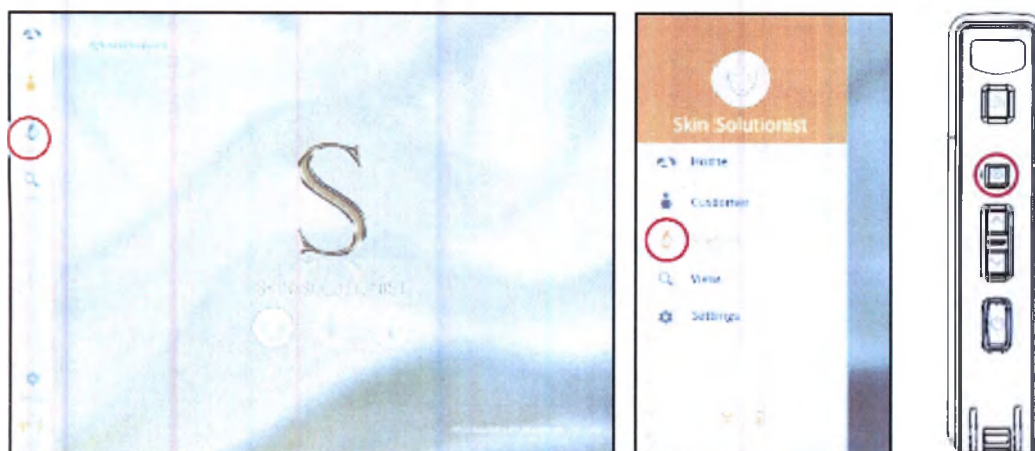- 1 Touch 'Analysis' to start.


Remote Control (Only Tablet)

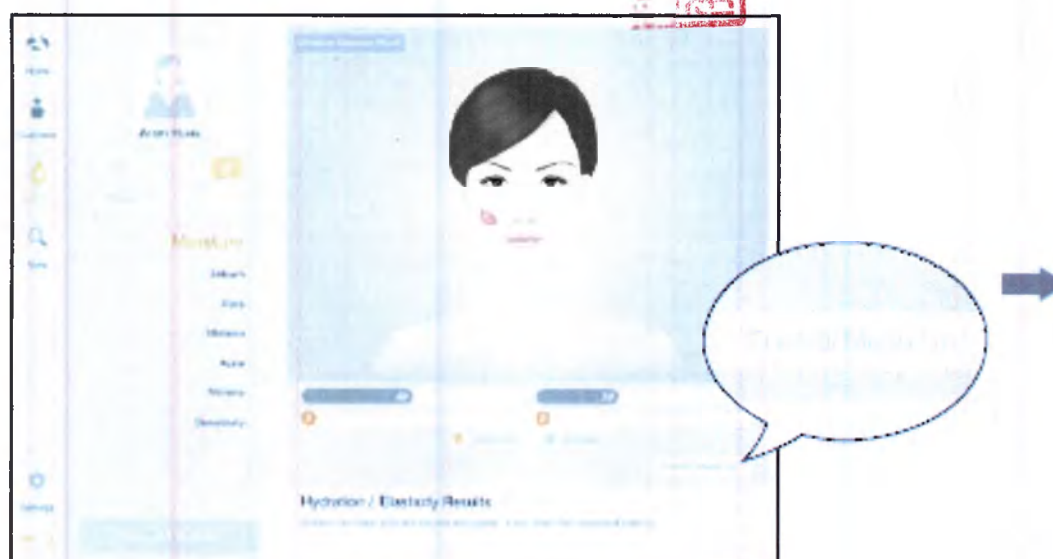- 1 During analysis, you can control the device in "Solutionist" program.
- 2 In screen, touch "Control Menu On" to activate "Remote".
- 3 You can make use of this function in all analysis pages.


CRM (Customer Relationships Management)

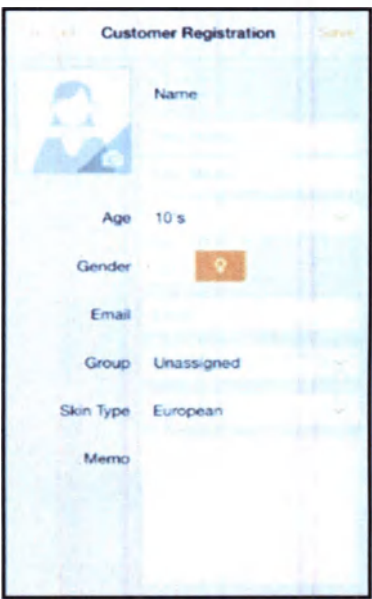
1. Input the information: - Full name, email address, gender and age group

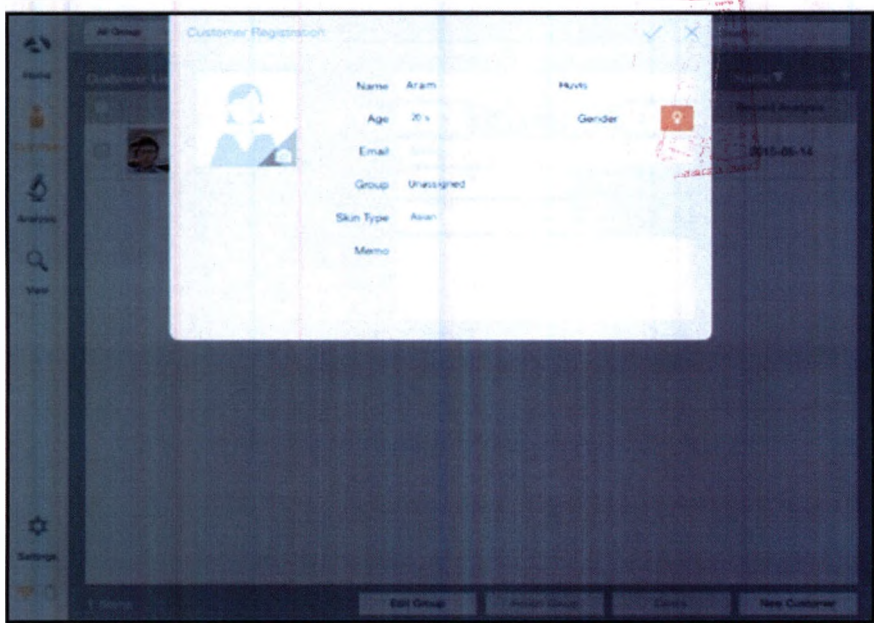
2. Take picture of customer's face with built-in camera or use the picture you already have in device

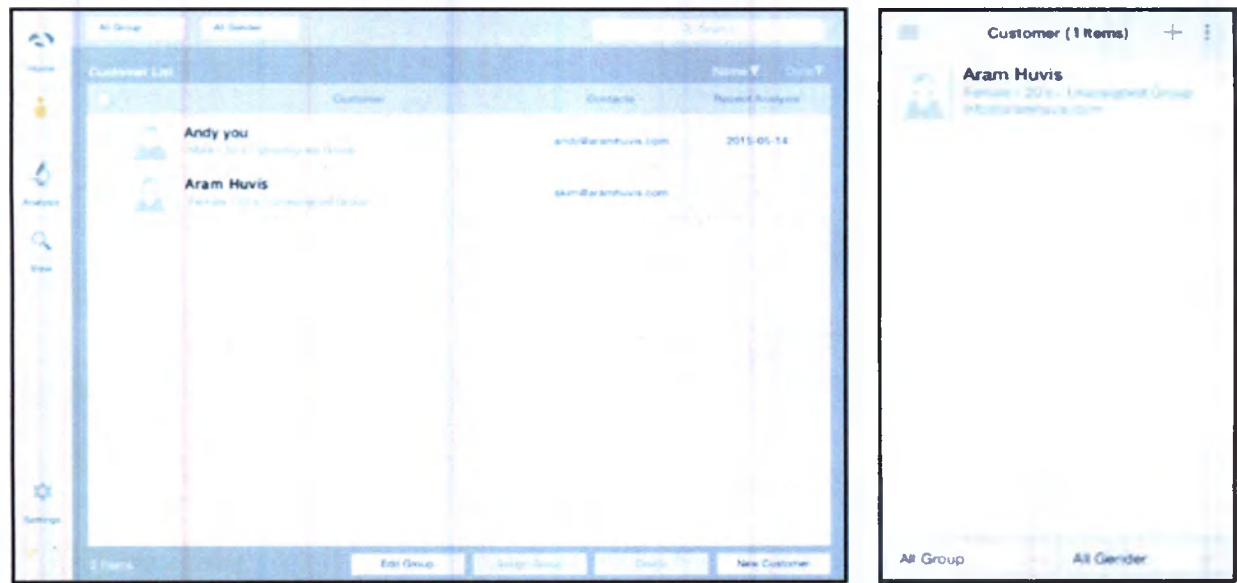3. Among the customer lists, select the client to check or start analysis

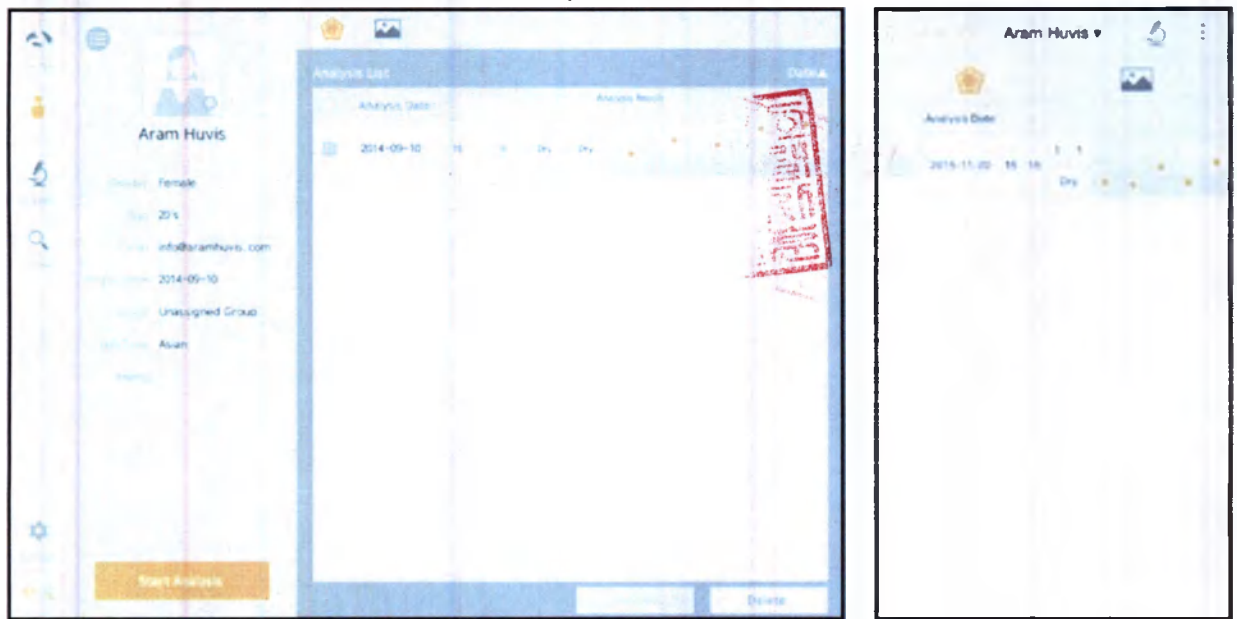4. Check the customer information and analysis result list

Skin Analysis / ASL-100 Lens

For the face skin analysis the ASL-100 lens with the built-in moisture sensor and the software application Solutionist are used.

The full analysis of the face skin will take about 10-15 minutes. The analysis includes the following tests: the determination of skin hydration, sebum (type of the skin according the quantity of sebum), skin pores size, quantity of brown spot, acne (under UV-light), wrinkles status, skin sensitivity.

Moisture Analysis

1. Press "capture" button and when you hear "beep" sound, please put the sensor onto your skin and wait until you hear "beep" sound again

Moisture sensor

2. The result will appear
To analyze the next step, press the below button

or

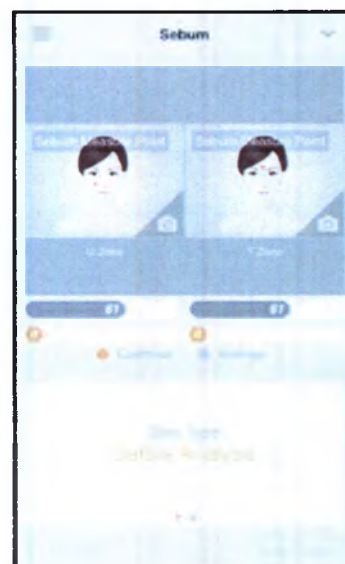
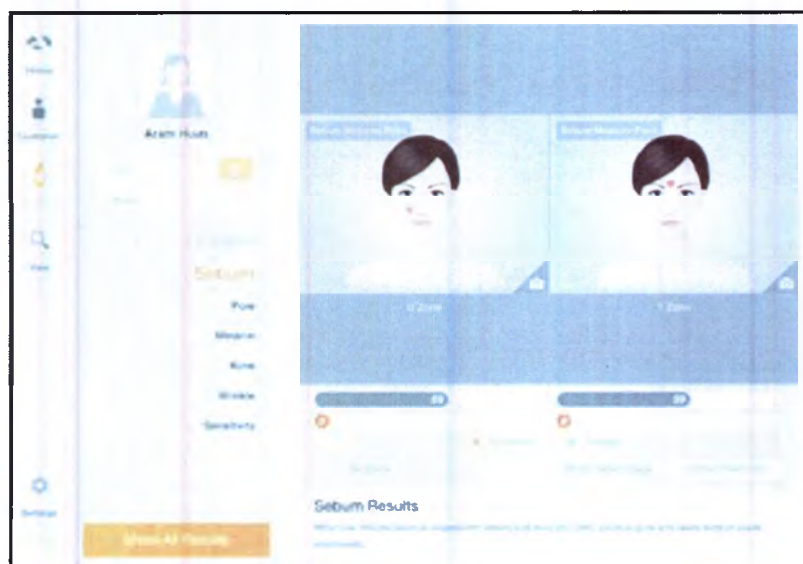
Sebum

1. Put the sebum paper onto the U-zone on the cheek, then press it for 2~3 seconds



U-zone



2. Place camera on the pink area and take picture. (U-zone)

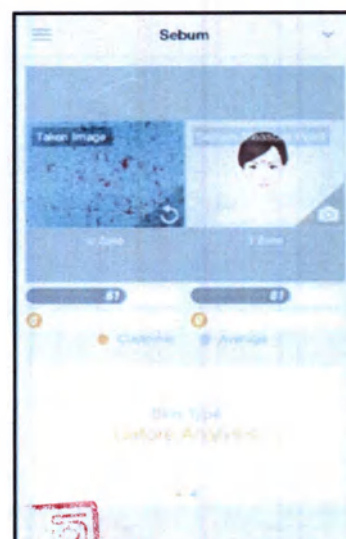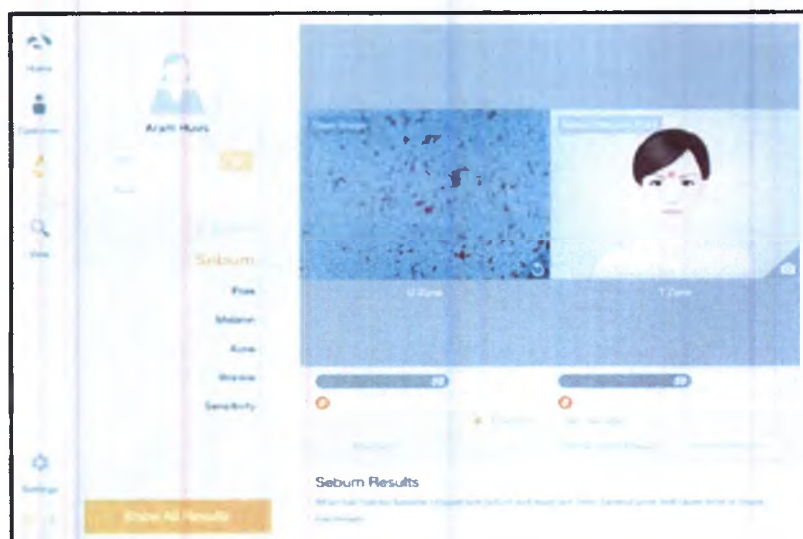Sebum paper

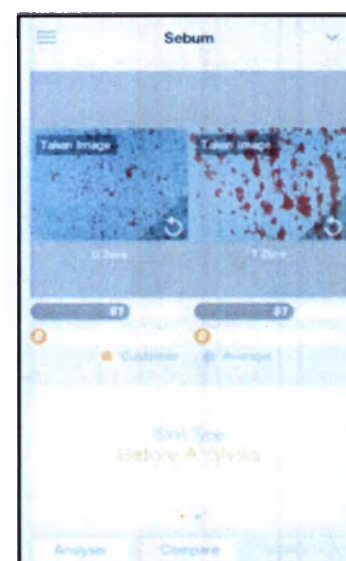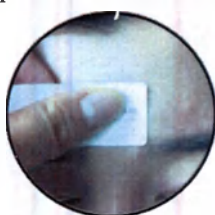
3. Put the sebum paper onto the T-zone on face, then press it for 2 ~3 seconds. Place camera on the pink area and take picture.(T-zone)

4. The result will appear.
To analyze the next step, press the below button.


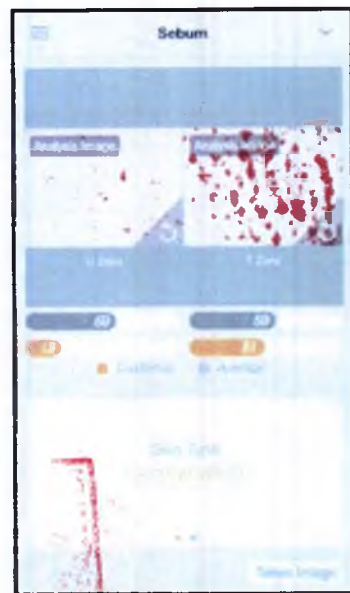
or




Pore

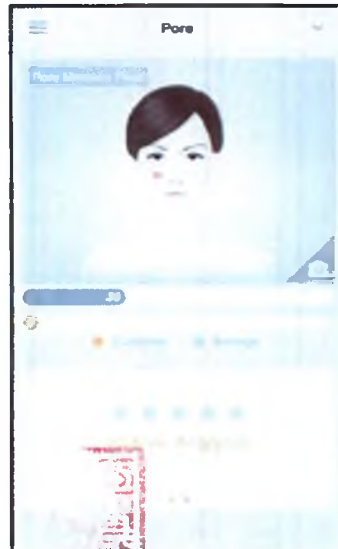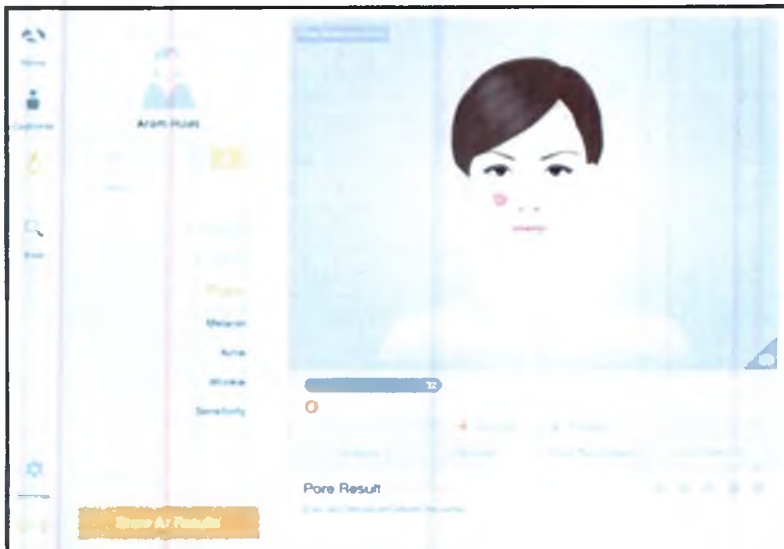
1. Press the button , to activate camera mode

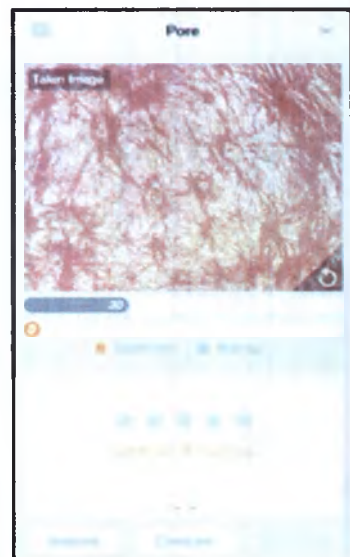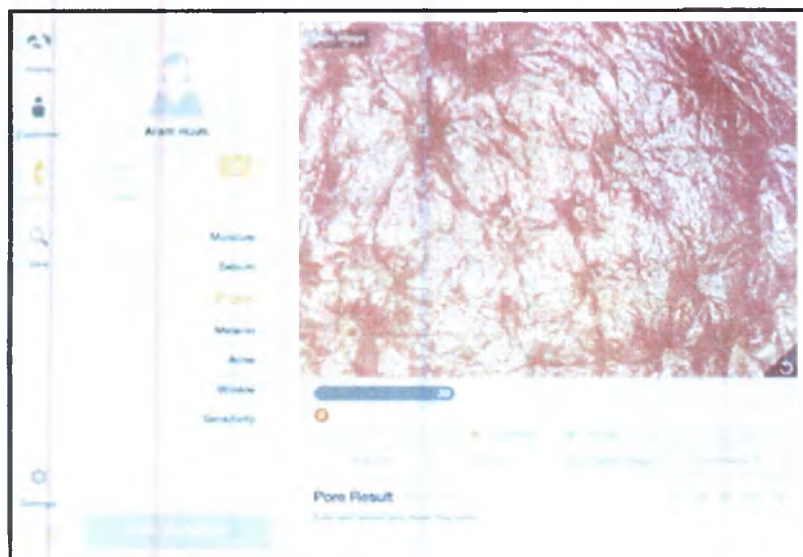
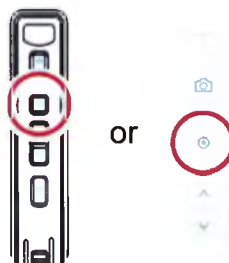
2. Put the camera onto the skin and press  button to capture the image. Then press button  to analyze it

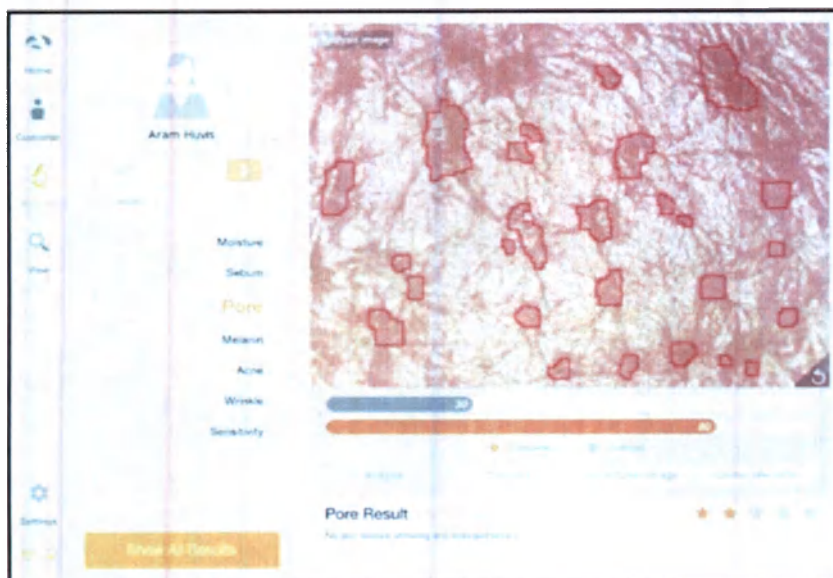3. The result will appear. Select "Compare" button to check the sample images as 4 .
To analyze the next step, press the below button

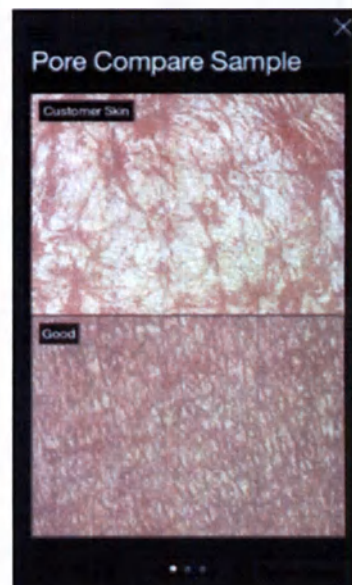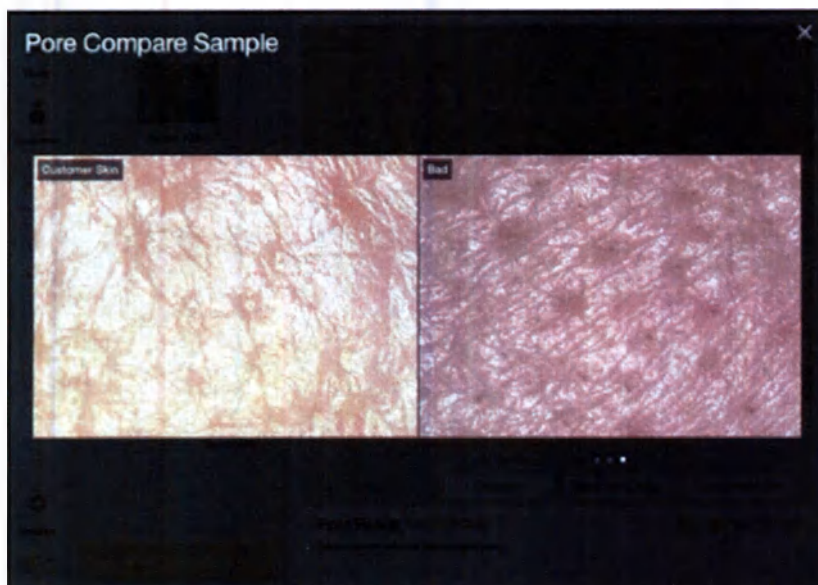
4. Compare your image with 3 sample images.
Good
Normal
Bad.




Melanin


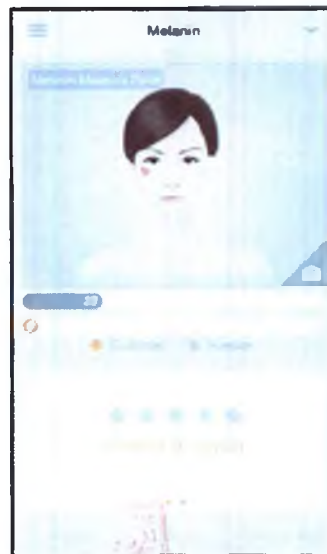
1. Press the button  , to activate camera mode

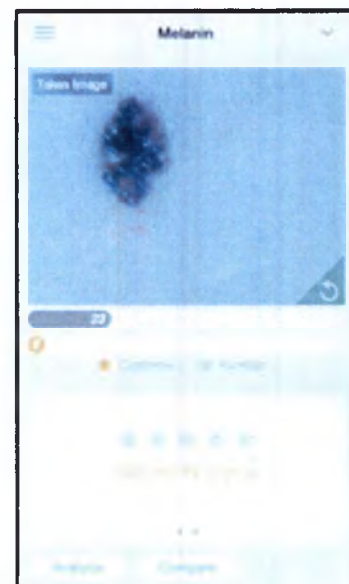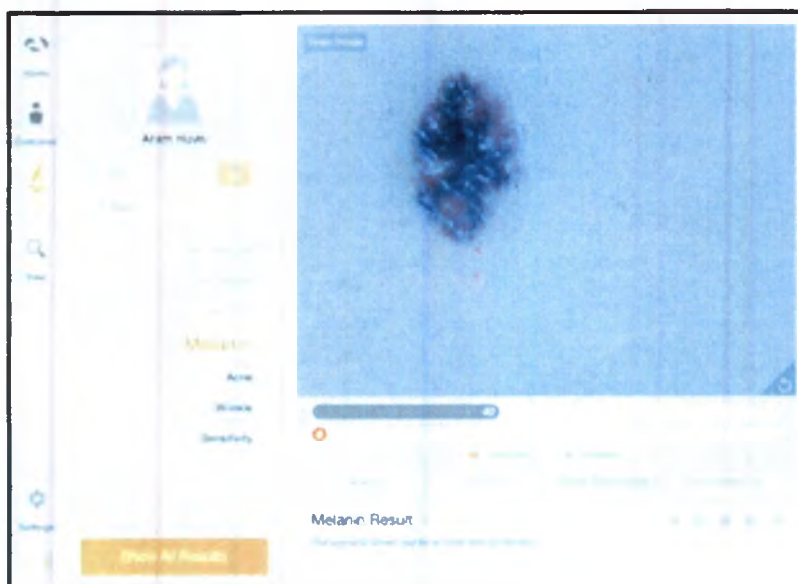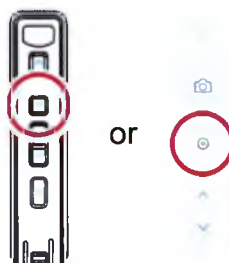
2. Put the camera onto the skin and press  button to capture the image. Then press button  to analyze it

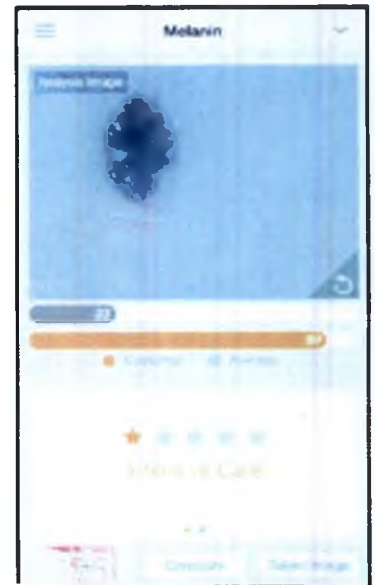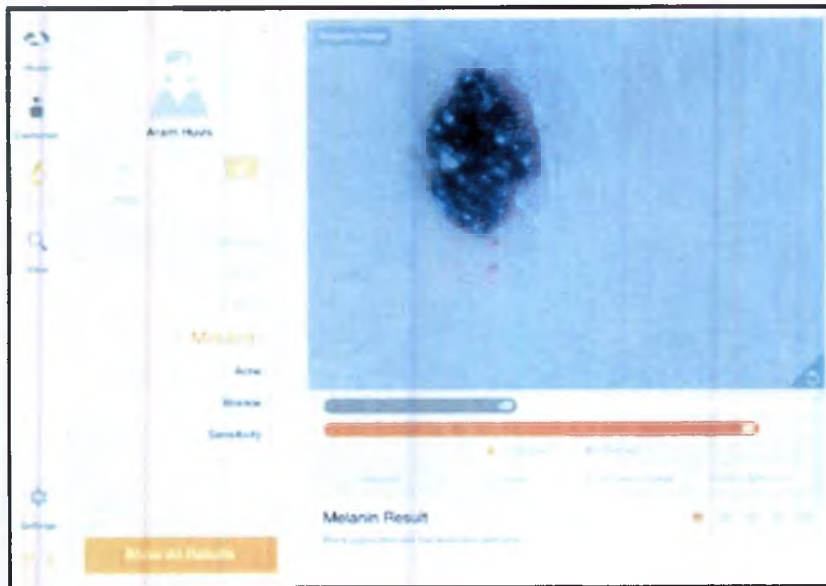3. The result will appear. Select "Compare" button to check the sample images as 4 .
To analyze the next step, press the below button

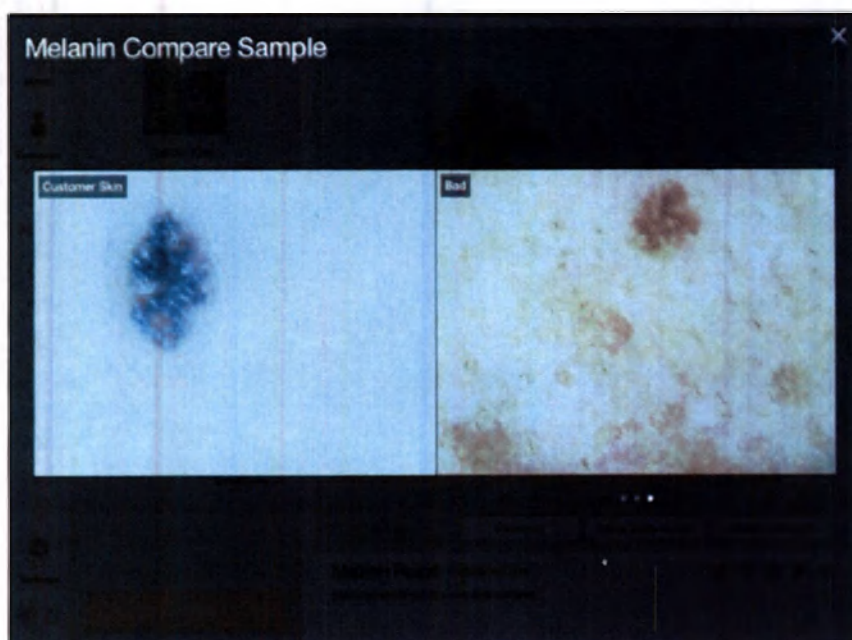4. Compare your image with 3 sample images.
Good
Normal
Bad.




Acne


1. Press the button  , to activate camera mode

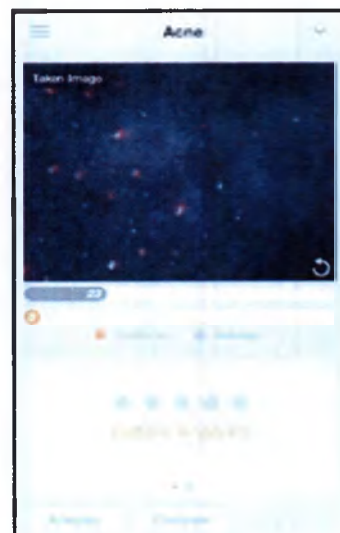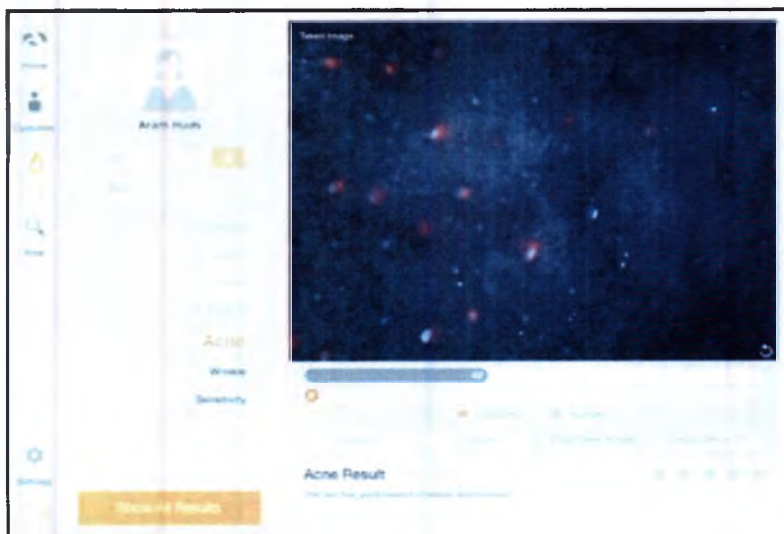
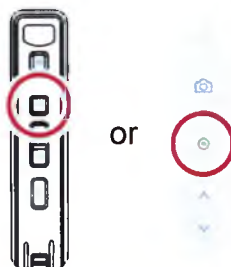
2. Put the camera onto the skin and press  button to capture the image. Then press button  to analyze it

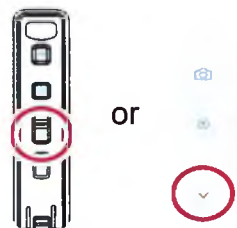3. The result will appear. Select "Compare" button to check the sample images as 4 .
To analyze the next step, press the below button

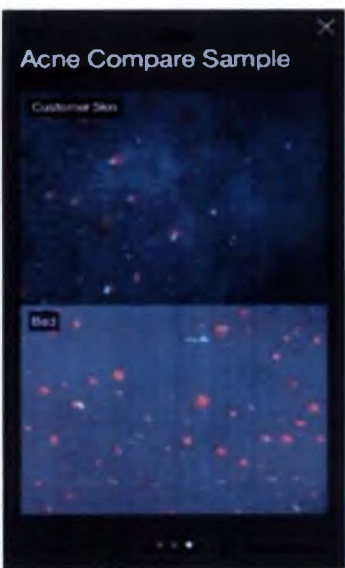
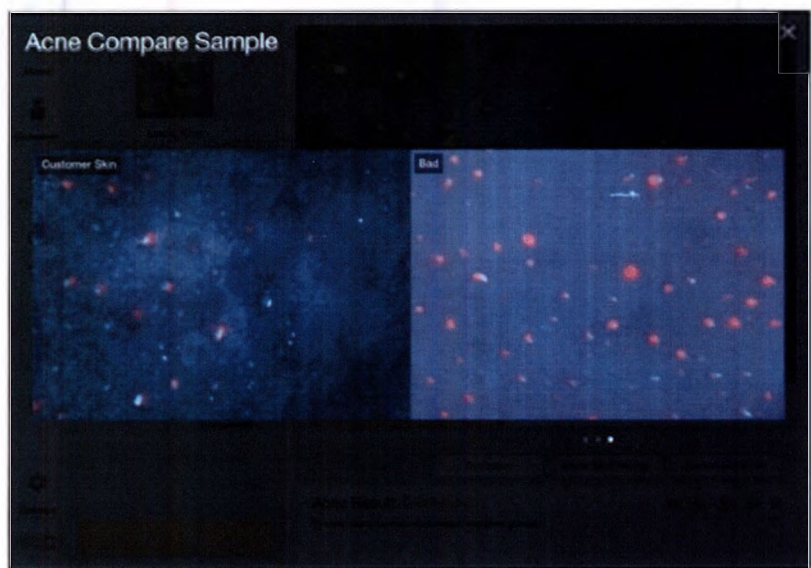
4. Compare your image with 3 sample images.
Good
Normal
Bad.




Wrinkles

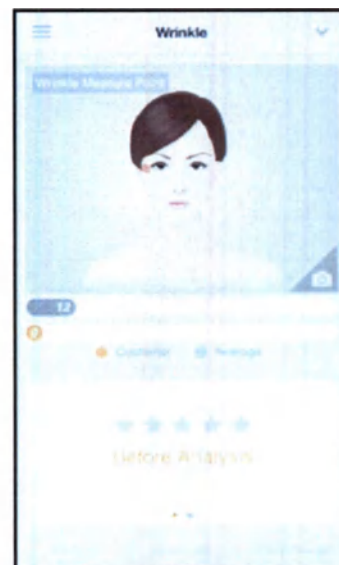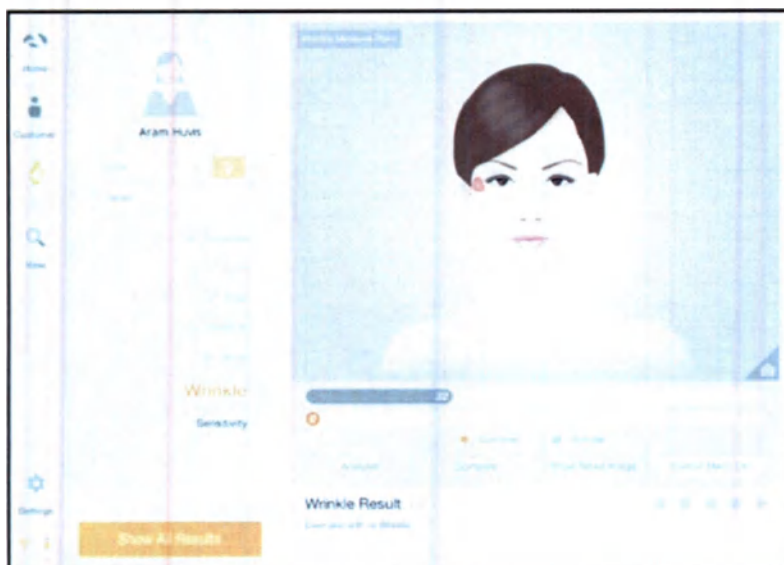1. Press the button , to activate camera mode

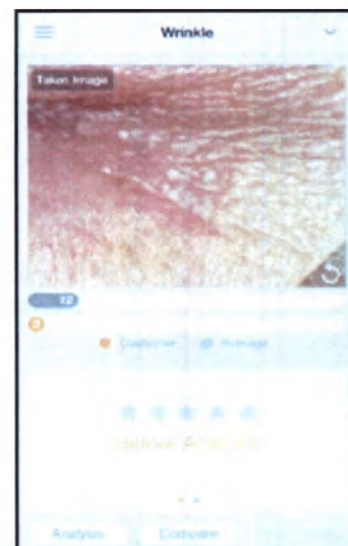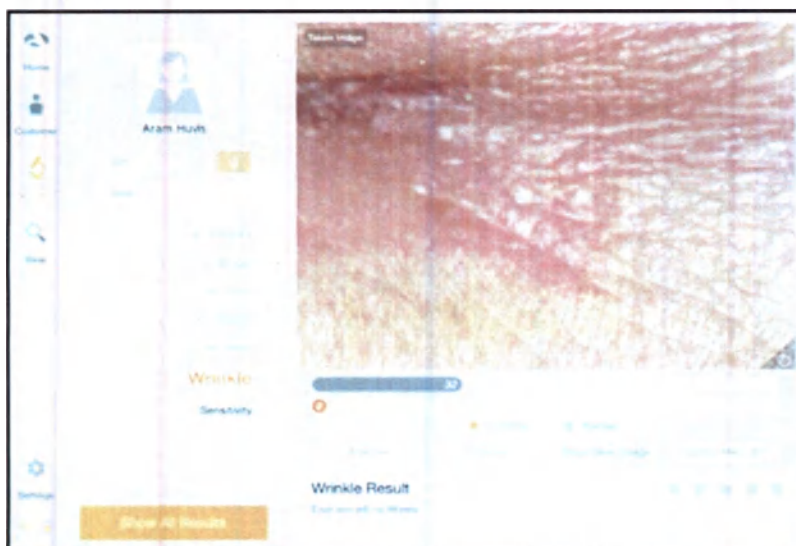
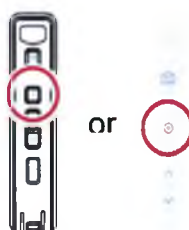
2. Put the camera onto the skin and press  button to capture the image. Then press button  to analyze it

3. The result will appear. Select "Compare" button to check the sample images as 4 .

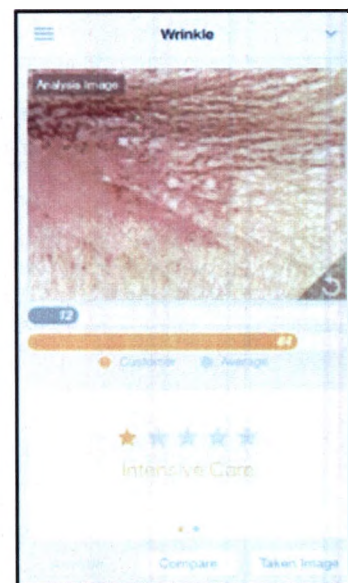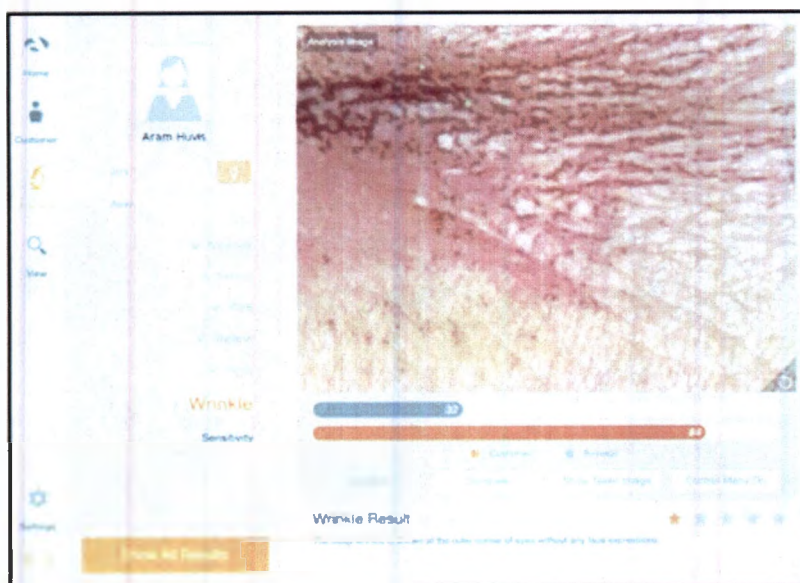To analyze the next step, press the below button


4. Compare your image with 3 sample images.

Good


Normal

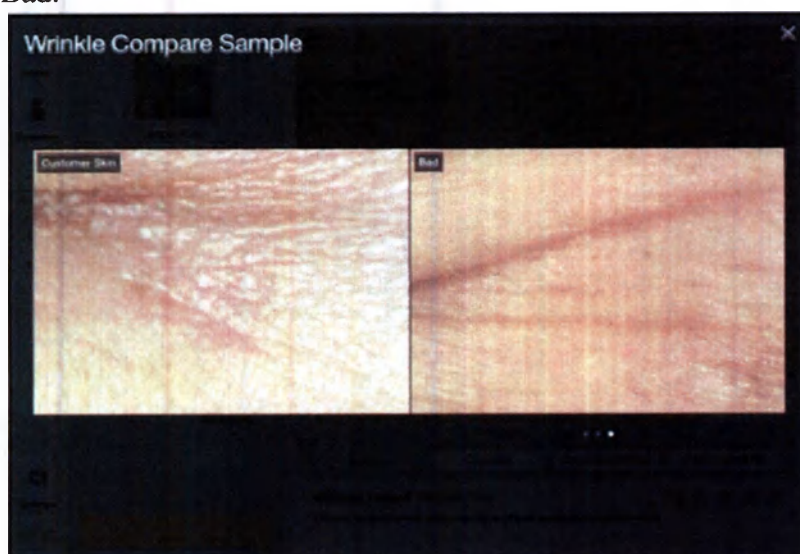Bad.


Sensitivity



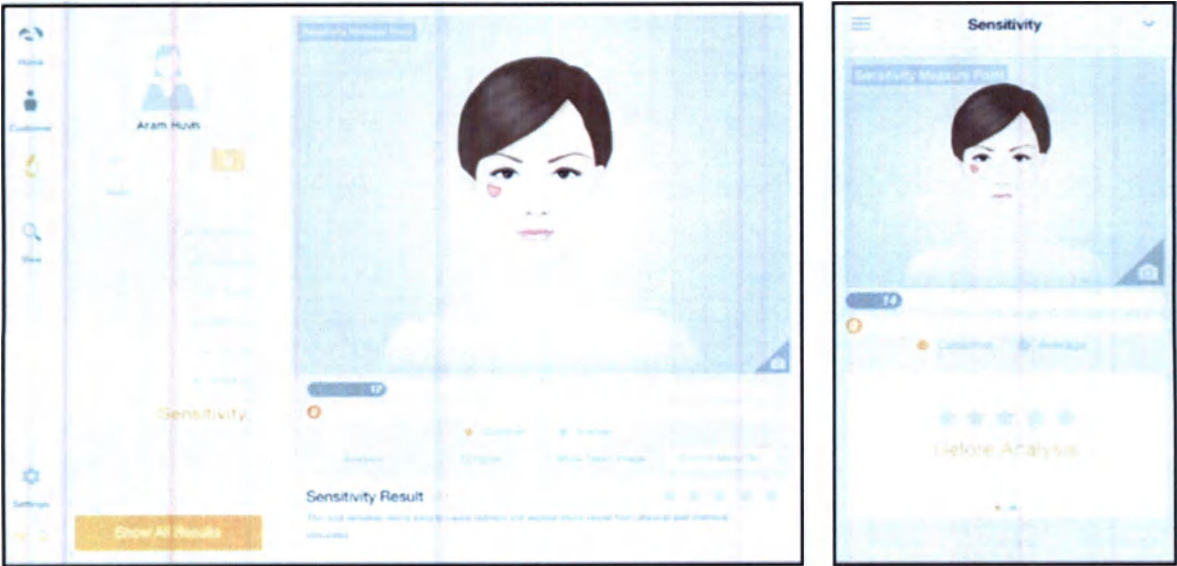
1. Press the button  , to activate camera mode

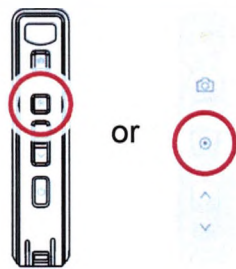2. Put the camera onto the skin and press  button to capture the image. Then press button  to analyze it

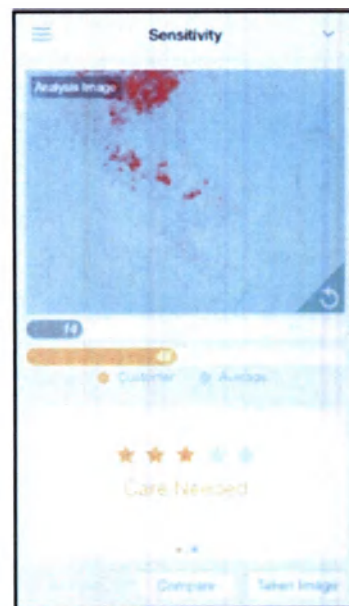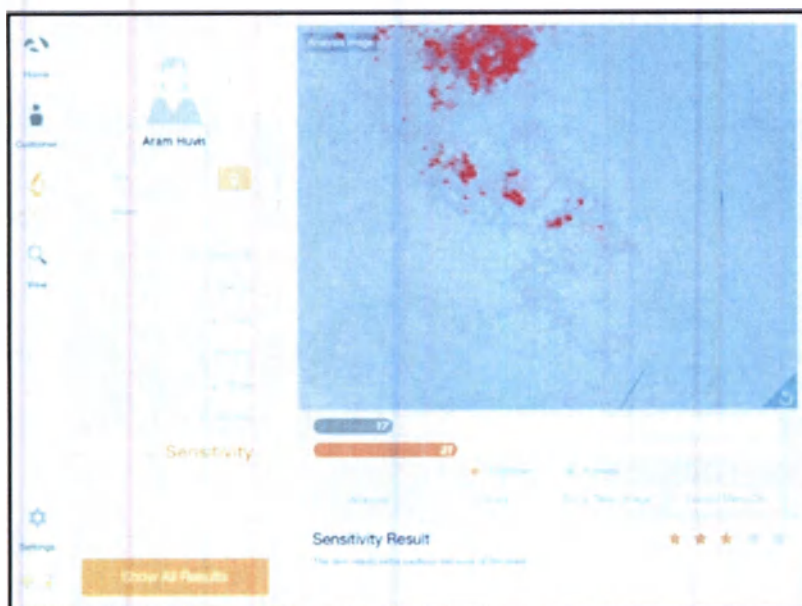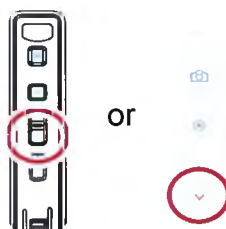3. The result will appear. Select "Compare" button to check the sample images as 4 .
To analyze the next step, press the below button


4. Compare your image with 3 sample images.

Good

Normal

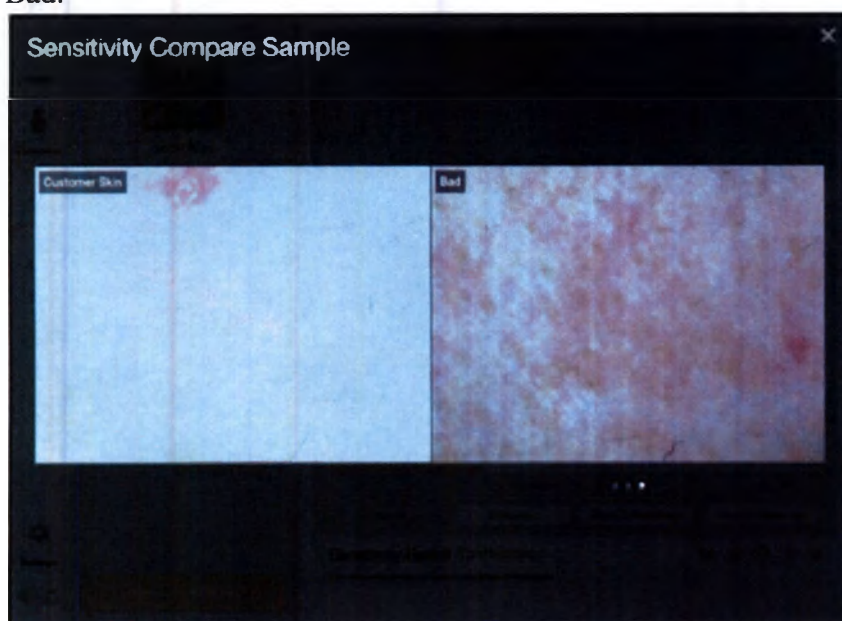Bad.



Hair Analysis / ASL-202 Lens

1. Push the Lens module change button.
2. Keeping the change button pushed, pull the ASL-100 lens module from the device body.
3. Mount the ASL-202 lens module at the device.

Hair Loss Status

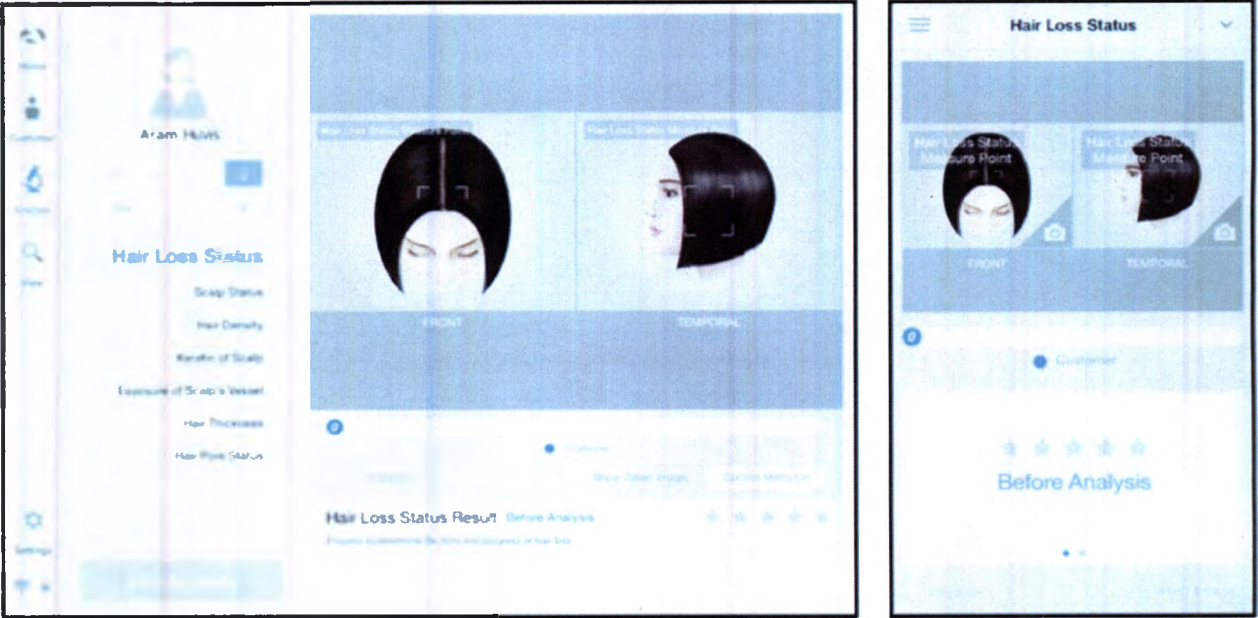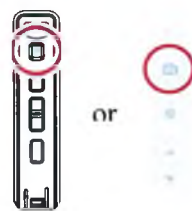1. Press the button  , to activate camera mode



2. Take a picture of front and temporal parts of a head by  button. Then press button  to analyze it

3. Then select the type which is similar with your hair loss shape.

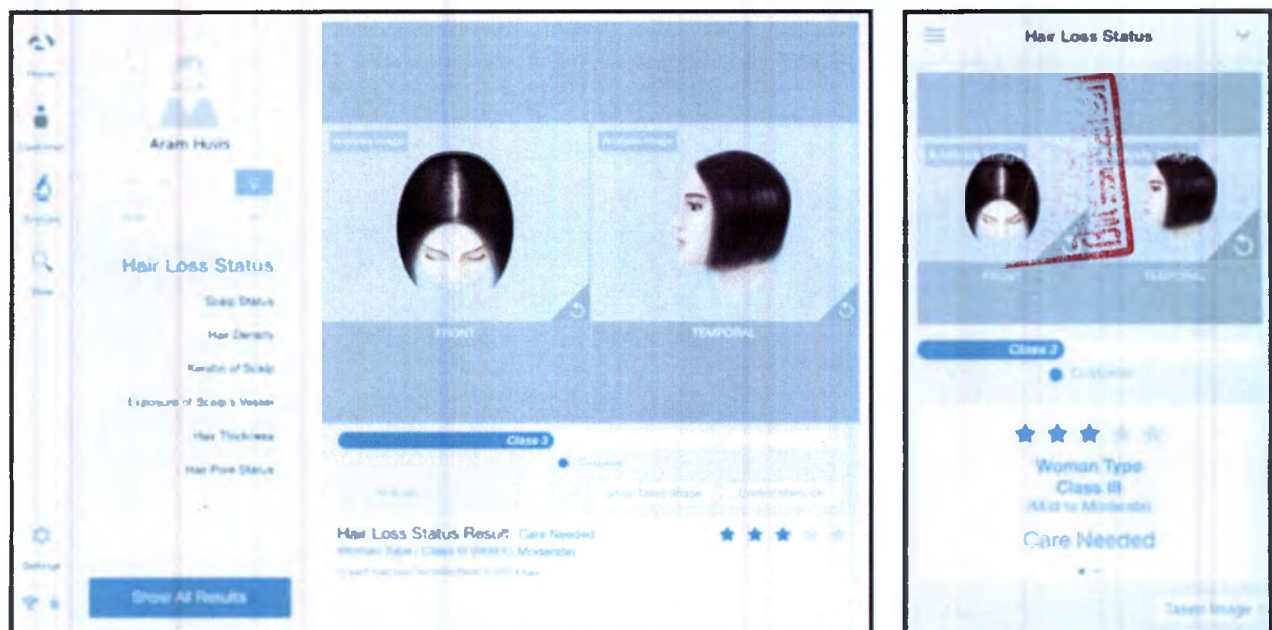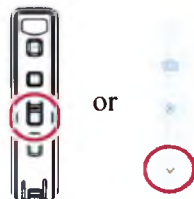
4. The result will appear. To analyze the next step, press the below button.


Scalp Status

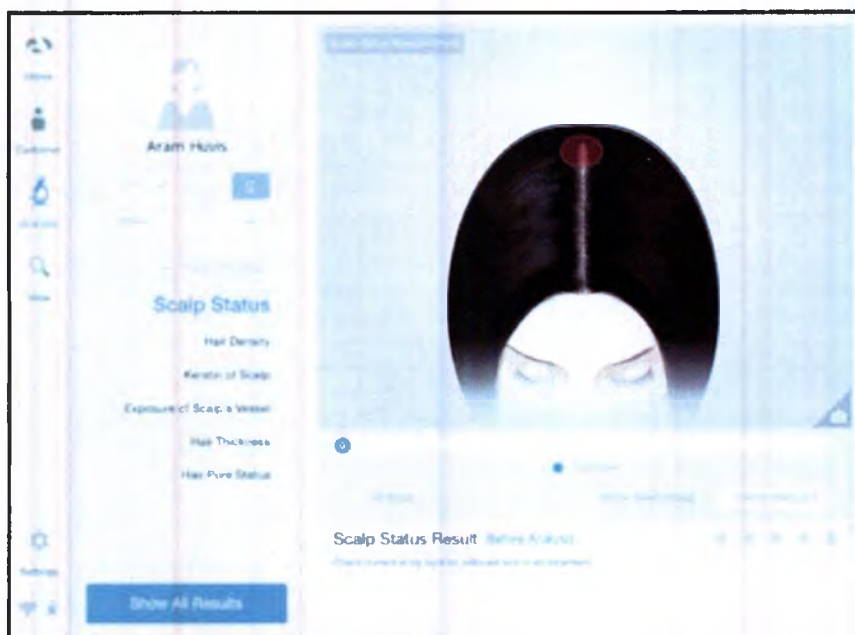
1. Press the button  , to activate camera mode


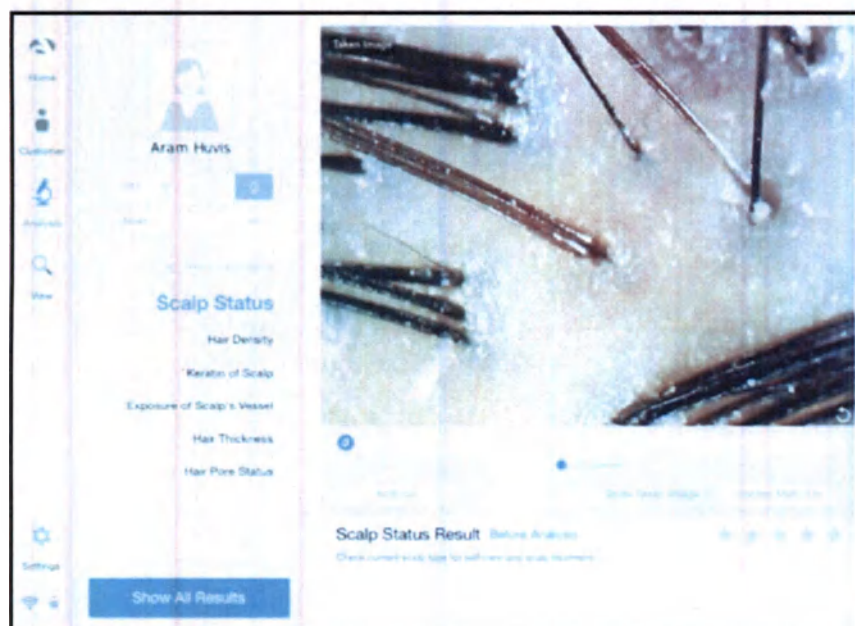
2. Take a picture of scalp by  button. Then press button  to analyze it

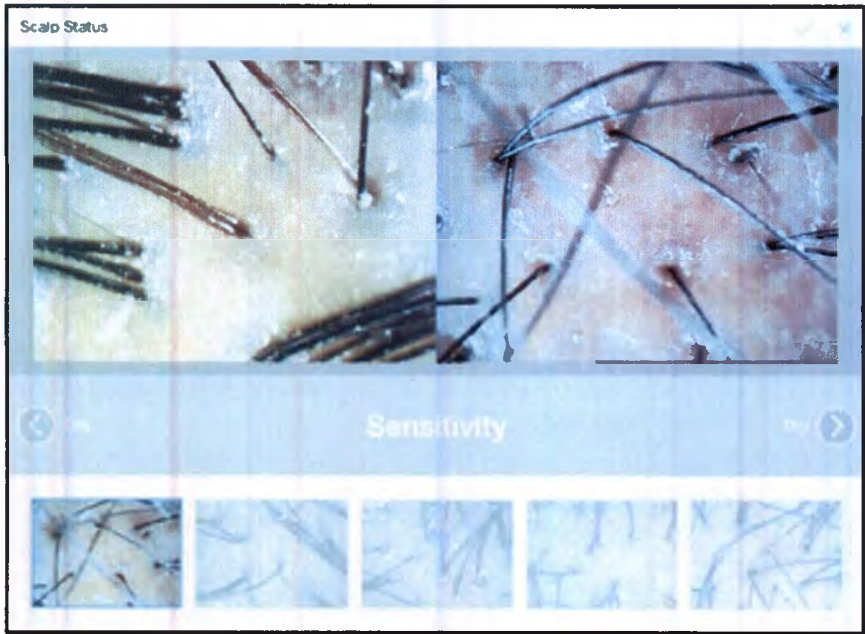3. Among the samples, choose the one which is similar with your image.

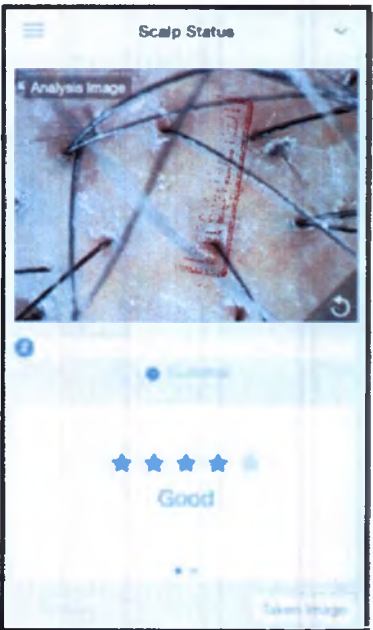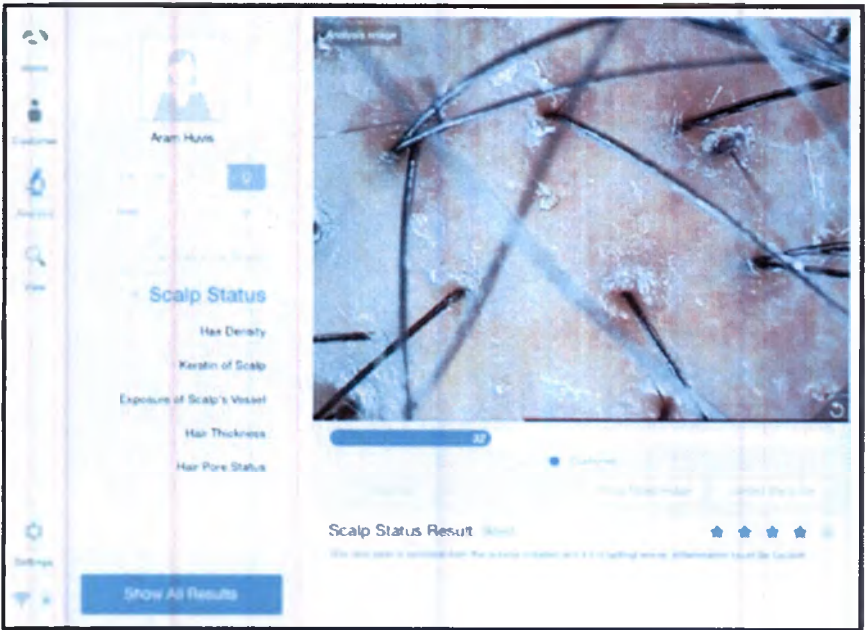4. The result will appear. To analyze the next step, press the below button.


Hair Density


1. Press the button  , to activate camera mode


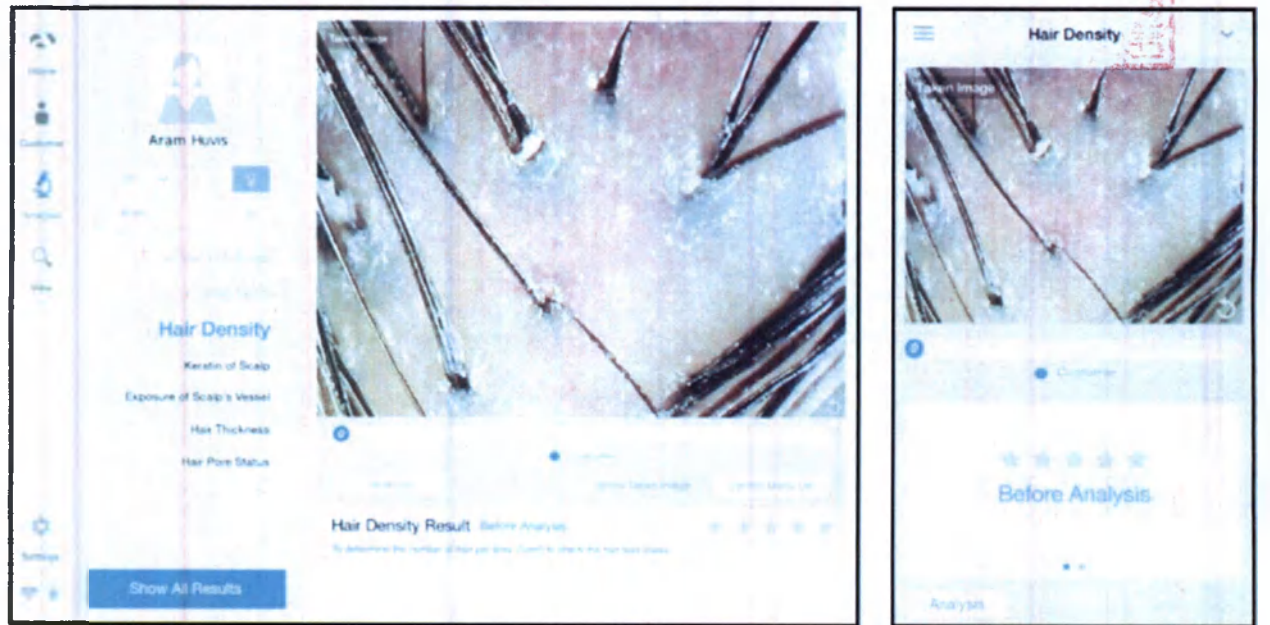
2. Take a picture of scalp by  button. Then press button  to analyze it

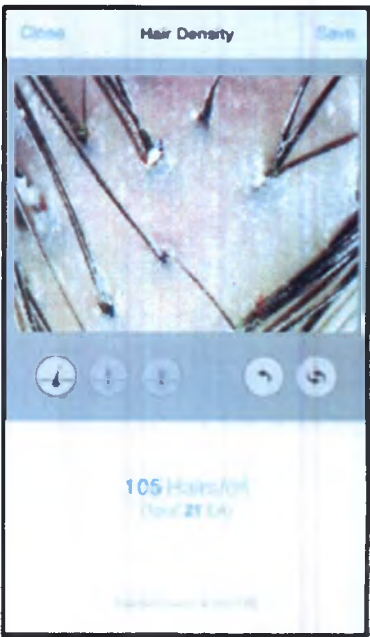3. Among the samples, choose the one which is similar with your image.

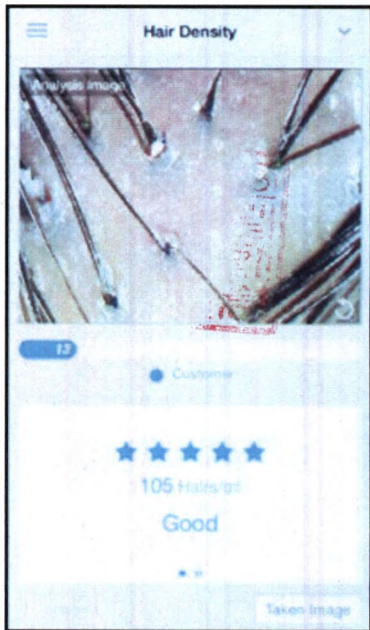
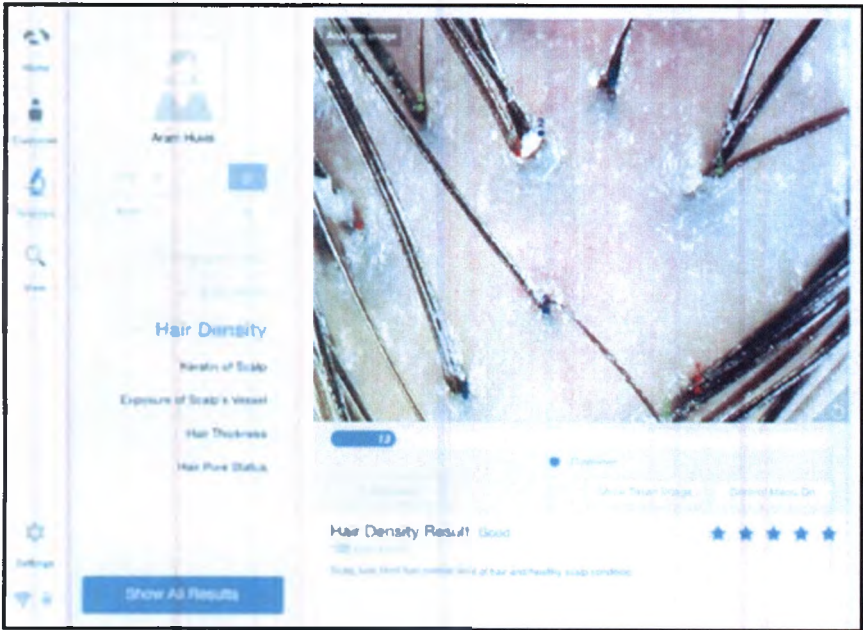
4. The result will appear. To analyze the next step, press the below button.




Keratin of Scalp

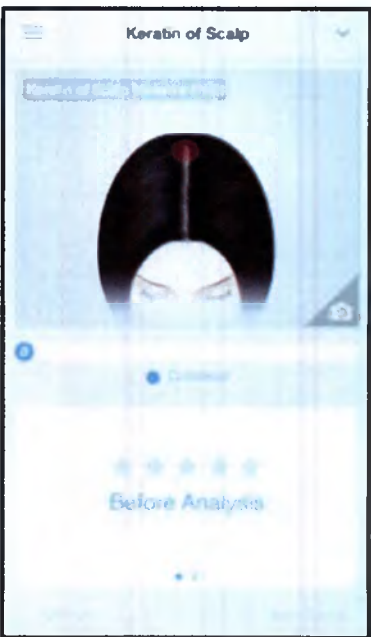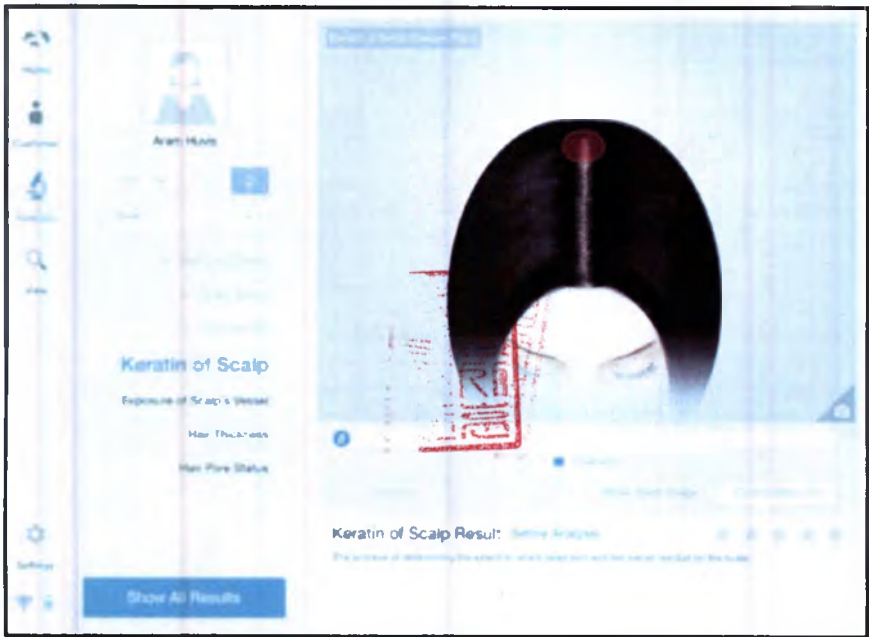
1. Press the button  , to activate camera mode

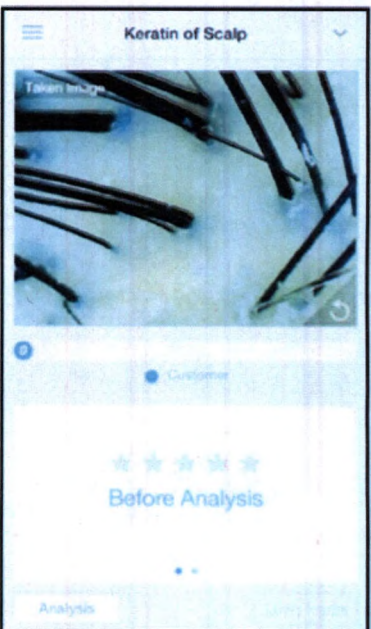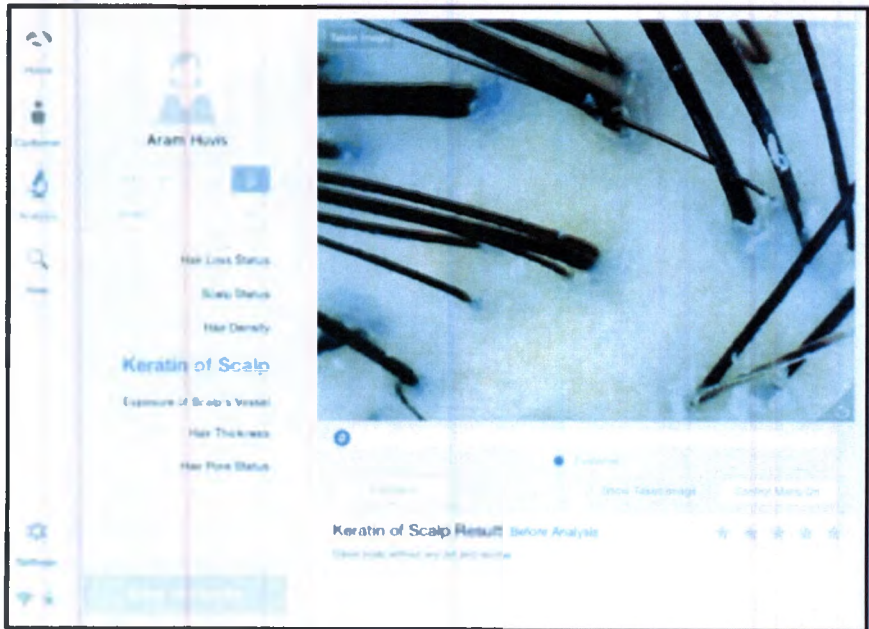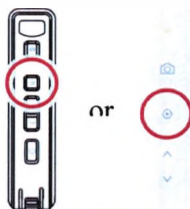
2. Take a picture of scalp by  button. Then press button  to analyze it

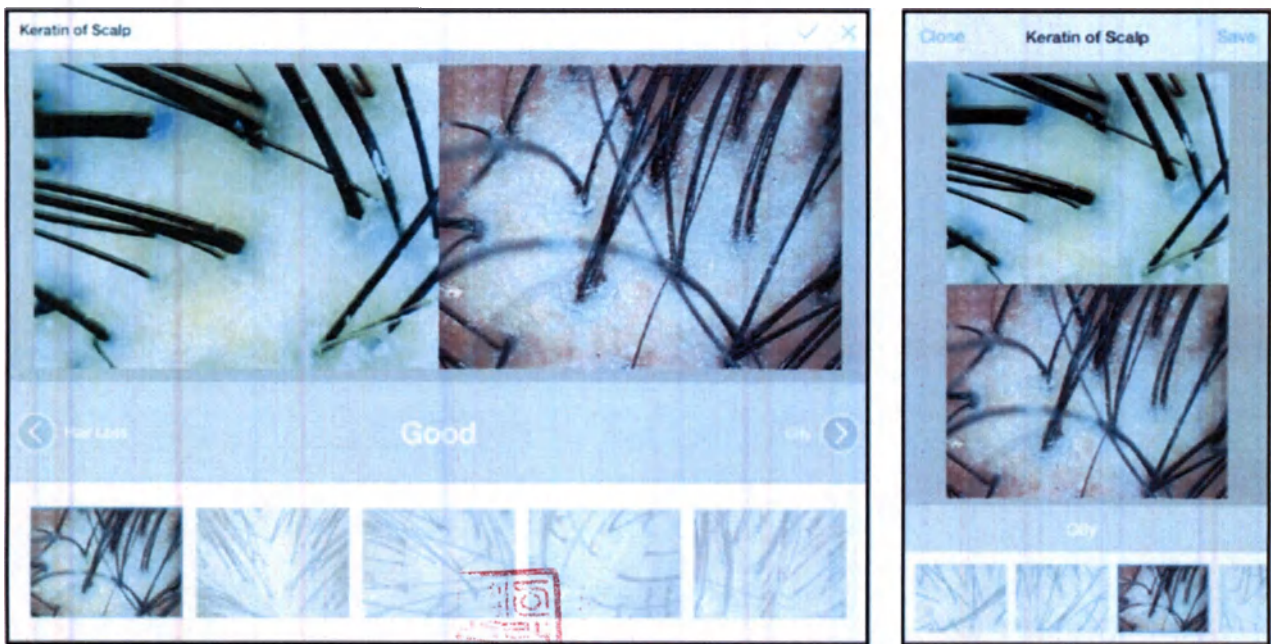3. Among the samples, choose the one which is similar with your image.

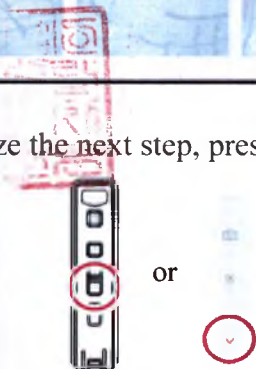4. The result will appear. To analyze the next step, press the below button.


Exposure of Scalp's Vessels



1. Press the button  , to activate camera mode



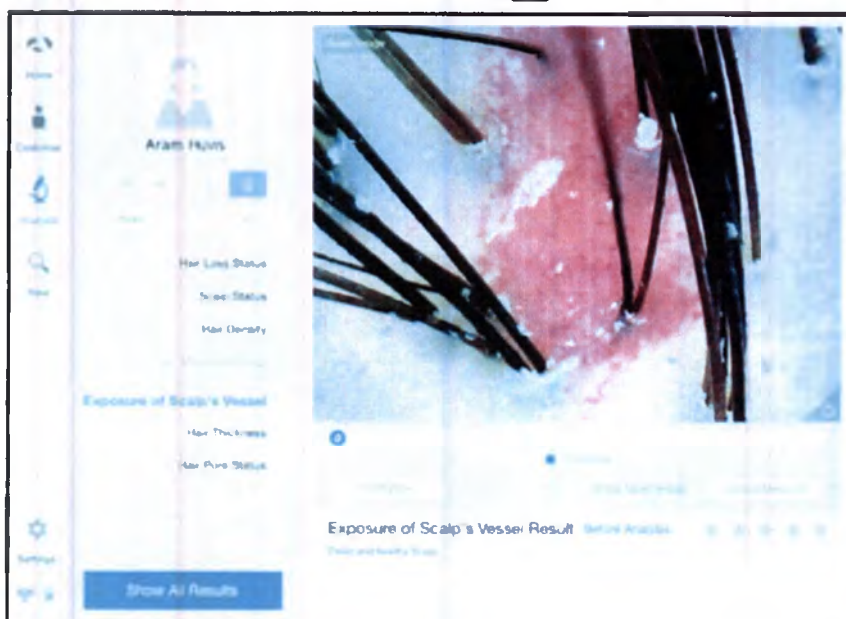
2. Take a picture of scalp by  button. Then press button  to analyze it

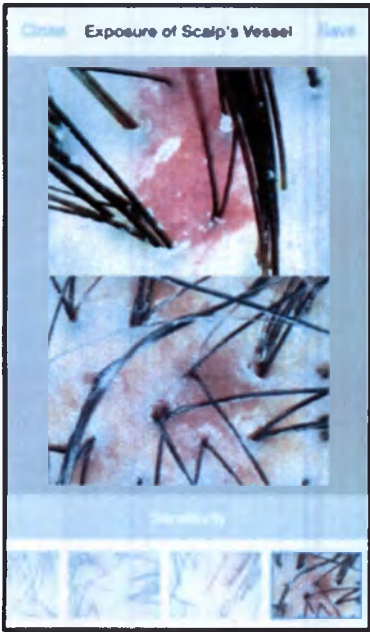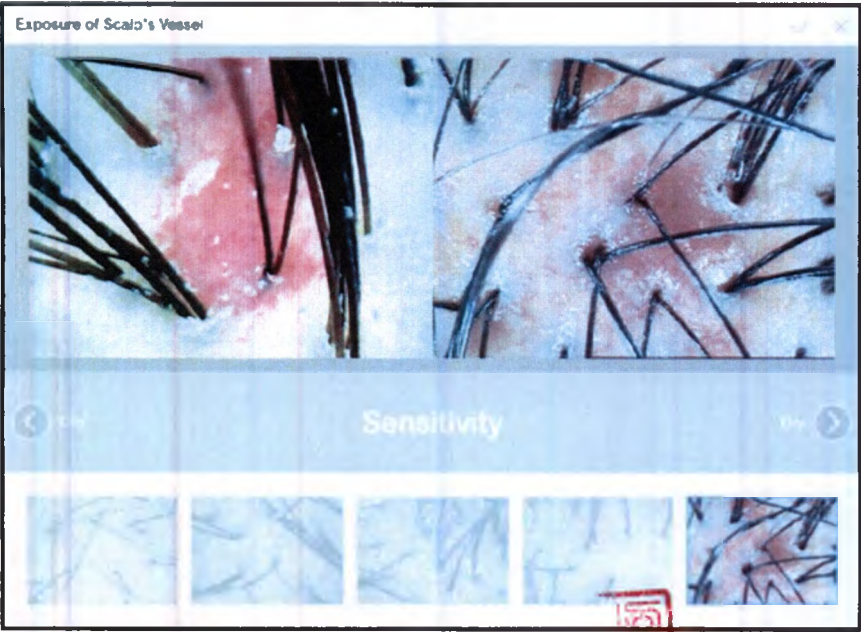3. Among the samples, choose the one which is similar with your image.

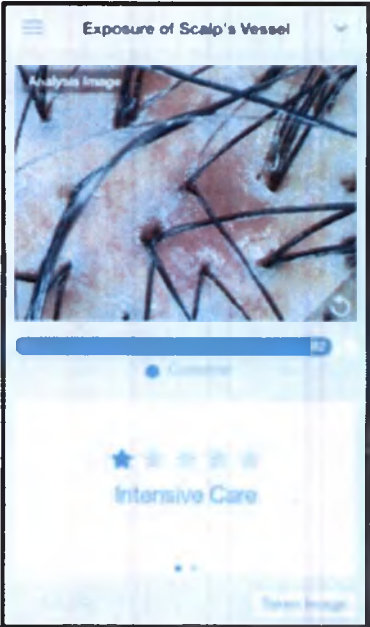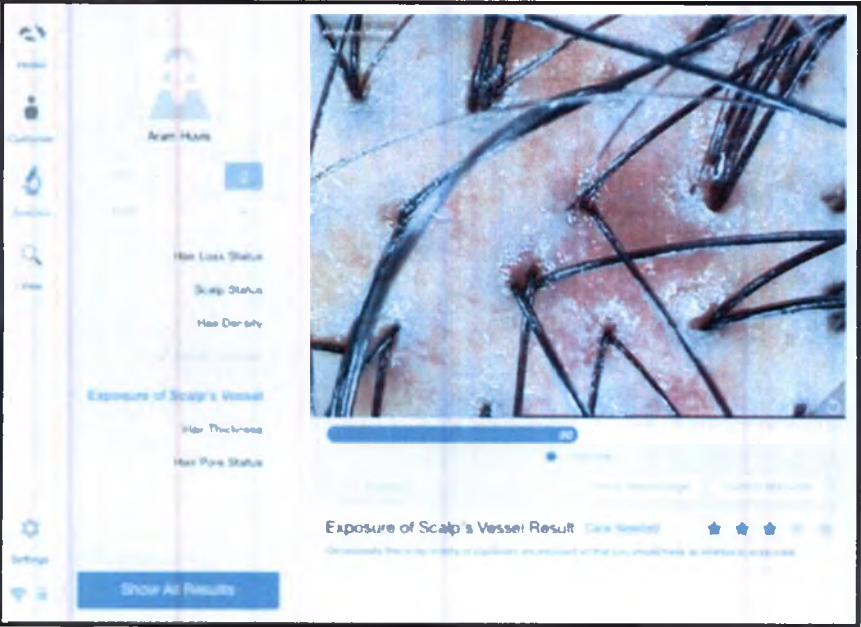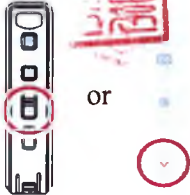4. The result will appear. To analyze the next step, press the below button.


Hair Analysis / ASL-203 Lens

1. Push the Lens module change button.
2. Keeping the change button pushed, pull the ASL-202 lens module from the device body.
3. Mount the ASL-203 lens module at the device.


Hair Thickness

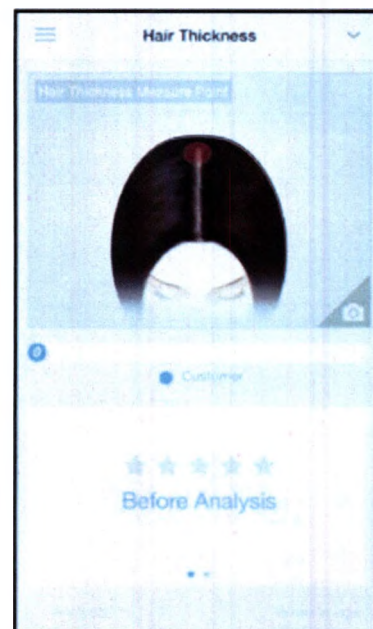
1. Press the button  , to activate camera mode


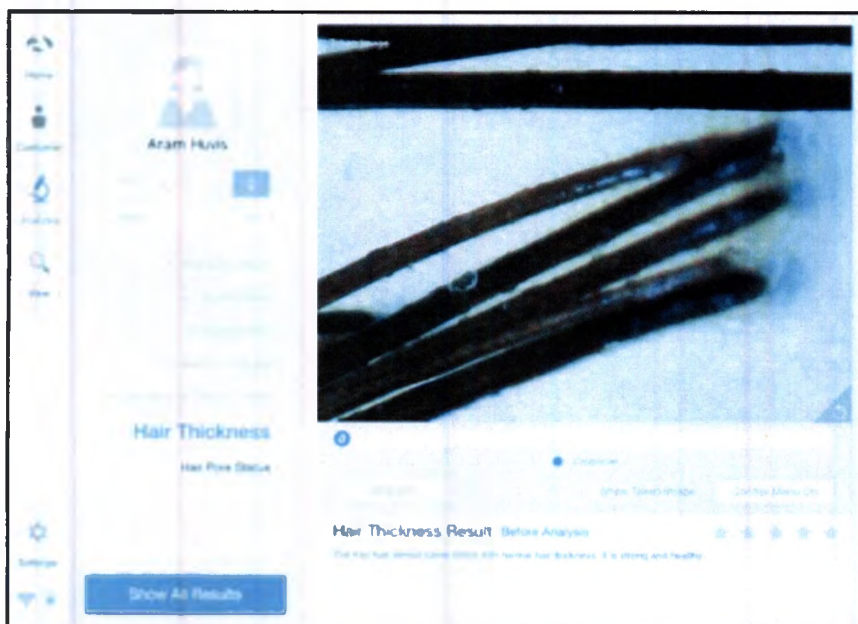
2. Take a picture of hair by  button. Then press button  to analyze it

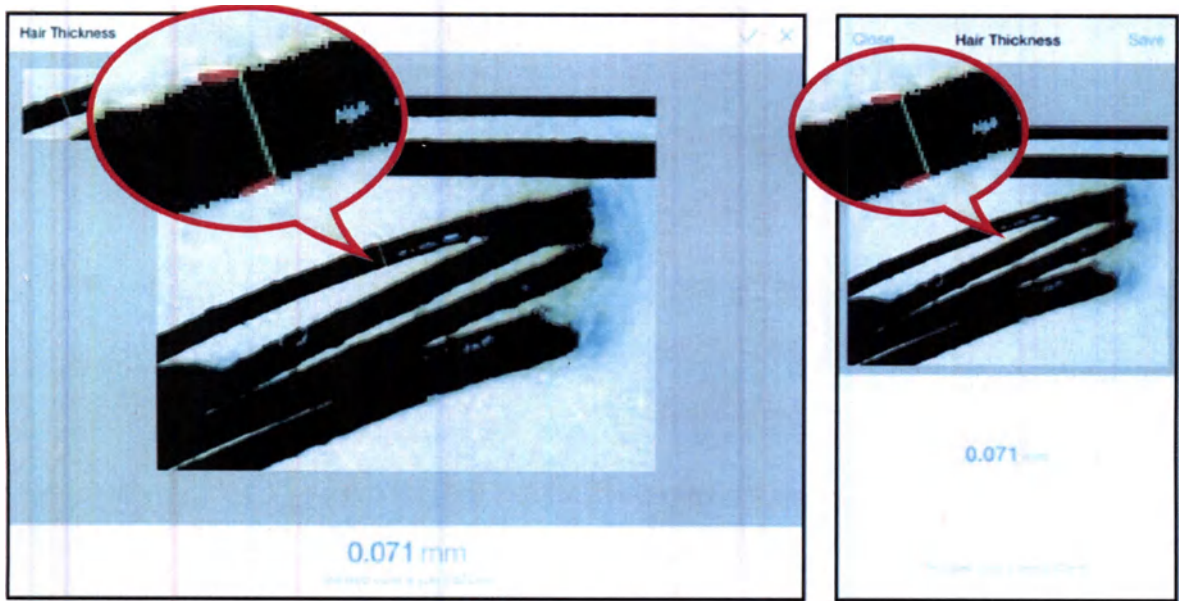3. Select the hair you want to measure. The value is automatically calculated.

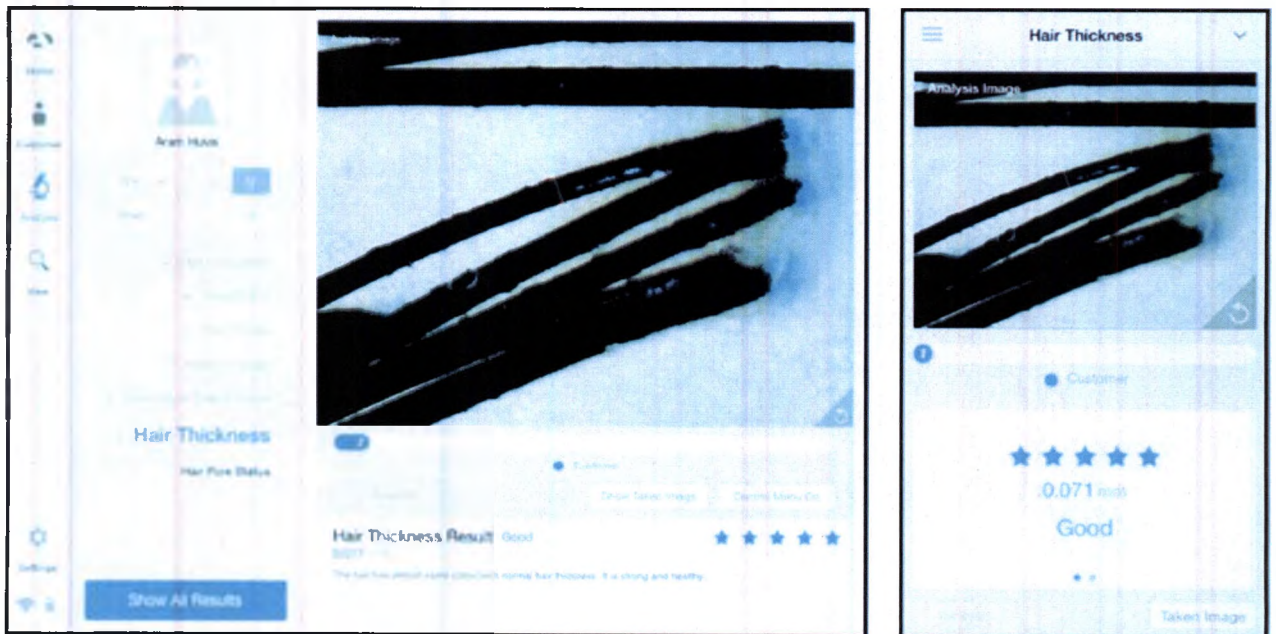
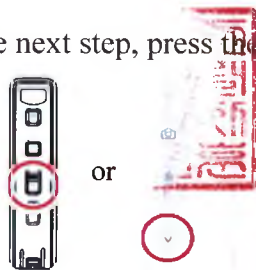
4. The result will appear. To analyze the next step, press the below button.


Hair Pore Status

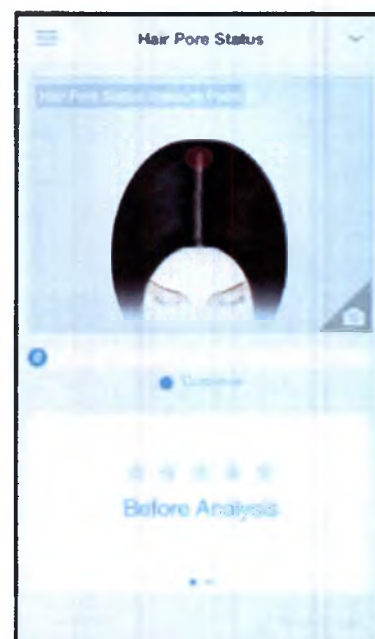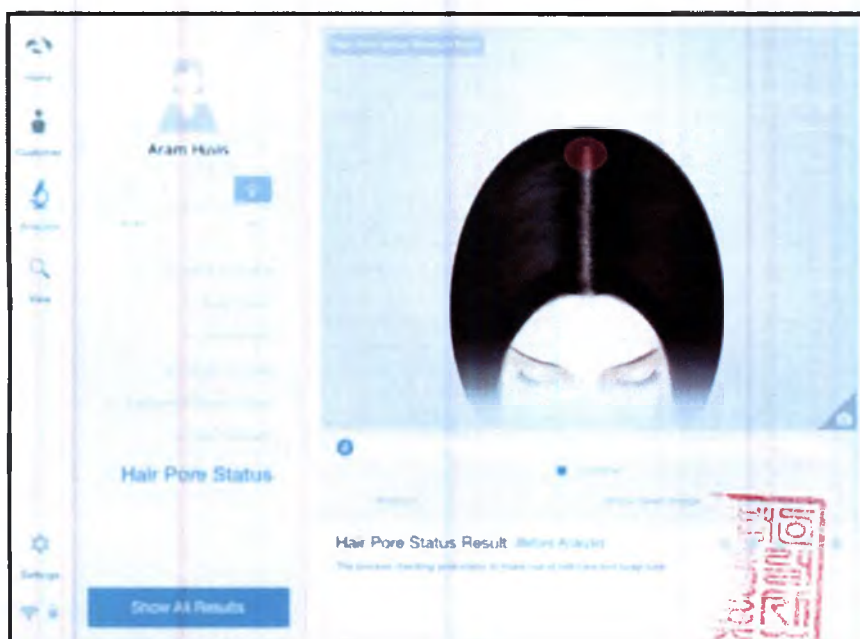
1. Press the button  , to activate camera mode


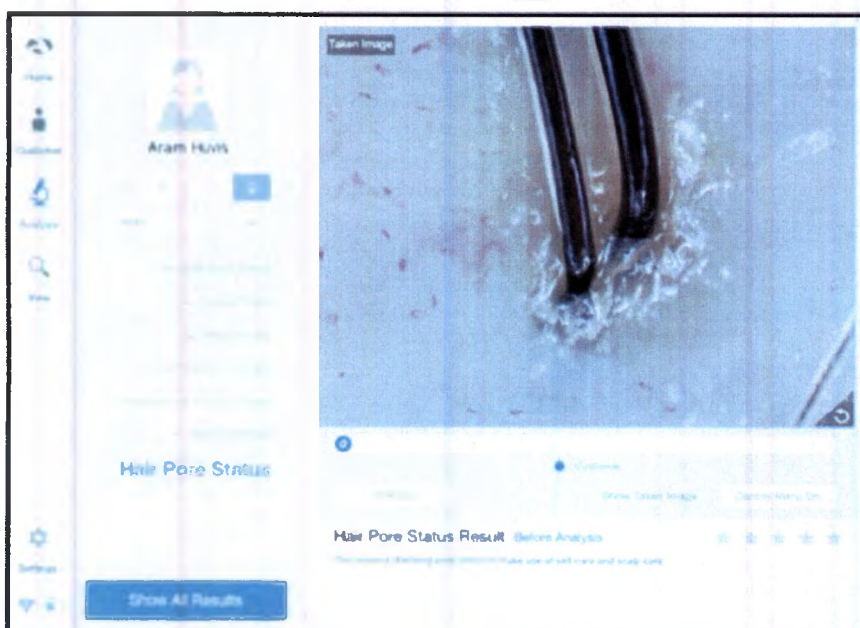
2. Take a picture of a hair pore by  button. Then press button  to analyze it

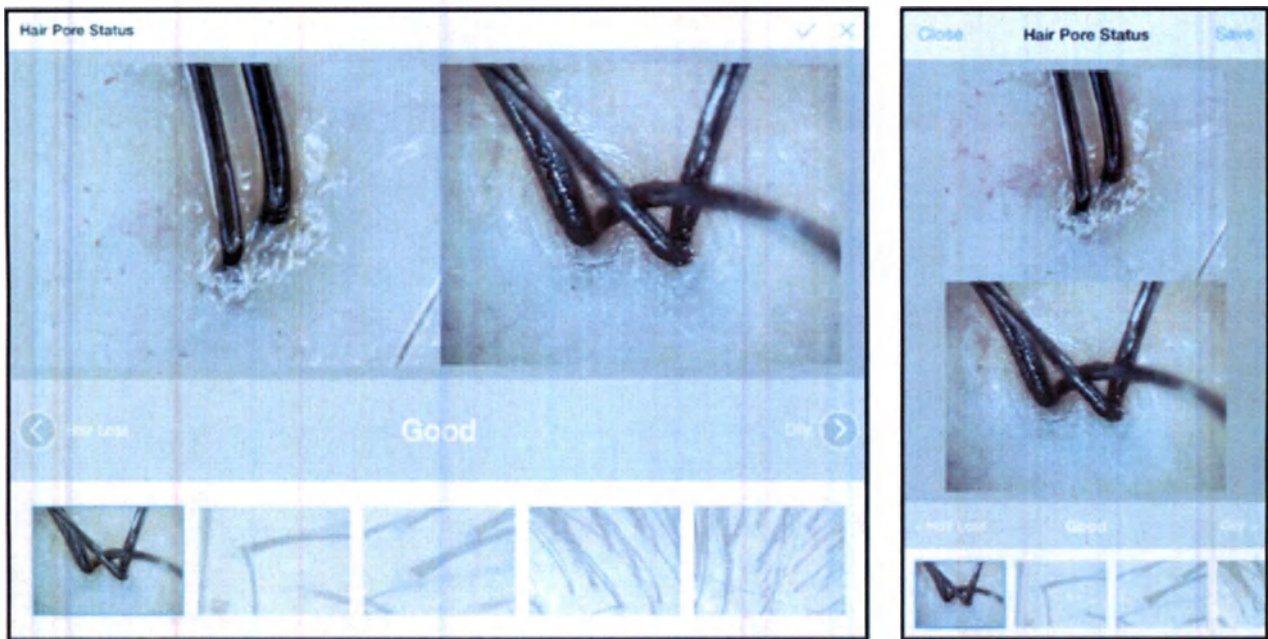3. Among the samples, choose the one which is similar with your image.

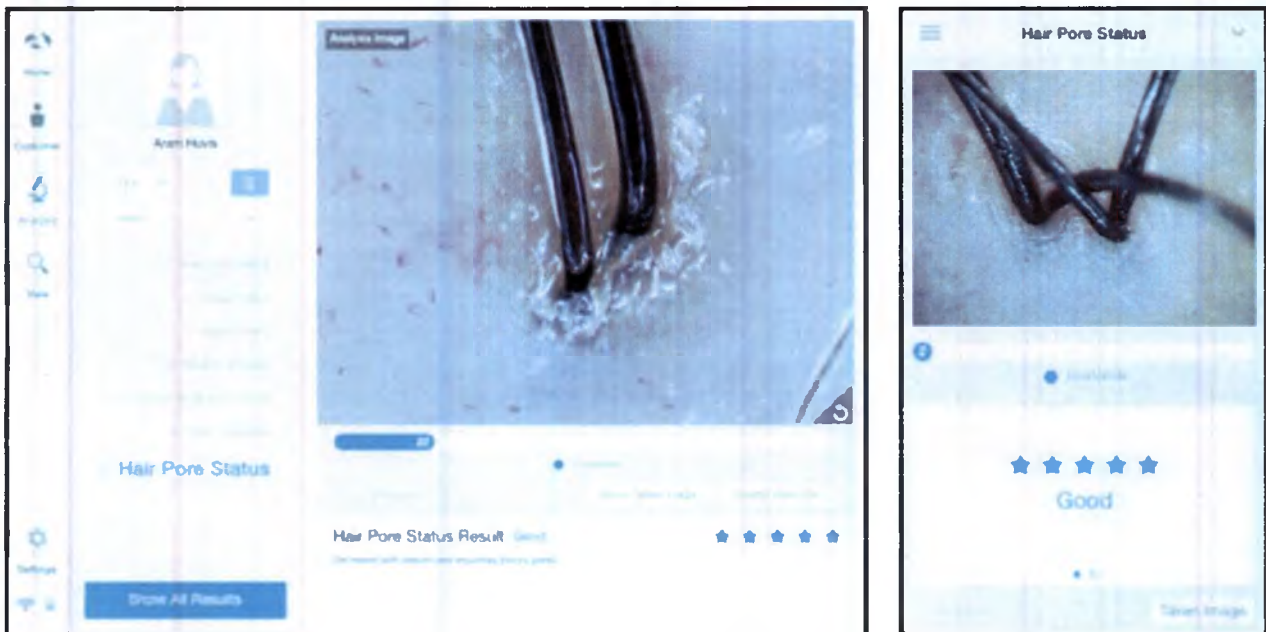
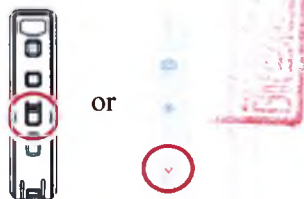
4. The result will appear. To analyze the next step, press the below button.



Cuticle Status

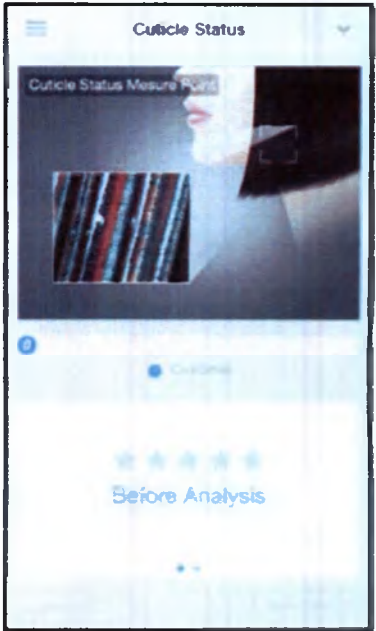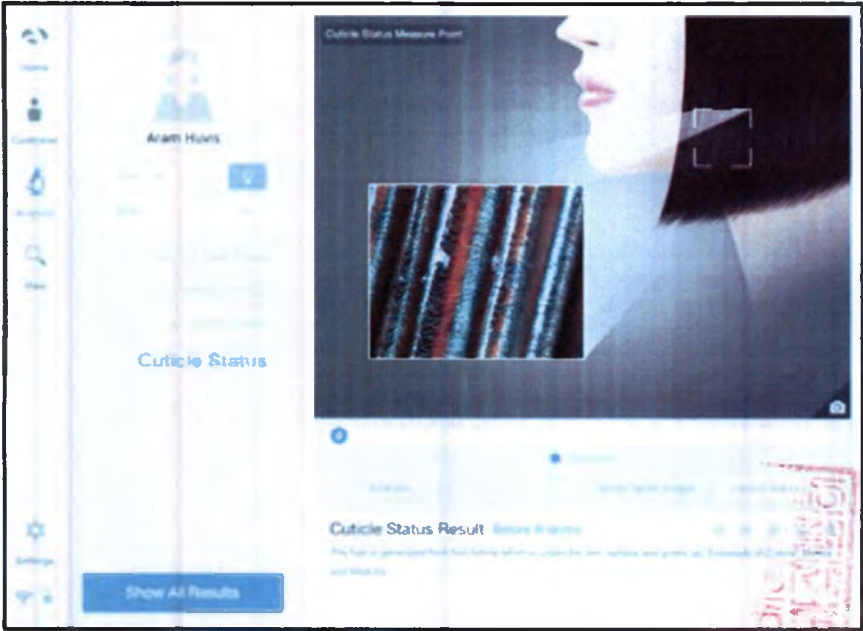
1. Press the button  , to activate camera mode


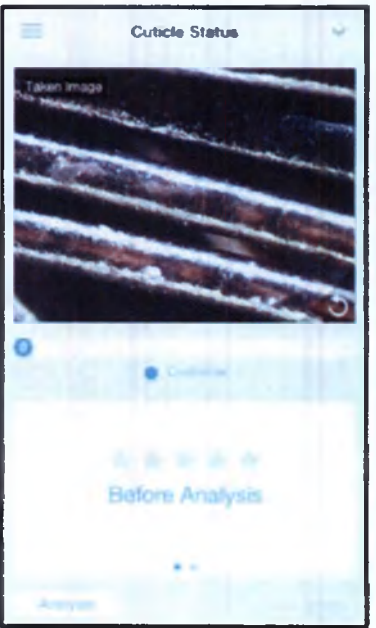
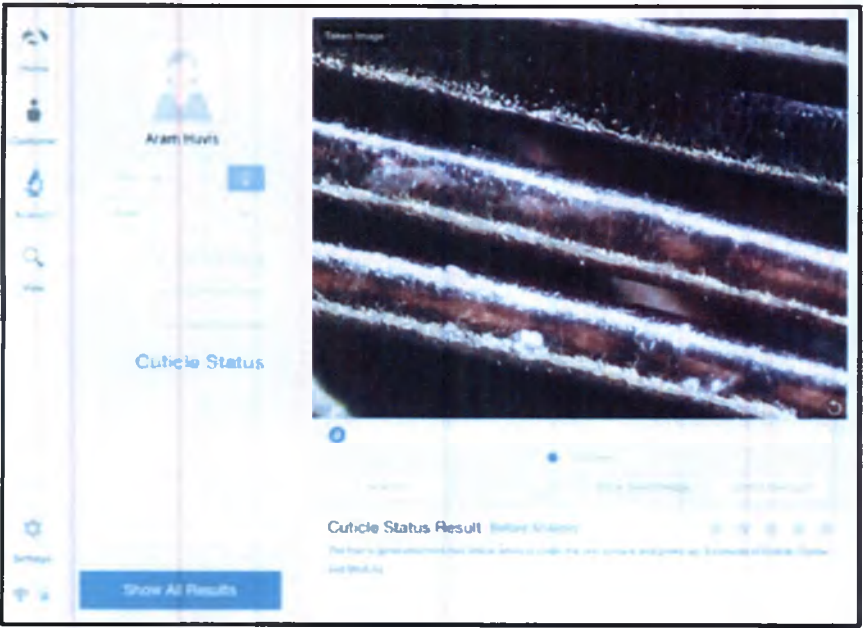
2. Take a picture of hair by  button. Then press button  to analyze it

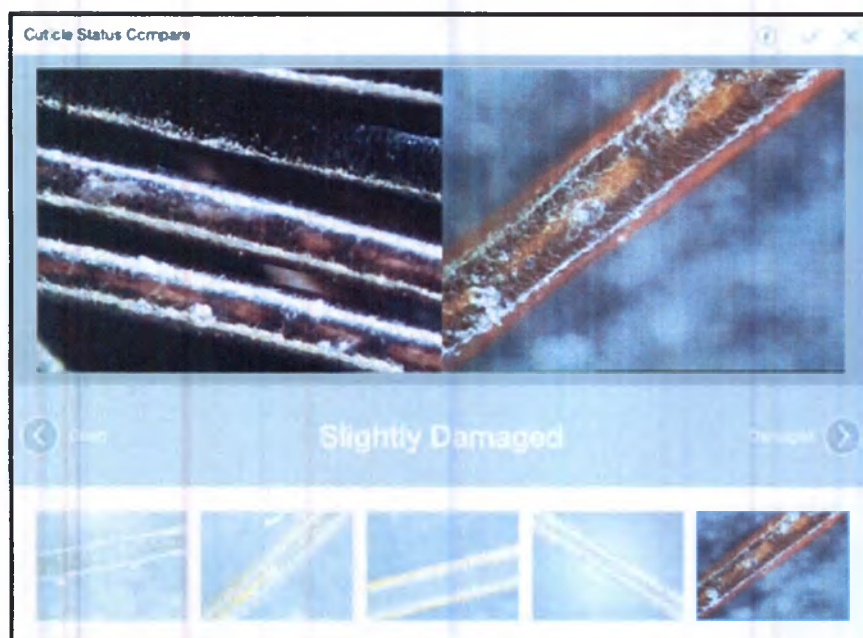3. Among the samples, choose the one which is similar with your image.

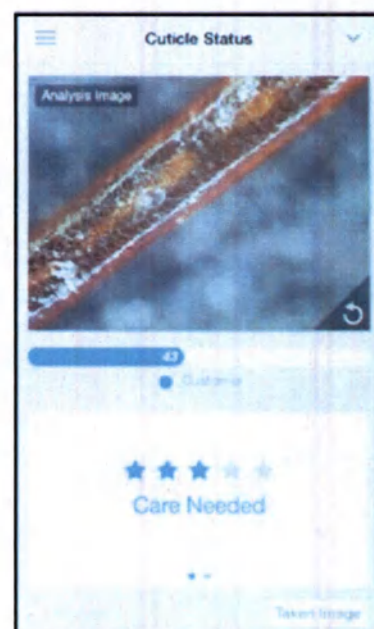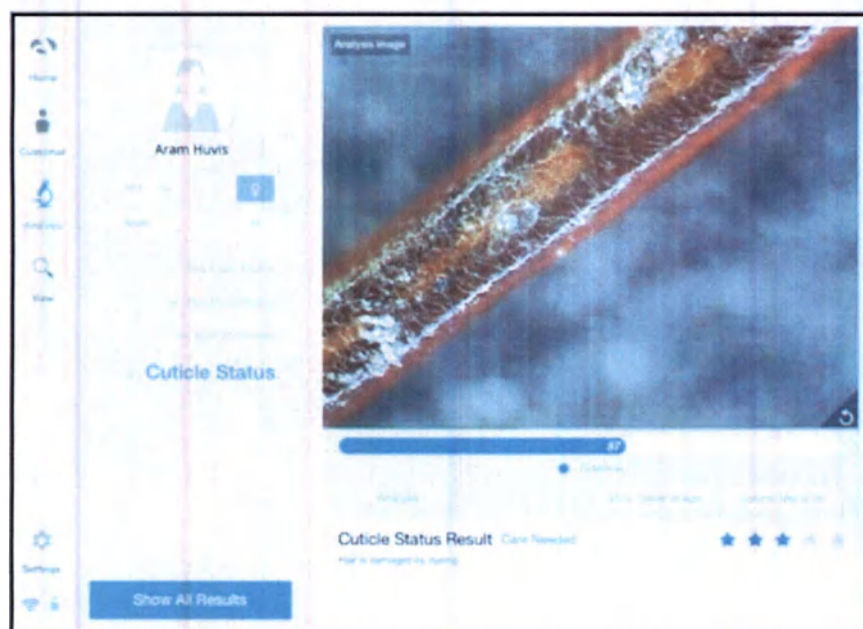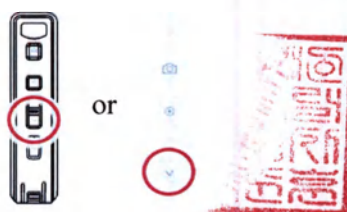
4. The result will appear. To see all results choose “Show All Results” or press the below button.


Summary of the Results

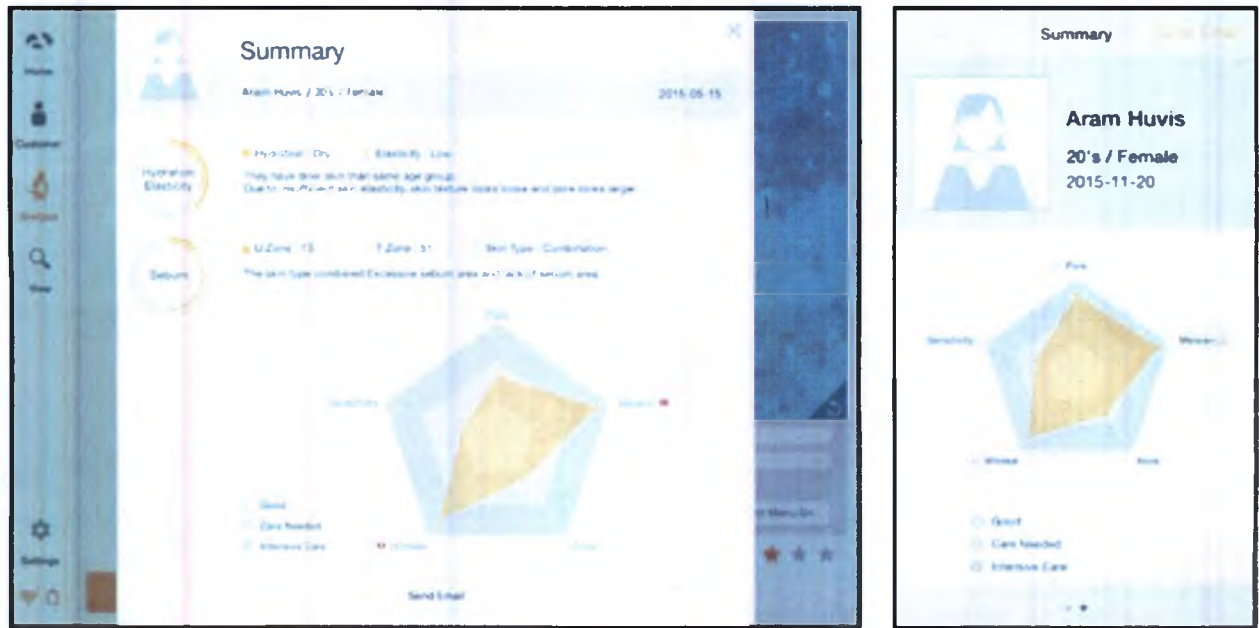1. Check the result

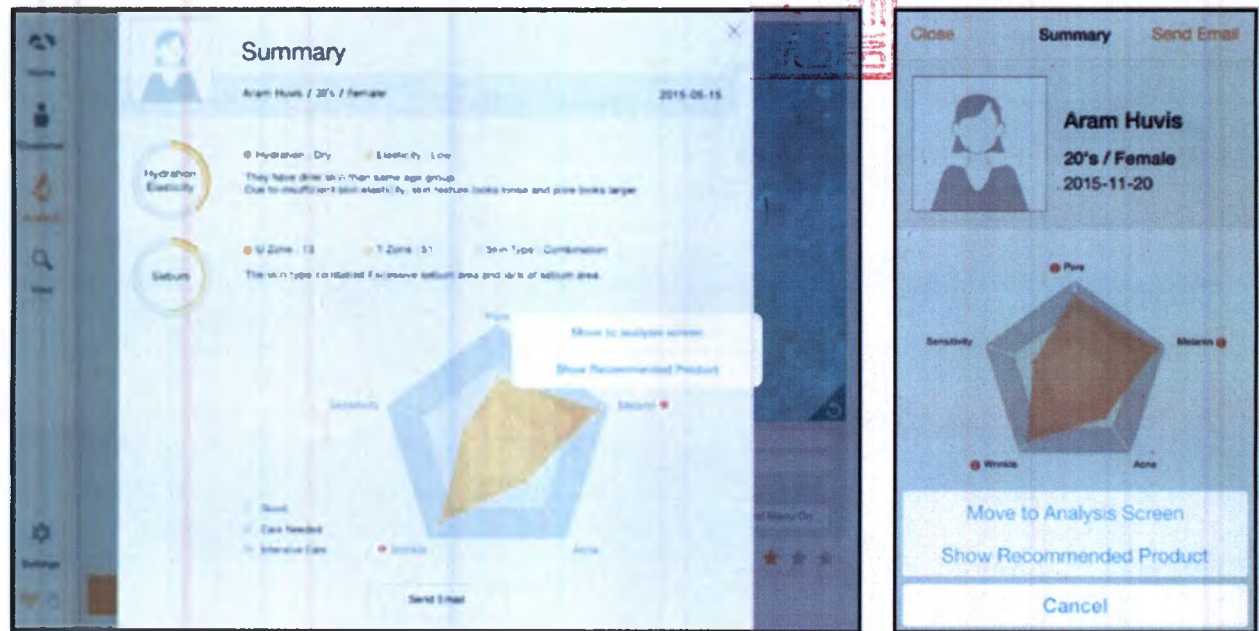2. Select  button or each parameter's name on pentagon graph. Once you select "Show Recommended Product", product image will pop up as 3


3. Available to check product image



View Mode

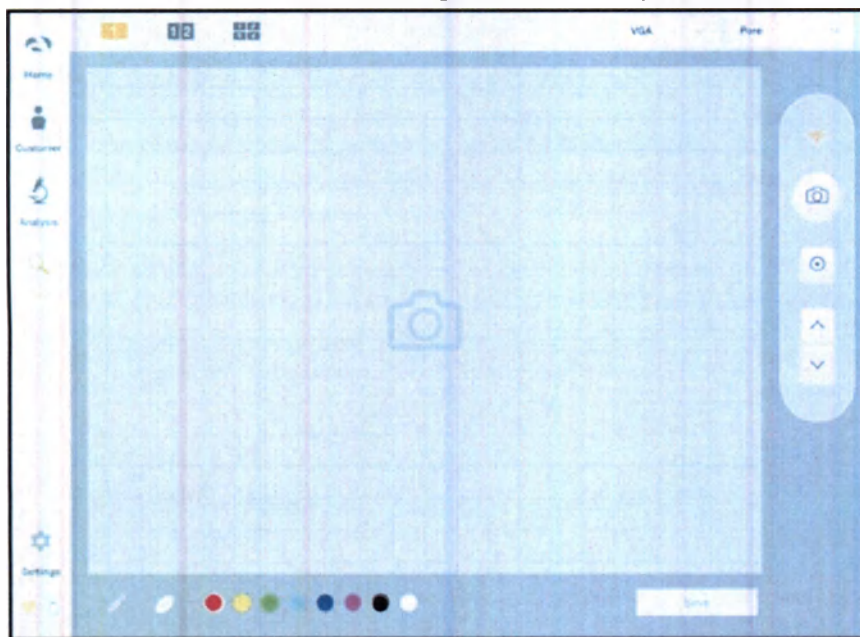1. In view mode, it is able to take pictures with capture button in many times.

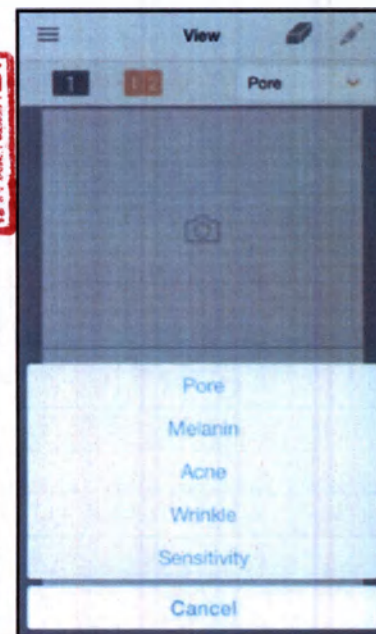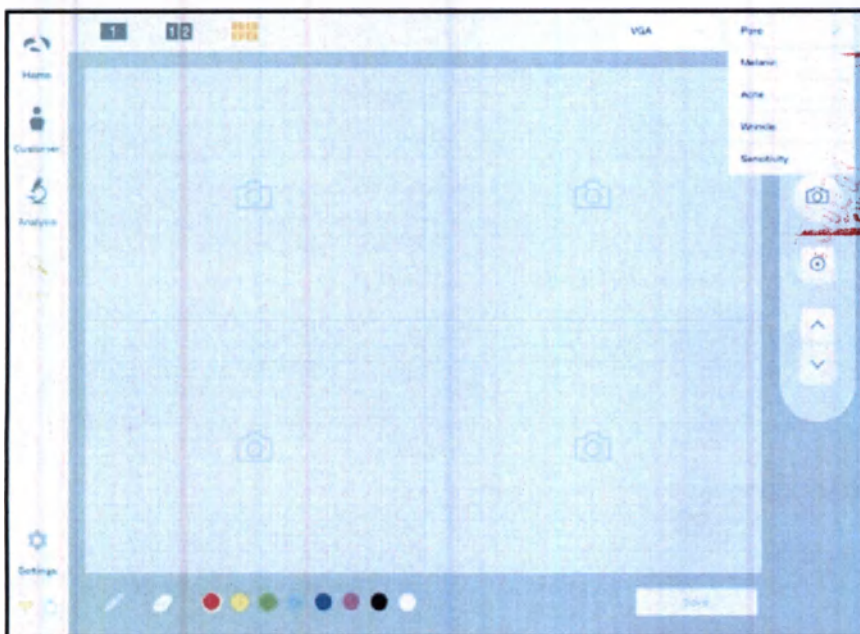
2. The user can select analysis parameter to take picture.

3. In one screen, you can see 4 different images maximum.

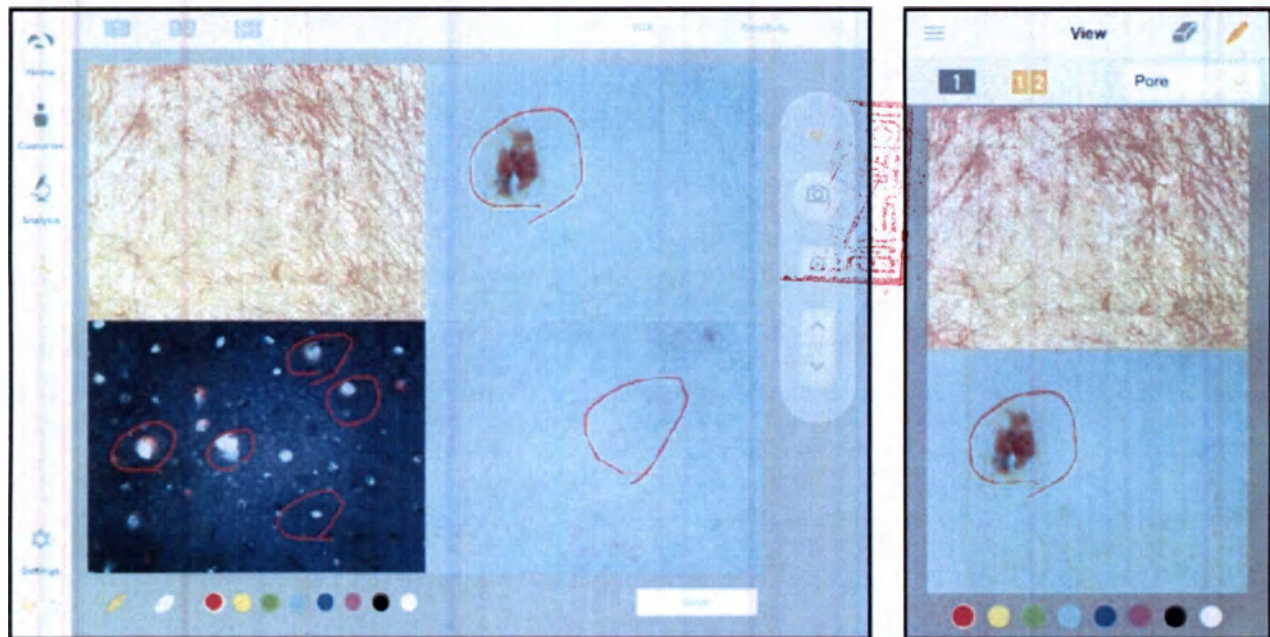4. The user can mark the defect or concern according to the image with note function.

14. Solutionist Software Troubleshooting & FAQ

1. Device is not available to connect to the program.

- Check the Wi-Fi connection statue in the setting of the device (Computer or tablet or smart phone).
- Please check the password of Wi-Fi.

2. I cannot click the menu on the Solutionist after connecting with device.

- You need to get a certification first before using Solutionist program.
- Menu will be activated once you get a certification.
- In order to get information how to get device certification, please refer the **Section 13. Directions of use → Product Activation** of this Abstract.

3. What is the Wi-Fi Combo on the manual?

- With this function user can use device and internet at the same time.
- User can send an email after analysis immediately.
- In order to get information how to make this function works and how to use this, please refer **Section 13. Directions of use → Wi-Fi Combo** of this Abstract.

4. Where can I update the Solutionist?

- Go to the “Setting” menu, click “About” and update the version after checking current software version.
- Version updating is only available in Wi-Fi connection.

15. Conditions of Safe Use

Humidity

Do not install the device at the sites with high humidity or where water can be spilled upon the device. Moisture ingress can cause damage or inflammation of the device. The relative humidity at the working area of the device must be no more than 80% without the possibility of moisture condensation.

Repair

Do not disassemble or change the device construction since it is hazardous for the users (for details please see the Section 18. Device Care, Maintenance and Repair).

Electricity

Connection of the device to the electrical supply must be straightly in accordance with the ways, described at the Manual. The violation of these recommendations can be hazardous (for instance – shocking hazard).

Environment

Do not install the device at the sites with too high or too low temperatures. Recommended temperatures must be within 15°C – 35°C.

Sunlight

Do not leave the device and lenses under direct sun light. It can cause device damage.

16. Storage Instructions.

Storage conditions:

- Temperature: (0 - +40°C) ±1 °C;
- Relative humidity: not more than 80%±2%;
- Atmospheric pressure: 760 ± 10 mm Hg.

The device must only be kept in original packaging.

The devices must be stored in a dry, dark and cool place away from direct sunlight, atmospheric precipitations and soil moisture in stands or cases 30 cm above the floor, away from direct sunlight, heaters and other sources of heat (not less than 1 m).

The packed devices must be protected from dirt, dust, moist, and damage.

17. Technical and Performance Characteristics of the Medical Device

	The device for the diagnostics of skin (dermatoscopy) and hair (trihoskopiya) under magnification Aramo Smart Lite (ASL) with the lens:		
Item	ASL-100	ASL-202	ASL-203
Wi-Fi Type	Wireless LAN IEEE 802.11 b/g/n		
Frequency range (MHz)	2412 MHz –2472 MHz, 2422 MHz –2462 MHz		
Modulation	DSSS, OFDM		
Antenna Type	PCB antenna		
Operation System Support	Android, iOS, Windows, Mac OS		
Image Sensor	¼” Inch Color C-MOS UXGA [2 Mega Pixels]		
Sensor pixels (effective)	1624 × 1212 pixels		
Image Capture Size	Wi-Fi Video Streaming VGA (640×480) / 2M (1600×1200)		
Image Frame Rate	VGA (640×480) – 30 (fps) 2M (1600×1200) – 15 (fps)		
Light Sensitivity	Light intensity 0,1 lx at F 2.0		
LED	Dip LED X8 & UV-Chip LED X8	Dip LED X8	Dip LED X8
Lenses: magnification	ASL-100 1.X30-NL* 2.X30-PL** 3.X30-UV*** 4.Moisturer Sensor	ASL-202 1.X60-NL 2.X60-PL 3.X200-NL 4.X200-PL	ASL-203 1.X200-NL 2.X600-NL
Focusing	Fixed type		
Protection class (according to (p. 6.2 IEC 60601-1-2005)	Class II medical equipment with an internal power source and type B applied parts		
Battery	Type: Li-ION /Typical Capacity: 1150 mAh Normal Voltage: 3,7 V ± 10%		
DC IN Adaptor	(5V DC/2A)±5%/		
Out Dimensions: Width×Hight×Depth, mm	(69×100×22) ± 0,5		
Weight, g	100 ± 5		
*NL – normal light **PL – polarized light ***UV – ultraviolet light			

18. Device Care, Maintenance and Repair

18.1. Device Care Recommendations

The device for the diagnostics of skin ASL is contacting with the intact human skin (lenses ASL-100; ASL-202; ASL-203), therefore under normal conditions of use it is necessary to disinfect the contacting surfaces after each contact with the patient's skin. For the lenses disinfection: wipe the lenses with a cotton cloth wetted with 70% solution of ethanol. For the moisture sensor disinfection: wipe the sensor with a cotton cloth wetted with 70% solution of ethanol, then wipe with the dry cloth.

Sebum paper is a single use part and does not need to be sterilized or disinfected. After use the sebum paper is utilized in accordance with the rules and requirements approved for the hospital.

Microfiber cloth is used to wipe the surfaces not contacting with a patient's skin. There is no need for sterilization or disinfection of the microfiber cloth. The cloth is cleaned while it is became dirty, but not less than 1 time per 6 months. To clean the cloth wash it with warm soap water, then – with clean water.

The lenses must be kept clean and protected from dust, dirt and contamination. If there is dirt or a finger print on the lens it is recommended to clean the lens with a cotton bud wetted with 70% solution of ethanol. To remove dust from the lenses it is recommended to wipe the surface with the cloth that is supplied with the device or to blow the dust off with a blower (for the lenses – it can be found in stores for accessories for video- and photo devices).

Cleaning the device surfaces not contacting with a patient's skin (device body, DC IN adaptor, and other accessories) can be done as required, but not less than once in a month with a clean slightly wetted cotton cloth. It is possible to use special cloths for cleaning of electronic devices.

ATTENTION!

Turn off and disconnect the device from power supply before cleaning.

18.2. Device Maintenance

The installation, setting and using of the device no special knowledge is required. It is enough to read the Manual carefully and follow its recommendations regarding the maintenance of the device.

18.3. Device Repair

Do not disassemble or change the device construction since it can cause the shocking hazard to the user. Please do not try to repair the device if any trouble occurs – instead contact with the manufacturer's representative.

If any trouble occurs

If any trouble occurs during the using of the device (the uncertain noise, smell, or fume) it is recommended to stop using of the device, turn it off and call to the manufacturer's representative.

Please turn off the device and bring it to the representative for the warranty services, if anything from the following circumstances occurs:

1. Power supply connector is damaged;
2. The battery is leaking;
3. Liquid (or other substance) was spilled at the device;
4. The device is failed to perform as it should (the not intended use of the device can cause its serious damaging);
5. The device was dropped down or seriously damages in some other way.

19. Description of the Materials Content of the Device Including the Materials Contacting with Human Body

Table 2. The Description of the Main Functional Parts of the Device for the Diagnostics of Skin ASL and its Accessories and their Materials

Name of the Item:		1. Device for the Diagnostics of Skin ASL	
№	Name of the Functional Part	Description, material	Contact with a user/doctor's (D) or patient's (P) skin
1	Main case (main deco, main cover, battery cover, cam case)	P.C LG SC1004A W0262P White UV Spray	D
2	Wi-Fi lens ¹	GE141R 82049, Clear, Polished	-
3	Side Button	P.C LG SC1004A W0262P White UV Spray	D
4	Deco Window	PC 2T, back side print	-
5	Eject Knob	P.C. LG SC1004A W0262P White	D
6	Locker	P.C. LG SC1004A W0262P	-
7	Sensor Filter Bracket	LG SC1004A-KPA1 (black)	-
8	IR filter	Glass 0.5T high resolution	-
9	POGO Pin – L	BSBM Au-plated	-
10	Eject Spring	SUS 0,4	-
11	Locker Spring	SUS 0,3	-
12	Shield Can	Daechang Metal 0,2T (Ni-Plated)	-
13	Camera Screw	PH Ø 1,0 x 2 x 1,5 (head) BK	-
14	Camera Screw	PH Ø 1,4 x 3 x 2,5 (head) BK	-
15	Camera Screw	PH Ø 1,7 x 4 x 4 (head) BK	-
16	Tapping Screw	PH Ø1.4 x 5 x 2.5 (head) Ni	-
17	Tapping Screw	PH Ø1.4 x 6 x 2.5 (head) Ni	-
18	Label Plate	PE Sheet 0,1T one side sticker	-
Name of the Item:		2. Cradle for device Aramo Smart Lite (ASL)	
1	Cradle case	P.C LG SC1004A W0262P White UV Spray	D
2	Cradle Press	EGI 2.0T Zn-plated	-
3	Weight Tape	45 X 35 TESA 4965 (double sided, self adhesive)	-
4	Foot	PORON 7.8 X 1.7T BLACK	-
5	Tapping Screw	PH T2 2 x 4 x 3 Ni	-
6	Tapping Screw	PH T2 2 x 8 x 3 Ni	-
Name of the Item:		3. Li-Ion Battery	
1	Li-Ion Battery	3,7V / 1150 mA	-
Name of the Item:		4. DC IN Adaptor 5V (cable, power supply, adapter)	
1	Adapter (case)	P.C LG SC1004A W0262P White	D
2	Adapter (5B, 2A)	5B/2A, Common type, white	D
3	USB Cable	Data Cable 1,2 m white	D

¹ Wi-Fi lens is a led light lens that shows the device is connected with the Wifi. Wi-Fi lens is on the upper side of the device (at the figure "ASL Design" (page 14) it is called "Wi-Fi Indicate lamp").

Name of the Item:		5. Lens ASL-100 with built-in moisture sensor	
1	PC Lens Cover (100)	LG SC1004A-KPA1 (Black) Al Spray Coating	D
2	Holder Water PCB	P.C LG SC1004A W0262P, White	-
3	MOIS Pin	BSBM Au-plated	-
4	PL Filter Holder	LG_SC1004A-KPA1 (black), matt spray	-
5	IR LED Holder A(Skin)	LG_SC1004A-KPA1 (black), matt spray	-
6	PL Filter Vertical	PL Filter	-
7	PL Filter Horizontal	PL Filter	-
8	X30 Lens	ASL-30 (common lens) – optical glass	P
9	0775-shaft	BSBM Au-plated, Ø 1.5 x 7.75 x MITAP	-
10	Lens Holder	Al, Black, matt, anodized	-
11	UV Chip LED Spring	SWP-B 12.5 x 1,25 x 0.15 Au-plated	-
12	Moisture Spring	SWP-B 10 x 1.7 x 0.15 Au-plated	-
13	Lens Body	P.C LG SC1004A W0262P, White, UV Spray	-
14	Lens Body Cover	P.C. LG SC1004A-KPA1 (black)	-
15	Lens Body Cover	P.C. LG SC1004A W0262P, White, UV Spray	-
16	Camera Screw	PH M1.2 x 3 x 2 (head) BK	-
17	Camera Screw	PH M1,0 x 8 x 1,8 (head) Ni	-
18	Camera Screw	PH M1,0 x 3 x 1,8 (head) Ni	-
19	Camera Screw	PH Ø 1,4 x 4 x 2,5 (head) BK	-
20	Camera Screw	PH 1,2 x 3 x 2 (head) BK	-
21	Machine Screw	PH Ø 2 x 4 x 3 (head) BK	-
22	Lens Tape-A	PET, 10 x 45 x 0.05T Black	-
23	Lens Tape-B	PET, 10 x 35 x 0.05T Black	-
Name of the Item:		6. Lens ASL-202	
1	Lens Tip (202)	GE141R 82049, clear, polished	-
2	1500 x Shaft	Ø 1.5 x 15.0 x 1.0 TAP Au-plated	-
3	PL Filter Holder	LG_SC1004A-KPA1 (black), matt spray	-
4	IR LED Holder A(Skin)	LG_SC1004A-KPA1 (black), matt spray	-
5	PL Filter Vertical	PL Filter	-
6	PL Filter Horizontal	PL Filter	-
7	X60 lens	AML-601S – optical glass	P
8	Lens Holder	Al, Black, matt, anodized	-
9	Lens Body	P.C LG SC1004A W0262P, White, UV Spray	-
10	Lens Body Cover	P.C. LG SC1004A-KPA1 (black)	-
11	Lens Body Deco	P.C. LG SC1004A W0262P, White, UV Spray	-
12	Camera Screw	PH M1.2 x 3 x 2 (head) BK	-
13	Camera Screw	PH M1,0 x 8 x 1,8 (head) Ni	-

14	Camera Screw	PH M1,0 x 3 x 1,8 (head) Ni	-
15	Camera Screw	PH Ø 1,4 x 4 x 2,5 (head) BK	-
16	Camera Screw	PH 1,2 x 3 x 2 (head) BK	-
17	Machine Screw	PH Ø 2 x 4 x 3 (head) BK	-
18	Lens Tape-A	PET, 10 x 45 x 0.05T Black	-
19	Lens Tape-B	PET, 10 x 35 x 0.05T Black	-
20	Лента линзы – В	ПЭТ, 10 x 35 x 0,05Т черный	-
Name of the Item:		7. Lens ASL-203	
1	Lens Tip (203)	GE141R 82049, clear, polished	-
2	1500 x Shaft	Ø 1.5 x 15.0 x 1.0 TAP Au-plated	-
3	LED Holder-B (hair)	LG_SC1004A-KPA1(black), matt-spray	-
4	X200 lens	AML-2002H – optical glass	P
5	Lens Holder	Al, Black, matt, anodized	-
6	Lens Body	P.C LG SC1004A W0262P, White, UV Spray	-
7	Lens Body Cover	P.C. LG SC1004A-KPA1 (black)	-
8	Lens Body Deco	P.C. LG SC1004A W0262P, White, UV Spray	-
9	Camera Screw	PH M1.2 x 3 x 2 (head) BK	-
10	Camera Screw	PH M1,0 x 8 x 1,8 (head) Ni	-
11	Camera Screw	PH M1,0 x 3 x 1,8 (head) Ni	-
12	Camera Screw	PH Ø 1,4 x 4 x 2,5 (head) BK	-
13	Camera Screw	PH 1,2 x 3 x 2 (head) BK	-
14	Machine Screw	PH Ø 2 x 4 x 3 (head) BK	-
15	Lens Tape-A	PET, 10 x 45 x 0.05T Black	-
16	Lens Tape-B	PET, 10 x 35 x 0.05T Black	-
Name of the Item:		8. Protective cap for lens	
1	Lens Cap	Urethane (Songwon P-3198A) 90 ⁰ (white)	D
Name of the Item:		9. Sebum Paper	
1	Paper Base	White paper 120 gsm	D
2	Sebum paper, pink	Oil paper (see Table 2 for details)	P
Name of the Item:		10. Microfiber cloth	
1	Cloth	Polyester 80% + Polyamide 20%, Korean brand “DS trade”	D
Name of the Item:		11. Hand strip	
1	Hand Strip	PU(Polyurethane), Korean brand “LIMIT”	D
Name of the Item:		12. Manual	
1	Cover	White art paper 80 gsm	D
2	Pages	White offset paper 60 gsm	D

Table 3. Materials of the Sebum paper, pink

CAS #	Name of the Substance	Mass%
65996-61-4	Cellulose pulp	65 ~ 80%
471-34-1	Calcium carbonate	10 ~ 20%
57301-22-1	Butadiene copolymer	3 ~ 5%
25767-47-9	Alkyl-acrylate copolymer	2 ~ 3%
7631-86-9	Silicate (silicone dioxide)	1 ~ 3%
13332-58-7	Kaolin	1 ~ 2%
65996-62-5	Oxidized starch	1 ~ 3%
–	Fluorescent pigment	0,5 ~ 1%

20. Device Labels Information

The ASL package design is shown at the figure 4. Stickers with information in Russian are put additionally at the device custom package. These stickers for the different ASL design styles (size 16x7±0,2 cm, put at the side of the box top) are shown at the figure 5, a-g, and a sticker common for all ASL design styles (size 20,5x3,5±0,2 cm, put at the box bottom) is shown at the figure 5, h.

The device package and labels (fig. 4-8) show the following required information:

- manufacturing company name;
- brand name and the model of the device;
- name and registered office address of the manufacturer;
- name and registered office address of the manufacturer's representatives;
- country of the manufacturer;
- manufacturing date;
- information of customers' support service;
- barcode;
- serial number;
- electrical parameters (on the cradle and DC IN Adaptor);
- type and class of protection against electrical shock;
- symbols, specifying the information necessary for the proper usage of the medical device (see tables 4-6 below);
- number and date of GOST-R Registration Certificate.


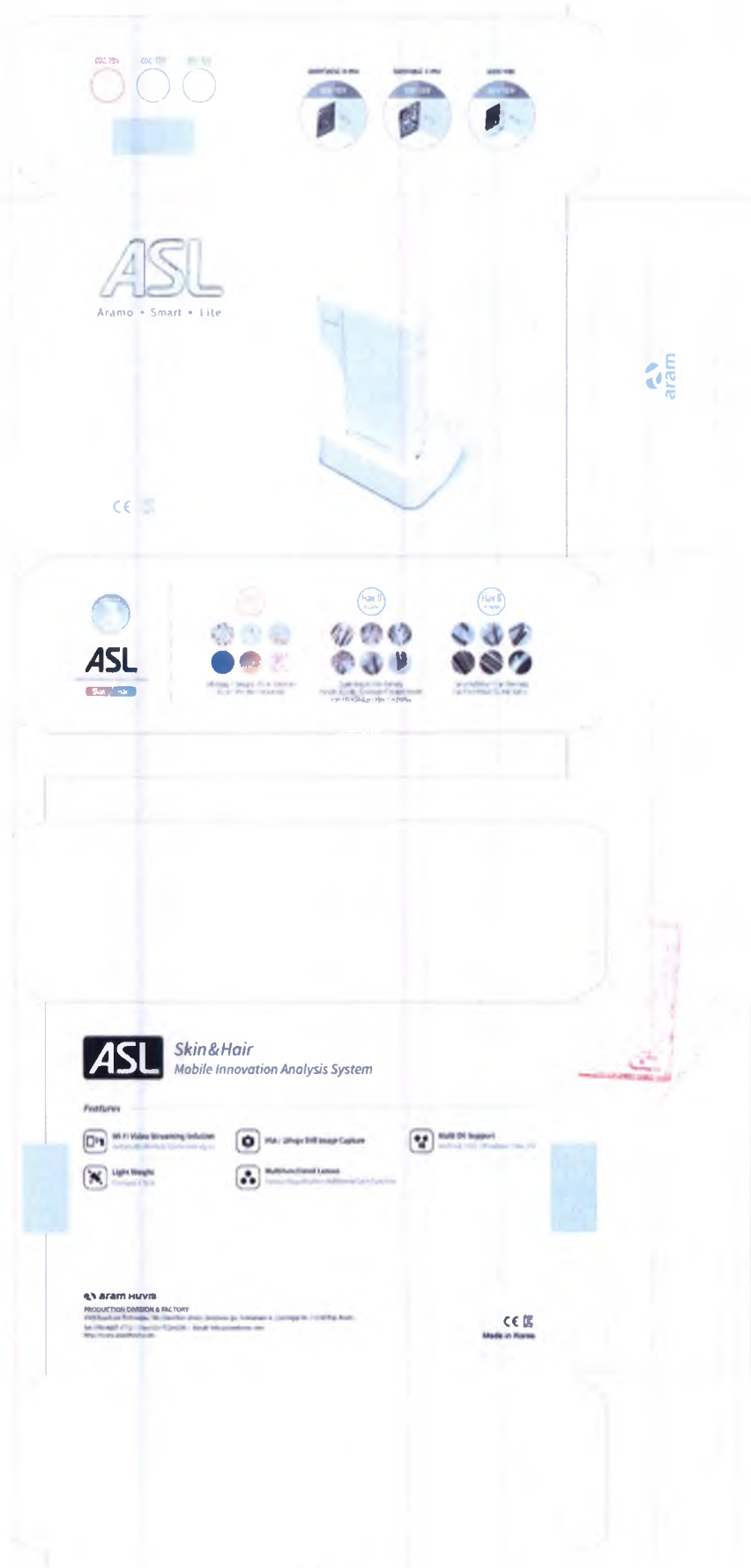
Fig. 4. Gift box design: **a** – general view; detailed view: **b** – top, central part; **c** – top, side part 1; **d** – top, side part 2; **e** – bottom.

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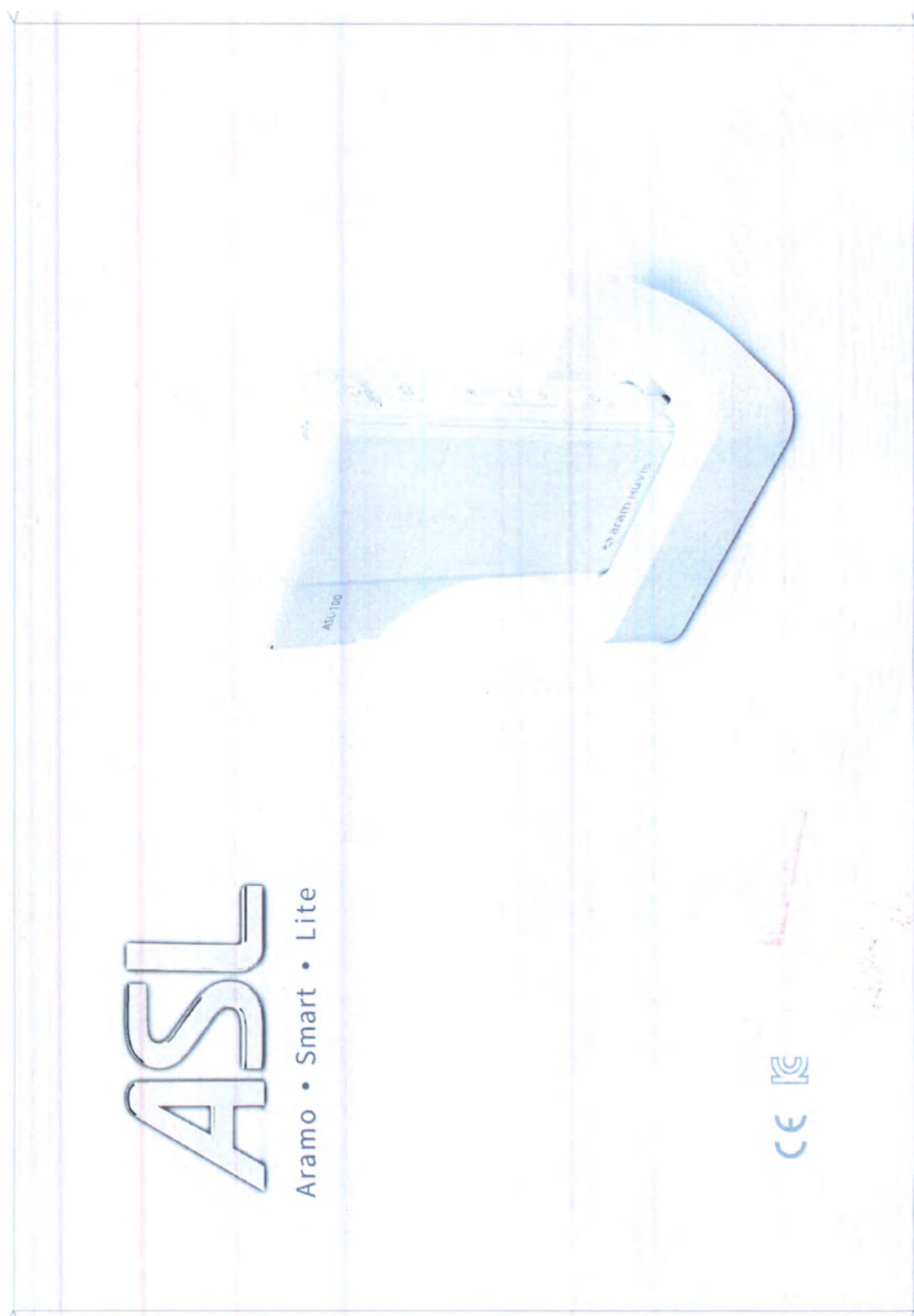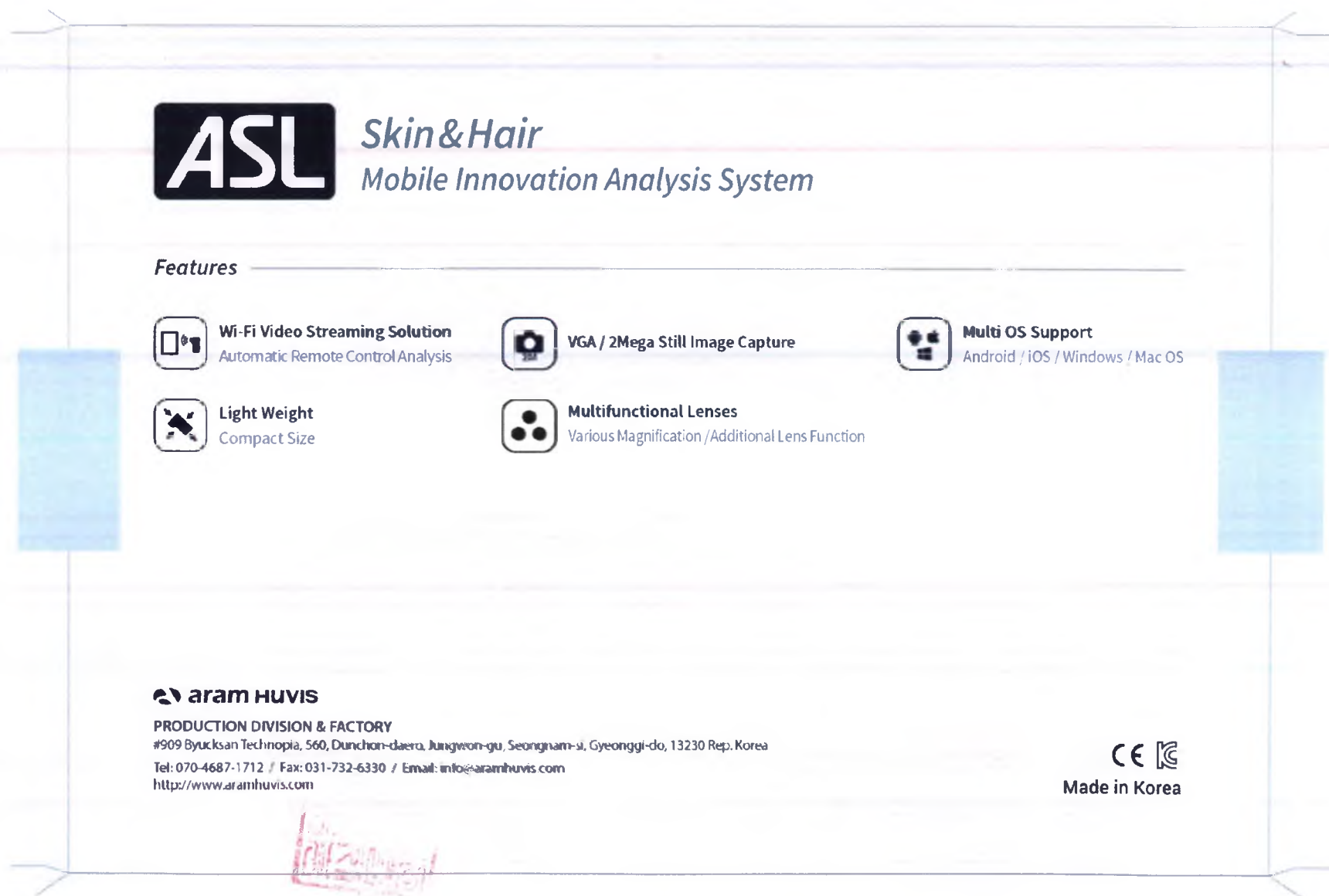
h

Fig. 5. Stickers with information in Russian: a, b, c, d, e, f, g – stickers for the different design styles (see section 12), h – common sticker (put at the bottom of the box). The symbols are explained in the table 5 below.

Fig. 6. Device label (placed under the battery). The symbols are explained in the table 4 below

Table 4. Device label legend

Symbol or Code	Description
CE 1137	EC Certification Mark
	KC Certification number
	Class II medical equipment with an internal power source and type B applied parts
MSIP-CMM-AHC-ASL	Model KC-code
CMIT ID	The ID of wireless authentication of China (SRRC)
Model Name	Name of the device model
Product SN	Device serial number (15 letters)
SSID	Wi-Fi LAN Identification name (SSID) (see for details – Section 13. Directions of Use → How to Connect with the Program)
PW	Wi-Fi LAN Password (see for details – Section 13. Directions of Use → How to Connect with the Program)
	CAUTION! Do not open the cover because of possible damage

Fig. 7. Cradle label (at the bottom).

Table 5. Cradle legend

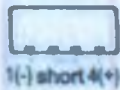
Symbol or Code	Description
Model Name	Name of the device model
Power	Electrical parameters: DC, 5V, 2A max.

Fig. 8. DC IN Adaptor (The symbols are explained in the table 7 below).

Таблица 6. DC IN Adaptor symbols legend

Symbol or Code	Description
Model	Name of the device model
Input	Input electrical parameters: Voltage 100-240V, variable., frequency 50/60 Hz, current 0,5A
Output	Output electrical parameters: Voltage 5V, direct., current 2,0A

	Pins of the output connector: 1 – negative, 2, 3 – short-circuit, 4 – positive
	Device safety mark, issued by TUV SUD
	Corresponded with the standards of the Federal Communications Commission, USA
FOR HOME OR OFFICE USE	For use in domestic or office conditions
	UL certified equipment
	Protection class II
	Low Power Supply

21. Safe Disposal and Utilization Instructions

Medical device must be disposed after usage, after the expiry date is reached, or if the package was damaged and the product cannot be used.

Medical devices must be disposed in accordance with the medical practices, which are regular for the medical institutions, and with the local state and federal regulatory documents. The choice of methods of safe disposal depends on the medical institution capacity and profile, availability of the waste decontamination/deactivation installations, the local routine ways of the waste decontamination/deactivation (burning, landfilling, utilization).

In Russian Federation the medical device must be disposed in accordance with the actual Russian Federation Sanitary Regulations and Standards for medical wastes management.

Since the device is information technology equipment it is recommended to use special utilization – the same as for computer and electronic appliances.

22. Shelf life of the Medical Device

Warranty service period – 1 year since the date of purchase or date of manufacturing, if the date of purchase is missed at the Warranty Card.

Shelf life – 3 year since the date of manufacturing.

23. Manufacturer's Warranty

Thank you for the choose products of the Company Aram Huvis Co., Ltd (South Korea). The product undergoes strict quality management and inspection process during production. In case of functional or operational defect occurring during the warranty period you have the right for warranty services – we will test the device for free and if the defect confirms – we will replace the device or its parts.

Model name		SN	
Date of purchase		Company name	
Address		Contact phone	

Please fill in the Warranty Card and provide it if any warranty event occurs.

Warranty service period – 1 year since the date of purchase.

Warranty Service – Warranty service provided for free if the warranty event occurs during the warranty period (1 year since the date of purchase).

Warranty Event	Service
Functional defect occurring under normal condition of use within 10 days after purchase	Replacement with a new product
Functional defect occurring under normal condition of use within 1 year after purchase	Warranty service
Damage in cases excluding the warranty service	Forwarding to the chargeable repair

Cases excluding the warranty service

- Damage because of violation of the rules of intended use;
- Mechanical damages (external and internal);
- Damage because of liquid ingress or unauthorized repair;
- Damage because of the power supply does not correspond to the federal standards quality of supply lines.

Service Request Form			
Date of purchase	/	/	Company
Contact person			
Address:			
Tel.:			
Product name, serial no:			
Problems:			

24. Manufacturer

Aram Huvis Co., Ltd.,
801, 434-6, Sangdaewon-dong, Jungwon-gu,
Seongnam-si, Gyeonggi-do, Korea

<http://aramhuvis.com/>

Tel.: +82 31 732 6320

Fax: +82 31 732 6330

info@aramhuvis.com

25. Manufacturer's Representative and Customers Service in Russian Federation

ООО «NATRIA»

115114, Moscow, Derbenevskaya street 5/1

Russian Federation

Tel. (495) 995-96-76

e-mail: natria@list.ru

The design styles, accessories, and spare parts of the Device for the diagnostics of skin (dermatoscopy) and hair (trihoskopiya) under magnification Aramo Smart Lite (ASL)

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Spare parts

1. Lens ASL-100 with built-in moisture sensor – 1 pce.
2. Lens ASL-202 – 1 pce.
3. Lens ASL-203 – 1 pce.
4. Lithium-Ion Battery – no more than 3 pcs.
5. DC IN Adaptor 5V (cable, power supply, adapter) – 1 pce.

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appeared before me and admitted
said principal`s subscription to
the attached INSTRUCTIONS FOR USE
OF THE MEDICAL DEVICE-----

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This is hereby attested on this
15th day of Sep. 2017 at this office.

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Belong to Seoul Central
District Prosecutor's Office
Korean Re Bldg. 5F, Ste. 502, 68, Jong-ro 5-gil
Jongno-gu, Seoul, Korea

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This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, May. 2009 Under Law No.16.

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