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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of B.H MEDICAL PRODUCTS CO., LTD. on the annexed INSTRUCTION FOR USE is genuine.

China Council for the Promotion
of International Trade

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Authorized

Signature:

Liu Yongcui

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(Date: Jan. 07, 2020)

证明 查询网址 Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

"APPROVED"

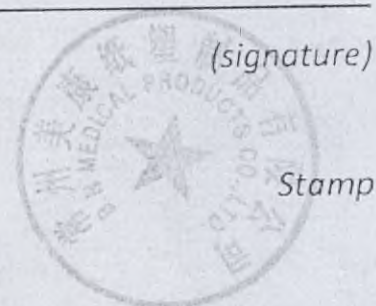
BH Medical Products Co., Ltd.

General Manager

Tao Xiao Hua

Tao Xiao Hua

(signature)

INSTRUCTION FOR USE

FOR MEDICAL DEVICE

2019

BH Medical Products Co., Ltd.

Tel: 0086-519-83978717

Add: NO.90 Shengjiang villagers' group, Zhangjiacunwei, Xilin Street, Zhonglou District,
Changzhou City, Jiangsu Province, People's Republic of China

Fax: 0086-519-83978717

Web: www.bhdisposable.com

Name on the market of the Russian Federation of the medical device:

Protective covers for the radiovisiography sensor "Dispodent", disposable, polymeric, non-sterile

Purpose

Protective covers are intended for protection of patient from cross contamination. They provide patient with sanitary protection during dental care procedures and comfort usage during radiovisiographic procedures.

Scope of application

The area of use for the product is dentistry. For professional dental treatment only. This product is intended to be used by highly qualified medical personnel in settings of dentist's room at clinics, hospitals and other medical dentistry organizations.

Indications

Indications for the use of the medical device are:

- dental care procedures
- radiographic procedures.

Contraindications

The product is contraindicated for use in patients who are:

- Claustrophobic.
- Suffering from severe asthma.
- Having difficulties breathing through the nose.
- Those with allergic reaction for materials used for manufacturing of covers.

Side effects

Patients can experience talking difficulties, when the product is in use. Allergic reaction due to intolerance of patient to material used for manufacturing of covers.

Precautions in the use of the product.

- Protective covers for the radiovisiography sensor "Dispodent", disposable, polymeric, non-sterile are intended only for single application.
- Do not use expired covers:
- Do not use damaged covers:
- It is necessary to follow rules and safety measures, applicable to this type of procedures in the field of profession dentistry application;
- User shall be familiar with components, used at covers fabrication, this will help to avoid patient's allergic reaction.

BH Medical Products Co., Ltd.

Add: NO.90 Shenzhou villagers' group, Zhongjiacunwei, Xilin Street, Zhongjiu District
Chongzhou City, Jiangsu Province, People's Republic of China

Tel: 0086-519-83978703

Fax: 0086-519-83978717
Web: www.bhdisposable.com

Operating principle

Operating principles of the product are based on isolation of oral cavity from external effects.

Risks associated with the use of the product

If the materials used to manufacture the device are inappropriate, the resulting device may be unfit for its intended use. The risks were identified at the stage of design and development (inadequate materials) and utilization (infection due to negligence). At the stage of design and development, the risks were managed (by careful choice of materials). The utilization risks are minimized by informing the user. Thus, all risks identified are mitigated, and the remaining risk is acceptable.

Correlation with other medical devices

- Protective covers for the radiovisiography sensor "Dispodent", disposable, polymeric, non-sterile match with all medical products, which are used in dentistry.

Technical specifications

Main technical specifications:

Item No.	Indicator description	Width x Length (a x b)	Tolerances
1	Dimension, (mm) (see Fig. 1)	35mm x 200mm 43mm x 200mm	±5% ±5%
2	Thickness (um)	25	±10%
3	Density, (kg/m ³)	918	±5%
4	Tensile Strength, (MPa(N/mm ²)) - in longitudinal direction - in transversal direction	Not less than 30 Not less than 25	-
5	Tensile yield stress, (MPa(N/mm ²))	11	±5%
6	Elongation at break, (%) - in longitudinal direction - in transversal direction	700 700	±10% ±10%
7	Welded joint tensile strength,	Not less than 60%	-

BH Medical Products Co., Ltd.

Add: NO 90 Shenyang villagers' group, Zhongjiao Wei, Xilin Street, Zhonglou District, Changzhou City, Jiangsu Province, People's Republic of China

Tel: 0519-83928717

Fax: 0519-83928717
Web: www.bhdisposable.com

BH MEDICAL PRODUCTS CO., LTD

	(MPa(N/mm ²))	of tensile strength in longitudinal direction	
8	Softening point, (°C)	100	±2°C
9	Melting point, (°C)	121	±2°C
10	Statistical friction coefficient	0.1-0.5	-
11	Surface resistivity, (Om)	Not more than 1x10 ¹⁶	
12	Weight, (g)		
	35mm x 200mm	0.29 g	±10%
	43mm x 200mm	0.36 g	±10%

Covers have transparent surface. Following is not allowed to be on surface of covers: foreign substances, which can be observed by bare eye, pressed-in wrinkles, unevenly cut edges, tears and holes. Welded joints of covers are even straight all its length, without burnt through areas and wrinkles. Covers don't give any foreign scent to distilled water and don't change color and transparency in distilled water. Inner surfaces of covers are not adhesive to one another. Covers are radiotransparent. Darkening deviation, measured by densimetric method during shooting of sample don't exceed 5%. Covers are resistant to the effects of biological fluids (saliva, blood).

Materials used for the product:

Low density polyethylene CAS number 9002-88-4,

Manufactured by: BH Medical Products Co., Ltd., China

– 98%,

Concentrate of Anti-blocking and sliding additives, Manufactured by - Jiangxi Zhilian Plastics & Chemicals Technology Co., Ltd., China - 2%

Product utilization process

- Unpack the product, free it from package material.
- Check the product for its integrity with the visual inspection.
- Use this product only for its intended use. Follow the instructions in this leaflet.

Application and maintenance requirements

BH Medical Products Co., Ltd.

Tel: 0086-519-83978705

Add: NO. 90 Shenyang villagers group, Zhongjiacunwei, Xian Street, Zhongqiu District
Changzhou City, Jiangsu Province, People's Republic of China

Fax: 0086-519-83978717
Web: www.bhdisposable.com

BH MEDICAL PRODUCTS CO., LTD

Products are used in dentist's room at clinics, hospitals and other medical dentistry organizations by the medical personnel, taught how to use covers, in accordance with requirements and procedures which are in action on the territory of the Russian Federation.

Lifetime and warning concerning the prohibition of medical device use after the expiration date.

The warranty service life of the product - 5 years from the date of manufacture.

Do not use after the expiration date.

Operation, storage and transportation conditions

Usage environment: the product withstands exposure to temperature $(+32 - +42) \pm 1^{\circ}\text{C}$ and RH of 80% at 25°C without quality and functional performance deterioration.

Products are stored in dry ventilated room at temperature from $+5^{\circ}\text{C}$ to $+40^{\circ}\text{C}$ and RH of 80% at temperature of $+25^{\circ}\text{C}$. Avoid direct sunlight.

Products may be transported by any roofed vehicle according to the rules and requirements for goods transportation depending on the vehicle type. Mode of sending: parcel post, truck, plane, container. Transportation conditions: at temperature from $+50$ to -50°C and 95% RH at $+25^{\circ}\text{C}$

Medical device packaging variations

Dimensions 35mm x 200mm: Cardboard package - 500 units, Leaflet for use - 1 unit.

Dimensions 43mm x 200mm: Cardboard package - 500 units, Leaflet for use - 1 unit.

Requirements for the maintenance and repair

The product is provided in its final form and requires no maintenance or repair, other than disposal, as per the standards of the medical institution.

Utilization and termination procedure

Products which haven't been utilized can be disposed of with household waste. Used products belong to Class B wastes and are to be disposed according to SanPiN 2.1.7.2790-10 Health and Hygiene Rules and Standards with regard to Medical Wastes Disposal (as approved by Russia's Chief State Medical Officer Regulation No. 163 dated December 9, 2010)

Warranties by manufacturer

Manufacturer guarantees quality of covers if rules for operation, storage and shipping were followed.

BH Medical Products Co., Ltd.

Add. NO.90 Shenjiang villagers' group, Zhongliadonwei, Xilin Street, Zhongliou District
Chongzhou City, Jiangsu Province, People's Republic of China

Tel: 0086-519-83978705

Fax: 0086-519-83978717
Web: www.bhdisposable.com

BH MEDICAL PRODUCTS CO., LTD

Disclaimer: Covers are intended for professional use in dentistry field and shall be used in strict adherence to current leaflet. Manufacturer doesn't responsible for non intended use of covers or for use violating states of leaflet.

AUTHORIZED REPRESENTATIVE in Russia.

Dispotrade LLC. 1/1 Vvedenskogo Street. Room 226. 117342. Moscow, INN 7728585728. KPP 772801001 Tel.: 8 (495) 589-13-30. E-mail: dispodent@mail.ru

INFORMATION ABOUT THE PRODUCT MANUFACTURER.

«BH Medical Products Co., Ltd». No 90 Shenjiang villagers' group Zhangjiacounwei, Xilin Street Zhonglou District, 213024 Changzhou City, Jiangsu Province, PEOPLE'S REPUBLIC OF CHINA

BH Medical Products Co., Ltd.

Tel: 0086-519-83978705

Add: NO.90 Shenjiang villagers' group, Zhangjiacounwei, Xilin Street, Zhonglou District,
Changzhou City, Jiangsu Province, People's Republic of China

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