
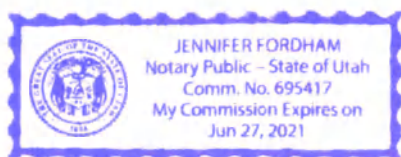
ACKNOWLEDGEMENT

State of Utah)
 §
County of Salt Lake)

On this 25th day of July, 2019, before me, Jennifer Fordham, a notary public, personally appeared Robert Wolfarth, Senior Manager, Regulatory Affairs proved on the basis of satisfactory evidence to be the person whose name is subscribed to the attached instrument, and acknowledge he executed the same.

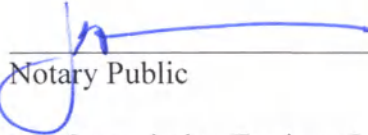Witness my hand an official seal.

Notary Public

My Commission Expires: June 27, 2021

Regulatory Affairs Sr. Manager
Merit Medical Systems, Inc.
USA

July 25, 2019

**Angiographic needles with
accessories
INSTRUCTIONS FOR USE**

INSTRUCTIONS FOR USE

INDICATIONS:

The Merit Advance Angiographic Needle is used for providing a puncture site in blood vessels for the introduction of vascular access devices.

Merit Advance Angiographic Safety Needle also incorporates a safety mechanism to minimize needlestick injuries.

The Majestik Needles are manufactured with a transparent polycarbonate hub, stylette key (notch in the hub), easy grip shield, stainless steel stylettes, cannulas and obturators.

WARNINGS:

- Do not use if package is opened or damaged.
- Do not alter this device.
- Failure to fully advance the safety mechanism until it locks when removing the needle may result in the needle tip re-emerging from the safety mechanism, potentially resulting in an accidental needlestick with a contaminated needle. A needlestick with a contaminated needle may cause infectious disease.
- Handle and discard in accordance with accepted medical practice and applicable regulations. After use, this product is a potential biohazard.

CAUTIONS:

- Carefully read and follow all instructions prior to use.
- Care and adherence to universal precautions must be taken to avoid accidental needlestick injuries.
- Do not withdraw guide wire through the needle.
- Do not attempt to override or defeat the safety mechanism.
- Only qualified healthcare practitioners should insert, manipulate and remove these devices.
- Follow standard infection control precautions as specified by the CDC (USA) or local equivalent.
- Sterile and non-pyrogenic only if packaging is not opened, damaged or broken.
- **Be ONLY:** Federal (USA) law restricts the sale of this device to or on the order of a physician.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or crossinfection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

CONTRAINDICATIONS:

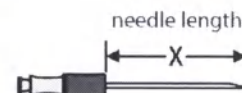
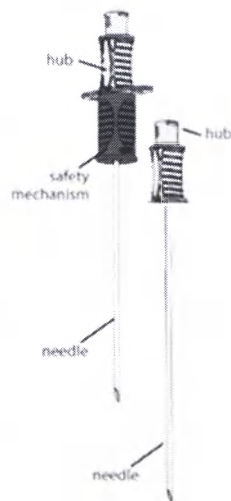
There are no contraindications associated with the Merit Advance Angiographic Needle.

INSTRUCTIONS FOR USE:

1. Identify the insertion site and prepare the site using proper aseptic technique and local anesthesia as required.
2. Remove the Merit Advance Angiographic Needle from package using proper aseptic technique.
3. Insert a Merit Advance Angiographic Needle using standard technique.
4. After use, advance the safety mechanism to the end of the needle until you feel the safety mechanism stop and the safety device locks over the needle tip.

Note: For guidewire applications, the safety mechanism may be activated before removing the needle from the guidewire.

5. Dispose of the Merit Advance Angiographic Needle in an approved sharps container while observing universal precautions.

DEVICE DESCRIPTION

ITEM	DESCRIPTION	
Needle Length	Advance:	4.0 cm (1 ½ inches) through 9.0 cm (3 ½ inches)
	Majestik:	3.8 cm (1 ½ inches) through 7.0 cm (2 ¾ inches)
Outside Diameter (OD)	Advance:	18G (1.2mm) through 22G (0.7mm)
	Majestik:	18G (1.2mm) through 22G (0.7mm)
Inside Diameter (ID)	Advance:	18G (1.2mm) through 22G (0.7mm)
	Majestik:	18G (1.2mm) through 22G (0.7mm)
Color coded hubs for gauge recognition	•	18G (1.2mm) needles are Pink or Clear
	•	19G (1.1mm) needles are Brown
	•	20G (0.9mm) needles are Yellow
	•	21G (0.8mm) needles are Green
	•	22G (0.7mm) needles are Blue

COMPONENT	MATERIAL SPECIFICATION
Merit Advance Angiographic Needles	
Needle Hub	Polycarbonate Medical Grade
Cannula	Stainless Steel AISI 304

PACKAGING

Product is packaged and labeled as sterile, single-use devices. Sterile barrier systems are specified per ISO 11607 to prevent ingress of microorganisms and allow aseptic presentation of product at point of use.

Applicable Symbols	
	This symbol means that the device fully complies with European Directive 93/42/EEC.
	Manufacturer
	Authorized Representative
	Single use
	Reference Number
	Lot Number
	Sterilization: Ethylene-Oxide Gas
	Caution: U. S. federal law restricts this device to sale by or on the order of a physician.
	Use by
	Caution: Consult Accompanying Documents
	Do not use if package is damaged.
	Non-pyrogenic

Use Conditions - in medical and preventive treatment facility.

SHELF LIFE

Shelf life - 3 years.

STERILIZATION

Ethylene oxide sterilization

DISPOSAL

Dispose the devices according to local environmental requirements.

Class of hazard for medical wastes by reference to specification of morphological composition (in accordance with Russian Sanitary Regulations and Standards 2.1.7.2790-10)
Class B

WARRANTY**Limited Warranty and Disclaimer**

Merit represents and warrants that the products shall materially conform to the specifications at the time of shipment to Customer and will remain materially free of defects in materials and workmanship for the period specified on the written limited warranty separately provided with each product. This warranty shall not apply to products that have been (i) modified, changed, repaired, refurbished, reprocessed, or altered by anyone other than Merit; (ii) subjected to misuse, mishandling, re-use, re-sterilization, accident, abuse, neglect or tampering; (iii) damaged by excessive physical, environmental or electrical stress; or that have had a serial number altered, defaced or removed; (iv) combined with any other product; (v) used outside the approved "Indications for Use" as cleared by the relevant competent authority, used contrary to the use outlined in the product specifications or in an application or environment for which such product was not designed or contemplated. Any technical product advice furnished by Merit is provided without duty or compensation and Merit assumes no obligation or liability, all such advice being given and accepted at Customer's risk. EXCEPT AS EXPRESSLY SET FORTH HEREIN MERIT MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, TO CUSTOMER, ITS END-USERS OR ANY THIRD PARTIES WITH RESPECT TO THE PRODUCTS. MERIT DISCLAIMS AND EXCLUDES ANY AND ALL OTHER EXPRESS, IMPLIED, AND STATUTORY WARRANTIES, INCLUDING WITHOUT LIMITATION, WARRANTIES AGAINST INFRINGEMENT, AND THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

Limitation of Liability

UNDER NO CIRCUMSTANCES SHALL MERIT BE LIABLE TO CUSTOMER OR ANY OTHER PERSON OR ENTITY FOR ANY PUNITIVE, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, WHETHER BASED UPON LOST GOODWILL, LOST RESALE PROFITS, WORK STOPPAGE, PRODUCT FAILURE, IMPAIRMENT OF OTHER PRODUCTS OR OTHERWISE AND WHETHER IN A CONTRACT, TORT OR OTHER ACTION FOR OR ARISING OUT OF BREACH OF WARRANTY, BREACH OF CONTRACT, DELAY, NEGLIGENCE, STRICT LIABILITY, OR OTHERWISE.

TRANSPORTATION, STORAGE AND OPERATION CONDITIONS

Transportation temperature	-35 °C to +55 °C
Storage temperature	+5 °C to +27 °C
Usage	+15 °C to +35 °C, relative humidity ≤80%, atm. Pressure 84.0 – 106.7 kPa (630-800 mm Hg).
Contact with the internal environment of the body at a temperature	32 °C - 42 °C

Customer Service

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095 U.S.A.
+ 1 801 253 1600

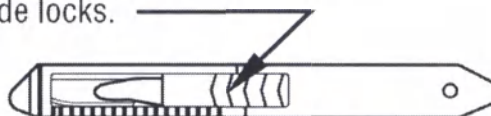
Merit Technologies LLC
119017 Moscow,
1st Kazachiy pereulok,
bld 7, 1 floor, room 6, Russian
Federation+ 7 495 221 89 02


Futura Safety Scalpel

INSTRUCTIONS FOR USE


Push forward with thumb
until blade locks.


OPENED LOCKED POSITION

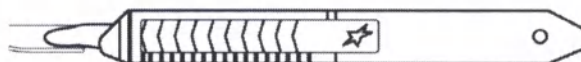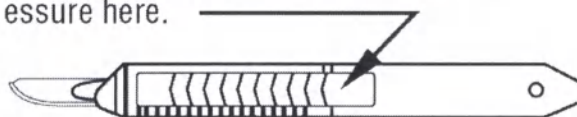Retract blade before passing scalpel
disposing scalpel. To retract apply
pressure here.

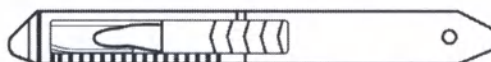Scalpel is now ready for passing
disposal.


WARNING: Used sharps are contaminated.
Handle carefully. Dispose scalpel in
accordance with all applicable local and
national laws and regulations.

Customer Service in the Russian Federation:
Limited Liability Company "Merit Technologies"
(Merit Technologies LLC), PSRN 1117746679533, TIN 7705960130
Legal address: 119017, Moscow, 1st Kazachiy pereulok,
bld 7, 1 floor, room 6
Tel / Fax: 7 (495) 221-89-02

Futura Safety Scalpel
Registration certificate № FSZ 2012/12606 dated 29.12. 2017

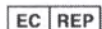Merit Medical Systems, Inc. South Jordan, Utah 84095 • www.merit.com
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Merit Medical Ireland Ltd. Parkmore Business Park West, Galway, Ireland
European Customer Service by Country: Belgium 0800 72906; France 0800 916030;
Germany 0800 1820871; Ireland 091 703700; Neth. 0800 0228184; U.K. 0800 973115

INSTRUCTIONS FOR USE

DESCRIPTION

Temporary sharps holder.

INDICATIONS FOR USE

The ShortStop temporary needle/scalpel holder is intended to prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION

Do not reuse. All needles should be flushed prior to use.

INSTRUCTIONS FOR USE

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop temporary sharps holder in the sterile field with the foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.
2. After use, the clinician may temporarily place the sharps device in the foam.
3. After completion of procedure, dispose of the sharps using universal precautions. Place the needle/scalpel in an OSHA compliant waste disposal system. After the sharps are removed from the foam, place the used ShortStop device in hospital compliant waste disposal system.

Device is sterilized as stated on the package label.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

	Sterile if package is unopened and undamaged
	Non-Pyrogenic
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Sterilized using Irradiation
	Caution: Consult accompanying documents.
	Catalog number
	Lot number
	Use by date: year-month-day
	Single use
	Manufacturer: Name & Address
	Authorized Representative in the European Community

Customer Service in the Russian Federation:
Limited Liability Company "Merit Technologies"
(Merit Technologies LLC), PSRN 1117746679533, TIN 7705960130
Legal address: 119017, Moscow, 1st Kozachly pereulok, bld 7, 1 floor, room 6

Tel / Fax: +7 (495) 221-89-02

Sharps Holder ShortStop
Registration certificate № FSZ 2012/12606 dated 29.12.2017

CE 0086

www.merit.com

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A. Customer Service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West
Galway, Ireland
EC Customer Service +31 43 3588222

400643003EN_002 ID 2017-08-15

INSTRUCTIONS FOR USE

DESCRIPTION

Temporary sharps holder.

INDICATIONS FOR USE

The ShortStop temporary needle/scalpel holder is intended to prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION

Do not reuse. All needles should be flushed prior to use.

INSTRUCTIONS FOR USE

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop temporary sharps holder in the sterile field with the foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.
2. After use, the clinician may temporarily place the sharps device in the foam.
3. After completion of procedure, dispose of the sharps using universal precautions. Place the needle/scalpel in an OSHA compliant waste disposal system. After the sharps are removed from the foam, place the used ShortStop device in hospital compliant waste disposal system.

Device is sterilized as stated on the package label.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

	Sterile if package is unopened and undamaged
	Non-Pyrogenic
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Sterilized using Irradiation
	Caution: Consult accompanying documents.
	Catalog number
	Lot number
	Use by date: year-month-day
	Single use
	Manufacturer: Name & Address
	Authorized Representative in the European Community

Customer Service in the Russian Federation:
Limited Liability Company "Merit Technologies"
(Merit Technologies LLC), PSRN 1117746679533, TIN 7705960130
Legal address: 119017, Moscow, 1st Kazachiy pereulok, bld 7, 1 floor, room 6

Tel / Fax: +7 (495) 221-89-02

Sharps Holder ShortStop
Registration certificate № FSZ 2012/12606 dated 29.12.2017

CE 0086

www.merit.com

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A. Customer Service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West
Galway, Ireland
EC Customer Service +31 43 3588222

400643003EN_002 ID 2017-08-15

INSTRUCTIONS FOR USE

DESCRIPTION

Temporary sharps holder with multiple access points.

INDICATIONS FOR USE

The ShortStop temporary sharps holder with multiple access points is intended to help prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION

- **RU ONLY:** Federal (USA) law restricts the sale of this device to or on the order of a physician.
- All needles should be flushed prior to use.
- When inserting sharps into the temporary sharps holder, do not support the device with your hands as inadvertent needle sticks may occur.
- Needle bevel contact with the plastic wall of the device, may result in damage to the needle.
- Sterile and non-pyrogenic only if packaging is unopened and undamaged.

REUSE PRECAUTION STATEMENT


For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of the infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

INSTRUCTIONS FOR USE

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop Advantage in the sterile field with the printed foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.
2. After use, the clinician may temporarily place a sharp device in either the top access compartment or side access compartment of the temporary sharps holder.
3. After completion of procedure, dispose of the sharp devices using universal precautions. Place the sharp devices in an OSHA compliant waste disposal system. After the sharp devices have been removed from the foam, place the used ShortStop Advantage device in a hospital compliant waste disposal system.

	Sterile if package is unopened and undamaged
	Non-Pyrogenic
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Sterilized using Irradiation
	Caution: Consult accompanying documents.
	Catalog number
	Lot number
	Use by date: year-month-day
	Single use
	Manufacturer: Name & Address
	Authorized Representative in the European Community

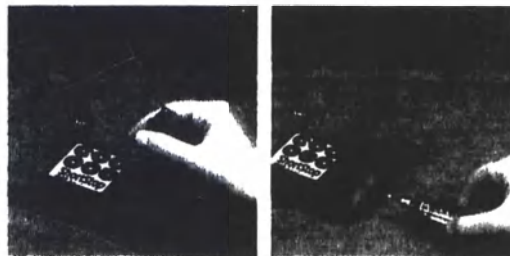Customer Service in the Russian Federation:
Limited Liability Company "Merit Technologies"
(Merit Technologies LLC), PSRN 1117746679533, TIN 7705960130
Legal address: 119017, Moscow, 1st Kazachiy pereulok, bld 7, 1 floor, room 6
Tel / Fax: 7 (495) 221-89-02

Sharps Holder ShortStop Advantage
Registration certificate № FSZ 2012/12606 dated 29.12.2017

CE 0086

MA **MERITMEDICAL**

www.merit.com

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A Customer Service 1-800-356-3748

EC REP

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland
EC Customer Service +31 43 3580222

400769006EN_002 2017-03-22

ShortStop GRANDSTAND

INSTRUCTIONS FOR USE:

DESCRIPTION: Temporary sharps holder

INDICATIONS FOR USE: The ShortStop temporary needle/scalpel holder is intended to prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION: Do not reuse. All needles should be flushed prior to use. Not intended to contain fluid.

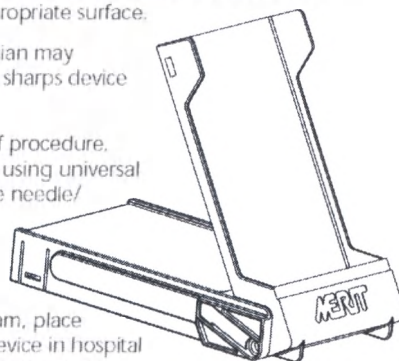
INSTRUCTIONS FOR USE: Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop temporary sharps holder in the sterile field with the foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.


2. After use, the clinician may temporarily place the sharps device in the foam.

3. After completion of procedure, dispose of the sharps using universal precautions. Place the needle/scalpel in an OSHA compliant waste disposal system.

After the sharps are removed from the foam, place the used ShortStop device in hospital compliant waste disposal system.

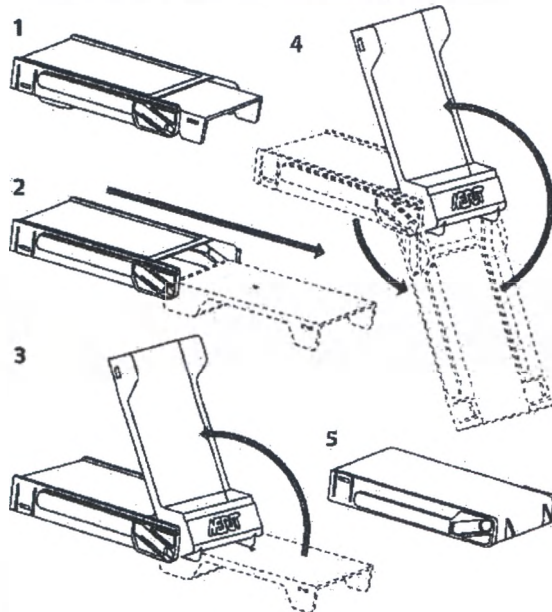Device is sterilized as stated on the package label.

Customer Service in the Russian Federation:

Limited Liability Company "Merit Technologies"

(Merit Technologies LLC), PSRN 1117746679533, TIN 7705960130

Legal address: 119017, Moscow, 1st Kazachiy pereulok, bld 7, 1 floor, room 6

Tel / Fax: 7 (495) 221-89-02

Sharps Holder Grandstand

Registration certificate № FSZ 2012/12606 dated 29.12.2017

 **MERITMEDICAL**

 0123

Merit Medical Systems, Inc., South Jordan, Utah 84095 • www.merit.com
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Merit Medical Ireland Ltd. Parkmore Business Park West, Galway Ireland

Representative Customer Service Country: Belgium 0800 72905, France 0800 916030;

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