POLIDENT d.o.o. Industrija dentalnih proizvodov - Dental Products Industry

Volčja Draga 42, 5293 Volčja Draga, SLOVENIJA
P: +386 (0)5 330 48 40 polident@polident.si
F: +386 (0)5 330 48 70 www.polident.si

INSTRUCTIONS FOR USE "Artificial Acrylic Teeth"

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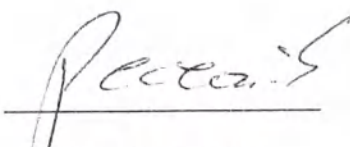
Dne: 13.11.2018

Branka Pahor

Instructions for Use for the medical product “Artificial acrylic Teeth”

Replacing missing teeth by making total and partial dentures.

	31	II	T1, SI
	32	I2, I3, L2	T1, T2, SI
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
3	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
	32I	I2, I3, L2	T1, T2, SI
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	0I	II	T1, SI
	02	II, I3, L2	T1, T2, T3, SI
	03	I3, I4	T3, T4, SI
0	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	4I	II	T1, SI
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
4	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

Potential customer: dental technicians, dentists. During the production of dental prostheses, the usual working procedures of the dental technician are required, so there is no need for detailed operating instructions.

Indications for use:

Replacing missing teeth by making total and partial dentures.

Contraindications:

There is no known contraindication to the use of artificial acrylic teeth.

Precautions:

The products are not marked with safety requirements, warnings or precautions, due to lack thereof.

Materials:

PRIMODENT artificial acrylic teeth consist of three layers. The composition is strongly cross-linked polymethyl methacrylate and pigments. The main raw materials used for polymerization are: polymethyl methacrylate powder, methyl methacrylate monomer, ethylene glycol dimethacrylate, peroxide and pigments. Teeth of the shade guide are made of the same material as all **PRIMODENT** teeth.

Release form:

PRIMODENT artificial acrylic teeth are available in the following design variants:

- A8 (upper lateral teeth) 8 pcs
- A8 (lower lateral teeth) 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth) 6 pcs
- A28 (full set: lateral, front teeth) 28 pcs

Accessories (optional):

- **PRIMODENT** shade guide
- **PRIMODENT** V shade guide

Packaging:

Each carrier with the teeth is packaged into carton box **PRIMODENT**. The number of teeth carriers in each box are:

- upper and lower anterior teeth: 32 pieces
- upper and lower posterior teeth: 22 pieces
- 4 complete (1 piece upper anterior teeth carrier, 1 piece lower anterior teeth carrier, 1 piece upper posterior teeth carrier, 1 piece lower posterior teeth carrier).

Minimum delivery quantity:

- A8 (upper lateral teeth): 8 pcs
- A8 (lower lateral teeth): 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth): 6 pcs

PRIMODENT synthetic polymer teeth are supplied complete with instructions for use.

Transport and storage:

There are no special transportation and storage conditions required. The products shall be stored in their original packaging, in a clean, dry place away from sunlight or a heat source.

Manufacturer:

POLIDENT d.o.o., Industrija dentalnih proizvodov, Dental Products Industry (POLIDENT d.o.o.)
Address: Volcja Draga 42, Volcja Draga, Slovenia
Tel: +00386 (0) 5 330 48 40
Fax: 00386 (0) 5 330 48 70
E-mail: polident@polident.si

Authorized representative of the manufacturer in the Russian Federation:

Joint Stock Company Omni Med (AO Omni Med)
Russia, 115598, Moscow, ul. Zagoryevskaya, 10, korp. 4, Et. tsokolnyy pom. I K 7-2 OF7
Tel: +7 (495) 255-16-13, +7 (499) 235-11-74
E-mail: 4547170@gmail.com

Warranty

The "Polident" company incurs liability for the products in the event of any failure in the products. The manufacturer's warranty liabilities are not available if the product is supplied in accordance

with the product quality characteristics and the required quantity.

Service life and shelf life

The term of storage of synthetic polymer teeth is unlimited if the products are processed and stored in normal and prescribed conditions.

Life in the human body (in the oral cavity) depends not only on the properties of the material teeth and the shape of teeth. There are many other factors that can affect the "life" of the product in the mouth, especially due to clinical abrasion, the state of the patient's mouth (pH value, smoking), effectiveness and strength of chewing, material and position of the antagonist, abrasiveness of the material in toothpastes and cleaning agents, the presence of bruxism, etc. Therefore, the manufacturer does not define an exact service life.

Criteria for the unfitness of the medical device for use

The product is a single use one and cannot be reused.

Disposal of the medical product:

The used products shall be disposed of in accordance with the current legislation as epidemiologically hazardous waste.

Products not used for their intended purpose for any reason belong to the class of epidemiologically safe waste.

Marking

Like the other products of the Polident Company, **PRIMODENT** teeth have a CE marking (*EU certification*):

Symbols used in the labelling of medical products

Manufacturer

Refer to the instructions for use

Catalog number

Batch code

**CROSS LINKED 2****Instructions for Use
for the medical product "Artificial acrylic Teeth"**

Artificial acrylic teeth **CROSS LINKED 2** are designed to be used by professionals : dental technicians and dentists in all types of prosthetic dentures and they perfectly imitate natural teeth. During denture manufacturing the common dental technician's operating procedures are required and therefore there is no need for detailed operating instruction for work. Shapes of **CROSS LINKED 2** artificial acrylic teeth are accompanied by a shade guide used in the process of manufacturing prostheses.

Intended use:

Replacing missing teeth by making total and partial dentures.

The available wide range of shapes meets even the most stringent criteria.

	31 32 33 34 35 36 37	11 12, 13 12, 13 13, 14 13, 14, 15 14, 15 15, 16	T1 T1, T2 T2, T3 T2, T3, T4 T3, T4 T3, T4 T4, T6
	01 02 03 04 05 06	11, 13 13, 14 14, 15 15, 16 16, 17	T1 T1, T2, T3 T3, T4 T3, T5 T3, T4, T5 T5, T6
	41 42 43 44 45	11 13, 14 13, 14 15, 16 15, 16	T1 T2, T3 T3, T4 T3, T4, T5, T6 T4, T5

The teeth shall be installed on the carriers (carrier for the front teeth and carrier for the lateral teeth). Six front teeth shall be installed on each front carrier. Eight lateral teeth shall be installed on each posterior carrier.

Available shapes of CROSS LINKED 2:

- A6 (upper front teeth) 41, 42, 43, 44, 45, 01, 02, 03, 04, 05, 06, 31, 32, 33, 34, 35, 36, 37
- A6 (lower front teeth): 11, 12, 13, 14, 15, 16, 17
- A8 (upper lateral teeth): T1, T2, T3, T4, T5, T6
- A8 (lower lateral teeth): T1, T2, T3, T4, T5, T6.

CROSS LINKED 2 artificial acrylic teeth are available according to the **CROSS LINKED 2** shade guide.

CROSS LINKED 2 shade guide: BL3, A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4.

The teeth are made of two layers and the collection consists of 18 different moulds of upper anterior teeth which are subdivided into 3 groups (the oval, square and

triangular group) of 7 different moulds of lower anterior and of 6 different moulds of posterior teeth.

Operating conditions:

No special operating conditions.

Application conditions: teeth are intended for the use in dental clinics.

Potential customer: dental technicians, dentists.

During the production of dental prostheses, the usual working procedures of the dental technician are required, so there is no need for detailed operating instructions.

The shade guide allows to choose a suitable shade of teeth during the manufacturing of prostheses.

Indications for use:

Replacement of the patient's natural teeth by full and partial prosthetics

Contraindications:

There is no known contraindication to the use of artificial acrylic teeth.

Precautions:

The products are not marked with safety requirements, warnings or precautions, due to lack thereof.

Materials:

CROSS LINKED 2 artificial acrylic teeth consist of two layers. The composition is strongly cross-linked polymethyl methacrylate and pigments. The main raw materials used for polymerization are: polymethyl methacrylate powder, methyl methacrylate monomer, ethylene glycol dimethacrylate, peroxide and pigments. Teeth of the shade guide are made of the same material as all **CROSS LINKED 2** teeth.

Release form:

CROSS LINKED 2 artificial acrylic teeth are available in the following design variants:

- A8 (upper lateral teeth) 8 pcs
- A8 (lower lateral teeth) 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth) 6 pcs
- A28 (full set: lateral, front teeth) 28 pcs

Accessories (optional):

- **CROSS LINKED 2** shade guide

Packaging:

Each carrier with teeth is packed in a **CROSS LINKED 2** cardboard box. Maximum number of carriers for teeth in each box:

- upper and lower anterior teeth: 32 pieces
- upper and lower posterior teeth: 22 pieces
- 4 complete (1 piece upper anterior teeth carrier, 1 piece lower anterior teeth carrier, 1 piece upper posterior teeth carrier, 1 piece lower posterior teeth carrier)

Minimum delivery quantity:

- A8 (upper lateral teeth): 8 pcs
- A8 (lower lateral teeth): 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth): 6 pcs

CROSS LINKED 2 artificial acrylic teeth are supplied complete with instructions for use.

Transport and storage:

There are no special transportation and storage conditions required. The products shall be stored in their original packaging, in a clean, dry place away from sunlight or a heat source.

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Fax: 00386 (0) 5 330 48 70

E-mail: polident@polident.si

Authorized representative of the manufacturer in the Russian Federation:

Joint Stock Company Omni Med (AO Omni Med)

Russia, 115598, Moscow, ul. Zagoryevskaya, 10, korp. 4, Et. tsokolnyy pom. I K 7-2 OF7

Tel: +7 (495) 255-16-13, +7 (499) 235-11-74

E-mail: 4547170@gmail.com

Warranty

The "Polident" company incurs liability for the products in the event of any failure in the products. The manufacturer's warranty liabilities are not available if the product is supplied in accordance with the product quality characteristics and the required quantity.

Service life and shelf life

The term of storage of artificial acrylic teeth is unlimited if the products are processed and stored in normal and prescribed conditions.

Life in the human body (in the oral cavity) depends not only on the properties of the material teeth and the shape of teeth. There are many other factors that can affect the "life" of the product in the mouth, especially due to clinical abrasion, the state of the patient's mouth (pH value, smoking), effectiveness and strength of chewing, material and position of the antagonist, abrasiveness of the material in toothpastes and cleaning agents, the presence of bruxism, etc. Therefore, the manufacturer does not define an exact service life.

Criteria for the unfitness of the medical device for use

The product is a single use one and cannot be reused.

Disposal of the medical product:

The used products shall be disposed of in accordance with the current legislation as epidemiologically hazardous waste.

Products not used for their intended purpose for any reason belong to the class of epidemiologically safe waste.

Marking

Like the other products of the Polident Company, **CROSS LINKED 2** teeth have a CE marking (EU certification):

Symbols used in the labelling of medical products

Manufacturer

Refer to the instructions for use

Catalog number

Batch code

REF-LINE

Instructions for Use for the medical product "Artificial Acrylic Teeth"

Artificial acrylic teeth **REF-LINE** are designed to be used by professionals : dental technicians and dentists in all types of prosthetic dentures and they perfectly imitate natural teeth. During denture manufacturing the common dental technician's operating procedures are required and therefore there is no need for detailed operating instruction for work. Shapes of **REF-LINE** artificial acrylic teeth are accompanied by a shade guide used in the process of manufacturing prostheses.

Intended use:

Replacing missing teeth by making total and partial dentures.

The available wide range of shapes meets even the most stringent criteria.

		
M1	V1,V3	Q1
M3	V3,V4,U3	R2,R4,Q1,Q3,F2,F4
M4	V4,U3,U4	R2,R4,Q3,F2,F4
M6	V4,V6,V8,U4	R4,R6,Q5,F4,F6
N3	V3,V4,U3	R2,R4,Q3,F2,F4
N4	V4,V6,U3,U4	R2,R4,Q3,F2,F4
N6	V6,V8,U4,U6	R4,R6,Q5,F4,F6
W3	V3,V4,U3	R2,R4,Q3,F2,F4
W4	V4,V6,U3,U4	R2,R4,Q3,F2,F4
W6	V6,V8,U4,U6	R4,R6,Q5,F4,F6
H3	V3,V4,U3	R2,R4,Q3,F2
H4	V4,V6,U3,U4	R2,R4,Q3,F2,F4
H6	V6,V8,U4,U6	R4,R6,Q5,F4,F6

The teeth shall be installed on the carriers (carrier for the front teeth and carrier for the lateral teeth). Six front teeth shall be installed on each front carrier. Eight lateral teeth shall be installed on each posterior carrier.

Available shapes of REF-LINE teeth:

- A6 (upper front teeth) M1, M3, M4, M6, N3, N4, N6, W3, W4, W6, H3, H4, H6
- A6 (lower front teeth): V1, V3, V4, V6, V8, U3, U4, U6
- A8 (upper lateral teeth): R2, R4, R6, F2, F4, F6, Q1, Q3, Q5.
- A8 (lower lateral teeth): R2, R4, R6, F2, F4, F6, Q1, Q3, Q5.

REF-LINE artificial acrylic teeth are available according to the **REF-LINE** shade guide. - **REF-LINE** shade guide: BL1, BL3, A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4.

Operating conditions:

No special operating conditions.

Application conditions: teeth are intended for the use in dental clinics.

Potential customer: dental technicians, dentists.

During the production of dental prostheses, the usual working procedures of the dental technician are required, so there is no need for detailed operating instructions.

The shade guide allows to choose a suitable shade of teeth during the manufacturing of prostheses.

Indications for use:

Replacing missing teeth by making total and partial dentures and implant supported dentures

Contraindications:

There is no known contraindication to the use of artificial acrylic teeth.

Precautions:

The products are not marked with safety requirements, warnings or precautions, due to lack thereof.

Materials:

REF-LINE artificial acrylic teeth consist of four layers. The composition is strongly cross-linked polymethyl methacrylate containing inorganic filler and pigments. The main raw materials used for polymerization are: polymethyl methacrylate powder, methyl methacrylate monomer, ethylene glycol dimethacrylate and other crosslinking agents of dimethacrylates, silica dioxide, peroxide and pigments. Teeth of the shade guide are made of the same material as all **REF-LINE** teeth.

Release form:

REF-LINE artificial acrylic teeth are available in the following design variants:

- A8 (upper lateral teeth) 8 pcs
- A8 (lower lateral teeth) 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth) 6 pcs
- A28 (full set: lateral, front teeth) 28 pcs

Accessories (optional):

- **REF-LINE** shade guide

Packaging:

Each carrier with the teeth is packaged into carton box **REF-LINE**. The number of teeth carriers in each box are:

- upper and lower anterior teeth: 18 pieces
- upper and lower posterior teeth: 12 pieces.
- 1 complete (1 piece upper anterior teeth carrier, 1 piece lower anterior teeth carrier, 1 piece upper posterior teeth carrier, 1 piece lower posterior teeth carrier).

Minimum delivery quantity:

- A8 (upper lateral teeth): 8 pcs
- A8 (lower lateral teeth): 8 pcs
- A6 (upper front teeth) 6 pcs
- A6 (lower front teeth): 6 pcs

REF-LINE artificial acrylic teeth are supplied complete with instructions for use.

Transport and storage:

There are no special transportation and storage conditions required. The products shall be stored in their original packaging, in a clean, dry place away from sunlight or a heat source.

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Fax: 00386 (0) 5 330 48 70

E-mail: polident@polident.si

Authorized representative of the manufacturer in the Russian Federation:

Joint Stock Company Omni Med (AO Omni Med)

Russia, 115598, Moscow, ul. Zagoryevskaya, 10, korp. 4, Et. tsokolnyy pom. I K 7-2 OF7

Tel: +7 (495) 255-16-13, +7 (499) 235-11-74

E-mail: 4547170@gmail.com

Warranty

The "Polident" company incurs liability for the products in the event of any failure in the products. The manufacturer's warranty liabilities are not available if the product is supplied in accordance with the product quality characteristics and the required quantity.

Service life and shelf life

The term of storage of artificial acrylic teeth is unlimited if the products are processed and stored in normal and prescribed conditions.

Life in the human body (in the oral cavity) depends not only on the properties of the material teeth and the shape of teeth. There are many other factors that can affect the "life" of the product in the mouth, especially due to clinical abrasion, the state of the patient's mouth (pH value, smoking), effectiveness and strength of chewing, material and position of the antagonist, abrasiveness of the material in toothpastes and cleaning agents, the presence of bruxism, etc. Therefore, the manufacturer does not define an exact service life.

Criteria for the unfitness of the medical device for use

The product is a single use one and cannot be reused.

Disposal of the medical product:

The used products shall be disposed of in accordance with the current legislation as epidemiologically hazardous waste.

Products not used for their intended purpose for any reason belong to the class of epidemiologically safe waste.

Marking

Like the other products of the Polident Company, **REF-LINE** teeth have a CE marking (*EU certification*):

Symbols used in the labelling of medical products

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