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Apostille

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1. Country (Pays): United States of America

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2. has been signed by (a été signé par) Ka Lia Xiong

3. acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota

**4. bears the seal / stamp of (est revêtu du sceau / timbre de)
Ka Lia Xiong, Notary Public, State of Minnesota**

Certified Attesté

5. at (à) St. Paul, Minnesota

6. Date (Date) 07/07/2020

7. by (par) Secretary of State, State of Minnesota

8. File No (Sous n°) 1166494900059

9. Seal / Stamp (Sceau / Timbre):

10. Signature (Signature):

Steve Simon

Steve Simon

Secretary of State
State of Minnesota

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Angie Draper
Angie Draper
Sr. Regulatory Affairs Associate
3M ESPE Dental Products

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Ka Lia Xiong
Notary Public
My Commission Expires: January 31, 2023

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3M ESPE RelyX Veneer Cement

General Information

3M ESPE RelyX™ Veneer Cement is a radiopaque, color stable, light-cured resin cement indicated for bonding veneers fabricated of porcelain or composite. It is used in conjunction with 3M ESPE RelyX™ Try-In Pastes, Adper Single Bond 2 Adhesive, Single Bond Universal Adhesive. An additional silane primer is not required when using Single Bond Universal Adhesive. The RelyX Veneer cement contains a resin system consisting of a TEGDMA/ BisGMA blend and a zirconia/silica filler with a particle size range of 0.2–3.0 microns and volume percentage of approximately 47%.

RelyX™ Veneer cement is designed to be used in conjunction with Adper Single Bond 2 Adhesive, Adper Scotchbond Multi-Purpose Adhesive, or Single Bond Universal Adhesive and RelyX™ Try-In pastes. It is recommended that RelyX Veneer cement be used only with 3M ESPE adhesive systems where veneer bonding is indicated. Please read the instructions for use of the 3M ESPE adhesive system you use for additional safety and dispensing information.

Note: 3M ESPE disclaims liability for claims arising from the use of RelyX™ Veneer cement with any adhesive other than Adper Single Bond 2 Adhesive, Adper Scotchbond Multi-Purpose Adhesive, and Single Bond Universal Adhesive.

Relyx Veneer can be supplied as kit and as refills:

RelyX Veneer cement kit includes: 1) Syringes of RelyX Veneer cement, 5 pcs (A1, A3, WO, TR, B0,5); Syringes of Relyx Try-in paste, 5 pcs (A1T, A3T, WOT, TRT, B0,5T), Bottle of Single Bond Universal Adhesive, 1 pcs; Syringes of Scotchbond Universal Etchant, 1 pcs; Etchant dispensing tips, 25 pcs; Microbrushes, 50 pcs; Mixing wells, 48 pcs.

Relyx Veneer cement in refills: 1) 1 Syringe of Relyx Veneer cement (A1 shade); 2) 1 Syringe of Relyx Veneer cement (A3 shade), 3) 1 Syringe of Relyx Veneer cement (B0,5 shade); 4) 1 Syringe of Relyx Veneer cement (TR shade); 5) 1 Syringe of Relyx Veneer cement (WO shade).

Indications

Permanent cementation of ceramic or composite veneers.

Contraindications

There are no known contraindications.

Potential Adverse Events (Potential Complications)

Allergic reactions in patients with known acrylate allergies.

Based on current complaint history 3M ESPE has been notified of no reportable adverse event for this product.

Scope of use

Scope of use

Dentistry. For professional use only.

Precautionary Information for Patients

These products contain substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of these products in patients with known acrylate allergies. If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of these products.

Precautionary Information for Dental Personnel

These products contain substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the product contacts the glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

Please refer to the instructions for use of the 3M ESPE adhesive system you use for additional precautionary information.

Hazardous Information

ETCHANT PRECAUTIONS: Contact with eyes or skin may cause burns. In case of contact, flush area with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection.

Adper Single Bond 2 Adhesive, RelyX Ceramic Primer and Single Bond Universal Adhesive are highly flammable. Avoid static discharge. Keep away from heat, sparks, open flame, pilot light, and other sources of ignition.

Preparation

1. To prepare the veneer, the internal bonding surface of a ceramic veneer surface should be micro-etched with a hydrofluoric acid etchant. The dental laboratory typically does this. The internal surface of a composite veneer should be roughened by sandblasting or air abrasion.
2. Remove the provisional restoration, if present, from the tooth. Clean tooth with pumice; rinse thoroughly and lightly air dry.
3. Check the fit and esthetics of the restoration by trying in the veneer with water or the Translucent shade of RelyX Try-In paste. If the fit and esthetics are acceptable, proceed directly to step 5 using the Translucent shade of RelyX Veneer cement.
4. If color adjustment is needed, select the appropriate shade of RelyX Try-In paste. Apply RelyX Try-In paste directly onto the internal surface of the veneer. Gently seat the veneer onto the tooth preparation. With multiple veneers, start at the midline and seat veneers sequentially working back. The shade of RelyX Try-In paste is formulated to precisely match

the corresponding cured shade of the cement.

5. Once the fit and esthetics are verified, remove the veneers and thoroughly rinse the RelyX Try-In paste from the tooth and restoration with a water spray and then dry.
6. The internal surface of the veneer should be thoroughly cleaned prior to applying the silane or adhesive to remove any contaminants. The surface can be cleaned by applying etchant (34%-35% phosphoric acid) for 15 seconds followed by rinsing and drying.
 - 6.1. For Adper Single Bond 2 Adhesive and Scotchbond Multipurpose Plus Adhesive: Dispense RelyX Ceramic Primer (silane) into a mixing well. Brush a layer of the silane onto the internal surface of the veneer. Lightly air-dry the surface for 5 seconds to evaporate the solvent.
 - 6.2. For Single Bond Universal Adhesive: Use the disposable applicator to apply Single Bond Universal to the etched veneer. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.**
7. Clean the prepared teeth using flour of pumice slurry. Rinse and lightly dry. Isolate the area to prevent contamination. Place matrix strips between teeth to prevent unwanted bonding to adjacent teeth.

Directions

1. Etch the tooth preparation. Apply etchant to both dentin and enamel for 15 seconds. Rinse for 10 seconds. Blot excess water leaving tooth moist. If the tooth is accidentally dried, rewet the bonding surface with water for 15 seconds and blot excess leaving tooth moist. Alternatively, a HEMA based desensitizing agent can be applied to the etched dentin according to the manufacturer's instructions.
2. Apply the adhesive.
 - 2.1. For Adper Single Bond 2 Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. **Do not light cure.** Apply 1 coat of Adper Single Bond 2 Adhesive to the silane treated veneer. Dry gently for 5 seconds. **Do not light cure.**
 - 2.2. For Adper Scotchbond Multi-Purpose adhesive: Apply Adper Scotchbond Multi-Purpose Primer to etched enamel and dentin. Dry gently for 5 seconds. Apply Adper Scotchbond Multi-Purpose Adhesive to the primed enamel and dentin. **Do not light cure.** Apply 1 coat Adper Scotchbond Multi-Purpose Adhesive to the silane treated veneer. **Do not light cure.**
3. For Single Bond Universal adhesive: Use the disposable applicator to apply Single Bond Universal immediately to the entire surface of the enamel and dentin and rub it in for 20 sec. Avoid contact between the adhesive and the oral mucosa. If necessary, rewet the disposable applicator during treatment. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.** Apply a thin layer of the selected shade of RelyX Veneer cement directly from the syringe onto the bonding surface of the veneer. **Protect the cement from direct operator**

light exposure.

4. Seat the veneer in place using gentle pressure. Excess cement will extrude out around the margins. Spot cure the veneer into place on the facial surface away from the margins using a small diameter light guide for 20 seconds to secure it in place.
5. Remove the excess cement from the margins using a blunt instrument or dry brush. Alternatively, the excess cement can be "tack-cured" for 3 seconds with a standard halogen light to allow the excess to gel and allow for easier cleanup.
6. Light cure the labial, lingual, interproximal and occlusal surfaces for 30 seconds each. The A5 Opaque/Dark shade will require a 40 second exposure for each surface. Curing recommendations will allow for cement cure depths not less than 0.5 mm for the opaque shades and not less than 1.5 mm for the nonopaque shades. **Note: The cure times are indicated for a standard halogen curing light with a minimum light output of 400 mW/cm².**
7. Remove matrix strips. Finish marginal areas with Sof-Lex Finishing and Polishing Strips and Discs, manufactured by 3M ESPE. Check and adjust occlusion as necessary.

Storage and use

This product is designed to be used at room temperature. The humidity and pressure are not regulated. If stored in cooler allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for expiration date.

Please refer to the instructions for use of the 3M ESPE adhesive system you use for specific storage and use information. Do not expose materials to elevated temperatures or intense light.

Do not store materials in proximity to eugenol-containing products.

Technical characteristics:

Relyx Veneer cement syringe: Length (mm): 117,5±11 (filled full assembly), Width (mm): 31,75 ± 3; Height (mm): 12,7 ± 1,6; Diameter (mm): 8,9 ± 1; Mass (g): 7,7 ± 5% (filled assembly).

Relyx Veneer cement cap: Length (mm): 12,7 ± 1,6, Diameter (mm): 4,8 ± 1; Mass (g): 0,24 ± 0,1 (cap only).

Relyx Veneer Try-in paste syringe: Length (mm): 117,5±11 (filled full assembly), Width (mm): 31,75 ± 3; Height (mm): 12,7 ± 1,6; Diameter (mm): 8,9 ± 1; Mass (g): 6,8 ± 5% (filled assembly).

Relyx Try-in Paste cap: Length (mm): 12,7 ± 1,6, Diameter (mm): 4,8 ± 1; Mass (g): 0,24 ± 0,1 (cap only).

Scotchbond Universal Etchant syringe: Overall Length (mm): 77,4±10%; Cylinder length (mm): 63,65 ± 0,98; Diameter (mm): 7,7 ± 5%; Mass (g): 8,0 ± 1.

Single Bond Universal Adhesive bottle: Length (mm): 59 ± 1, Diameter (mm): 21 ± 1; Mass (g): 8,0 ± 1.

Etchant dispensing tips: Length (mm): $30,0 \pm 1,2$; Diameter (mm): $6,90 \pm 0,30$; Mass (g): $0,20 \pm 0,02$.

Microbrushes: Length (mm): 84 ± 1 ; Diameter (mm): $2,0 \pm 0,5$; Mass (g): $0,30 \pm 0,05$.

Mixing wells: Length (mm): $46,05 \pm 5\%$; Width (mm): $28,58 \pm 5\%$; Height (mm): $9,52 \pm 5\%$; Thickness (mm): $1,27 \pm 5\%$.

List of standards: ISO 10993-1: 2003(E), ISO 10993-3: 2003, ISO 10993- 5: 1999, ISO 10993-6: 2007, ISO 10993-10: 2002, ISO 10993-11: 2006, ISO 7405: 1997, ISO 13485: 2003 +Cor, 1:2009, ISO 14971: 2000, ISO 4049: 2009, ISO 15223-1: 2012, EN 980: 2008, EN 1041: 2008/ Amd 1:2013

Labelling information

Products are supplied in packaging labeled with the following information: Product name and REF number; name and address of the manufacturer; Do not reuse; European Authorized Representative sign; Contents; LOT number; Storage conditions; Shelf life; Information about trademarks and symbols - Caution, See accompanying document, Symbol CE; Flammable; Corrosive substance; Ecological packaging, Recycling sign.

Cleaning and Disinfection

Disinfect the products using an intermediate level disinfection process (liquid contact) as recommended

Medicinal substances: Medical devices do not contain or deliver medicinal substances.

Human Blood Derivatives: Medical devices do not contain or deliver human blood derivatives.

Animal-Derived Substances: No materials of human origin are intentionally being added to the products.

Disposal methods

Dispose of content/container in accordance with the local/regional/national/international/ regulations.

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Maintenance and repair

This medical device isn't subject to maintenance and repair.

Manufacturer of the medical device

- Developer

3M ESPE Dental Products

Address: 2510 Conway Avenue, St, Paul, Minnesota, 55144-1000, USA

Phone: 651-7334365

- Manufacturer

3M ESPE Dental Products

Address: 2510 Conway Avenue, St, Paul, Minnesota, 55144-1000, USA

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Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Apostille

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1. **Country (Pays): United States of America**
This public document (Le présent acte public) Document from 3M
2. **has been signed by (a été signé par) Ka Lia Xiong**
3. **acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota**
4. **bears the seal / stamp of (est revêtu du sceau / timbre de)**
Ka Lia Xiong, Notary Public, State of Minnesota

Certified Attesté

- | | |
|---|---|
| 5. at (à) St. Paul, Minnesota | 6. Date (Date) 07/07/2020 |
| 7. by (par) Secretary of State, State of Minnesota | 8. File No (Sous n°) 1166494900020 |
| 9. Seal / Stamp (Sceau / Timbre): | 10. Signature (Signature): |

Steve Simon
Secretary of State
State of Minnesota

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3M ESPE RelyX™ Try-In Paste

3M ESPE RelyX™ Try-In Pastes are used to guide the dentist in the selection of the shade required for the final cementation with 3M ESPE RelyX Veneer cement and RelyX Ultimate adhesive resin cement. The shades of RelyX Try-In paste specifically match the final cured shade of RelyX Veneer cement and RelyX Ultimate cement. RelyX Try-In paste contains a polyethylene glycol resin, a zirconia/silica filler and pigments. The RelyX Try-In pastes are water soluble for easy clean up.

Relyx Try-In Paste can be supplied as component of kit and as refills:

1) 1 Relyx Try-In Paste (A1T shade); 2) 1 Relyx Try-In Paste (A3T shade), 3) 1 Relyx Try-In Paste (B0,5T shade); 4) 1 Relyx Try-In Paste (TRT shade); 5) 1 Relyx Try-In Paste (WOT shade).

Indications

- Color matching guide for RelyX Veneer cement and RelyX Ultimate adhesive resin cement.

Instructions for use

Preparation

1. To prepare the veneer, the internal bonding surface of a ceramic veneer surface should be micro-etched with a hydrofluoric acid etchant. The dental laboratory typically does this. The internal surface of a composite veneer should be roughened by sandblasting or air abrasion.
2. Remove the provisional restoration, if present, from the tooth. Clean tooth with pumice; rinse thoroughly and lightly air dry.

Directions

1. Check the fit and esthetics of the restoration by trying in the veneer with water or the Translucent shade of RelyX Try-In paste. If the fit and esthetics are acceptable, proceed directly to the bonding procedure using the Translucent shade of RelyX Veneer cement.
2. If color adjustment is needed, select the appropriate shade of RelyX Try-In paste. Apply RelyX Try-In paste directly onto the internal surface of the veneer. Gently seat the veneer onto the tooth preparation. With multiple veneers, start at the midline and seat veneers sequentially working back. The shade of RelyX Try-In paste is formulated to precisely match the corresponding cured shade of the cement.
3. Once the fit and esthetics are verified, remove the veneers and thoroughly rinse the

RelyX Try-In paste from the tooth and restoration with a water spray and then dry.

4. The veneer is now ready for final bonding and cementation. Refer to the appropriate product's instruction for use.

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Do not expose the paste to elevated temperatures.

Technical characteristics:

Relyx Veneer Try-in paste syringe: Length (mm): 117,5±11 (filled full assembly), Width (mm): 11,75 ± 3; Height (mm): 12,7 ± 1,6; Diameter (mm): 8,9 ± 1; Mass (g): 6,8 ± 5% (filled assembly).

Relyx Try-in Paste cap: Length (mm): 12,7 ± 1,6, Diameter (mm): 4,8 ± 1; Mass (g): 0,24 ± 0,1 (cap only).

List of standards: ISO 10993-1: 2003(E), ISO 10993-3: 2003, ISO 10993- 5: 1999, ISO 10993-6: 2007, ISO 10993-10: 2002, ISO 10993-11: 2006, ISO 7405: 1997, ISO 13485: 2003 Cor, 1:2009, ISO 14971: 2000, ISO 4049: 2009, ISO 15223-1: 2012, EN 980: 2008, EN 1041: 2008/ Amd 1:2013

Labelling information

Products are supplied in packaging labeled with the following information: Product name and EF number; name and address of the manufacturer; Do not reuse; European Authorized representative sign; Contents; LOT number; Storage conditions; Shelf life; Information about trademarks and symbols - Caution, See accompanying document, Symbol CE; Flammable; Corrosive substance; Ecological packaging, Recycling sign.

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Developer

3M ESPE Dental Products

Address: 2510 Conway Avenue, St. Paul, Minnesota, 55144-1000, USA

Phone: 651-7334365

Manufacturer

3M ESPE Dental Products

Address: 2510 Conway Avenue, St. Paul, Minnesota, 55144-1000, USA

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Scotchbond™ Universal Etchant

Product Description

Scotchbond™ Universal Etchant is an etching gel for etching enamel and dentin using the acid etching technique. Scotchbond Universal Etchant contains 32% phosphoric acid by weight.

Indications

- Etching of enamel and dentin using the acid etching technique.

Precautionary Measures

Scotchbond Universal Etchant: Contact with eyes or skin may cause burns. Wear suitable protective clothing, gloves and eye/face protection. In case of contact, flush area with plenty of water and seek medical advice.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- ▶ Follow the customary procedures to clean unprepared tooth structure (e.g., with pumice paste, polishing paste, or power blasting) and rinse thoroughly with a water spray.
 - Cleaning pastes containing fluoride or oil should not be used because they can impair the effectiveness of the etching process.
- ▶ When working with prepared tooth structure, use a water spray to remove any loose preparation debris and dry the die or cavity with water- and oil- free air or with cotton pellets. Do not overdry.
- ▶ A dry area for treatment is a prerequisite for acid etching. We recommend using a rubber dam to ensure that the area remains adequately dry.

Application

- ▶ Remove the Scotchbond Universal Etchant syringe from the foil pouch, unscrew the cap, and attach an Etchant Dispensing Tip.
- ▶ Apply the etching gel to the precise area of the enamel and/or dentin and allow to react for 15 sec. Rinse thoroughly for 15 sec with water, using a suction at the same time.
- ▶ Dry with a cotton pellet; do not overdry.
 - Depending on the adhesive system being used, air drying may not be recommendable. You will find precise instructions for drying in the instructions for use of the specific product.
- ▶ Avoid any contamination of the etched surfaces. If contamination occurs, repeat the etching, rinsing, and drying.
- ▶ Continue immediately with the bonding process.
- ▶ After application, remove the application tip from the syringe and replace the cap.

Hygiene

- ▶ Use common agents to clean Scotchbond Universal Etchant syringes and disinfect them in accordance with the commonly used hygienic regulations.

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Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is

responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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State of Minnesota - Secretary of State

This Certificate is not valid for use anywhere within the United States of America, its territories or possessions.

Apostille

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1. **Country (Pays): United States of America**

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2. **has been signed by (a été signé par) Ka Lia Xiong**

3. **acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota**

4. **bears the seal / stamp of (est revêtu du sceau / timbre de)**

Ka Lia Xiong, Notary Public, State of Minnesota

Certified

Attesté

5. **at (à) St. Paul, Minnesota**

6. **Date (Date) 07/07/2020**

7. **by (par) Secretary of State, State of Minnesota**

8. **File No (Sous n°) 1166494900046**

9. **Seal / Stamp (Sceau / Timbre):**

10. **Signature (Signature):**

Steve Simon

Steve Simon

Secretary of State

State of Minnesota

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.

This Apostille does not certify the content of the document for which it was issued.

To verify the issuance of this Apostille, see <https://apostille.sos.state.mn.us>.

This certificate does not constitute an Apostille under the Hague Convention of 5 October 1961, when it is presented in a country which is not a party to the convention. In such cases, the certificate should be presented to the consular section of the mission representing that country.

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To whom it may concern,

In my capacity as Senior Regulatory Affairs Associate, 3M ESPE Dental Products,
2510 Conway Avenue, St. Paul, MN, 55144-1000, USA, I certify that attached IFU for:

Single Bond Universal Adhesive

in English language is truthful and accurate.

Angie Draper

Angie Draper
Sr. Regulatory Affairs Associate
3M ESPE Dental Products

03 July 2020

Date

State of Minnesota)
County of Ramsey) SS

Subscribed and sworn to before me this 3rd day of July 2020.

Kalia Xiong
Kalia Xiong, Notary Public

My Commission Expires: January 31, 2023

Instruction for use

For component of dental materials introductory kit Relyx™ Veneer

Single Bond Universal Adhesive, Single Bond Universal DCA Adhesive

Product Description

Single Bond Universal is a single-component, light-curing adhesive which is available in L-Pop blisters for single dosing or in bottles for multiple doses.

Depending on the indication, the adhesive is used:

- In a "self-etching" procedure to enable the shortest possible treatment time and to minimize post-operative sensitivities.
- With selective enamel etching to maximize the adhesion to the tooth enamel and to minimize post-operative sensitivities.
- In a "total etching" procedure with a prior phosphoric acid etching step, e.g., if the etching cannot be restricted to the tooth enamel.

Single Bond Universal DCA activates the dual curing mechanism of Single Bond Universal and is mixed with the adhesive whenever dual-cure or self-cure composite filling materials, cements, or core build-up materials are being used. The addition of the activator to the adhesive is not required when using RelyX™ Ultimate Adhesive Resin Cement because the activator is already integrated into the cement.

These Instructions for Use must be kept for reference for the duration of product use. Please see the pertinent information for use for details on all of the products mentioned below.

Indications

- All classes of fillings (according to Black) with light-curing composite or compomer filling materials
- Cementation of indirect restorations when combined with RelyX Ultimate Adhesive Resin Cement
- Cementation of veneers when combined with RelyX Veneer Cement
- Bonding of core build-ups made of light-curing composite or core build-up materials
- Bonding of dual-cure cements and core build-up materials and self-cure composites when combined with Single Bond Universal DCA
- Repair of composite or compomer fillings
- Intraoral repair of composite restorations, porcelain fused to metal, and all-ceramic restorations without extra primer
- Root surface desensitization
- Sealing of cavities prior to cementation of amalgam restorations
- Sealing of cavities and preparation of tooth stumps prior to temporary cementation of indirect restorations
- Bonding of fissure sealants
- Protective varnish for glass ionomer fillings

Precautionary Measures

For Patients and Dental Personnel

- Single Bond Universal: contact with eyes may cause severe eye damage. Wear eye protection to prevent injury. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

For Patients

- Single Bond Universal contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.
- If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use.

For Dental Personnel

- Single Bond Universal contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured adhesive. If skin contact occurs, wash skin with soap and water.
- The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If Single Bond Universal contacts the glove, remove and discard the gloves. Wash your hands immediately with soap and water and re-glove.
- If allergic reaction occurs, seek medical attention as needed.
- Single Bond Universal and Single Bond Universal DCA are highly flammable. Avoid static electricity build-up. Keep away from heat, sparks, open flame, pilot lights, and other ignition sources.

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

Precautions during Processing

- Seat temporary restorations using a eugenol-free product (e.g., with RelyX™ Temp NE). The use of temporary cements containing eugenol can inhibit the polymerization process of Single Bond Universal during the final cementing.
- For the best possible bond, do not treat the surface of Zirconia, aluminum oxide, or metal restorations with phosphoric acid.

Pulp Protection

To avoid pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material. To avoid bite increases during the cementing of the later restoration, the pulp protection should be applied prior to taking an impression for the final restoration.

Tooth Preparation

Preparation of the Cavity/Tooth Stump

- Remove loose preparation debris by spraying with water, and lightly air dry the stump or the cavity in only 2-3 bursts of water-free and oil-free air, or use cotton pellets to dry it off. Do not overdry.

- The cavity should be just dry enough that the surface has a slightly glossy appearance. Overdrying can lead to post-operative sensitivity.

We advise against the use of substances such as desensitizers, disinfectants, astringents, dentin sealants, rinsing solutions containing EDTA, hydrogen peroxide, etc. (with the exception of chlorhexidine solutions) after the final cleaning. The residues of these substances can be detrimental to the bonding strength and setting reaction of the adhesive, and especially of dual- and self-cure materials.

Unprepared Tooth Enamel

- Clean the tooth enamel as usual (e.g., with polishing paste or powder blasting), and thoroughly rinse by spraying with water.
- Then etch and rinse, see section "Selective Enamel Etching".
- Continue with the application of Single Bond Universal Adhesive.

Etching of the Tooth Structure (optional)

Clinically adequate adhesive bonds are achieved by the application of the adhesive. Even higher adhesive values can be achieved by selectively etching the enamel or using the "total etching procedure".

Selective Enamel Etching

Accidental etching of the dentin does not have any detrimental effect on the adhesion.

- Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond™ Universal - Etchant to the prepared and unprepared (if present) tooth enamel and allow to react for 15 sec.
- Rinse thoroughly with water and dry with water-free and oil-free air or with cotton pellets; do not overdry.

Total Etching Procedure

- Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond Universal Etchant, to the prepared and unprepared (if present) tooth structure (enamel and dentin) and allow to react for 15 sec.

- Rinse thoroughly with water and dry with water-free and oil-free air or with cotton pellets; do not overdry.

Root Surface Desensitization

- Clean the root surface as usual (e.g., with polishing paste or powder blasting).

- Clean thoroughly by spraying with water, and dry with water-free and oil-free air or with cotton pellets.

- Continue with the application of Single Bond Universal Adhesive.

Pre-treatment of Fillings or Restorations During Repair

(made of composite, glass ceramic, Zirconia ceramic, aluminum oxide ceramic, metal (incl. amalgam))

- Thoroughly roughen the surface of the restoration which is to be luted, preferably by blasting with aluminum oxide $\leq 40 \mu\text{m}$; if ceramic, remove the glaze up to 1 mm over the margin.

- Clean thoroughly by spraying with water, and dry with water-free and oil-free air or with cotton pellets.

- Do not etch with phosphoric acid because this can weaken the adhesion to certain materials (e.g., Zirconia, aluminum oxide, metal).

- Continue immediately with the application of Single Bond Universal Adhesive. The use of an additional primer is unnecessary.

Dosing Single Bond Universal Adhesive

Dosing from the Bottle

The dosing bottle cap has a thumb depression so that it can be opened to one side and closed again with one hand. The cap will remain open if the opening angle is $>90^\circ$.

- To open, take the bottle in your hand, place your thumb in the thumb depression, and push the cap to the side until it remains in the open position.

- Hold the bottle upside down in a vertical position and pour the required quantity of Single Bond Universal in a mixing well. Protect the adhesive in the mixing well from light.

- After use, carefully close the bottle again by applying pressure to the lever from above.

- The cap tangibly snaps into place when pressed closed and a small, even gap between the cap and bottle can be seen all the way around.

Dosing from the L-Pop Blister

Attention: Do not squeeze the L-Pop blister without the disposable applicator.

- Hold the shaft of the disposable applicator with one hand, covering the blister opening with your thumb. This will prevent the etching liquid from spraying out.

- With the thumb and index finger of the other hand, squeeze the reservoir of the L-Pop blister, starting with the outer end, in the direction of the disposable applicator.

- Turn the disposable applicator back and forth in the liquid to cover it completely.

- For easy application in the cavity, the disposable applicator can be bent while it is still in the L-Pop blister. In order to do this, only pull the disposable applicator out of the blister far enough so that the narrow section of the applicator shaft can be seen. Fold the applicator shaft at this location over your thumbnail.

- During the treatment, press all of the liquid into the open depression end with your thumb and index finger to rewet the applicator.

Dosing Single Bond Universal DCA Activator

- Unscrew the lid of the bottle.

- Mix the required quantity of Single Bond Universal DCA in a ratio of 1-to-1 (drops) with Single Bond Universal in a mixing well and mix for 5 sec immediately before application. Protect the liquid in the mixing well from light.

- Carefully close the bottle tightly after dosing. If necessary, disinfect the bottle in accordance with local hygiene regulations.

Application

- Avoid any contamination with blood, saliva, or sulcus fluid during application. We recommend the use of a rubber dam to ensure that the treated area is kept sufficiently dry and to prevent exposure of any soft tissue.

Procedure for direct restorations with light-curing composites and compomers, intraoral repair using light-curing composites, root surface desensitization, sealing of cavities before filling with amalgam, sealing of cavities and stump preparations before temporary attachment of indirect restorations

- Use the disposable applicator to apply the adhesive to the entire tooth structure and **rub it in** for 20 sec. Avoid contact between the adhesive and the oral mucosa.
- If necessary, rewet the disposable applicator during treatment.
- Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely.
- Harden the adhesive with a commonly used curing light for 10 sec.
- As appropriate for the indication, continue with the desired material in accordance with the pertinent instructions for use.

Procedure for dual-cure and self-cure composite and core build-up materials and cements

When using RelyX Ultimate, the activator is not required because it has been integrated into the cement. Please follow the RelyX Ultimate instructions for use.

- Place one drop each of Single Bond Universal and Single Bond Universal DCA in a mixing well and mix for 5 sec.
- Use the disposable applicator to apply the mixture to the entire tooth structure and **rub it in** for 20 sec. Avoid contact between the adhesive and the oral mucosa.
- If necessary, rewet the disposable applicator during treatment.
- Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely.
- Harden the adhesive with a commonly used curing light for 10 sec.
- As appropriate for the indication, continue with the desired material in accordance with the pertinent instructions for use.

Procedure when cementing indirect restorations and veneers using RelyX Ultimate Adhesive Resin Cement

When using RelyX Ultimate, Single Bond Universal DCA is not required. Please follow the RelyX Ultimate instructions for use.

Use of Single Bond Universal as a primer for indirect restorations

When using RelyX Ultimate, Single Bond Universal DCA is not required. Please follow the RelyX Ultimate instructions for use.

- Prepare the surface of the restoration:
 - Etch glass ceramic surfaces with hydrofluoric acid in accordance with manufacturer's instructions.
 - Roughen composite, Zirconia ceramic, and metal surfaces, preferably by sandblasting.
 - Do not etch with phosphoric acid because this can weaken the bonding to certain materials (e.g., Zirconia, aluminum oxide, metals).
- Thoroughly rinse the surface with water and dry with water-free and oil-free air or with cotton pellets.
- In combination with other composite cements:
 - Place one drop each of Single Bond Universal and Single Bond Universal DCA in a mixing well and mix for 5 sec.
 - Immediately after mixing, use the disposable applicator to apply the adhesive to the entire surface of the restoration to be cemented and allow it to **react** for 20 sec. Do not light-cure.
 - Follow the instructions for use from the manufacturer to apply the cement.

Using RelyX Ultimate to cement posts

When using RelyX Ultimate, Single Bond Universal DCA is not required. Please follow the RelyX Ultimate instructions for use.

Using other composite cements to cement posts

- Prepare the endodontically treated root canal for the post cementing.
- Trial fit the post; it should fill 2/3 of the root canal.
- Place one drop each of Single Bond Universal and Single Bond Universal DCA in a mixing well and mix for 5 sec.
- Use the disposable applicator to apply the mixture to the entire tooth structure and **rub it in** for 20 sec.
- Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. Soak up excess liquid with a paper tip.
- Harden the adhesive with a commonly used curing light for 10 sec.
- Follow the instructions for use for the relevant product to cement the post.

Procedure when cementing veneers using 3M ESPE RelyX™ Veneer Cement

- The surface of the glass ceramic veneer which is to be cemented should be etched using hydrofluoric acid in accordance with manufacturer's instructions. An additional silane primer is not required when using Single Bond Universal.
- Before applying the adhesive, clean the prepared tooth structure with pumice paste, thoroughly rinse with water, and dry with water- and oil-free air or with cotton pellets.
- Trial fit the veneer using RelyX™ Try-In Paste. Use water to rinse off the paste from the tooth and veneer and dry all of the surfaces with water- and oil-free air or with cotton pellets. Isolate the adjacent teeth from the veneer preparation using transparent template strips.
- Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond Universal Etchant, to the enamel and dentin and allow to react for 15 sec. Then rinse thoroughly with water for 10 seconds and dry with cotton pellets. The surface should be shining and not have any puddles on it.
- Use the disposable applicator to apply Single Bond Universal immediately to the entire surface of the enamel and dentin and **rub it in** for 20 sec. Avoid contact between the adhesive and the oral mucosa. If necessary, rewet the disposable applicator during treatment. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.**
- Use the disposable applicator to apply Single Bond Universal to the etched veneer. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.**
- Carefully cement the veneer with RelyX™ Veneer Cement following the pertinent instructions for use. Remove any excess from the veneer edges and light-cure in accordance with manufacturer's instructions. We recommend light-curing the gingival edges first, then the middle and the incisal edges. Avoid eye contact with the light source.

Procedure for light-cure sealing of fissures

In contrast to a composite filling, light-cure fissure sealants can be bonded to untreated tooth enamel using Single Bond Universal without prior etching with phosphoric acid.

- Clean the surface to be sealed with polishing paste or by powder blasting (e.g., using Clinpro Prophyl Powder).
- Use the disposable applicator to apply Single Bond Universal to the surface to be sealed and **rub it in** for 20 sec, applying pressure. Continue to rub in for a longer period of time when sealing larger surfaces. **Do not reduce this time.** Applying and distributing the adhesive is not adequate.
- Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely.
- Apply a light-cure fissure sealant, e.g., Clinpro Sealant.
- Polymerize the fissure sealant and the adhesive together with a commonly available light-curing device; the time is dependent on the fissure sealant used.

Use as protective varnish for glass ionomer fillings

- If a template was used for the shaping, first remove the template.
- Use the disposable applicator to apply Single Bond Universal to the fully modeled filling and **rub it in** for 20 sec. Avoid contact between the adhesive and the oral mucosa.
- If necessary, rewet the disposable applicator during treatment.
- Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely.
- Use a commonly available light-curing device to polymerize the adhesive for 10 sec on every surface of the filling.
- When the filling has been completely cured, grind down the occlusion and reapply the adhesive to the ground places to protect the filling from drying out.
- Use a damp cloth or cotton pellet to remove the oxygen inhibition layer.

Hygiene and Disinfection

Clean any contaminated adhesive and activator bottles with commonly used agents and disinfect them in accordance with the commonly used hygienic regulations.

Notes

- Single Bond Universal can prematurely polymerize when exposed to natural or artificial lighting. Avoid intensive light exposure during application.

Storage and Stability

Store Single Bond Universal and Single Bond Universal DCA at 02-25°C/36-77° F.

Do not use after the expiration date.

Disposal methods: Dispose of content/container in accordance with the local/regional/national/international/regulations.

Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B.

Environmental precautions

Place in a metal container approved for transportation by appropriate authorities. Dispose of collected material as soon as possible.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

International symbols:

CE	CE Mark, conformity to European Directives
LOT	Batch code
REF	Part Number, Catalog Number

Temperature limitation Use by date

Caution, consult accompanying documents or Attention: see instructions for use

Do not re-use, single use or use only once

Consult instructions for use

Manufacturer

Information valid as of November 2019

P

Customer Service

Authorized representative of the manufacturer in the Russian Federation:

АО "3M Russia", 108811, Moscow, Poselenie Moskovskiy, Kievskoe shosse, 22 km. 6, bld.1.
Russia /phone +7 (495) 578-74-74

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