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The press

Clamps of the lower limb

Orthosis on the knee joint, elastic OK-11 (serial number 001)

Orthosis on the knee joint elastic with Velcro OK-12 (serial number 002)

Orthosis on the knee of high elasticity OK-13 (serial number 003)

Indications:

- prophylaxis of knee joint injuries in sports;
- bruises, knee joint injuries;
- extension of muscles and ligaments;
- pain syndrome in chronic joint diseases;
- Prevention of hypothermia of joints when working in difficult climatic conditions.

Result of application:

- heats the joint and adjacent tissues;
- has a micro massage effect and compression;
- provides the fixation of joints;

Reduces pain syndrome;

- unloads muscles and ligaments of the knee joint.

Features: made of elastic material, creating a compression above the 2nd class, ensuring a tight fitting of the product and the necessary impact on the joint. The web is made of special viscous with sections of different density according to the anatomical structure of the joint. Products OK-12 has one velcro-strap, which provides a more durable fixation. The fabric of the OK-13 product has a multidirectional netting, which ensures a perfect fit and the correct distribution of pressure. The products are breathable moisture separators that promote the intense evaporation of moisture from the surface of the skin, maintains heat and improves blood circulation in the joint.

The weight of the products is 40-55 gr. depending on the size.

Color: gray, beige, white, black, blue. Dimensions: XS, S, M, L, XL, XXL (measurement of the midpoint of the knee in the standing position, cm).

The composition of products OK-11, OK-12, OK-13: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the knee joint elastic, reinforced with stiffeners OKE-11 (serial number 004)

Orthosis on the knee joint is elastic, reinforced with stiffeners with Velcro OCE-12 (serial number 005)

Orthosis on the knee of high elasticity, reinforced with stiffeners OKE-13 (serial number 006)

Indications:

- prophylaxis of knee joint injuries in sports;

- bruises, knee joint injuries;

- extension of muscles and ligaments;

- pain syndrome in chronic joint diseases;

prevention of hypothermia of joints when working in difficult climatic conditions.

Result of application:

- heats the joint and adjacent tissues;

- has a micro massage effect and compression;

- provides the fixation of joints;

Reduces pain syndrome;

- unloads muscles and ligaments of the knee joint;

- helical stiffeners reduce impact loads on the knee joint.

Features: Made of elastic material, creating compression above the 2nd class, ensuring a tight fit of the product and the necessary impact on the joint. The canvas has an air permeable moisture-removing effect, is made by special viscous with sections of different density according to the anatomical structure of the joint. The product OKE-12 has one velcro-velcro, which provides a more durable fixation. The fabric of the product OKE-13 has a multidirectional netting, providing an excellent fit and the correct distribution of pressure. There are lateral stiffeners.

The weight of the products is 70-90 gr. depending on the size.

Color: gray, beige, white, black, blue. Dimensions: XS, S, M, L, XL, XXL (measurement of the midpoint of the knee in the standing position, cm).

The composition of the products: OKE-11, OKE-12, OKE-13: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the knee joint elastic with stiffeners and silicone pad OKS-11 (serial number 007)

Orthosis on the knee of a high elasticity with a silicone pad OKS-12 (serial number 008)

Orthosis on the knee joint of high elasticity with stiffeners and silicone pad OKS-13 (serial number 009)

Orthosis on the knee joint is tight-elastic with stiffeners and silicone pad OKS-14 (serial number 010)

Indications:

- bruises and sprains of ligaments of the knee joint;
- dislocation of the patella;
- inflammatory diseases of the knee and patella;
- rehabilitation after injuries of the knee and patella;
- prophylaxis of knee joint injuries during physical activity.

Result of application:

- heats the joint and adjacent tissues;
- has a micro massage effect and compression;
- provides the fixation of joints;

Reduces pain syndrome;

- unloads muscles and ligaments of the knee joint;
- helical stiffeners reduce impact loads on the knee joint;
- silicone pad fixes the knee cap.

Features: Made of elastic material, creating a compression above the 2nd class, ensuring a tight fitting of the product and necessary impact on the joint. The canvas has an air permeable moisture-removing effect, is made by special viscous with sections of different density according to the anatomical structure of the joint. The fabric OKS-12, OKS-13 has a multidirectional netting, which ensures perfect fit and proper distribution of pressure. OKS-14 exerts a rigid fixation and compression action on the knee joint, increasing its resistance to stress. Products OKS-11, OKS-13, OKS-14 are equipped with lateral stiffeners. The silicone pad fixes the patella.

The weight of the products is 85-105 gr. depending on the size.

Color: beige, gray, black, white, blue. Dimensions: XS, S, M, L, XL, XXL (measurement of the midpoint of the knee in the standing position, cm).

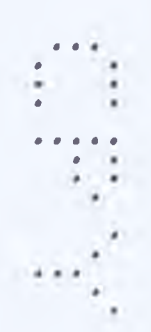
The composition of the products: 67% - nylon, 23% - elastic, 10% - lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Knee-type neoprene bracelet with silicone insert OKN-21 (serial number 011)

Neoprene knee brace with silicone insert and stiffeners OKN-22 (serial number 012)

Indications:

- inflammatory and degenerative diseases;
- prophylaxis of knee and patella injuries;
- bruises, sprains;
- boli in the knee joint;
- period of rehabilitation after knee joint injuries;
- arthritis-arthritis, bursitis, synovitis.

Result of application:

- fixation of the knee and patella, stabilization of the knee in the frontal plane, moderate compression;
- microphone action;
- heat-saving effect improves blood circulation;
- The elastic roller-ring stabilizes the patella;
- helical stiffeners in OKN-21 model reduce impact loads when walking.

Features: Made of porous aeroprene, possessing a heat-saving property, without interfering with skin breathing. The hole on the front surface of the bandage and the elastic roller-ring reliably limit the mobility of the patella. Spiral stiffeners in the product OKN-22 make it possible to use it for damages of the cruciate ligament of the knee joint.

The weight of products is 130-150 gr. depending on the size.

Color: beige, black, white, blue, gray. Size: XS, S, M, L, XL, XXL (measurement of the circumference of the middle of the knee in standing position, cm).

The composition of the products: 30% - neoprene, 27% - cotton, 23% - elastic, 20% - lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household rubbish

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for patella fixing OP-21 (serial number 013)

Orthosis for patella universal OP-22 (serial number 014)

Indications:

- osteochondropathy tuberosity of the tibia (Osgut-Schlatter disease);
- a habitual (repetitive) dislocation of the kneecap (patella);
- prophylaxis of patella dislocation with increased loads on the knee joint;
- Professional and sports support loads on the knee joint.

Result of application:

- Strengthens the support of the knee by fixing the tendons of the quadriceps muscle of the thigh;
- OP-21, additional tab provides stronger fixation of the patella;
- OP-22, the non-slip base maintains the correct compression in the position of the joint movement.

The weight of products is 35-40 gr. depending on the size.

Features: the product can be worn directly on the body. It consists of an elastic cuff, a silicone cushion on the region of tuberosity of the tibia and one tightening with a velcro closure.

Color: black, blue, beige, white, gray. Size: universal.

The composition of the products: 60% - polyester, 20% nylon, 12% - steel, 3% -cotton, 5% -polyvenyl chloride, one tape-Velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the knee joint with side plates OK-21 (serial number 015)

Orthosis on the knee joint with side springs OK-22 (serial number 016)

Indications:

- Bole syndrome with injuries and joint diseases;
- instability of the knee joint;
- traumas of the patella;
- damage to the meniscus;
- prophylaxis of diseases and injuries of the knee joint.

Result of application:

- The side plates (removable polycentric hinges) ensure normal movement of the knee joint;

-the motorization immobilizes (creates rest) the knee joint in the period of inflammatory disease, with acute trauma or after surgical treatment;

-The knee joint is fixed and unloaded;

-Compression and massage effects are provided;

A heat-saving effect provides increased blood circulation of muscles and improved absorption of drugs;

-orthosis helps reduce edema and improve blood supply to soft tissues adjacent to the joint.

The weight of the products is 330-450 gr. depending on the size.

Features: Made of hypoallergenic neoprene, which also has a pronounced heat-saving effect, which improves blood circulation, facilitates the rapid absorption of external medications. Pelot in the patella region fixes the joint. OK-21 has side plates provide lateral stabilization of the knee joint. Spiral stiffeners on the sides of the product OK-22 provide additional support and soft fixation of the knee joint and tendons. The degree of fixation is regulated by the tension of the two straps.

Color: black, gray, white, beige, blue. Size: S, M, L, XL, XXL (measurement of the midpoint of the knee in the standing position, cm).

The composition of the products: 60% - polyester, 20% nylon, 12% - steel, 3% -cotton, 5% polyvinyl chloride, tape-Velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the knee joint with side plates split OKP-21 (serial number 017)

Orthosis on the knee joint with side springs detachable OKP-22 (serial number 018)

Indications:

- Conservative treatment of arthrosis of the knee joint;

- instability of the knee joint;

-Stabilization after removal of plaster casts;

- rehabilitation after surgery;

-prophylaxis of joint injuries during exercises related to the load on the knee joint;

it is convenient to put on aorta with a significant difference in the circumference of the leg above and below the knee, with excess weight or back pain, and also with regular physiotherapy procedures.

Result of application:

- The side plates (removable polycentric hinges) ensure normal movement of the knee joint;

-the motorization immobilizes (creates rest) the knee joint in the period of inflammatory disease, with acute trauma or after surgical treatment;

- The knee joint is fixed and unloaded;

- Compression and massage effects are provided;

- A heat-saving effect provides increased blood circulation of muscles and improved absorption of drugs;

-orthosis helps reduce edema and improve blood supply to soft tissues adjacent to the joint.

Features: made of hypoallergenic neoprene, which also has a pronounced heat-saving effect, which improves blood circulation, promotes rapid absorption of external medications.

Pelot in the patella region fixes the joint. OKP-21 has side plates that provide lateral stabilization of the knee joint. Spiral stiffeners along the sides of the OKP-22 product provide additional support and soft fixation of the knee and tendons. The products are equipped with two velcro straps.

The detachable design facilitates the donning of the orthosis to the knee joint, including with strong edema and over bandages.

The weight of the products is 330-450 gr. depending on the size.

Color: black, gray, white, beige, blue. Dimensions OKP-21: XS, S, M, L, XL, XXL; OKP-22: S / M, L / XL (measurement of the circumference of the middle of the knee joint in the standing position, cm).

The composition of the products: 60% - polyester, 20% nylon, 12% - steel, 3% -cotton, 5% polyvinyl chloride, tape-Velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the knee joint with a flexion angle regulator, shortened OR-31 (serial number 019)

Orthosis on the knee joint with a flexor angle regulator, universal OR-32 (serial number 020)

Orthosis on the knee joint with a flexor angle regulator, elongated OR-33 (serial number 021)

Indications:

-chronic instability of the knee;

- injuries and ruptures of the lateral and cruciate ligaments of the knee joint;

-trauma of menisci;

- step-by-step adjustable expansion of the volume of movements in the knee joint for rehabilitation after injuries and restorative surgical operations;

-rehabilitation measures after knee arthroplasty.

Result of application:

-fixing the knee joint;

-Stabilization and the possibility of metered movements of the knee joint;

-with the help of a protractor, the parameters of the volume of movements are specified in the development of contractures of the knee joint of a mixed genesis.

Comprises:

- four cuffs made of synthetic material with a cloth coating on the shin and thigh;

- two metal tires with a hinge that regulates the volume of movements in 10-degree increments; Immobilizes the knee joint at an angle of 10, 20, 30 degrees. Sets the amount of motion in the range from 0 to 120 degrees.

four belts with Velcro fasteners

The retractable tire allows to place the orthosis on the lower limb: on the inner surface of the thigh - under the inguinal fold area, outside - to the region of the hip joint; The length of the tire is adjustable in the range of 52-72 cm in increments of 2.5 cm.

OR-31- The grip lever is reduced,

OR-33 - increased grip lever.

The weight of products is 1500-2000 gr. depending on the size.

Color: black, gray, blue, white, beige. Size: universal.

The composition of the products: 60% - polyester, 20% nylon, 12% - steel, 3% -cotton, 5% polyvinyl chloride, tape-Velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis stabilizing on the knee joint OKR-31 (serial number 022)

Orthosis stabilizing the knee joint elongated OKR-32 (serial number 023)

Indications:

-stability of the knee joint of a pronounced degree;

-infix deformations of the knee joint (valgus or Varus);

- patella injury;

- damage to the meniscus;

- State after operation on the knee joint and postoperative pain;
- rehabilitation after injuries and restorative operations on the meniscus;
- immobilization of the knee during transportation.

Result of application:

- the basis of the action of the orthosis is the principle of stabilization, which makes it possible to exclude, substantially reduce or correct the axial load on the joint.

Comprises:

- three parts made of synthetic material with a soft, pleasant to the touch fabric coating;
- the five metal tires of the aluminum alloy;
- Equipped with a system of four or six puffs (depending on the size), equipped with velcro fasteners, respectively.

due to the elongation of the web and the plates, provides greater girth and a more complete fixation of the joint, which allows you to leave the joint in complete immobility.

The weight of the products is 800-1100 gr. depending on the size.

Color: black, white, gray, beige, blue. Size: S, M, L, XL, XXL (measuring the circumference of the thigh, cm).

The composition of the products: 60% - polyester, 20% nylon, 12% - steel, 3% -cotton, 5% polyvinyl chloride, tape-Velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Specifications (admissions $\pm 10\%$)

name	Overall dimensions, cm	Weight, g	Color
OK-11, Size: XS, S, M, L, XL, XXL;	circle of the middle of a knee joint changes from 30 to 50 with walk 5, height 23	40-55 with walk 0,83 (depending on the Size)	Grey, beige, white, black, dark blue
OK-12, Size: XS, S, M, L, XL, XXL;			
OK-13, Size: XS, S, M, L, XL, XXL;			
OKE-11, Size: XS, S, M, L, XL, XXL;		70-90 with walk 3,33 (depending on the Size)	
OKE-12, Size: XS, S, M, L, XL, XXL			
OKE-13, Size: XS, S, M, L, XL, XXL;			

OKS-11, Size: XS, S, M, L, XL, XXL;	middle circle knee joint changes from 30 to 50 with walk 5, height 26	85-105 with walk 3,33 (depending on the Size)	
OKS-12, Size: XS, S, M, L, XL, XXL			
OKS-13, Size: XS, S, M, L, XL, XXL			
OKS-14, Size: XS, S, M, L, XL, XXL			
OKN-21, Size: XS, S, M, L, XL, XXL	circle of the middle of a patellar joint changes from 30 to 50 with walk 5, height 26	130-150 with walk 3,33 (depending on the Size)	Beige, black, white, dark blue, grey
OKN-22, Size: XS, S, M, L, XL, XXL			
OP-21	Height 6, length 33	35	Black, dark blue, beige, white, grey 1 velcro fastener
OP-22;	Height 4,5, length 36	40	
OK-21, Size: S, M, L, XL, XXL;	circle of the middle of a patellar joint changes from 30 to 55 with walk 5, height 31	330-450 with walk 20 (depending on Size)	Black, grey, white, beige, dark blue 2 velcro fastener
OK-22, Size: S, M, L, XL, XXL;			
OKP-21, Size: XS, S, M, L, XL, XXL;	circle of the middle of a knee joint changes from 30 to 55 with walk 5, height 32,5	330-450 with walk 20 (depending on Size)	Black, grey, white, beige, dark blue
OKP-22, Size: S, M, L, XL			
OR-31	Height 62, length 80	1500-2000	Black, grey, dark blue, white, beige
OR-32;			
OR-33			
OKR-31, Size: S, M, L, XL, XXL;	circle of the middle of a knee joint changes from 43 to 62 with walk 6, the length changes from 42 to 70 with walk 8	800-1100 with walk 60 (depending on Size)	Black, white, grey, beige, dark blue
OKR-32, Size: S, M, L, XL, XXL			
OPD;	Height 13, length 22	30-55	White, beige, grey, black
FA-11, Size: S, M, L, XL, XXL	Circle of an anticnemion above anklebone bones: 15-20, 20-25, 25-30, более30	25-35	White, black, beige, grey, dark blue
OA-11, Size: XS, S, M, L, XL, XXL;	Circle of an anticnemion above anklebone bones: to 15, 15-20, 20-25, 25-30, 30-35, More than 35 Height 23,5	30-55 with walk 4,16	Grey, beige, white, dark blue, black
OA-12, Size: XS, S, M, L, XL, XXL;			
OA-13, Size: XS, S, M, L, XL, XXL;			
OA-13, Size: S, M, L, XL, XXL			
OAR-11, Size: S, M, L, XL, XXL	Circle of an anticnemion above anklebone bones: 15-20, 20-25, 25-30, 30-35, more than 35 Height 23,5	40-60 with walk 4	White, black, beige, grey, dark blue
OAR-12, Size: S, M, L, XL, XXL;			
OAR-13, Size: S,			

M, L, XL, XXL;					
OAS-11, Size: S, M, L, XL, XXL;	Circle of an anticnemion above anklebone bones: 15-20, 20-25, 25-29, 29-34, more than 34 Height 23,5			65-90 with walk 5	Grey, beige
OAS-12, Size: S, M, L, XL, XXL;					
OAS-13, Size: S, M, L, XL, XXL;					
OA-21, Size: S, M, L, XL	Circle of an anticnemion above bones lodyzhki:17-20, 20-23, 23-26, 26-28			200-250 with walk 12,5	Black, white, grey, beige, dark blue
OA-22, Size: S, M, L, XL				250-300 with walk 12,5	
OAP-311	Height 45, length 45			350	White, dark blue, black, beige, grey
OAS-312, Size: S, M, L, XL;	Size	Size of man's footwear	Size of female footwear	150-200 with walk 10	White, beige, black, grey, dark blue
	S	35-38	To 37		
	M	39-43	37-39		
	L	43-45	39-42		
	XL	45-47	43-45		
	Height 23,5				
OA-313, Size: S, M, L, XL	Height 30			3500-4000 with walk 125	Black, white, beige, dark blue, grey
	Size	Size of footwear (European)			
	S	To 37			
	M	37-41			
	L	41-44			
	XL	More than 44			
OA-314, Size: S, M, L	Height 35				
	Size	Size of footwear (European)			
	S	To 37			
	M	37-41			
	L	41-44			
	XL	More than 44			
OT-11 (Right/left), Size: S, M, L, XL, XXL;	Grasp of hips 80-130 with walk 10, a dyne 78			2000-2500 with walk 100	White, beige, black, dark blue
OT-21 (Right/left), Size: XS, S, M, L, XL;	Size (A grasp on a pelvic ring (on 5 sm below a belly-button), sm)			550-700 with walk 30	Black, beige, white

	XS 70-80 S 80-90 M 90-100 L 100-110 XL 110-120 Height 17		
0-21, Size: XS, S, M, L	Size Age of the child, month XS 3-6 S 6-12 M 12-24 L 20-30 Height 9	100-150 with walk 12,5	White, black, beige
OT-31, Size: S, M, L;	Size Distance between subnodal poles in position of abduction in hip joints, sm S 16-20 M 21-30 L 31-40 Length 36	450-500 with walk 10	White, black, grey, beige

The thumb lock of the OPD foot (serial number 24)

Description:

- the retainer consists of a plastic frame with polyurethane foam overlays and 2 velcro fasteners, allowing you to pull your thumb inside;
- Night bandage bandage on the thumb, made of durable plastic;
- the plastic frame keeps the foot and thumb in the correct position during the night rest;
- Soft inner lining reduces pain and makes treatment comfortable;
- the tension of the bandage is gradually strengthened until complete cure (correction of the deformation of the finger);

The weight of the products is 30-55 gr. depending on the size.

Purpose:

- conservative treatment of valgus deformity of the big toe;
- fixation in the correct position after surgical treatment.

Benefits:

- A durable plastic frame holds the thumb in the correct position during sleep.

Recommendations for use:

Only for night use. When walking, do not apply! Do not use with open and bleeding wounds.

Size: universal.

Color: white, beige, gray, black.

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene-butylene acetate copolymer, 15% -fused polyurethane, 5% -cotton, velcro tape.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

The ankle-fixing ankle in the form of an 8-shaped bandage FA-11 (serial number 025)

Indications:

- partial damage to the ligaments and articular capsule of the ankle;
- prophylaxis of ankle joint injuries while playing sports.

Result of application:

- limited fixation of the Achilles tendon, ankle and foot;
- Easy compression;

It is made of medical elastic in the form of an 8-shaped dressing. Compression is achieved due to the elastic band and the possibility of adjusting the pulling by means of a velcro fastener over the ankles.

The weight of the products is 25-35 g. depending on the size.

Color: white, black, beige, gray, blue. Dimensions: S, M, L, XL (measurement of shin circumference above ankle bones, cm).

Ingredients: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for ankle joint elastic OA-11 (serial number 026)

Orthosis on the ankle elastic, with Velcro OA-12 (serial number 027)

Orthosis on ankle joint of high elasticity OA-13 (serial number 028)

Indications:

- trauma, bruises, sprains of ankle joints;

- prophylaxis of injuries and exacerbations of inflammatory diseases of the ankle joint under physical exertion.

Result of application:

- microphone action;

- even light compression;

- provides the fixation of the joint;

- Disease of pain syndrome.

It is made of an elastic material, which creates compression above the 2nd class, ensuring tight fitting of the product and the necessary impact on the joint. The web is made of special viscous with sections of different density according to the anatomical structure of the joint. The OA-12 has 1 velcro strap, which provides a more durable fixation. The fabric of the product OA-13 has a multidirectional netting, which provides an excellent fit and the correct distribution of pressure. Breathable, moisture draining promotes intensive evaporation of moisture from the surface of the skin, preserves heat and improves blood circulation in the joint.

The weight of the products is 30-55 gr. depending on the size.

Color: gray, beige, white, blue, black. Dimensions: XS, S, M, L, XL, XXL (measurement of shin circumference above ankle bones, cm)

Ingredients: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for ankle joint elastic reinforced OAR-11 (serial number 029)

Orthosis on ankle joint tight-elastic with stiffeners OAR-12 (serial number 030)

Orthosis on the ankle joint elastic reinforced, with Velcro OAR-13 (serial number 031)

Indications:

- bruises and sprains of ankle joints;

- prophylaxis of ankle joint injuries and exacerbations of inflammatory diseases;

- prophylaxis of exacerbations of chronic course of inflammatory diseases of the ankle;

- prophylaxis of injuries in sports and physical activities.

Result of application:

- Moderate fixation of the ankle;

- uniform compression;

- The stiffeners reduce impact loads on the joint.

It is made of a dense elastic material, which creates compression above the 2nd class, ensuring tight fitting of the product and the necessary impact on the joint. The canvas has an air permeable moisture-removing effect, is made by special viscous with sections of different density according to the anatomical structure of the joint. Products OAR-13 has 1 velcro-sticky tape that provides a more durable fixation. The fabric of the OAR-12 product has a multidirectional netting, which ensures perfect fit and proper distribution of pressure. There are lateral stiffeners.

Spiral spring stiffeners reduce impact loads on the joint.

The mass of products is 40-60gr. depending on the size.

Color: white, black, beige, gray, blue. Size: S, M, L, XL, XXL (measurement of shin circumference above the ankle bones, cm).

Ingredients: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the ankle joint elastic with silicone inserts OAS-11 (serial number032)

Orthosis on the ankle joint elastic with silicone inserts and Velcro OAS-12 (serial number 033)

Orthosis on the ankle joint of high elasticity with silicone inserts OAS-13 (serial number 034)

Indications:

- extension of muscles and ligaments of the joint;

- rehabilitation after dislocations and subluxations of the ankle;

- prophylaxis of exacerbations and conservative treatment of arthrosis, arthritis, bursitis, synovitis;

- prophylaxis of injuries with ankle load;

- reduction of swelling after injuries and surgeries.

Result of application:

- Fixation of the ankle joint, silicone inserts reliably protect the ankles, reduce edema and stimulate proprioceptive sensitivity (deep sensitivity).

Made of dense elastic material. Silicone pads in the area of the bony protrusions fix the ankle joint. One Velcro strap on products OAS-12 provides a more dense and reliable fixation.

The fabric of the OAS-13 is made of high-elastic material, which has a multi-directional weave, which ensures perfect fit and proper distribution of pressure.

The weight of the products is 65-90 gr. depending on the size.

Color: gray, beige. Dimensions: S, M, L, XL, XXL (measurement of shin circumference above the ankle bones, cm).

Ingredients: 67% - nylon, 23% - elastic, 10% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on the ankle joint with lacing, rigid fixation OA-21 (serial number 035)

Orthosis on the ankle joint with lacing, soft fixation OA-22 (serial number 036)

Indications:

- damage to the ligaments of the ankle;
- Ankle injury;
- chronic ankle instability of pronounced degree;
- the stifling paralysis of the foot;
- rehabilitation after injuries and operations on the ankle;
- if excessive mobility (permanent dislocations) and with "hanging" the foot;
- prophylaxis of damage to the Achilles tendon, ankle and foot in sports, heavy loads.

Result of application:

- Fixation of the ankle and lower third of the shin;
- OA-21 provide a strong, tight fixation of the ankle;
- OA-22, due to softer and pleasant material provides a soft, at the same time, tight fixation of the joint.

Made of dense breathable material. Lateral rigid metal tires reliably fix the joint. Lacing facilitates individual adjustment of the size of the orthosis. Suitable for wearing in casual shoes. By appointment, the orthosis can be used instead of a cast bandage. Universal, it is possible to apply both on the right and on the left leg.

Color: black, white, gray, beige, blue. Dimensions: S, M, L, XL (measurement of shin circumference above ankle bones, cm).

The composition of the product OA-21: 60% - polyester, 20% nylon, 12% - steel, 5% cotton, 5% polyvinyl chloride, tape, velcro.

The composition of the product OA-22: 13% cotton, 50% polyester, 37% elastic.

Weight of OA-21 is 200-250 gr. depending on the size.

Weight OA-22 is 250-300g. Depending on the size.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for ankle joint (pneumatic tire) OAP-311 (serial number 037)

Indications:

- rehabilitation after dislocations, subluxations of the ankle joint;
- Bole syndrome and swelling after operations and injuries of bones and soft tissues of the ankle joint.

Result of application:

- rigid fixation of the ankle and lower third of the shin;
- immobilizes and cools the ankle joint during the period of inflammatory disease, with acute trauma or after surgical treatment.

It is made of two plastic tires of anatomical shape and two gel pillows for additional fixation and cooling of the soft tissues of the ankle joint. Plastic anatomical tires exactly repeat the contour of the leg and are attached to the shins and using two wide adjustable straps with a velcro closure. One of the two straps with Velcro fastening located on the sole allows you to adjust the orthosis according to the individual dimensions of the patient's leg. The gel pillows have the ability to keep up to two hours of low temperature (after cooling in the freezer for 15-20 minutes).

The weight of products is 350 gr.

Color: white, blue, black, beige, gray. Dimensions: universal.

The composition of the product: 45% - polypropylene, 30% - nylon, 5% - copolymer of ethylene and vinylacetate, 5% - foamed polyurethane, 5% - cotton, velcro tape.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for the ankle (stop holder) OAS-312 (serial number 038)

Indications:

- the hip stop syndrome after the suffered neuritis and damage of the peroneal nerve, in trauma, after strokes, spinal cord injuries, cerebral palsy, neuroinfections and spinal cord tumors;

Result of application:

- fixation of the foot, ankle and upper third of the leg in a functionally advantageous position;
- The art is designed for wearing in shoes.

Made of plastic. The plastic frame (sleeve) covers the shin in the gastrocnemius area and passes into a plastic half-cup. The plastic half-stem has the support of the inner longitudinal arch and high sides to stabilize the natural position of the foot. Two belts with velcro fastening: one with a soft gasket is fixed on the shank of the shank, the second strap can be attached both above the ankles, and on the rise of the foot. Designed for people weighing not more than 110 kg.

The weight of the products is 150-200. depending on the size.

Color: white, beige, black, gray, blue. Dimensions: S, M, L, XL right / left (matched to the size of the shoe).

The composition of the product: 45% polypropylene, 30% nylon, 5% ethylene / vinyl acetate copolymer, 5% polyurethane foam, 5% cotton, 2 adhesive tape.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis on ankle joint adjustable extended OA-313 serial number 039)

Orthosis on the ankle joint is not adjustable shortened OA-314 serial number 040)

Indications:

- Early functional therapy for injuries of ankle ligaments, ankle injuries, intraarticular fractures and reconstructive surgeries;
- damage to the Achilles tendon;
- paralysis and paresis of the foot;
- Postoperative and conservative therapy of crural and foot fractures.

Results of application:

- dosage load on the joint in the postoperative period;
- provides the unloading of the foot;
- Allows you to fix the position of the rear or plantar flexion.

Made of two metal tires with volume controllers (the ability to set the angle of flexion-extension or the necessary range of movements in the joint in 10-degree increments). Three cuffs on the shins and two cuffs on the foot with Velcro fasteners for adjusting the degree of abutment of the orthosis to the body. Equipped with a sole with a roll, which provides the patient almost normal gait. There is a detachable socks on the velcro fastener for use in cold weather. OA-313 has an elongated cuff that allows it to be used by people of high growth. OAH-OA-314, has a shortened cuff, which allows it to be used by people of low stature.

The weight of the products is 3500-4000 gr. depending on the size.

Color: black, white, beige, blue, gray. Size: S, M, L, XL (matched to the size of the shoes).

Composition of products: 60% -polyester, 20% nylon, 12% -stal, 3% -cotton, 5% -polyenyl chloride, ribbon-velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

name	Overall dimensions, sm		Weight, rp	Execution variants on colour scale		
OS-21	Size	Circle of a thorax with an adressed hand (sm)	450-500	Black, white, beige, dark blue, grey		
	S/Î	To 100				
	L/XL	More than 100				
OS-11	Size	Length forearms (sm)	30-55	Black, white, dark blue, beige, grey		
OS-12						
OS-13					S	30-33
					M	33-36
					L	36-39
	XL	More than 39				
R-2	Width 8, length of straps - 48-56 with walk 2		55-60 with walk 1,25	White, black, beige, dark blue, grey		
R-1	Width 8, length of straps - 48-					

2022

	56 with walk 2					
R-3	Height 15 – 21 with walk 2, width 23-29 with walk 2					
OSR-21	Szie	Thorax circle at level of an axillary recess (sm)	90-105 walk 15	with	Black, white, beige, dark blue, grey	
	S/M	To 90				
	L/XL	More than 90				
	Height 16,5					
OSR-22	Size	Thorax circle at level of an axillary recess (sm)				
	S	75-80				
	M	80-85				
	L	85-90				
	X	90-95				
	Height 16,5					
OEU-11	Height 7,5, length 60		35-40		Black, white, beige, dark blue, grey	
OEU-12	Height 3,5. Length 60					
OEE-11	Size	Circle of an elbow joint (cm)	40-55 walk 1	with	Grey, beige, white, black, dark blue	
OEE-12						
OEE-13						
XS						To 24
S						24-26
M						26-28
L	28-31					
XL	More than 31					
Height 23,5						
OEH-21	Size	Circle of an elbow joint (sm)	70-85 walk 3	with	Grey, beige, White, black, dark blue	
OEH-22						
OEH-23						
S						To 26
M						26-28
L						28-31
XL	31-35					
XXL	More than 35					
Height 22,5						
OER-31	Height 43, width 35		2000-2500		Black, white, beige, dark blue, grey	
OHE-11	Height 10		25-35 walk 2	with	Grey, beige, white, dark blue, black	
	Size	Circle wrist joint (cm)				
	S	To 17				
	M	17-19				
	L	19-22				

	XL	22-24			
	XXL	More than 24			
OHE-12	Size	Circle wrist joint (cm)			
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-24			
	Height 10				
OHE-13	Height 10				
	Size	Circle wrist joint (cm)			
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-24			
OHG-21	Size	Circle wrist joint (cm)	25-40 with шаром3	Grey, beige, white, black, dark blue	
OHG-22					
OHG-23	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-26			
	XXL	More than 26			
	Height 17				
OMR-21	Height 16, length 34,5				
OMR-22	Size	Circle wrist joint (cm)			Grey, beige, white, black, Dark blue
	S/M	17-21			
	L/XL	21-26			
	Height 8				
OMR-23	Height 16, length 34,5				
OC-311	Height 26		90-110	Grey, beige, white, black, dark blue	
	Size	Circle wrist joint (cm)			
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-26			
	XXL	More than 26			
OC-312	Height 19		100-125		
	Size	Circle wrist joint (cm)			


	S/M	17-21			
	L/XL	21-26			
OC-313	Height 26			200-250	
	Size	Circle wrist joint (cm)			
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-26			
	XXL	More than 26			
OC-321	Size	Circle wrist joint (cm)		100-150	Grey, beige, white, black, dark blue
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-26			
	Height 19				
OC-322	Height 18, width 12			160	
OC-323	Size	Circle wrist joint (cm)		160	
	S	To 17			
	M	17-19			
	L	19-22			
	XL	22-26			
	Height 19				
OHR-31	Size	Circle wrist joint (cm)		100-150	Grey, beige, White, black, dark blue
	S	To 17			
	M	17-19			
	L	19-22			
	XL	More than 22			
	Height 26				
OHR-32	Height 21, length 27				
OHR-33	Size	Circle wrist joint (cm)			
	S	To 17			
	M	17-19			
	L	19-22			
	XL	More than 22			
	Height 26				

Orthosis for the hip joint OT-11

(right / left) (serial number 041)

Indications:

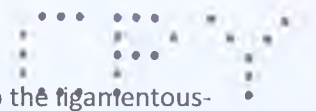- instability of the hip joint;

- The rehabilitation period after operations, changes in hip dislocations, damage to the ligamentous-muscular apparatus;

- Prevention of injuries of the hip joints, neck of the hip; prevention and treatment of loosening of the hip endoprosthesis.

Result of application:

- Hard fixation of the hip joint with the ability to adjust the angle of flexion-extension and reduction-retraction.

- prevention of development of contractures and muscle hypotrophy.

Made of plastic panel, equipped with soft textile inlays and two velcro strips, as well as one main strap. Thanks to the features, the orthosis can easily be mounted on the right and left hip joint. The modern high-tech hinge provides the possibility of independent adjustment of the flexion-extension volume in 10-degree increments and reduction-withdrawal in the range from minus 5 to 40 degrees.

The weight of the products is 2000-2500 gr. depending on the size.

Color: white, beige, black, blue. Dimensions: S, M.L, XL, XXL (the size is selected according to the hips).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride, tape, velcro.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for the hip joint of aeroprene with stiffeners OT-21

(right / left) (serial number 042)

Indications:

- period of rehabilitation after reconstructive operations (endoprosthetics, arthroplasty, etc.);

- isolated edge lines, detached, incomplete fractures of the large trochanter (by agreement of the attending physician);

- fresh fractures of the femoral neck, including in elderly patients;

- degenerative-dystrophic changes in the hip joint (aseptic necrosis of the femoral head, cystic perestroika, coxarthrosis, etc.);

- arthropathy;

- damage to the musculoskeletal system, directional dislocation, including in case of a sports injury;

- for prolonged physical and sports loads.

Result of application:

- orthosis ensures fixation, stabilization and partial unloading of the hip joint;
- improvement of coordination of work of muscles;
- orthosis can be used in combination with medical compression jersey;
- With exercises LFK bandage can not be removed.

Made of aeroprene - a soft, breathable, highly elastic neoprene with additional perforations all over the surface. In the pelvic component of the construction, depending on the size, 3-4 flexible metal, parallel stiffeners are used, as well as an elastic screed for adjusting the tension of the structure in the pelvic region. The girth and degree of fixation of the femoral component is also carried out by means of an elastic screed. On the coupler of the pelvic and femoral components there is a tape-tension indicator. Belt fastening is carried out with the help of two velcro fasteners.

The weight of products is 550-700gr. depending on the size.

Color: black, beige, white. Size: XS, S, M, L, XL (girth around the pelvic ring (measured 5 cm below the navel, cm)).

Ingredients: 30% neoprene, 27% cotton, 23% elastic, 20% lycra.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis for hip joint O-21 (serial number043)

Indications:

- Prostrate hip dislocation;
- dysplasia of the hip joint;
- subluxation of the hip.

Result of application:

- fixation of the hip joint in a functionally advantageous position without any strain on the shoulder girdle and spine of the child;
- promotes the correct formation and development of the acetabulum in children with hip dysplasia;
- Provides fixation of the hips in the position of bending and retraction at an angle of 90 degrees or more.

It is made of hypoallergenic, non-irritating skin polymeric materials.

The weight of the products is 100-150 gr. depending on the size.

Color: white, black, beige. Size: XS, S, M, L (the age of the child).

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene-butylene acetate copolymer, 15% fused polyurethane, 5% -cotton, velcro tape.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis to the hip joint reinforced with OT-31 (serial number 044)

Indications:

- dysplasia of the hip joints;
- subluxation of the thigh;
- Prostate hip dislocation;
- rehabilitation in the postoperative period.

Result of application:

- Provides fixation of the hip joints without any load on the shoulder girdle and spine of the child.

Made of plastic with metal construction. Inside there are soft gaskets.

In the model, the position of the lead in the hip joint can be changed step by step in steps of 10 degrees. The adjustable design allows 6 months to increase the volume of the tire belt following the growth of the child without moving to the next size.

The weight of the products is depending on the size.

Color: white, black, gray, beige. Size: S, M, L (the distance between the popliteal pits in the position of the lead in the hip joints, cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride, 3 Velcro tapes.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. Ability to use orthoses longer than the prescription

Orthosis soft supporting for cervical vertebrae (adult / child) CCA-1, CCV-1 (serial number 081 and 082)

Indication: used when

- Muscular torticollis;
- trauma of cervical spine;

- postoperative condition on the cervical spine;

correction of displacement of cervical vertebrae.

Result of application:

- The bandage limits the rotational, flexural and extension movements of the neck;
- Thermal action helps improve blood circulation, accelerate recovery and shorten recovery time.

Made of high-quality foam rubber, anatomical shape, one velcro closure, top protected with a sheath of cotton fabric.

The weight of the products is 50-100 gr. depending on the size.

Color: beige, black, white, gray, blue, cream.

Dimensions are selected by measuring the circumference of the neck and the height (from the chin to the jugular fossa, cm):

adult (CCA-108 (8.00 cm), CCA-109 (9.00 cm), CCA-110 (10.00 cm), CCA-111 (11.00 cm)),; Child (CCV-103 (3, 00 cm), CCV-104 (4.00 cm), CCV-105 (5.00 cm), CCV-106 (6.00 cm), CCV-107 (7.00 cm)).

Composition of products: 13% cotton, 50% polyester, 37% elastic.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Orthosis rigid supporting for cervical vertebrae CC-3 (serial number 83)

Indications:

- trauma of cervical spine;
- at the stage of transport evacuation;
- increased mobility of cervical vertebrae;
- displacement of cervical vertebrae;
- osteoporosis;
- osteomyelitis;
- oncological diseases;
- osteochondrosis;
- Postoperative period in the cervical spine.

Result of application:

- fixes and stabilizes the cervical spine.
- Unloads the ligamentous apparatus and the muscles of the cervical region.
- Creates a rigid support to the head.

It is made of a material on a person's plaster positive, it is fixed by two belts with a velcro closure, an opening in the area of the thyroid cartilage allows you to monitor the condition of the tracheostomy.

The weight of the products is 100-120 g. depending on the size.

Color: beige. Dimensions: S, M, L, XL (matched in the same way as bandages are soft).

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene / venil acetate copolymer, 15% -fused polyurethane, 5% cotton.


Storage conditions: The orthosis should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: dispose of the orthosis with household rubbish.

Shelf-life: not established. The possibility of using an orthosis longer than prescribed by the course of treatment should be agreed with the doctor. Do not discontinue the use of orthosis without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

name	Overall dimensions, sm	
CP-23	Size	Waist circle
	S	To 85
	M	85-95
	L	95-105
	XL	105-120
	XXL	120-135
	Height 23	
CP-31	Size	Waist circle
	S	To 85
	M	85-95
	L	95-105
	XL	105-120
	XXL	120-135
	Height 31	
KLS-1	Size	The sum of circles of a waist and hips, деленная half-and-half, sm
	XS	To 70
	S	70-85

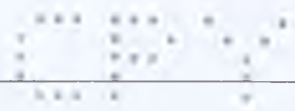	M	85-97	
	L	97-105	
	XL	105-115	
	XXL	More than 115	
	Height 30		
KL-2	Size	The sum of circles of a waist and hips, деленная half-and-half, sm	
	XS	To 70	
	S	70-85	
	M	85-97	
	L	97-105	
	XL	105-115	
	XXL	More than 115	
	Height 20		
KL-3	Size	The sum of circles of a waist and hips, деленная half-and-half, sm	
	XS	To 70	
	S	70-85	
	M	85-97	
	L	97-105	
	XL	105-115	
	XXL	More than 115	
	Height 20		
KTL-2	Size	Growth, The sum of circles of a waist and hips, деленная half-and-half, sm	
	S	150-160 60-75	
	S1	160-170 60-75	
	M	160-170 70-85	
	M1	170-180 70-85	
	L	170-180 80-95	
	L1	180-190	

		80-95	Height 42,5-57/47,5-58,5 with walk 5
	XL	180-190 90-110	
	XL1	190-200 90-110	
	XXL	200-210 110-130	
KTL-4	Size	Growth, The sum of circles of a waist and hips, деленная half- and-half, sm	
	S	100-120 45-52	
	M	120-140 52-58	
	L	140-160 58-65	
	Height 30-40		
KTL-5	Size	Growth Sm	The sum окружностей a waist and hips, деленная half-and-half, sm
	S	150-160	60-75
	S1	160-170	60-75
	M	160-170	70-85
	M1	170-180	70-85
	L	170-180	80-95
	L1	180-190	80-95
	XL	180-190	90-110
	XL1	190-200	90-110
	XXL	200-210	110-130
KS-2	T Size	The sum of circles of a waist and hips, деленная half-and-	

		half, sm	
	S	75-90	
	M	90-105	
	L	105-120	
	XL	120-135	
	Height 34-39		
KS-1			
	Size	The sum of circles of a waist and hips, деленная half-and-half, sm	
	S	75-90	
	M	90-105	
	L	105-120	
	XL	120-135	
	Height 46-54		
G-1	Size	Waist circle, sm	
	The universal	To 150 sm	
	Height 42		
G-2	Size	Circle of hips, sm	
	S	60-75	
	M	75-90	
	L	90-105	
	XL	105-115	
	XXL	115-125	
	Height 45		
KTL-3	Size	Waist circle	
	S	60-80	
	M	80-100	
	L	100-120	
	XL	120-140	
	Height 40		
PP-2	Size	Waist circle	
	S/M	70-95	
	L/XL	95-120	
	Height 33-39 with walk 6		
PP-3	Size	The sum Circles of a waist and hips, деленная half-and-half, ростом	
	XS	60-75 150-160	

	S	60-75 160-170
	M	70-85 160-170
	L	70-85 170-180
	XL	85-95 170-180
	XXL	90-110 170-190
	Height 42,5-57/47,5-58,5	
PPC-2	Size	The sum of circles of a waist and hips, деленная half-and-half, sm growth
	S	45-52 100-120
	M	52-58 120-140
	L	58-65 140-160
	Height 30-40 with walk 5	
KL-4	Size	The sum of circles of a waist and hips, деленная half-and-half, sm
	XS	To 70
	S	70-85
	M	85-97
	L	97-105
	XL	105-115
	XXL	More than 115
	Height 20	
KLU-1	Size	The sum of circles of a waist and hips, деленная Half-and-half, sm
	S/M	70-95
	L/XL	95-110
	Height 25	
KLS-2	Circle from 60-110 with walk 10, height 20	

KLS-3	Size	Waist circle	
	S	64-72	
	M	72-80	
	L	80-90	
	XL	90-100	
	XXL	100-110	
	Height 30		
KT-2	Size	The sum of circles of a waist and hips, деленная half-and-half, sm	
	XS	65-75	
	S	75-85	
	M	85-95	
	L	95-105	
	XL	105-115	
	XXL	115-125	
	XXXL	125-135	
OUC-11	Height 6, length 47		
OUC-12	Height 3, length 38		
OGH-1	Size	Waist circle	
	S	To 85	
	M	85-95	
	L	95-105	
	XL	105-115	
	XXL	More than 115	
CCA-1	Size are selected by measurement of a circle of a neck and heights (from a chin to яремной poles, sm): Adult (CCA-108 (8,00 sm), CCA-109 (9,00 sm), CCA-110 (10,00 sm), CCA-111 (11,00 sm); children's (CCV-103 (3,00 sm), CCV-104 (4,00 sm), CCV-105 (5,00см), CCV-106 (6,00 sm), CCV-107 (7,00см)).		
CCV-1			
CC-3	Size	Neck height	Neck circle
	S	5,7	25-33
	M		33-41
	L		41-46
	XL		46-48 and more
	S	8,3	25-33
	M		33-41
	L		41-46
	XL		46-48 and more
	S	10,8	25-33
M	33-41		
L	41-46		
XL	46-48 and more		
S		25-33	

	M L XL	13,3	33-41 41-46 46-48 sm and more	
--	--------------	------	--	--

Corset soft on the abdominal wall (narrow) 23 cm (CP-23) (serial number 084)

Corset soft on the abdominal wall (wide) 31 cm (CP-31) (serial number 085)

Indications:

- uncomplicated neuropathies of myopathy of the lower back;
- radiculitis and radiculosis;
- fixes the anterior and lateral abdominal walls;
- rehabilitation after surgery on the abdominal wall;
- The weakness of the abdominal wall.

Result of application:

- Moderate support, protection from cooling back muscles;
- Early mobilization after operations on the abdominal and abdominal organs;
- prophylaxis of the formation of hernia of the abdominal wall.

Manufactured by:

Equipped with a double valve for better fit of the belt to the body.

The mass of products CP-23 is 140-220g. depending on the size.

The mass of products CP-31 is 200-280g. depending on the size.

Color: white, beige, gray, black. Dimensions: S, M, L, XL, XXL (selected by measuring the circumference of the waist, cm).

Composition of products: 13% cotton, 50% polyester, 37% elastic.

Storage conditions: orthoses should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: orthoses should be disposed of with household garbage.

Shelf-life: not established. The possibility of using orthoses longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using orthoses without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Corset semi-rigid lumbosacral KLS-1 (serial number 086)

Indications:

-radiculitis, osteochondrosis, spondylarthrosis, spondylitis, spondylolisthesis, protrusion of intervertebral discs;

- rehabilitation after injuries and surgeries;

-functional therapy for pain in the lower back and lower part.

Result of application:

Hyperextension, reclination and discharge of the lumbosacral spine with compensation of the function of the back muscles;

-dosed fixation and unloading of the spine;

- with physical exertion has a stabilizing effect on the lumbosacral spine, stimulating the effect on the muscles of the back.

Made of durable high-quality material with breathable properties. The corset has two ties, equipped with a velcro fastener, to strengthen the fixation. The rear wall includes 4 ribs and grips the sacral section.

The weight of the products is 345-475 gr. depending on the size.

Color: beige, black. Dimensions: XS, S, M, L, XL, XXL (the sum of waist circumferences and hips, divided in half, cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset semi-rigid lumbar KL-2 (serial number 087)

Indications:

-Bole syndrome on a background of muscle tension and muscular imbalance of vertebral displacements (spondylolisthesis I degree);

-functional therapy for lumbargia and ishalgia;

-prophylaxis of injuries and diseases when working with weights.

Result of application:

-hyperextension, reclining and unloading of the lumbar spine with compensation of the function of the back muscles;

-dosed fixation and unloading of the spine;

- with physical exertion has a stabilizing effect on the lumbar spine, stimulating effect on the muscles of the back.

Made of elastic mesh material. The corset has two ties, equipped with a velcro fastener, to strengthen the fixation. The location of the 4 stiffeners relieves muscle tension and, as a result, reduces pain. The width of the product is 20 cm.

The weight of the products is 210-265 gr. depending on the size.

Color: beige, black, white, gray, blue.

Dimensions: XS, S, M, L, XL, XXL (the sum of waist circumferences and hips, divided in half, cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride, tape, velcro.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset semi-rigid lumbar reinforced KL-3 (serial number 088)

Indications:

- radiculitis;
- root syndrome;
- Localized osteochondrosis;
- spondyloarthrosis and spondylitis;
- intervertebral hernia;
- displacement of vertebrae;
- functional therapy for back pain.

Result of application:

- hyperextension, reclining and unloading of the lumbar spine with compensation of the function of the back muscles;
- dosed fixation and unloading of the spine;
- with physical exertion has a stabilizing effect on the lumbar spine, stimulating effect on the muscles of the back.
- Stabilizes the joint-ligament apparatus of the lumbar region.

Made of elastic mesh material. The corset has two ties, equipped with a velcro fastener, to strengthen the fixation. The location of the 6 stiffeners relieves muscle tension and, as a result, reduces pain. The width of the product is 25 cm. It is possible to wear a corset in a standard way or with a reclining effect.

The weight of the products is 330-440 gr. depending on the size.

Color: beige, black, white, gray, blue.

Dimensions: XS, S, M, L, XL, XXL (the sum of waist circumferences and hips, divided in half, cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Cortex chest-lumbar (adult) KTL-2 (serial number 089)

Corset chest-lumbar (child) KTL-4 (serial number 090)

Indications:

- formation of correct posture in adults and children;
 - bending of the thoracic spine;
 - osteochondrosis of the thoracic spine, osteoporosis;
 - protection of the stoop with a load on the spine (with sedentary work);
 - rehabilitation after injuries and operations of the thoracic spine;
- gizzards of intervertebral discs of the thoracic region.

Result of application:

Improves posture without restricting freedom of movement;

-reklation and unloading of the upper- and mid-thoracic sections with strengthening of the muscular corset of muscle tone.

Backrest made of non-elastic material, which does not interfere with muscle activity. The corset is fixed on the abdomen with a single velcro closure. There are 2 simulated stiffeners, located paravertebrally. Twisting of the shoulders is supported by two inextensible straps with velcro fasteners.

The weight of KTL-2 products is 300-440 gr. depending on the size.

The weight of KTL-4 products is 200-250 gr. depending on the size.

Color: beige, black, white, gray, blue.

Dimensions KTL-2: S, S1, M, M1, L, L1, XL, XL1, XXL.

Dimensions KTL-4: S, M, L.

The composition of the products: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: corsets should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Conditions for disposal: corsets should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using corsets longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using corsets without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Cortex chest-lumbar reinforced KTL-5 (serial number 091)

Indications:

- violations of posture (stoop), accompanied by a pain syndrome in the lumbar region;
- curvature of the thoracic spine (kyphosis, scoliosis I-II degree);
- prevention of fractures on the background of osteoporosis;
- rehabilitation after operations and spinal injuries;
- Multiple herniated intervertebral discs of the thoracic region.

Result of application:

- improves posture, without limiting freedom of movement;
- Reclamation and discharge of the upper and middle thoracic parts with strengthening of the muscular corset, muscle tone;
- removal of muscle tension and, accordingly, elimination of pain syndrome.

It is made: a back from an inelastic material that does not interfere with muscular activity. The corset is fixed on the abdomen with one velcro closure. There are 2 simulated stiffeners, located paravertebrally. The reversal of the shoulders is supported by two non-stretchable straps with velcro fasteners. The design provides tension tapes for relieving muscle tension and, accordingly, eliminating pain syndrome.

The weight of the products is 390-540 gr. depending on the size.

Color: beige, black, white, gray, blue.

Dimensions KTL-5: S, S1, M, M1, L, L1, XL, XL1, XXL.

Methods of donning:

The straps must be passed under the armpits and crossed on the back. Then, to straighten the patient's shoulders and fix the position of the shoulder belt, having secured the straps with velcro fasteners on the belt of the corset.

Recommended wearing regimen for posture correction:

- Starting from 1-1,5 hours on the first day, gradually within two weeks, increase the time to 6 hours per day;
- permanent wearing for 6 hours every day for 2 months;
- gradually within 2 weeks reduce to 1-1.5 hours on the last day of use;
- After 3 months, it is NECESSARY to consult a doctor.

Individual treatment and terms of wearing are determined by the attending physician!

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset lumbosacral hard KS-2 (serial number 092)

Indications:

Immobilization after surgery;

- grains of intervertebral discs of the lumbar and lumbosacral spine;
- vertebrae fractures in the lumbar region;
- spondylolisthesis with the threat of trauma to the roots of the spinal cord;
- instability in the lumbar spine for various reasons.

Result of application:

Hyperextension, reclination and discharge of the lumbosacral spine with compensation of the function of the back muscles.

Made of lightweight and durable materials. Inside the rigid corset is a soft lining. Six fixing belts are equipped with velcro fasteners.

The mass of products is 400-550 gr. depending on the size.

Color: gray, black, beige, white, blue. Dimensions: S, M, L, XL (the sum of waist circumferences and hips, divided in half, cm).

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene / venil acetate copolymer, 15% -fused polyurethane, 5% cotton.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset chest-lumbar hard KS-1 (serial number 093)

Indications:

Immobilization after surgery;

-grains of intervertebral discs of thoracic and lumbar spine;

vertebrae fractures in the thoraco-lumbar region;

-spondilolistesi with the threat of trauma to the roots of the spinal cord;

-instability in the chest-lumbar spine for various reasons.

Result of application:

Hyperextension, reclining and unloading of the thoraco-lumbar spine with compensation of the function of the back muscles.

Made of lightweight and durable materials. Inside the rigid corset is a soft lining. Eight fixing belts are equipped with velcro fasteners.

The weight of the products is 500-650 gr. depending on the size.

Color: gray, black, beige, white, blue. Dimensions: S, M, L, XL (the sum of waist circumferences and hips, divided in half, cm).

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene / venil acetate copolymer, 15% -fused polyurethane, 5% cotton.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset hard (hyperextensor) G-1 (serial number 094)

Indications:

- stable compression fractures of the vertebral bodies of the inferior thoracic and lumbar regions (Th10-L2) without neurologic symptoms.

Result of application:

- supports the hyperextension position in the thoraco-lumbar spine;
- helps to normalize the muscle tone in the area of damage;
- reduces thoracic kyphosis and increases lumbar lordosis, effectively unloading the anterior parts of the vertebral bodies;
- relieves or eliminates pain syndrome;
- a functional-advantageous position accelerates the restoration of the bone tissue of the vertebra in the fracture zone.

Result of application:

- helps maintain the necessary position of the vertebra in the sagittal plane due to the work of the patient's own muscles;

Feature: three-point support system, pelotas are attached in the region of the waist, sternum and symphysis; Modular design allows for precise adjustment; has a small weight.

The weight of products is 1500-2000 gr. depending on the size.

Color: gray, black, beige, white, blue. Dimensions: universal (allows to use orthosis in patients of different height and build, with waist circumference up to 150 cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset rigid (hyperextensor frame) G-2 (serial number 095)

Indications:

- stable compression fractures of the vertebral bodies of the inferior thoracic and lumbar regions (Th10-L2) without neurologic symptoms.

Result of application:

- supports the hyperextension position in the thoraco-lumbar spine;
- helps to normalize the muscle tone in the area of damage;
- reduces thoracic kyphosis and increases lumbar lordosis, effectively unloading the anterior parts of the vertebral bodies;
- relieves or eliminates pain syndrome;
- limits lateral flexion and rotational movements of the trunk;
- a functional-advantageous position accelerates the restoration of the bone tissue of the vertebra in the fracture zone.

Feature:

- frame design provides reliable stabilization, both in the sagittal and in the frontal plane;
- 3-point support system helps to restore the normal bending of the spine due to the work of your own muscles;

The modular design of the corset allows for precise individual fit;

- The sliding pelvic girdle can be fixed to the required level (it is convenient for full patients);
- the location of the sternal pelota, the height and width of the frame are easily adjustable;
- in contact with the surface of the chest, the angle of installation of the sternal pelota is automatically regulated by the movable block;
- Replaceable soft foam pads are fastened with Velcro fasteners;
- Corset has a small weight, easy to clean.

The weight of the products is 2300-2800 gr. depending on the size.

Color: gray, black, beige, white, blue. Dimensions: S, M, L, XL, XXL (hip circumference, see).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset chest-lumbar, semi-rigid KTL-3

(serial number 096)

Indications:

- osteochondrosis, spondylosis, spondylolisthesis;
- radiculitis;
- hernia of intervertebral discs of the lower thoracic, lumbar and sacral spine;
- the rehabilitation period after trauma and operations of the lower thoracic and lumbosacral spine;
- uncomplicated fractures of the processes of the bodies of the lumbar vertebrae.
- instability in the lumbar spine for various reasons.

Result of application:

Hyperextension, reclining and unloading of the thoraco-lumbosacral spine with compensation of the function of the back muscles.

Made of lightweight and durable elastic materials. The corset has two additional ties, equipped with a velcro closure, to strengthen the fixation. The location of the 4 stiffeners relieves muscle tension and, as a result, reduces pain. The width of the product is 35 cm. It is possible to wear a corset in a standard way or with a reclining effect.

The weight of the products is 210-265 gr. depending on the size.

Color: white, black, gray, beige, blue. Dimensions: S, M, L, XL (selected by measuring the circumference of the waist, cm).

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset Corrector Posture PP-2 (serial number 097)

Indications:

- violations of posture (stoop), accompanied by pain in the thoracic spine;
- curvature of the thoracic spine (kyphosis, scoliosis I-II degree);
- prevention of fractures on the background of osteoporosis;
- rehabilitation after operations and injuries of the thoracic spine;
- Multiple herniated intervertebral discs of the thoracic region.

Result of application:

- two shoulder straps fixed on the back of the corrector in the form of a triangle

Improves posture without restricting freedom of movement;

- reclining and unloading the upper and middle chest areas with strengthening of the muscular corset, muscle tone;
- removal of muscle tension and, accordingly, elimination of pain syndrome

A back is made of an elastic breathable material, which does not interfere with muscle activity. The corset is fixed on the stomach with one velcro strap. There are 2 stiffeners located on each side of the spine. The back of the shoulders is supported by two stretchable velcro straps.

The weight of the products is 300-350 gr. depending on the size.

Color: beige, black, gray, white, blue.

Size: S / M, L / XL (selected by measuring waist circumference and height, see)

Ingredients: 45% polypropylene, 30% nylon, 5% ethylene / vinyl acetate copolymer, 15% -fused polyurethane, 5% cotton.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset Corrector Posture PP-3, PPC-2 (serial number 098, 099)

Indications:

- formation of correct posture in adults and children;
- bending of the thoracic spine;
- osteochondrosis of the thoracic spine, osteoporosis;
- protection of the stoop with a load on the spine (with sedentary work);
- rehabilitation after injuries and operations of the thoracic spine;
- gizzards of intervertebral discs of the thoracic region.

Result of application:

- improves posture, without limiting freedom of movement;
- reclamation and discharge of the upper and middle thoracic parts with strengthening of the muscular corset of muscle tone;
- reliable fixation, which allows to form the correct beautiful posture
- possibility to use as an independent method of preventing postural disorders;
- Convenience in use (proofreader is invisible under clothes).

The back is made of cotton material, which does not interfere with muscle activity. The corset is fixed on the abdomen with a single velcro closure. There are 2 simulated stiffeners located parallel to the spine. Twisting of the shoulders is supported by two non-stretch straps with velcro fasteners.

The weight of PP-3 products is 390-540 gr. depending on the size.

The weight of PPC-2 products is 200-250 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions PP-3: S, M, L, XL, XXL.

Dimensions PPC-2: S, M, L (selected by measuring waist circumference and height, see)

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Lumbar corset KL-4 (serial number 100)

Indications:

- osteochondrosis, spondylosis, spondylolysis in the lumbosacral spine, including with pain syndrome;
- radiculitis, myositis, bruises of the lumbosacral spine;
- The recovery period after injuries and operations in the lumbosacral spine;
- prevention of damage to the lumbosacral spine during intensive physical exertion

Result of application:

- semi-rigid fixation of the lower thoracic and lumbosacral spine;
- unloading the back muscles;
- analgesic effect.

Made of elastic "breathable" material that does not cause skin irritation; elastic double screed with Velcro fastener provides additional fixation; The required rigidity is provided by 4 metal fins.

The weight of the products is 330-440 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions KL-4: S, M, L, XL, XXL. (selected by measuring the circumference of the waist, see)

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Lumbar corset KLU-1 (serial number 101)

Indications:

- ablation of the abdominal cavity;
- myositis, lumbalgia;
- pain in degenerative diseases of the lumbosacral spine (osteochondrosis, spondylosis, etc.)
- rehabilitation after injuries and operations in the lumbar spine;
- painful syndrome in osteoporosis;
- herniated intervertebral discs of the lumbar spine.
- preventive use in case of exercise and sports

Result of application:

- provides a semi-rigid fixation of the lumbosacral spine;
- analgesic effect;
- unloads muscles and restores the physiological position of the spine;
- facilitates and eliminates pain syndrome;
- supports the anterior abdominal wall.

Made of elastic air-and moisture-permeable material, which does not cause skin irritation; has two types of size, which is regulated by an additional one removable cuff on 1 velcro fastener; is equipped with four metal plates that take the shape of the body.

The weight of the products is 220-270 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions KL-3: S / M, L / XL (selected by measuring the circumference of the waist, see)

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset lumbosacral KLS-2 (serial number 102)

Indications:

- rehabilitation after operations on the sacral and lumbar spine;
- acute and chronic pain due to spinal injuries;

- uncomplicated fractures of the processes of the bodies of the lumbar vertebrae;
- lumbosacral radiculitis, lumbago, lumbargia;
- back pain caused by excess weight and muscle failure;
- hernia of intervertebral discs;
- inflammation of the sacroiliac joint;
- degenerative displacement of vertebrae;
- bruises of the lumbosacral spine;
- after manual therapy and medical gymnastics;

Results of application:

- strong fixation of the lumbosacral spine;
- 4 simulated, removable stiffeners allow to achieve anatomically correct corset position and provide the maximum support;
- unloads the spiny-sacral department of the spine;
- improves sensorimotor control, provides stabilization due to activation of own muscles;
- helps to reduce pain.

Manufactured: breathable moisture separating material with a high content of cotton. 3 convenient velcro fasteners (one main and two additional) for stronger fixation, four metal tires.

The weight of the products is 220-270 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions: S, M, L, XL, XXL (selected by measuring the circumference of the waist, see)

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Corset lumbosacral KLS-3 (serial number 103)

Indications:

- Rehabilitation after injuries and rehabilitation in the lumbosacral region of the spine;
- painful syndrome in osteoporosis;

- Pain syndrome in osteochondrosis and other degenerative diseases of the lumbosacral spine;
- back pain caused by excess weight and muscle failure;
- hernia of intervertebral discs;
- inflammation of the sacroiliac joint;
- degenerative displacement of vertebrae;
- bruises of the lumbosacral spine.

Results of application:

- creates a functionally advantageous position;
- unloads the muscles of the back;
- reduces the phenomenon of inflammation and stimulates the recovery processes, thereby facilitating and eliminating pain syndrome;
- slows the progression of the disease.

Made of: a durable elastic air and moisture-permeable hypoallergenic material. A straight, high model with four functionally curved plates is fixed in front with a robust single velcro fastener; On the back wall there are two crossing reinforcement bands, which allow to regulate the degree of fixation; is equipped with four metal plates.

The weight of the products is 220-270 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions: S, M, L, XL, XXL (selected by measuring the circumference of the waist, see)

Ingredients: 60% polyester, 20% nylon, 12% steel, 3% cotton, 5% polyvinyl chloride.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Breast corset (thoracic) CT-2 (serial number 104)

Indications:

- Rehabilitation period after operations on the thorax;
- trauma of the chest.

Results of application:

- complete fixation of the thorax;

- creates optimal conditions for the healing of postoperative scars;
- reduction of pain in the wound area.

Made: the corset is made of elastic material, which helps to tighten the chest and prompt healing of the wound. One Velcro fastener is located either on the front surface of the chest or on the side. The corset has a special anatomical shape, which allows to optimally arrange the bandage on the chest. On the back of the corset has two plastic stiffeners, which are located symmetrically on each side of the spine. Two straps, thrown over the shoulder straps, contribute to better fixation of the bandage.

The weight of products is 550-700 gr. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions: XS, S, M, L, XL, XXL, XXXL (volume of the chest, see)

Ingredients: 30% neoprene, 27% cotton, 23% elastic, 20% lycra.

Storage conditions: The corset should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: corset should be disposed of with domestic garbage.

Shelf-life: not established. The possibility of using a corset for longer than prescribed by the course of treatment should be agreed with the doctor. It is not allowed to stop using the corset without consulting with the attending physician.

Warranty period: 6 months from the date of sale of the corset while observing the instruction and operating rules.

Orthosis umbilical (child) OUC-11 and OUC-12 (serial number 105 and 106)

Indications:

- Conservative treatment of umbilical hernias in children under 6 years of age;
- prophylaxis of prolapse and infringement of a hernial sac

It is made of: a durable and elastic combined material and very pleasant to the touch, made of environmentally friendly and hypoallergenic materials. It is fixed on the body with a single reliable velcro fastener. It is recommended to put on lying down, after the correction of the hernial protrusion.

The weight of OUC-11 is 35 g. depending on the size.

The weight of OUC-12 is 40 g. depending on the size.

Color: beige, black, gray, white, blue.

The sizes are universal.

Ingredients: 13% cotton, 50% polyester, 37% elastic.

Storage conditions: The orthosis should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: dispose of the orthosis with household rubbish.

Shelf-life: not established. The possibility of using an orthosis longer than prescribed by the course of treatment should be agreed with the doctor. Do not discontinue the use of orthosis without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis, subject to the instruction and operating rules

Orthosis of inguinal OGH-1 (serial number 107)

Indications:

-Postoperative period

- Inward hernia of the inguinal and femoral regions

- Hernias formed after operations on the groin area

- prevention of infringements and relapses of inguinal hernias.

Results of application:

Due to the design of the inguinal bandage and its peloids, mechanical support is provided for the formation of inguinal hernias and prevention of the formation of strangulated hernias, as well as an increase in the hernial sac.

Made of: from perforated material, does not cause irritation. Equipped with four removable pelots, which can either increase or decrease the pressure on the area of the hernial protrusion.

Two-sided groin bandage is put on lying, on underwear, with full correction of the hernia. Then in the "pockets" is installed a removable pelot, and the belt is fixed with a single velcro fastener.

Product weight 100-150g. depending on the size.

Color: beige, black, gray, white, blue.

Dimensions: S, M, L, XL, XXL (the volume of the circumference between the waist and hips, see)

Ingredients: 13% cotton, 50% polyester, 37% elastic.

Storage conditions: The orthosis should be stored at room temperature, in dry rooms, away from heat sources, direct sunlight and dust.

Disposal conditions: dispose of the orthosis with household rubbish.

Shelf-life: not established. The possibility of using an orthosis longer than prescribed by the course of treatment should be agreed with the doctor. Do not discontinue the use of orthosis without agreement with the attending physician.

Warranty period: 6 months from the date of sale of the orthosis while observing the instruction and operating rules.

Cleaning and disinfection

The surface of products with rigid fixing should be wiped with a damp sponge, detergent solutions in accordance with GOST 25644 and dried at a temperature not exceeding 40 ° C.

Do NOT dry near heating appliances and avoid direct sunlight. products can lose shape.

Coverings of foamed polymeric material should be wiped with a soft cloth moistened with household detergent, dried at a temperature of 18-20 °C away from the heaters.

Instructions for washing:

- When the first wash, wash the products separately from other laundry.
 - The products should be hand washed (40 ° C), especially if you apply ointments to treat the feet.
 - After washing, rinse the product thoroughly and repeatedly.
 - Use detergent without bleach (shampoo, baby soap).
 - When washing, do not use products that soften the laundry.
 - After washing, dry your products in air or in a washing machine in a gentle mode.
- Do not dry damp products on a towel or in direct sunlight.
- Do not clean the product.

1. Sanitary treatment of bandages on the cervical spine:

Wash the bandage as it gets dirty, at least once a month, the amount of washes is up to 150 times, an additional cover for the bandage is erased daily.

2. Sanitary treatment of compression knitwear:

Daily washing only by hand. Withstands up to 200 washes.

3. Sanitary treatment of fixatives on the upper and lower limbs:

Washing as it is contaminated, 3 times a month, can withstand up to 250 washes.

4. Sanitary treatment of fixatives on the upper and lower limbs with silicone inserts:

Washing as it gets dirty, 3 times a month, can withstand up to 200 washes.

5. Sanitary treatment of elastic retainers on the upper and lower limbs:

Washing as much as possible, at least 3 times a month, can withstand up to 150 washes.

6. Sanitary treatment of elastic belts

Laundry is not less than once a week, the maximum amount of washing is 100.

7. Sanitary treatment of corsets on the thoracic, lumbar and sacral spine:

Laundry as contamination, at least once a month, the maximum amount of washing 180 times.

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此公文書
This public document

2. 簽署人
has been signed by 江佩瑜
Chiang, Pei-yu

3. 簽署人職務
acting in the capacity of 民間公證人
Notary public

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9. 章戳
Seal/stamp: 10. 簽署
Signature:

Fan, Guang-fu

Director, Southwestern Taiwan Branch Office, BOCA

For The Minister of Foreign Affairs

11. 附註：
remarks:

This document authentication only certifies the authenticity of the signature, seal or stamp and the capacity of the person who has signed the attached document. It does not validate the contents of the document for which it was issued.

本證明書之真實性，請向下列網址查詢：
To verify the issuance of this authentication,
please peruse the following website：
<https://docauth.boca.gov.tw/BOCAWeb/index4.jsp>

G030162

Transportation of packaged products must be carried out in closed railway wagons, containers, cars, in ship holds, heated and sealed aircraft compartments in accordance with the requirements of the rules applicable to this mode of transport, under the following environmental conditions:

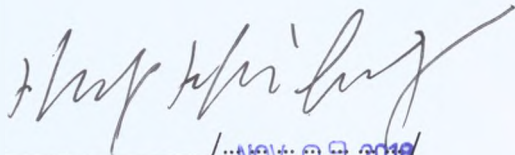
- ambient temperature from -10 to + 40 ° C;
- Relative humidity - no more than 70%;
- conditions of product transportation - according to group 5 according to GOST 15150.


Products must be stored in clean, dry, closed rooms under the following conditions:

- ambient temperature from -10 to + 40 ° C;
- Relative humidity - no more than 70%;
- Protect from indirect and direct exposure to sunlight and artificial light.

Authorized representative of the manufacturer in the Russian Federation (name, address, telephone, mail)

Limited Liability Company "Athlett" (LLC "Athlett")
Address: 420044, Russia, Republic of Tatarstan, Kazan, ul. Korolenko, 73, Apt. 52
Phone: + 7-843-515-12-95
Mail: atlett@bk.ru

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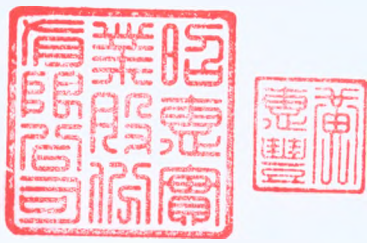For The Contents Of This Document
The Notary Public Assumes No Responsibility.

The Chiang, Pei-Yu Notary Public Office,
Taiwan Chiayi District Court, R.O.C.
Address : No.209, Linsen E. Rd., East Dist.,
Chiayi City, Taiwan (R.O.C.)
Tel:886-5-2751123 Fax:886-5-2753123

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