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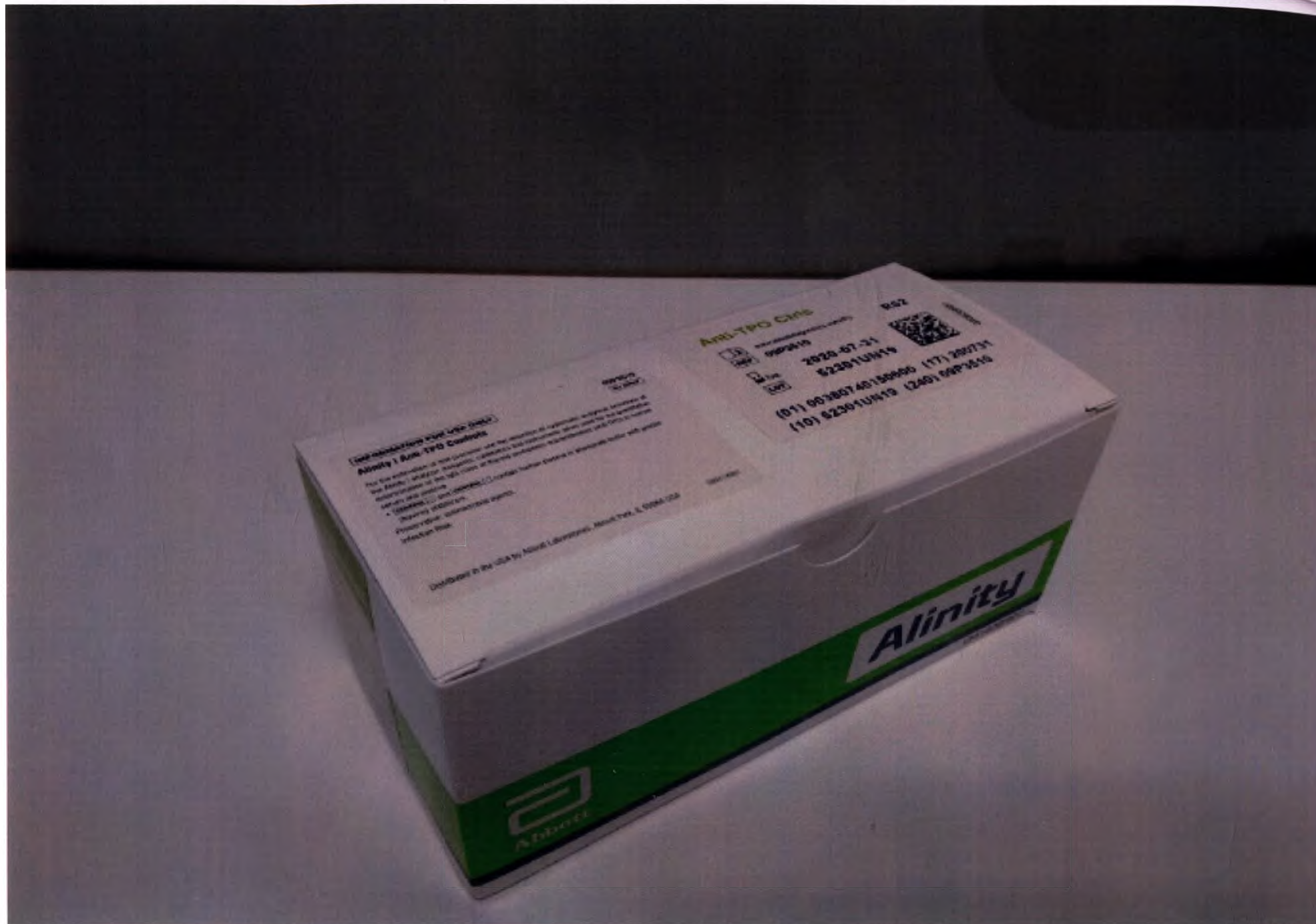
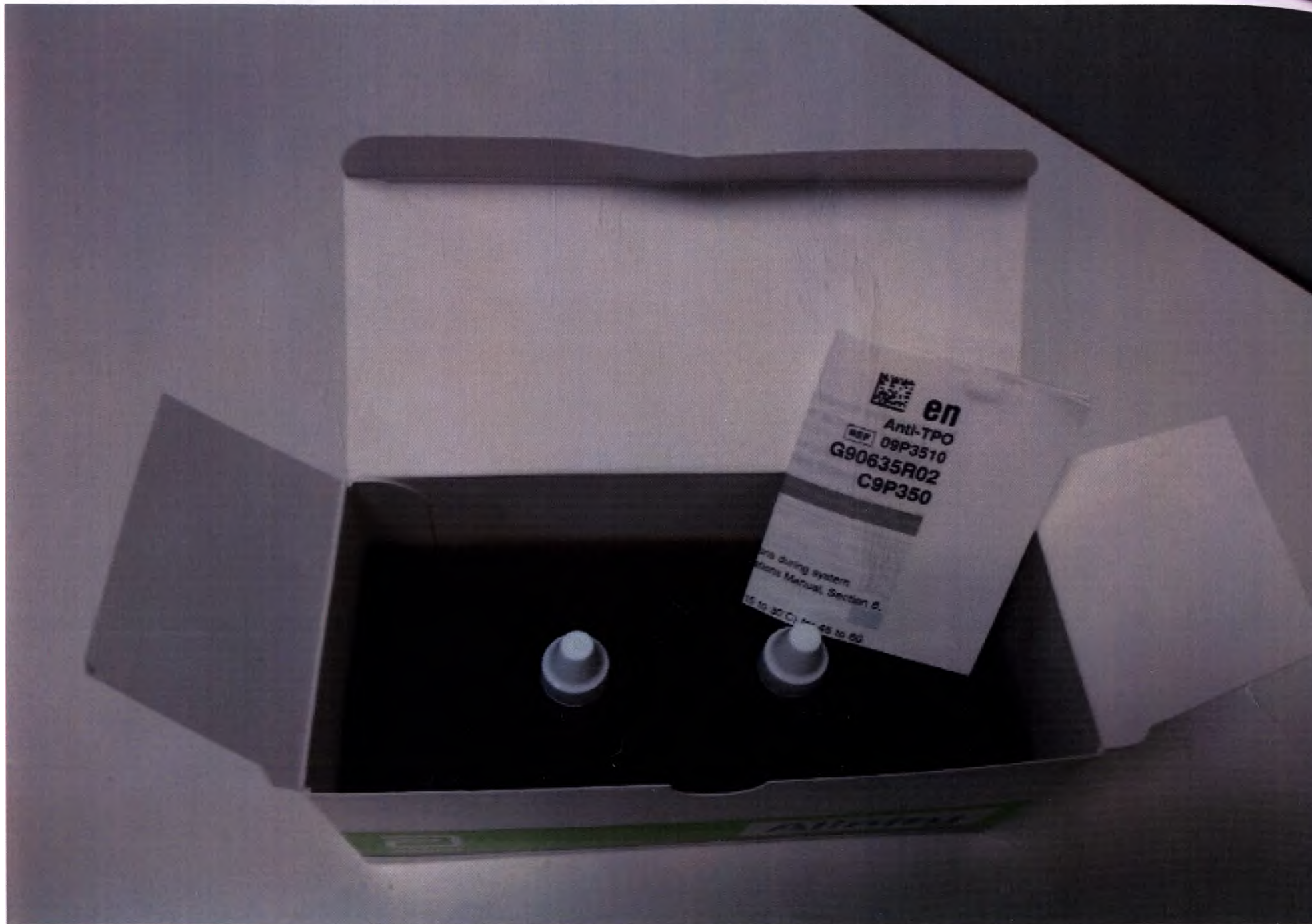
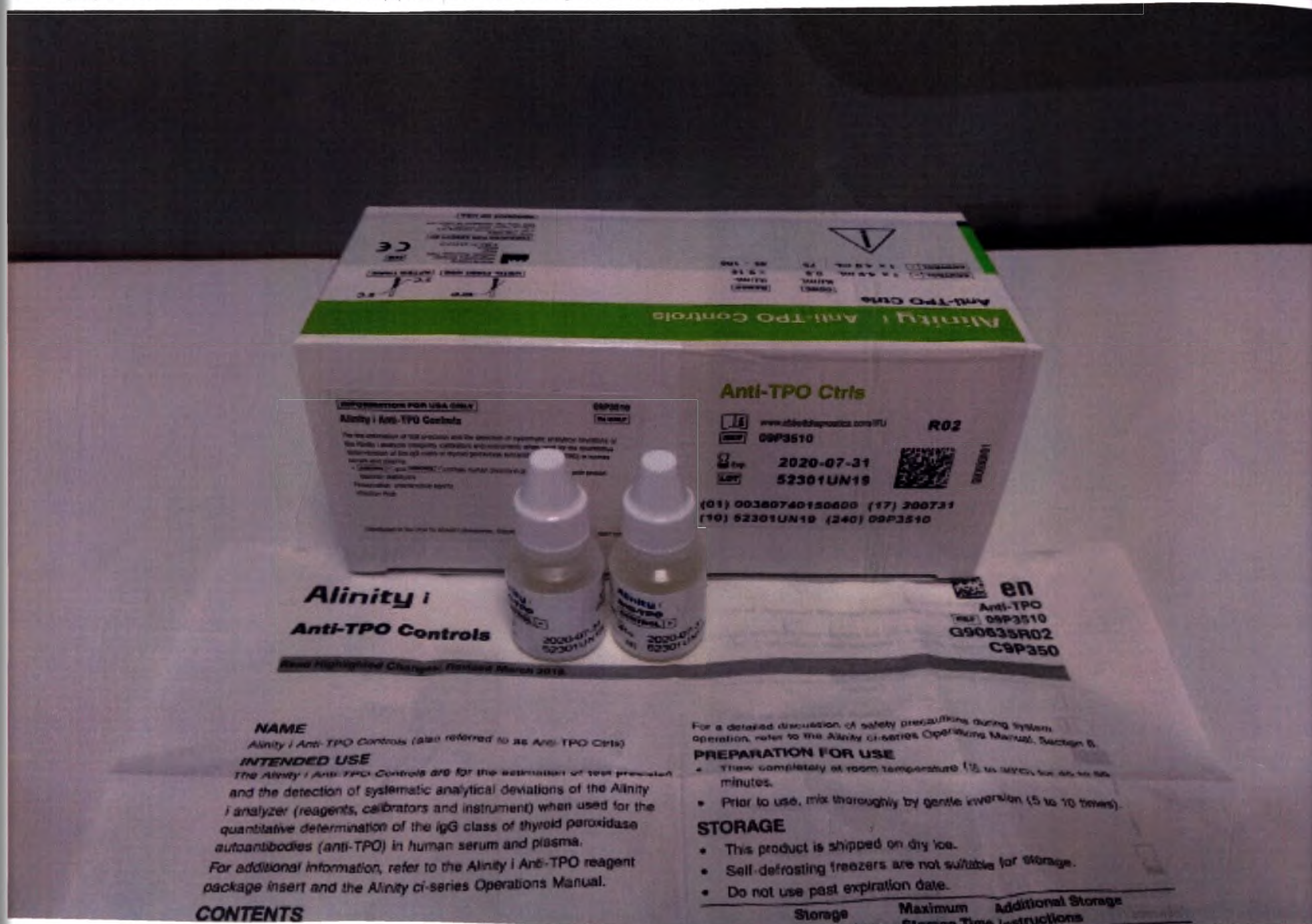
NAME
Alinity i Anti-TPO Controls (also referred to as Anti-TPO Ctrl's)

INTENDED USE
The Alinity i Anti-TPO Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer (reagents, calibrators and instrument) when used for the quantitative determination of the IgG class of thyroid peroxidase autoantibodies (anti-TPO) in human serum and plasma.

For additional information, refer to the Alinity i Anti-TPO reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 6.

PREPARATION FOR USE

- Thaw completely at room temperature (15 to 25°C) for 30 to 60 minutes.
- Prior to use, mix thoroughly by gentle inversion (5 to 10 times).

STORAGE

- This product is shipped on dry ice.
- Self-defrosting freezers are not suitable for storage.
- Do not use past expiration date.

Storage Temperature	Maximum Storage Time	Additional Storage Instructions

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A Site of Pinner Scientific Company LLC