BAUSCH+LOMB

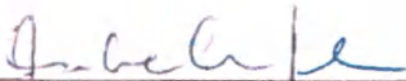
See better. Live better.

1400 North Goodman Street
Rochester, NY 14609
585.338.6000
www.bausch.com

For Russia
Federal Service on Surveillance in Healthcare

Certified True Copy Letter

We, Bausch & Lomb Incorporated, hereby swear (or affirm) that the attached Instruction for Use for Bausch + Lomb Ultra® (samfilcon A) are true and complete photocopies of the documents on file at Bausch & Lomb Incorporated, 1400 North Goodman Street, Rochester, NY 14609, USA.


Isabelle B. Lefebvre, M.Sc. RA
Vice President, Regulatory Affairs

27 January 2020
Date


State of New Jersey

County of Union

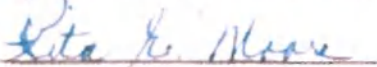Subscribed and sworn to (or affirmed) before me on this 27th day of January, 2020,
by Isabelle B. Lefebvre, proved to me on the basis of satisfactory evidence to be the
person(s) who appeared before me.

Signature of Notary Public

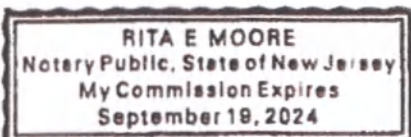BAUSCH + LOMB ULTRA® (samfilcon A)

MoistureSeal™ technology

INSTRUCTIONS FOR USE

Production: Rochester

Market: Russian Federation

BAUSCH + LOMB ULTRA®*MoistureSeal™ technology***⚠ INSTRUCTIONS FOR USE**

This leaflet is intended to inform users of Bausch + Lomb (samfilcon A) hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

PRODUCT NAME	MATERIAL	WATER CONTENT	PACKAGING SOLUTION
Bausch + Lomb Ultra® (samfilcon A)	samfilcon A	46% by weight	Borate buffered saline with Poloxamine

Description and wear

The lenses are designed for daytime or prolonged wear, and should be replaced at the interval recommended by a specialist. Single use of the lenses with subsequent disposal is also allowed. If used repeatedly, the lenses should be cleaned, washed, and disinfected before re-insertion.

Recommended wearing period:

Daytime wear (lenses removed at bedtime): 30 days.

Prolonged wear (lenses not removed at bedtime): up to 7 days and 6 nights of continuous wear; after that, the lenses should be removed, cleaned, washed, and disinfected before re-insertion. Replacement period of the lenses with prolonged wear: up to 30 days, as recommended by the ophthalmologist.

Single wear

When Soft contact lenses BAUSCH+LOMB ULTRA® (samfilcon A) are prescribed for single wear, they should be disposed of after removal.

Always follow the wear schedule recommended by your ophthalmologist.

These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam. If the packaging is damaged or opened DO NOT USE.

Insertion and removal instructions

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps.

BAUSCH + LOMB ULTRA® (samfilcon A)

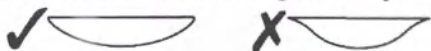
MoistureSeal™ technology

INSTRUCTIONS FOR USE

Production: Rochester

Market: Russian Federation

Pour the lens into the palm of your hand. Make sure the lens is the right way out.

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens onto the white

of the eye. Use a rewetting drop if the lens is difficult to remove.

Caring for your lenses

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

Warnings and precautions

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise

to greater risk of serious eye infections.

Do not wear lenses while sleeping.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes

Contact your eye care practitioner before wearing contact lenses when:

- Dry eyes (inadequate tear fluid)
- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication

If you experience any of the following problems, remove your lenses immediately:

BAUSCH + LOMB ULTRA® (samfilcon A)

MoistureSeal™ technology

INSTRUCTIONS FOR USE

Production: Rochester

Market: Russian Federation

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal and insertion of a new lens, then remove the lens and contact your eye care practitioner. These symptoms can lead to more serious conditions if ignored.

Shipping, storage, and use conditions

Storage: between 5 °C and 40 °C. Do not freeze.

Conditions to avoid:	Temperature:	Avoid temperatures above 40 °C and below 5 °C, do not freeze. Store at ambient temperature.
	Light:	Avoid exposure to direct sunlight.
	Shock:	Avoid strong shock.
	Humidity:	Avoid humidity above 80%, store below 60%.
Substances to avoid:	Water:	Avoid contact with water.
	Saliva:	Avoid contact with saliva.
	Tobacco smoke:	Avoid long stay in tobacco smoke-filled rooms.

Shelf life

5 years.

Recycling:

Soft contact lenses are non-toxic and do not create an excessive load on a person or the environment during operation, storage or transportation. To prevent damage to the environment, it is necessary to comply with safety requirements, storage conditions, transportation and operation.

Soft contact lenses are sent for disposal in accordance with the hygienic requirements of SanPin 2.1.7.2790-10 "Sanitary and epidemiological requirements for the management of medical waste" adopted in the Russian Federation.

BAUSCH + LOMB ULTRA® (samfilcon A)

MoistureSeal™ technology

INSTRUCTIONS FOR USE

Production: Rochester

Market: Russian Federation

Soft contact lenses to be disposed of after use, as well as in connection with the expiration of the shelf life or in the case of violation of the integrity of the package, belong to the hazard class A. Disposal into containers with household garbage is allowed.

Warranty

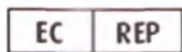
No warranty is provided for the soft contact lenses.

Reference guide for symbols labels and cartons

Fee paid for waste management per 94/62/EC

Bausch & Lomb Incorporated
1400 North Goodman Street
Rochester NY 14609, USA

Bausch & Lomb Incorporated
Cork Road Industrial Estate
Waterford, X91 V383, Ireland

Date of Issue: 2018-07

Indicates the CE conformity marking and the notified body number

®/™ denote trademarks of Bausch & Lomb Incorporated. © Bausch & Lomb Incorporated.

Authorized representative of the manufacturer in the Russian Federation:

Limited Liability Company "Bausch Health"

Russia, 115162, Moscow, st. Shabolovka bld. 31, p. 5, tel / fax + 7-495-510-28-79

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