EWAC

MEDICAL

We get you moving**EWAC MEDICAL**

Dulleweg 18

1721CT Broek op Langedijk

Operating Instruction Manual

Underwater Treadmill Pooltrack

Instruction Manual

Part 1:

User Guide

Read and follow all instructions.

This user guide is a part of the appliance and must be always ready at hand.

Introduction

This user guide is intended to provide a first-time user with the essential information needed to use the EWAC underwater treadmill safely, without going into too much detail. This guide must be read by anyone about to use the underwater treadmill before starting an exercise or treatment programme. Alternatively, the person supervising the programme can explain the safety requirements listed below to their client. This guide forms part 1 of the instruction manual, the full operating instructions are in part 2. Underwater treadmill Pooltrack may be hereafter referred to as the underwater treadmill, treadmill or Pooltrack.

Indications for use:

- trauma (fractures, damage to ligaments and muscles) and their effects.
- diseases of the musculoskeletal system
- neurological disorders, consequences of stroke and traumatic brain injuries related to the violation of the stereotype of the movement.
- vascular disorders (including endarteriit), such as vegetative-vascular dystonia, migraine.
- emotional disturbances: neurosis, chronic fatigue syndrome, depression and anxiety.
- in paediatric practice rehabilitation exercises in the pool, useful for neurological disorders, developmental delay, scoliosis, cerebral palsy, emotional anxiety and even autism.

Contraindications:

- Acute period of the disease
- Colds
- Skin diseases.

Safety Precautions Before Using The Underwater Treadmill

- You must be supervised when using the underwater treadmill unless you have read and understood the full operating instructions or have been trained by someone familiar with them.
- Individuals weighing more than 250 kg (386 lbs) with 50% weight bearing (buoyancy) must not use the underwater treadmill.
- Always wear footwear suitable for exercise in water.
- If you have a medical condition or are pregnant, you must consult your doctor before using the underwater treadmill. You must also inform the person supervising the exercise programme about the condition.
- Never use the underwater exercise treadmill after a heavy meal or when under the influence of alcohol or sedatives.

Safety Measures While Using The Underwater Treadmill

- Never use the underwater treadmill yourself unless trained by the person supervising you, or unless you have read the full operating instructions.
- You should start at the lowest speed before increasing the speed.
- You can stand astride the treadmill on the side platforms. Once the treadmill is running, you can gradually step onto the belt. You can use the same technique to step off the treadmill onto the side platforms.
- You can stop the treadmill at any time by pressing the emergency stop button. This will decelerate and stop the treadmill.
- Never attempt to correct a fault with the underwater treadmill, report the fault to the person supervising.
- Avoid submerging your head in the water.

Instruction Manual Part 2:

Operating Instructions


1. List of Contents

1. List of Contents
2. Introduction
- 2.1 The Layout of the Accompanying Documentation
- 2.2 Responsibilities of Users
3. Safety When Using the Treadmill
- 3.1 Weight of the User
- 3.2 Age of the User
- 3.3 Emergency stop
- 3.4 Personal Protective Equipment for the Client
- 3.5 Personal Protective Equipment for the Supervisors
4. Operation
- 4.1 Starting the Underwater Treadmill
- 4.2 Touch-Screen Operation
5. Touch-Screen Information Messages
6. Touch-Screen Settings & Administration
7. Specifications
8. Disposal
9. Storing / Transportation
10. Transport package
11. Packaging and marking
12. Warranty
13. Maintenance and repair

2. Introduction



The EWAC Medical underwater treadmill provides an ideal environment for reduced weight-bearing exercise, a personal hydrotherapy environment where the client's progress can be easily monitored. It is used to provide early-stage post-stroke, post-traumatic or post-surgical rehabilitation for patients of any age group.

The exercise Treadmill Standing areas are provided on either side of the treadmill, enabling the user to step quickly and easily off the treadmill. High-visibility markers on the treadmill belt indicate the speed of the treadmill. Support handrails are provided for user comfort and safety.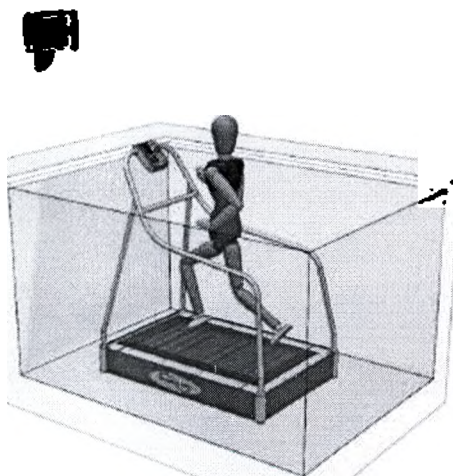

The Control system. The external touch screen mounted on the treadmill and easily reached by the operator provides intuitive adjustment of treadmill speed with minimal training. The removable remote control device, mounted outside the pool, may be managed with the remote control. Treadmill speed can be quickly adjusted to either pre-set values for convenience or manually adjusted to give intermediate speeds.

External treadmill mounted touch screen	Operator control is via removable remote control
---	--

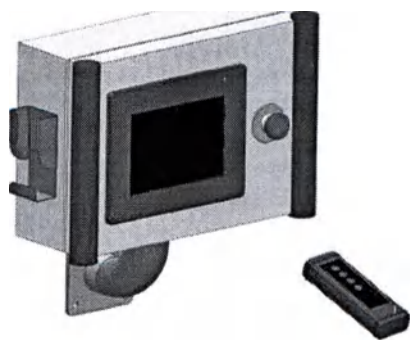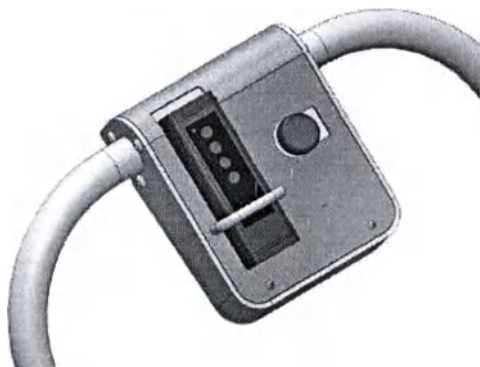2.1 The Layout of the Accompanying Documentation

The instructions for the EWAC Medical underwater treadmill system are split into four sections:

- 1 User Guide: simplified instructions intended for clients.
- 2 Operating Instructions: these instructions, for day to day operation of the treadmill.
- 3 Maintenance Instructions: the technical details of the system, together with the maintenance schedules.
- 4 Installation instructions: specialised instructions for installation and decommissioning.

The users of the EWAC Medical underwater treadmill fall into four separate groups:

- 1 Clients on the treadmill undertaking exercise or undergoing treatment.
- 2 Supervisors monitoring performance or administering treatment outside the pool.
- 3 Individuals responsible for servicing and maintaining the system.
- 4 Individuals transporting, installing, commissioning, decommissioning or disposing of the system.

Each section of the instruction manual is primarily addressed to one group of users. The table below shows which sections of the manual should be read by each user group.

2.2 Responsibilities of Users

	Clients	Supervisors	Maintenance Personnel	Installation Personnel
User Guide	X	X	X	-
Operating Instructions	-	X	X	X
Maintenance Instructions	-	-	X	X
Installation Instructions	-	-	-	X

Set out below is a guide to the responsibilities of each group of users.

Clients undertaking exercise or treatment

- The requirements of the safety notices and user guide must be complied with by the client or patient using the exercise treadmill.
- The client must not attempt to correct a fault or troubleshoot the treadmill system.
- If the client is using the underwater treadmill without professional assistance, they must consult with their medical practitioner to assess whether they are fit enough to undertake exercise without risk to themselves.
- If the client is using the underwater treadmill without professional assistance, then their responsibilities include those ascribed to supervisors.

Supervisors

- The safety notices and user guide must be made available to the client before an exercise or treatment programme commences. The supervisor must ensure that the requirements of the safety notices and user guide are understood by the client.
- The supervisor must have read and understood these operating instructions before using the underwater treadmill.
- The supervisor must only attempt to correct or troubleshoot those faults that are described in these operating instructions.
- If the treadmill is used to treat a physical medical condition, then risks to the individual undertaking treatment must be evaluated by the person administering the treatment.
- Likewise, the responsibility for assessing the suitability of a fitness training program for a client lies with the person specifying the program.

3. Safety When Using The Treadmill

3.1 Weight of the User

The underwater treadmill has been load tested to 125 kg in dry conditions. Individuals weighing more than 250 kg (550 lbs) with 50% weight bearing (buoyancy) must not use the treadmill.

3.2 Age of the User

It is recommended that children under 12 years old do not use the underwater treadmill. The risks to an individual under this age must be evaluated by the person administering the treatment or prescribing exercise programme.

3.3 Emergency stop

Pressing the emergency stop mounted on the operator panel stops the treadmill. The emergency stop located on the main control panel operates in the same way. When pressed the emergency stop push-button will latch in place. To reset the button, twist it in the direction indicated by the arrows on the button, this will allow it to pop out again. The "reset" button on the operator panel touch screen will then need pressing. See section 5 for more details on the emergency stop operation.

3.4 Personal Protective Equipment for the Client

Appropriate footwear must be worn when using the underwater treadmill, as there is a risk of minor injury to feet coming into contact with the treadmill platforms. If this footwear is intended for general use it should be manufactured from non absorbent material that can be properly cleaned between users. For regular users personal footwear is recommended.

3.5 Personal Protective Equipment for the Supervisor

No personal protective equipment is required for supervisors working outside the pool.

4. Operation

4.1 Starting the underwater treadmill

Normally the underwater treadmill is left on continuously. If the power is switched off the procedure for powering up the system is described below:

- 1 Turn on the power at the mains supply point and at the isolator on the main control panel.
- 2 Press the "reset" push-button on the touch screen.
- 3 The system it is ready for use.

4.2 Touch-Screen Operation


The underwater treadmill is controlled using the touch-screen located on the remote control. This section shows a typical sequence of operations for using the underwater treadmill touch-screen.

1. The start screen

To start the underwater treadmill, lightly press anywhere on the touch screen.

2. The menu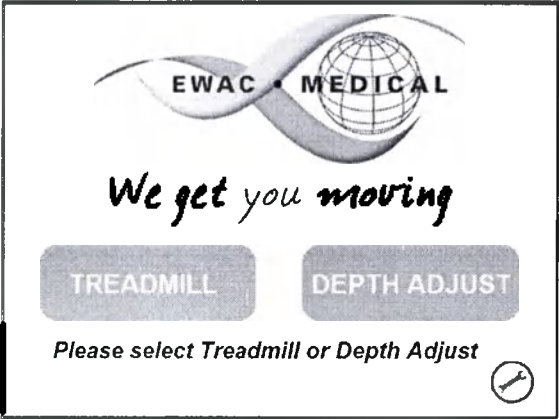


Two options are displayed TREADMILL or DEPTH ADJUST.

TREADMILL: This option enables the user to control the treadmill.

DEPTH ADJUST: This option enables the user to control the adjustable floor in the pool if your treadmill is mounted on the EWAC MEDICAL adjustable floor. The adjustable floor is user supplied. If DEPTH ADJUST options is selected continue to step 3. If TREADMILL option is selected move to step 5.


3. The Ligon

To access DEPTH ADJUST the user name and password are required.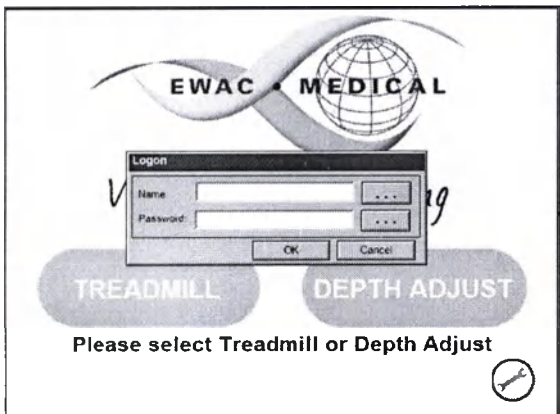

Enter the name "**MASTER**" and the password "**CONTROL**" then press OK.

4. The Depth Adjust menu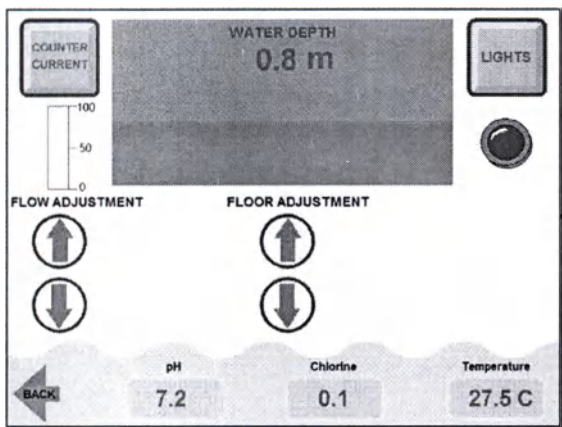


This menu shows the pool control options.

- Pressing the FLOOR ADJUSTMENT arrow buttons will alter the water depth.
- Pressing the COUNTER CURRENT button will turn on/off the water jet, FLOW ADJUSTMENT arrows will alter the water flow.
- Pressing the LIGHTS button will turn on/off the pool lights.

The water pH, Chlorine and Temperature are displayed in the windows at the bottom of the screen.

-Press the BACK arrow to return to the main menu (Step 2).

5. The Treadmill menu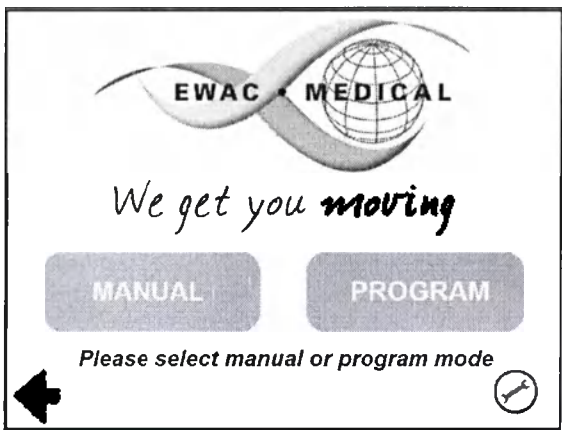

Two options are displayed. The choice is to use the treadmill in either MANUAL or PROGRAM mode.


MANUAL: This option enables the user to select the desired treadmill speed manually.

PROGRAM: This option enables the user to create training programs, or use previously stored training programs.

If **MANUAL** option is selected continue to step 3. If **PROGRAM** is selected move to step 6.

6. The treadmill speed menu

This menu shows the options treadmill speed.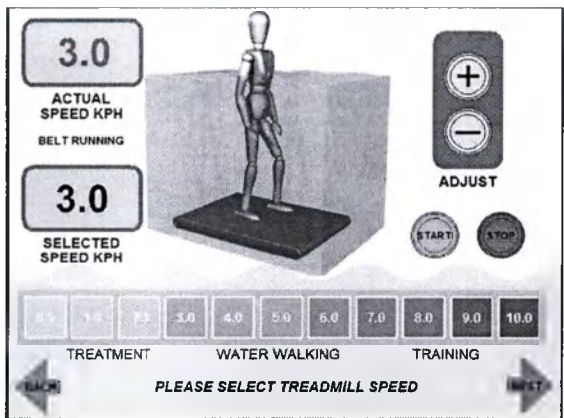


-Pressing any of the pre-set speeds on the scale and then pressing START will start the treadmill.

-The + / - buttons can be used to fine tune the treadmill speed.

-The display window in the top left of the screen indicates the actual treadmill speed.

-Pressing the button marked STOP will slow down and stop the treadmill at any time.

7. Current profile screen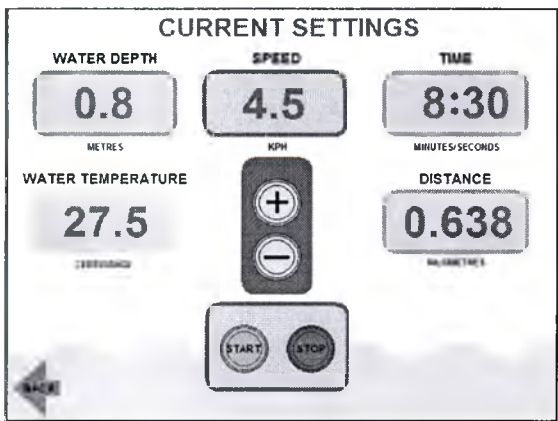

This screen summarises the current settings of the treadmill. The + / - buttons can be used to fine tune the treadmill speed.

-Pressing the button marked STOP will slow down and stop the treadmill at any time.


8. Remote mode screen

The REMOTE CONTROL can be used to adjust the current settings of the underwater treadmill.

-Pressing the START button will start the treadmill.

-Pressing the STOP button will slow down and stop the treadmill at any time.

-Pressing SPEED UP will increase the treadmill speed.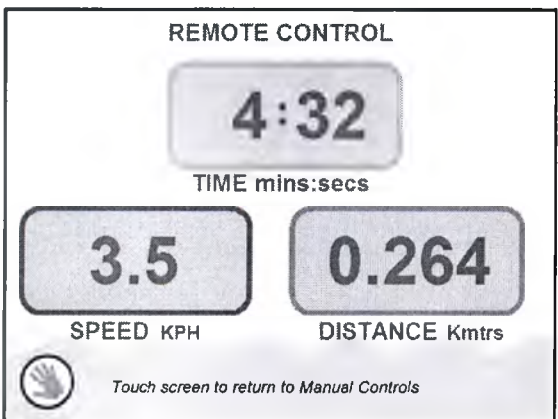

-Pressing the SLOW DOWN will decrease the treadmill speed.

If PROGRAM is selected


9. Training Program Selection and Setting

or create a new program.

This screen allows the user to run a stored training program, modify an existing program

-A list of stored training programs are show in the table on the left hand side of the screen, to navigate between these use the up and down buttons. The details of the program are displayed in the tables on the right of the screen.

-Training programs can be divided into stages, from 1 to3.
-Once the desired training program has been chosen press LOAD and then RUN.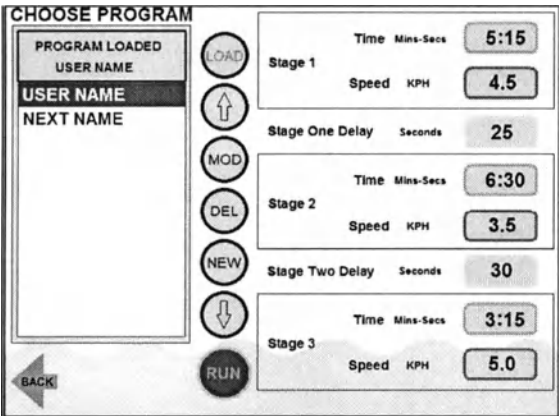


10. Training program, profile screen

This screen summarises the current settings of the training program running.


-The treadmill speed can be adjusted with the

+ & - buttons.
-Pressing the STOP button will end the program.
-Pressing the NEXT arrow will display all stages of the current training program.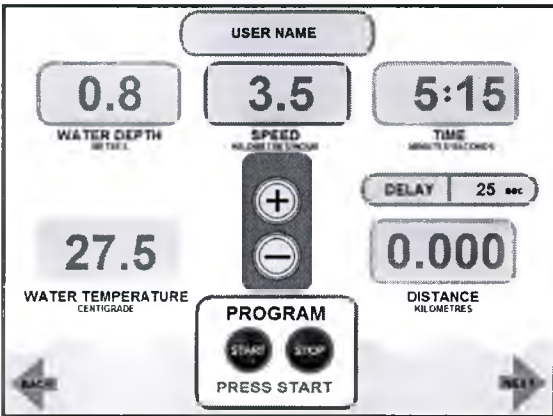

11. Current training program profile screen

This screen summarises all stages of the current training program running.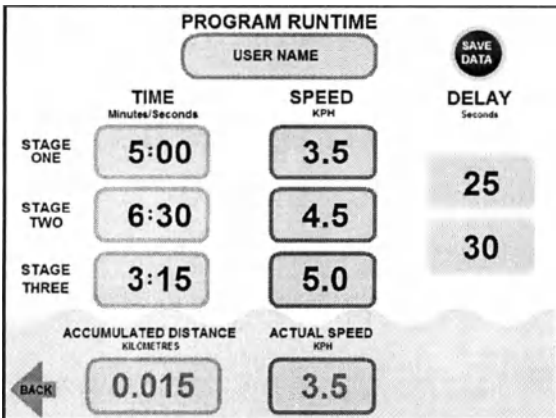

12. Training program new and modification

From the Training Program Selection and Setting screen (image 9). Press buttons marked MOD or NEW to change or create new training program.

-Press the individual windows to change the values.

-Press SAVE DATE to save program.

-Press the BACK arrow to return to the training program selection screen.

13. Training program deletion

From the Training Program Selection and Setting screen (step 9). Press buttons marked DEL to delete a training program.

-A list of stored training programs are shown in the table on the left hand side of the screen

-To scroll up and down the list use the up and down arrow buttons.

-Confirmation of your selection is requested when deleting a program.

-Press the BACK arrow to return to the training program selection screen.

5. Touch-Screen Information Messages

If a fault occurs in the underwater treadmill control system an information message is displayed on the touch screen. Figure 2 shows a typical message.

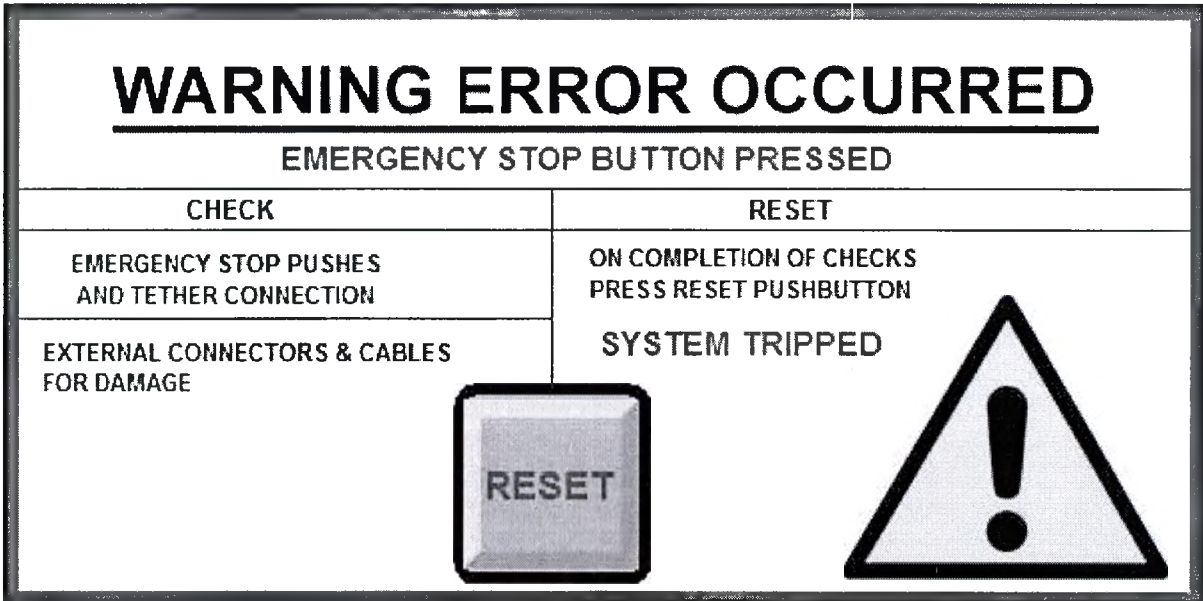
Figure 2

There are many information messages and a full list is included in the maintenance section. For quick reference, the more common messages that could be encountered during day to day operation are included in the table below.

Message	Cause	Remedy
Emergency stop button pressed.	1/ The emergency stop on the operator panel is pressed in. 2/ The emergency stop on the main electrical control panel is pressed in.	Press ACKNOWLEDGE on the screen. Release the e-stop by twisting it so that it springs outwards. Then press RESET on the settings & administration screen. Then press RETURN to return to the screen displayed before the error or press MODE SELECTION to return to the introductory menu.


6. Touch-Screen Settings & Administration

From most screens the Settings and Administration menu can be accessed by pressing the

button.

1. Settings & Administration Menu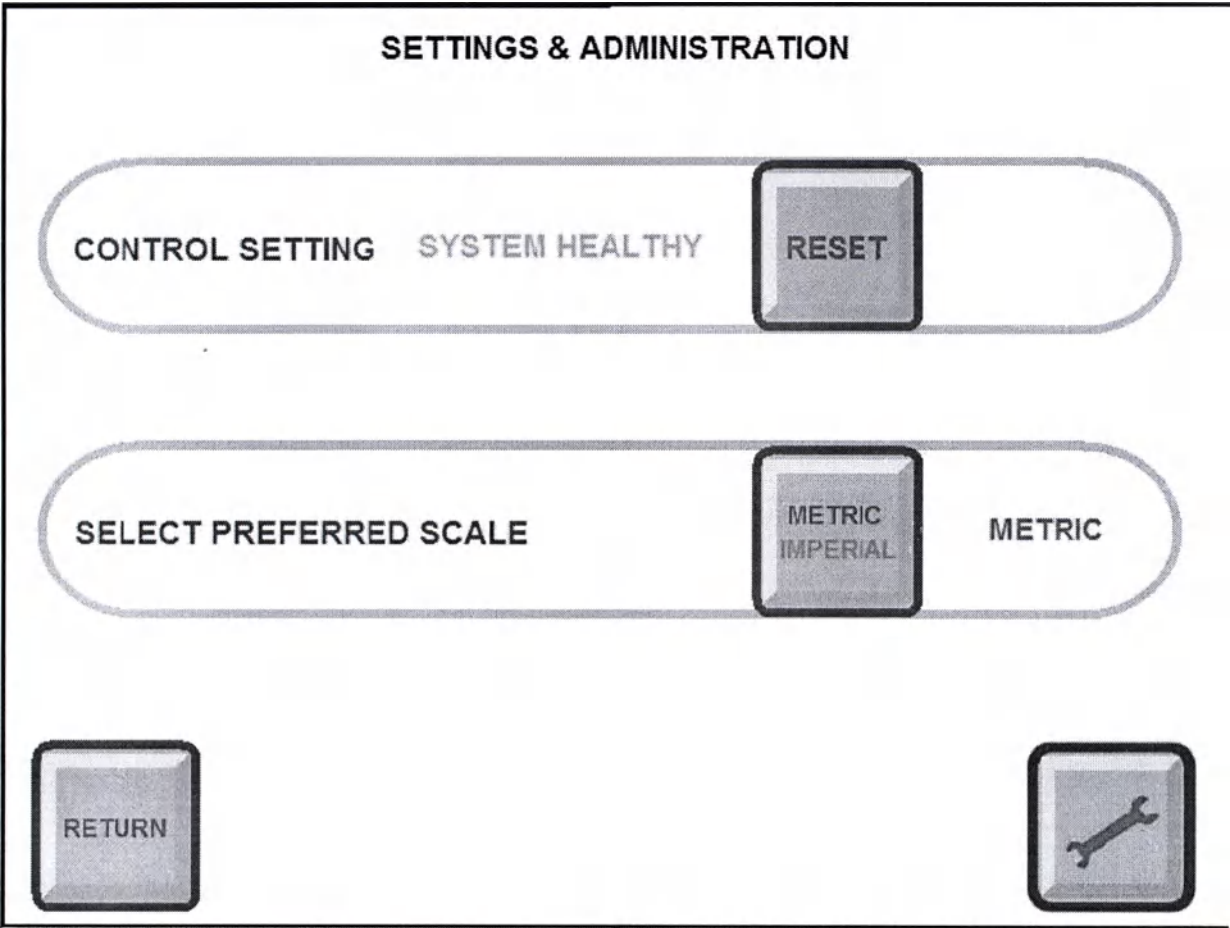

Press the RETURN button to return to the screen displayed before. Pressing the spanner symbol will take you to the Maintenance 01 screen.

2. Maintenance 01 screen

This screen displays the system condition.

MAINTENANCE 01

INPUTS

I1

SPEED SENSOR

I2

EMERGENCY STOP

I3

SPARE

I4

SPARE

I5

IR TREADMILL REMOTE START

I6

IR TREADMILL REMOTE STOP

I7

IR TREADMILL REMOTE INCREASE

I8

IR TREADMILL REMOTE DECREASE

I9

SPARE

I10

SPARE

I11

SPARE

I12

SPARE

I13

SPARE

RETURN

OUTPUTS

Q1

RESET RELAY

Q2

TREADMILL ENABLE

Q3

SPARE

Q4

TREADMILL FORWARD

Q5

SPARE

Q6

SPARE

Q7

SPARE

Q8

SPARE

Q9

SPARE

Q10

SPARE

LOG ON

Full details of the maintenance 02 (including the security PIN) are included in the maintenance instructions. Press the RETURN tab to return to the Settings and Administration menu.

7. Specifications

Specifications	Value	Tolerances
Underwater Treadmill Pooltrack		
Treadmill load capacity, 17V AC Drive	250 kg (550lbs) with 50% weight bearing	±5%
Treadmill speed range	0,1 – 11.0 km/h (0.06 – 7 mph)	±5%
Weight of the assembled treadmill	200 kg	±5%
Dimensions	Length – 1,540 mm, width – 1,040 mm, height – 1,740 mm (including support handrails)	±5%
Overall dimension of outer screen	12x27x32 cm	±5%
Treadmill acceleration	0.25 m/s ²	±5%
Treadmill deceleration	0.25 m/s ²	±5%
Weight of the assembled transformer	17 kg	±5%
Hydraulic environment	Potable water	-
Mains voltage	3 x 230/400 V 50 Hz	-
Testing voltage	12 V	-
Electrical supply to main control panel	16 A 220/240 Volt Single Phase 50/60Hz	-
IP ratings, submerged components	IP68	-
Voltage of TREADMILL MOTOR	17 V AC	-
Voltage at OPERATOR PANEL	12 V DC	-
Voltage of MAIN CONTROL PANEL	220/240 V AC	-

Safety class: type B, class of parts connecting with patient I by classification EN 60601-1.

Instruction Manual Part 3:

Maintenance Instructions

1. List of Contents

- 1. List of Contents
- 2. Safety During Maintenance
- 3. Routine Inspection and Maintenance
 - 3.1 Treadmill
 - 3.2 Side Platforms
 - 3.3 Treadmill drive
- 4. Summary Tables
- 5. Touch-Screen Maintenance mode
- 6. Troubleshooting
 - 6.1 Touch-screen error messages
- 7. Lifespan of the underwater treadmill
- 8. List of Accompanying Documentation

2. Safety During Maintenance

- Do not bring mains voltage equipment, such as lighting, heaters or power tools into the pool area.
- Do not use mains voltage equipment around the pool area unless the pool is drained.
- Ensure that the power supply to the main control panel is isolated before carrying out work inside the panel.
- Employ a safe system of work when working around the pool.
- All maintenance personnel must be familiar with the underwater treadmill.

3. Routine Inspection And Maintenance

3.1 Treadmill

Inspection:
Check the surface of the treadmill belt daily for signs of wear. If present stop using the underwater treadmill and contact the local distributor or EWAC Medical for advice.
Routine Maintenance: None.
PPE required when performing maintenance: None.

3.2 Side Platform

Inspection:
Check the position and fit daily for signs of movement. If present stop using the underwater treadmill and contact the local distributor or EWAC Medical for advice.
Routine Maintenance: None.
PPE required when performing maintenance: None .

3.3. Treadmill drive

Inspection:

Every 12 months the treadmill drive motor should be removed from the treadmill and inspected for any obvious damage to the shaft seals. The oil filling plug should be removed and the oil inspected to see if any evidence of water ingress, such as milky or emulsified oil, is present. If evidence of water ingress is present, stop using the underwater treadmill and contact the local distributor or EWAC Medical for advice.

Routine Maintenance: None, maintenance free.

PPE required when performing maintenance: None.

4. SUMMARY TABLES

The following tables summaries maintenance schedules of section 3.

Inspection and Maintenance - Daily		
Maintenance procedure	Appendix	Page no.
3.1 Treadmill <i>Inspection:</i> Check the surface of the treadmill belt for signs of wear		
3.2 Treadmill side platforms <i>inspection:</i> Check the treadmill side platforms/guards are correctly fitted		
Inspection and Maintenance - 12 months		
Maintenance procedure	Appendix	Page no.
3.3 Treadmill drive <i>Inspection:</i> The treadmill drive motor should be removed and inspected for seal condition and water ingress.		

5. Touch-Screen Maintenance Mode

To access Maintenance 01 screen press the spanner symbol on the Settings & Administration screen.

1. Maintenance 01

Pressing the **LOGON** button brings up the maintenance 02 access panel. Enter the Name **"Master"** and Password **"Control"**. (mind the capitals, use the SHIFT button!). Then press the NEXT button to access MAINTENANCE 02 screen.

MAINTENANCE 01

INPUTS

I1

SPEED SENSOR

I2

EMERGENCY STOP

I3

SPARE

I4

SPARE

I5

IR TREADMILL REMOTE START

I6

IR TREADMILL REMOTE STOP

I7

IR TREADMILL REMOTE INCREASE

I8

IR TREADMILL REMOTE DECREASE

I9

SPARE

I10

SPARE

I11

SPARE

I12

SPARE

I13

SPARE

RETURN

OUTPUTS

Q1

RESET RELAY

Q2

TREADMILL ENABLE

Q3

SPARE

Q4

TREADMILL FORWARD

Q5

SPARE

Q6

SPARE

Q7

SPARE

Q8

SPARE

Q9

SPARE

Q10

SPARE

LOG ON

2. Maintenance and service 02

Displayed are the factory settings.

MAINTENANCE 02	
pH SCALING 4060 Default Setting ANALOGUE MULTIPLY 0.008 MINUS 3160 7.2	CHLORINE SCALING 3188 Default Setting ANALOGUE MULTIPLY 0.008 MINUS 3176 0.1 %
HELP	DEPTH CALIBRATION 2586 Default Setting ANALOGUE DIVISION 3224 MINUS 7 0.8 M
TEMPERATURE SCALING 6587 Default Setting ANALOGUE MULTIPLY 0.008 MINUS 3150 81.5 F 27.5 C	COUNTER CURRENT PUMP ON ENABLE DISABLE LIGHTS ON ENABLE DISABLE
PREV	EXIT SERVICE

NOTE: Before changing any of these settings you must contact the local distributor or EWAC Medical.

If any of the options stated on the screen are available, in short the following can be done:

-Calibrating pH, CI-and temperature :

- ○ ANALOGUE MULTIPLY has to be 0.008 (AM)
- ○ The value on the top left side is defined as the sensor output value (SOV)
- ○ The MINUS value should only be adjusted (M)
- ○ Read out the true value of the pH, CL-or temperature in the control room (TV).
- ○ Calculate M and enter. $M = SOV - TV/AM$ (pH -> $4060 - 7.2/0.008 = 3160$)

-Depth calibration :

- ○ ANALOGUE DIVISION has to be 3224 (AD)
- ○ Calculate M and enter. $M = SOV - TV \times AM$ (f.e. depth -> $2586 - 0.8 \times 3224 = 6.8$, round this to 7)

-Counter current pump and lights can be enabled or disabled.

Pressing the SERVICE button will display MAINTENANCE 03 screen.

3. Maintenance 03

Displayed are the factory settings.

SERVICE

Version: Fusion AC EWAC01PX

SET SERVICE START DATE

DAY01

MONTH10

YEAR2011

SERVICE REQUIRED

SERVICE RESET

MACHINE CYCLE COUNT

32

SERVICE REQUIRED

COUNT RESET

PREV

EXIT


NOTE: Before changing any of these setting you must contact the local distributor or EWAC Medical.

6. Troubleshooting

6.1 Touch-screen error messages

If a fault occurs in the underwater treadmill control system an information message is displayed on the touch screen. The figure hereunder shows a typical message.

The table below lists all the error messages, together with the cause and remedial action required.

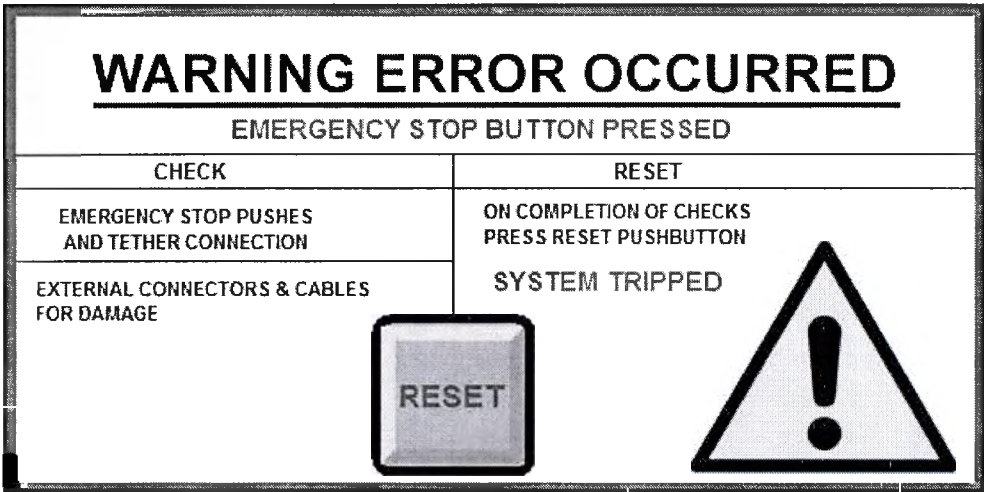Emergency stop button pressed.	The emergency stop on the operator panel is pressed in.	Press ACKNOWLEDGE on the screen. Release the e-stop by twisting it so that it springs outwards. Then press RESET on the settings & administration screen. Then press RETURN to return to the screen displayed before the error or press MODE SELECTION to return to the introductory menu.
Emergency stop button pressed.	The emergency stop on the main electrical control panel is pressed in.	Press ACKNOWLEDGE on the screen. Push the plastic cap on the end of the tether back over the switch. Then press RESET on the settings & administration screen. Then press RETURN to return to the screen displayed before the error or press MODE SELECTION to return to the introductory menu.

7. Lifespan o the Underwater Treadmill

"Lifetime" is defined in section 4 of the essential requirements of 93/42/EEC as the time when the characteristics and performance of the device could be affected to a degree that could compromise safety. From experience with the materials and methods of construction of the device, the maximum lifetime of the sub-assemblies of the device is estimated to be as follows:

Sub-assembly	Maximum Estimated Lifetime
The exercise treadmill, including any electrical equipment fixed to it.	10 years
The electrical control system, including all cabling to the other sub-assemblies.	10 years

If component parts of a sub-assembly are replaced during the estimated lifetime, the subassembly must still be disposed of at the end of its lifetime. If a sub-assembly is replaced entirely, then the estimated lifetime of that sub-assembly starts from the date of its installation.

8. List of Accompanying Documentation

Description	Reference
Declaration of Conformity	DECLARATION OF CONFORMITY
User Guide	Manual part 1
Operating instructions	Manual part 2
Maintenance instructions	Manual part 3
Installation instructions	Manual part 4

Instruction Manual Part 4:

Installation Instructions

1. List of Contents

1. List of Contents
2. Safety During Installation
3. Transportation and Packaging
4. Treadmill Installation
5. Test Procedure
6. Decommissioning & Disposal
7. Intellectual Property and Contact Details

2. Safety During Installation

2.1 Personal Protective Equipment (PPE)

Installation personnel are to wear PPE as required by national or regional regulations. Examples of PPE that may be required are:

- High visibility work wear.
- Safety shoes with toe protection.
- Face mask.
- Safety harness if working at a height.
- Gloves.
- Safety glasses.
- Ear defenders.
- Safety helmet.

2.2 Special safety requirements in pool environment

- Do not bring mains voltage equipment, such as lighting, heaters or power tools into the pool area.
- Do not use mains voltage equipment around the pool area unless the pool is drained.
- Ensure that the power supply to the main control panel is isolated before carrying out work inside the panel.
- Employ a safe system of work when working around the pool.
- All maintenance personnel must be familiar with the underwater treadmill system.

3. Transportation and Packaging

3.1 Shipping details

Shipping crates are used for the EWAC Medical underwater treadmill system. These crates are manufactured from treated wood suitable for export. They include a pallet base to enable unloading with a two tonne capacity fork lift truck.

3.2 Removal of packaging

The shipping crates for the treadmill have sides which are screwed in place. After opening the crates, all internal packaging should be carefully removed from the components. The treadmill can then be removed from the shipping crates using a fork lift truck. At all times great care should be taken when unpacking and moving the treadmill to avoid damage.

3.3 Disposal of packaging

The medical device marking includes the following information:

- an address of the corporate manufacturer;
 - the date of manufacture;
 - a number of the device according to the numbering system of the manufacturing company;
 - the name of the medical device;
 - the electric network parameters;
 - the marking of corresponding the device to the EC Directive 93/42/EEC.
- The transport marking is performed by applying the manipulation signs such as «Fragile. With care», «Protect from moisture», «Protect from sunlight».

Symbol	Meaning
	The date of manufacture
	The marking of corresponding the device to the EC Directive 93/42/EEC
	With care! Refer to the associated documents.
	Refer to the operation manual

4. Treadmill Installation


The Underwater Treadmill should be installed by a local qualified representative of EWAC Medical.

4.1 Mains Electrical connection

6 Amp, 220/240Volt, Single Phase, to the main control panel. Connection to be made to an IP44 standard socket.

4.2 Lifting of free standing treadmill

The free standing treadmill has a weight of 200 kg. This should be lifted using the appropriate lifting equipment, following the guide below.

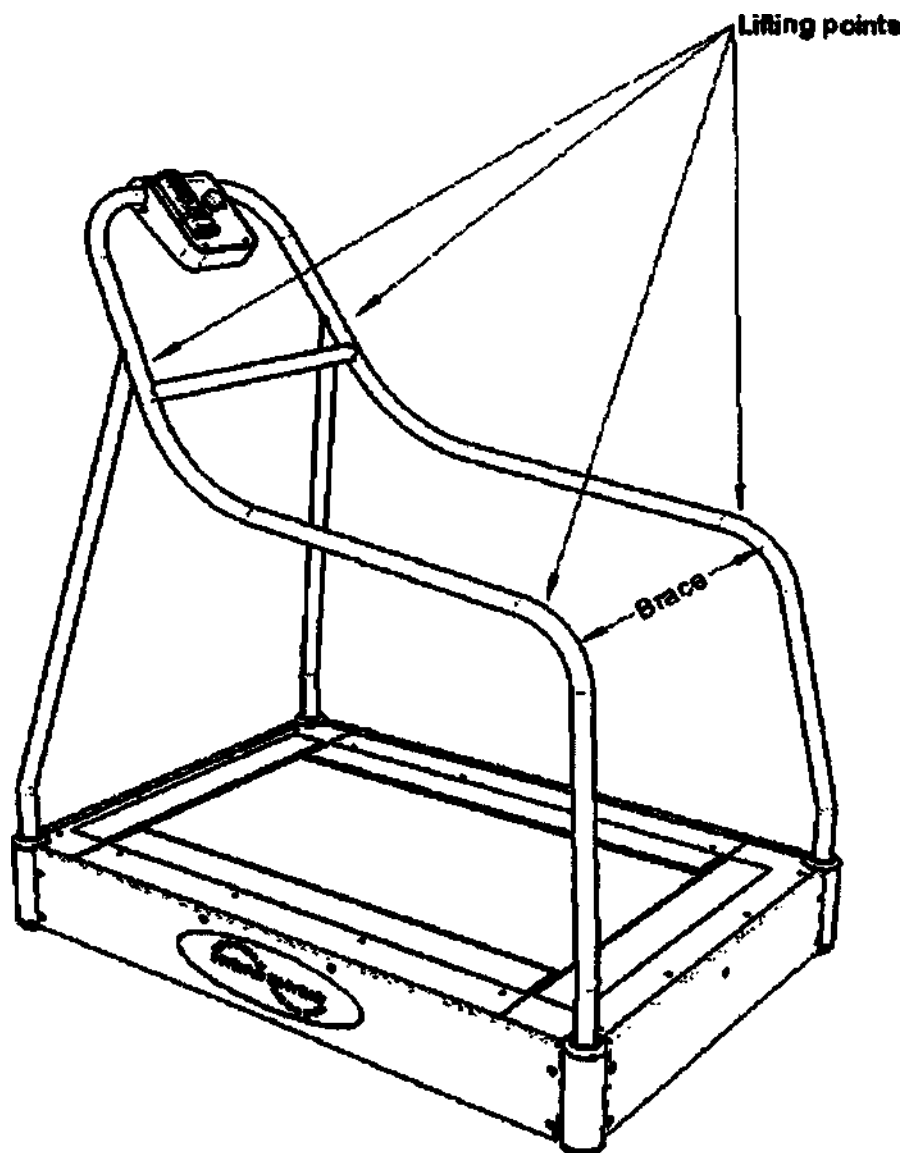5. Test Procedure

After the system has been set up the test procedure, ref. QC1044 Underwater Treadmill Site Test Procedure, must be performed to validate the installation. EWAC Medical must be contacted to obtain a copy of this form. Once the testing is done, the completed form must be sent back to EWAC Medical.

6. Decommissioning & Disposal

6.1 Lifetime of the underwater treadmill

The maximum lifetime of the sub-assemblies of the device is estimated to be as follows:

Sub-assembly	Maximum Estimated Lifetime
The exercise treadmill, including any electrical equipment fixed to it.	10 years
The electrical control system, including all cabling to the other sub-assemblies.	10 years

If component parts of a sub-assembly are replaced during the estimated lifetime, the subassembly must still be disposed of at the end of its lifetime. If a sub-assembly is replaced entirely, then the estimated lifetime of that sub-assembly starts from the date of its installation.

6.2 Decommissioning Procedure

- 1 The pool system is to be drained.
- 2 The electrical supply is to be isolated at the wall socket. No mains voltage equipment such as power tool heaters or lighting must be brought into the pool room until all water has been drained from the system.
- 3 The system is then separated into its component sub assemblies.
- 4 The sub assemblies are then packed into the shipping crates.

6.3 Disposal

The packed up components should be returned to EWAC Medical for disposal. Should this not be possible, the system must be disposed of in accordance with whatever regulations are in force at the time and place of disposal.

7. Intellectual Property & Contact Details

In case of questions, remarks, failures, etc. kindly contact the local distributor or EWAC Medical. The contact details are:

EWAC Medical Dulleweg 18 1721 CT Broek op Langedijk The Netherlands

Tel. +31 (0)226 313457 Fax +31 (0)226 313543 E-mail: office@ewac.nl Website: www.ewac.com

The Human Underwater Treadmill System, Type FOCUS-003 565/001004 and designs

derived from it, are the subject of patent protection. EU Patent App. 06709688.3 US Patent App. 11/815 951 Community Design No. 000547070-0001 US Design Pending App. 29/270 321

8. Disposal

Underwater Treadmill must be disposed of by local public law institutions according to the local requirements and the local disposal regulations.

9. Storing / Transportation

Pooltrack treadmills for hydrotherapy are transported by all kind of transport in closed vehicles in accordance with the requirements and rules of carriage of goods for the every kind of transport.

While transportation, Pooltrack treadmills for hydrotherapy is resistant to the influence of climatic factors at temperature range from -20 up to +40° with relative humidity up to 90% without mechanical impacts during the vibratory loads in the frequency range from 10 to 55 Hz, displacement amplitude 0.35 mm and shock loads with the peak acceleration 10g at duration of shock acceleration action 16 m/s. Acceptable slope - up to 10°.

While storing, Pooltrack treadmills for hydrotherapy is resistant to the influence of climatic factors at temperature range from - 20 up to +40° with relative humidity up to 90%.

10. Transport package

Pooltrack treadmill for hydrotherapy is delivered with the following kit: the Pooltrack treadmill, a connecting cable, 10 m. length, a reducing transformer, an outer screen, size 12x27x32 cm, a remote control, a operating manual, detachable handrails, size 150x65 cm, 2 pieces, a remote detachable control unit. Pooltrack treadmill for hydrotherapy is packed in a polyethylene film and in wooden boxes.

11. Packaging and marking

The medical device marking includes the following information:

- an address of the corporate manufacturer;
- the date of manufacture;
- a number of the device according to the numbering system of the manufacturing company;

- the name of the medical device;
 - the electric network parameters;
 - the marking of corresponding the device to the EC Directive 93/42/EEC.
- The transport marking is performed by applying the manipulation signs such as «Fragile. With care», «Protect from moisture», «Protect from sunlight».

Symbol	Meaning
	The date of manufacture
	The marking of corresponding the device to the EC Directive 93/42/EEC
	With care! Refer to the associated documents.
	Refer to the operation manual

12. Warranty

The warranty is valid for 12 months from the date of purchase.

13. Maintenance and repair

An annual inspection of the equipment should be performed to ensure in its proper operating and safety for the user. We recommend contacting your local qualified service representative for maintenance and repairs. Moreover, we recommend keeping records of all action for technical checkup and current maintenance. Personnel who have not desired qualifications should not open the equipment for any reason. Representatives of the qualified personnel may request a maintenance manual from a local distributor to ensure quality and proper maintenance of the device. EWAC company assumes no responsibility for any inconvenience for the operator or patient because of negligent technical checkup and repair.

All complaints for the device should be addressed to: LLC «Octopus» (Октопус), Russia, Moscow, 129344 Eniseyskaya (Енисейская) 1 street, buid. 1, office 325.

In accordance with the approved standards of safety, always use original spare parts if for any reason the unit needs repair.

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