State of Minnesota - Secretary of State

This Certificate is not valid for use anywhere within the United States of America, its territories or possessions.

Apostille

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1. Country (Pays): United States of America

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2. has been signed by (a été signé par) Irma Vargas**3. acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota****4. bears the seal / stamp of (est revêtu du sceau / timbre de)**

Irma Vargas, Notary Public, State of Minnesota

Certified**Attesté****5. at (à) St. Paul, Minnesota****6. Date (Date) 08/02/2019****7. by (par) Secretary of State, State of Minnesota****8. File No (Sous n°) 1094965600047****9. Seal / Stamp (Sceau / Timbre):****10. Signature (Signature):**

Steve Simon

Steve Simon

Secretary of State
State of Minnesota

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.

This Apostille does not certify the content of the document for which it was issued.

To verify the issuance of this Apostille, see <https://apostille.sos.state.mn.us>.

This certificate does not constitute an Apostille under the Hague Convention of 5 October 1961, when it is presented in a country which is not a party to the convention. In such cases, the certificate should be presented to the consular section of the mission representing that country.

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3M

To whom it may concern


In my capacity as International Regulatory Affairs Associate, 3M Health Care Business, St. Paul, MN, USA, I certify that the attached IFU (Instruction for Use) for for 3M™ Littmann® Cardiology IV™ in English is truthful and accurate.


Regina Feferman
International Regulatory Affairs Associate
3M Health Care

7/30/19
Date

State of Minnesota)
County of Ramsey) SS

Subscribed and sworn to before me this 30th day of July 2019

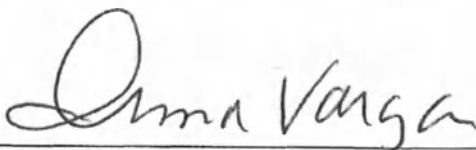Irma Vargas, Notary Public
My Commission Expires: January 31, 2020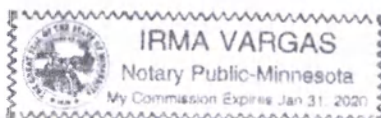

Instruction for use

Littmann® Cardiology IV™ Stethoscope

Intended use

The 3M™ Littmann® Cardiology IV™ Stethoscope can be used for auscultation of heart, lung and other body sounds.

Scope of use:

3M™ Littmann® Cardiology IV™ Stethoscope is intended for medical diagnostic purposes only. This product is not designed, sold or intended for use except as indicated.

Contraindications

No contraindications have been identified for these products.

Potential complications/side effects

No potential complications/ side effects are associated with this product.

Precautions

Do not immerse your stethoscope in any liquid.

Do not subject your stethoscope to any sterilization process.

Avoid storing your stethoscope in extreme heat.


To prevent staining of stethoscope tubing, avoid contact with pens, markers, newsprint, or other printed materials.

Contents (Included in box):


- I. Littmann® Cardiology IV™ Stethoscope (version 56 cm) contents:
 1. Littmann® Cardiology IV™ Stethoscope
 2. Snap Tight Soft-Sealing Eartips – (Small -2 pc., Large -2 pc.) (if necessary)
 3. Nonchill Bell Sleeve (if necessary)
 4. Instruction for use
- II. Littmann® Cardiology IV™ Stethoscope (version 69 cm) contents:
 1. Littmann® Cardiology IV™ Stethoscope
 2. Snap Tight Soft-Sealing Eartips – (Small -2 pc., Large -2 pc.) (if necessary)
 3. Nonchill Bell Sleeve (if necessary)
 4. Instruction for use

Instructions for Use


Insert eartips in a forward direction.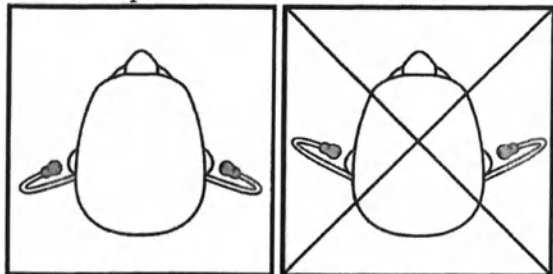

Adjust the Headset Tension

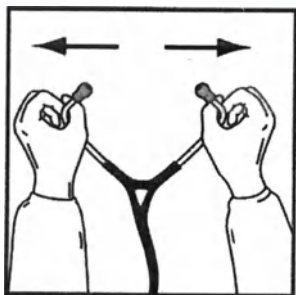To reduce the spring tension, pull the eartubes apart.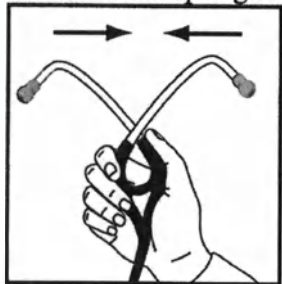


To increase the spring tension, squeeze the eartubes together.


Select the Active Side of the Chestpiece

Only one side of the chestpiece is acoustically active at a time, identified by a mark on the stem. Rotate the chestpiece to change which side is active.

Listen to Low and High Frequency Sounds with Littmann® Tunable Diaphragms on Each Side of the Chestpiece. Use the large side for adults and the small side for pediatric or thin patients, around bandages and carotid assessment.

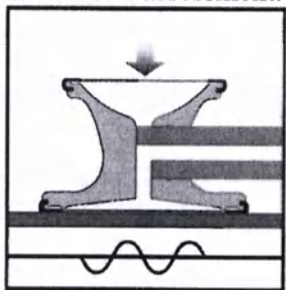
Low frequencies: apply light pressure to the chestpiece.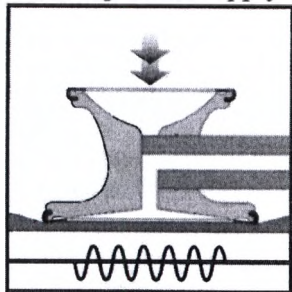

High frequencies: apply firm pressure to the chestpiece

Convert to a Traditional Open Bell

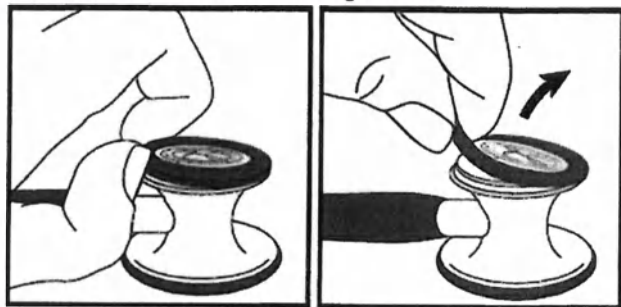Pinch the rim of the diaphragm on the small side; pull and lift from the chestpiece.

Attach the sleeve by placing one side on the edge and stretching it over the chestpiece.


To install diaphragm, remove the nonchill sleeve. Insert the edge of the diaphragm into the groove and roll the rim around the chestpiece.

Cleaning and Disinfection

Clean your stethoscope between exams using a 70% isopropyl alcohol wipe or a disposable wipe with soap and water. A 2% bleach solution may be used to disinfect your stethoscope; however, the tubing may experience a slight discoloration after exposure to bleach. The tunable diaphragms, nonchill bell sleeve and eartips may be removed for thorough cleaning.

To remove the eartips from the headset, pull firmly. To assemble the eartips, push the small side of the eartip firmly onto the eartube until it snaps fully into place. Ensure all parts and surfaces are dry before reassembly. Be sure the eartips are firmly attached to the headset before inserting them into your ears.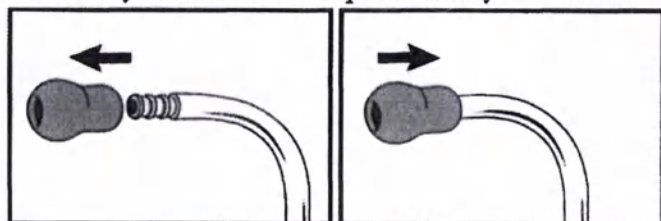

Storage, Transportation and Working conditions:

Operating range is -35° to 60° C, 15 to 93% relative humidity. The pressure is not regulated.

Storage and transport range is -40° to 55° C, 15 to 93% relative humidity. The pressure is not regulated.

During transportation the products may experience short term temperature variations without affecting

product safety and performance.

Transportation of medical device is carried out by all type of transport.

Technical specification:

Dimensions with tolerances.

3M Littmann® Stethoscope Family	Soft Tubing Diameter (cm)	Chest Piece Diameter (cm)		Total Length (cm)	Total Weight (grams))
		Adult Side	Pediatric Side		
Cardiology IV™ (69 cm)*	1.2 ± 0.2	4.6 ± 0.2	3.6 ± 0.2	69.9 ± 2.0	177 ± 12
Cardiology IV™ (56 cm)*	1.2 ± 0.2	4.6 ± 0.2	3.6 ± 0.2	57.2 ± 2.0	167 ± 12

*3M™ Littmann® Stethoscope Cardiology IV™ exist in 2 versions.

Tubing (binaural) Length (cm): 69.9 ± 2.0, 57.2 ± 2.0

Outside Diameter of Tubing (Single-piece tubing) (cm): 1.2 ± 0.2

Outside Diameter of Tubing (Dual-lumen tube) (cm): 1 ± 0.2

Outside Diameter of Tubing (connection part) (cm): 1 ± 0.2

Wall thickness of Tubing (binaural) (all parts of tubing) minimum 1,7 mm

Length of eartube (from binaural to eartip) (cm) 14±1

Outside Diameter of eartube (mm) 6±0,5

Wall thickness of of eartube (mm) 1-2

Thickness of ChestPiece (cm) 2,72 ± 10%

Outside Diameter of stem (cm) 1± 10%

Inside Diameter of stem (mm) 3-4

Length of stem (from binaural to ChestPiece) (cm) 1 - 1,8

Snap Tight Soft-Sealing Eartips Small: Diameter (mm) 12,01 ± 5 %; Length (mm) 19,76 ± 5 %. Weight (gram) 1,26 ± 5 %.

Snap Tight Soft-Sealing Eartips Large: Diameter (mm) 12,99 ± 5 %; Length (mm) 19,87 ± 5 %. Weight (gram) 1,32 ± 5 %.

Nonchill Bell Sleeve: Outer Diameter (mm) - 35,86 ± 5 %; Outer Diameter (mm) - 29,24 ± 5 %; Thickness (mm) - 3,75 ± 5 %, Weight (gram) - 0,94 ± 5 %.

Headset tension must be adjustable. Force required to change of Headset tension ≤22N (5 lbf)

Fixing strength of ear tips (tearing force): from 22 N to 89 N (5 to 20 lbf (pound force))

Force required to attach ear tips to eartubes: minimum 53 N (12 lbf)

Fixing strength of stem to tubing (binaural) (tearing force): from 89 N to 177 N (20 to 40 lbf (lbf force))

Acceptable frequency range 20-1000 Hz

Pressure level maximum 129 dB peak sound

Medical devices do not contain or deliver medicinal substances.

Medical devices do not contain or deliver human blood derivatives.

No materials of animal origin are intentionally being added to the products.

Maintenance and repair

Repairs must be done by a qualified service person. See Warranty.

Disposal

Dispose of content/container in accordance with the local/regional/national/international regulations.

Medical devices in Russian Federation must be disposed of in the local public entities in accordance with

SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B).

Explanation of Symbols

Manufacturer

European Conformity mark

EC Representative

Batch code

Consult instructions for use.

Not made with natural rubber latex.

LICENSE SYMBOL OF EUROPEAN NETWORK OF INDUSTRY-FUNDED SYSTEMS FOR RECYCLING THE PACKAGING MATERIAL

Littmann® Stethoscope Service and Warranty Program

Please register your stethoscope online at: <http://www.littmann.com/warranty-registration>.

The Littmann® Cardiology IV™ stethoscope is warranted against any defects in material and workmanship for a period of seven (7) years.

Within the warranty period, repairs will be made without charge upon the return of the stethoscope to 3M, except in cases of obvious abuse or accidental damage.

For service and repair in the U.S.A., please visit www.littmann.com/service or call 1-800-292-6298. If you are outside of the U.S.A., please visit www.littmann.com for the contact information of your local 3M office.

3M Company, 3M Health Care,

3M Center, 2510 Conway Avenue, Bldg. 275-5W-06, St. Paul, Minnesota, 55144, USA

1 800 364-3577

www.3M.com

Manufacturing site:

3M Company, 3M Health Care

5400 Paris Road, Columbia, Missouri, 65202, USA

Authorized representative on the territory of the Russian Federation:

AO "3M Russia", Russia

108811, Moscow, Poselenie Moskovskiy, Kievskoe shosse, 22 km, 6, bld.1

+7 (495) 784 7474 / +7 (495) 784 7475 (fax)

e-mail 3mra@mmm.com

For more information, visit www.littmann.com.

3M, Littmann, the L Littmann logo, the L, and Cardiology IV are trademarks of 3M.

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© 3M. All rights reserved.