PROCURADORIA-GERAL DA REPÚBLICA

Ref.ª/réf./ref.

APOSTILLE

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1. País/Pays/Country : **Portugal**
Este documento público/Le présent acte public/This public document
2. Foi assinado por/a été signé par/has been signed by **Filipa Romão**
3. Agindo na qualidade de/agissant dans la qualité de/acting in the capacity of **Advogada 52295L**
4. E tem o carimbo de/est revêtu du timbre de/bears the stamp of **Filipa Romão**

Reconhecido/Attesté/Certified

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This Apostille only certifies the signature, the capacity of the signer and the seal/stamp it bears. It does not certify the content of the document for which it was issued.

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---I recognize the signature on the document annexed of Mr. Eng Carlos de Matos Quaresma Setra, as the legal representant of the commercial company "Laboratório Edol – Produtos Farmacêuticos, S.A.". With sufficient powers to act, certificate by the commercial certify with code number 4468-8842-2529.--

----- I also certify and checked the quality, the powers to act and the identity of the person, by the exhibition of the above mentioned certificate and the display of the citizen card No. 01259040, valid until 01.04.2020, issued by SIC Lisboa.-----

----- The document consists of three sheets, and refers to a Instructions for use, Ophthalmic solution Tayfon soft tear.-----

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The lawyer, holder of the professional certificate No. 52295L, issued by the Lisbon District Council of the Bar Association

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REGISTO ON-LINE NA ORDEM DOS
ADVOGADOS PORTUGUESES SOB O N.º
52295L/636
RECOGNITION
REGISTRATION ON-LINE ON LAWYERS
OF ORDER UNDER PORTUGUESE No.
52295L/636

Filipa Romão
Advogada
Cédula nº 52295L
Rua Actor Chaby Pinheiro, nº 4, R/C Dto
2795-060 Linda-a-Velha
Telm: 967434379 / 913580985
Email: filipa.romao@gmail.com

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---I recognize the signature on the document annexed of Mr. Eng Carlos de Matos Quaresma Setra, as the legal representant of the commercial company "Laboratório Edol - Produtos Farmacêuticos, S.A.". With sufficient powers to act, certificate by the commercial certify with code number 4468-8842-2529.--

----- I also certify and checked the quality, the powers to act and the identity of the person, by the exhibition of the above mentioned certificate and the display of the citizen card No. 01259040, valid until 01.04.2020, issued by SIC Lisboa.-----

----- The document consists of three sheets, and refers to a Instructions for use, Ophthalmic solution Tayfon soft tear.-----

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INSTRUCTIONS FOR USE

Ophthalmic solution Tayfon soft tear

- Moisturizing of eye in case of dryness and eye fatigue
- Provision of comfort conditions while wearing contact lenses
- Quick moisturizing of dry and tired eyes
- Restores the protective properties of the tear film
- Elimination of eye discomfort during prolonged work at the computer
- Protects the eyes from harmful effects of external factors
- Compatible with all types of contact lenses
- Gives your eyes a long-lasting feeling of comfort
- You can bury without removing the contact lenses

Description:

Ophthalmic solution Tayfon soft tear is a sterile, high viscosity solution.

Properties:

Ophthalmic solution Tayfon soft tear provides long maintenance of the ocular surface of the protective film with moisturizing and softening properties.

The lacrimal fluid has an important role in ensuring the health of the ocular surface. Normal tear film continuous layer covering the surface of the eye and protects the cornea and conjunctiva from drying and damage, promotes nutrition, providing their normal functioning and a clear vision. Under the influence of various factors the stability of the tear film can be disturbed, which is manifested by sensation of dryness, burning, foreign body and fatigue in the eye, irritation and redness of the eyes, in some cases, and lacrimation.

Hydroxypropylmethylcellulose, included in the formulation, restores, stabilizes and reproduces the optical characteristics of the tear film, alleviate symptoms of excessive dryness and irritation of the eye.

Composition per 1 ml:

Components	Quantity
Hypromellose (hydroxypropylmethylcellulose)	2.5 mg
<u>Excipients:</u>	
sodium chloride, sodium acetate anhydrous, sodium citrate, potassium chloride, calcium chloride dihydrate, magnesium chloride, ethylenediamine tetra-acetic acid EDTA, chlorhexidine bigluconate, highly purified water, 10% hydrochloric acid or 40% sodium hydroxide up to pH 6.2-7.0	

Indications:

Ophthalmic solution Tayfon soft tear is intended for:

- Additional moistening of the anterior surface of the eye (cornea and conjunctiva) with a feeling of dryness, irritation, foreign body sensation, burning in the eyes;
- Ensuring a comfortable eye condition, moisturizing and eliminating the discomfort that occurs when wearing soft and hard contact lenses;
- Protection of the cornea, elimination of discomfort and eye irritation with reduced secretion of lacrimal fluid, prolonged visual stress (work at the computer, driving vehicles, watching TV,

etc.), as well as from adverse environmental factors (air conditioning, negative climatic conditions, air pollution, etc.) ;

application in the complex treatment of eye diseases, accompanied by a violation of the stability of the tear film (with deformations of the eyelids, lagophthalmus, after operations on the eyelids, dry eye syndrome, etc.);

- Protection of the cornea and relief of discomfort after surgical operations, injuries and burns of the cornea and conjunctiva.

Recommendations and usage:

It is recommended to drip Ophthalmic solution Tayfon soft tear 2 drops in each eye as needed.

The frequency and duration of treatment set individually, depending on the sensations or according to the recommendations of an ophthalmologist or specialist in contact lenses.

Tilt your head back slightly, gently take your finger lower lid, drip 2 drops of solution into the conjunctival SAC, to blink 2-3 times to ensure that the solution is evenly distributed over the surface of the eye.

Recommendations when wearing contact lenses:

While wearing soft and hard contact lenses can be used to Ophthalmic solution Tayfon soft tear in the conjunctival sac without removing the lenses.

For extra comfort before putting on contact lenses every recommended to apply 1-2 drops of solution.

Contraindications:

Hypersensitivity to any component.

Warnings and precautions:

The bottle do not use if the protective membrane cap on the dropper bottle is damaged or missing before first use.

Before each use first drop tools let idle without using.

To avoid infection avoid contact of the tip of the dropper bottle with eyes or any other surface.

After each use, tightly close the bottle-dropper cap.

The solution is intended for personal use only: do not use the same vial more than one person.

Side effects and interaction:

In the case of joint use with ophthalmic drugs, it is recommended to observe a pause of at least 10 minutes between the instillation of eye drops and application of Ophthalmic solution Tayfon soft tear. Eye ointments should always be applied after instillation.

Presentation:

Dropper bottles, 10 ml. 1 dropper bottle with leaflet in a cardboard pack.

Storage conditions:

Keep out of reach of children, store at the temperature of not higher than 25°C. Avoid freezing.

on date:

s. Do not use after expiry date. After first opening, use within 28 days.

Available without a prescription.

Disposal:

After use of the solution, dropper bottles are disposed together with domestic waste.

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Name of the organization that accepts claims / Made-to-order:

JSC Taufarm, 123312, Moscow, Testovskaya St., 10, 11th floor, build I, office18.

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