certified copy

2/2

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Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

Tel.: +49 (0)8152 700-0
Fax: +49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

3M

To whom it may concern

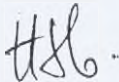
October 14, 2019

Registration documents for Impression Tray – IFU

We, 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that following pages are an excerpt of the Information for Use.

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Instructions for use

Directed Flow Disposable Impression Tray in various sizes and in variants

2019

Directed Flow Disposable Impression Tray

Instructions for Use

Product Description

Impression Tray is a disposable impression tray made of plastic that can be used for all types of impressions. Due to the special design of the tray, the pressure during impression taking is higher than with commonly used standardized trays. This reduces the tendency to form fin-shaped voids caused by the material's flow characteristics. Tungsten carbide burs can be used to customize the tray. Impression Tray is available in three sizes for upper and three sizes for lower jaw. Instructions for Use should not be discarded for the duration of product use.

Indications

Suitable impression materials:

- Elastomeric impression materials of medium and high consistency (excluding putty)
- Alginates

Types of impression:

- All types of preliminary and precision impressions
- Impressions for the production of temporary restorations
- Opposing jaw impressions
- Impressions for the production of splints

Preparatory Steps

Selection of a Tray of Suitable Size

- Select a tray and check the size inside the patient's mouth.
There must be sufficient space between the wall of the tray and the row of teeth to ensure that no permanent deformation results when the tray is removed from the row of teeth. As a rule of thumb, this space should be 3-4 times the depth of the undercut of the row of teeth viewed from occlusal.
- Interfering areas, e.g., at the lip frenulum or dorsal end of the tray, can be removed with a tungsten carbide bur. Remove any fins or grinding burs.
- Clean the tray after fitting and before taking the impression and dry thoroughly with a stream of air.
- The tray has dorsal stops so that additional blocking is not required as often as when using other standardized trays. If blocking is nevertheless required, the tray can be blocked at the distal end using wax or putty silicone.

Adhesive

The Impression Tray is equipped with a self-adhesive fleece, making the use of an adhesive unnecessary.

Impression Taking

In the course of positioning commonly available standardized trays in the patient's mouth, fin-shaped voids caused by the material's flow characteristics are often introduced. These voids later become apparent in the model.

The palatal and lingual areas of Impression Tray are specially shaped so that the impression material flows towards the teeth when the tray is inserted into the patient's mouth. This increases the pressure during impression taking and reduces flow-related voids. In order to make use of this effect, however, the tray must be filled with impression material completely to the retention edge. The upper jaw tray is designed to contain an additional palatal groove serving to take up any

excess impression material filled into the tray. This groove prevents the excessive impression material from flowing into the pharynx, where it would cause a gag reflex.

- Fill the tray with impression material completely to the retention edge.
- Slowly insert the tray parallel to the tooth axes.
- Allow the impression material to set.
- Once the impression material has completely set, remove the tray from the mouth.

Disinfection of the Impression

- Place the impression in a standard disinfectant solution.
Allow the disinfectant solution to act for the period of time recommended by the manufacturer.
For details, please refer to the respective Instructions for Use.
- After disinfection, rinse the impression under running tap water for approx. 15 sec.

Disposal

The impression tray is made of polypropylene and can be recycled in accordance with local regulations after the impression material has been removed. It can be discarded along with the impression material in the usual practice waste.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty contract, negligence, or strict liability.

Intended use

Create a precise reproduction of the clinical situation within dental workflows.

Contraindications

Contraindications are not available.

Scope of use:

Dentistry. For professional use only.

Side effects

Side effects are not available.

Warnings and Precautions

For this product no specific precautions have to be followed.

Technical characteristics

Directed Flow Disposable Impression Tray for upper jaw:

size: S: length (mm) $94,2 \pm 0,5$; Width (mm) $69,5 \pm 0,5$; height (mm) $21,0 \pm 0,5$;
size: M: length (mm) $98 \pm 0,5$; Width (mm) $76,4 \pm 0,5$; height (mm) $21,8 \pm 0,5$;
size: L: length (mm) $101,6 \pm 0,5$; Width (mm) $78,5 \pm 0,5$; height (mm) $22,6 \pm 0,5$;

Directed Flow Disposable Impression Tray for lower jaw:

size: S: length (mm) $91,8 \pm 0,5$; Width (mm) $65,6 \pm 0,5$; height (mm) $27,5 \pm 0,5$;
size: M: length (mm) $95,9 \pm 0,5$; Width (mm) $73,1 \pm 0,5$; height (mm) $28,6 \pm 0,5$;
size: L: length (mm) $99,6 \pm 0,5$; Width (mm) $77,1 \pm 0,5$; height (mm) $30,7 \pm 0,5$;

Contents

Directed Flow Disposable Impression Tray in various sizes and in variants:

- I. Directed Flow Disposable Impression Tray for upper jaw in various sizes:
 1. Directed Flow Disposable Impression Tray for upper jaw, size S (10 pcs.); Instruction for use, 1 piece
 2. Directed Flow Disposable Impression Tray for upper jaw, size M (10 pcs.); Instruction for use, 1 piece.
 3. Directed Flow Disposable Impression Tray for upper jaw, size L (10 pcs.); Instruction for use, 1 piece.
- II. Directed Flow Disposable Impression Tray for lower jaw in various sizes:
 1. Directed Flow Disposable Impression Tray for lower jaw, size S (10 pcs.); Instruction for use, 1 piece.
 2. Directed Flow Disposable Impression Tray for lower jaw, size M (10 pcs.); Instruction for use, 1 piece.
 3. Directed Flow Disposable Impression Tray lower jaw, size L (10 pcs.); Instruction for use, 1 piece.

Sterilization

Sterilization is not applicable. This product is not labelled nor otherwise represented as sterile, nor is it intended to be sterilized by the user.

Shelf life

The shelf life of the product is 36 months. See shelf life expiry date on the packaging.

Storage Conditions

No specific storage conditions need to be considered for this product.

Operating Environment

This product is designed to be used at room temperature, no specific humidity and atmosphere pressure rates are considered.

Transportation

During transportation the products may be affected short term temperature variations with no impact to product safety and performance.

Transportation of medical device is carried out by all type of transport.

Maintenance and repair

This medical device isn't subject to maintenance and repair.

Medicinal substances

Medical devices do not contain or deliver medicinal substances.

Human Blood Derivatives

Medical devices do not contain or deliver human blood derivatives.

Animal-Derived Substances

No materials of animal origin are intentionally being added to the products.

Disposal

Dispose of content/container in accordance with the local/regional/national/international/regulations.

Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" (to the waste class B).

Applicable national standards.

A list of applicable national standards is available on request.

Symbols used on packages

The products are supplied in a package with the marking containing the following information and symbols:

	Use by date		Do not reuse
	Catalogue number		Lot number
	Caution	CONTENTS	Contained items
	Recycling code (Low-density polyethylene)		Green dot
	Manufacturer		Symbol for Impression Tray for upper jaw
	Conformity sign		Symbol for Impression Tray for lower jaw
	2-D barcode		
	Product meets the requirements of the applicable EC directives		

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Dental Products

Carl-Schurz-Str. 1

41453 Neuss - Germany

3M and ESPE are trademarks of 3M or 3M Deutschland GmbH.
© 2019, 3M. All rights reserved.

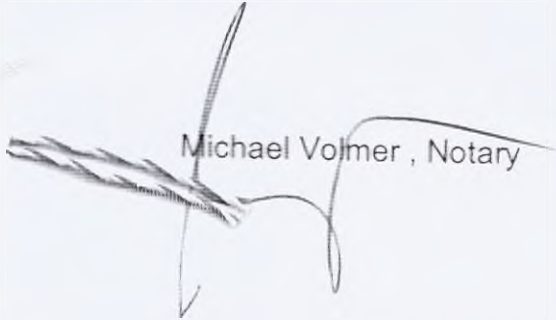
Date of the Information: October 2019

Authorized representative of the manufacturer in RF:

AO "3M Russia", 108811, Moscow, Poselenie Moskovskiy, Kievskoe shosse, 22 km, 6, bld.1
T. +7 (495) 578-74-74

I hereby certify that the document
affixed hereto is a true copy
of the original.

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