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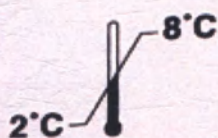
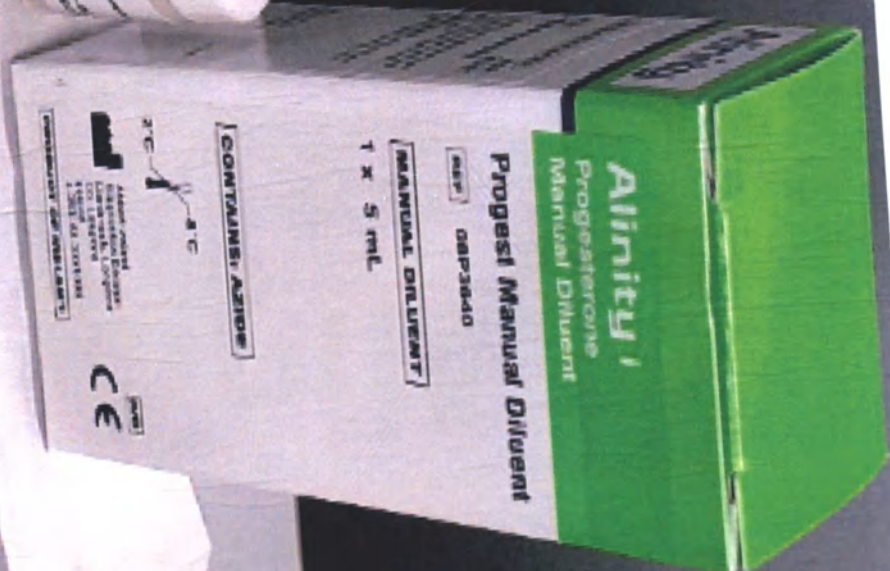
PRODUCT OF IRELAND

Alinity i Progesterone Manual Diluent

Read Instructions Carefully. Reprinted April 2018.

EN
Progesterone
Manual Diluent
REF: DBP3640
Q72862502
DBP360

NAME
Alinity i Progesterone Manual Diluent (also referred to as progesterone manual diluent)

INTENDED USE
The Alinity i Progesterone Manual Diluent is used for manually diluting specimens for testing using the Alinity i Progesterone reagent kit on the Alinity i analyzer. For additional information, refer to the Alinity i Series Operations Manual, package insert and the Alinity i Series Operations Manual.

CONTENTS


DBP3640
1 x 5 mL

STORAGE

Do not use past expiration date		
Storage Temperature	Maximum time storage	Instructions
Unopened 2 to 8°C	Until expiration date	
Opened 2 to 8°C	Until expiration date	Remove tightly capped. Return to refrigeration storage after use.

INDICATIONS OF INSTABILITY OR DEGRADATION
Instability or deterioration should be suspected if there are precipitates, visible signs of leakage or swelling.

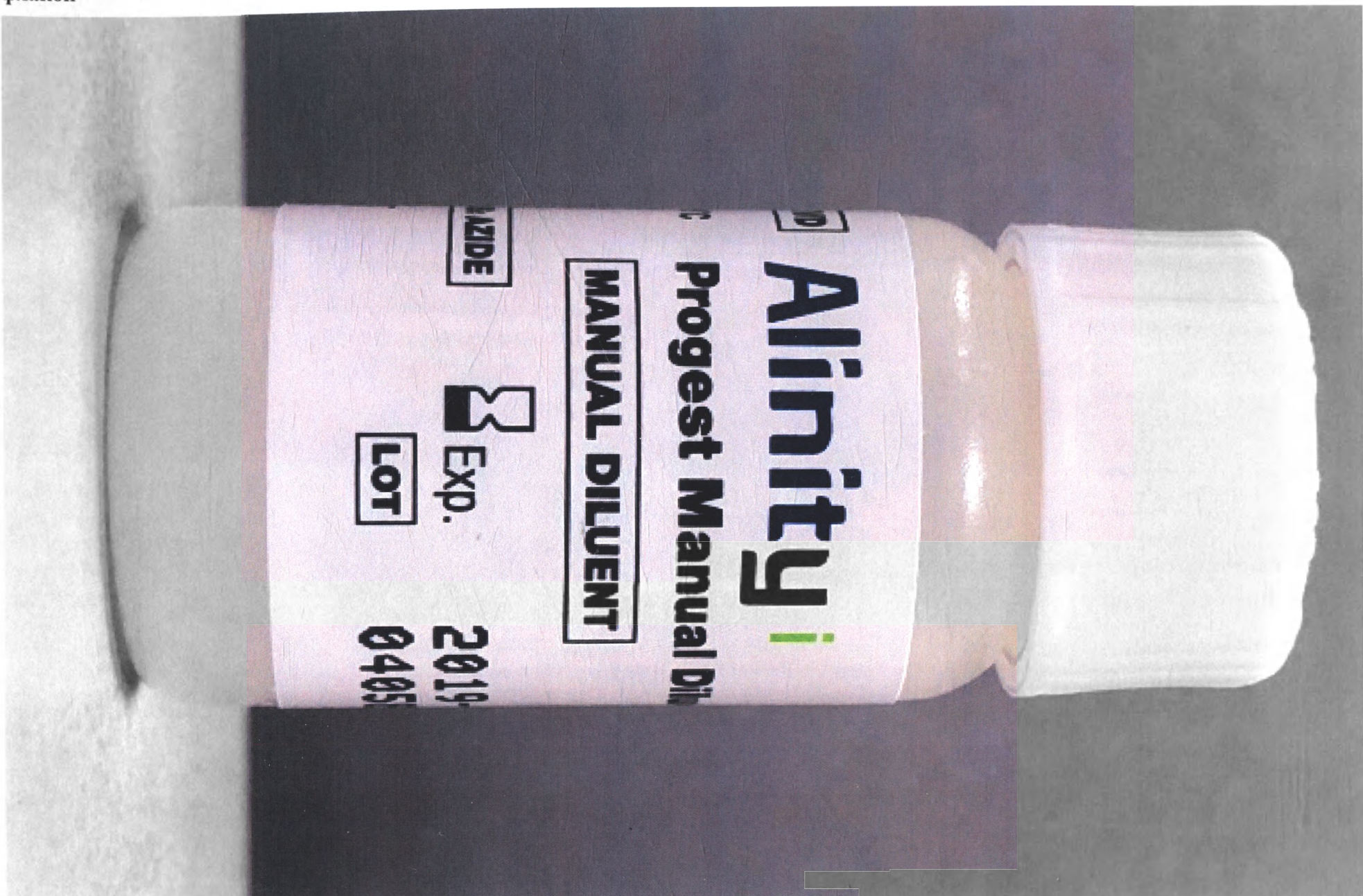SAMPLE DILUTION PROCEDURES
Refer to the Sample Dilution Procedures section of the Alinity i Series Operations Manual.

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