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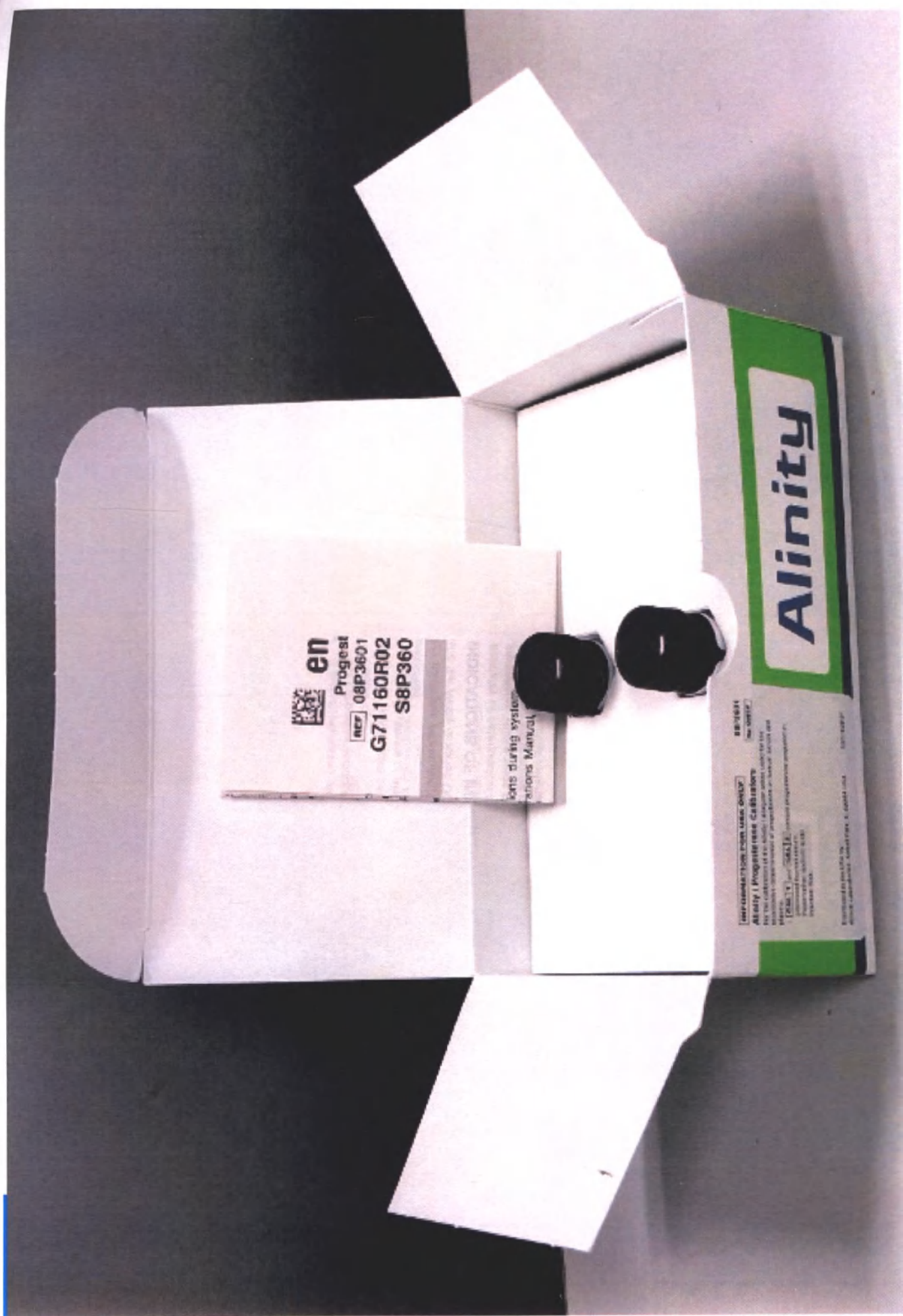
Alinity i Progesterone Calibrators

Read Highlighted Changes: Revised April 2018

NAME

Alinity i Progesterone Calibrators (also referred to as Progest Cals)

INTENDED USE

The Alinity i Progesterone Calibrators are for the calibration of the Alinity i analyzer when used for the quantitative determination of progesterone in human serum and plasma.

For additional information, refer to the Alinity i Progesterone reagent package insert and the Alinity i-series Operations Manual

CONTENTS

The **CAL 1** and **CAL 2** contain progesterone prepared in processed

3D

Alinity i Progesterone Calibrators

Abbott

Progest Cals

CAL 1	1 X 3.0 mL	0.7	2.2
CAL 2	1 X 3.0 mL	40.0	127.2

UNTIL FIRST USE

2°C -10°C

AFTER FIRST USE

ng/mL nmol/L

CONC

LOT 08P3601

EXP 2020-03-25

REF 03521U100

QTY (10) 03521U100 (240) 08P3601 (91) Progest

R02

CONTAINS ALZINE

For a detailed discussion of safety precautions during system operation, refer to the Alinity i-series Operations Manual, Section B.

PREPARATION FOR USE

- Prior to first use, thaw completely at room temperature (15 to 30°C) for 1 to 2 hours, or overnight at 2 to 8°C.
- Prior to use, mix thoroughly by low speed vortex or inversion.

STORAGE

- This product is shipped on dry ice.
- Do not use past expiration date.

Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened -10°C or lower	Until expiration date	This product is stable for 3 weeks after thaw.

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