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NIGHT

7

7. Keep hold of the outer applicator tube. With your index finger, push the smaller tube all the way into the bigger tube (or use your other hand to push in the smaller tube). One tube is now inside the other.
8. Carefully remove the two tubes. The tampon should now be comfortably inside you, with the removal cord hanging outside your body.
9. To dispose of the plastic applicator, you can use the wrapper as a disposal bag. Do not flush it as it is made of plastic.

TO REMOVE YOUR TAMPON

Bend your knees a little with your feet apart and relax. Pull the removal cord forwards at the same angle you used to insert the tampon. The tampon may be flushed down the toilet if there is no litter bin.

ABSORBENCY AND WHEN TO CHANGE

Every woman has her own unique menstrual flow pattern which can vary from day to day, and is often heavier in the first days. For best protection and comfort choose the combination that is right for you. Always use the lowest absorbency to meet your needs. Using a higher absorbency than needed could result in an unpleasant removal experience. Tampax® tampons are available in a range of absorbencies:

Lites: light flow.

Absorbency: <6g. 1 EDANA droplet
Normal/Regular - light to medium flow.

Absorbency: 6-9g. 2 EDANA droplets
Super - medium to heavy flow.

Absorbency: 9-12g. 3 EDANA droplets
Superplus - very heavy flow.

Absorbency: 12-15g. 4 EDANA droplets

A tampon should be changed every 4-8 hours. If you need to change more frequently, try a higher absorbency. If however, you remove the tampon after the recommended time and white fibre is still visible, try a lower absorbency.

Always check that the last tampon has been removed before inserting a new one and make sure to remove the last tampon you use at the end of your period.

Always wash your hands before and after inserting or removing a tampon.

Use tampons during menstruation only - never in anticipation of your period or between periods.

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Do I have to remove a tampon when going to the loo?

No. Tampons are worn in the vagina, which is separate from the urinary opening and the anal opening.

What should you feel when the tampon is inserted? If

the tampon has been inserted correctly, you should not feel it. The removal cord can be hanging freely outside your body, but it should not be felt. If the tampon feels uncomfortable, most likely it has not been inserted deep enough. Remove the tampon and use a new one, making sure to insert it deeper.

Can I use a tampon if I am still a virgin? Yes you can.

The tampon won't affect your virginity at all. Also, the hymen is a flexible membrane with a small opening through which the menstrual blood passes as it leaves the body - it is through this opening that tampons can be inserted and removed.

Can a tampon get lost in my body? No. There's no place for it to go. The opening at the top of the vagina is far too small to let a tampon go through.

Is a Tampax® tampon easy to take out? Yes. The cord on Tampax® tampons is sewn up through the entire length of the tampon. Just grasp the cord and pull gently: the tampon will slide out.

If it is hard to remove, it either means that your flow level required a lower absorbency or that you removed the tampon before it was full. If it slides down easily, it means you chose the right absorbency for your flow level, and that you removed it when it was full.

WHAT YOU SHOULD KNOW ABOUT TOXIC SHOCK SYNDROME (TSS)

Toxic Shock Syndrome (TSS) is a rare but serious illness. It is caused by toxins produced by the bacterium *Staphylococcus aureus* which is commonly found in the nose and vagina. It is a recognisable and treatable illness that occurs in men, women and children.

In the United States, where 65% of women use tampons, the Centre for Disease Control (CDC) report an overall TSS incidence rate of only 0.0005%, of which half occur in menstruating women using tampons. It is more likely to occur in teenage girls and women under 30 than in older women. Even though it is so rare, it's best to know about it so you can take action if you think you have it.

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Early recognition and treatment of the symptoms are important. Therefore, here's a list of symptoms to keep an eye out for: **Sudden high fever (usually 39°C or more), vomiting, diarrhoea, rash that looks like sunburn, dizziness, muscle aches, fainting or near fainting when standing up.** Not all of the signs may be present. Skin peeling may occur in later stages of the illness.

TSS can occur anytime during menstruation or shortly thereafter. It can rapidly progress from flu-like symptoms to a serious illness that can be fatal. **If you have one or more of these symptoms, you must immediately remove the tampon and see a doctor.** Tell your doctor you have been using a tampon and are concerned about TSS.

You should talk to your doctor before using tampons if you have had symptoms of TSS in the past. You can reduce the risk of menstrual TSS by using the tampon with the minimum absorbency required to meet your needs and changing it regularly, every 4-8 hours, and by using a pad instead of a tampon at least once for several hours every day during your menstrual period. You can also essentially eliminate the risk of menstrual TSS by not using tampons

AN INVITATION:

If you would like any further information, please write or call us on our consumer care line. Or, if you are not entirely happy with the product, return it together with the portion of the box embossed with a code number to:

Insert Consumer Contact Address and phone number here

If no phone number available:

If you would like any further information, please write us. Or, if you are not entirely happy with the product, call us or return it together with the portion of the box embossed with a code number to:

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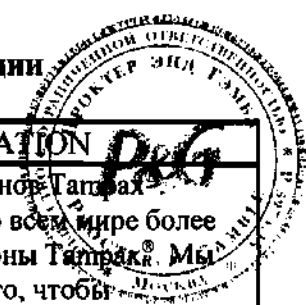
Тел. (8-800) 200-20-20. www.procterandgamble.ru

ENGLISH MASTER COPY

Thank you for buying Tampax® tampons. Tampons have been successfully used by more than 100 million women around the world for over 60 years. With this leaflet we want to make using a tampon as easy as possible for you.

HOW A TAMPAX® TAMPON WORKS

A Tampax Compak tampon is made of an absorbent material (rayon). It is worn inside your vagina and expands gently to fit the shape of your body comfortably. As the flow is absorbed before it leaves the body, there should be no menstrual odor.

The applicator, made from smooth plastic, helps you to insert the tampon easily and hygienically. It is discreet to carry due to its shorter applicator. The wrapper can be used for easy disposal of the plastic applicator.

HOW TO INSERT YOUR TAMPAX® TAMPON

Before you start: Relax and take your time. If you are tense, your muscles will tighten up. Read the instructions first and then put the leaflet down where you can see it so that you have both hands free. Wash your hands. Next find a position that's most comfortable for you: sit on the toilet seat with knees wide apart, or squat slightly with knees bent and far apart, or stand with one foot on the toilet seat.

1. Remove the applicator from the wrapper.
2. The plastic applicator is made of two tubes, one fitting inside the other.
3. You need to extend the applicator to its full size prior to insertion of the tampon.
4. Pull the inner tube out of the outer tube till it clicks into place. You are now ready to insert the tampon.
5. Find the grooved rings at the bottom of the outer tube. Use your thumb and middle finger to hold the applicator firmly between these rings. The removal cord should be hanging down from the inner tube. Use your other hand to fold back the skin that covers the opening of your vagina. Place the rounded tip of the tampon to your vaginal opening. Point the tip towards your lower back.
6. Remember your vagina slopes towards the bottom of your back, not straight up. Take a few breaths and then gently push the applicator in the vagina. As soon as your fingers touch your body, the applicator is in the right place. If the applicator doesn't slide in easily, gently rotate it or move it to the right or left as you push.

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Tampons are designed to absorb menstrual flow only. Do not use a tampon in anticipation of your period or to absorb non-menstrual vaginal discharge.

HOW TO INSERT YOUR TAMPAX TAMPON?

When to try it for the first time?

- Preferably during a heavy flow day, as the applicator slides most easily inside the vagina.
- Preferably during a moment when you have time and are relaxed. Being relaxed is important to have a comfortable insertion because your muscles around the vagina are also relaxed.

What are the best positions for insertion?

Sitting on the toilets, standing with knees slightly bent or standing with one leg up.

What are the steps for insertion?

1. After washing you hands remove the tampon from the wrapper. Hold with your thumb and middle finger the applicator firmly on the grooved rings (b).
2. With the removal cord (a) hanging down, insert the closed rounded tip of the outer insertion tube into the vaginal opening pointing slight upward angle towards you lower back.
3. Then gently push the outer tube (c) in the vagina until your fingers touch your body. If the applicator doesn't slide in easily, gently rotate or move to the right or left as you push.
4. Keeping hold of the grooved rings, push the inner tube (d) with your pointer finger completely into the outer tube and the tampon (e) will slide into the correct position in the vagina.

How to dispose of the applicator?

Applicator is made of paper so they can be flushed away if there is no litter bin.

What should you feel when the tampon is inserted?

If the tampon has been inserted correctly you should not feel it. The removal cord will be hanging freely outside you body but it should not be felt. If the tampon feels uncomfortable, most likely it has not been inserted deep enough. Remove the tampon and use a new one, making sure to insert it deeper.

WHEN SHOULD YOU CHANGE YOUR TAMPAX TAMPON?

Remember to change the tampon regularly, every 4-8 hours. During the heavy flow days, you might have to change it every 4-6 hours, while during the light flow days, changing it every 6-8 hours is usually enough.

Always take the tampon out before inserting a new one.

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Remember always to remove the last tampon you used at the end of your period.

HOW TO REMOVE YOUR TAMPAX TAMPON?

Pull the cord gently downwards at the same angle as you used to insert it. If it is hard to remove, it either means that your flow level required a lower absorbency or that you removed the tampon before it was full. If it slides down easily, it means you chose the right absorbency for your flow level, and that you removed it when it was full.

HOW TO DISPOSE YOUR TAMPAX TAMPON?

Tampon is flushable and biodegradable so it can be flushed away if there is not litter bin.

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Toxic Shock Syndrome (TSS) is a rare but serious illness. It is caused by toxins produced by the bacterium *Staphylococcus aureus*, which is commonly found in the nose and vagina. TSS can occur in men, women and children. Approximately one-half of the cases occur in menstruating women and girls. Menstrual TSS is associated with tampon use. It is more likely to occur in teenage girls and women under 30 than in older women. Early recognition and treatment of the symptoms are important.

Symptoms:

TSS can occur anytime during menstruation or shortly thereafter. It can rapidly progress from flu-like symptoms to a serious illness that can be fatal. The warning signs of TSS are listed below, but all of them may not be present:

- * High fever
- * Vomiting
- * Diarrhea
- * Rash that looks like sunburn
- * Dizziness
- * Muscle aches
- * Fainting or near fainting when standing up

Skin peeling may occur in later stages of the illness

If you have one more of these symptoms, you must immediately:

1. Remove Tampon
2. Seek medical attention
3. Tell your doctor if you have been using a tampon

and are concerned about TSS

If you had TSS once, you can get it again. You should talk to your doctor before using tampons if you have had symptoms of TSS in the past.

Ways to Reduce the risk of TSS:

You can reduce the risk of menstrual TSS by using a pad

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instead of a tampon at least once each day during your menstrual period and by using the tampon with the minimum absorbency required to meet your needs. You can also essentially eliminate the risk of menstrual TSS by not using tampons.

AN INVITATION:

If you would like any further information please visit our site **www. Tampax.com** where you can find even more info/answers to questions more asked by beginners.

Consumer service address : Local consumer service address or contact tel number.

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