State of Minnesota - Secretary of State

This Certificate is not valid for use anywhere within the United States of America, its territories or possessions.

Apostille

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1. **Country (Pays): United States of America**

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2. **has been signed by (a été signé par) Ryan J. Bartz**

3. **acting in the capacity of (agissant en qualité de) Notary Public, State of Minnesota**

4. **bears the seal / stamp of (est revêtu du sceau / timbre de)
Ryan J. Bartz, Notary Public, State of Minnesota**

Certified Attesté

5. **at (à) St. Paul, Minnesota**

6. **Date (Date) 11/26/2019**

7. **by (par) Secretary of State, State of Minnesota**

8. **File No (Sous n°) 1120679200063**

9. **Seal / Stamp (Sceau / Timbre):**

10. **Signature (Signature):**

Steve Simon

Steve Simon

Secretary of State
State of Minnesota

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears.

This Apostille does not certify the content of the document for which it was issued.

To verify the issuance of this Apostille, see <https://apostille.sos.state.mn.us>.

This certificate does not constitute an Apostille under the Hague Convention of 5 October 1961, when it is presented in a country which is not a party to the convention. In such cases, the certificate should be presented to the consular section of the mission representing that country.

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November 25, 2019

To whom it may concern

In my capacity as Senior Regulatory Affairs Associate, 3M ESPE Dental Products,
2510 Conway Avenue, St. Paul, MN, 55144-1000, USA, I certify that attached IFU for:

3M ESPE RelyX™ Veneer Cement

in English language is truthful and accurate

Angie Draper

Angie Draper
Sr. Regulatory Affairs Associate
3M ESPE Dental Products


25 Nov 2019

Date

State of Minnesota)
County of Ramsey) SS

Subscribed and sworn to before me this 25th day of November 2019.

Ryan Bartz
Ryan Bartz, Notary Public
My Commission Expires: January 31, 2024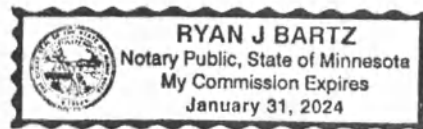

Instructions for Use

3M ESPE RelyX Veneer Cement

General Information

3M ESPE RelyX™ Veneer Cement is a radiopaque, color stable, light-cured resin cement indicated for bonding veneers fabricated of porcelain or composite. It is used in conjunction with 3M ESPE RelyX™ Try-In Pastes, Adper Single Bond 2 Adhesive, Single Bond Universal Adhesive. An additional silane primer is not required when using Single Bond Universal Adhesive. The RelyX Veneer cement contains a resin system consisting of a TEGDMA/ BisGMA blend and a zirconia/silica filler with a particle size range of 0.2–3.0 microns and volume percentage of approximately 47%.

RelyX™ Veneer cement is designed to be used in conjunction with Adper Single Bond 2 Adhesive, Adper Scotchbond Multi-Purpose Adhesive, or Single Bond Universal Adhesive and RelyX™ Try-In pastes. It is recommended that RelyX Veneer cement be used only with 3M ESPE adhesive systems where veneer bonding is indicated. Please read the instructions for use of the 3M ESPE adhesive system you use for additional safety and dispensing information.

Note: 3M ESPE disclaims liability for claims arising from the use of RelyX™ Veneer cement with any adhesive other than Adper Single Bond 2 Adhesive, Adper Scotchbond Multi-Purpose Adhesive, and Single Bond Universal Adhesive.

Relyx Veneer can be supplied as kit and as refills:

RelyX Veneer cement kit includes: 1) Syringes of RelyX Veneer cement, 5 pcs (A1, A3, WO, TR, B0,5); Syringes of Relyx Try-in paste, 5 pcs (A1, A3, WO, TR, B0,5), Bottle of Single Bond Universal Adhesive, 1 pcs; Syringes of Scotchbond Universal Etchant, 1 pcs; Etchant dispensing tips, 25 pcs; Microbrushes, 50 pcs; Mixing wells, 48 pcs.

Relyx Veneer cement in refills: 1) 1 Syringe of Relyx Veneer cement (A1 shade); 2) 1 Syringe of Relyx Veneer cement (A3 shade), 3) 1 Syringe of Relyx Veneer cement (B0,5 shade); 4) 1 Syringe of Relyx Veneer cement (TR shade); 5) 1 Syringe of Relyx Veneer cement (WO shade).

Indications

Permanent cementation of ceramic or composite veneers.

Contraindications


There are no known contraindications.

Potential Adverse Events (Potential Complications)

Allergic reactions in patients with known acrylate allergies.

Based on current complaint history 3M ESPE has been notified of no reportable adverse event for this product.

Scope of use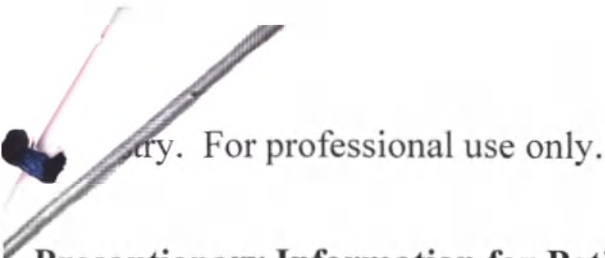

try. For professional use only.

Precautionary Information for Patients

These products contain substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of these products in patients with known acrylate allergies. If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of these products.

Precautionary Information for Dental Personnel

These products contain substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the product contacts the glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

Please refer to the instructions for use of the 3M ESPE adhesive system you use for additional precautionary information.


Hazardous Information

ETCHANT PRECAUTIONS: Contact with eyes or skin may cause burns. In case of contact, flush area with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection.

Adper Single Bond 2 Adhesive, RelyX Ceramic Primer and Single Bond Universal Adhesive are highly flammable. Avoid static discharge. Keep away from heat, sparks, open flame, pilot light, and other sources of ignition.

Preparation

1. To prepare the veneer, the internal bonding surface of a ceramic veneer surface should be micro-etched with a hydrofluoric acid etchant. The dental laboratory typically does this. The internal surface of a composite veneer should be roughened by sandblasting or air abrasion.
2. Remove the provisional restoration, if present, from the tooth. Clean tooth with pumice; rinse thoroughly and lightly air dry.
3. Check the fit and esthetics of the restoration by trying in the veneer with water or the Translucent shade of RelyX Try-In paste. If the fit and esthetics are acceptable, proceed directly to step 5 using the Translucent shade of RelyX Veneer cement.
4. If color adjustment is needed, select the appropriate shade of RelyX Try-In paste. Apply RelyX Try-In paste directly onto the internal surface of the veneer. Gently seat the veneer onto the tooth preparation. With multiple veneers, start at the midline and seat veneers sequentially working back. The shade of RelyX Try-In paste is formulated to precisely match the corresponding cured shade of the cement.

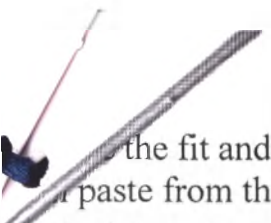- 
- If the fit and esthetics are verified, remove the veneers and thoroughly rinse the RelyX Try-It paste from the tooth and restoration with a water spray and then dry.
6. The internal surface of the veneer should be thoroughly cleaned prior to applying the silane or adhesive to remove any contaminants. The surface can be cleaned by applying etchant (34%-35% phosphoric acid) for 15 seconds followed by rinsing and drying.
- 6.1. For Adper Single Bond 2 Adhesive and Scotchbond Multipurpose Plus Adhesive: Dispense RelyX Ceramic Primer (silane) into a mixing well. Brush a layer of the silane onto the internal surface of the veneer. Lightly air-dry the surface for 5 seconds to evaporate the solvent.
- 6.2. For Single Bond Universal Adhesive: Use the disposable applicator to apply Single Bond Universal to the etched veneer. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.**
7. Clean the prepared teeth using flour of pumice slurry. Rinse and lightly dry. Isolate the area to prevent contamination. Place matrix strips between teeth to prevent unwanted bonding to adjacent teeth.

Directions

1. Etch the tooth preparation. Apply etchant to both dentin and enamel for 15 seconds. Rinse for 10 seconds. Blot excess water leaving tooth moist. If the tooth is accidentally dried, rewet the bonding surface with water for 15 seconds and blot excess leaving tooth moist. Alternatively, a HEMA based desensitizing agent can be applied to the etched dentin according to the manufacturer's instructions.
2. Apply the adhesive.
 - 2.1. For Adper Single Bond 2 Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. **Do not light cure.** Apply 1 coat of Adper Single Bond 2 Adhesive to the silane treated veneer. Dry gently for 5 seconds. **Do not light cure.**
 - 2.2. For Adper Scotchbond Multi-Purpose adhesive: Apply Adper Scotchbond Multi-Purpose Primer to etched enamel and dentin. Dry gently for 5 seconds. Apply Adper Scotchbond Multi-Purpose Adhesive to the primed enamel and dentin. **Do not light cure.** Apply 1 coat Adper Scotchbond Multi-Purpose Adhesive to the silane treated veneer. **Do not light cure.**
3. For Single Bond Universal adhesive: Use the disposable applicator to apply Single Bond Universal immediately to the entire surface of the enamel and dentin and rub it in for 20 sec. Avoid contact between the adhesive and the oral mucosa. If necessary, rewet the disposable applicator during treatment. Subsequently direct a gentle stream of air over the liquid for about 5 sec until it no longer moves and the solvent has evaporated completely. **Do not light-cure.** Apply a thin layer of the selected shade of RelyX Veneer cement directly from the syringe onto the bonding surface of the veneer. **Protect the cement from direct operatory light exposure.**
4. Seat the veneer in place using gentle pressure. Excess cement will extrude out around the margins. Spot cure the veneer into place on the facial surface away from the margins using a

- ... diameter light guide for 20 seconds to secure it in place.
- Remove the excess cement from the margins using a blunt instrument or dry brush. Alternatively, the excess cement can be “tack-cured” for 3 seconds with a standard halogen light to allow the excess to gel and allow for easier cleanup.
6. Light cure the labial, lingual, interproximal and occlusal surfaces for 30 seconds each. The A5 Opaque/Dark shade will require a 40 second exposure for each surface. Curing recommendations will allow for cement cure depths not less than 0.5 mm for the opaque shades and not less than 1.5 mm for the nonopaque shades. **Note: The cure times are indicated for a standard halogen curing light with a minimum light output of 400 mW/cm².**
7. Remove matrix strips. Finish marginal areas with Sof-Lex Finishing and Polishing Strips and Discs, manufactured by 3M ESPE. Check and adjust occlusion as necessary.

Storage and use

This product is designed to be used at room temperature. The humidity and pressure are not regulated. If stored in cooler allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for expiration date.

Please refer to the instructions for use of the 3M ESPE adhesive system you use for specific storage and use information. Do not expose materials to elevated temperatures or intense light.

Do not store materials in proximity to eugenol-containing products.

Technical characteristics:

Relyx Veneer cement syringe: Length (mm): 117,5±11 (filled full assembly), Width (mm): 31,75 ± 3; Height (mm): 12,7 ± 1,6; Diameter (mm): 8,9 ± 1; Mass (g): 7,7 ± 5% (filled assembly).

Relyx Veneer cement cap: Length (mm): 12,7 ± 1,6, Diameter (mm): 4,8 ± 1; Mass (g): 0,24 ± 0,1 (cap only).

Relyx Veneer Try-in paste syringe: Length (mm): 117,5±11 (filled full assembly), Width (mm): 31,75 ± 3; Height (mm): 12,7 ± 1,6; Diameter (mm): 8,9 ± 1; Mass (g): 6,8 ± 5% (filled assembly).

Relyx Try-in Paste cap: Length (mm): 12,7 ± 1,6, Diameter (mm): 4,8 ± 1; Mass (g): 0,24 ± 0,1 (cap only).

Scotchbond Universal Etchant syringe: Overall Length (mm): 77,4±10%; Cylinder length (mm): 63,65 ± 0,98; Diameter (mm): 7,7 ± 5%; Mass (g): 8,0 ± 1.

Single Bond Universal Adhesive bottle: Length (mm): 59 ± 1, Diameter (mm): 21 ± 1; Mass (g): 8,0 ± 1.

Etchant dispensing tips: Length (mm): 30,0±1,2; Diameter (mm): 6,90±0,30; Mass (g): 0,20±0,02.

Microbrushes: Length (mm): 84±1; Diameter (mm): 2,0±0,5; Mass (g): 0,30±0,05.

Mixing wells: Length (mm): 46,05±5%; Width (mm): 28,58±5%; Height (mm): 9,52±5%; Thickness (mm): 1,27±5%.

List of standards: ISO 10993-1: 2003(E), ISO 10993-3: 2003, ISO 10993- 5: 1999, ISO 10993-6: 2007, ISO 10993-10: 2002, ISO 10993-11: 2006, ISO 7405: 1997, ISO 13485: 2003 +Cor, 1:2009, ISO 14971: 2000, ISO 4049: 2009, ISO 15223-1: 2012, EN 980: 2008, EN 1041: 2008/ Amd 1:2013

Labelling information

Products are supplied in packaging labeled with the following information: Product name and REF number; name and address of the manufacturer; Do not reuse; European Authorized Representative sign; Contents; LOT number; Storage conditions; Shelf life; Information about trademarks and symbols - Caution, See accompanying document, Symbol CE; Flammable; Corrosive substance; Ecological packaging, Recycling sign.

Cleaning and Disinfection

Disinfect the products using an intermediate level disinfection process (liquid contact) as recommended

Medicinal substances: Medical devices do not contain or deliver medicinal substances.

Human Blood Derivatives: Medical devices do not contain or deliver human blood derivatives.

Animal-Derived Substances: No materials of human origin are intentionally being added to the products.

Disposal methods

Dispose of content/container in accordance with the local/regional/national/international/ regulations.

Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B.

Maintenance and repair

This medical device isn't subject to maintenance and repair.

Manufacturer of the medical device

– *Developer*

3M ESPE Dental Products

Address: 2510 Conway Avenue, St, Paul, Minnesota, 55144-1000, USA

651-7334365

– Manufacturer

3M ESPE Dental Products

Address: 2510 Conway Avenue, St. Paul, Minnesota, 55144-1000, USA

Phone: 651-7334365

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information valid as of November 2019

Customer Service

Authorized representative of the manufacturer in the Russian Federation:

AO "3M Russia", 108811, Moscow, Poselenie Moskovskiy, Kievskoe shosse, 22 km, 6, bld.1,
Russia /phone +7 (495) 578-74-74

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