● ○ ○ ○ ●
IDALINA AMADOR
Notaria

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This Apostille only certifies the signature, the capacity of the signer and the seal/stamp it bears. It does not certify the content of the document for which it was issued.

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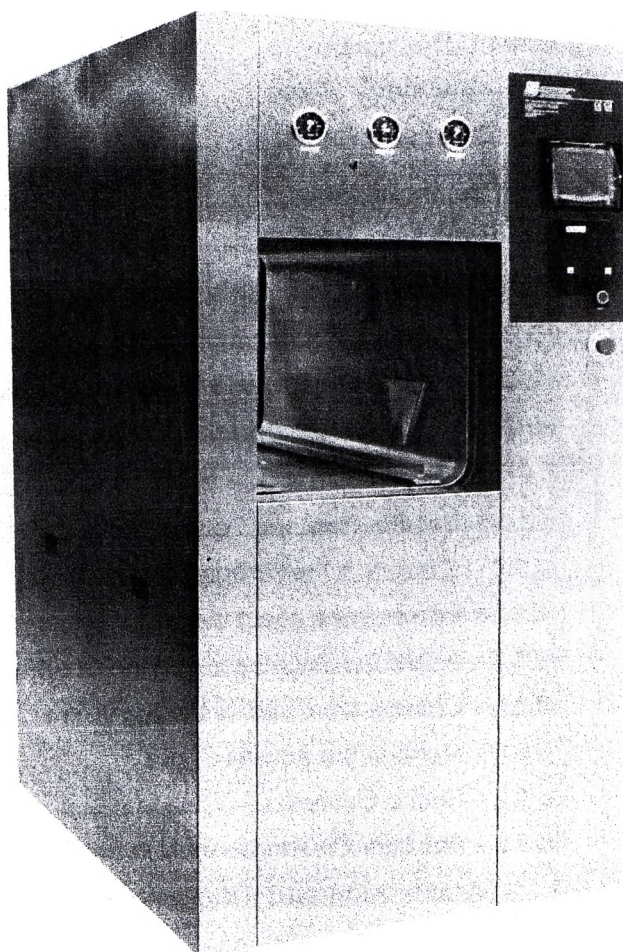
CE
0120

1 – Index

1 – Index	1
2 – Introduction	2
2.1 – Generic Description	2
2.2 – Manufacturer	2
2.3 – Technical Characteristics	2
2.4 – Symbol Description	2
3 – Accessories	3
4 – Installation Requirements	4
4.1 – Installation of the Power Supply	4
4.2 – Installation of Cold Water	4
5 – Equipment Description	5
5.1 – Equipment Views	5
5.2 – Command Panel	5
6 – Functioning Instructions	6
6.1 – Sterilizer's Start	6
6.2 – Navigate in the Programs Menu	6
6.2.1 – Execute a Standard Program	6
6.2.2 – Execute the Bowie & Dick Test, the Air Leakage Test or Cleaning the Sterilizer	14
6.2.3 – Execute a Configurable Program	15
6.3 – The Sterilization's Cycle	16
6.3.1 – Set the Time and Date	16
6.3.2 – Execute a Test Program	17
6.3.3 – Informations about the Manufacturer	18
6.3.4 – Adjust the Brightness and the Contrast of the Display	19
6.3.5 – Change the Client or the Sterilizer	19
6.3.6 – Manage the Access Codes	19
6.3.7 – Sound Options	19
6.3.8 – Sterilizer Cleanness (only in two doors sterilizers)	21
6.4 – Available Standard Programs	21
6.5 – Programs Description	22
6.5.1 – Program 121°C Rubbers / 134°C Textiles / Instruments / Containers / Prions	23
6.5.2 – Program 134°C Fast	23
6.5.3 – Air Leakage Test Program	24
	25

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Fax: +351 229 015 900

prohs@prohs.pt | www.prohs.pt

6.6 – Preparing the Instruments to Sterilize	26
6.6.1 – Unwrapped Materials.....	26
6.6.2 – Wrapped Materials	27
6.7 – Rules for Opening the Sterile Zone Door.....	28
6.8 – Rules for Placing the Cargo in the sterilizer.....	29
7 – Errors	31
8 – Maintenance	32
8.1 – Usual Maintenance	32
8.1.1 – Cleaning the External Surfaces.....	32
8.1.2 – Cleaning the Chamber's Interior, Load Structure.....	33
8.1.3 – Cleaning the Door Gaskets	34
8.2 – Preventive Maintenance.....	35
9 – Factory Predefined Access Codes	35
10 – Printer	36
11 – Air Compressor	36
11.1 – Main Components.....	36
11.2 – Instructions for Use	37
11.3 – Periodic Maintenance.....	37
11.3.1 – Oil Change.....	38
11.3.2 – The Compressor does not Start	38
11.3.3 – The Compressor Works, but doesn't Reached the Maximum Pressure	38
11.3.4 – The Compressor Works, but the Pressure doesn't Increase (or increases very slowly)	38
11.3.5 – The Compressor Works, but doesn't Fill	38
11.3.6 – During the Operation the Compressor Stops	38
11.3.7 – Air Leakage	38
11.3.8 – Leakage in the Valve below the Pressure Switch	38
11.3.9 – The Compressor Starts when there isn't use of Air.....	39
11.3.10 – The Compressor Starts and Stops more Frequently than Usual	39
11.3.11 – The Compressor doesn't Start when the Pressure is below the Minimum and / or do not Stop when Reach the Maximum Pressure	39
11.3.12 – The Compressor is Very Hot and Consumes too much Oil.	39

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

2 – Introduction

The company **PROHS** thanks the confidence deposited in us, by acquiring our products hopes that they completely fulfill your needs.

The **JSM** equipments are manufactured according the effective Security Norms and Quality Control, thus guaranteeing the Process and Use, as long as the instructions of this manual are used and respected.

2.1 – Generic Description

The **JSM** Horizontal Steam Sterilizers have as basis of its function the control of the temperature and pressure seeking the complete elimination of all living microorganisms. Through fast and uniform heat transference, the **JSM** Horizontal Steam Sterilizers are an indispensable tool in places where high levels of sterilization are required.

2.2 – Manufacturer

The **JSM** Horizontal Steam Sterilizers brand is manufactured by:

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3 - Technical Characteristics

Brand	JSM										
Model	70L	100L	110L	145L	175L	250L	340L	360L	490L	610L	740L
Number of doors	One or Two (1/2 PD)										
Type of sterilization	Saturated Steam										
Steam Generator	With or without steam generator built in										
Programs	Heating Program 134°C Normal (Textiles) 121°C (Rubbers) 134°C Fast 134°C Instruments 134°C Containers 134°C Prions Bowie Dick test Air Leakage Test										
Types of Drying	Possibility to elaborate up to a hundred new programs*										
Capacity (litres)	High Vacuum Drying High Vacuum and air pulses Drying High Vacuum and steam pulses Drying										
Usable Capacity (U.E. ^{3h})	70	100	110	145	175	250	340	360	490	610	740
Useful dimensions (cm)	32x32x70	32x32x100	40x40x70	40x40x90	50x50x70	50x50x100	70x70x70	60x60x100	70x70x100	70x70x125	70*70*150
Exterior dimensions (height x width x depth) (cm)	170x77x92	170x77x118	180x84x96	180x84x112	180x94x96	180x94x127	1195x116x96	180x106x127	195x116x127	195x116x152	195x116x177
Water Consumption (litres)	110	115	120	120	130	145	170	170	200	225	
Power	Without Steam Generator	1,3kW – 3x3A			2,1kW 3x4A		2,1kW – 3x4A		2,75kW – 3x5A		
	With Steam Generator	24kW – 35A			26kW 35A		30kW – 45A		45kW – 65A		
Power Supply (V/Hz)	3~ 400V / 50Hz										
Water Supply (inches)	3/4"										
Sewer Resistant to the Temperature	2"										
Approximate Weight (kg)	260	370	435	500	570	685	750	800	830	1000	1200
Sterilization Temperature (°C)	121°C e 134°C										
Sterilization Time (min.)	134°C Normal (Textiles) – 4m 121°C (Rubbers) – 16m 134°C Fast – 3m 134°C Instruments – 4m 134°C Containers – 4m 134°C Prions – 18m Bowie Dick Test – 3m30s										
Certification Standards	CE Certificate (Directive 93/42 CE, NP EN 285) CE Certificate (Directive 97/23 CE, BS 3970, PD 5500)										

PROHS - Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

2.4 – Symbol Description

This sign indicates the presence besides of the manufacturer name and address.

This sign indicates the need of caution. Read accurately all the notes beside this sign.


3 – Accessories

The JSM Horizontal Steam Sterilizers comes with a number of accessories, presented next:


PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
Rua do Castanhal n° 316 Zona Industrial Maia I Sector II
Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
Telf.: +351 229 059 170 Fax: +351 229 015 900
prohs@prohs.pt | www.prohs.pt

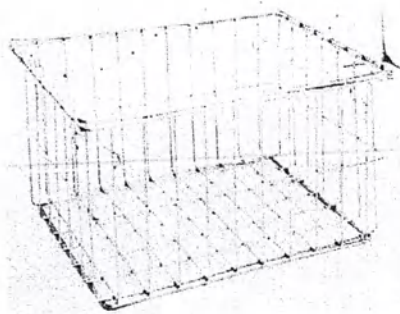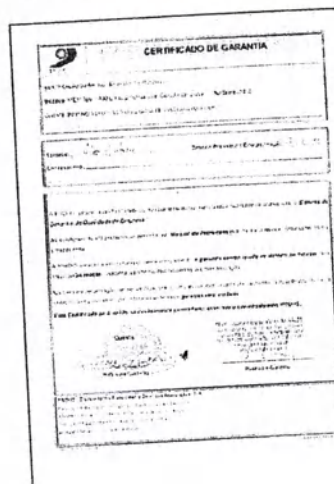1. Stainless steel baskets of different sizes depending the Sterilizer's model;
2. Load car and platform;
3. Instructions Manual;
4. Warranty certificate.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

4 – Installation Requirements

The Horizontal Sterilizer **JSM**, during its manufacturing, has gone through several tests before being forwarded.

For a correct installation the following aspects must be taken in to attention:

- Must be placed in a plain and levelled surface;
- Laterally, a free space of approximately 40 cm must be kept. This is an essential space for a correct access for maintenance;
- The installation place must be well ventilated (4 to 6 renewals / hour);
- It must not be installed in places where the possibility to release gases with explosive characteristics to the environment exists. Care must be taken in handling liquids that might damage the exterior plating.

4.1 – Installation of the Power Supply

Once placed in a proper place, following the previous conditions, the installation of the power supply, must fulfill the following requirements:

- The tension, power and frequency characteristics, described in the technical characteristics list, for the selected model must be proved;
- The sterilizer must be connected to a net protected by an earth connection, as the applicable norms.

4.2 – Installation of Cold Water

Fulfill the following requirements to feed and exit the water:

- General cold water supply (DN ½", 4 to 6 MPa and hardness around 7 °F);
- Sewer in heat resistant material, preferably alloy (DN50);

Notes:

- For any additional clarifications, please consult us.
- PROHS** is not responsible for any damages caused to people or things, by not following the procedures described in the previous points.

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
5 – Equipment Description

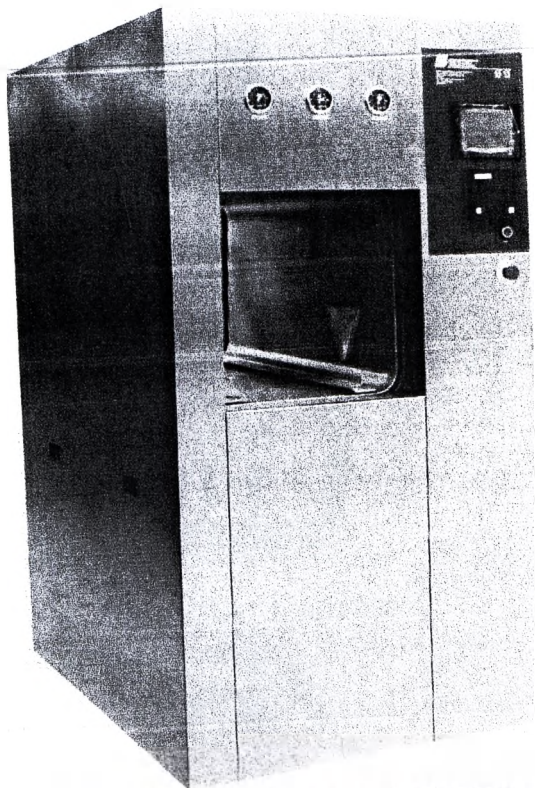
5.1 – Equipment Views

In the Non-Sterile zone (entrance of the material to sterilize, eventually, also exit of the material, when the Sterilizer has only one door), the Horizontal Sterilizer has a command panel constituted of:

“Touch Screen” Display, Printer, On/Off Button and Pressure Indicators (Gauges)

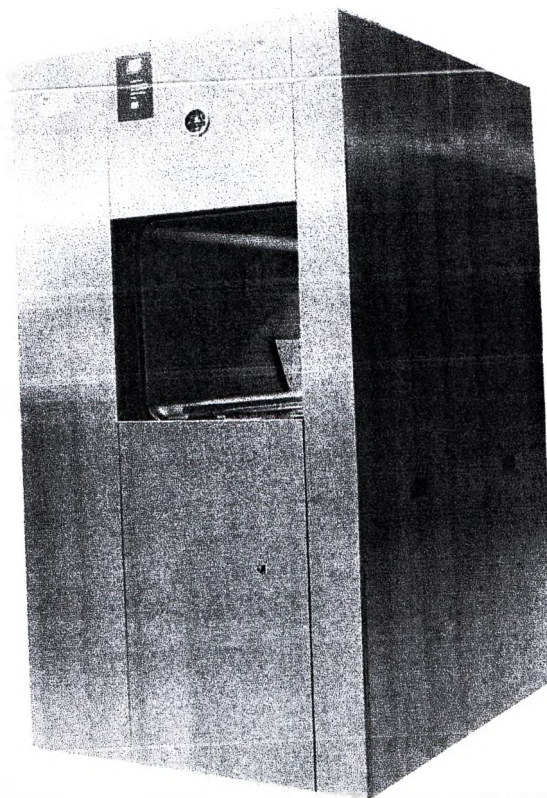The existence of a command panel, it's essentially for a correct monitoring of the sterilization cycles.

In the Sterile zone, (Exit of the Sterile Material from the Horizontal Sterilizer, when it has two doors), the Horizontal Sterilizer has a command panel and a gauge.

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

prohs@prohs.pt | www.prohs.pt

5.2 – Command Panel

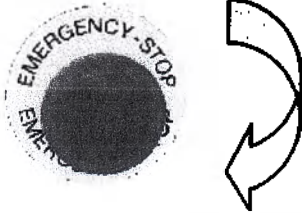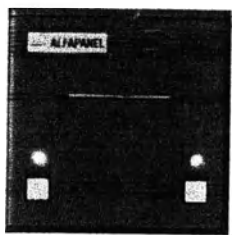It's constituted by the following components:

 5" "Touch Screen" Display – Allows the visualization of all stages of the Sterilization Process, being all operative commands automatically done in the Display through touch.	 Power switch, to turn the Sterilizer On or Off.
 Conventional gauges to an easy visualization of the work pressure in the chamber, jacket and steam generator.	 Emergency Stop – Allows the immediate stop of the Sterilizer. To turn the Sterilizer back on, turn the switch in the direction indicated by the arrow (Clockwise)
 In a two door sterilizer, there's a panel that allows the correct monitoring of the sterilization cycle.	 Printer – Fast printer with graphical register and conclusive results in real time.

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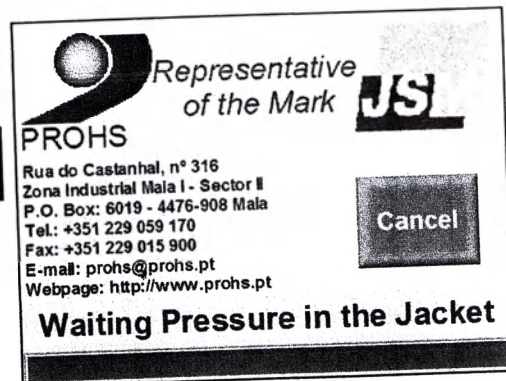
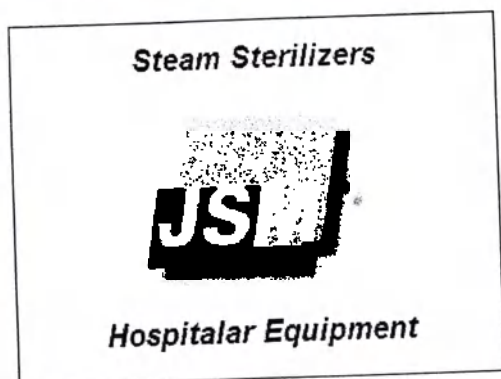
6 – Functioning Instructions

6.1 – Sterilizer's Start

- Turn the Sterilizer On in the button **INT. GERAL**



- Wait for the welcome screen to be presented in the "touch screen" display.

- In Sterilizers with a steam generator, wait for it to reach a pressure of approximately 4 Bar (See the Steam Generator Pressure Gauge).

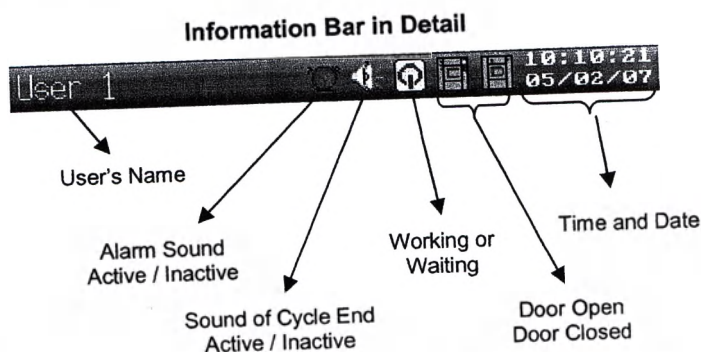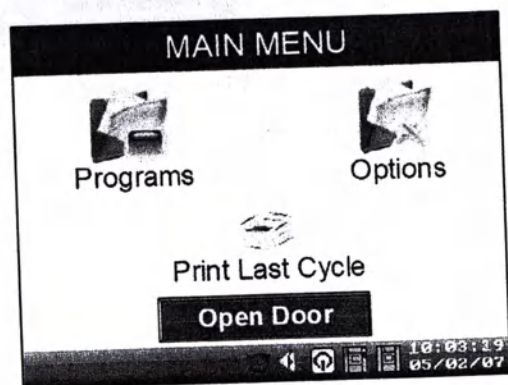- When this window shows in the "touch screen" display the sterilizer is ready to execute a cycle. You have three alternatives, access to the Programs in **Programs** (6.2), access to the options in **Options** (6.4), or print a copy of the last cycle done, pressing the **Print Last Cycle** button. You can also open the Non-Sterile's Zone Door.

- There are 2 access levels to the Sterilizer's Menu, The Responsible level (higher) and the User level (Lower).
- To access the menu **Programs** (6.2), you need to enter an access code, User or Responsible. To access the **Options** (6.4) menu, you need an access code of Responsible level.


PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal n° 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt


6.2 – Navigate in the Programs Menu

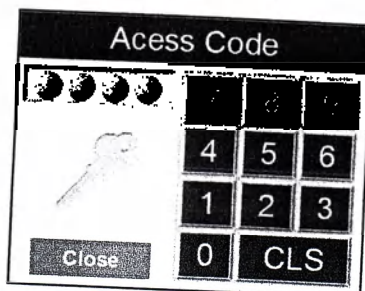- To execute a sterilization cycle, press the button **Programs** in the **Main Menu**, you will be immediately asked to insert an access code, it can be an **User** or **Responsible** code.
- Insert the code, in case the code is not accepted you will remain in the **Main Menu**. If the code is accepted it enters in the **Programs** menu and in the left inferior corner, the name associated with the code entered will appear. In case you make a mistake entering the access code press the button Clean to clear the inserted digits.

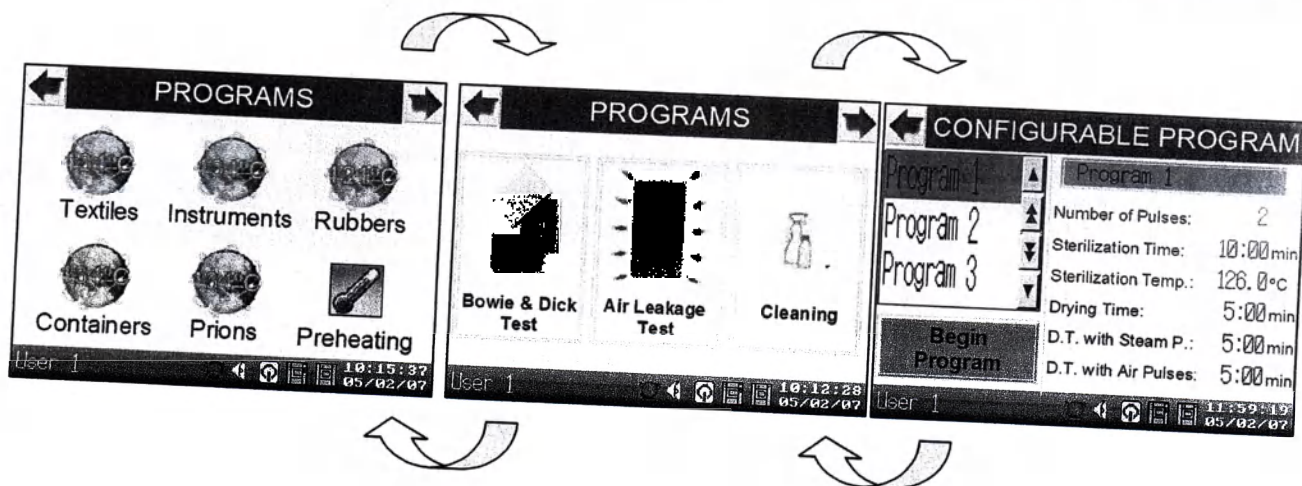- After the access code acceptance, you're presented with the standard programs menu, there are six programs and they're described in the point 6.6 of this manual. If you press the arrow in the upper right corner, you enter in the menu of Bowie & Dick test, Air Leakage test and Cleaning. Pressing again the upper right arrow takes you to the Configurable Programs menu.

- You can freely navigate between these three Program menus, for that use the upper arrows, the left to go back and the right to go forward.

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
prohs@prohs.pt | www.prohs.pt






6.2.1 – Execute a Standard Program

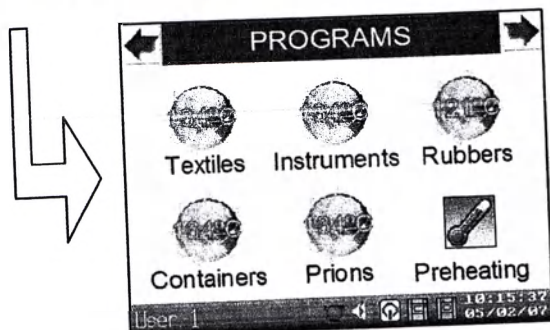- To execute a standard program, press the button corresponding to the program you wish to execute.

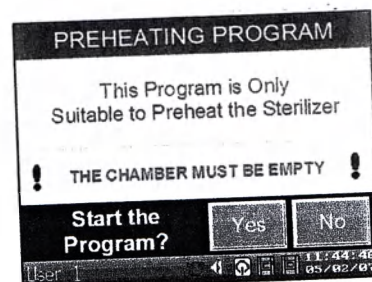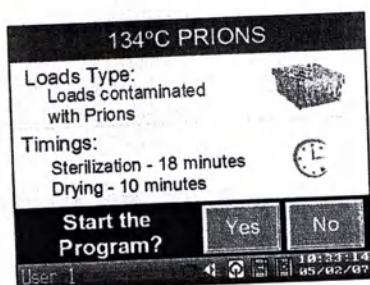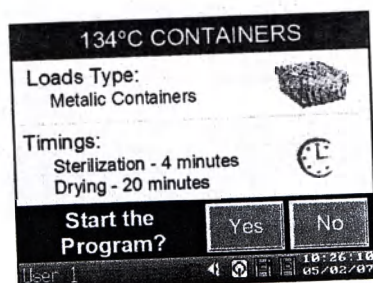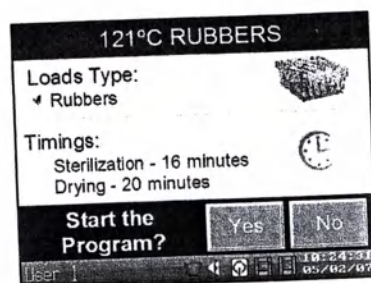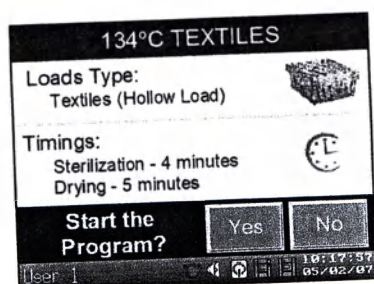- These are the six different windows shown when the corresponding program is pressed. To confirm the start of the cycle press **Yes**, to cancel press **No**.


- Notice that, before the start of any cycle, this is described in its main details, such as, sterilization time, drying time, and, especially, the type of load that must be sterilized with the chosen program. The last program is the exception, this is destined exclusively to the sterilizer's pre-heat, as such, it shouldn't be used to sterilize any material. This cycle should be made before the Bowie & Dick test, and its only purpose is to pre-heat the sterilizer's chamber.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal n° 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

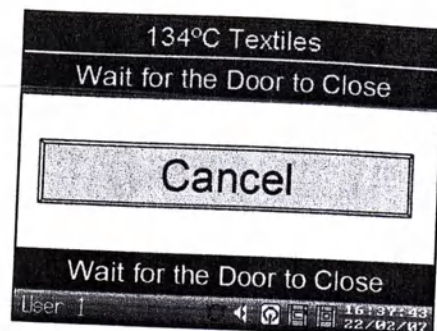
- After confirming the start of the cycle, the door opens, and you're asked to insert the material to be sterilized. Insert the material and press the button **Close**. Wait for the door to close. In the Pre-Heat program the door doesn't open and the cycle begins immediately.



- After the door is closed, the sterilization cycle begins; this is described in detail in the point 6.3.

6.2.2 – Execute the Bowie & Dick Test, the Air Leakage Test or Cleaning the Sterilizer

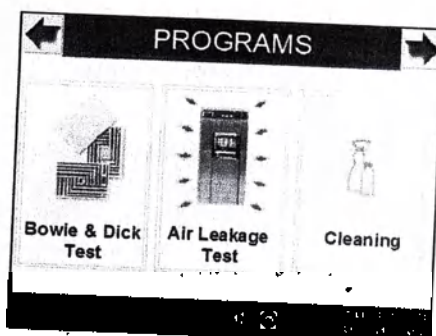- To execute the Bowie & Dick test (this test must be preceded by a Pre-heat cycle) press the corresponding button. Regarding the Air Leakage Test and the Sterilizer's Cleaning, these are described in the points 6.4.2 and 6.4.8 respectively.

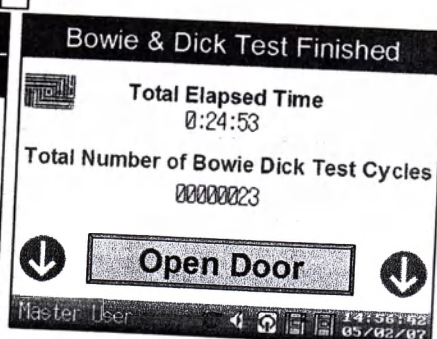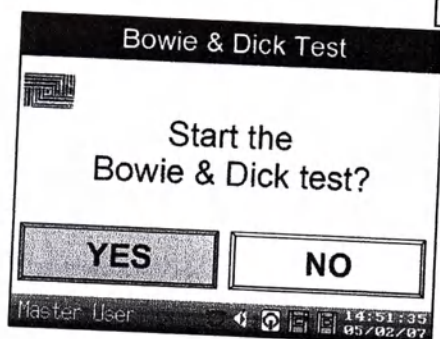- Confirm the cycle's start with the button **Yes**, wait the opening of the door, insert the test package and give the order to close the door. The sterilization cycle begins, this stage is described in detail in the point 6.3.

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
prohs@prohs.pt | www.prohs.pt


6.2.3 – Execute a Configurable Program

- The configurable programs are programs that can be configured by an Authorized Technician, at request from the Responsible of the Sterilization Central. These must be in agreement with the minimum parameters demanded by the effective norms. Up to a hundred distinct programs can be configured. The parameters that can be configured are the number of pulses, the sterilization's time, the sterilization's temperature and the time and type of drying.

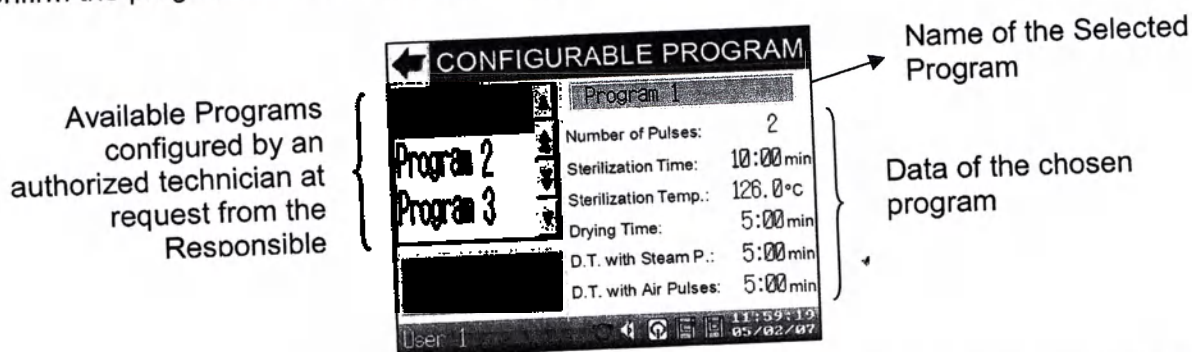- To execute a configurable program, choose it using the up and down arrows (double arrows to bigger steps), when you find the program you want to execute press it (it turns to green), confirm the program in the button **Begin Program**.

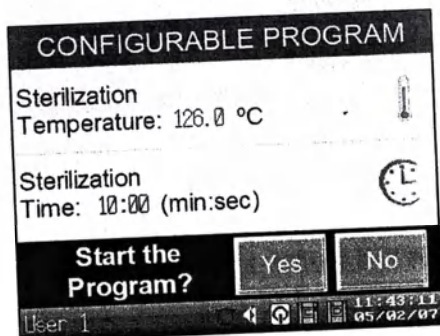- After confirming the Program, you're asked to confirm the start of the cycle, in this window you're presented with the sterilization's temperature and time. Confirm the start of the cycle, the door opens and you're asked to insert the material to sterilize. Insert the material and press the button Close. Wait for the door to close.

- The Sterilization cycle begins, this stage is described in detail in the point 6.3.


PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt


6.3 – The Sterilization's Cycle

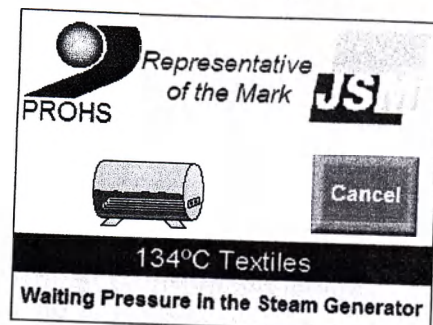- After you give the order to close the door in points 6.2.1, 6.2.2 and 6.2.3, the screen to Wait for Pressure in the Jacket will show. This stage can vary in time depending of the chosen cycle, the sterilizer's size and the moment when the cycle is being executed. For example, in the first cycle of the day, it will take more time. When the steam injection to the jacket is finished and the steam generator is at full working pressure, the sterilization cycle begins.

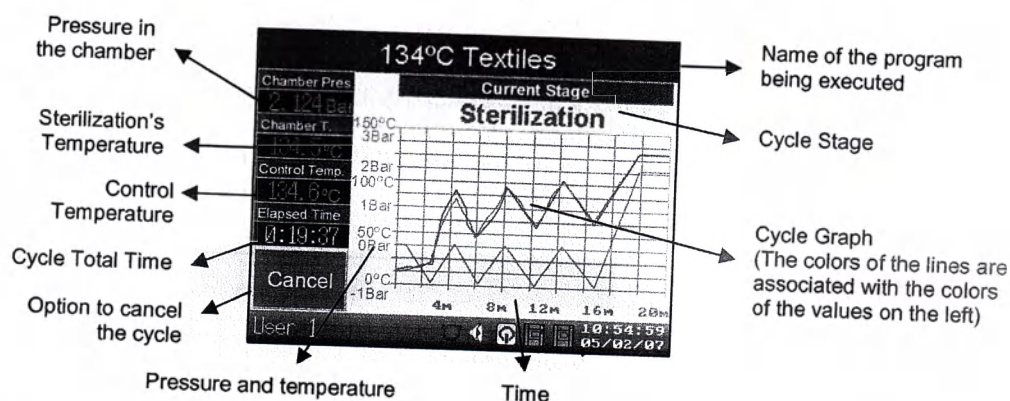- This is the cycle's window, in this stage the sterilization cycle started. In the image below you can see with more detail the informative fields available in this window.

- In the end of the cycle the door automatically opens and an indication of the cycle's total time and the total number of cycles is shown in the image.


- After removing the material from the sterilizer press the button **Close the Door** (When it's a one door sterilizer), or press the button **Door** (When it's a two doors sterilizer) in the command panel of the Sterile zone to go back to the Main Menu and the sterilizer's respective door is closed.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal n° 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

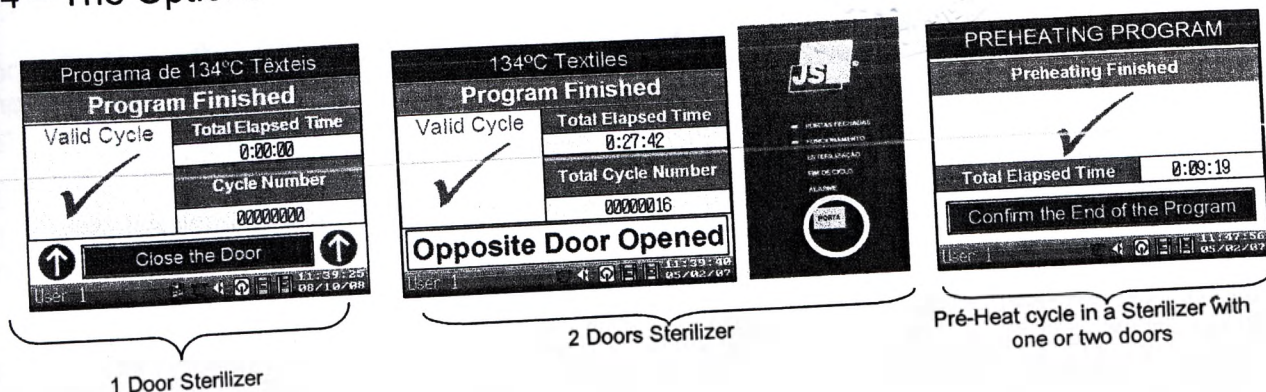
6.4 – The Options


- Selecting **Options** in the Main Menu, you will be asked for an access code, it must be obligatorily the code from the Sterilization Central Responsible.

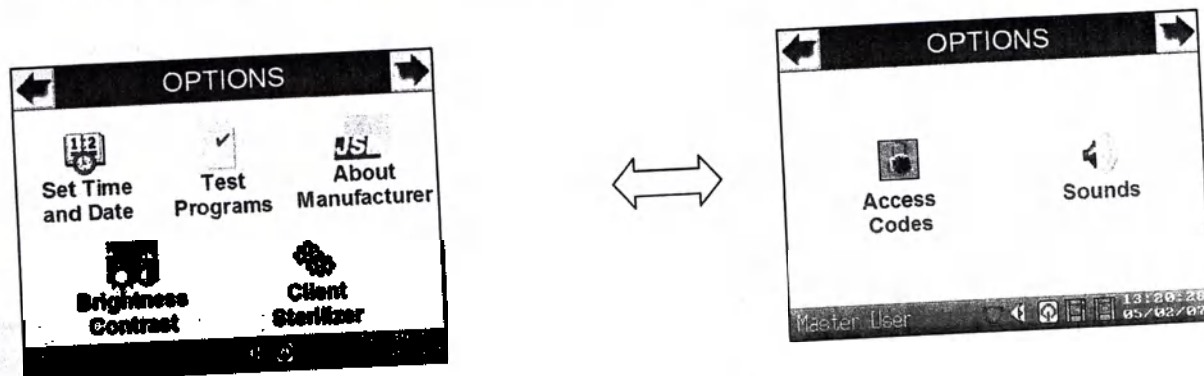- In the screens presented below, you can see all available options, these are described in the following points. The options are split in two distinct Windows.

6.3.1 – Set the Time and Date

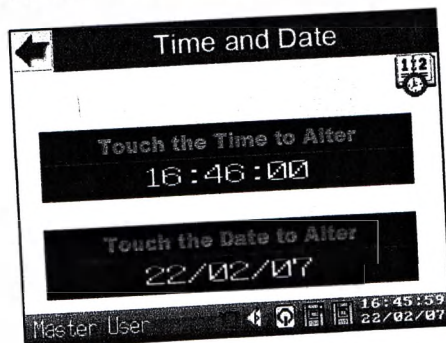- When pressing the button Set Time and Date, in the previous screen, it enters in the window to set the hour and date respectively. Press directly in the time or in the date to set them, insert the time in the format "hour.minutes.seconds" (hh.mm.ss), don't forget to put the point (.) as separator, the date is inserted in a similar way, "year.month.day" (yyyy.mm.dd). In the end press the left upper arrow to return to the Options menu.


PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt


6.3.2 – Execute a Test Program

- Pressing the button **Test Programs** in the Options menu, it enters in the window of possible tests to be executed, in this case the Air Leakage Test. The Program 134°C Fast, not properly being a test program, is used to sterilize material, however this program is only active in situations where the sterilizer is installed in Surgery Rooms.

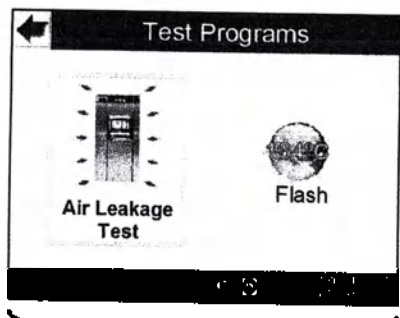
- Press the button of the Program to execute, the Program 134°C Fast is used to sterilize unwrapped instruments in emergency cases in surgery rooms, this program can only be executed by the Responsible.

Sterilizer installed in a surgery room

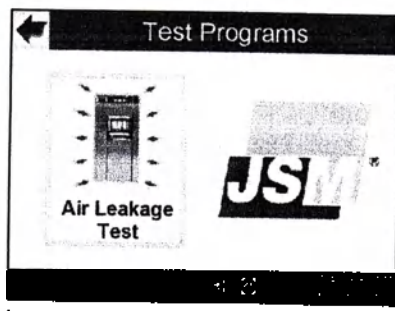
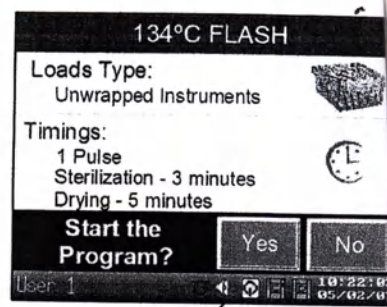
Sterilizer Not installed in a surgery room

- To execute the Program 134°C Fast press the corresponding icon, a window to confirm the beginning of the program will appear, after this point the cycle's execution is, in all ways, similar to the standard cycles already described in points 6.2.1 and 6.3.

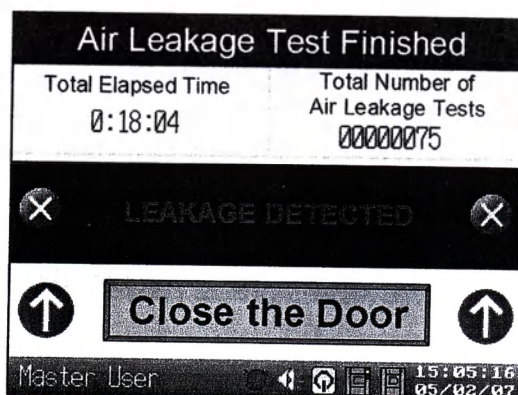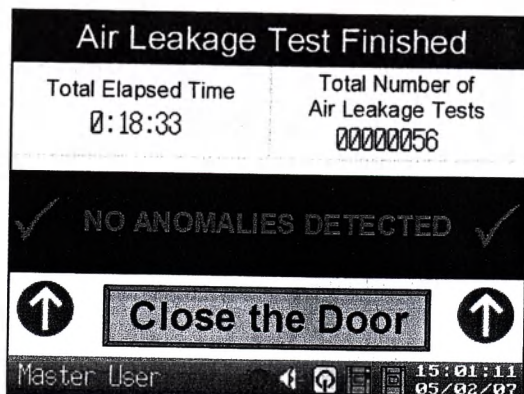- After you start the Air Leakage Test, wait for the ending of the cycle. In the end you're presented with the result, which can be positive or negative. In case of a negative result, that might mean that the sterilizer has a leak in its vacuum system, repeat the test, and if it comes negative again, please contact the Technical Assistance.

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Fax: +351 229 015 900

prohs@prohs.pt | www.prohs.pt

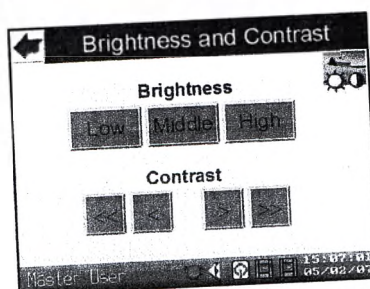
6.3.3 – Informations about the Manufacturer

- By pressing in **About Manufacturer**, in the Options menu, it enters in the window with informations about the Manufacturer, in this menu you have the possibility to see a map of how to get to PROHS.



6.3.4 – Adjust the Brightness and the Contrast of the Display

- By pressing **Brightness and Contrast**, in the options menu, it enters in the window to adjust these parameters. Adjust the contrast in small steps (" $\leftarrow$ " " $\rightarrow$ "), or big steps (" $\leftarrow\leftarrow$ " " $\rightarrow\rightarrow$ "). Regarding the Brightness, you have the choice of three different levels of brightness.

6.3.5 – Change the Client or the Sterilizer

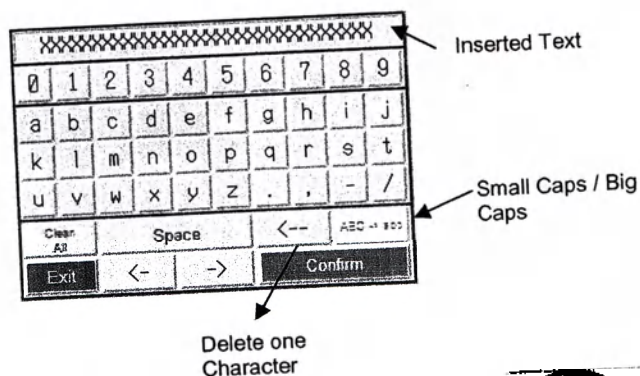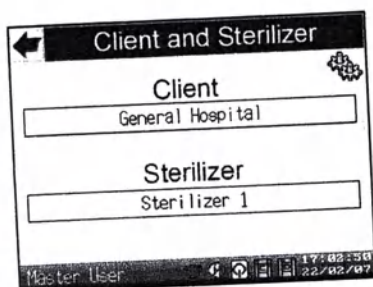- If you want to change the name of the Client and/or the sterilizer that prints in the header of the printed cycles press the button Client / Sterilizer in the Options menu, to change the data press the respective field to show the alphanumeric keyboard. A maximum of 24 characters can be entered.

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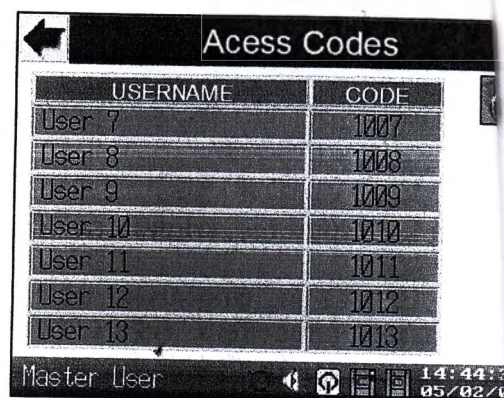
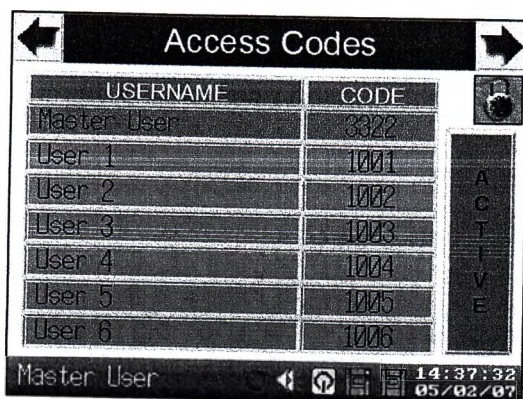
6.3.6 – Manage the Access Codes

- To manage the sterilizer's Access codes press the respective icon in the second Op screen. Once in that page, you are presented with the names and Access codes currently assigned.



- The orange field belongs to the responsible of the sterilization Central, he is the only one can change the name and the Access codes.

1st – Press the name you want to change and the alphanumeric keyboard appears.

2nd – To change the Access codes associated to the names, press the respective code.

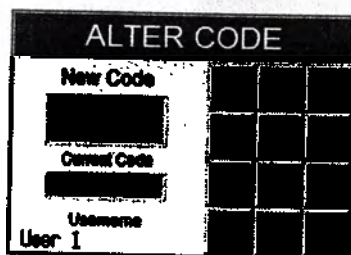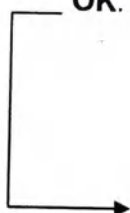3rd – The window to alter the code will show, enter the code (four numeric digits) and press **OK**. If you get an Invalid Code message, please insert a different code and try again.

- Repeat this operation for all the users you need to alter.

- If wanted, there is the possibility to deactivate the Access code request to enter the Programs, for such press the button **Active** / **Inactive**.

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
prohs@prohs.pt | www.prohs.pt


6.3.7 – Sound Options

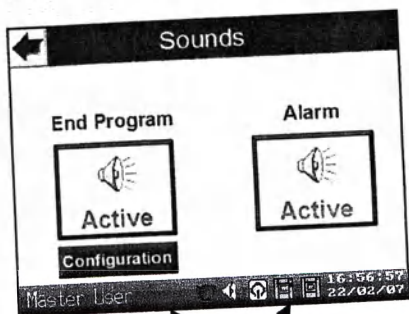- Pressing the Icon **Sounds** in the second Options menu, you enter in the window of possible configurations, regarding the sounds. There are two options described next.
- The sound of End of Cycle has the possibility of being configured, this is, it can be configured to sound continuously or just for a few seconds (programmable), if the last one is selected, you can even opt to repeat, or not, the sound after some determined seconds (programmable).


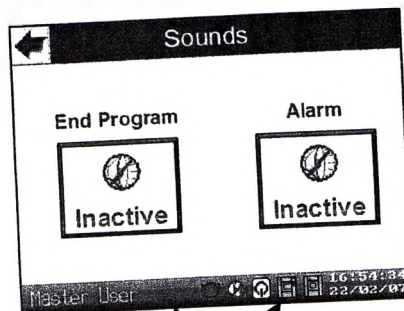
To activate / deactivate the sounds press the respective buttons


The sound of End of Cycle and Alarm are deactivated here

6.3.8 – Sterilizer Cleanness (only in two doors sterilizers)

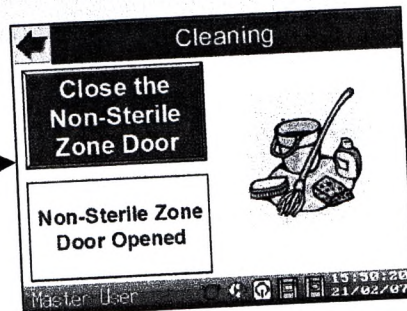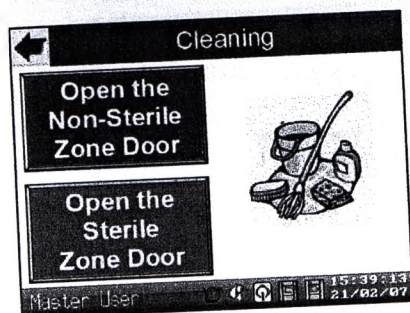- Pressing the icon **Cleaning** in the second Options menu, you enter the menu with the options to open the doors thus to be able to do an effective cleanness in the sterilizer's chamber. The doors can only be open one at a time, alternating, and it's not possible to open the two doors at the same time.

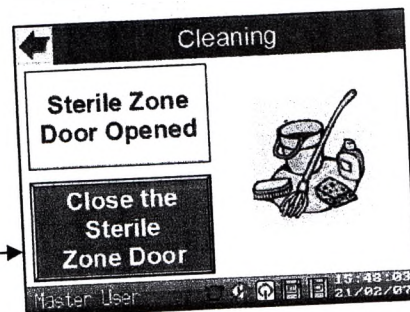Press this button to open the Non-Sterile zone door.

Press this button to open the sterile zone door.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

6.4 – Available Standard Programs

- All the sterilization programs are “Standard”, made by **PROHS**, enclosing a large k materials to sterilize.

- The following tables have the purpose to help the user choose the most adequate pro to its necessities.

Parameters \ Programs	Preheating	134°C Textiles	121°C Rubbers	134°C Instruments	134°C Containers	134°C Prions	134°C Fast	
N.º of Pulses	-	3	3	3	3	3	1	3
Sterilization Temperature (°C)	134	134	121	134	134	134	134	13
Sterilization Time (min)	3	4	16	4	4	18	3	3.3
Drying / Vacuum (min)	1	5	-	-	-	10	5	5
Drying with air Pulses (min)	-	-	20	20	-	-	-	-
Drying with Steam Pulses (min)	-	-	-	-	20	-	-	-

Programs	Type of material
Preheating	Used only to preheat the sterilizer
134°C Textiles	Textiles (Hollow Loads)
121°C Rubbers	Thermo sensitive materials (Rubbers, Glasses, etc.)
134° C Instruments	Wrapped Instruments
134°C Containers	Metallic containers with instruments
134°C Prions	Material contaminated with prions (Creutzfeldt-Jakob)
134°C Fast	Unwrapped materials (Only in the surgery room)
Bowie & Dick	Specific daily test (After Preheating)

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
prohs@prohs.pt | www.prohs.pt

6.5 – Programs Description

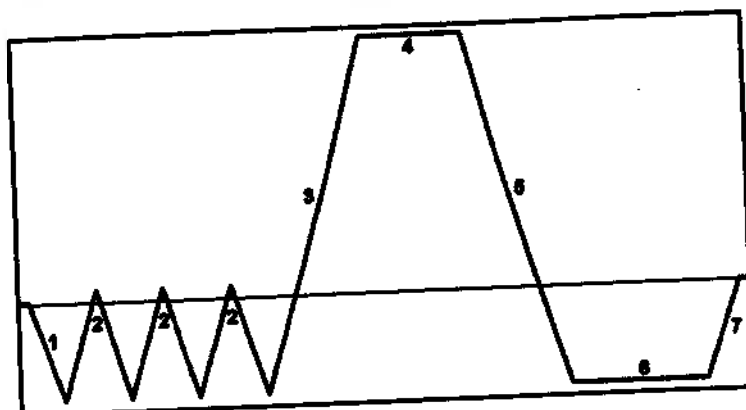6.5.1 – Program 121°C Rubbers / 134°C Textiles / Instruments / Containers / Prions

The Programs 134°C Textiles, 121°C Rubbers, 134°C Instruments, 134°C Containers, 134°C Prions and Bowie & Dick have the following stages:


- The vacuum production and the steam injection in pulses, three times, to achieve a complete elimination of air in the chamber. (1 – Pré Treatment, 2 – Pulses);
- Steam injection until the corresponding values for pressure and sterilization temperature are reached. (3 – Chamber's Steam);
- 4 – Sterilization. This stage refers to the maintenance of the referred parameters, during their respective sterilization times, to determine the death of all living micro organisms;
- Steam exit from the chamber. (5 – Decompression);
- Coinciding with the adequate values of vacuum, time period and drying type (normal or with injection of steam or air pulses), so that the processed material, free himself from all the moistness residues. (6 – Drying);
- The equalizing of Pressures is done through the entrance of sterile air (7 – Air Entrance), in the chamber, passing through a bacteriological filter.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

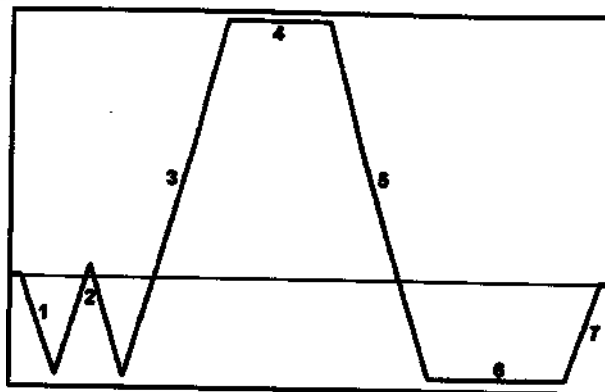6.5.2 – Program 134°C Fast

The Program **134°C Fast** has the following stages:

- The vacuum production and the steam injection in one pulse, to achieve an elimination of air in the chamber. (1 – Pre Treatment, 2 – Pulse);
- Steam injection until the corresponding values for pressure and sterilization temperature are reached. (3 – Chamber's Steam);
- 4 – Sterilization. This stage refers to the maintenance of the referred parameters, during their respective sterilization times, to determine the death of all living micro organisms;
- Steam exit from the chamber. (5 - Decompression);
- Coinciding with the adequate values of vacuum and time period, so that the processed material, free himself from all the moisture residues. (6 – Drying);
- The equalizing of Pressures is done through the entrance of sterile air (7 – Air Entrance), in the chamber, passing through a bacteriological filter.

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
prohs@prohs.pt | www.prohs.pt

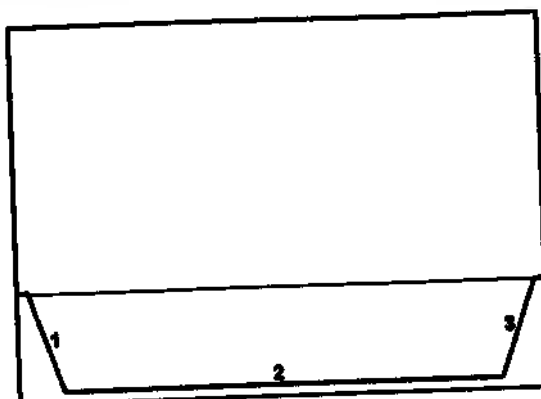5.5.3 – Air Leakage Test Program



The Air Leakage Test has the following stages:

- Vacuum production to -0,93 Bar (1 - Vacuum);
- Vacuum test, during 15 min. (2 – Test stage);
- Air Entrance (3 – Air Entrance).

NOTE:

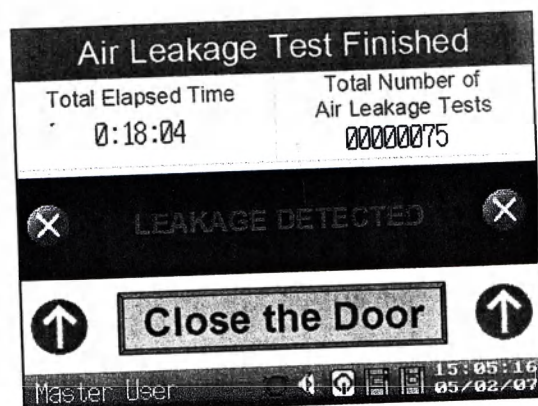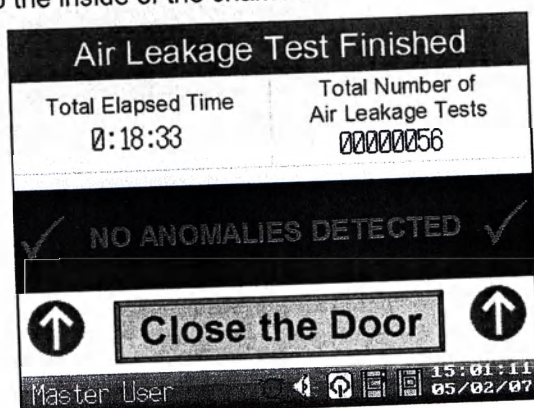- If a variation superior to 13 mbar occurs in the last 10 minutes of the test, it's because a leakage was detected to the inside of the chamber.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal n° 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

6.6 – Preparing the Instruments to Sterilize

The instruments to sterilize must be clean and disinfected, to allow their correct sterilization.

For such it's recommended to:

- Proceed to the decontamination, laundering and disinfection (using the proper equipment ex.: Washer / Disinfector and ultrasonic bath), seeking to eliminate any kind of organic substance and living microorganisms.
- The instruments, after being washed and disinfected, must be submitted to a drying process, before being wrapped and sterilized.

6.6.1 – Unwrapped Materials

If you sterilize unwrapped materials, they must be used immediately after being sterilized, you cannot use them outside this conditions.

Some advices to obtain a correct sterilization:

- Do not sterilize chromed instruments, or instruments that tend to oxidise when in contact with the water steam.
- Materials made from different metals, must be separated and placed in different trays because some metallic elements have corrosive influences between them.
- The metallic instruments not made of stainless steel, must not be in direct contact with the basket, and must be separated by paper, to guarantee that there are no direct contacts between them.
- The instruments must have enough distance between them, allowing their entire surface to have contact with the steam.

6.6.2 – Wrapped Materials

To keep all materials sterile, after their sterilization, they must be wrapped.

Some advices to obtain a correct sterilization:

- Do not overload packages with material.
- Place the packages in the basket, in a way that their plastic parts are in contact. This way you ease the drying of the materials.
- Do not overload the baskets, to improve the steam circulation through the material.

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Fax: +351 229 015 900

prohs@prohs.pt | www.prohs.pt

6.7 – Rules for Opening the Sterile Zone Door

The sterile zone door can only be open in certain situations, to open it press the button

DOOR in the command panel in the Sterile zone.

When the sterile zone door is opened, this information will be presented in the touch screen and will not be possible to do anything else on the screen until the door close

The door can only be open:

- After the end of a sterilization cycle considered valid (to the exception of the Bowie & Dick test and the preheating cycle);
- If the console presents the Main Menu.

The door cannot be open:

- After the end of the Security Program;
- After the end of a Bowie & Dick test;
- After a preheating cycle;
- After the end of an Air Leakage Test;
- After the opening of the non sterile zone door;
- After the On / Off of the Sterilizer;
- If the Display is in a menu, other than the Main Menu.

When the door cannot be open, because one of the above reasons, the luminous pointer of **Closed Doors** will blink repeatedly, whenever the button **DOOR** is pressed.



PROHS - Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt



6.8 – Rules for Placing the Cargo in the sterilizer

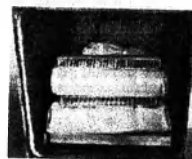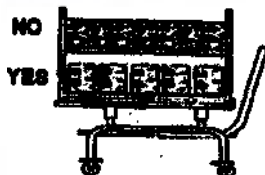- The volume of the material to be placed must not exceed 75% of the sterilizer's total capacity.
- It's recommended to maintain a minimum space of approximately 25mm between packages and the chamber walls, to allow the steam to flow with easiness and in a way in the entire chamber.


- Articles of distinct materials react in different ways to the steam, as such, they should be sterilized together. Whenever possible execute cycles just for one type of material (textiles, just instruments or just containers, etc...).
- The smaller and lighter packages must be placed in the upper part of the chamber. In mixing load the instruments must always be placed in the lower part of the chamber (heavier).
- The instruments, when placed in a basket or in a container, should be uniform distributed to avoid the excessive condensation in certain parts of the materials.
- All containers capable to accumulate condensation (baby bottles, basins, etc...) should stay with the surface that accumulates water placed down.

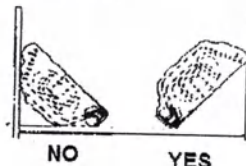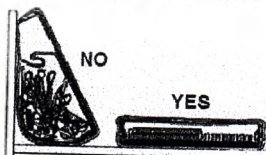- Instruments wrapped in sterilization pouches, should be placed vertically with the paper side turned to the paper side of the adjacent package, and the plastic side placed to the plastic side of the adjacent package, and thus successively.

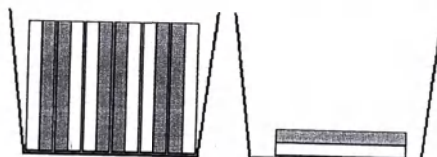Plastic with Plastic
Paper with Paper

If its not possible to put the bags on the vertical, put them in horizontal but with the paper side to the bottom and the plastic to the top

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prohs@prohs.pt | www.prohs.pt

7 – Errors

In any of the Errors, in case a cycle's being executed, first wait for the safety program to execute, and this can vary in time depending of which cycle is being executed. Only after its correct execution the button to Continue will show (Confirm the alarm).

Error Number	Description	What to do
01	Failure in the Steam Generator	The Steam Generator is taking too long to generate Steam. Check the failure and try again. If the alarm continues contact the technical services.
02	Pressure failure in the jacket	The stipulated time to place the pressure in the jacket at the right functioning pressure was exceeded. Confirm the failure and try again. If the alarm continues contact the technical services.
03	Jacket Pressure Extreme / Insufficient	Anomaly detected in the jacket's pressure. Check the Steam Line that feeds the sterilizer. If the sterilizer has an incorporated steam generator check the pressure in the gauge.
04		Confirm the anomaly. If the alarm continues contact the technical services.
05 07 09 11 17 18 19	Failure in making Vacuum	Too much time to do vacuum in the chamber. Check for any leaks. Check if the sewer is clean. Try again. If the alarm continues contact the technical services.
06 08 10 12	Pressure failure in the chamber	Too much time to do the pressure in the chamber, try again. If the alarm continues contact the technical services.
13	Chamber's Temperature Extreme / Insufficient	Anomaly detected in the chamber's temperature. Extreme increase or reduction of the chamber's temperature. Confirm the anomaly. Try again.
14		If the alarm continues contact the technical services.
15	Pressure in the chamber Extreme / Insufficient	Anomaly detected in the chamber's pressure. Extreme increase or reduction of the chamber's pressure. Confirm the anomaly. Try again.
16		If the alarm continues contact the technical services.
20	Serious problem making the vacuum	Confirm the anomaly. Contact the technical services.
21	Serious problem in the air entrance	Confirm the anomaly. Contact the technical services.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
 Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
 Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
 Telf.: +351 229 059 170 Fax: +351 229 015 900
 prohs@prohs.pt | www.prohs.pt

Error Number	Description	What to do
22	Failure in the vacuum pump	Anomaly detected in the vacuum pump. Contact the technical services.
23	Failure in closing the door	Anomaly detected in closing the door. Air leakage in the door. It took more than three minutes to close. Check if there's something stopping the correct closing of the door.
24		Confirm the anomaly. Try again. If the alarm continues contact the technical services.
25	Failure in the door	Anomaly detected in the door. Air leakage in the door.
26		Confirm the anomaly. Try again. If the alarm continues contact the technical services.
27	Failure in opening the door	Anomaly detected in the door. The gasket might not be totally retracted.
28		Confirm the anomaly. Try again. If the alarm continues contact the technical services.
29	Alarm of water lack	Anomaly detected in the line's water pressure that feeds the sterilizer. Check if there's water. If the alarm continues contact the technical services.
30	Alarm in the Air Compressed	Anomaly detected in the pressure of the compressed air. Check if there's pressure in the compressed air line that feeds the sterilizer. If the alarm continues contact the technical services.
31 32	Alarm of sensor failure	Error detected in the sensors
33 34		Confirm the anomaly. If the alarm continues contact the technical services.
35	Power down	Failure in the power supply.
Program Canceled	Program Cancelled	Confirm the anomaly. Program cancelled by the user. Confirm.

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prohs@prohs.pt | www.prohs.pt

8 – Maintenance

As all electric devices, the Horizontal Sterilizer requires a correct use and a periodical maintenance, contributing with this, to a continuous, safe and efficient functioning of the device.

To avoid any risk to the user, it is necessary to regularly control the device, and the maintenance made by a specialized **PROHS** technician.

General Recommendations:

1st - Don't start the Sterilizer if there is no:

1.1 – Water, Compressed Air or Steam.

2nd - Execute daily the **Bowie & Dick** test.

PROCEDURE:

(THIS TEST SHOULD BE MADE DAILY, IN THE BEGINNING OF EACH DAY AND AFTER A MALFUNCTION RESOLUTION)

- Execute a preheating cycle;
- Place the Bowie & Dick package horizontally in the lower centre of the sterilizer's chamber;
- Execute the cycle's specific test (Bowie & Dick test);
- When the cycle is over, remove the package from the sterilizer's chamber;
- Open the package, remove the test sheet and analyse it, the test must have changed its colour uniformly;
- If the result is incorrect, repeat the test two more times. If the results are, once again, incorrect do not use the sterilizer and contact **PROHS**.
- If the result is correct, the Sterilizer is ready to be used.

3rd – Execute weekly the “**Biological Control Test**”

4th – Position the cargo to sterilize in a way that it doesn't touch the chamber walls.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
Telf.: +351 229 059 170 Fax: +351 229 015 900
prohs@prohs.pt | www.prohs.pt

8.1 – Usual Maintenance

This kind of maintenance must be done by the users.

Frequency	Maintenance
Daily	General cleanness of the external surfaces. " Bowie & Dick " test.
Weekly	Interior cleanness of the chamber. "Biological Control" test.
Monthly	In the different stages of each sterilization program, check the values for pressure, Temperature and Time. Cleanness of the door gaskets.
When necessary	Place paper in the printer
	Set the time and date

Notes:

- Before doing any kind of maintenance, check if the equipment is disconnected from the power line.
- The company **PROHS** don't take any kind of responsibility by problems caused on the equipment, in case of components replaced by non-original components.

8.1.1 – Cleaning the External Surfaces

For a good functioning of the device, its external surfaces must be periodically cleaned with a wet cloth and a detergent inert to stainless steel.

8.1.2 – Cleaning the Chamber's Interior, Load Structure

All this accessories must be periodically cleaned, with a wet cloth inert to stainless steel. Then with alcohol followed by water.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
Telf.: +351 229 059 170 Fax: +351 229 015 900
prohs@prohs.pt | www.prohs.pt

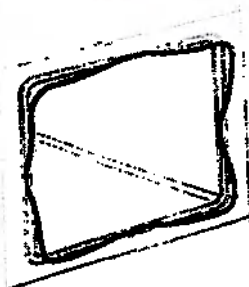
8.1.3 – Cleaning the Door Gaskets

The gaskets must be cleaned to avoid calcify. To clean the door interior, use a non abrasive cloth and a detergent inert to stainless steel.

PROCEDURE:

- Remove the mirror of the door placed on top of it;
- Remove the gullet gasket of the sterilizer;
- Clean the gullet of the sterilizer with a cotton cloth or a compress.

- Check if there aren't any residues in the gullet of the sterilizer that can damage the gasket;
- Check if the gasket as any damage before placing it in the sterilizer's gullet;
- Place the gasket in the sterilizer's gullet;
- Check if the gasket is well placed;
- Place the mirror back on the door.

PROHS -Equipamento Hospitalar e Serviços Associados, S.A.
Rua do Castanhal nº 316 Zona Industrial Maia I Sector II
Apartado 6019 EC Outeiro 4476- 908 Maia - Portugal
Telf.: +351 229 059 170 Fax: +351 229 015 900
prohs@prohs.pt | www.prohs.pt

8.2 – Preventive Maintenance

This type of maintenance must be done by a specialized **PROHS** technician.

Frequency:	Maintenance:
Monthly	- Purge the desumifier of the regulation glass of the Compressed Air. - Clean the filters. - Check the door's pneumatic filter. - Check the tubes and accessories of the compressed air. - Check the electric board. - Manually turn the safety valve. - Refining the door's safety device. - Vacuum test.
Semester	- Replace the jacket steam trap. - Replace gaskets (or every 200 cycles). - Lubrication of the displacement bars of the door - Replace sealants of the door's cylinder. - Check the electromagnetic valves. - Check the pneumatic valves. - Check and clean the safety valves. - Clean the filters. - Check and clean the air compressed valves. - Check the steam traps. - Replace the bacteriological filter. - Check the vacuum pump. - Check and re-squeeze the door's safety bar.

NOTE:

Since the Horizontal Sterilizer has pressure and temperature sensors, to its correct functioning, it may be necessary to do a periodical check. For such, please consult our technical services.

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
9 – Factory Predefined Access Codes

User Name	Access Code
Nurse	3322
User 1	1001
User 2	1002
User 3	1003
User 4	1004
User 5	1005
User 6	1006
User 7	1007
User 8	1008
User 9	1009
User 10	1010
User 11	1011
User 12	1012
User 13	1013

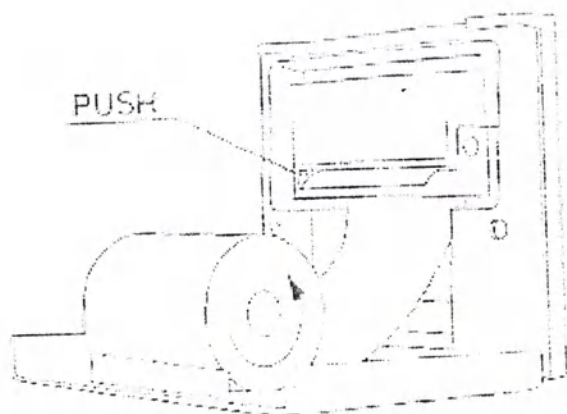10 – Printer

To replace the paper, open the front cover and do as indicated in the above Picture.

To replace the printing tape open the front cover, remove the used tape e place the new one, press PUSH, as shown in the above picture.

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
11 – Air Compressor

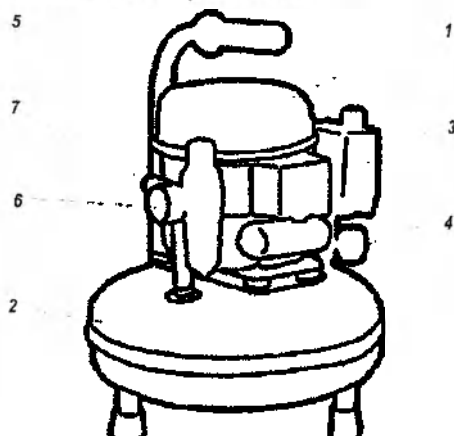11.1 – Main Components

- 1- Electric motor;
- 2- Air deposit;
- 3- Pressure switch;
- 4- Deposit pressure gauge;
- 5- Air regulator;
- 6- Operative pressure gauge;
- 7- Security Valve.

11.2 – Instructions for Use

- Always use the compressor on a flat surface
- The main switch is over the pressure switch (Fig.1);
- Turn the switch to position "0" (Fig.1);
- Connect the power and put the compressor to operate turning the switch in position "1" (Fig.1);
- The functioning is automatic;
- The Pressure switch turns off the compressor when the pressure of the tank reaches the maximum value (8bar) and starts again when the minimum value is restored (6bar);
- Adjust the air pressure in the regulator (Fig.2);
- The pressure is indicated in the gauge placed next to the regulator;

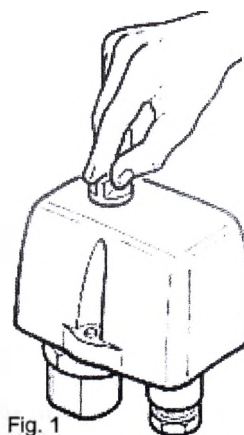
Fig. 1

Attention!

- The compressor must be connected to an electric plug protected by a circuit breaker;

- This table shows the value of consumption of the compressor:

Voltage	230V
Frequency	50Hz
Consumption	2,4A

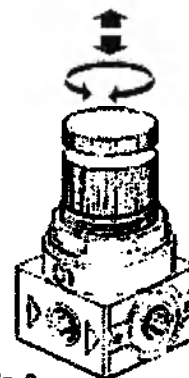
Fig. 2

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

prohs@prohs.pt | www.prohs.pt

11.3 – Periodic Maintenance


All operations listed below must be made by technicians.

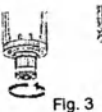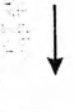 Fig. 3  Fig. 4  Fig. 6  Fig. 7	Once a Week	Once a Month	Once a Year
• Control de oil level.	X		
• With the engine stopped, the level that you see on the display must be the indicated by the label.			
• Remove the water accumulated on the filter glass as indicated in Fig.3 (the operation should be done with the tank at low pressure): • Turn the knob clockwise; • Press the knob for the discharge of condensation in the cup; • When finished, turn the knob rotate counter-clockwise.		X	
• Remove the condensation that creates on the tank, to drain the reservoir, remove the pressure, putting him in a place where the water will not damage the floor and open the tap (Fig.4).		X	
• Controlling the compressor efficiency:		X	
• Check the joints from the racords, the pressure tube, tighten the screws and check the electric parts.		X	
• Control the filter of air aspiration. Replace it if necessary.		Every 3 Months	
• Clean the compressor with a smooth cloth. The dust and dirt avoids a good cooling.		X	
• Disassemble and clean the air filter cartridge contained in the regulator (Fig.6). This shall be done without pressure in the tank.		Every 6 Months	
• Control the filter regulator and the efficiency of the filter element.			X
• Monitor the safety valve (Fig.7) taking slowly the ring with pressure in the tank.			X
• Total replacement of oil.			X

11.3.1 – Oil Change

Proceed as in Figure:

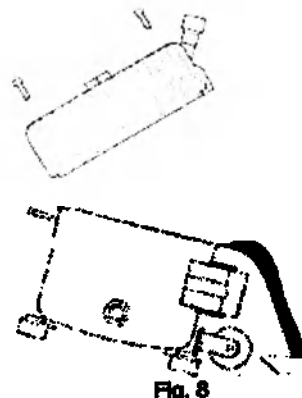1. Disassemble the engine of the tank if necessary;
2. Remove the top unscrewing the 4 screws;
3. Lean a little bit the engine and hold the internal block;
4. Remove the oil;
5. Check the ORing from the top of the compressor;
6. Restore the lid and check when in operation if the ORing is well positioned to ensure a perfect seal when closed;
7. Fastening the 4 screws from the lid;
8. Mount the engine in the tank.

Always use oil "ROLOIL-SINCOM/32E". Another oil can cause serious mechanical damage in the compressor.
Consequently the guarantee is valid only if you use the oil mentioned.

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prohs@prohs.pt | www.prohs.pt

11.3.2 – The Compressor does not Start

- a) Lack of energy. Check the circuit break and the installation;
- b) When the reservoir is at low pressure. The compressor does not start until the pressure does not reach the minimum values of pressure;
- c) The non-return valve has fled. Disassemble the flexible tube to see if air comes out of the valve (Fig.9, item 1). If so, unscrew the lid of the valve, clean the disk of rubber (Fig.9 item 2) with a dry cloth and a re-pieces. If continues a replace valve is needed;
- d) The start relay is damaged. Contact the seller;
- e) Condenser defect. Replace it;
- f) The thermal relay makes noise in the compressor due to high temperature. When cool, the compressor works automatically.

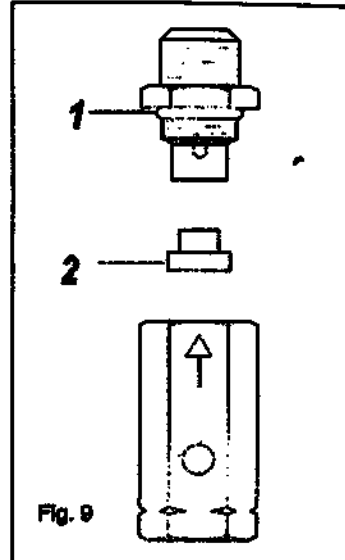
Fig. 9

11.3.3 – The Compressor Works, but doesn't Reached the Maximum Pressure

- a) Make sure there is no air leakage;
- b) Check the efficiency of the pressure switch and adjusted it if necessary;
- c) The non-return valve is blocked and creates a restriction of flow. Replace it.

11.3.4 – The Compressor Works, but the Pressure doesn't Increase (or increases very slowly)

- a) The lid was not taken from the tube or orifice of the aspiration of air;
- b) The oil filter is damaged. Clean it or replace it;
- c) Make sure there is no loss of air.

11.3.5 – The Compressor Works, but doesn't Fill

The defect may be due to rupture of the valve seals. Immediately replace the damaged parts.

11.3.6 – During the Operation the Compressor Stops

The engine contains a thermal relay that closes automatically, the compressor stops when it reaches a high temperature. The compressor restart past 15/20 minutes.

11.3.7 – Air Leakage

It may be due to a bad sealing of some accessories. Check all records wetting them with soap water.

11.3.8 – Leakage in the Valve below the Pressure Switch

The valve is damaged, replace it;

The defect may be due to a bad seal in the valve.

11.3.9 – The Compressor Starts when there isn't use of Air

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Fax: +351 229 015 900

prohs@prohs.pt | www.prohs.pt

11.3.10 – The Compressor Starts and Stops more Frequently than Usual

- a) There is condensation in the deposit. Make a discharge;
- b) Air leakage.

11.3.11 – The Compressor doesn't Start when the Pressure is below the Minimum and / or do not Stop when Reach the Maximum Pressure

Defect in the pressure switch. Change it.

11.3.12 – The Compressor is Very Hot and Consumes too much Oil.

- a) Check the oil level;
- b) The oil is not the correct. Use only the oil recommended by manufacturer;
- c) Air leakages;
- d) The air filter is damaged. Clean it or replace it;
- e) Ambient temperature too high. Never install the compressor inside a closet because it will not have proper ventilation. The compressor should be installed in a fresh zone;
- f) The compressor is overloaded. Make sure that the model is the appropriate for the needs;

**Attention!**

NEVER PASS THE MAXIMUM OIL LEVEL

DO NOT USE OTHER OIL BUT THE INDICATED BY THE SUPPLIER ON PENALTY OF INVALIDATE THE WARRANTY

Save the remaining oil in the bottle for future use. Enter the filter in the suction tube or in the hole, exerting a simple pressure.

After the introduction of oil is essential not to shake or turn the compressor to prevent the oil overflow.

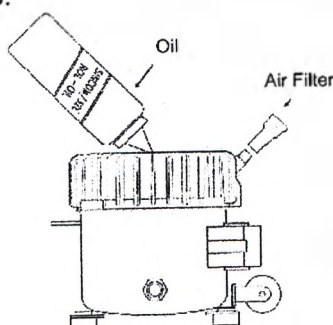
**Attention!**

The safety valve is calibrated and sealed by the manufacturer.

DO NOT TRY TO MODIFY THE COMPRESSOR.

During the operation of the engine group, the output tube may reach high temperature. Avoid contact with it, because it can cause burns.

Failure to comply with these recommendations, may cause serious damage to the compressor or the users.

PROHS - Equipamento Hospitalar e Serviços Associados, S.A.
Rua do Castanhal, 316 - Z. Ind. Maia I, Sector II
Apartado 6019, EC Outeiro • 4476-908 MAIA
Telf: +351 229 059 170 • Fax: +351 229 015 900
e-mail: prohs@prohs.pt
site: www.prohs.pt

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