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FLOW 90[®] Wand Instructions for Use

SYSTEM DESCRIPTION

The WEREWOLF COBILATION[®] System is an electrocautery system consisting of the:

- Bipolar radiofrequency (RF) Connector;
- Disposable, sterile Wand;
- Reusable, non-sterile foot control; and
- Reusable, non-sterile power cord.

The components are designed to be operated as a single unit.

DESCRIPTION

The FLOW 90 Wand (wand) is a bipolar, RF electrocautery device used for specific indications in orthopedic and arthroscopic surgeries. The Wand features AMBIENT[®] technology which provides accurate ($\pm 3^{\circ}\text{C}$ between 20-60 $^{\circ}\text{C}$) real-time temperature monitoring of the intra-articular irrigating fluid. The Wand is designed for use up to 7 minutes of cumulative active ablation and is to be used only with the WEREWOLF[®] Controller. The Wand is capable of operating in five Modes (Table 1).

Table 1. Modes and Levels for the FLOW 90 Wands

Action	Mode	Level	Mode Level Indicator
Ablation	Lo	Lo Minus (-)	⬇️
		Lo (+)	⬆️
		Lo Plus (+)	⬆️
Ablation	Med	Med Minus (-)	⬇️
		Med (+)	⬆️
		Med Plus (+)	⬆️
Ablation	Hi	Hi Minus (-)	⬇️
		Hi (+)	⬆️
		Hi Plus (+)	⬆️
Coagulation	Coag	Coag (-)	⬇️
		Coag Plus (+)	⬆️
Vacuum	Yes	Not Adjustable	N/A

* Default Modes and Levels for FLOW 90

Note: Lo Mode is recommended for the debridement of articular cartilage.

Table 2. Output Voltage for the WEREWOLF COBILATION System with the FLOW 90 Wands

Output Voltage (V) at 100% duty	Power (W) at 100% duty
100	100
150	150
200	200

System Preparation

1. Prepare the patient pre-operatively according to standard procedures.
2. Install the WEREWOLF Controller according to the instructions in the User Manual. Press the Main Power switch (MPS) on the front of the Controller to activate the Controller.
3. Connect a new sterile Wand connector to the Controller. The Controller will prompt you with animation.
4. Open the Controller door. The Controller will prompt you with animation.
5. Insert the Wand tubing. Match the connector colors on the tubing with the colors of the fittings on the Controller. The Controller will prompt you with animation. Note: If the Wand tubing is not properly installed, the Controller will not allow activation of RF energy.
6. Close the door. The Controller will prompt you with animation.
7. Attach the tubing connector at the end of the tubing to a waste collection receptacle.
8. If desired, connect the foot control to the Controller. Follow the instructions provided in the User Manual.
9. When the Wand is installed in the Controller, the Controller sets the Wand Coag Mode and Ablation Mode and Level to default. Select the appropriate settings on the Controller touch screen following the instructions provided in the User Manual.

Wand Use

Note: The electrode at the distal tip of the Wand is applied to tissue to effect ablation and coagulation. The Wand is bipolar. A neutral electrode is not used with this Wand.

CAUTION: Use the lowest Controller setting necessary to achieve the desired tissue effect.

CAUTION: The Wand is not designed for mechanical displacement of tissue through applied force. Do not use the Wand as a lever to enlarge a surgical site or gain access to tissue as this may result in a bent or detached electrode, damage to the device, and/or a cracked spacer leading to patient injury.

1. Insert the Wand into the treatment area through a prepared entry point. Ensure the Wand tip is surrounded by irrigating fluid when in use.
2. CAUTION: Use caution when inserting and removing the Wand with a cannula. Connecting the Wand electrode and/or insulation against edges of the cannula may lead to Wand damage.

Ablation

2. To activate an Ablation Mode using the integrated finger switch, press and hold the YELLOW button. The Ablation Mode and level will be displayed on the Controller and a tone will be heard.

The Wand should be activated prior to contacting the tissue and then moved into light contact with the targeted tissue. While activated, the Wand should be kept in constant motion. Refer to the User Manual for the Controller for how to activate using the foot control.

Note: Lo Mode is recommended for the debridement of articular cartilage. For treating articular cartilage, the Wand should be positioned on its side (45 to 90 degrees) so that only the edge of the electrode contacts the targeted tissue. This will help to control ablation depth and enhance visibility.

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