3M
Opticlude™

Orthoptic Eye Patch for occlusion therapy

GB

Description:

3M™ Opticlude™ Orthoptic Eye Patch consists of a soft nonadherent pad bonded to a skin-friendly breathable and conformable backing coated with a hypoallergenic adhesive. The dressing is breathable and conforms to body contours.

Intended Use:

3M™ Opticlude™ Orthoptic Eye Patch is used in eye occlusion treatment of Amblyopia and Strabismus in children.

Precautions:

Follow directions provided by your eye care professional regarding use of 3M™ Opticlude™ Orthoptic Eye Patch.

In case of any skin reactions, immediately stop using the patch and consult your doctor. For patients with sensitive skin we recommend to use Opticlude™ Silicone eyepatches.

Reused product does not fulfill claims and can not be used as intended.

Instructions for use:

Application

1. For best adhesion, first clean and dry skin.

Remove paper liner from the eye patch.

2. Gently close both eyes keeping facial muscles relaxed.

Wrinkles around the eye should be avoided.

3. Apply 3M™ Opticlude™ Orthoptic Eye Patch with the narrow end towards the nose. Do not stretch the patch, or pull skin.

4. Secure the pressure sensitive adhesive tape by applying light pressure with fingertips around the outside of the eye patch.

Removal

1. At the nose side of eye patch, grasp and loosen the outer edge of the eye patch.

2. Support the skin with your fingers. Remove the tape low and slow, peeling the eye patch towards the ear, keeping it close to the skin surface and pulled back over itself. Please be aware that removing the eye patch at an angle will pull the skin surface and increase the risk of redness and irritation.

3. As the patch is removed, continue to support the newly exposed skin.

Storage and Shelf Life:

For best results store the 3M™ Opticlude™ Orthoptic Eye Patch at normal room temperature. Shelf life is provided by the expiration date printed on the box.

Made in Poland

3M Poland Manufacturing Sp. z o.o.

ul. Kwidzyska 6

51-416 Wrocław, Poland

3M and Opticlude are trademarks of the 3M Company

© 2016, 3M

All rights reserved.

YP-2763-0348-1 / 0716

Date of the latest revision of the instruction for use 2016-07

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