**3M Medipore + Pad Adhesive Wound Dressing**

1. 5 x 7.2 cm (50 dressings per box)
2. 5 x 9.2 cm (5 dressings per box)
3. 6 x 10 cm (50 dressings per box)
4. 10 x 10 cm (25 dressings per box)
5. 10 x 15 cm (25 dressings per box)
6. 10 x 20 cm (25 dressings per box)
7. 10 x 25 cm (25 dressings per box)
8. 10 x 30 cm (25 dressings per box)
9. 10 x 35 cm (25 dressings per box)

**Description:** The 3M Medipore + Pad Adhesive Wound Dressing consists of a non-adherent absorbent pad bonded to a larger soft cloth backing coated with a border of hypoallergenic, water-resistant adhesive.

The dressing is hypoallergenic and presented sterile.

Dressing is protected by a silicone paper and packed in paper/paper pouch

**Indications for Use**

The 3M Medipore + Pad is non-invasive device which come into contact with injured skin and by covering wounds is used as a mechanical barrier and for absorption of exudates.

Dressings are intended to be used by professionals and at home.

The 3M Medipore + Pad adhesive wound dressing is designed to be used as a mechanical barrier and for absorption of exudates for acute wounds such as cuts, burns, abrasions, IV catheter sites and surgical incisions.

Follow your "gauze and tape" protocol for use of this dressing. This product is not designed, sold or intended for use except as indicated.

**Contraindications**

No contraindications have been identified for these products.

**Potential complications**

No potential complications have been identified for these products.

**Warnings**

- 1) The 3M™ Medipore™ + Pad adhesive dressing is not intended to replace sutures or other primary wound closure methods.
- 2) The dressing should be changed if adhesion is compromised or exudate penetrates through the soft cloth backing.
- 3) The site should be observed frequently for signs of infection. If infection occurs (which may be signaled by fever, pain, redness, swelling, or an unusual odor or discharge), remove the dressing and take standard precautions

**Precautions**

- 1) Haemostasis of the catheter site or wound should be achieved before applying the dressing.
- 2) The skin should be dry and free of detergent residue to ensure good adhesion.
- 3) Mechanical skin trauma may result if tension is applied.
- 4) When applied on areas of high mobility or considerable tissue swelling is anticipated, tension-related skin injuries may occur.

- 5) Medipore + Pad is not intended to be resterilized.
- 6) Reused product does not fulfil claims and can not be used as intended.

### Directions for Use

Preparation: In accordance with standard hospital protocol, prepare the site for catheter insertion or wound dressing. To ensure best dressing adhesion, clipping of excess hair at the site may be necessary. Shaving is not recommended. Apply to a clean, dry site. For best results, the dressing should be large enough to have a margin which adheres to dry, healthy skin around the wound site.

1. Open package and remove dressing.
2. Peel the paper liner from the dressing, exposing the adhesive surface.
3. Position the dressing over the wound site or the catheter insertion site and apply dressing. To avoid skin tension, flex joints before application.
4. Smooth down the dressing edges. Seal securely around catheter or wound site. Firmly smooth adhesive border to the skin.
5. Check tension on the dressing regularly, particularly if the wound is subject to swelling.
6. Change the dressing should exudate penetrate through the soft cloth backing.
7. To remove the dressing, gently grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

### Storage/Shelf Life

The shelf life of the packaged product (3M Medipore + Pad) is 2 years from the date of adhesive coating / laminating.

Do not use after the expiration date, which is printed on each package.

### Information about repair and maintenance

This medical device isn't subject to maintenance and repair.

### Utilization

Dispose of the device in accordance with applicable local/regional/national/international regulations. Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B).

### Transport conditions

Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance.

Transportation of medical device is carried out by all type of transport.

### Warranty

3M Poland Manufacturing Sp. z o.o. warrants this product will be free from defects in material and manufacture.

User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Poland Manufacturing Sp. z o.o. sole obligation shall be repair or replacement of the 3M Poland Manufacturing Sp. z o.o. product.

Explanation

Do not

This rubber

Caution

Do not

For further information visit [www.3m.com](http://www.3m.com)

Made in  
3M  
ul. F

## Explanation of Symbols

Do not use if package is damaged or open

This product and package do not contain natural rubber latex.

Caution, see instructions for use

Do not reuse


Use by date

Batch code

Manufacturer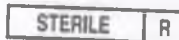

Sterilized using irradiation

For further information, please contact your local 3M representative or contact us at [www.3M.com](http://www.3M.com) and select your country.

0459

Made in Poland

**3M Poland Manufacturing Sp. z o.o.**  
ul. Kwidzyńska 6, 51-416 Wrocław Poland

Authorised representative of the  
manufacturer in Russia:  
ZAO "3M Russia", Russia 121614,  
Moscow, Krylatskaya str.  
bldg.17/3  
T.+74957847474

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