
Acknowledged as True and Correct
General Manager
Shandong Ming Yuan Latex Co., Ltd.
Han Xiujuan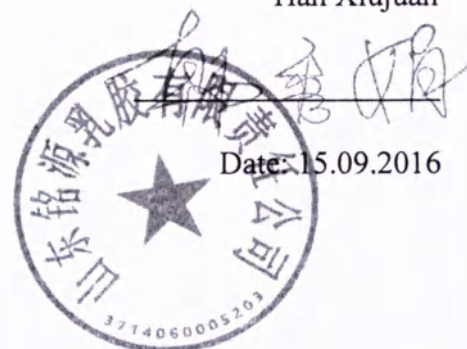

Natural latex Condoms R and J

Direction for use

Russian edition

2016

CONTENT

1.	INTENDED USE	3
2.	INDICATIONS	3
3.	CONTRAINDICATIONS	3
4.	PRECAUTIONS	3
5.	INTENDED APPLICATION	4
5.1.	GENERAL PROVISIONS	4
5.2.	PRE-STARTING PROCEDURES	4
5.3.	PROCEDURE OF WORK WITH DEVICE	4
6.	SPECIFICATIONS	6
7.	DISPOSAL AND ENVIRONMENTAL PROTECTION	8
8.	TRANSPORTATION AND STORAGE	8
9.	PACKAGE CONTENTS	8
10.	MARKING	8
11.	MANUFACTURER WARRANTY	9
12.	MANUFACTURER INFORMATION	9
13.	RESPONSIBLE SUPPLIER	9

1. INTENDED USE

This Direction for use is applicable to natural latex condoms R and J (hereinafter referred to as condoms) for full cover of penis during sexual intercourse to prevent penetration of sperm into female genital tracts and/or to prevent sexually transmitted diseases

Domain of usage: general medical and preventing treatment facilities as well as home conditions.

Natural latex condoms R and J are manufactured in the following types: (see annex 1).

2. INDICATIONS:

- to prevent from unintended pregnancy;
- to prevent from HIV and STDs;
- occasional sexual intercourses and/or late reproductive age;
- temporary means of protection during break in use of intrauterine device or hormonal tablets.

3. CONTRAINDICATIONS:

This device is made of natural latex that may cause allergic reactions of some people. If you are allergic to latex, consult the doctor before using such condoms.

POSSIBLE SIDE EFFECTS

Runny nose, itching, sneezing, dizziness, rough breathing; hypoesthesia or erectile dysfunction.

4. PRECAUTIONS:

- Only use a condom once.
- Store condoms in a cool dark place, away from direct sunlight.
- Don't use the condom if it's deformed, brittle or obviously damaged, if the external wrapper of condom is torn or individual pack is torn.
- This device contains silicon oil lubricant. If you want to use additional lubrication use the lubricant recommended for common use with condom. Avoid using oil-based lubricants like vaseline, baby oil, shower gel, massage oil, solid oil, oleomargarine and etc. that can easily damage latex condoms.
- Before any sexual contact, place the condom on the head of the erect penis, after that you may start sexual intercourse to avoid sexually transmitted diseases and unintended pregnancy.
- If sperm got into vagina immediately use emergency contraception.
- This device should be used before its expiration date
- Condom doesn't provide 100% protection from sexually transmitted diseases and unintended pregnancy.
- Use condoms of proper size and shape and be careful using it. While opening the foil avoid pricking condom with nails, jewellery not to damage it.
- Consult the doctor or pharmacist about medications you can use together with the condom.

5. INTENDED APPLICATION:

5.1 General provisions

Condoms are the only effective remedy at preventing pregnancy and HIV/AIDS and other sexually-transmitted infections, don't require engagement of highly experienced medical staff.

If the integrity of the condom is broken (e.g. if it is punctured, cut), toss the condom out and use a new one.

It is used as hygiene product for personal use to prevent sexually transmitted diseases and unintended pregnancy.

5.2 Pre-starting procedures:

5.2.1 Before condom using make sure of package integrity. In case of revealing of tear or holes, it is necessary to take a new condom.

5.3 Procedure of work with device:

5.3.1 Use a new condom before each sexual intercourse. While opening the package don't use sharp objects. Put a condom on prior to sexual intercourse

5.3.2 Method of application.

1. Open the package carefully so that there's less chance of ripping it: tear the package open from the serrated edge.

2. Place the condom on the head of the erect penis such that free space is on the top.

3. Pinching the tip of the condom, roll down the condom until it reaches the base of the penis.

4. Make sure that there is free space on the top and the condom isn't damaged.

5. After ejaculation and while the penis is still erect, pull the erect penis out holding onto the condom at the base of the penis.

5.3.3 Removal method

Remove the condom carefully, roll it in a paper tissue and throw it away in a bin.

6. SPECIFICATIONS

6.1. Condom sizes (fig. 1).

6.1.1. Condom length should be:

- length L1: ≥ 180 mm;
- length L2: ≥ 160 mm;
- length L3: 18 ± 2 mm.

6.1.2 Condom width W should be: 54 ± 2 mm.

6.1.3 Rim width V should be: $1,5 \pm 0,3$ mm.

6.1.4 Wall thickness C should be: $0,06 + 0,03 / - 0,01$ mm.

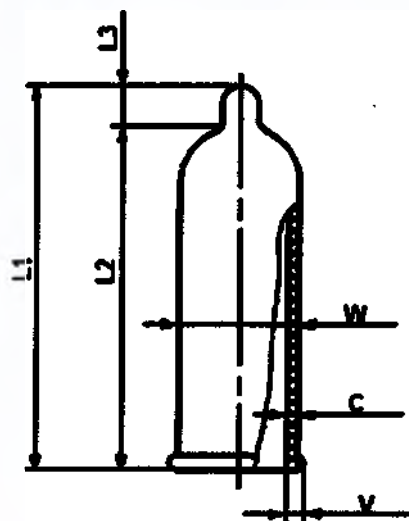
Figure 1 – Condom sizes.

6.2 Condom weight should be: $1,4 \pm 0,3$ g.

6.3 Physical and mechanical properties of the condom comply with the Table 1

Table 1

Property	Value	
	before heat ageing	after heat ageing
1. Strength condition under tension, MPa, (kgf/cm ²), not less	20(200)	17(170)
2. Elongation at break, %, not less	700	600

6.4 Condoms are manufactured from materials shown at Table 2.

Table 2 Materials (Raw materials), which have a contact with healthcare personnel and (or) patient

No	Name	Used material		Manufacturer
1	Natural latex condoms R and J 3 in 1 Elite Anatomic (contoured anatomic ribbed and dotted) - 3 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
2	Natural latex condoms R and J 3 in 1 Elite Anatomic (contoured anatomic ribbed and dotted) - 10 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
3	Natural latex condoms R and J Delicate (ribbed) - 3 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
4	Natural latex condoms R and J Delicate (ribbed) - 10 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	

5	Natural latex condoms R and J Natural Classic Thin (classic thin) -3 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
6	Natural latex condoms R and J Natural Classic Thin (classic thin) - 10 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
7	Natural latex condoms R and J Tender (ultra soft) -3 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
8	Natural latex condoms R and J Tender (ultra soft) - 10 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
9	Natural latex condoms R and J Tender (ultra soft) -144 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
10	Natural latex condoms R and J ULTRA THIN (natural feels) -3 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	
11	Natural latex condoms R and J ULTRA THIN (natural feels) - 10 pcs./package;	Condom	see Annex 2	Shandong Ming Yuan Latex Co., Ltd.
		Lubricant	Dimethicone (CAS No. 9016-00-6)*	

* - pharmacopoeial monograph [9016-00-6] according to RF State Pharmacopoeia.

6.5 Specific features of the device are the following:

- natural latex condoms R and J 3 in 1 Elite Anatomic (contoured anatomic ribbed and dotted) 3 pcs./package and 10 pcs./package are available in anatomic shape and have a texture (ribbed and dotted) , uncolored.


- natural latex condoms R and J Delicate (ribbed) 3 pcs./package and 10 pcs./package have a ribbed texture, uncolored.

- natural latex condoms R and J Natural Classic Thin (classic thin) 3 pcs./package and 10 pcs./package; natural latex condoms R and J ULTRA THIN (natural feels) 3 pcs./package and 10 pcs./package have thin walls, uncolored.

- natural latex condoms R and J Tender (ultra soft) 3 pcs./package, 10 pcs./package and 144 pcs./package with high-lubricant, uncolored.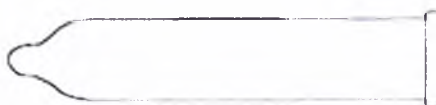

7. DISPOSAL AND ENVIRONMENTAL PROTECTION

Condoms should be disposed in accordance with household waste disposal rules. Disposal should be carry out in accordance with local legislation current standards and requirements.

8. TRANSPORTATION AND STORAGE

8.1 The transportation of the condoms should be carry out by any means of transport in covered vehicles in accordance with rules of cargo transportation valid for this transport.

8.2 While transporting at sub-zero temperatures, condoms should be unpacked after holding at normal environmental conditions during 24 hours from receipt.

8.3 Condoms should be stored packed in closed warehouse room at a temperature 0 to 25°C and relative humidity not above 80% in the distance of no more than 1 m the heaters.

While storing condoms should be protected from direct sunlight, should not be impacted by vaseline, oils, organic solvents, petroleum product, carbolic acid, acids and alkalies.

9. PACKAGE CONTENTS

№	Description	Quantity, pc.
1	Natural latex condoms R and J *	1 package
2	Direction for use	1
3	Warranty	1

*- the customer can choose any condom type of a list of mentioned ones, see Annex 1

10. MARKING

The symbols used in marking.

- Lot No:

(number)

- " SINGLE USE ONLY "

- " Manufacturing date "

(year/month)

- " Best by ..."

(year/month)

- " The product is made of natural latex rubber, it may cause an allergy "

- CE marking

- " It is necessary to throw the used device into the special litter bins and containers."

- "Green Dot"

11. MANUFACTURER WARRANTY

The manufacturer guarantees compliance of the condoms to the national standards requirements if the operating, transportation and storage conditions are satisfied by a customer.

Guaranteed shelf-life of condoms – 5 years as from the date of production.

12. MANUFACTURER INFORMATION:

Shandong Ming Yuan Latex Co., Ltd., Middle Licheng Road, Development Zone of Linyi County 251500, Dezhou City, Shandong Province, China. Telephone: 86-156-0547-8830/86-534-2147767, E-MAIL: oemcondom@gmail.com.

13. RESPONSIBLE SUPPLIER:

Limited liability company GLOBUS-MARINE (GLOBUS-MARINE LLC), 10H/9, lit. A, 1/32, Ropshinskaya street, Saint Petersburg, Russian Federation, 197198. Telephone: +7(812)242 88 18.

ANNEX 1

I Natural latex condoms R and J, possible types:

1. Natural latex condoms R and J 3 in 1 Elite Anatomic (contoured anatomic ribbed and dotted)
-3 pcs./package;
2. Natural latex condoms R and J 3 in 1 Elite Anatomic (contoured anatomic ribbed and dotted)
- 10 pcs./package;
3. Natural latex condoms R and J Delicate (ribbed) -3 pcs./package;
4. Natural latex condoms R and J Delicate (ribbed) - 10 pcs./package;
5. Natural latex condoms R and J Natural Classic Thin (classic thin) -3 pcs./package;
6. Natural latex condoms R and J Natural Classic Thin (classic thin) - 10 pcs./package;
7. Natural latex condoms R and J Tender (ultra soft) -3 pcs./package;
8. Natural latex condoms R and J Tender (ultra soft) - 10 pcs./package;
9. Natural latex condoms R and J Tender (ultra soft) -144 pcs./package;
10. Natural latex condoms R and J ULTRA THIN (natural feels) -3 pcs./package;
11. Natural latex condoms R and J ULTRA THIN (natural feels) - 10 pcs./package.

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