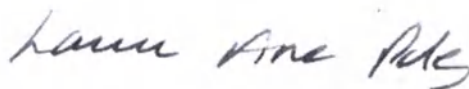
BE IT KNOWN that I Laura Anne Pawley, Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG, a duly authorised Notary Public, duly admitted, sworn and practice throughout England and Wales.

CERTIFY ONLY that Brian Michael Howes, a United Kingdom Citizen, who is well known to me, who, is duly authorised by Wendy Mejia of Reckitt Benckiser Healthcare (UK) Limited ('the Company') to represent them in this matter, has today caused the annexed Maintenance Documentation (Instruction for Use)- Reflex Natural Rubber Latex Lubricated Condoms for use in Russia to be produced to me and that they have represented to me on behalf of the Company that the said document is a true electronic copy of the original document.

SIGNED and sealed at Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG on the 31st of March 2023.

Protocol No: 2023-2032

A handwritten signature in black ink that reads "Laura Anne Pawley".

Laura Anne Pawley
Notary Public

Dear Sir / Madam

I Wendy Mejia of Reckitt Benckiser Healthcare (UK) Limited CERTIFY that the attached Maintenance Documentation (Instruction for Use) - Reflex Natural Rubber Latex Lubricated Condoms is a true electronic document for use in The Russian Federation.

Name: Wendy Mejia

Signature: *Wendy Mejia*

Electronically signed by: Wendy Mejia
Reason: I have reviewed this document.
Date: Mar 29, 2023 18:41 GMT+1

Date: 29-Mar-2023

**Reckitt Benckiser Healthcare (UK)
Limited**

103-105 Bath Road
Slough, Berkshire SL1 3UH
United Kingdom
T +44 (0)1753 217800
F +44 (0)1753 217899

Approved by:

Andrew Brownhill

Electronically signed by: Andrew Brownhill

Reason: I approve this document.

Date: Mar 24, 2023 11:08 GMT

Andrew Brownhill

Senior Global Regulatory Manager

24-Mar-2023

Date: _____

Maintenance documentation

(Instruction for use)

REFLEX NATURAL RUBBER LATEX LUBRICATED CONDOMS

NAME OF THE MEDICAL DEVICE

Reflex natural rubber latex lubricated condoms

Variants of medical device

1. Reflex Classic natural rubber latex lubricated condoms – 3 pcs/pack, 12 pcs/pack
2. Reflex Dotted natural rubber latex lubricated condoms – 3 pcs/pack, 12 pcs/pack
3. Reflex Light natural rubber latex lubricated condoms – 3 pcs/pack, 12 pcs/pack

MANUFACTURER

Head Office Address:

Reckitt Benckiser Healthcare (UK) Limited
103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK

Location address

Reckitt Benckiser Healthcare (UK) Ltd.
Dansom Lane, Hull, HU8 7DS, UK

MANUFACTURING SITE*

SSL Manufacturing (Thailand) Ltd.
Wellgrow Industrial Estate 96&100, Moo 5, Bangna -Trad Rd KM.36,
Bangsamak, Bangpakong, Chachoengsao, 24180, Thailand

**information about the place of manufacture is presented in the outer side of the secondary/consumer packaging in the format "The country of origin: [country]"*

AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

Reckitt Benckiser Healthcare LLC, 115114, Moscow, 4 Slusovaya emb., Floor 3, Russia

PURPOSE OF MEDICAL DEVICE

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

POTENTIAL CONSUMERS OF THE MEDICAL DEVICE

Medical Device is intended to be used by laypersons.

MEDICAL DEVICE COMPOSITION, DESCRIPTION AND MAIN TECHNICAL CHARACTERISTICS

1) Reflex Classic natural rubber latex lubricated condoms

Reflex Classic natural rubber latex lubricated condoms - colourless, smooth, of anatomical shape, teat-ended, lubricated with silicone. Nominal sizes: length – 190 mm, width – 52.5 mm.

2) Reflex Dotted natural rubber latex lubricated condoms

Reflex Dotted natural rubber latex lubricated condoms - colourless, with dotted structure, straight-walled, teat-ended, lubricated with silicone. Nominal sizes: length – 180 mm, width – 52 mm.

3) Reflex Light natural rubber latex lubricated condoms

Reflex Light natural rubber latex lubricated condoms - colourless, smooth, straight-walled, teat-ended, lubricated with silicone. Nominal sizes: length – 180 mm, width – 52 mm.

PACKAGE CONTENTS OF THE MEDICAL DEVICE**

The medical device is packed in a secondary / consumer package of 3 pcs. a pack or 12 pcs. a pack. It is accompanied by the instruction for consumers that can be enclosed in the package or printed in the inside and / or outside of the package.

***quantity of condoms is presented in the outer side of the secondary/consumer packaging, the instruction for consumers is printed in the inner side of the secondary/consumer packaging*

LABELLING INFORMATION

The following information is labelled on the primary packaging:

- trade mark;
- LOT number;
- expiry date.

The following mandatory information is labelled in the outer and inner sides of the secondary packaging:

- the name of the medical device
- quantity of condoms in consumer package
- description of condom
- nominal length and width of condom
- indication that the condom is lubricated
- indication that the lubricant does not contain spermicide
- storage conditions
- information about manufacturer and authorized representative of manufacturer
- information about manufacturing site
- batch number (LOT)
- expire date (EXP)
- instruction for use

Symbols used for labelling and what they mean:

Symbol	Interpretation
	Use by/Expiry date
	Lot number
	Contains latex
	For single use only
	Read instruction before use

Additional information can be printed on the outer surface of the secondary/ consumer package including reference to international standards.

INDICATIONS

- prevention of unwanted pregnancy
- protect from HIV and other sexually transmitted diseases

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition.

PRECAUTIONS FOR USE

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. Don't use condom if its expiration date has expired. Do not use a condom if its individual package is broken.

If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Use a lubricant that is recommended for use with condoms.

Petroleum- or oil-based lubricants (petroleum jelly, baby oil and some pessaries) and some topical medicines can damage condoms.

Stop use if you feel discomfort or irritation while using the condom

Consult a specialist:

- about compatibility of condoms with other contraceptives
- if you are using topical medicines
- if irritation /discomfort persist after use
- if a condom leaks or bursts during use (do this as soon as possible and within 72 hours)

For anal sex or it is necessary use additional lubrication on the outside of condom.

Condoms may present a choking hazard. Keep out of reach of children.

If several types of sex (oral, vaginal or anal) are practiced during intercourse - it is recommended to use a new condom for each type of sex.

Condoms are for single use only.

No contraception method can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

POSSIBLE SIDE EFFECTS

Condom are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if user is allergic to latex.

APPLICATION

Carefully extract the condom from individual packaging by tearing the package at the toothed edge. Make sure that condom is not inside out.

squeeze the teat of the end of condom and place the condom on the top of the erect penis before sexual intercourse. Roll down to its base. If the condom doesn't unroll, it may be on backwards, discard and use a new one.

If a condom compresses or slips, a different size of condom should be used, as this can cause damage to the condom. Once the intercourse is over, remove the penis by holding the condom tightly at its base, gently remove the condom and throw into a disposal bin. Do not put the used condom down the toilet. For anal sex or in case if it is necessary, use an additional lubricant.

STORAGE CONDITIONS

Condoms should be stored in packed form, at temperature from 0 to 25 ° C and relative air humidity not higher than 80% at not less than 1 m from heating appliances. During storage, condoms should be protected from direct sun light and not exposed to petroleum products, phenols and derivatives thereof, oils of petroleum origin, gasoline, kerosene and other organic substances.

SHELF LIFE

Shelf life - 5 years

INFORMATION ABOUT MEDICINAL PRODUCTS AND PHARMACEUTICAL SUBSTANCES

Device does not contain medical products and pharmaceutical substances.

LIST OF MATERIALS OF ANIMAL AND (OR) HUMAN ORIGIN***

Compounded natural rubber latex contains casein ($0.022 \pm 10\%$ of contents of it, functions as a stabilizer). Casein (milk protein) is a natural by-product of milk. It has no known toxic effects although it may cause a reaction in individuals with an allergy to milk. The exposure from the condoms is considered not to pose any additional risk to users of the condoms.

The device does not contain any materials of human origin

****warning about possible side effect in the form of the allergic reaction because of the natural latex contained in the medical device composition is presented in the inner side of the secondary/consumer package*

USAGE WITH OTHER MEDICAL DEVICES****

Condoms can be used on their own to prevent from STIs and pregnancy and aren't intended to be used in conjunction with other medical devices.

However, consumers can use condoms with condom compatible lubricants if needed.

*****recommendation to use compatible with condoms lubricant when it is necessary is presented in the inner side of the secondary/consumer packaging*

LIST OF APPLIED STANDARDS

ISO 13485

ISO 14971

ISO 4074

SOFTWARE DESIGN, DEVELOPMENT AND VALIDATION

Not applicable

MAINTENANCE AND REPAIR

Not applicable

GUIDELINES FOR DESTRUCTION AND DISPOSAL

The devices should be used in accordance with the instruction for use. Disposed of together with household waste. Do not put the used condom in the sewer. In all other cases, the condoms are disposed of in accordance with local requirements.

ENVIRONMENT SAFETY REQUIREMENTS

With proper usage, storage and disposal Reflex natural rubber latex lubricated condoms have no negative impact on environment.

MANUFACTURER WARRANTY

The manufacturer guarantees compliance with the quality of the medical product with compliance with the requirements for transportation, storage and application of the product specified in the technical documentation.

For all issues related to the circulation of a medical product in the territory of the Russian Federation, it is necessary to contact the Authorized Representative of the manufacturer:

Reckitt Benckiser Healthcare LLC,
Floor 3, 4 Shlyuzavaya emb., Moscow, 115114, Russia.

Information
to be printed in the inner side
of the secondary/ consumer package

Please, read the instruction carefully before use.

PURPOSE/ INDICATIONS

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition

POSSIBLE SIDE EFFECTS

Condom are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if user is allergic to latex.

PRECAUTIONS FOR USE

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. Don't use condom if its expiration date has expired. Do not use a condom if its individual package is broken.

If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Use a lubricant that is recommended for use with condoms.

Petroleum- or oil-based lubricants (petroleum jelly, baby oil and some pessaries) and some topical medicines can damage condoms.

Stop use if you feel discomfort or irritation while using the condom

Consult a specialist:

- about compatibility of condoms with other contraceptives
- if you are using topical medicines
- if irritation /discomfort persist after use
- if a condom leaks or bursts during use (do this as soon as possible and within 72 hours)

For anal sex or it is necessary use additional lubrication on the outside of condom

Condoms may present a choking hazard. Keep out of reach of children.

If several types of sex (oral, vaginal or anal) are practiced during intercourse - it is recommended to use a new condom for each type of sex

Condoms are for single use only.

No contraception method can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

APPLICATION *(the scheme of application could be presented in the pack)*

- 1) Carefully extract the condom from individual packaging by tearing the package at the toothed edge. Make sure that condom is not inside out.
- 2) Squeeze the teat of the end of condom and place the condom on the top of the erect penis before sexual intercourse. Roll down to its base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. If a condom compresses, or slips, a different size of condom should be used, as this can cause damage to the condom.
- 3) Once the intercourse is over, remove the penis by holding the condom tightly at its base, gently remove the condom.
- 4) Throw the condom into a disposal bin. Do not put the used condom down the toilet.

For anal sex or in case if it is necessary, use an additional lubricant.

Date of the latest revision of the instruction:

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