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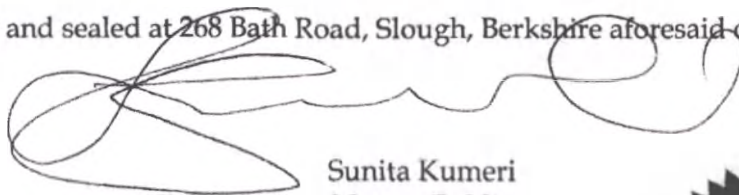
BE IT KNOWN that I, Sunita Kumeri of, 268 Bath Road, Slough, Berkshire United Kingdom a duly authorised Notary Public

CERTIFY ONLY that Brian Michael Howes who is well known to me and who is duly authorised by Reckitt Benckiser Healthcare (UK) Ltd ("the Company") to represent them in this matter has today caused the annexed Maintenance Documentation of the Medical Device Durex® Invisible Stimulation Natural Rubber Latex Condoms to be produced to me and has represented to me on behalf of the company that the said document is a true copy of the original electronic document produced to him.

SIGNED and sealed at 268 Bath Road, Slough, Berkshire aforesaid on 16th July 2020.

Sunita Kumeri
Notary Public
England and Wales

Protocol No. 1/20

Approved by:

Andrew Brownhill

Andrew Brownhill (Jun 11, 2020 13:58 GMT+1;

Andrew Brownhill

Global Regulatory Affairs Manager

Date: Jun 11, 2020

MAINTENANCE DOCUMENTATION OF THE MEDICAL DEVICE

Durex® Invisible Stimulation natural rubber latex condoms

Reckitt Benckiser Healthcare (UK) Ltd., UK

NAME OF THE MEDICAL DEVICE

Durex Invisible Stimulation natural rubber latex condoms

MANUFACTURER

Reckitt Benckiser Healthcare (UK) Ltd.

Registered address: 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK.

Phone: +44 (0) 1482 582438

MANUFACTURING SITE

Qingdao London Durex Co Ltd., No 1-13 Shangma Part, Aodong Road, High-tech Industrial Development Zone, Qingdao City, Shandong Province, P. R. China.

AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

Reckitt Benckiser Healthcare LLC, Floor 3, 4 Shlyuzavaya emb., Moscow, 115114, Russia.

Phone: +7 (495) 961 2565; Fax: +7 (495) 961 2566.

PURPOSE OF THE MEDICAL DEVICE

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

POTENTIAL CONSUMERS OF THE MEDICAL DEVICE

Medical Device is intended to be used by laypersons. Not for clinical use.

MEDICAL DEVICE OVERVIEW (DESCRIPTION)

The condoms are manufactured from natural latex, smooth, colourless, straight-walled.

TECHNICAL CHARACTERISTICS

Durex Invisible Stimulation	
Technical parameter	Specification
Condom length, mm	180 ± 10
Condom width, mm (measured at 30 mm from the bead)	52 ± 2
Teat length, mm	16 ± 7
Thickness, mm	0.043 - 0.052
Bead thickness, mm	1-2
Condom weight (after removed from the primary package), g	1.20 ± 0.5
Condom weight in primary package, g	2.0 ± 0.5
Elongation, %	Min 700

Durex Invisible Stimulation	
Technical parameter	Specification
Elongation, % (aged)	Min 600
Freedom from holes	Impermeable
Tensile strength, mPa	Min 20
Tensile strength, mPa (aged)	Min 17

LABELLING INFORMATION

Each individual package is labelled with:

- trademark;
- lot number (LOT);
- shelf life formatted as "Expiry date (EXP)".

Symbols used for labelling and what they mean:

	Use by/Expiry date
	Lot number

Additional information can be printed on the individual package including reference to international standards.

The following information is applied to the outer surface of the secondary/ consumer package:

- condom description (surface, colour, form, with teat/without teat);
- number of condoms in consumer package;
- nominal condom length;
- nominal condom width;
- indication if condom contains/does not contain lubricant;
- indication if the lubricant contains/does not contain spermicide;
- storage guidelines;
- name and address of the manufacturer;
- country of origin (if applicable);
- shelf life formatted as "Expiry date (EXP)".

Additional information can be printed on the outer surface of the secondary/ consumer package including reference to international standards.

Outer or inner surface of consumer packages and/or package insert contains prescribing information, which includes:

- reminder to handle the condoms carefully, including when removing them out of the package;
- indications, contraindications, precautions for use, possible side effects;
- method of application;
- necessity to remove condom right after sexual intercourse;
- necessity to exclude petroleum- and oil-based lubricants;
- necessity to use safe additional lubricant if needed;
- necessity to consult a specialist on compatibility with topical medications;

- disposal instructions for used condoms.

INDICATIONS

- prevention of unwanted pregnancy;
- protection from HIV and other sexually transmitted diseases.

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition.

Not suitable for anal sex.

PRECAUTIONS FOR USE

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Use the lubricant recommended for use with condoms. Petroleum- and oil-based lubricants (petroleum jelly, baby oil and some pessaries) and some topical medicines that are applied to the penis or vagina can damage condoms.

Stop use if you feel discomfort or irritation while using the condom.

Consult a specialist if:

- about the compatibility of condoms with other contraceptives;
- you are using topical medicines applied to penis or vagina;
- irritation/discomfort persists after use;
- you are unsure if you are allergic to latex;
- a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

If it is necessary use additional lubrication on the outside of the condom.

If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows.

Use a condom only once. Reuse may increase risk of failure or infection.

None of the contraception methods can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

POSSIBLE SIDE EFFECTS

Condoms are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if user is allergic to latex.

APPLICATION

Tear the package at the toothed edge and handle the condom carefully as it can be damaged by fingernails and sharp objects.

Make sure that the condom is not inside out. Having squeezed the teat place the condom on the top of the erect penis. Roll down to its base. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage.

Soon after ejaculation pull the penis out holding the condom firmly at the base of the penis and carefully take the condom off.

Dispose in a bin. Do not flush the condom down the toilet.

TRANSPORTATION

There are no specific conditions of transportation.

EXPLOITATION CONDITONS

There are no specific conditions required when the medical device is used for its purpose and as indicated in the labelling.

MEDICINAL OR PHARMACEUTICAL SUBSTANCE CONTAINED IN THE MEDICAL DEVICE

Not applicable.

LIST OF MATERIALS OF ANIMAL AND (OR) HUMAN ORIGIN

Compounded natural rubber latex contains casein (0,0152% of the contents of it, functions as a stabilizer) that is considered as a derivative from tissue/materials of animal origin as it is extracted from milk.

Casein (milk protein) is a natural by-product of milk. It has no known toxic effects although it may cause a reaction in individuals with an allergy to milk. The exposure from the condoms is considered not to pose any additional risk to users of the condoms.

The device does not contain any human blood derivatives.

STERILIZATION

The medical device is presented as a non-sterile device and is not intended to be sterilized thereafter.

USAGE WITH OTHER MEDICAL DEVICES

Condoms can be used on their own to prevent from STIs and pregnancy and aren't intended to be used in conjunction with other medical devices.

However, consumers can use condoms with condom compatible lubricants if needed.

INSTALLATION, MAINTENANCE, CALIBRATION

Not applicable.

CLEANING, DISINFECTION, STERILISATION OF THE MEDICAL DEVICE

Not applicable.

INTERNATIONAL STANDARDS

ISO 13485

ISO 14971

ISO 4074

PRECAUTIONS TO BE TAKEN BY USER RELATED TO THE DISPOSAL OF THE DEVICE

Durex Invisible Stimulation natural rubber latex condoms should be disposed in a bin. The condoms should not be flushed down the toilet.

DATE OF THE ISSUE / THE LATEST REVISION OF THE MAINTENANCE DOCUMENTATION

The medical device includes an indication of the latest revision within the instruction for use.

GUIDELIENSS FOR DESTRUCTION AND DISPOSAL

The devices should be used in accordance with the instructions for use.

In household use, the product is disposed of as household waste. Do not put it down the drain.

In all other cases, the condoms are disposed of in accordance with local requirements.

Approved by:
Andrew

ENVIRONMENT SAFETY REQUIREMENTS

With proper usage, storage and disposal Durex® condoms have no negative impact on the environment.

Approved by:

Andrew Brownhill

Andrew Brownhill (Jun 11, 2020 13:58 GMT+1)

Andrew Brownhill

Global Regulatory Affairs Manager

Date: Jun 11, 2020

Signature:

INSTRUCTION for CONSUMERS

Durex® Invisible Stimulation natural rubber latex condoms

Reckitt Benckiser Healthcare (UK) Ltd., UK

Read the instruction carefully before use, especially if condoms are used for oral or anal sex.

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections.

ATTENTION: Check the expiry date on the condom wrapper before you use it. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

PURPOSE/ INDICATIONS

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

APPLICATION

1. Tear the package at the toothed edge and handle the condom carefully as it can be damaged by fingernails and sharp objects.
2. Make sure that the condom is not inside out. Having squeezed the teat place the condom on the top of the erect penis.
3. Roll down to its base. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage.
Soon after ejaculation pull the penis out holding the condom firmly at the base of the penis and carefully take the condom off.
4. Dispose in a bin. Do not flush the condom down the toilet.

If it is necessary use additional lubrication on the outside of the condom.

If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows.

Use a condom only once. Reuse may increase risk of failure or infection.

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition.

Not suitable for anal sex.

POSSIBLE SIDE EFFECTS

Condoms are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if user is allergic to latex.

PRECAUTIONS FOR USE

Use the lubricant recommended for use with condoms. Petroleum- and oil-based lubricants (petroleum jelly, baby oil), some pessaries and some topical medicines that are applied to the penis or vagina can damage condoms.

Stop use if you feel discomfort or irritation while using the condom.

Consult a specialist if:

- about the compatibility of condoms with other contraceptives
- you are using topical medicines applied to penis or vagina;
- irritation/discomfort persists after use;
- you are unsure if you are allergic to latex;
- a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

None of the contraception methods can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

STORAGE AND SHELF LIFE *(could be printed in the secondary/ consumer package)*

Store the product in a dry place at 0-25 °C. Avoid direct sunlight.

Shelf life - 5 years.

MAIN TECHNICAL CHARACTERISTICS *(could be printed in the secondary/ consumer package)*

Nominal sizes: length – 180 mm, width – 52 mm;

Description: straight-walled, teat-ended, smooth, colourless, made of natural latex, with silicone lube containing stimulating agent, extra thin.

Date of the latest revision of the instruction:

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