un as pyhymul

Pfizer Consumer HealthCare
P. O. Box 26609
1211 Sherwood Avenue
Richmond, VA 23220

Rodger Hudson
Quality Systems Manager
Pfizer Global Supply

1 September 2016

For Use in Russia

To whom it may concern,

I, the undersigned, **Rodger Hudson, Quality Systems Manager**, Pfizer Global Supply attest to the fact that, to the best of my knowledge, the attached 7 page document is accurate and correct in describing **the Universal ThermoCare® HeatWraps for use on** different areas of the body.

01 Sep 2016

Rodger Hudson
Quality Systems Manager
PGS – Global Quality Operations

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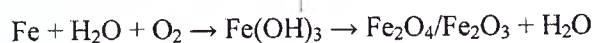
Description

The ThermaCare® HeatWraps is a portable,
convenient for use product for relief of pain
sensations in the region of the back, neck, knee,
elbow, wrist. Individual **HeatWraps** are firmly
fixed to the body and should remain at the site of
application and not move during use. They are a
family of disposable **HeatWraps** designed for
adjoining different anatomical parts of the body.

The products are manufactured with different
numbers of individual cells with a dry mixture
which generates heat during controlled
accelerated exothermic reaction of iron oxidation
which occurs immediately after opening the
package and contact with oxygen.

The reaction rate and the subsequent generation
of heat (40 °C) are determined by the rate of
oxygen penetration through the perforated upper
layer. The cells are firmly fixed in a thermal
insulating pack made from two polymer
materials. These different layers are covered with
a glue film. Discs are able of generating
controlled heat for up to 12 hours for temporary
relief of mild pains and unpleasant sensations in
the muscles and joints caused by overstrain,
sprains, dislocations, arthritis and muscular low
back pain. The reaction stops after oxidation of
all free iron or cessation of oxygen supply at
repeated rolling of the product.

Composition

Mixture of thermal cell (target value – 2.0 g/cell)

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Purpose

For relief of pain in the muscles, joints and tendons due to:

- overstrain,
- sprain,
- dislocation,
- arthritis.

In chronic or acute pain.

Size "Universal ThermaCare® HeatWraps for
different areas of the body": 188 mm x 97.3 mm

Universal ThermaCare® HeatWraps for different
areas of the body.

Only for external application!

Disposable.

One size.

How to use

1. Open the package with the HeatWraps just before use. Cells are heated within 30 min after the opening of the pack.
2. Apply the HeatWraps to the site of pain sensations with dark heating cell to the skin.
3. Fix the HeatWraps.

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Relief of pain without drugs for up to 8 hours!

- The ThermaCare[®] HeatWraps provides prolonged effect of pain relief without taking drugs due to local warming action.
- Prolong, uniform heat impact contributes to deep muscle relaxation, improvement of blood flow, acceleration of natural processes of recovery of the damaged tissues, relief of pain
- Heating of the HeatWraps cells begins immediately at opening the pack with the device
- Very simple and convenient for use: easy to put on and remove
- Thin, unseen under the clothing, does not hamper
- In contrast to warming ointments, is odourless and may be used during office hours
- For achieving the maximum effect it is recommended to wear the HeatWraps for 8 hours.

Patients with diabetes, a circulatory disorder, cardiovascular system disorder, rheumatoid arthritis and pregnant women must consult their doctor before using the device.

Do not use

- If the heating cell and/or pack are damaged
- Concomitantly with warming ointments or other warming agents
- On the damaged areas of the skin
- On open wounds, bruises occurring within the last 48 hours
- On the areas of the skin which lost sensitivity to heat
- For persons who cannot follow all available

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instructions or remove the device themselves, including children, newborns and some elderly people

- Do not use on the face or head, breasts, in the elbow bend or in the axillary region, on the fingers, in the lower abdomen, in the inguinal region, popliteal region, on the feet or the toes

Precautions

- If during the use of the HeatWraps sensation of excessive warmth occurs, it is necessary to stop the device's use or fix it above the underwear or fabric, avoiding direct contact with the skin.
- The use of the HeatWraps can produce burn. It is necessary to check regularly the skin condition during the use of the device. If you see irritation or burn on the skin, immediately remove the HeatWraps.
- The risk of burn increases with age. Persons over 55 years or older are recommended to wear the HeatWraps above the underwear or fabric and not to use it during sleep.
- Do not produce excessive pressure on the HeatWraps and do not wear the belt above the device.
- Do not use the HeatWraps for more than 8 hours during a day (within 24 hours).
- While using the HeatWraps avoid overlapping of the thermal cells.

Special warnings and precautions

- Each thermal cell contains iron (about 2 g). At accidental swallowing immediately see the doctor.
- At contact of iron with the skin or eyes rinse them with water, remove the HeatWraps and immediately see the doctor.
- Never heat the HeatWraps in a microwave oven and do not try to heat it again to use for the second time as the pack can inflame.

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- To avoid the increased risk of burns, the HeatWraps should be removed before conduct of MRT (magnetic resonance tomography).

Stop using the device and consult the doctor

- If you feel discomfort, burning sensation and also if swelling, rash or other changes of the **skin** appear during contact with the HeatWraps.
- If pain sensations remain the same or increase after 7 days of using the HeatWraps.

How supplied

A universal HeatWraps for different areas of the **body**.

One device in an individual vacuum pack. 3 packs with the device in a carton with the enclosed instruction on use.

Patch size: one-size-fits-all

Shelf life

3 years.

Do not use the HeatWraps after the expiration date printed on the package.

Storage

At room temperature.

Keep out of the reach of children and family **pets**.

How dispensed

Over-the-counter device, through the network of **pharmacies and specialized** departments of trade **networks**.

Terms of conveyance

The products should be transported in covered transport vehicles in accordance with the rules of transportation of freights with observation of **temperature regimen from minus 50°C to plus 50°C** (at 25°C with 80% relative humidity of air). Avoid mechanical impact and action of direct sun rays.

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SAFETY REQUIREMENTS

Keep out of the reach of children.

REQUIREMENTS REGARDING ENVIRONMENTAL DURING THE USE OF THE MEDICAL DEVICE

The medical device does not exert a negative effect on the human body and environment within its life cycle.

WARRANT OBLIGATIONS

The manufacturer ensures for the final consumer good functioning of the medical device within the shelf life.

GUIDELINES ON DISPOSAL OF THE MEDICAL DEVICE

To dispose the device in compliance with the requirements of the national legislation and also the regulations of the federal and regional authorities.

After the use or after the expiration date burn the device at the sites allocated for its disposal.

In private households dispose the device together with the domestic waste.

Manufacturer

Pfizer Consumer Healthcare, 1231 Wyandotte
Dr., Albany, Georgia, 31705, USA.

Send consumers' claims at the address:

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