Instruction for use

HEAT WRAPS

ThermaCare®

Lower Back & Hip

Name of the medical product

Heat Wraps ThermaCare® Lower Back & Hip

Composition:

Iron powder, water, sodium chloride, sodium thiosulfate, activated carbon, absorbent thickener.

Indications:

Relief of muscular, joint and ligament pains associated with:

- overexertion
- strains
- sprains
- arthritis.

For chronic and acute pain.

Action:

ThermaCare Heat Wraps promotes deep muscle relaxation, improved blood flow, pain relief.

Pain relief up to 16 hours without medicines!

- The ThermaCare heat wrap provides a long-lasting pain relief effect without the use of medicines due to the deep warming effect
- The effect of intense heat promotes deep muscle relaxation, improvement of blood flow, pain relief
- For maximum effect, it is recommended to wear a heat wrap for 8 hours.
- When using a heat wrap for the back, the pain relief effect lasts up to 16 (8 hours while you wear it and up to 8 hours after you take it off)
- Heating of the heat wrap cells begins immediately when opening the package with the heatwrap
- Very simple and easy to use: easy to put on and take off

- Thin, discreet under clothing, does not restrict movement
- Unlike warming ointments, it has no smell and can be used during the working day

Safety precautions:

- **Consult your healthcare professional** before use if you have diabetes, poor circulation, heart disease, rheumatoid arthritis, or are pregnant.
- If there is a feeling of excessive heat when using the heat wrap, it is necessary to stop using it or fix it on top of underwear or fabric, avoiding direct contact with the skin.
- Using a heat wrap may cause burns. It is necessary to regularly check the condition of the skin during the use of the heat wrap. If you find irritation or burn on the skin, you should immediately remove the heat wrap.
- Prior to use during sleep, it is necessary to use a heat wrap during the day to determine individual sensitivity to heat..
- The risk of getting burned increases with age. **If 55 and older, wear a heat wrap over a layer of clothing or cloth and do not wear a heat wrap while sleeping.**
- Do not place extra pressure over the product and do not wear under a waistband.
- Do not use more than 8 hours in a 24-hour period.

Do not use:

- If heat cell and /or wrap is damaged
- With warming ointments and with other forms of heat
- On damaged or broken skin
- On areas of bruising or swelling that have occurred in the last 48 hours
- On areas of the body where you can not feel heat
- On people unable to follow all the instructions or unable to remove the product, including children, infants and some elderly.

Directions for use:

For external use only! Single use.

1. Tear open pouch with a heat wrap when ready to use.
2. Place over pain area with darker heat cells toward skin.
3. Fix the heat wrap.

Stop use and consult a doctor:

- If you experience any discomfort, burning, swelling, rash or other changes in your skin that persist where the wrap is worn
- If after 7 days of using the heat wrap your pain gets worse or remains unchanged

Special warnings:

- Each heat cell contains iron (~2 grams). In case of accidental ingestion seek professional assistance immediately
- If it comes in contact with your skin or eyes, remove wrap, rinse affected area with water and seek professional assistance immediately
- Never heat product in a microwave or attempt to reheat, as wrap could catch fire
- To avoid increased risk of burns, remove product before undergoing any MRI (magnetic resonance imaging) procedures.

Storage: At room temperature. Keep out of the reach of children and pets.

Shelf life: 3 years Do not use the heat wrap after the expiration date indicated on the package

Applied standards

Applicable standards for Heat Wraps ThermaCare are indicated below.

MEDDEV 2.7-1 Rev.4: Clinical Evaluation: a guide for Manufacturers and Notified Bodies

MEDDEV 2.12-2 Rev.: 2 Post Market Clinical Follow-up studies

EN ISO 13485:2016 Medical devices - Quality management systems - Requirements for regulatory purposes (ISO 13485:2016)

EN ISO 14971:2019 Medical devices - Application of risk management to medical devices

EN 62366-1:2015 Medical devices - Application of usability engineering to medical devices

ISO 15223-1:2021 Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements.

EN 1041:2008 + A1:2013 Information supplied by the manufacturer with medical devices

ISO 10993-1:2018 Biological evaluation of medical devices — Part 1: Evaluation and testing within a risk management process

ISO 10993-5:2009 Biological evaluation of medical devices — Part 5: Tests for in vitro cytotoxicity

ISO 10993-10:2010 Biological evaluation of medical devices — Part 10: Tests for irritation and skin sensitization

ISO 10993-23:2021 Biological evaluation of medical devices — Part 23: Tests for irritation

ISO 14155 2020 Clinical Investigation Of Medical Devices For Human Subjects - Good Clinical Practice

ASTM F1886 Standard Test Method for Determining Integrity of Seals For Flexible Packaging By Visual Inspection

Disposal requirements: dispose the device in compliance with the requirements of the national legislation and the regulations of the national authorities.

After the use or after the expiration date, burn the device at the sites for its disposal. In private household dispose together with domestic waste (SanPiN 2.1.3684-21 Medical waste class – A).

Manufacturer: Angelini Pharma Inc., 9200, Corporate Blvd., Suite 100, Rockville, MD 20850, USA.

Manufacturing site: Angelini Pharma Inc., 1231 Wyandotte Drive, Albany, GA, 31705, USA.

Authorized representative of the manufacturer/ importer: Angelini Pharma Rus LLC, Russia, 123001, Moscow, Trekhpudny pereulok, 9, bld. 2, tel.: +7 (495) 933 3950, fax: +7 (495) 933 3951292, e-mail: complaints@angelini.ru.

Consumer claims should be sent to the importer.

The heatwrap dimensions are 1109 mm x 145 mm.

2 disposable heat wraps. In the form of a soft belt. Universal size.

Symbols used on the label:

- Do not reuse

- Manufacture

- European conformity


- Batch code

- Use by date

Rebecca
Ethridge

Digitally signed by Rebecca
Ethridge
Date: 2023.09.22 08:11:59
-04'00'

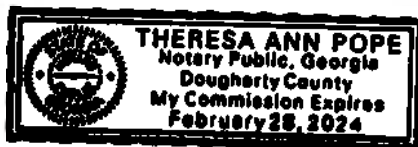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Instruction for use**HEAT WRAPS****ThermaCare®****Neck, Shoulder and Wrist****Name of the medical product**

Heat Wraps ThermaCare® Neck, Shoulder and Wrist

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Pain relief up to 12 hours without medicines!

- The ThermaCare heat wrap provides a long-lasting pain relief effect without the use of medicines due to the deep warming effect.
- The effect of intense heat promotes deep muscle relaxation, improvement of blood flow, pain relief.
- For maximum effect, it is recommended to wear a heat wrap for 12 hours.
- Heating of the heat wrap cells begins immediately when opening the package with the heat wrap.
- Very simple and easy to use
- Thin, discreet under clothing, does not restrict movement.
- Unlike warming ointments, it has no smell and can be used during the working day.

Safety precautions:

- **Consult your healthcare professional** before use if you have diabetes, poor circulation, heart disease, rheumatoid arthritis, or are pregnant.
- If there is a feeling of excessive heat when using the heat wrap, it is necessary to stop using it or fix it on top of underwear or fabric, avoiding direct contact with the skin.
- Using a heat wrap may cause burns. It is necessary to regularly check the condition of the skin during the use of the heat wrap. If you find irritation or burn on the skin, you should immediately remove the heat wrap.
- When applying to the wrist do not overlap the heat cells.
- Prior to use during sleep, it is necessary to use a heat wrap during the day to determine individual sensitivity to heat.
- The risk of getting burned increases with age. **If 55 and older do not wear a heat wrap while sleeping.**
- Do not place extra pressure over the product
- Do not use more than 12 hours in a 24-hour period.

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Directions for use:

For external use only! Single use.

1. Tear open pouch with a heat wrap when ready to use.
2. Peel away protective film.
3. Place over pain area with darker heat cells toward skin.
4. Fix the heat wrap.

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Special warnings:

- Each heat cell contains iron (~2 grams). In case of accidental ingestion seek professional assistance immediately
- If it comes in contact with your skin or eyes, remove wrap, rinse affected area with water and seek professional assistance immediately
- Never heat product in a microwave or attempt to reheat, as wrap could catch fire
- Do not use the ThermoCare after the expiration date indicated on the package.
- To avoid increased risk of burns, remove product before undergoing any MRI (magnetic resonance imaging) procedures

Storage: At room temperature. Keep out of the reach of children and pets

Shelf life: 3 years.

Applied standards

Applicable standards for Heat Wraps ThermoCare are indicated below.

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- Use by date

Rebecca Ethridge

Digitally signed by Rebecca
Ethridge

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