22

I, Sarah McLoughlin, a notary public authorised to practise throughout England and Wales, with an office The Friarage 25a Rickford's Hill Aylesbury HP20 2RT England, CERTIFY AS FOLLOWS:

1. Intersurgical Limited is a private company limited by shares registered under the Companies Acts 1948 to 1976 by the Registrar of Companies for England and Wales under number 01488409, with registered office at Crane House Molly Millars Lane, Wokingham, Berkshire RG41 2RZ, England (the "Company");
2. The attached document entitled "Instructions for Use" and relating to "Mask anaesthetic facial ClearLite, in versions" dated 23 June 2020 bears the true signature of Ivan Seniut.

ISSUED under my seal of office and signed by me in Aylesbury, England, on this third day of July two thousand and twenty.

Sarah McLoughlin

Sarah McLoughlin
Notary Public
25a Rickford's Hill
Aylesbury Bucks HP20 2RT
07789 623052

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INSTRUCTION FOR USE
for Medical Device:

Mask anesthetic facial ClearLite, in versions
Manufacturer: Intersurgical Ltd., UK

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____ Intersurgical Ltd _____
Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK

Group Quality and Regulatory Affairs Director _____
Ivan Seniut _____
23.06.2020 _____

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INTERSURGICAL LIMITED
Crane House, Molly Millars Lane
Wokingham, Berkshire, RG41 2RZ
www.intersurgical.com info@intersurgical.com

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2.2. Intended use of the medical device

ClearLite anesthetic facial mask with many design options is designed to deliver anesthetic gases into patient (during the anesthesia administration and maintenance period), as well as to provide manual ventilation (during resuscitation, for example)

2.2.1 Description of the medical device

ClearLite anesthetic face mask with many design options is a flexible molded product of an inversed (face fitting) anatomical shape with an uninflatable curved flat cuff along the edge of the mask. The mask body has a double-side contractions for better finger gripping. In the (top view) projection, the mask has a shape of a smooth oval with the nose-area protrusion. The mask architecture is optimized in regards to the minimum priming volume, or dead space. On top of the mask there is a 15M (15 mm diameter) or 22F (22 mm diameter) connector, depending on its size, and ISO 5356 cone-type connectors. The mask body is transparent. The mask can be supplied with or without the mask holder ring based on the appropriate sizes (Fig.1). The mask holder ring has four retaining pins. The coupling size (inner diameter) of the mask holder ring is 22 mm. The cuff is profiled, with an extension to the nose area. In the nose area, the cuff is transversely reinforced with the ribs made of the cuff material to ensure a tighter fitting in this area. The mask body is marked with the manufacturer's brand name and size. To range the masks by cuff size, a color-coded indication is used (Fig.1).

The device is non-sterile and of single use. Each mask is individually packaged. The product in a package is clinically clean, produced and packaged in clean rooms and in controlled environment. The Intersurgical production facilities are certified under ISO 9001, ISO 13485 and Directive 93/42/EEC.

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2.3. Indication for use

The need to provide ventilatory support during anesthesia (induction of anesthesia), non-invasive ventilation of the lungs, emergency situations, etc. Mask anesthesia is used for low-traumatic, short-term operations that do not require muscle relaxation. Mask anesthesia is performed saving spontaneous breathing of patient.

2.4. Contraindications

It can not be used in case of respiratory disorders and disorders of other vital organs. During mask anesthesia, careful monitoring of the patient's condition is required.

2.5. Scope of application of the medical device

Anesthesiology, surgery, intensive care, emergency care, otolaryngology.

2.6. Conditions of use

The medical device is used in medical institutions.

2.7. Possible side effects when using a medical device

When using masks, there is a risk of obstruction of upper airway (tongue retraction, vomit aspiration, laryngospasm), as well as the possibility of respiratory depression.

The use efficiency of anesthetic masks depends on their correct size selection.

2.8. Potential user

Qualified medical personnel.

3. PRINCIPLES OF MEDICAL DEVICE OPERATION AND

ClearLite anesthetic face mask with many design options is connected to instrumental respiratory system as the endpoint interface for patient during inhalation anesthesia and (or) artificial lung ventilation (ALV). The mask is applied to the patient's face. The mask is oronasal. To supply breathing gas, a breathing circuit or resuscitation breathing bag is connected to the mask. By using the mask, it is possible to provide non-invasive ventilation of the patient during anesthesia and ALV. After applying the mask to the patient's face, providing the correct size is chosen, the space under the mask shall be air-tight sealed. This can be achieved due to the cuff deformation when the mask is pressed down. Thank to the improved anatomical design of the body and cuff shape, ClearLite anesthetic face mask provides a better seal with a minimal pressure on the facial tissue, reducing the risk of unnecessary discomfort or potential injury. Its high airtightness allows the Clinician to provide, among other things, optimal manual ventilation of the patient both during anesthesia and ALV, as well as in emergency situations. The mask design is optimized so as to reduce the priming volume, or dead space, for the reduction of

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1. Mask anesthetic facial ClearLite, for newborns, 15M, size 0 (cuff color-blue);
2. Mask anesthetic facial ClearLite, for infants, 15M, size 1 (cuff color-gray);
3. Mask anesthetic facial ClearLite, for children, with ring, 22F, size 2 (cuff color-white);
4. Mask anesthetic facial ClearLite, for children, without ring, 22F, size 2 (cuff color-white);
5. Mask anesthetic facial ClearLite, for adults small, with ring, 22F, size 3 (cuff color-yellow);
6. Mask anesthetic facial ClearLite, for adults small, without ring, 22F, size 3 (cuff color-yellow);
7. Mask anesthetic facial ClearLite, adult medium, with ring, 22F, size 4 (cuff color-green);
8. Mask anesthetic facial ClearLite, adult medium, without ring, 22F, size 4 (cuff color-green);
9. Mask anesthetic facial ClearLite, adult large, with ring, 22F, size 5 (cuff color-orange);
10. Mask anesthetic facial ClearLite, adult large, without ring, 22F, size 5 (cuff color-orange);
11. Mask anesthetic facial ClearLite, for adults very large, with ring, 22F, size 6 (cuff color-red);
12. Mask anesthetic facial ClearLite, for adults very large, without ring, 22F, size 6 (cuff color-red).

5.2 The main components and materials for Medical device

Materials	Brand and manufacturer
Polypropylene (PP)	PURELL RP373R (7547A), Germany
Thermoplastic elastomer (TPE)	Evoprene™ LF 6158, Germany
Low Density Polyethylene (LDPE)	ICORENE 1613, USA
Blue pigment	Riverdale Global, Blue pigment, USA
Gray pigment	Riverdale Global, Grey pigment, USA
Yellow pigment	Riverdale Global, Yellow pigment, USA
Dark Green pigment	Riverdale Global, Dark-green pigment, USA
Orange pigment	Riverdale Global, Orange pigment, USA
Red pigment	Riverdale Global, Red pigment, USA

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Mask anesthetic facial ClearLite are IVF products. In their manufacture, polyvinyl chloride (PVC) plastics and phthalates are not used, which significantly minimizes the possible harmful effects on the patient and the environment.

5.3. Key parameters and characteristics of the medical device

Name of MD	Mask anesthetic facial ClearLite, for newborns, 15M, size 0	Mask anesthetic facial ClearLite, for infants, 15M, size 1
Parameter MD		
Patient size	Neonate	Infant
Size	0	1
Hook ring	No	No
Colour	Blue	Grey
Approximate fill volume (ml)	39	56
Length (mm)	62.8	79.4
Width (mm)	58.1	59.5
Weight (g)	6.19	7.28
Connection	15M	15M

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

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Name of MD	Mask anesthetic facial ClearLite, for children, with ring, 22F, size 2	Mask anesthetic facial ClearLite, for children, without ring, 22F, size 2
Parameter MD		
Patient size	Paediatric	
Size	2	
Hook ring	Yes	No
Colour	White	
Approximate fill volume (ml)	80	
Length (mm)	93.7	
Width (mm)	70.5	
Weight (g)	10.61	
Connection	22F	

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

Name of MD	Mask anesthetic facial ClearLite, for adults small, with ring, 22F, size 3	Mask anesthetic facial ClearLite, for adults small, without ring, 22F, size 3
Parameter MD		
Patient size	Small Adult	
Size	3	
Hook ring	Yes	No
Colour	Yellow	
Approximate fill volume (ml)	112	
Length (mm)	107.9	
Width (mm)	81.4	
Weight (g)	13.96	
Connection	22F	

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

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Name of MD	Mask anesthetic facial ClearLite, adult medium, with ring, 22F, size 4	Mask anesthetic facial ClearLite, adult medium, without ring, 22F, size 4
Parameter MD		
Patient size	Medium Adult	
Size	4	
Hook ring	Yes	No
Colour	Dark Green	
Approximate fill volume (ml)	123	
Length (mm)	115.4	
Width (mm)	87.0	
Weight (g)	16.96	
Connection	22F	

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

Name of MD	Mask anesthetic facial ClearLite, adult large, with ring, 22F, size 5	Mask anesthetic facial ClearLite, adult large, without ring, 22F, size 5
Parameter MD		
Patient size	Large Adult	
Size	5	
Hook ring	Yes	No
Colour	Orange	
Approximate fill volume (ml)	178	
Length (mm)	123,0	
Width (mm)	92,8	
Weight (g)	19,21	
Connection	22F	

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

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Name of MD	Mask anesthetic facial ClearLite, for adults very large, with ring, 22F, size 6	Mask anesthetic facial ClearLite, for adults very large, without ring, 22F, size 6
Parameter MD		
Patient size	Extra Large Adult	
Size	6	
Hook ring	Yes	No
Colour	Red	
Approximate fill volume (ml)	216	
Length (mm)	134.6	
Width (mm)	101.3	
Weight (g)	22.31	
Connection	22F	

Numerical parameter values are nominal values. Limit deviations shall not exceed $\pm 10\%$.

5.4. Information about compatible medical devices

ClearLite anaesthetic face masks can be connected to respiratory circuit directly or via additional connectors. The configuration may include virus and bacterial filters, heat and moisture exchangers, which can also be connected directly to the mask. Oral Guedel airways can be used to provide non-invasive ventilation by using ClearLite anaesthetic face mask.

The connecting dimensions of the respiratory circuit elements to be connected to the mask and the mask connector shall comply with the requirements of ISO5356-1:2015 standard "Anaesthetic and respiratory equipment". Conical connectors. Part 1. Cones and sockets".

6. MEDICAL DEVICE LABELING AND PACKAGING

Scope of supply for the medical device

The medical device is delivered in individual packaging. Its transportation is performed in the cardboard shipping boxes made of thick corrugated cardboard. The shipping boxes have an additional information on the number of units in the

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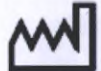
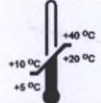
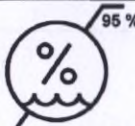
box.
Instructions for use are supplied with the multi-unit package of devices.

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6.2 Packaging Details

The medical device is supplied in individual packaging made of polymer material. Its transportation is performed in the cardboard shipping boxes made of thick corrugated cardboard. The shipping boxes have an additional information on the number of units in the box.

The materials used for packaging does not have harmful effects on its contents. The package design and material have such their corresponding properties in order to:

- ensure the safety of the device when it is stored in dry, clean conditions;
- minimize the risk of contamination of its contents during the opening procedure.

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Data on the packaging of a medical device (according to the decision of the manufacturer, the number of products in the transport box can be changed):

Version of the medical device	Quantity of pcs in the transport box	Pallet quantity pcs
Mask anesthetic facial ClearLite, for newborns, 15M, size 0;	40	2560
Mask anesthetic facial ClearLite, for infants, 15M, size 1;	30	1920
Mask anesthetic facial ClearLite, for children, with ring, 22F, size 2;	25	1600
Mask anesthetic facial ClearLite, for children, without ring, 22F, size 2;	25	1600
Mask anesthetic facial ClearLite, for adults small, with ring, 22F, size 3	35	1120
Mask anesthetic facial ClearLite, for adults small, without ring, 22F, size 3	35	1120
Mask anesthetic facial ClearLite, adult medium, with ring, 22F, size 4	35	1120
Mask anesthetic facial ClearLite, adult medium, without ring, 22F, size 4	35	1120
Mask anesthetic facial ClearLite, adult large, with ring, 22F, size 5	35	1120
Mask anesthetic facial ClearLite, adult large, without ring, 22F, size 5	35	1120
Mask anesthetic facial ClearLite, for adults very large, with ring, 22F, size 6	20	640

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In case of any questions related to the quality of the medical device, please contact the Authorized Representative of the Manufacturer in the Russian Federation at the following address:

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