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APPROVED

SIGVARIS AG

Gröblistrasse 8, 9014 St.Gallen, Switzerland

INSTRUCTION FOR USE

in accordance with the order of the Ministry of Health № 11N

Medical compression products "SIGVARIS" in versions:

- anti-embolic stockings THROMBO X (THROMBO X);
- compression socks JAMES (JAMES);
- compression socks (diabetic) (DIABETIC COMPRESSION SOCKS).

MANUFACTURER

SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

PRODUCTION SITE

JAMES:
SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

DIABETIC COMPRESSION SOCKS

SIGVARIS INC.
1119 Highway 74 South
Peachtree City, GA 30269
USA

THROMBO X

Sanyleg
Via Albania 1/3
46042 Castel Goffredo MN, Italien

1 PRODUCT NAME

Medical compression products "SIGVARIS" in versions:

- anti-embolic stockings **THROMBO X (THROMBO X);**
- compression socks **JAMES (JAMES);**
- compression socks (diabetic) **(DIABETIC COMPRESSION SOCKS).**

2 MANUFACTURER

SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

2.1 PRODUCTION SITE

JAMES:
SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

DIABETIC COMPRESSION SOCKS
SIGVARIS INC.
1119 Highway 74 South
Peachtree City, GA 30269
USA

THROMBO X
Sanyleg
Via Albania 1/3
46042 Castel Goffredo MN, Italien

3 Application pf the medical product

Medical compression products "SIGVARIS" in version:

- anti-embolic stockings THROMBO X (THROMBO X)

Degressive pressure profile and compression of ~18 mmHg around the ankle region

Size identification in sole area

Closed toe with an inspection hole

Skin friendly and breathable

can be reused

-compression socks JAMES (JAMES)

prevention and treatment of varicose veins, to create a graduated pressure on the vessels of the limbs (bandage effect), is based on a simple and effective mechanical principle, which is the use of elastic material. In this case, the vessels narrow, what accelerates the movement of blood, reduces the permeability of their walls, reducing swelling and stagnation of blood. As a result, the limbs are less tired, sore and swollen, improves health and reduces the risk of various complications.

compression socks (diabetic) (DIABETIC COMPRESSION SOCKS)

Specialized compression socks (18-25mmHg) for diabetics for whom require compression therapy. DIABETIC COMPRESSION SOCKS should only be used under medical supervision.

4 Indications, contraindications

Indications:

4.1. compression socks JAMES (JAMES)

- Functional disorders such as heaviness and fatigue in the legs, without existing varicosis (C0)s
- Mild varices with subjective manifestations (small varicose dilatation of the cutaneous veins, reticular varices) (C1–C2)s
- Varicosis during pregnancy, without edema (C2)s
- Varicosis during pregnancy, with edema (C3)s

- Edema in the evening, due to local causes (venous)
- (C3)s
- Primary varicosis with edema (C3)s
- Advanced primary varicosis with edema (stem and branch varices) (C3)s
- Chronic venous insufficiency (CVI)s
- Grade I* (C3)s corona phlebectatica paraplantis
- Grade II* (C4)s chronic edema**, eczema**, erythema**, hypodermatitis**, dermatosclerosis dermite ocre, atrophie blanche
- Grade IIIa* (C5)s healed ulcer cruris (relapse prophylaxis)
- * According to Widmer & Marshall
- ** After completion of basic therapy
- Prophylaxis of thrombosis/embolism
- Thrombophlebitis or varicothrombosis (superficial thrombosis), possibly with edema
- Thrombophlebitis (deep-vein thrombosis)
- Post-thrombotic edema with or without secondary follow-up therapy
- To maintain success of therapy:
 - a) after sclerotherapy; b) after ambulant selective varicose surgery; c) after stripping
- Before sclerotherapy or before surgery to prevent edema
- Ulcer prevention and ulcer therapy
- Postoperative edema
- Post-traumatic edema after completion of basic therapy
- Reversible lymphedema after complex physical decongestant therapy
- Irreversible lymphedema with pronounced induration after complex physical decongestant therapy
- Lipedema grade II and higher
- Cyclic idiopathic edema
- Congestion caused by immobility (arthrogenic congestion syndrome, paresis and partial paresis of the extremities)
- Angiodysplasia
- *According to CEAP classification

4.2 - anti-embolic stockings THROMBO X (THROMBO X)

All situations that increase the risk of thrombosis:

- During and after surgery (especially abdominal-pelvic and orthopedic surgery)
- Fractures or injuries of the lower limbs
- Impaired mobility

Anti-embolism stockings must only be used by patients when lying down!

4.3 - compression socks (diabetic) (DIABETIC COMPRESSION SOCKS)

- 1 Put on the foot slip (for all models with open toe).
- 2 Ease it across the heel using rubber gloves. 3/4 Do not pull at the top of the stocking.
- 3/4 Using the rubber gloves, spread the fabric evenly over the leg with the flat of the hand, working upwards towards the knee. With the thigh stocking, the same movement should be continued upwards towards the thigh. The foot part of the stocking should be placed at the base of the toes. The toes should remain free (for all models with open toe). The foot part has to be properly in place.
- 5 Pull out the foot slip.

Correctly worn SIGVARIS compression stockings:

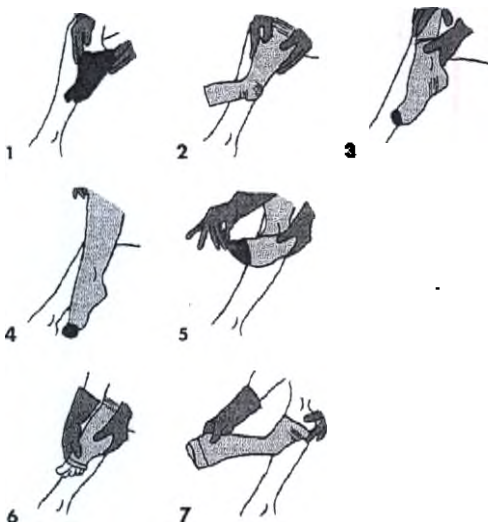
- The calf-length stocking fits correctly when the upper end is right under the knee.
- Thigh-length stockings: the rubber part must reach the groin.

6.2 To remove the stockings:

Removing the stockings is made much easier by also using rubber gloves. Take hold of the stocking at the upper edge and pull downwards towards the heel. With the flat of the hand, work the stocking over the heel and remove.

Important:

Sharp fingernails, rings and bracelets can cause defects. Avoid these dangers when putting the stockings on and removing them by always wearing rubber gloves! Do not pull or cut loose ends which sometimes appear on the inside of the stockings. The materials used in manufacturing SIGVARIS are synthetic elastic fibers (elastane). Ointments, oil, grease and direct sunlight will rarely harm the fabric. Regular checks by a doctor are necessary to maintain the success of the therapy. Extremely infrequently, compression stockings may produce allergic reactions. If acute pain or skin irritation occur while wearing the stockings, consult your doctor immediately.

Important information

- If recommended by a doctor and fitted correctly, compression socks usually do not have any side-effects.
- In neuropathic patients, compression-induced pressure sores or impaired blood flow can go unnoticed. Therefore, we recommend to check the feet several times a day for pressure sores, signs of impaired blood flow or new skin lesions. If any of these are present, the socks must be removed. In addition, you should consult a doctor.

- Diabetics frequently suffer from arterial circulatory disorders. If you have ABI (ankle-brachial index) between 0.5 and 0.8, mild compression as with DIABETIC COMPRESSION SOCKS can be used after consulting a specialist.
- We recommend to only wear DIABETIC COMPRESSION SOCKS after consulting a specialist. Check-ups with a doctor are extremely important and mandatory.

Care instructions

- Wash the socks daily. They can be washed in the washing machine in a laundry bag at up to 95° Celsius.
- Use a mild detergent that does not contain any fabric conditioner.
- Do not tumble dry in order to avoid overheating and static electricity. Wrap the compression socks in two dry towels to remove excess water and dry in a shaded and airy place. Do not place on a heater.
- Do not iron.
- Do not dry-clean.
- Sharp fingernails and rings with sharp edges can easily cause damage. Please be sure to pay attention to this when putting on and removing the socks.

No liability can be accepted in the event of failure to observe these contraindications

7. Conditions of use

- SIGVARIS medical compression products should be used at a temperature of +50 to +270 C and a relative humidity of 0~50%, a pressure of 850 to 1060 HPa and at an altitude of <3000m above sea level.
Can be used on board aircraft.

8. Precautions when using medical stockings

- Do not wear medical compression socks on open wounds.
Use only after consultation with a specialist!

9. Possible side effects when using medical stockings

- Individual intolerance of materials used in the manufacture of medical products
- If used incorrectly, medical compression stockings can cause skin damage and poor arterial circulation.

10. The warranty for medical DIABETIC COMPRESSION stockings is 6 months when used properly.

In strict compliance with the instructions of the manufacturer side effects do not exist!

Sigvaris medical compression products in versions anti-embolic stockings THROMBO X (THROMBO X)	81 % polyamid 19 % elastane	 filament thread of polyamide 6.6, trade name "Timbrelle®", and of polyamide with antimicrobial impregnation 6.6, trade name "Timbrelle®- Hygenic", flat thread of polyamide 6.6 "TIMBRELLE®- EDY", thread of polyamide 6.6 "SMART DRY", thread of polyamide 6.6 "BLACKBRILLIANC E", thread of polyamide 6.6 "TIMBRELLE® FASHIONblend", thread of 6.6 "silktouch" polyamide thread, 6.6 "Timbrelle® PerformanZ" polyamide thread, 6.6 "GREEN FC-Free" polyamide thread, flat natural white Silk thread dye white brand 74dgc Dominik GEORG LUH technogرافit GmbH, Germany  the thread made of polyamide 6.6 "GREEN FC-FREE», flat natural silk thread beige dye: brand 86DGC Dominik Georg Luh TECHNOGRAFIT GmbH, Germany	Contact with intact skin up to 18 hours
--	---	---	---

		 polyamide thread 6.6 "GREEN FC-FREE", flat natural silk thread pink dye: brand 3478DGC Dominik Georg Luh TECHNOGRAFIT GmbH, Germany	
		 6.6 "GREEN FC- FREE" polyamide thread, flat natural silk thread blue color dye brand CI 76645, Nan Ya Plastic Corporation, Taiwan	
		 polyamide thread 6.6 "GREEN FC-FREE", flat natural silk thread yellow dye CIB 645, Nan Ya Plastic Corporation, Taiwan Knit structure: uniform knit, seamless hosiery, made of yarn with elastic inner thread, inner thread runs to the knee	

		Toe and heel structure Heel is knitted with in and out movement with with inspection hole (with a patch at the top) Seam type – No seams	
Medical compression products "SIGVARIS" in version: - compression socks JAMES	81 % Polyamide 19 % Elastan	filament thread made of polyamide 6.6 , trademark «Timbrelle®», and of antibacterial-impregnated polyamide 6.6, trademark «Timbrelle®-Hygenic», flat thread made of polyamide 6.6 , trademark «TIMBRELLE®-EDY», thread of polyamide 6.6 «SMART DRY», thread from polyamide 6.6 «BLACKBRILLIANCE», thread of polyamide 6.6 «TIMBRELLE® FASHIONblend», thread of polyamide 6.6 «SILKTOUCH», thread of polyamide 6.6	Contact with undamaged skin up to 18 hours

		«TIMBRELLE® PerformanZ», thread of polyamide 6.6 «GREEN FC-FREE», white flat natural silk thread, dye trademark: 74DGC Dominik Georg Luh TECHNOGRAFIT GmbH, Germany Knitting structure: homogeneous knitting, seamless hosiery, made of yarn with elastic inner thread, inner thread extends to the knee Toe and Heel structure Heel is knitted with in and out movement with inspection hole (with a patch at the top) Seams Type – No seams	
Medical compression products "SIGVARIS" in version:- compression socks JAMES	65% Polyester 11% Cotton 19% Nylon 5% Spandex	polyester – trademark Dewspo, nylon -70 DENIER, spandex - Spandex DuPont, cotton - Organic cotton, white (white dye, trademark CI 77891, Nan Ya Plastic Corporation, Taiwan) Knitting structure: homogeneous knitting, seamless hosiery, made from yarn with elastic inner thread, inner thread extends to the knee Toe and Heel structure	Contatct with undamaged skin up to 18 hours

Heel is knitted with in and out movement with closed toe


Seams type

No seams

Primary packaging	Polythene film	Polythene film PET/PE (35709-G)	Mediated contact
Secondary packaging	Reversible chalked folding cardboard	Reversible chalked folding cardboard FBB (Frovi white FBB)	Mediated contact

15.1. Medical compression products "SIGVARIS" in version: - compression socks JAMES:

Model	Feet form	Rim	Compression Index	Colour
A – D	Closed toe	Soft top	2	cyanide carbon

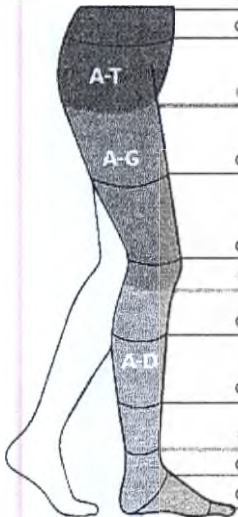Circumference in cm		XXS	XXS PLUS	XS	XS PLUS	S	S PLUS	M	M PLUS	L	L PLUS	XL	XL PLUS
	cT	60-75	65-80	65-80	75-90	75-90	85-100	85-100	95-110	95-110	105-120	105-120	115-130
	cH	75-90	80-95	80-95	85-100	85-100	95-110	95-110	105-120	105-120	115-130	115-130	125-140
	without grip top	42-54	47-59	47-59	52-64	52-64	56-68	56-68	61-74	61-74	66-79	66-79	73-85
	cG with grip top	42-52	47-56	47-56	52-61	52-61	56-65	56-65	61-71	61-71	66-76	66-76	73-85
	cF	38-49	41-52	41-52	44-55	44-55	49-60	49-60	53-64	53-64	59-70	59-70	63-78
	cE	29-36	33-40	33-40	37-44	37-44	39-46	39-46	41-48	41-48	45-52	45-52	46-53
	cD	23-30	27-34	27-34	31-38	31-38	34-41	34-41	37-44	37-44	41-48	41-48	42-50
	cC	25-32	29-36	29-36	33-40	33-40	35-42	35-42	38-45	38-45	42-49	42-49	43-51
	cB1	22-26	22-26	24-29	24-29	26-32	26-32	29-35	29-35	31-37	31-37	33-39	33-39
	cB2	18-20	18-20	20-22	20-22	22-24	22-24	24-26	24-26	26-29	26-29	29-32	29-32
	cY	27-31	27-31	28-33	28-33	29-35	29-35	31-37	31-37	32-38	32-38	33-39	33-39
	cA	18-20	18-20	20-22	20-22	22-24	22-24	24-26	24-26	26-29	26-29	29-32	29-32

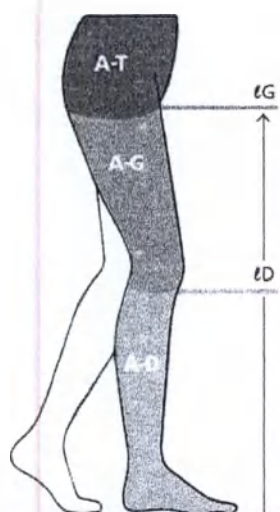
A-T = Pantyhose; A-G = Thigh; A-D = Calf

A-T = pantyhose, A-G = thigh-high stockings, A-D = stockings

Circumference in cm = Leg circumference in cm

Without grip top = soft top

With grip top = knobbed grip top

length in cm

Short*	31-34	Short	56-64	Short **	56-64
Normal	35-40	Normal	65-74	Normal	65-74
Long	41-45	Long	75-84	Long	75-84


* short length is available only in COTTON models with soft rim, CCL 2, in cyanide carbon and beige.


** short length is available only in COTTON models with soft rim, CCL 2 and 3, in beige.

15.2 ASSORTMENT THROMBO X


Modell	Foot	Rim	CCl.	colour
A - D	with inspection hole	soft top	~18 mmHg	white
A - G	with inspection hole	knobbed grip top	~18 mmHg	white
A - G	with inspection hole	Waist attachment	~18 mmHg	white

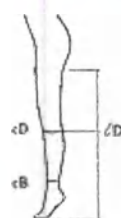Length - ℓ (cm)	small	medium	large	x-large
short 33 - 38				
normal 38 - 43	cB 18 - 22 cm	cB 22 - 26 cm	cB 26 - 29 cm	cB 29 - 32 cm
long 43 - 48	cD 29 - 35 cm	cD 34 - 40 cm	cD 39 - 45 cm	cD 44 - 51 cm

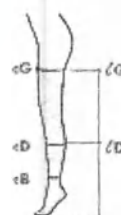Length - ℓ (cm)	small	medium	large	x-large
short 60 - 70	cB 18 - 22 cm	cB 22 - 26 cm	cB 26 - 29 cm	cB 29 - 32 cm
normal 70 - 80	cD 29 - 35 cm	cD 34 - 40 cm	cD 39 - 45 cm	cD 44 - 51 cm
long 80 - 90	cG 43 - 54 cm	cG 48 - 60 cm	cG 54 - 66 cm	cG 60 - 72 cm

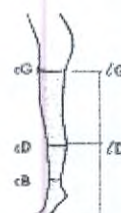
Length - ℓ (cm)	small	medium	large	x-large
short 60 - 70	cB 18 - 22 cm	cB 22 - 26 cm	cB 26 - 29 cm	cB 29 - 32 cm
normal 70 - 80	cD 29 - 35 cm	cD 34 - 40 cm	cD 39 - 45 cm	cD 44 - 51 cm
long 80 - 90	cG 43 - 54 cm	cG 48 - 60 cm	cG 54 - 66 cm	cG 60 - 72 cm

short
normal
long

15.3 ASSORTMENT DIABETIC COMPRESSION SOCKS

Modell	Foot	Rim	Compression	Colour
for women				
Calf A – D	closed toe	soft top	18 – 25 mmHg	white
for men				
Calf A – D	closed toe	soft top	18 – 25 mmHg	white

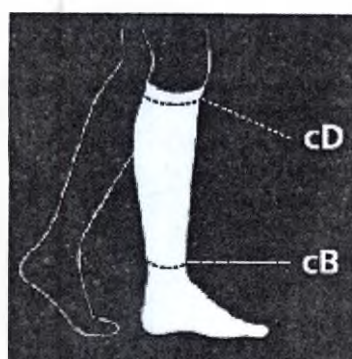Circumference in cm			
S	M	L	XL*
27 – 33	31 – 39	35 – 45	41 – 51
18 – 22	22 – 26	26 – 29	29 – 36

* size XL only for men.

There are compression stockings with different degrees of compression, depending on the severity of the disease to be treated. The pressure exerted by compression stockings at ankle level is measured in millimeters of mercury (mmHg.art.). The compression class is defined according to the official standards: GZG RAL standard in Europe.

Depending on the disease, the doctor prescribes a certain compression class to the patient. Light compression corresponds to class 1 or 2, strong compression to class 3 or 4 depending on the standard used.

CCL. I

Compression level 18-21 mmHg

Indications

- Heaviness and fatigue of the leg
- Mild varicose veins during pregnancy
- Mild varicose veins without significant oedema
- Prophylaxis of thrombosis and embolism for immobile patients
- Prophylaxis during travel (Economy Class Syndrome)

CCL. II

Compression level 23-32 mmHg

Indications

- More severe varicose veins during pregnancy
- Varicose veins with mild oedema
- Post superficial and deep venous thrombosis
- Post sclerotherapy, venous surgery or thermal ablation
- Chronic venous insufficiency (CVI)
- Management of healed venous ulcer
- Prevention of DVT and oedema
- Post-thrombotic syndrome (PTS)
- Prevention of PTS
- Post-traumatic oedema

CCL. III

Compression level 34-46 mmHg

Indications

- Pronounced varicosis with severe oedema
- Severe chronic venous insufficiency
- Treatment of active ulcers and recurrent ulcers
- Severe post-traumatic oedema
- Lymphoedema

CCL. IV

Compression level >49 mmHg.

Indications

- Severe post-thrombotic syndrome
- Severe lymphoedema

15.4 Medical compression products "SIGVARIS" in versions:

- anti-embolic stockings **THROMBO X (THROMBO X);**
- compression socks **JAMES (JAMES);**
- compression socks (diabetic) (**DIABETIC COMPRESSION SOCKS**).

16. Marking and packaging

Medical compression products "SIGVARIS" in versions:

- anti-embolic stockings **THROMBO X (THROMBO X);**
- compression socks **JAMES (JAMES);**
- compression socks (diabetic) (**DIABETIC COMPRESSION SOCKS**).

MANUFACTURER

SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

PRODUCTION SITE

JAMES:
SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen
Switzerland

DIABETIC COMPRESSION SOCKS

SIGVARIS INC.
1119 Highway 74 South
Peachtree City, GA 30269
USA

THROMBO X

Sanyleg
Via Albania 1/3
46042 Castel Goffredo MN, Italien

It is supplied in individual consumer cardboard packaging:

16.1. Medical compression products "SIGVARIS" in versions:

- compression socks **JAMES (JAMES);**
- compression socks (diabetic) (**DIABETIC COMPRESSION SOCKS**).

It has the following dimensions: 23.5 x 12.6 x 3.7 cm / ± 0.5 cm.

16.2. Medical compression products "SIGVARIS" in versions:

- anti-embolic stockings **THROMBO X (THROMBO X)**;
available in plastic bags: 24 x 19 cm / ± 0.5 cm.

It has a conformity mark (CE).

It also contains a photographic image of the medical product, an image of the contents of the package. On the package there is a brief description, marking of disposal, conditions of transportation, storage and use.

On consumer cardboard packaging data on the manufacturer, a bar code and a serial number are specified.

17. SYMBOLS AND MEANING

	Manufacturer
	Recyclable
	Use before (except THROMBO X) Best before date 3 years since manufacture For THROMBO X – Best before date not specified
	Lot number (except THROMBO X) sewed in tag has lot number (ex. 1927101) , except THROMBO X
	AD – Stockings AG – Thigh-length stockings
	Delicate mode / Synthetic
	Do not bleach
	Do not dry in drying machine

	Do not iron
	Do not dry clean
	Quality mark Association for Medical Compression Hosiery (except DIABETIC and THROMBO X)
	Oeko-Tex 100 Standard (except DIABETIC)
Authorized Manufacturer Representative	OOO «Veno+» 123007, Russia, , Moscow, 2nd Magistral'nyy Tupik, 7a TIN 7714804095 Tel: +7(495) 645-33-47 venoplus@venoplus.ru
Product license No.	
Disposal requirements for Russian Federation	“Medical compression products "SIGVARIS" in options” are to be disposed of in accordance with Sanitary Regulations and Norms 2.1.7.2790-10.
Storage requirements	“Medical compression products "SIGVARIS" in options” from +5 to +27 C and atmosphere relative humidity 0~50%, pressure from 850 to 1060 GPa and at height <3000 m above sea level. Do not use near other equipment or store vertically.

18. List of risks identified during risk analysis

Based on performed risk analysis results it was identified that implemented measures of risk mitigation are effective and residual risk is acceptable.

Manufacturer provided Risk Analysis 1,2.

18.1 Performed tests:

Compression test

- **Preconditioning**

The samples are washed according to DIN EN ISO 6330/4G:2013-02, dried at room temperature (Regulation C of this standard). The samples are further matured within standard environment for 24 hours according to DIN EN ISO 139:2011-10, section 3.1.

- **Stretch, pressure and residual pressure**

Test samples: 2

Tests were performed at $(20 \pm 2)^{\circ}\text{C}$ and relative humidity $(65 \pm 5)\%$ using HOSY.

Load curve was examined. Results are given in the table and diagram below. The table also specifies length and circumference.

Note:

HOSY tests stipulate that fabric and stocking grip top connection must be located next to the nearest connection between two clips.

The tables list values obtained at the highest measurement points from the clip that held the fabric only. This measurement point correlates with the specified point (on the stocking grip top) in the pressure distribution sheet.

18.2 Sizes

- **Length**

Length measurement in the highest point varies within 15%. This means that according to Section 4.5.2 RAL-GZ 387 / 1:2008-01^A an average product length should be measured.

- **Circumference**

Minimum and maximum circumference values differ more than by 10%. Therefore, at first, minimum, and then maximum circumference values were measured.

18.3 Packaging Type

Folding box. Packaging protects stockings from contamination and light exposure.

19. Marking

- **Label**

Label has all necessary information. However, calf size should be highlighted.

- **Packaging**

Label has all necessary information. However, calf size should be highlighted.

▪ **User Manual**

User Manual contains all necessary information.

Test Report No. 16.2.20.12832/2/1 from July 18 2017

Page 3 of 4

20. Ecological Safety

Certificate Oeko-Tex® (No. CPP 18868).

Fiber structure and seams

Seamless facial knitting with elastic inner thread in each row.

Material Composition

- **Textile fiber**
- **Elastic thread**

Outer coating

Inner coating

Additional thread

- **Filling**

Outer coating

Inner coating

No

Elastan, approx. 44 decitex

Total yarn density approx. 234 decitex

Polyamide

Polyamide

Polyamide

Elastan, approx. 612 decitex

Polyamide

Polyamide

Heel and toe structure

Heel and toe structure complies with specifications.

Elasticity

Direction	Measurement point	Elasticity, %	Actual stretching as transverse tension around the circumference rate	
			min	max
lateral	B	93	-	-
transverse	B	174	27	35
	D	127	36	58

Compression

Pressure is 3.93 kPa and 4.20 kPa accordingly in min and max circumference measurement points. Product complies with Compression class II.

Residual pressure and pressure distribution dynamics

Higher points residual pressure is not higher than lower points residual pressure in min and max circumference measurement points.

Residual pressure in min and max circumference measurement points from B1 to D is within set interval (See Table 8 RAL-GZ 387 / 1:2008-01A).

Test Report No. 16.2.20.12832/2/I from July 18 2017
Page 4 of 4

Permissible variations in the product quality

Incomplete marking.

Residual pressure around the minimum circumference of the grip top below 50%.

Actual stretching as the ratio of transverse tension around maximum circumference in D measuring point is higher than provided in the guideline.

Conclusions

Based on test results, the product is deemed to comply with requirements.

Hohenstein, July 18, 2017

Head
/signature/
Dipl. eng. Florian Girmond

Head of compression test acceptance department
/signature/
Dipl. eng. Christine Doege

/Seal of Hohenstein Laboratories/

Test results correlate solely to the submitted samples. This report shall be reproduced only in complete. No extracts can be made from this report. Using this report for advertising or publishing free interpretation of this report is subject to prior consent of the Hohenstein Institute. This report shall be legally valid only if signed by authorized persons. Documents and materials shall be submitted by the customer for 3 years if possible. In cases governed by law, 10 year storage period shall be applied.

All methods specified in the Appendices are accredited (refer to www.hohenstein.de/de/about_hohenstein/accreditation/accreditation.html) — marked with "A" in the report.

Table No. 1.1

Test Report No. 16.2.20.12832/2/I

Hohenstein compression ratio measurement system						
Manufacturer:		SIGVARIS AG				
Product:		JAMES Ccl.2				
Manufacturer's Size:		AD 22-24 (34-41/35-40) # S+				
Special Details:		With cuff - min. circumference				
Point	Circumference (cm)	Height (cm)	Stretching during wearing (%)	Tension (N/cm)	Pressure (kPa)	Residual pressure (%)
B	22.0	11.0	47.1	1.38	3.93	100.0
B1	26.0	21.0	38.9	1.26	3.05	77.6
C	35.0	30.0	44.7	1.23	2.21	56.1
D	35.3	31.0	45.2	1.19	2.13	54.1
1	22.4	6.0	36.8	1.01	2.84	72.1
2	22.0	11.0	47.1	1.38	3.93	100.0
3	22.8	16.0	46.3	1.37	3.79	96.3
4	26.0	21.0	38.9	1.26	3.05	77.6
5	31.7	26.0	43.3	1.27	2.52	64.1
6	35.3	31.0	45.2	1.19	2.13	54.1
7	34.5	36.0	51.1	.93	1.69	43.0

Based on 2 measurements results.

Table No. 1.2

Test Report No. 16.2.20.12832/2/1


Hohenstein compression ratio measurement system						
Manufacturer:		SIGVARIS AG				
Product:		JAMES Ccl.2				
Manufacturer's Size:		AD 22-24 (34-41/35-40) # S+				
Special Details:		With cuff - min. circumference				
Point	Circumference (cm)	Height (cm)	Stretching during wearing (%)	Tension (N/cm)	Pressure (kPa)	Residual pressure (%)
B	24.0	11.0	60.5	1.61	4.20	100.0
B1	32.0	21.0	71.0	1.75	3.43	81.6
C	42.0	30.0	73.7	1.76	2.64	62.8
D	42.3	31.0	73.9	1.73	2.57	61.2
1	24.6	6.0	49.6	1.20	3.07	73.1
2	24.0	11.0	60.5	1.61	4.20	100.0
3	26.6	16.0	70.6	1.74	4.11	97.9
4	32.0	21.0	71.0	1.75	3.43	81.6
5	38.6	26.0	74.6	1.81	2.94	69.9
6	42.3	31.0	73.9	1.73	2.57	61.2
7	41.5	36.0	81.7	1.50	2.27	53.9

Based on 2 measurements results.

Graph No.1

Test Report No. 16.2.20.12832/2/1

Hohenstein compression ratio measurement system	
Manufacturer:	SIGVARIS AG
Product:	JAMES Ccl.2
Manufacturer's Size:	AD 22-24 (34-41/35-40) # S+
Special Details:	With cuff

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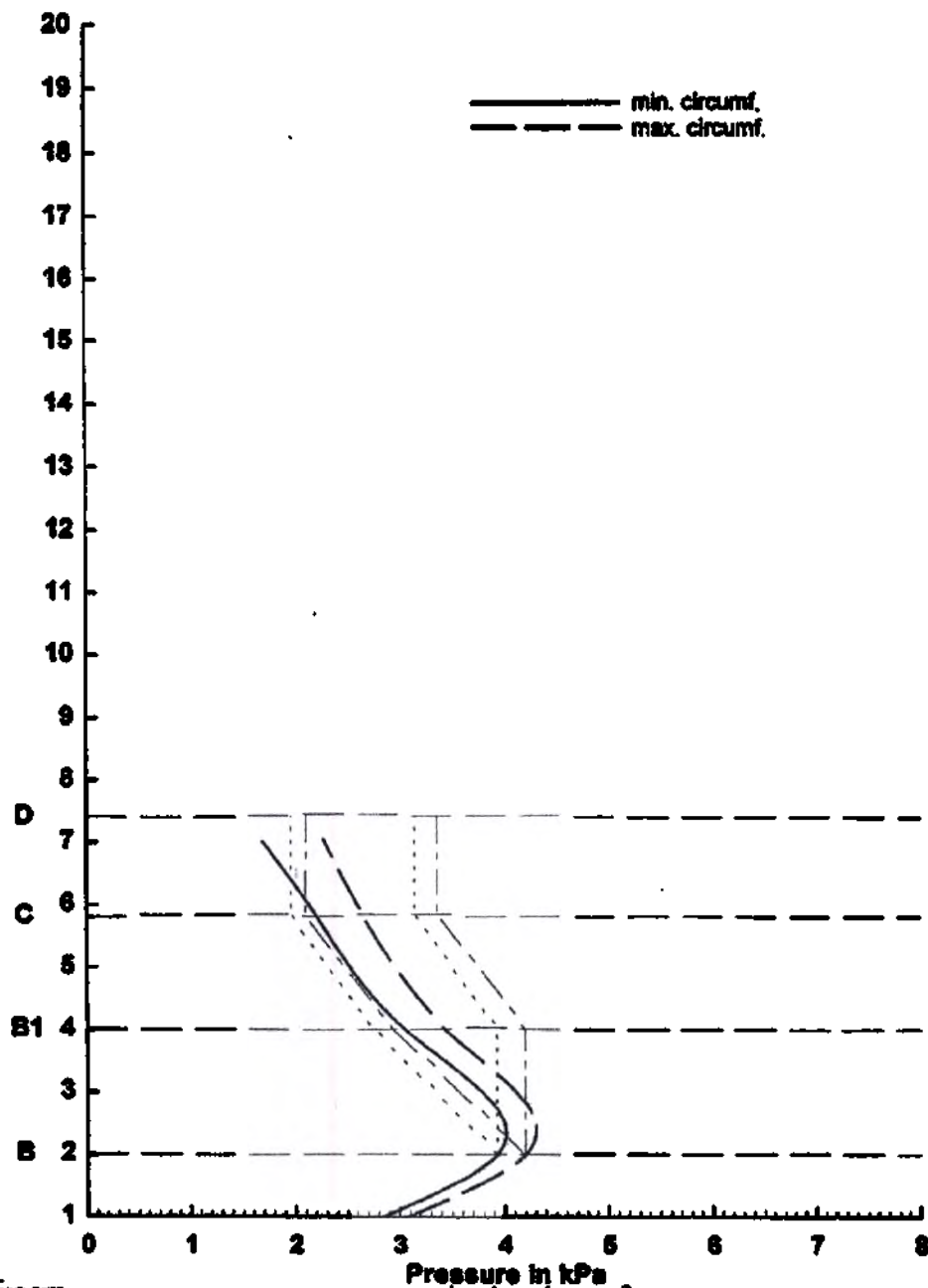Post Market Survey

Sales and claims data

We hereby confirm validity of attached document:

Sales and claims data for SIGVARIS AG

This document includes following information:

- Claims and complaints (on materials, sizes, presence of defects)
- Product returns (trial orders, on consignment, incorrect delivery, incorrect order)

-Product replacement

Attachment No. 3

21. Verification and validation of medical device data

Medical compression products "SIGVARIS" in options are supplied not sterile.

22. Materials of animal or human use with information their biological compatibility.

Manufacturing company SIGVARIS AG, Switzerland under full liability states that in manufacturing of medical product "Medical compression products "SIGVARIS" in options" no materials of animal or human use are used.

23. Previous modifications of medical product and similar modifications of currently marketed medical product.

In Russian Federation, there are following duly registered and currently marketed modifications: Medical compression products "SIGVARIS": stockings "Cotton", "Traditional", "Comfort", "Magic"; Thigh-high stockings "Cotton", "Traditional", "Comfort", "Magic"; "Thrombo-Sigvaris". Product license No. ФС3 2011/11272

Medical compression products SIGVARIS" in options are developed and manufactured according to Law of USA and EN ISO 13485 Standard, and also according to requirements of 93/42/EU Regulation.

Manufacturer is recording reports on all manufacturing processes within the company!

24. Medical substances contained in medical product.

Manufacturing company SIGVARIS AG, Switzerland under full liability states that in manufacturing of medical product "Medical compression products "SIGVARIS" in options" no medical substances are used.

25. Sterilization method.

Medical device is supplied not sterile.

26. Maintenance and repair

For medical product "Medical compression products SIGVARIS in options" maintenance is not required.

27. Disposal requirements for Russian Federation.

"Medical compression products "SIGVARIS" in options" are to be disposed of in accordance with Sanitary Regulations and Norms 2.1.7.2790-10.

28. Transportation

"Medical compression products "SIGVARIS" in options" are delivered ready to use, packed in transport packaging, transported by all means of transportation according to transportation requirements for transport type.

Temperature from +5 to +27°C

Relative humidity 0~50%,

Atmosphere pressure from 850 to 1060 GPa.

Transport packaging is resistant to transportation bounce with maximum acceleration 30 m/s² and frequency from 2 to 3 Hz.

29. Information on medical product care:

- Wash the socks daily. They can be washed in the washing machine in a laundry bag at up to 95°C.
- Use a mild detergent, free of fabric conditioner.

- Do not tumble dry the socks in order to avoid overheating and static electricity. Wrap the compression socks in two dry towels to remove excess water and dry in a shaded and airy place. Do not place on a heater.
- Do not iron.
- Do not dry-clean.
- Sharp fingernails and rings with sharp edges can easily cause damage. Please be sure to pay attention to this when putting on and removing the socks.

30. Storage requirements

Storage requirements: "Medical compression products "SIGVARIS" in options" from +5 to +27 C and relative humidity 0~50%, pressure from 850 to 1060 GPa and at height <3000 m above sea level.

Do not use near other equipment or store vertically.

31. Warranty, service life of medical product

There is a 6-month guarantee for "Medical compression products "SIGVARIS" in options" in case of manufacturing defects. Guarantee also does not cover damages caused by incorrect use of medical products.

32. Service life of compression stockings in sealed packing is 3 years since manufacture. After the start of usage stockings will provide proper pressure force for 6 months.

To keep bandage effect for all service life it is important to maintain the rules of care.

The medical benefit of SIGVARIS compression stockings is guaranteed for 6 months if they are used correctly.

33. Russian Federation Authorized Manufacturer Representative:

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123007, Russia, Moscow, 2nd Magistral'nyy Tupik, 7a

TIN 7714804095

Tel: +7(495) 645-33-47 venoplus@venoplus.ru

St. Gallen, 10th December 2019

SIGVARIS AG

Jürgen Sigg
CEO

Markus Hämmerling
Head of Regulatory Affairs

Official certification no. CI/192

Inspected for legalisation of the signature of the firm SIGVARIS AG, St.Gallen, Switzerland, signed by Mr. Jürgen Sigg, CEO.

St.Gallen, Switzerland, 06/05/2020

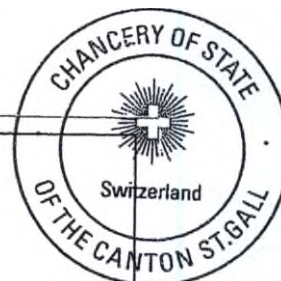
Population Services City of St.Gallen
Legalisations

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2. has been signed by Claudine Lüthi

3. acting in the capacity of Legalisationsbeamtin
в качестве чего

4. Bears the seal/stamp of legalizing officer
Со штампом

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