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
BE IT KNOWN that I, Sunita Kumeri of, 268 Bath Road, Slough, Berkshire United Kingdom a duly authorised Notary Public

CERTIFY ONLY that Brian Michael Howes who is well known to me and who is duly authorised by Reckitt Benckiser Healthcare (UK) Limited ("the Company") to represent them in this matter has today caused the annexed Operational Documentation for Durex® Play Lubricants in Variations to be produced to me and has represented to me on behalf of the company that the said document is a true copy of the original electronic document produced to him.

SIGNED and sealed at 268 Bath Road, Slough, Berkshire aforesaid on 6th August 2020.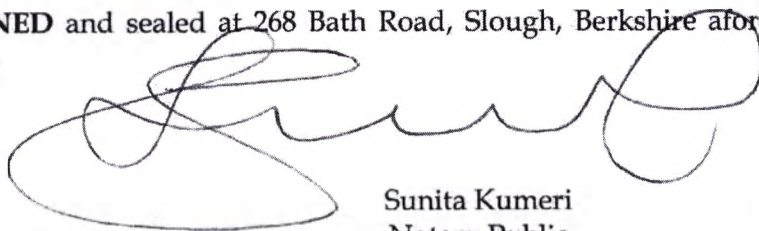

Sunita Kumeri
Notary Public
England and Wales

Protocol No. 5120

Approved by

DocuSigned by:
Mark Ainsworth
 Signer Name: Mark Ainsworth
Signing Reason: I approve this document
Signing Time: 28-07-2020 | 08:58 BST
BC84284D7A8448A5AA7891DC39BD6CF0

Date: 28-07-2020 | 09:05 BST

Name of Responsible Person Mark Ainsworth

Operational documentation:

Durex® Play lubricants

Manufactured by:

Reckitt Benckiser Healthcare (UK) Ltd., UK

Operational documentation

Durex® Play lubricants in variations

Version 02

MEDICAL DEVICE NAME

Durex® Play lubricants in variations

1. Durex® Play Feel lube;
2. Durex® Play Heat lube;
3. Durex® Play Sweet Strawberry lube.
4. Durex® Play Very Cherry lube.

LEGAL MANUFACTURER

Reckitt Benckiser Healthcare (UK) Ltd., United Kingdom

Head office address: 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK

Location address: Dansom Lane, Hull, HU8 7DS, UK

Phone: +44 (0) 1482 582438.

Manufacturing site(s):

- Reckitt Benckiser Healthcare Manufacturing (Thailand) Ltd., 65 Moo 12, Lardkrabang-Bangplee Rd, Bangplee, Samutprakarn, 10540, Thailand
- Meiyume Manufacturing (Thailand) Limited, № 21/7 Moo 6, Tambol Kookot, Amphur Lamhukka, Pathumthani Province, 12130, Thailand

Authorized representative of the manufacturer of the medical device

Name: Reckitt Benckiser Healthcare Limited Liability Company.

Abbreviated name: Reckitt Benckiser Healthcare LLC

Address: floor 3, 4 Shlyuzovaya nab., Moscow, Russia, 115114.

Telephone: +7(495)9612565.

Fax: +7(495)9612566

e-mail: contact_ru@rb.com

MEDICAL DEVICE APPLICATION:

Durex® Play lubricants in variations are used to ease vaginal dryness and discomfort during sexual intercourse.

DEVICE DESCRIPTION

Durex® Play lubricants are personal lubricants intended to ease vaginal dryness and intimate discomfort. These lubricants are applied to the intimate areas prior to sexual intercourse or as required. The products are suitable for all types of sex (vaginal, anal and oral), as indicated on instructions for use.

- Durex® Play Sweet Strawberry and Durex® Play Very Cherry are flavored lubricants.
- Durex® Play Feel and Durex® Play Heat are flavorless lubricants.

The flavors in Durex® Play Sweet Strawberry, Durex® Play Very Cherry are achieved by the following ingredients, such as Strawberry flavoring agent - Strawberry sherbet A0818111, Cherry flavoring agent - Cherry FH2A/FIR580650T, respectively.
Durex® Play Heat has a specific warming effect, which is achieved by hygroscopic effect in contact with the skin due to the high concentration of glycerol.
Durex® Play Feel – classic water base lubricants.

TECHNICAL CHARACTERISTICS

Device specification

№	Technical parameter	Quality requirements for release	Quality requirements for shelf life
1	Durex® Play Feel		
	Appearance	Clear gel free from lumps and extraneous matter	Clear gel free from lumps and extraneous matter
	Odor test	Odorless	Odorless
	pH at 25°C	5.1-5.8	5.1-5.8
	Benzoic acid, %w/w	0.143-0.157	0.130-0.157
	Total aerobic bacterial count, CFU/g	NMT 200	NMT 200
	Total yeasts and molds count, CFU/g	NMT 20	NMT 20
2	Durex® Play Heat		
	Appearance	Clear, colourless gel free from lumps and extraneous matter	Clear, colourless gel free from lumps and extraneous matter
	Odor test	Slightly sweet	Slightly sweet
	pH at 25°C	3.8 – 4.8	3.8 – 4.8
	Benzoic acid, %w/w	0.143 – 0.157	0.130 – 0.157
	Total aerobic bacterial count, CFU/g	NMT 200	NMT 200
	Total yeasts and molds count, CFU/g	NMT 20	NMT 20
3	Durex® Play Sweet Strawberry		
	Appearance	Clear, colourless to slightly pink gel, free from lumps and extraneous matter	Clear to hazy, colourless to slightly pink gel, free from lumps and extraneous matter
	Odor test	Characteristic of strawberry	Characteristic of strawberry
	Taste test	Characteristic of strawberry	Characteristic of strawberry
	pH at 25°C	3.8 – 4.8	3.8-4.8
	Benzoic acid, % w/w	0.143 – 0.157	0.130 – 0.157
	Total aerobic bacterial count, CFU/g	NMT 200	NMT 200

	Total yeasts and molds count, CFU/g	NMT 20	NMT 20
4	Durex® Play Very Cherry		
	Appearance	Clear, colourless to slightly yellow gel, free from lumps and extraneous matter	Clear, colourless to slightly yellow gel, free from lumps and extraneous matter
	Odor test	Characteristic of cherry	Characteristic of cherry
	Taste test	Characteristic of cherry	Characteristic of cherry
	pH at 25°C	3.8 – 4.8	3.8 – 4.8
	Benzoic acid, %w/w	0.143 – 0.157	0.130 – 0.157
	Total aerobic bacterial count, CFU/g	NMT 200	NMT 200
	Total yeasts and molds count, CFU/g	NMT 20	NMT 20

MAIN PARAMETERS AND CHARACTERISTICS




Durex® Play lubes can be shipped in 50 ml bottles (№ 1-2), 100ml bottles (№ 3) and 5 ml sachet (for Durex Play Feel only).

Packaging Specification:

PACK SIZE: Bottle №1: Bottle without Dispenser (Flip Top Cap) 50 ml.

- Durex® Play Feel, the nominal volume is not less than 50 ml and the lube weight is not less than 54.5 g
- Durex® Play Heat, the nominal volume is not less than 50 ml and the lube weight is not less than 61.0 g.
- Durex® Play Strawberry, the nominal volume is not less than 50 ml and the lube weight is not less than 61.0 g.
- Durex® Play Very Cherry, the nominal volume is not less than 50 ml and the lube weight is not less than 61.0g.





Bottle №1: Drawing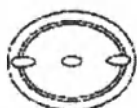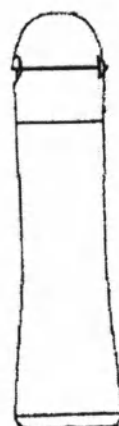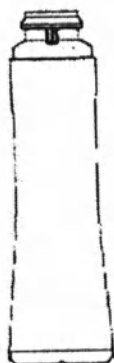

Bottle №1: Specification of Bottle

Parameter	Value	Tolerance	U.O.M
Overall Height	112,2	±1	mm
Diameter Minor Axis	31,5	±0,5	mm
Weight	8,5	±0,5	g

Bottle №1: Drawing of Cap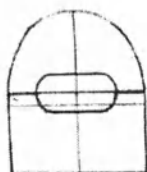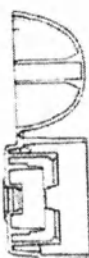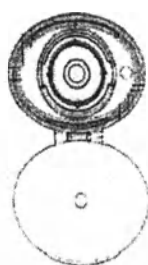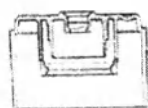


Bottle №1: Specification of Cap

Parameter	Value	Tolerance	U.O.M
Overall Height	35,5	±0,5	mm
External Diameter at Base	33,8×25,8	±0,5	mm
Internal Diameter at Base	32,2×24.2	±0,5	mm

PACK SIZE:**Bottle №2: Bottle without Dispenser (Flip Top Cap) 50 ml.**


- Durex® Play Feel, the nominal volume is not less than 50 ml and the lube weight is not less than 62.6g.
- Durex® Play Heat, the nominal volume is not less than 50 ml and the lube weight is not less than 69.9 g.
- Durex® Play Sweet Strawberry, the nominal volume is not less than 50 ml and the lube weight is not less than 69.9 g.
- Durex® Play Very Cherry, the nominal volume is not less than 50 ml and the lube weight not less than 69.9 g.

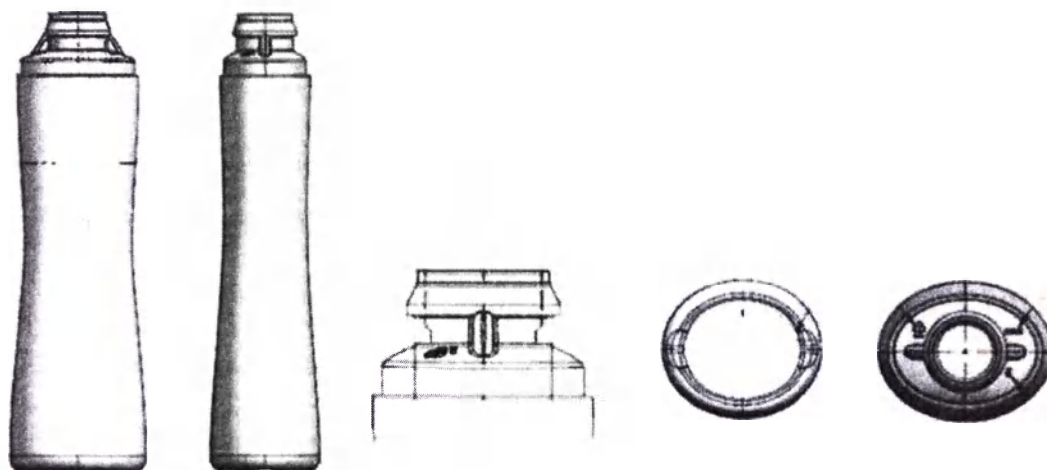Bottle №2: Drawing

Bottle №2: Specification of Bottle

Parameter	Value	Tolerance	U.O.M
Overall Height	116.2	$\pm 1,0$	mm
Diameter Minor Axis	31,5	$\pm 0,5$	mm
Wall Thickness	Minimum 0.4	-	mm
Weight	9.0	± 1.0	g

Bottle №2: Drawing of Cap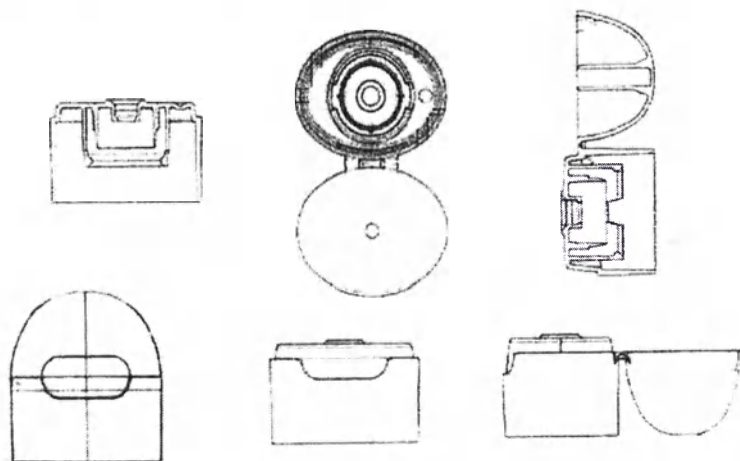

Bottle №2: Specification of Cap


Parameter	Value	Tolerance	U.O.M
Overall Height	35,5	$\pm 0,5$	mm
External Diameter at Base	33,8×25,8	$\pm 0,5$	mm
Internal Diameter at Base	32,2×24.2	$\pm 0,5$	mm

PACK SIZE:

Bottle №3: Bottle without Dispenser (Flip Top Cap) 100 ml.

- Durex® Play Feel, the nominal volume is not less than 100 ml and the lube weight is not less than 110 g.
- Durex® Play Heat, the nominal volume is not less than 100 ml and the lube weight is not less than 121 g.
- Durex® Play Sweet Strawberry, the nominal volume is not less than 100 ml and the lube weight is not less than 121 g.
- Durex® Play Very Cherry, the nominal volume is not less than 100 ml and the lube weight is not less than 121 g.


Bottle №3: Drawing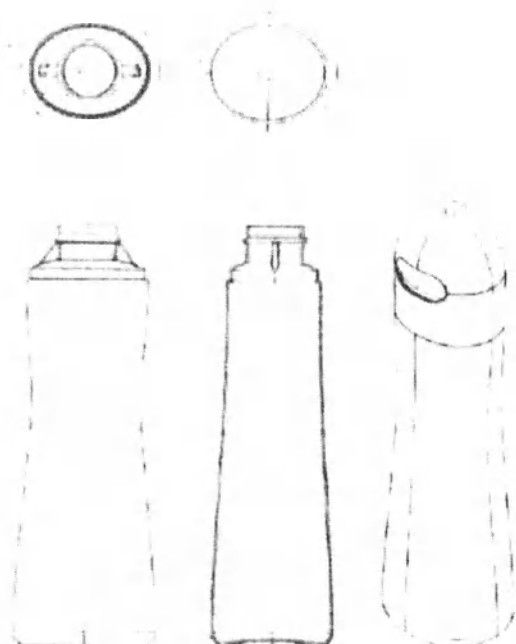

Bottle №3: Specification of Bottle

Parameter	Value	Tolerance	U.O.M
Overall Height	132.5	$\pm 1,0$	mm
Diameter Minor Axis	38.0	$\pm 0,5$	mm
Wall Thickness	Minimum 0.4	-	mm
Weight	12.5	$\pm 0,5$	g

Bottle №3: Drawing of Cap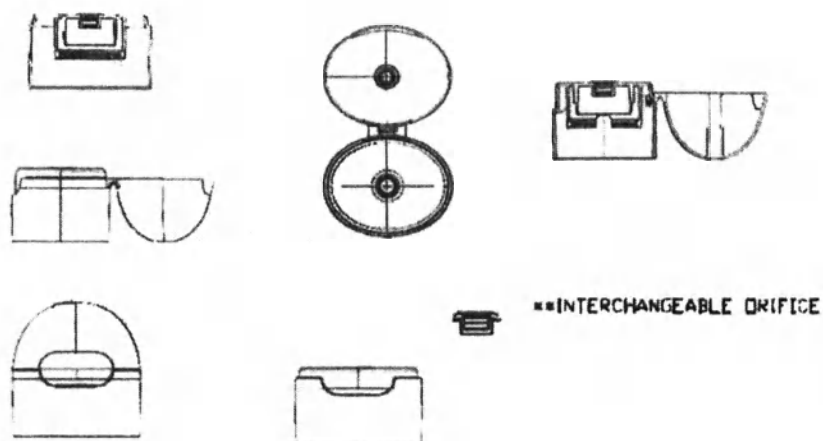

Bottle №3: Specification of Cap

Parameter	Value	Tolerance	U.O.M
Overall Height	40	$\pm 0,5$	mm
External Diameter at Base	38×30	$\pm 0,5$	mm
Internal Diameter at Base	36,4×28,4	$\pm 0,5$	mm

PACK SIZE: 5ml Sachet


Durex® Play Feel, the nominal volume is not less than 5 ml and the lube weight is not less than 5.6 g

5ml Sachet: Specification of Sachet

Parameter	Value	Tolerances	U.O.M
Bag Size (Width x Length)	54 x 80 mm	± 1	mm
Weight	0.79 per sachet	± 0.06	g

5ml Sachet: Image/drawing of sachet

Sealing area is 5 +/-1 mm for each edge/side.

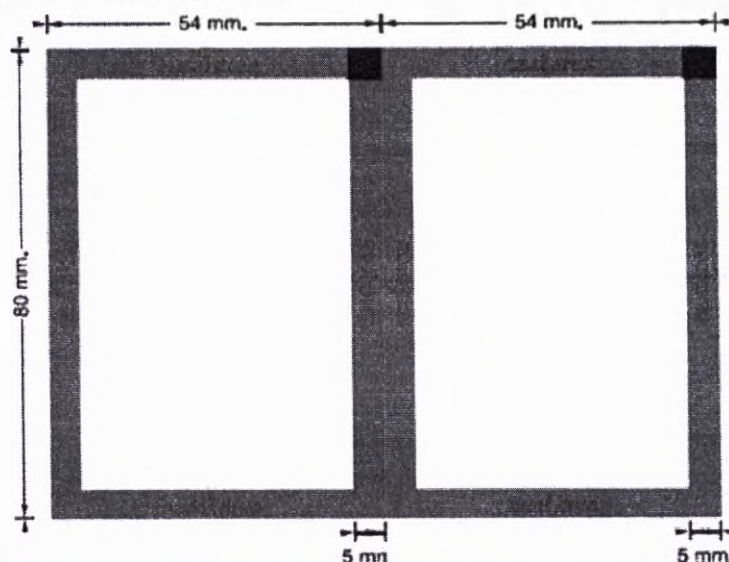5ml Sachet is additionally packed in a secondary carton package.

RAW MATERIALS

Packaging Materials for Bottle № 1-3

Package part	Material
Bottle:	HDPE:6140B SCG
Cap:	PP: RP 348N HMC

Packaging Materials for 5ml Sachet

Packaging	Material
Foil	PET 12 µm/ Ink + Adhesive 5 µm/ DL /Aluminium 9 µm/ LDPE 50 µm PET: A.J. Plast Public Company Limited Ink: TOYO INK (THAILAND) CO., LTD Adhesive: Novacote Flexpack Aluminium: JIANGSU ZHONGJI LAMINATION MATERIALS LDPE: SCG Chemicals Co., Ltd.

*The suppliers of raw materials can be changed based on manufacturer decision. The quality, safety and efficacy of medical device will be assessed by legal manufacturer before change will be implemented.

PP – polypropylene

HDPE – high-density polyethylene

LDPE – low-density polyethylene

PET – polyethylene terephthalate

DL - dry lamination

FORMULA RAW MATERIAL/LUBRICANT COMPOSITION:

Device name	Chemical compound	Weight fraction, %
Durex® Play Feel	Purified water	58.850
	Propylene glycol	20.000
	Glycerol	20.000
	Carbomer	0.600
	Xanthan Gum – CG-SFT	0.200
	Sodium hydroxide	0.160
	Benzoic acid	0.150
	Citric acid	0.040

Durex® Play Heat	Glycerol	79.805
	Purified water	17.5575
	Propylene glycol	2.000
	Hydroxyethylcellulose Natrosol 250 Pharm**	0.400
	Benzoic acid	0.150
	Saccharin sodium	0.075
	Sodium hydroxide*	0.0125
Durex® Play Sweet Strawberry	Glycerol	79.805
	Purified water	16.5575
	Propylene glycol	2.000
	Hydroxyethylcellulose Natrosol 250 Pharm**	0.400
	Benzoic acid	0.150
	Saccharin sodium	0.075
	Sodium hydroxide*	0.0125
	Strawberry flavoring agent Strawberry sherbet A0818111	1.000
Durex® Play Very Cherry	Glycerol	79.000
	Purified water	18.270
	Propylene glycol	2.000
	Hydroxyethylcellulose Natrosol 250 Pharm**	0.400
	Benzoic acid	0.150
	Saccharin sodium	0.075
	Sodium hydroxide*	0.025
	Cherry flavoring agent Cherry FH2A /FIR 580650T	0.080

* Sodium hydroxide is included in the formulation as a pH adjuster in. The amount of sodium hydroxide can vary up to 0.025% for formulation Durex® Play Sweet Strawberry, Durex® Play Heat, and Durex® Play Very Cherry. The amount of purified water will be adjusted accordingly to ensure correct batch weight is obtained.

** Amount of Natrosol may be adjusted depending on raw material viscosity. The amount of purified water will be adjusted accordingly to ensure correct batch weight is obtained.

APPLICATION

Bottle №1-3	Sachet
Flick cap open and squeeze gently apply.	Tear sachet open and squeeze gently to apply. For single use.
The lubricant can also be applied to the outside surface of a condom*. The products are suitable for all types of sex, as indicated on instructions for use.	

CONDOM COMPATIBILITY

*

Durex® Play lubes in variations:	Condom compatibility		
	NRL	PI	PU

Durex® Play Feel lube	Compatible	Compatible	Compatible
Durex® Play Heat lube	Compatible	Compatible	Compatible
Durex® Play Sweet Strawberry lube	Compatible	Compatible	Compatible
Durex® Play Very Cherry lube	Compatible	Compatible	Compatible

POTENTIAL CONSUMERS

Lubricants are intended for home use.

APPLICATION CONDITIONS

There are no specific temperature conditions required for the operation/use of the products.

TRANSPORTATION AND STORAGE, SHELF LIFE

Transportation:

Products are being transported by all types of transport in pallet.

Transportation is allowed under the following temperature conditions from -10 up to +60°C

Storage:

Store at the following temperature 10-30°C, out of reach of children and away from direct sunlight. An opened package should be used within three months.

Shelf life:

Durex® Play Feel – 3 years for lube in bottles №№1-3 and 2 years for lube in 5 ml sachet;
 Durex® Play Heat – 3 years for lube in bottles №№1-3;
 Durex® Play Sweet Strawberry - 2 years for lube in bottles №№1-3;
 Durex® Play Very Cherry- 3 years for lube in bottles №№1-3;

LABELLING

Each consumer package contains the following information:

- trademark;
- batch number;
- expiry date (year and month);
- lubricant description;
- volume of lubricant in a package;
- lubricant composition
- name of legal manufacturer;
- legal manufacturer company address;
- manufacturing site address;
- storage guidelines;
- application method;

- precautions for use;
- RC information

LABELING SYMBOLS

Symbol	Interpretation
	Name: Batch number
 EXP	Name: Use by

Additional information can be printed on the individual/consumer package including reference to international standards, marketing information – see “Labelling design”.

GUIDELINES FOR DESTRUCTION AND DISPOSAL

Medical device should be used in accordance with the instruction for use.

Durex® Play lubes are intended as a personal lubricant for home use only, and can be disposed of as household waste. No specific requirements are applied for disposal of this product.

MAINTENANCE OPERATION AND REPAIR

This section is not applicable for this range of medical devices.

ENVIRONMENT SAFETY REQUIREMENTS

With proper transportation, usage and storage, the medical device does not harm the human body and the environment.

ENVIRONMENTAL PROTECTION

With proper use, storage and disposal, Durex® Play lubricants in variations have no negative impact on the environment.

Approved by

DocuSigned by:
Mark Ainsworth
Signer Name: Mark Ainsworth
Signing Reason: I approve this document
Signing Time: 28-07-2020 | 08:57 BST
BC84284D7A8446A5AA7891DC39BD6CF0

Date: 28-07-2020 | 09:05 BST

Name of Responsible Person

Mark Ainsworth

Manufactured by:

Reckitt Benckiser Healthcare (UK) Ltd., UK

CONSUMER INSTRUCTION FOR USE FOR MEDICAL DEVICE:

Durex® Play lubricants in variations:

manufactured by

Reckitt Benckiser Healthcare (UK) Limited, UK

1. Durex® Play Feel lube'

Durex® Play lubricants are intended for use as personal lubricant to ease vaginal dryness and intimate discomfort.

Before using: lubricant is suitable for all types of sex (vaginal, anal and oral). COMPATIBLE WITH CONDOMS. Store at 10-30°C, out of reach of children and away from direct sunlight. Use within three months of opening.

Application:

Bottle №1-3	Sachet
Flick cap open and squeeze gently apply.	Tear sachet open and squeeze gently to apply. For single use.

The lubricant can also be applied to the outside surface of a condom.

Precautions: Use as directed. It's not a contraceptive and it doesn't contain spermicide. It may slow sperm down. Avoid contact with eyes, broken skin or wounds.

If you experience irritation, stop using this medical device. Consult your doctor if irritation continues, you experience persistent vaginal dryness, are pregnant or breast-feeding.

Ingredients: Purified water, Propylene glycol, Glycerol, Carbomer, Xanthan Gum, Sodium hydroxide, Benzoic acid, Citric acid

Head office of manufacturer: Reckitt Benckiser Healthcare (UK) Ltd., 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK.

Manufacturing sites:

- 1). Reckitt Benckiser Healthcare Manufacturing (Thailand) Ltd., 65 Moo 12, Lardkrabang-Bangplee Rd, Bangplee, Samutprakarn, 10540, Thailand
- 2). Meiyume Manufacturing (Thailand) Limited, № 21/7 Moo 6, Tambol Kookot, Amphur Lamlukka, Pathumthani Province, 12130, Thailand

Authorized representative in Russia: Reckitt Benckiser Healthcare LLC, floor 3, 4 Shlyuzovaya emb., Moscow, Russia, 115114. Hotline phone (free in Russia): 8-800-200-82-20, e-mail: contact_ru@rb.com.

Shelf life: 3 years (for bottle №№1-3) / 2 years (for sachet)

Use by (EXP: ГГГГ.ММ) and batch number (LOT) see on the package.

Instruction for use from: 00/0000

Registration certificate № ФС3 0000/00000 from 00.00.0000

(for bottle №№1-3) /

(for sachet)

Volume 50/100/5 ml

'Feel (in Russian)

2. Durex® Play Heat lube'

Durex® Play lubricants are intended for use as personal lubricant to ease vaginal dryness and intimate discomfort.

Before using: lubricant is suitable for all types of sex (vaginal, anal and oral). COMPATIBLE WITH CONDOMS. Store at 10-30°C, out of reach of children and away from direct sunlight. Use within three months of opening.

Application: Flick cap open and squeeze gently apply. The lubricant can also be applied to the outside surface of a condom.

Precautions: Use as directed. It's not a contraceptive and it doesn't contain spermicide. It may slow sperm down. Avoid contact with eyes, broken skin or wounds. If you experience irritation, stop using this medical device. Consult your doctor if irritation continues, you experience persistent vaginal dryness, are pregnant or breast-feeding.

Ingredients: Glycerol, Purified water, Propylene glycol, Hydroxyethylcellulose, Benzoic acid, Saccharin sodium, Sodium hydroxide.

Head office of manufacturer: Reckitt Benckiser Healthcare (UK) Ltd., 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK.

Manufacturing sites: 1). Reckitt Benckiser Healthcare Manufacturing (Thailand) Ltd., 65 Moo 12, Lardkrabang-Bangplee Rd, Bangplee, Samutprakarn, 10540, Thailand

2). Meiyume Manufacturing (Thailand) Limited, № 21/7 Moo 6, Tambol Kookot, Amphur Lamlukka, Pathumthani Province, 12130, Thailand

Authorized representative in Russia: Reckitt Benckiser Healthcare LLC, floor 3, 4 Shlyuzovaya emb., Moscow, Russia, 115114. Hotline phone (free in Russia): 8-800-200-82-20, e-mail: contact_ru@rb.com.

Shelf life: 3 years.

Use by (EXP: ГГГГ.ММ) and batch number (LOT) see on the package.

Instruction for use from: 00/0000

Registration certificate ΦC3 0000/00000 from 00.00.0000

Volume 50/100 ml

'Heat (in Russian)

3. Durex® Play Sweet Strawberry lube'

Durex® Play lubricants are intended for use as personal lubricant to ease vaginal dryness and intimate discomfort.

Before using: lubricant is suitable for all types of sex (vaginal, anal and oral). COMPATIBLE WITH CONDOMS. Store at 10-30°C, out of reach of children and away from direct sunlight. Use within three months of opening.

Application: Flick cap open and squeeze gently apply. The lubricant can also be applied to the outside surface of a condom.

Precautions: Use as directed. It's not a contraceptive and it doesn't contain spermicide. It may slow sperm down. Avoid contact with eyes, broken skin or wounds.

If you experience irritation, stop using this medical device. Consult your doctor if irritation continues, you experience persistent vaginal dryness, are pregnant or breast-feeding.

Ingredients: Glycerol, Purified water, Propylene glycol, Hydroxyethylcellulose, Benzoic acid, Saccharin sodium, Sodium hydroxide, Strawberry flavoring agent Strawberry sherbet.

Head office of manufacturer: Reckitt Benckiser Healthcare (UK) Ltd., 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK.

Manufacturing sites: 1). Reckitt Benckiser Healthcare Manufacturing (Thailand) Ltd., 65 Moo 12, Lardkrabang-Bangplee Rd, Bangplee, Samutprakarn, 10540, Thailand

2). Meiyume Manufacturing (Thailand) Limited, № 21/7 Moo 6, Tambol Kookot, Amphur Lamlukka, Pathumthani Province, 12130, Thailand

Authorized representative in Russia: Reckitt Benckiser Healthcare LLC, floor 3, 4 Shlyuzovaya emb., Moscow, Russia, 115114. Hotline phone (free in Russia): 8-800-200-82-20, e-mail: contact_ru@rb.com.

Shelf life: 2 years.

Use by (EXP: ГГГГ.ММ) and batch number (LOT) see on the package.

Instruction for use from: 00/0000

Registration certificate ΦC3 0000/00000 from 00.00.0000

Volume 50/100 ml

'Sweet Strawberry (in Russian)

4. Durex® Play Very Cherry lube'

Durex® Play lubricants are intended for use as personal lubricant to ease vaginal dryness and intimate discomfort.

Before using: lubricant is suitable for all types of sex (vaginal, anal and oral). COMPATIBLE WITH CONDOMS. Store at 10-30°C, out of reach of children and away from direct sunlight. Use within three months of opening.

Application: Flick cap open and squeeze gently apply. The lubricant can also be applied to the outside surface of a condom.

Precautions: Use as directed. It's not a contraceptive and it doesn't contain spermicide. It may slow sperm down. Avoid contact with eyes, broken skin or wounds.

If you experience irritation, stop using this medical device. Consult your doctor if irritation continues, you experience persistent vaginal dryness, are pregnant or breast-feeding.

Ingredients: Glycerol, Purified water, Propylene glycol, Hydroxyethylcellulose, Benzoic acid, Saccharin sodium, Sodium hydroxide, Cherry flavoring agent Cherry.

Head office of manufacturer: Reckitt Benckiser Healthcare (UK) Ltd., 103-105 Bath Road, Slough, Berkshire, SL1 3UH, UK.

Manufacturing sites: 1). Reckitt Benckiser Healthcare Manufacturing (Thailand) Ltd., 65 Moo 12, Lardkrabang-Bangplee Rd, Bangplee, Samutprakarn, 10540, Thailand

2). Meiyume Manufacturing (Thailand) Limited, № 21/7 Moo 6, Tambol Kookot, Amphur Lamlukka, Pathumthani Province, 12130, Thailand

Authorized representative in Russia: Reckitt Benckiser Healthcare LLC, floor 3, 4 Shlyuzovaya emb., Moscow, Russia, 115114. Hotline phone (free in Russia): 8-800-200-82-20, e-mail: contact_ru@rb.com.

Shelf life: 2 years.

Use by (EXP: FTTT.MM) and batch number (LOT) see on the package.

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