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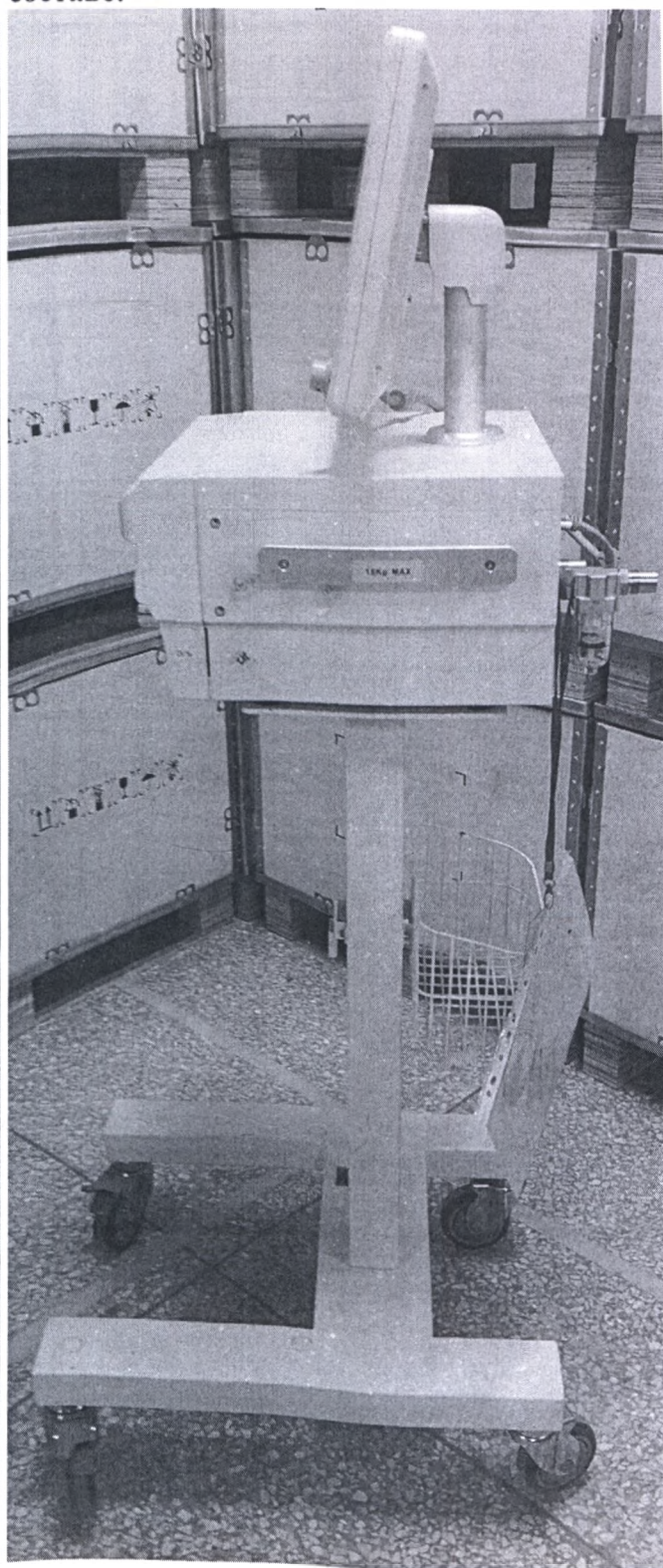
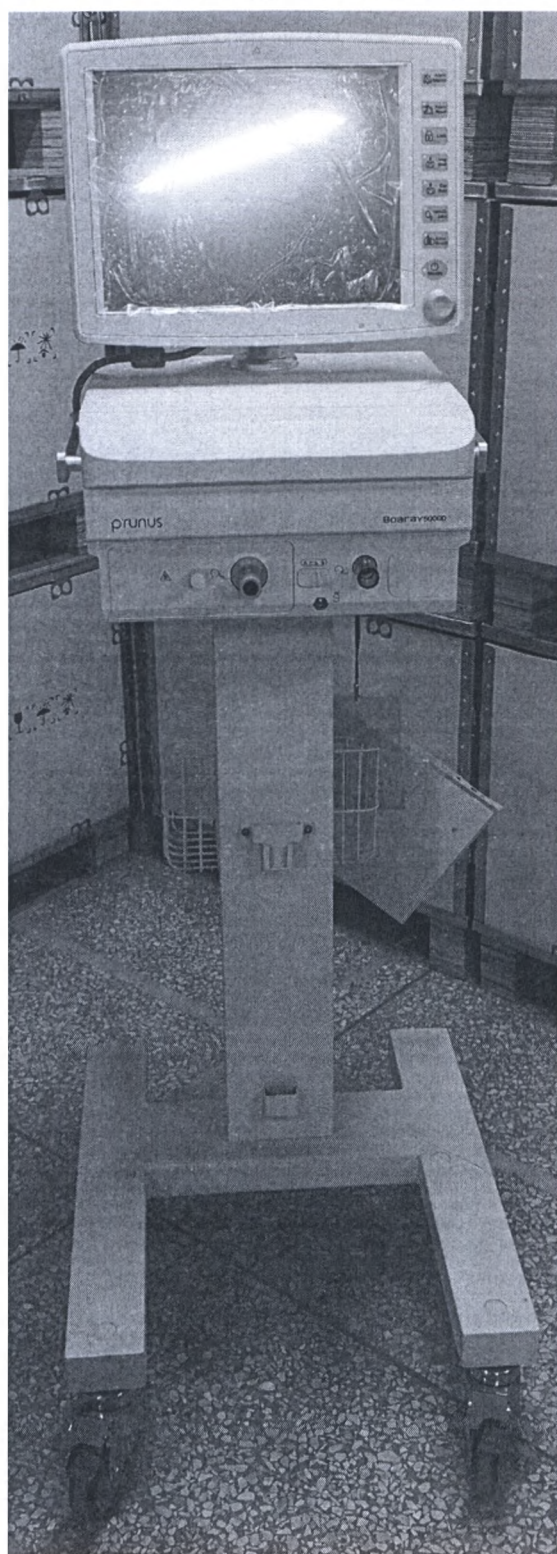
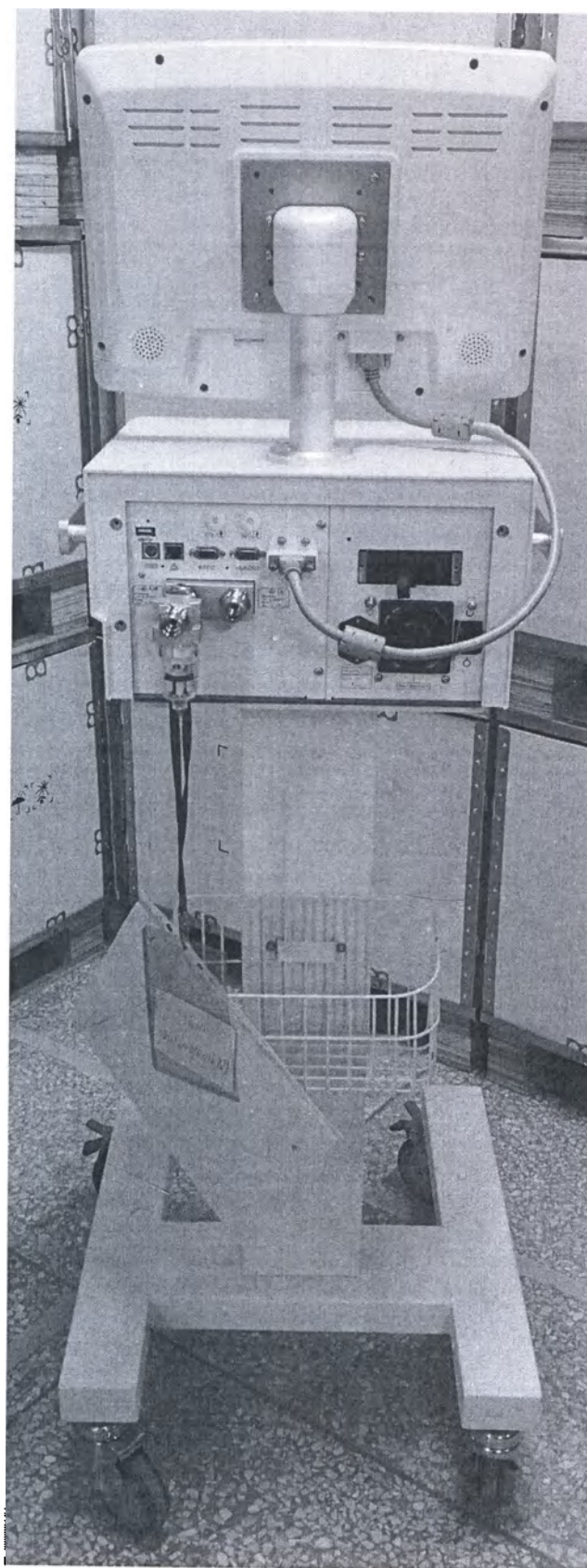
9.2 Disassemble the O-ring from the slot of the aluminum base.

Base O-ring

9.3 Install a new O-ring to the slot of the aluminum base.

9.4 Use the hand to press the O-ring evenly into the slot.

9.5 Assemble the aluminum base back to the chamber body.

⚠ Avoid using nail or sharp tool to scratch or damage the seal-ring.

10. Product Specification

	VHC-25 Adult Male/ Male	VHC-25 Adult Male/ Female
Inlet	22mm Male	22mm Female
Outlet	22mm Male	22mm Male
Compressible Volume (Empty)	640 ml	640ml
Compressible Volume (Filled)	140ml	140ml
Maximum Water Level	500ml	500ml
Compliance (Empty)	0.85 ml/cmH2O	0.85 ml/cmH2O
Compliance (Filled)	0.4 ml/cmH2O	0.4 ml/cmH2O
Maximum Flowrate	90LPM	90LPM
Maximum Operation Pressure	165cmH2O	165cmH2O
Gas leakage rate under maximum operation pressure	<5ml / min	<5ml / min

11. Explanation of Symbols

Caution	Consult instructions for use	Manufacturer	Date of manufacture	Use-by date	Authorized representative in the European Community
Batch code	Catalogue number	Fragile, handle with care	This way up	Caution: Federal law restricts this device to use by or on the order of a licensed physician.	

12. Customer Service Contact Information

Vincent Medical Manufacturing Co., Ltd.

Flat/RM B2, 7/F., Hang Fung Industrial Building, Phase 2, 2G Hok Yuen Street, Hung Hom, Kowloon, Hong Kong

Tel: +852 2365 5688

Fax: +852 2765 8428

Website: www.inspired-medical.com

Rx ONLY

CE 0197

Vincent Medical Manufacturing Co., Ltd.

Flat/RM B2, 7/F., Hang Fung Industrial Building, Phase 2, 2G Hok Yuen Street, Hung Hom, Kowloon, Hong Kong

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