BIOTEKNE SRL

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BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

BIOTEKNE SRL

Tassi Fabio

Legal Representative

BIOTEKNE SRL

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40033 CASALECCHIO DI RENO (BO)

Tel. 051 56 52 11 - Fax 051 61 82 524

e-mail: info@biotekne.it

C.F. 10826570151 - P.I. 02027231204

Instructions for use

MESO-RELLE Sterile Needle for Mesotherapy

Manufactured by:

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2019

Table of contents

1. Name
2. Design options
3. Intended use
4. Intended use
5. Indications
6. Contraindications
7. Adverse effects
8. Safety requirements, precautions
9. General information
10. Method of application
11. Conditions of use
12. Storage and transportation conditions
13. Scope of supply
14. Packaging and labeling
15. Sterilization
16. Disposal procedure
17. Warranty obligations
18. Shelf life
19. Authorized representative of the manufacturer in the Russian Federation

BIOTEKNE s.r.l.

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40033 Casalecchio di Reno (BO)

Tel. +39 051 565211 • Fax +39 051 6182524

e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204

R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

1. Name

MESO-RELLE sterile needle for mesotherapy (hereinafter referred to as the "MESO-RELLE needle")

2. Design options

27 G - outer diameter 0.40mm; length 4mm, 6mm, 12mm, 25mm, 40mm

30 G - outer diameter 0.30 mm; length 4mm, 6mm, 12mm, 25mm, 40mm

31G - outer diameter 0.26mm; length 4mm, 6mm, 12mm, 25mm

32G - outer diameter 0.23mm; length 4mm, 6mm, 12mm, 25mm

33G - outer diameter 0.20 mm; length 4mm, 12mm

3. Intended use.

Sterile MESO-RELLE needle for mesotherapy (hereinafter referred to as the "MESO-RELLE Needle") is a disposable needle intended for subcutaneous injections and administration of a special "cocktail", a set of various drugs - to a depth of 0.5 to 6 mm, depending on the patient's problem.

MESO-RELLE needle is intended for administration of fillers into the skin or subcutaneous layers to add additional tissue volume for cosmetic or reconstructive purposes.

4. Intended use

Cosmetology, aesthetic medicine (mesotherapy).

5. Indications

for administering fillers into the skin or subcutaneous layers

6. Contraindications

- Skin diseases.
- Chronic vascular pathology (3rd degree hypertension, 3rd degree IHD); chronic renal failure, nephropathy.
- Pregnancy 3 trimesters.
- Gallstone disease.
- Acute febrile illnesses.
- Blood coagulation disorders due to illness (hemophilia), as well as the use of anticoagulants and antiplatelet agents (heparin, coumarins, aspirin, ticlopidine).

BIOTEKNE s.r.l.

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40033 Casalecchio di Reno (BO)

Tel. +39 051 565211 • Fax +39 051 6182524

e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204

R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

Contraindication as to the use of MDs also include contraindications as to administration of fillers in general, particularly:

General contraindications:

- allergic reactions or individual intolerance to the components of the drug or product;
- autoimmune diseases and systemic lesions of connective tissue;
- mental disorders; infectious diseases;
- severe somatic diseases; exacerbation of any chronic diseases.; anaphylactic shock or history of "rejection" of a foreign body; hemophilia, administration of anticoagulants and antiplatelet agents; bleeding disorder;
- recurrent tonsillitis and fever.

Local contraindications:

- inflammatory process, violation of the integrity of the skin and availability of a non-resorbable implant in the area where it is planned to make a puncture;
- availability of other fillers, biodegradable materials at the sites of injection based on polylactic acid, collagen or calcium hydroxyapatite;
- malignant neoplasms at the site of injection; recent chemical peeling, laser or mechanical resurfacing procedures;
- propensity for keloid and hypertrophic scars;
- viral and fungal skin diseases.

7. Adverse effects

- Pain
- Hematoma
- Local edema
- Erythema
- Induration
- Teleangiectasia
- Infectious complications

8. Safety requirements, precautionary measures.

- before use, check the packaging of the needle for integrity. Do not use the needle, if the packaging has been damaged.
- check the needle before use;
- do not use a damaged needle;
- after the procedure, drop the needle into the container for disposal or into its analogue;
- handle needles in accordance with the rules adopted by your institution;
- do not use an outdated needle;
- try not to put caps on the used needles;

BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

- before working with needles, plan your actions in advance, including those related to the disposal of needles;
- throw out the used needles into a special waste container in a timely manner;
- immediately notify of all injuries when working with needles and other sharp objects - this will help you to get the necessary medical care on time.

9. General information.

MESO-RELLE needles represent an effective tool for intradermal administration of fluids, especially in the fields of mesotherapy, sclerotherapy and injection with fillers;

The MESO-RELLE needle consists of a metal tube pointed at one end. At the other end, the needle is attached to a connector that fits onto a syringe or an injection device. The needle has a safety cap and is packed in a sterile package.

10. Administration

A physician chooses the route of administration and possible combinations of agents in each individual case, being guided by approval documentation for the use of the respective agent/agents. To increase the efficiency of mesotherapy prior to the start of treatment, it is advisable to conduct a course of surface peeling.

Preparation of a patient immediately before the mesotherapy session consists in cleansing and disinfecting the skin with 0.05% chlorhexidine or 76% alcohol (according to indications). A mesotherapy session is carried out by administering mesotherapeutic agents into the skin using various injection techniques or using special needles for mesotherapy.

The procedure is carried out using a syringe with a needle. Due to precise control over the depth and site of injection, a physician may carry out the procedure on the most delicate or inaccessible areas: eyelids (by palpating the bone edge of the orbit), wings of the nose, etc. The injection site is completely controlled by the physician, therefore the likelihood of damage to the blood and lymph vessels is sharply reduced and, as a result, the invasiveness of the procedure is reduced.

The needle is inserted into the skin at an angle of 30-45 degrees, with the bevel up, while keeping a distance of 10 mm between the injections.

Table for the needle selection, depending on the intended use

Dimensions	Intended use	Depth of injection
33G	Recommended for delicate correction of: - lips and paraoral zone - purse-string wrinkles - periorbital zone - glabella region - nose bridge - лоб	Superficially subcutaneously
32G	- puppet wrinkles - lacrimal groove	Non-deeply subcutaneously

BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000.00 di cui versato € 33.500.00

	- forehead - back of hands - glabella region - crow's feet wrinkles - area around the eyes - purse-string wrinkles - nasolabial fold - lips	
31G	- eyebrow wrinkles - nasolabial folds - neck and décolleté - corners of the mouth - lacrimal groove - nose - the middle third of the face, cheeks, cheekbones, nasojugal fold - face contour - lips - chin	Non-deeply subcutaneously
30G	- cheeks/cheekbones/chin - nasolabial and glabellar wrinkles - nose - face contour - lateral and medial sides of the thigh - belly - infrabuttock crease - inner side of the shoulder - paravertebrally	Into the adipose tissue (subcutaneous fat), periosteally, subcutaneously
27G	- cheeks/cheekbones/chin - nasolabial and glabellar wrinkles - nose - face contour	Into the adipose tissue (subcutaneous fat), periosteally, subcutaneously

11. Conditions of use

Air temperature + 15 °C - + 25 °C;
Relative humidity (not more than 70) ± 2%;
Atmospheric pressure 755 ± 20 mm Hg

12. Storage and transportation conditions

BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

Transportation conditions

Air temperature: +2 °C - +25 °C
Relative humidity: no more than 70±2%;
Atmospheric pressure: 755±20 mm Hg

Storage conditions

Air temperature: +2 °C - +25 °C
Relative humidity: no more than 70±2%;
Atmospheric pressure: 755±20 mm Hg
Store in the original packaging, in closed, dry, cool premises. Avoid direct sunlight, high temperatures and high humidity. Packaging must be protected from damage.
Keep out of the reach of children. Storage temperature +2 °C - + 25 °C.

13. Scope of supply

"MESO-RELLE sterile needles for mesotherapy" are packed in a group package in the form of a cardboard box in the amount of 100 pieces. A large shipping box contains 2 small shipping boxes, each of which contains 50 group packs of needles (5,000 pcs.).

Large shipping box: weight 14 kg, size 43x45x43 cm

Small shipping box: weight 6.3 kg, size 42x20x40 cm

14. Packaging and labeling

Individual sterile packaging

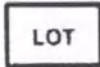
- Product Denomination and Logo
- Name of Manufacturer
- Country of Origin
- Model
- Needle size
- Sterility
- Batch code with indication of sterilization date
- Shelf life (best before)
- Warning about the need to check the integrity of the package (see the symbol)

Explanation of symbols used when labeling any individual packaging:

Symbol	Explanation
	European Union Conformity Mark
	Needle sterilization with ethylene oxide

BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

	Reprocessing is not allowed
	Do not use, if the packaging has been damaged.
	Shelf life
REF	Catalog number
	Lot

15. Sterilization

"MESO-RELLE sterile needles for mesotherapy" are sterilized with ethylene oxide.

16. Disposal considerations

Disposal must be performed in accordance with regulatory and local laws and the protocols of a particular medical institution.

According to the hazard class of medical waste during its disposal, the medical product may be classified as epidemiologically hazardous waste (infected and potentially infected waste contaminated with blood and/or other biological fluids).

17. Warranty obligations

Shelf life subject to storage conditions is 3 years from the date of manufacture, on condition of compliance with all storage requirements specified by the manufacturer. The warranty obligations cover the entire declared shelf life of the product and apply to the products subject to compliance with the conditions of storage, transportation, operation.

The manufacturer guarantees that the present medical product has been manufactured and packaged with due care prescribed in the EU Directive 93/42/EEC for medical products. The manufacturer has EU Certificates, indicating that the company's quality management system complies with the requirements of Appendix II (excluding section 4) of Directive 93/42/EEC, ISO 13485: 2003.

The medical products have been individually tested and checked with due diligence. The manufacturer will replace any product in case of detection of the defects made during manufacture and packaging (evidence may be provided), based on the reports on such defects sent to the address of the Manufacturer or its authorized representative or distributor. This warranty obligations apply in lieu of any other warranty, verbal or written, express or implied, and also excludes it.

The physician using this product is responsible for visual determination of suitability of the product for use on patients.

The manufacturer and its authorized representative or distributors are not responsible for any harm or damage of any nature that occurred directly or indirectly when using the product.

BIOTEKNE s.r.l.
Via della Bastia, 9
40033 Casalecchio di Reno (BO)
Tel. +39 051 565211 • Fax +39 051 6182524
e-mail: info@biotekne.it

C.F.: 10826570151 • P.IVA: 02027231204
R.I. BO: 65263/1999 • R.E.A.: 388619 • C.S.: € 80.000,00 di cui versato € 33.500,00

If this product has become defective within the warranty period, your exclusive option and the Manufacturer's sole responsibility should be to improve or replace the product. Liability with regard to the product ceases, if any damage occurs due to transportation, storage or improper handling by the user, as well as other factors beyond the manufacturer's control.

18. Shelf life

Shelf life - 5 years from the date of manufacture.

19. Authorized representative of the manufacturer in the Russian Federation

Information about the authorized representative on the issues of circulation of a medical product in the territory of the Russian Federation:

Trium Trade Limited Liability Company

1 Meshchanskaya str., building 3, Floor I, office I, rooms 1B, 1B, 3B, 3B, 3F, the city of Moscow, 129090

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*We certify that the signer is a duly authorised signatory on
behalf of the Company for any documents and acts to be used
in trade with foreign countries.*

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Nadia Rimondi

Chamber of Commerce
Industry, Handicraft and
Agriculture of Bologna

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