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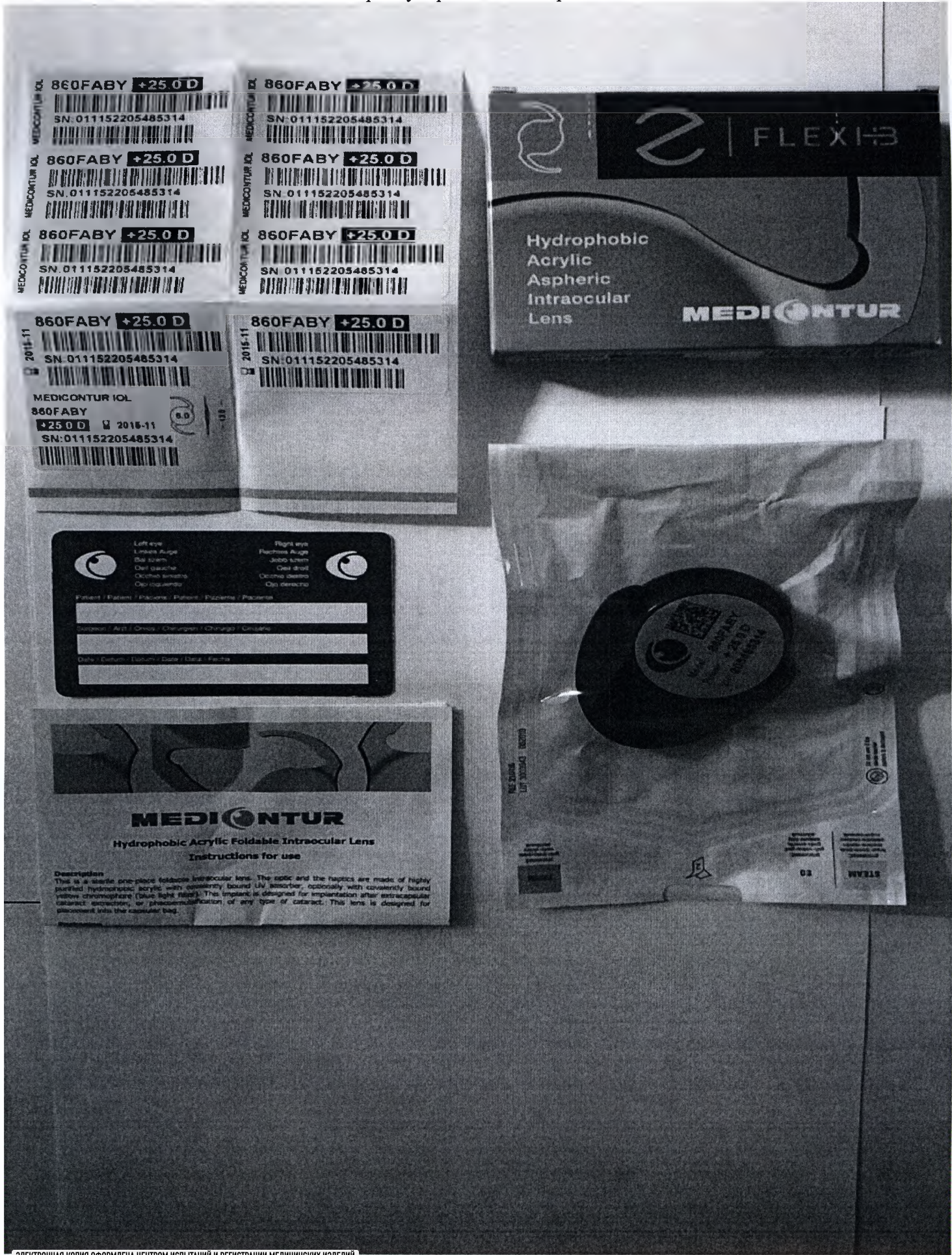
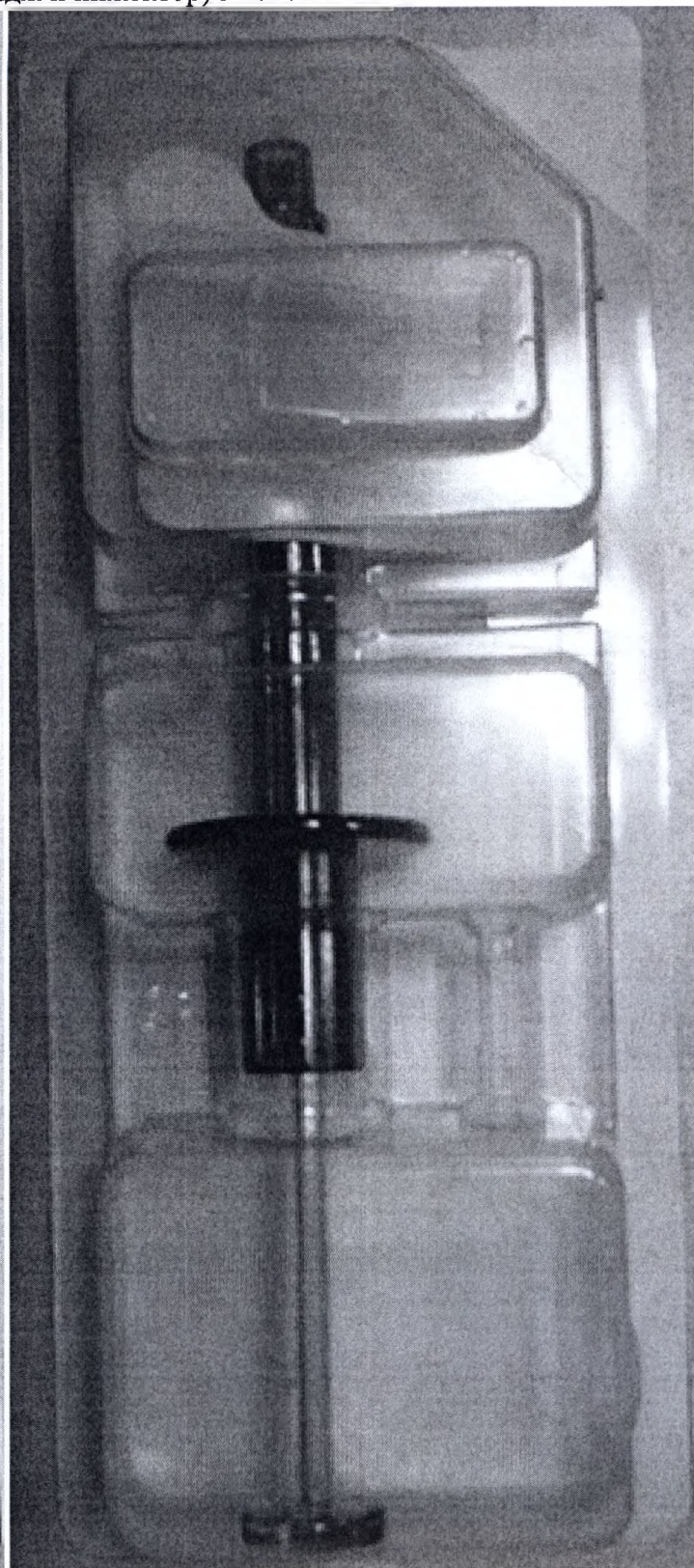
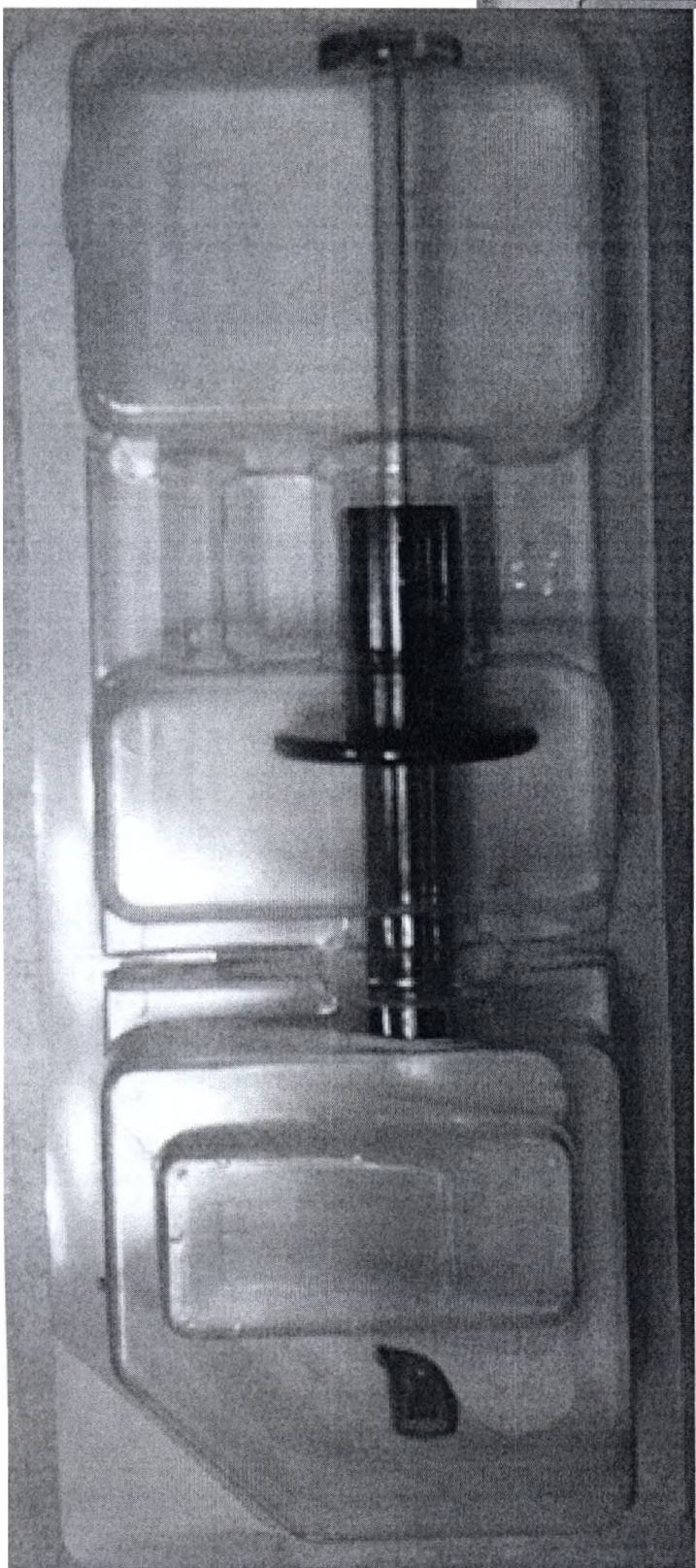
MEDI CONTUR

Hydrophobic Acrylic Foldable Intraocular Lens
Instructions for use

Description

This is a single-piece foldable intraocular lens. The optic and the haptics are made of highly purified hydrophobic acrylic with covalently bound UV absorber, optionally with covalently bound yellow chromophore (blue light filter). This implant is designed for implantation after the cataract surgical extraction, or phacolytic cataract of any type of cataract. This lens is intended for placement into the capsular bag.

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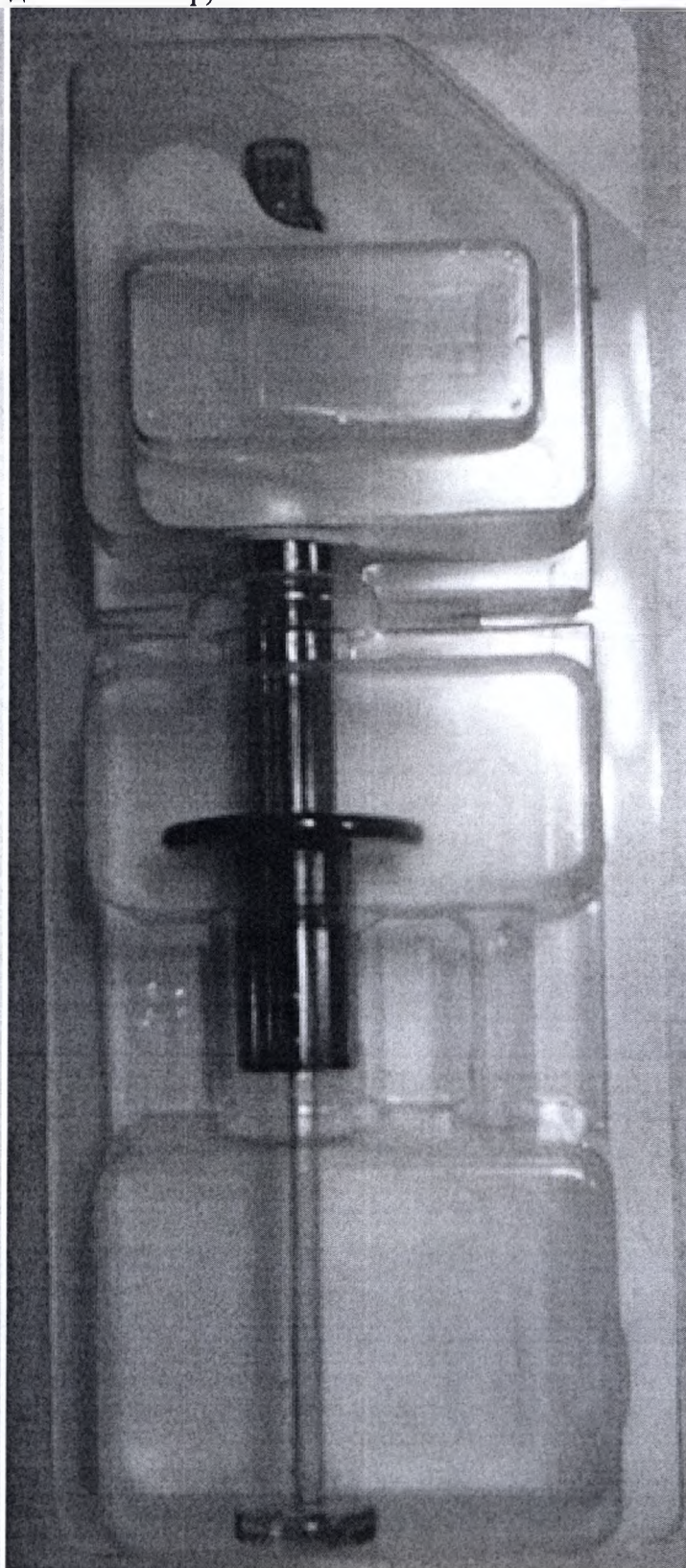
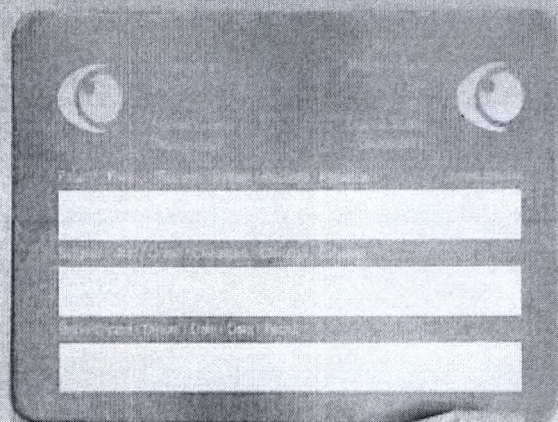
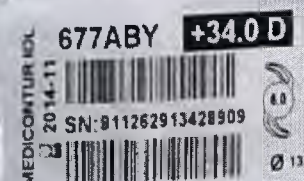
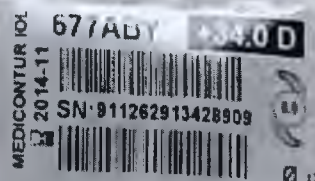
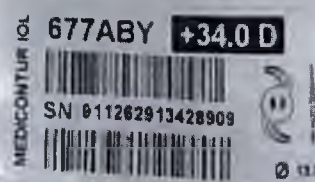
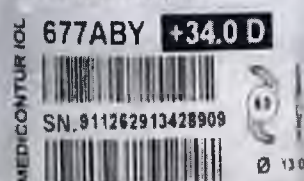
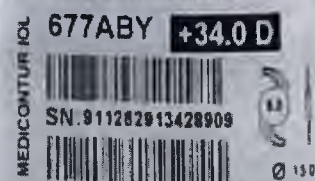
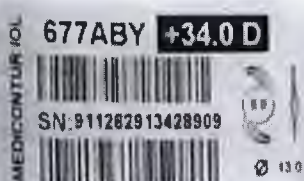
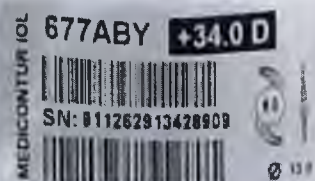
MEDICONTUR

Acrylic Foldable Intraocular Lens
Instructions for Use

Contents:

One acrylic foldable intraocular lens consisting of highly purified hydrophilic acrylic with covalently bound UV absorber, optionally with yellow chromophore up blue light filter.

Description:

The intraocular lens is an optical product of highest precision. The manufacturing and the quality management system of MEDICONTUR is in accordance with international standards and is certified according to ISO 9001 ISO 13485 and ISO 13495. The tolerance for the refractive power of a MEDICONTUR intraocular lens is ± 0.25 D in the range $+25.0$ D and ± 0.5 D in the range ± 25.0 D.

Packaging:

The hydrophilic lens is held by a lens holder (also lens folder) fixed in the cap of a vial containing sterile water. The vial is wrapped in a sterile pouch (folded bag).

Sterilization:

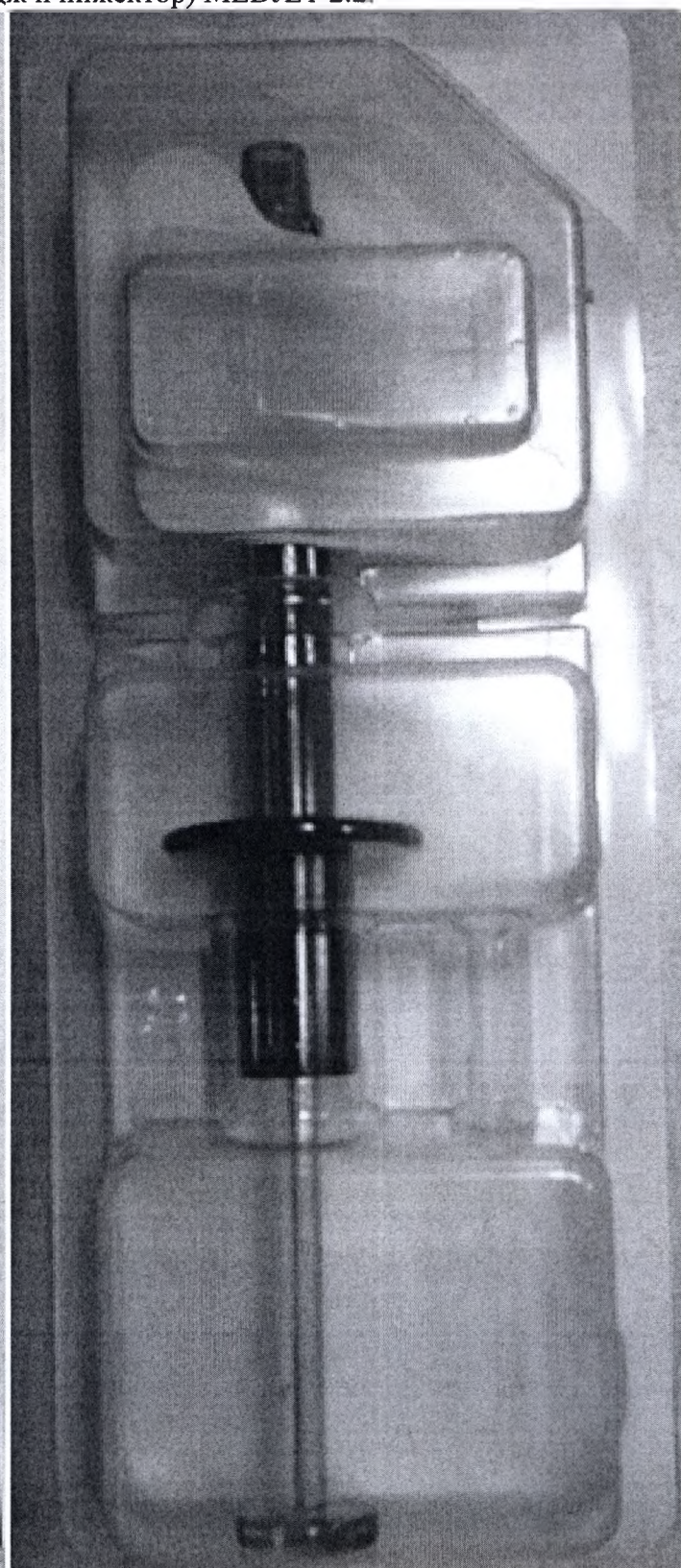
The hydrophilic lens has been sterilized by steam autoclave according to international standards.

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