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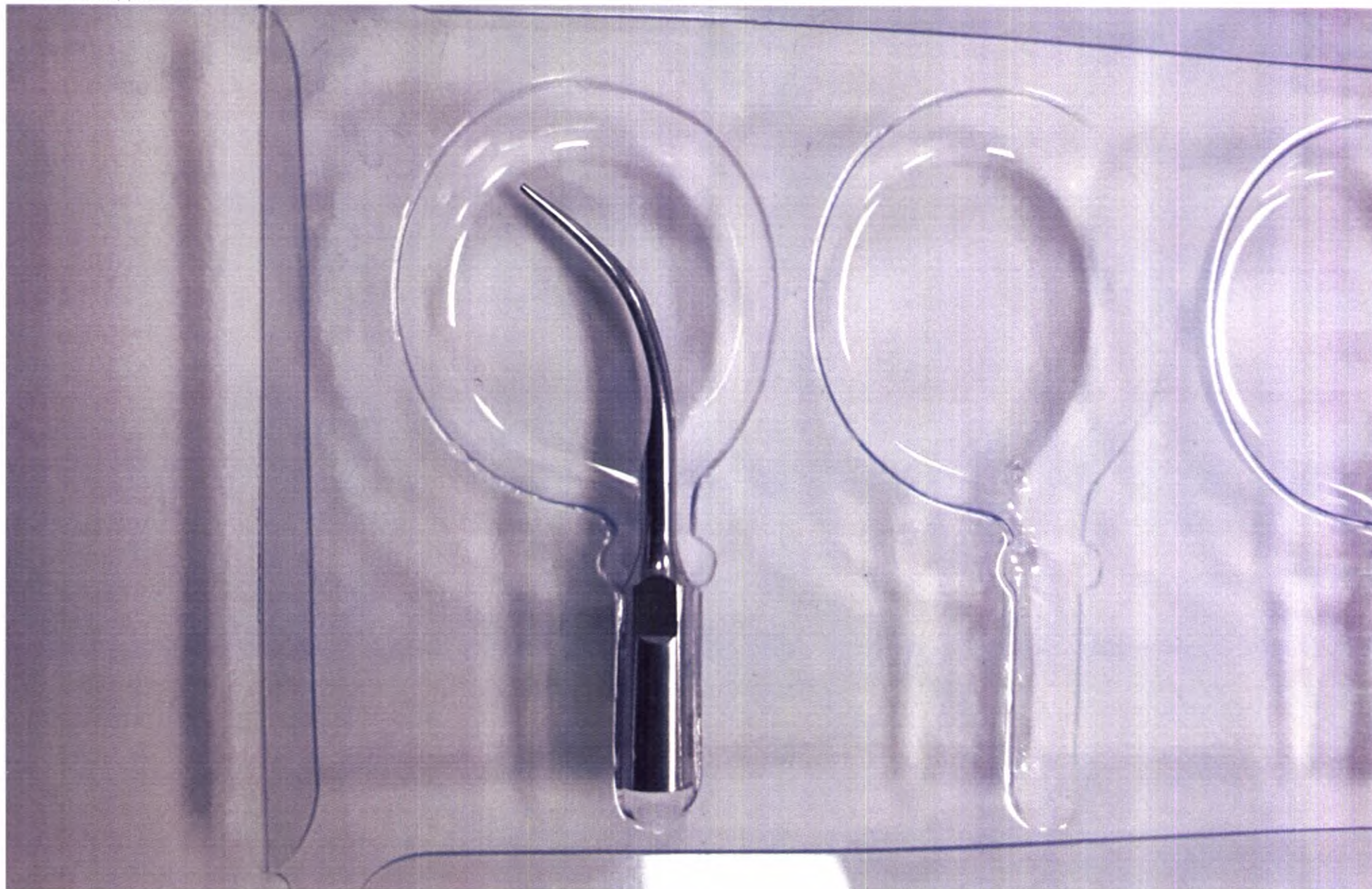
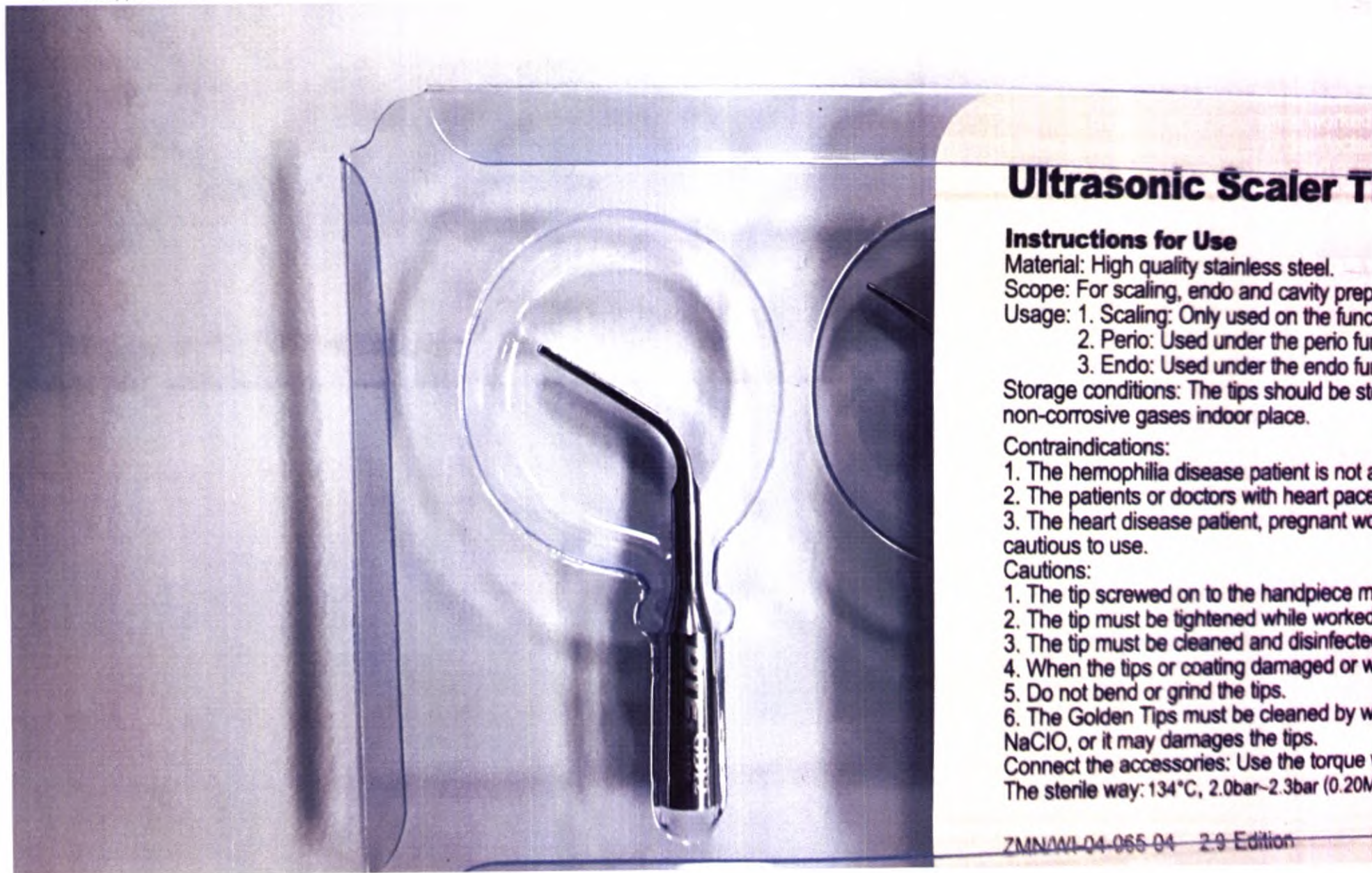
Ultrasonic Scaler T

Instructions for Use

Material: High quality stainless steel.

Scope: For scaling, endo and cavity prep.

Usage: 1. Scaling: Only used on the func

2. Perio: Used under the perio fur

3. Endo: Used under the endo fur

Storage conditions: The tips should be str
non-corrosive gases indoor place.

Contraindications:

1. The hemophilia disease patient is not a

2. The patients or doctors with heart pacer

3. The heart disease patient, pregnant wo
cautious to use.

Cautions:

1. The tip screwed on to the handpiece m

2. The tip must be tightened while worked

3. The tip must be cleaned and disinfected

4. When the tips or coating damaged or w

5. Do not bend or grind the tips.

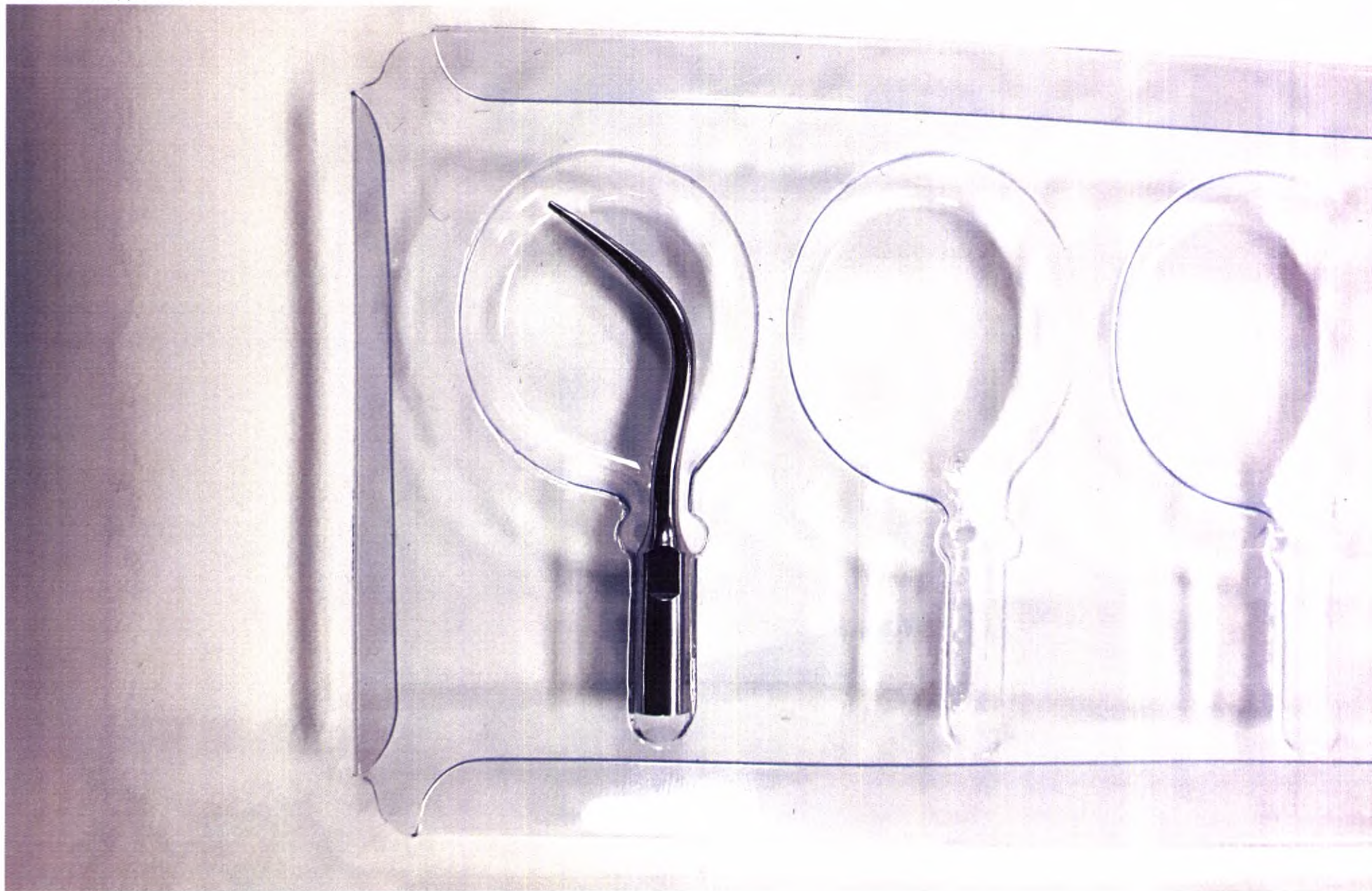
6. The Golden Tips must be cleaned by w
NaClO, or it may damages the tips.

Connect the accessories: Use the torque v

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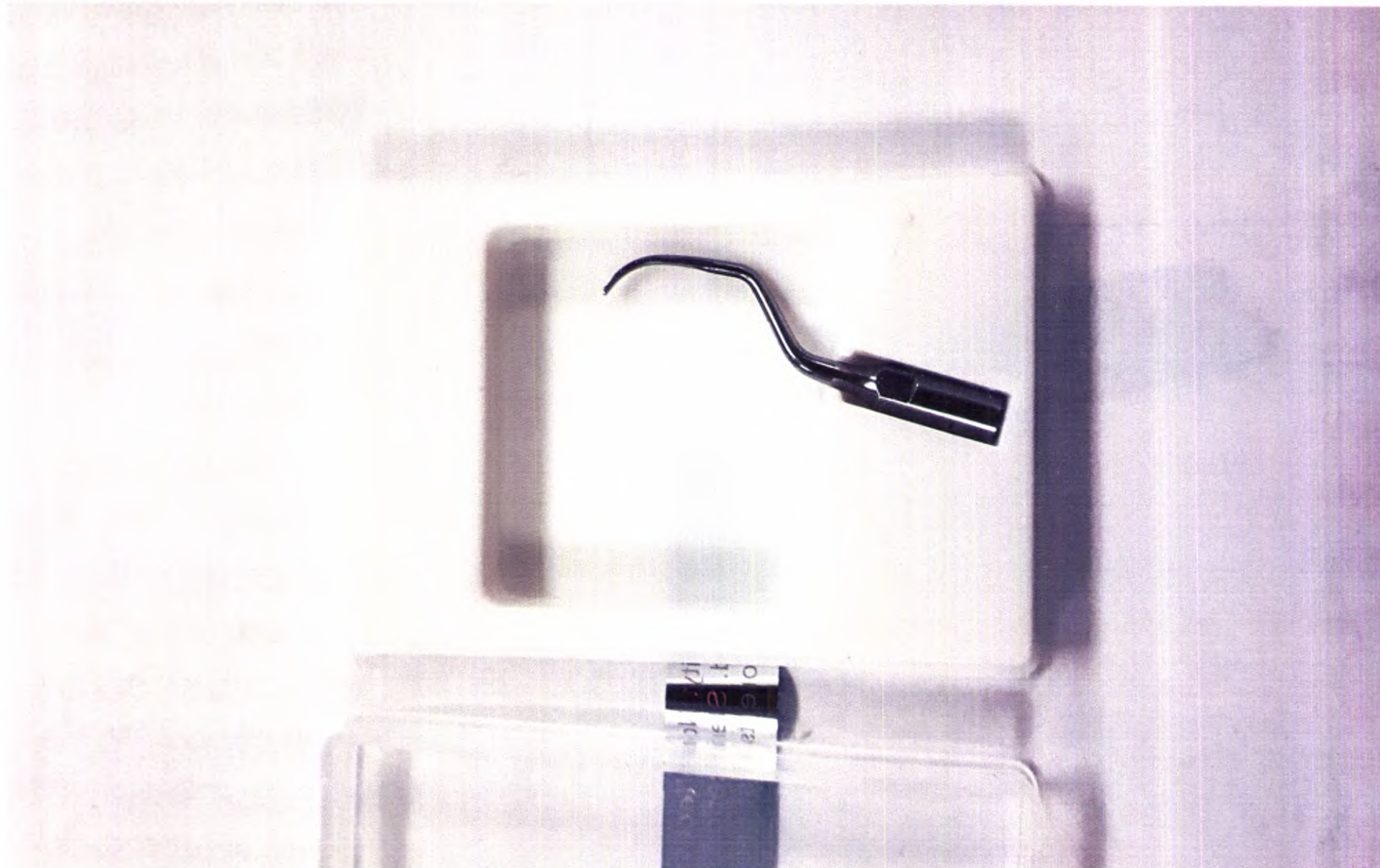
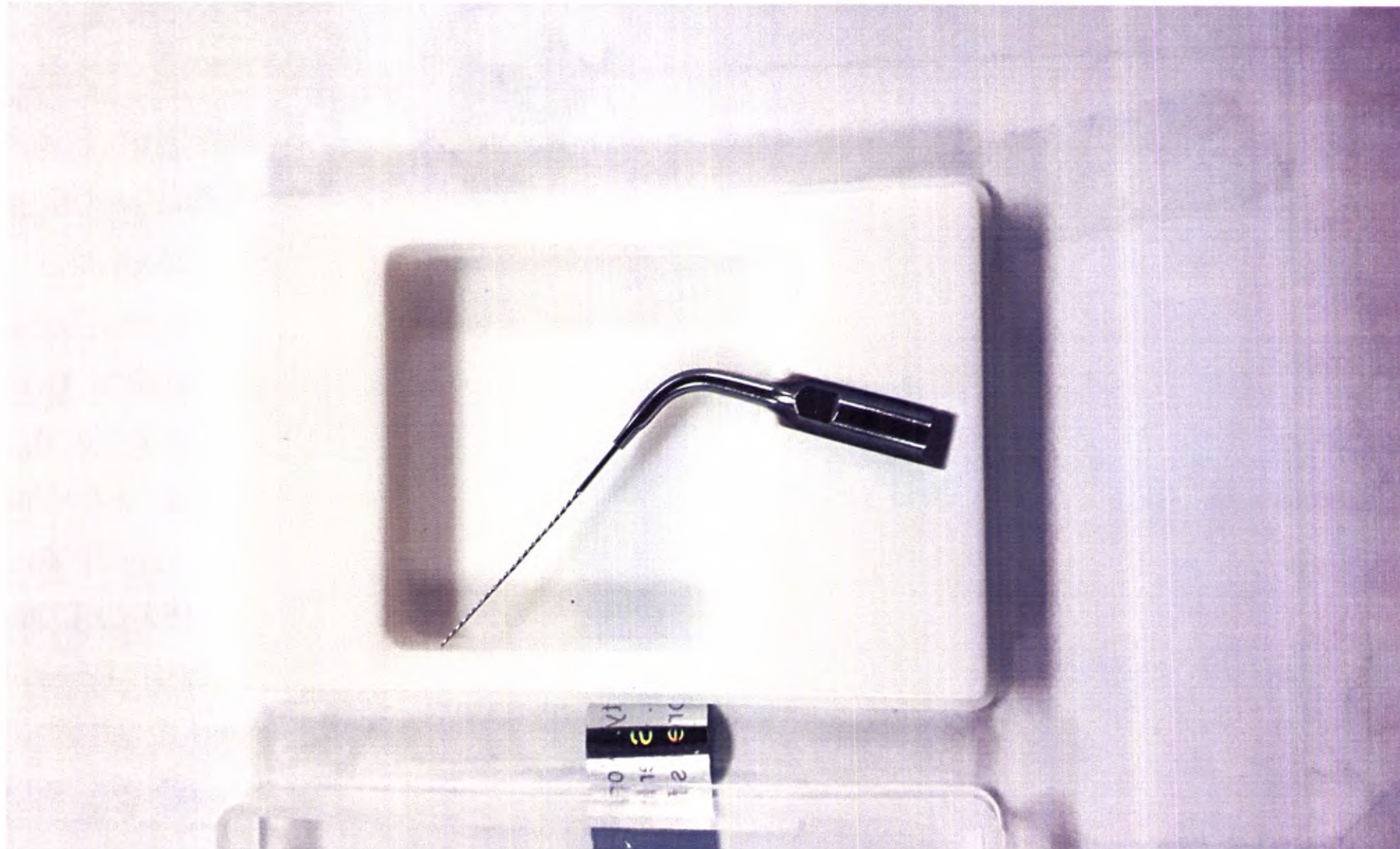
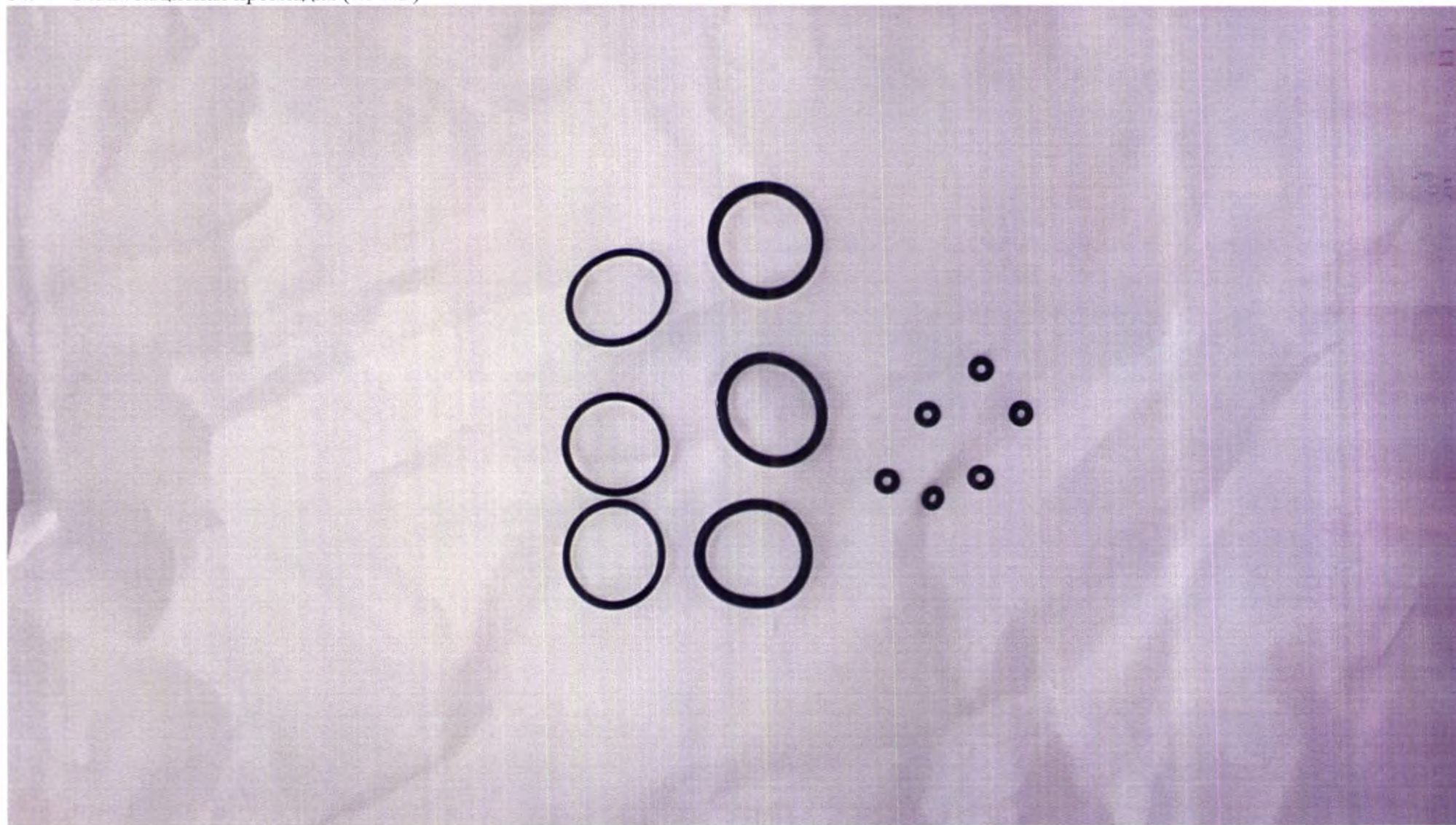
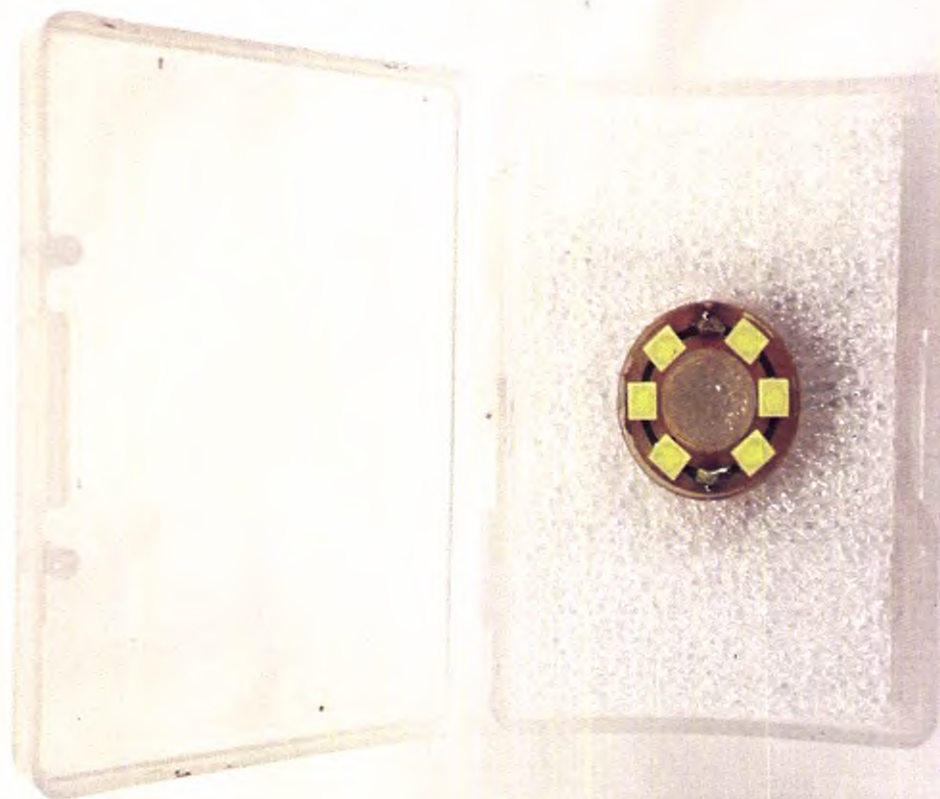
S6 LED ULTRASONIC SCALER INSTRUCTION MANUAL

CE 0197

Guilin Woodpecker Medical Instrument Co., Ltd.

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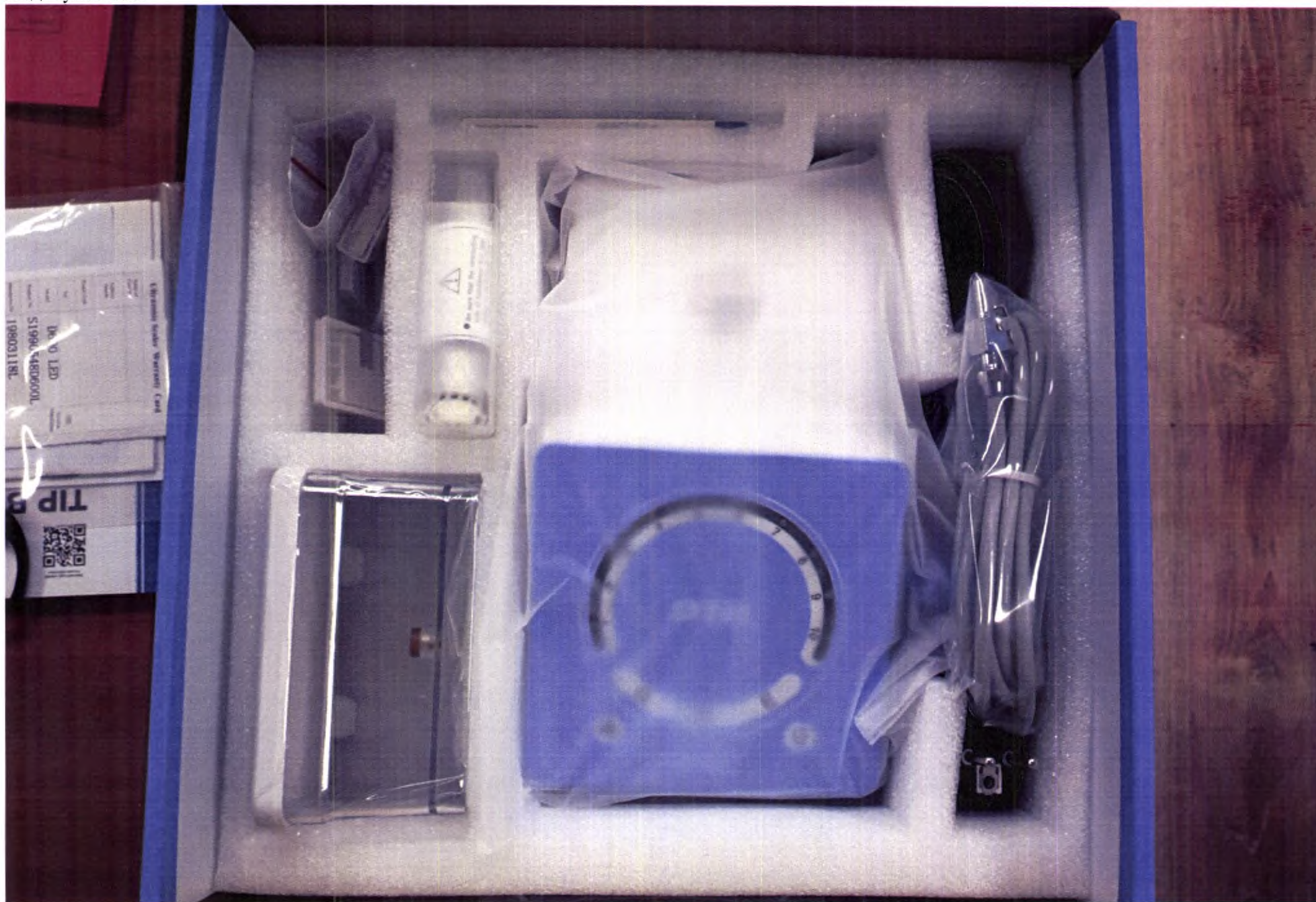
Product Name: Ultrasonic Scaler Model: D600 LED
 Guilin Woodpecker Medical Instrument Co., Ltd.
 Information Industrial Park, Guilin National
 High-Tech Zone, Guilin, Guangxi, 541004 P. R. China
 Website: <http://www.glwoodpecker.com>

  MedNet GmbH
 Borkstrasse 10 · 48163 Muenster · Germany

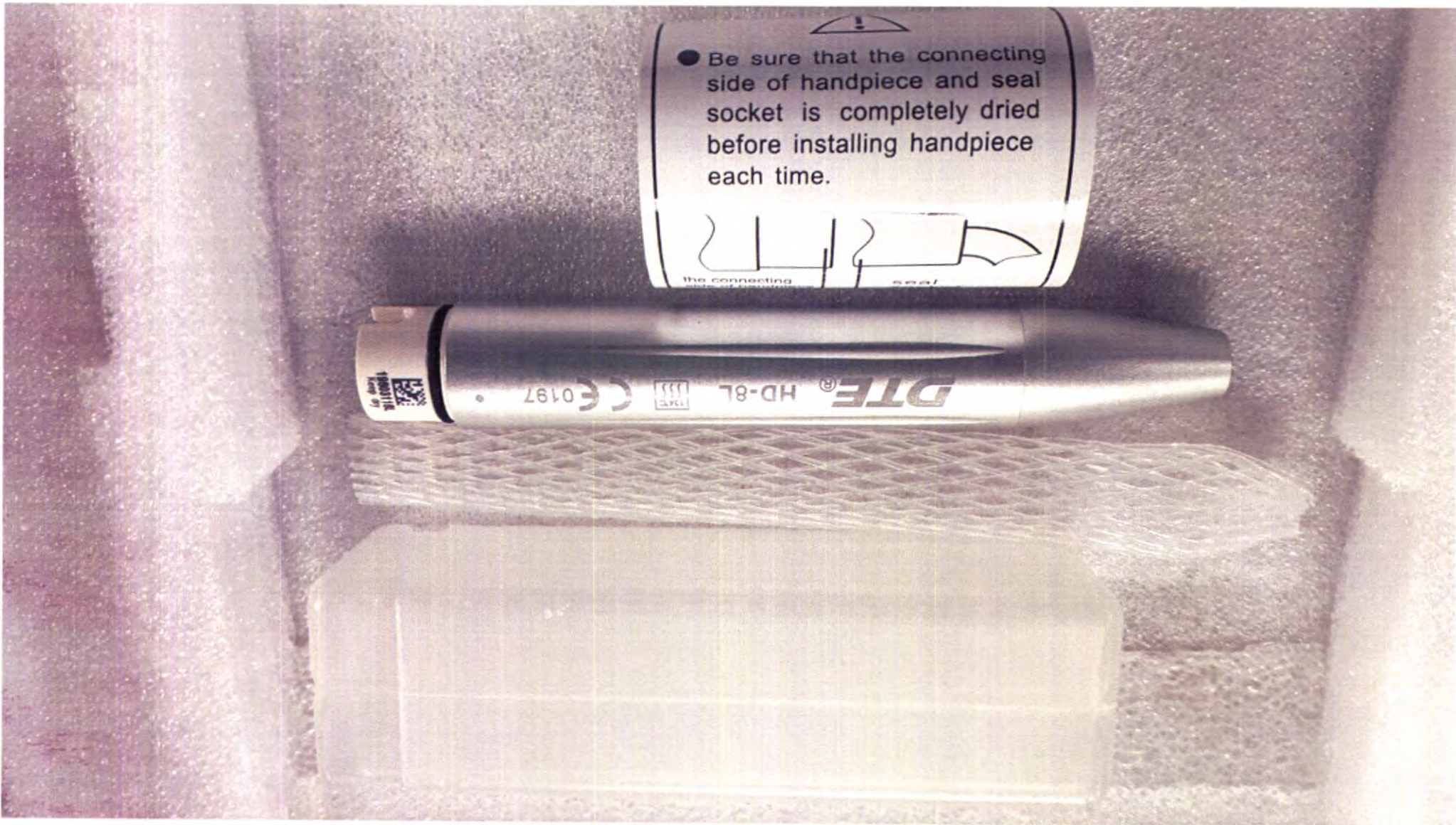
Rated input: 220-240V~ 50Hz/60Hz 150mA

2019 



 0197

S1990548D600L

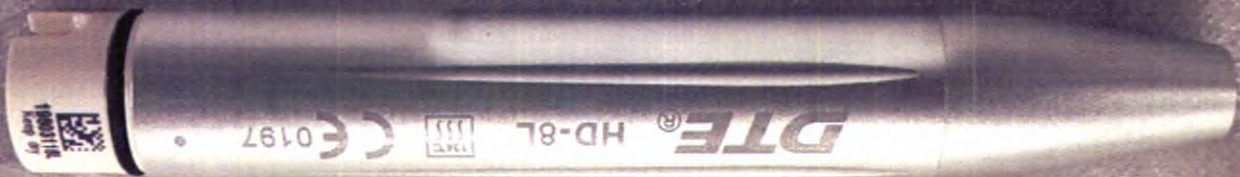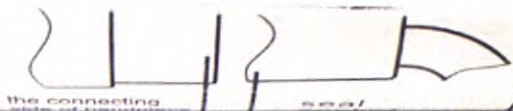● Be sure that the connecting side of handpiece and seal socket is completely dried before installing handpiece each time.

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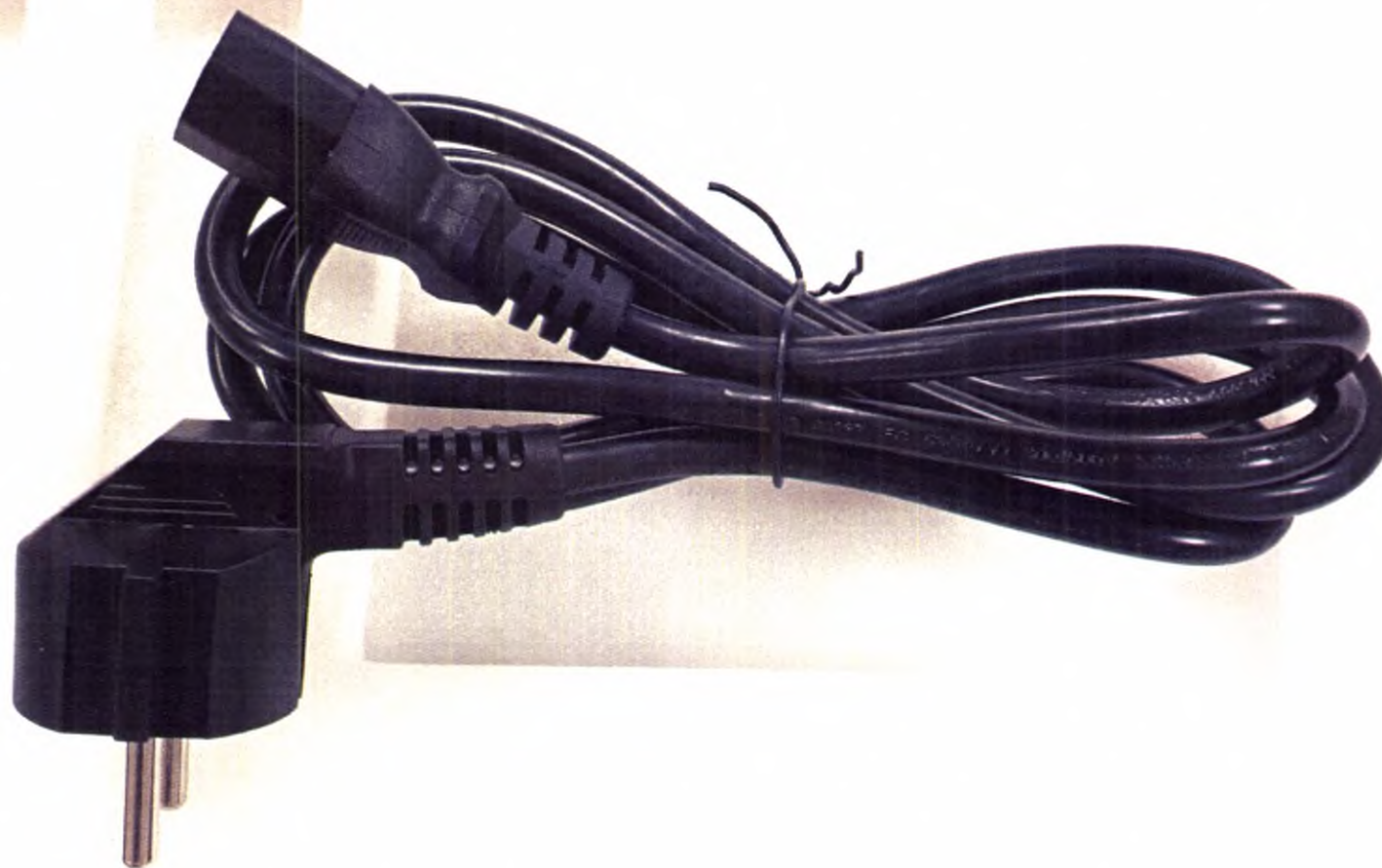
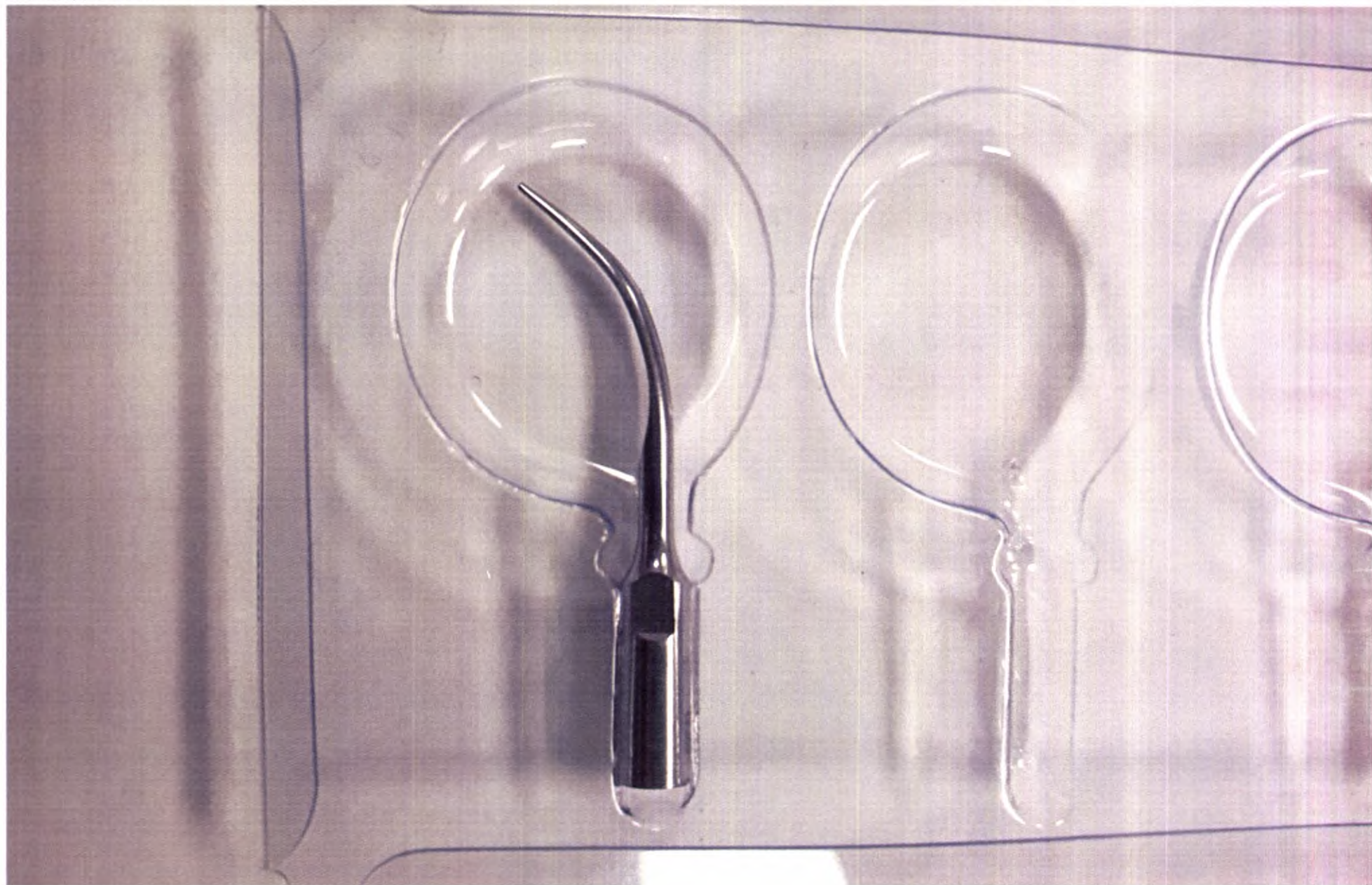
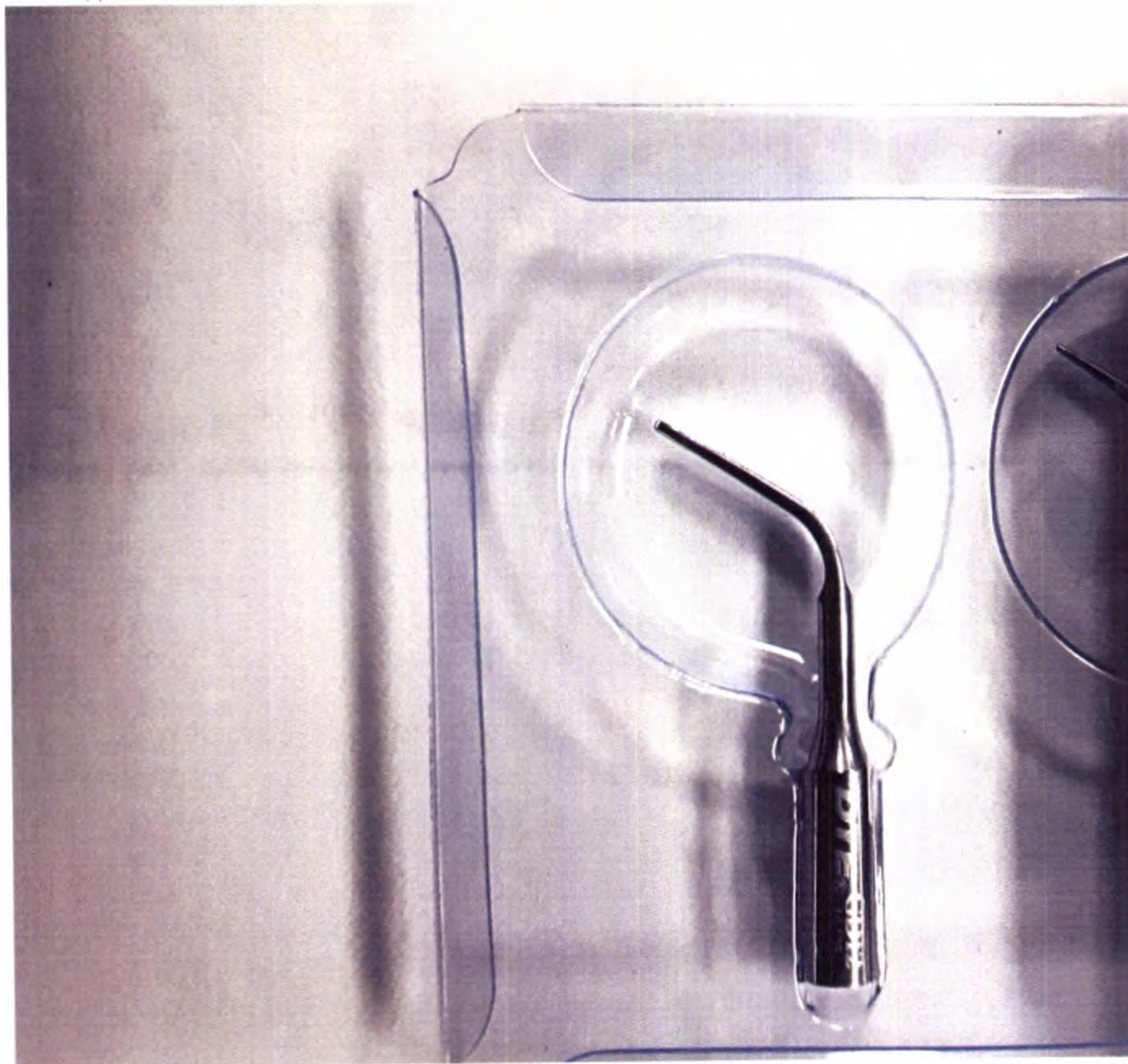
Ultrasonic Scaler Tip

Instructions for Use

Material: High quality stainless steel.

Scope: For scaling, endo and cavity prep.

Usage: 1. Scaling: Only used on the fund

2. Perio: Used under the perio fur

3. Endo: Used under the endo fur

Storage conditions: The tips should be stored in a dry, non-corrosive gases indoor place.

Contraindications:

1. The hemophilia disease patient is not a

2. The patients or doctors with heart pacer

3. The heart disease patient, pregnant women should be cautious to use.

Cautions:

1. The tip screwed on to the handpiece must be

2. The tip must be tightened while worked

3. The tip must be cleaned and disinfected

4. When the tips or coating damaged or worn, it must be replaced.

5. Do not bend or grind the tips.

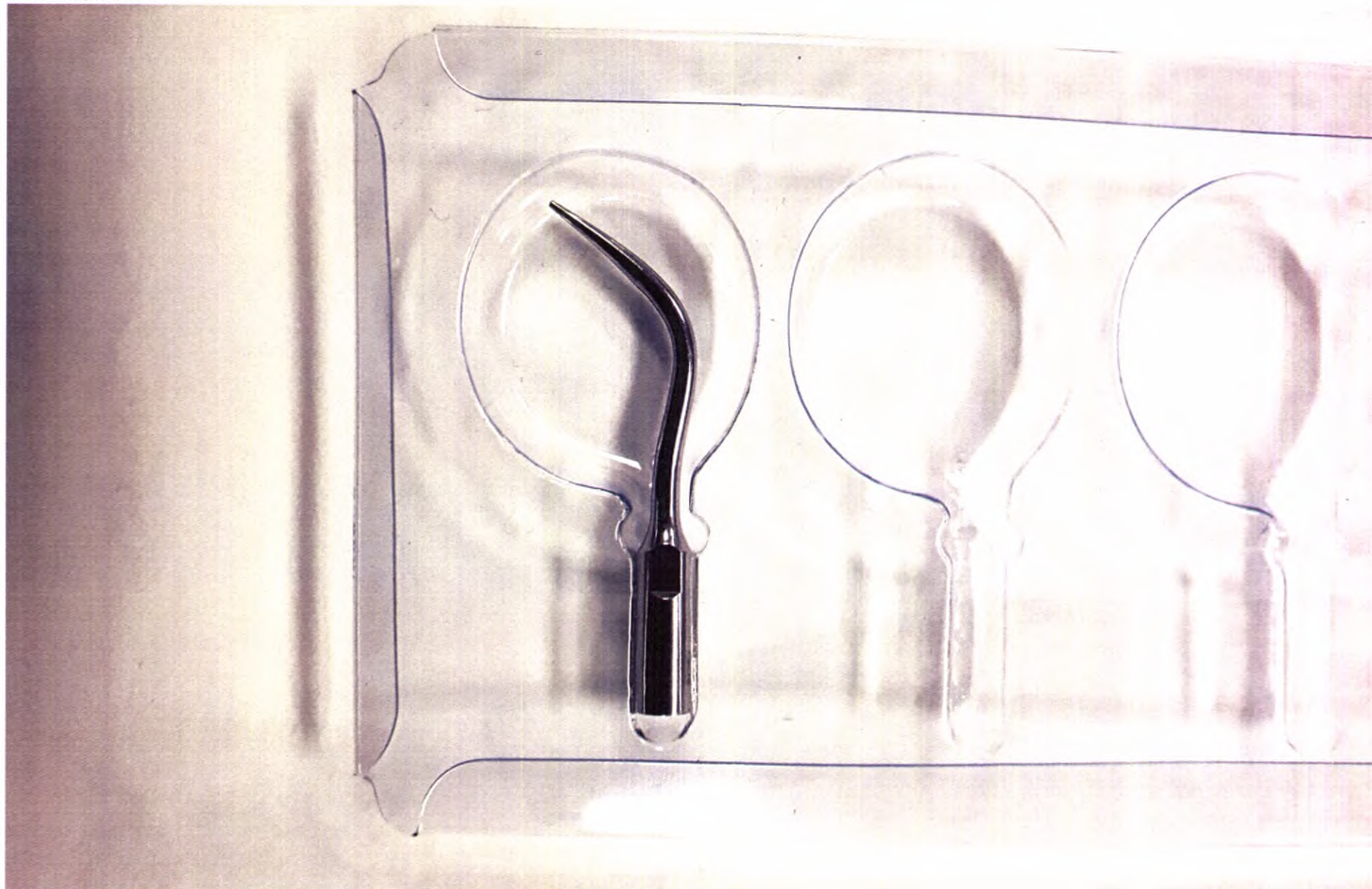
6. The Golden Tips must be cleaned by weak acid solution (e.g. NaClO), or it may damage the tips.

Connect the accessories: Use the torque wrench to tighten the tips.

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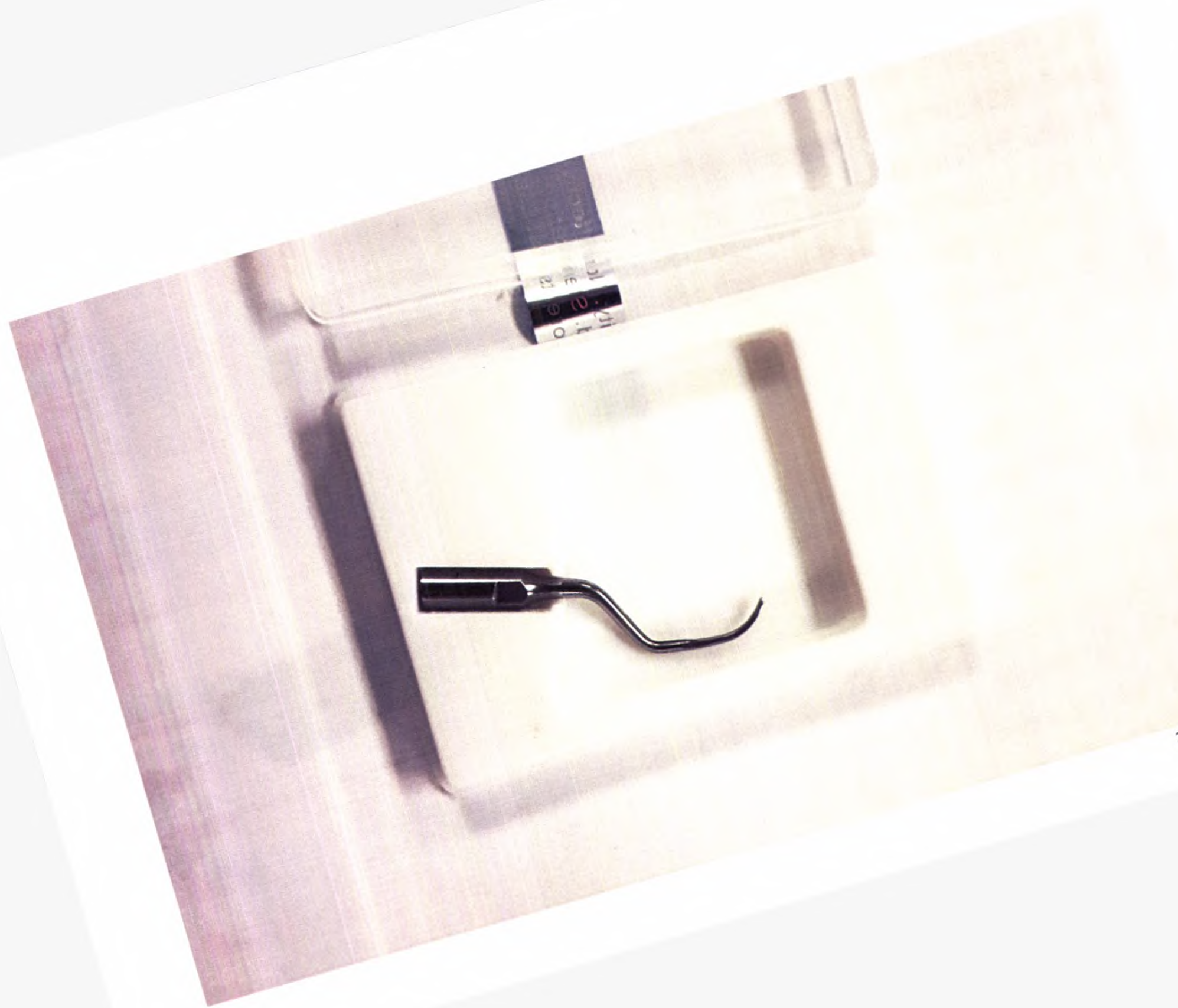
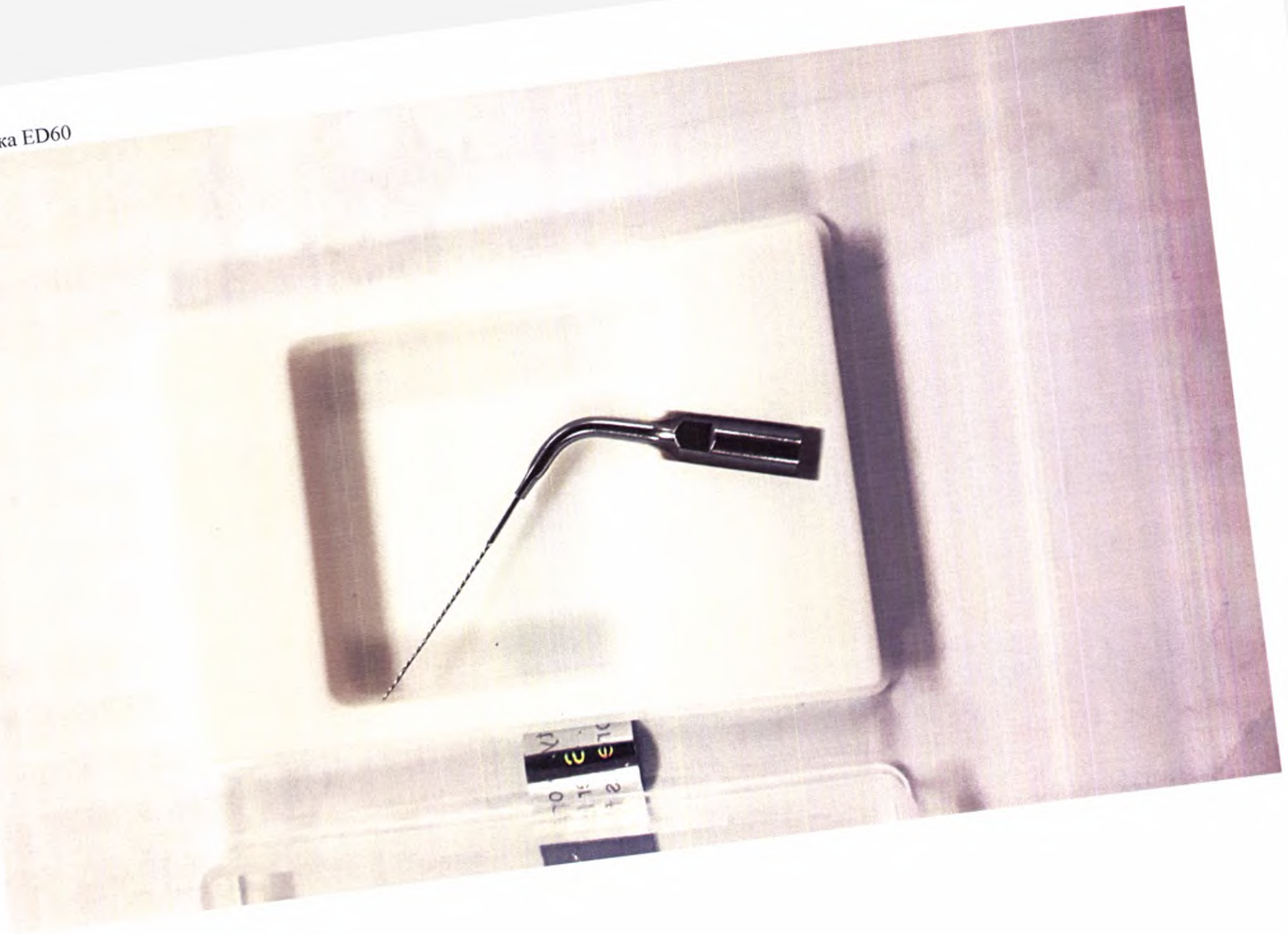
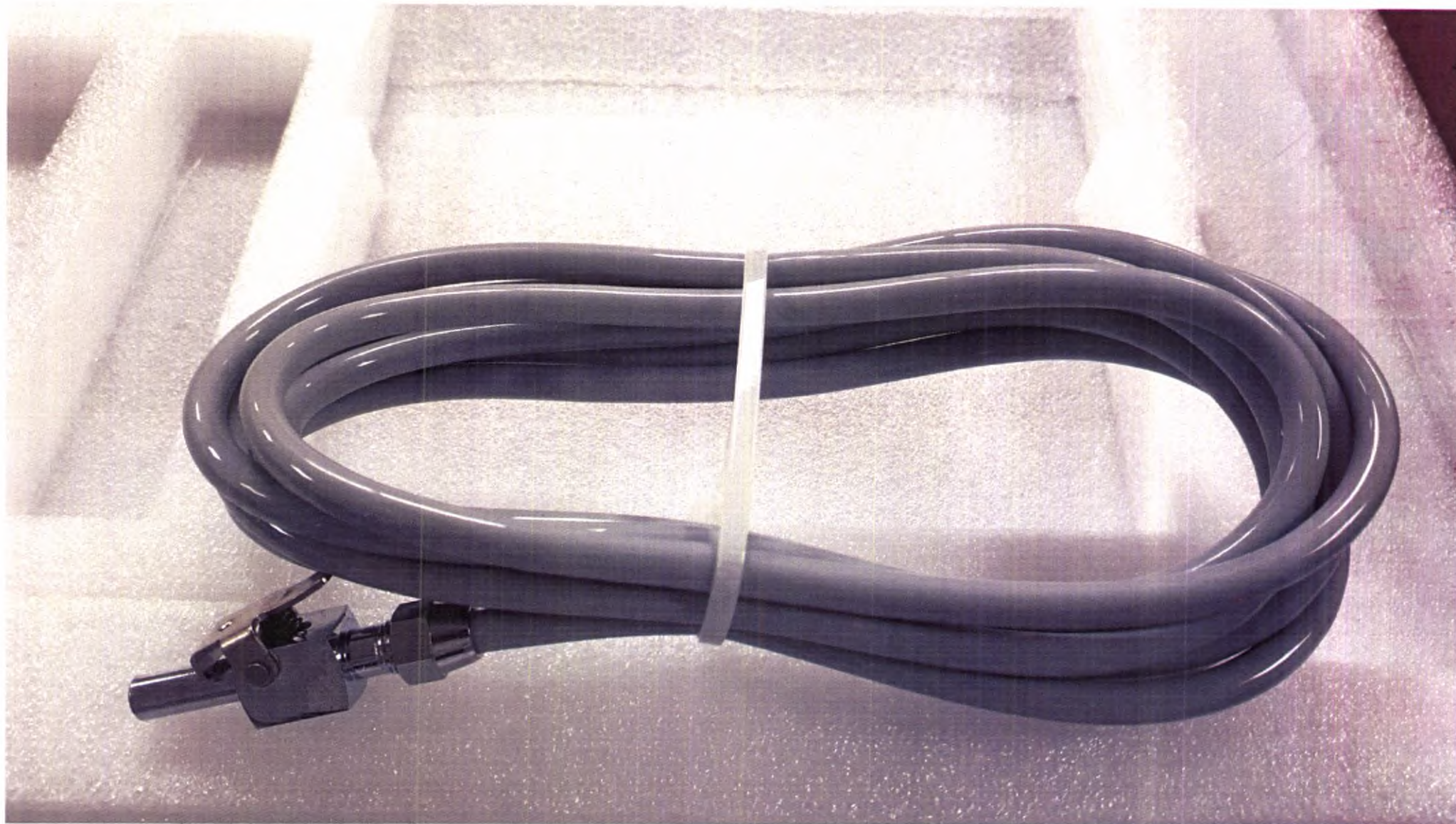
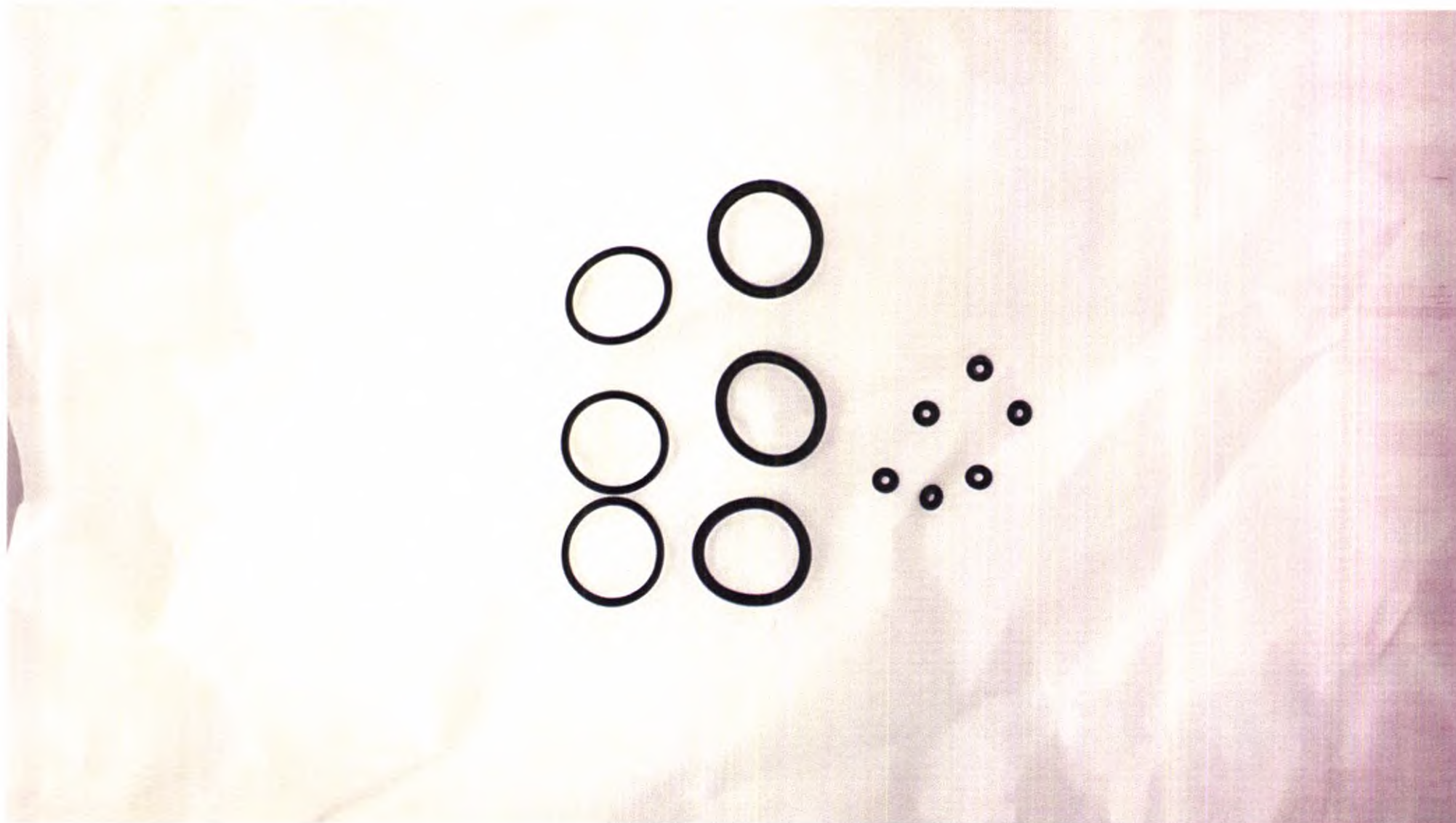
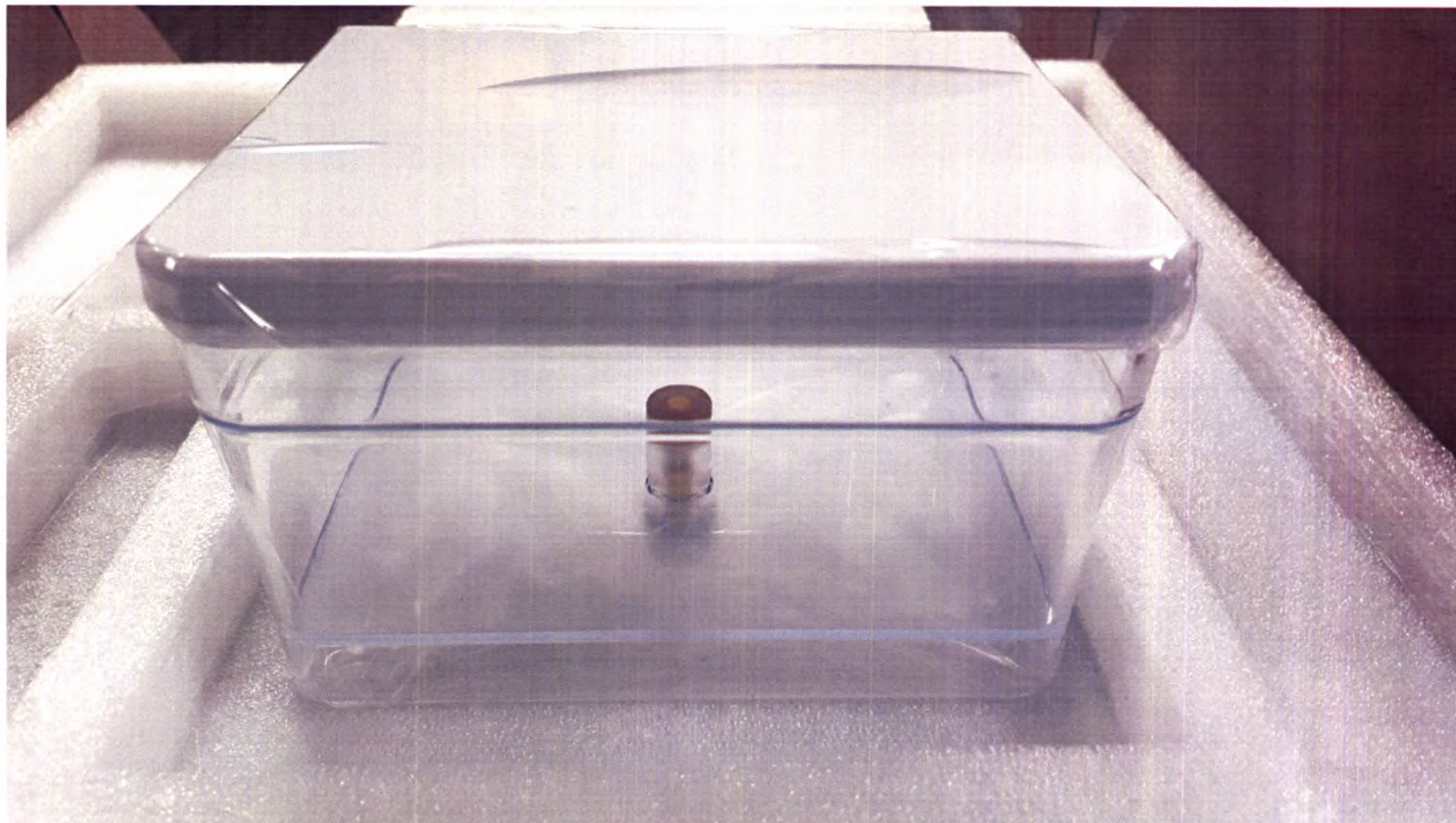
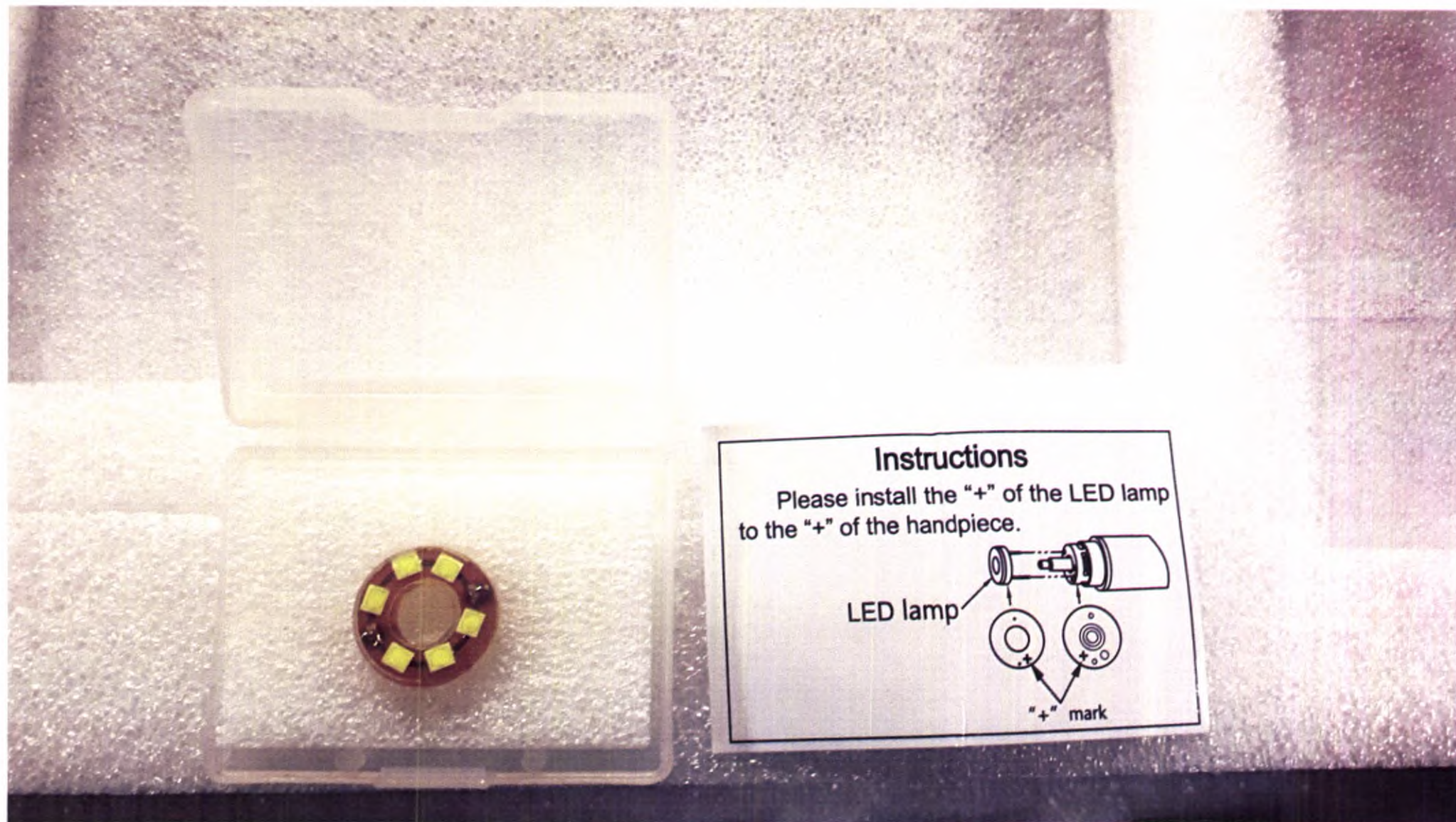
DTE

FDA CE 0197

Please read this manual before operating

D600 LED ULTRASONIC SCALER
INSTRUCTION MANUAL

www.glwoodpecker.com

GUILIN WOODPECKER MEDICAL INSTRUMENT CO., LTD.

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