Ullueh

## **PROTAPER NEXT**

Direction for use

***FOR DENTAL USE ONLY***

**PROTAPER NEXT Files for endodontic treatment.**

### **Manufactured by:**

**Maillefer Instruments Holding Sarl (Dentsply Maillefer), Chemin du Verger,  
3, CH-1338, Ballaigues, Switzerland. Tel.: +41(0)21 843-97 66, Fax: +41(0)  
21 843-9293**

1. Description
2. Photo
3. Indications
4. Contraindications
5. Warnings
6. Precautions
7. Adverse reactions
8. Step by step instruction
9. Storage conditions, shelf-life
10. Transportation conditions
11. Environment's protection
12. Utilization
13. Manufacturer's guarantee
14. Notified representative

  
Frédéric Mottier  
QA RC Director

Date

11.10 2022

  
Abdallah Rizki  
Director, Operations

MAILLEFER INSTRUMENTS HOLDING Sarl  
CH – 1338 BALLAIGUES

## 1.Description.


Protaper Next is a single use instrument (file) for endodontic treatment.

| Size/technical characteristics | X1            | X2            | X3            | X4            | X5            |
|--------------------------------|---------------|---------------|---------------|---------------|---------------|
| Length                         | 21, 25, 31 mm | 21, 25, 31 mm | 21, 25, 31 mm | 21, 25, 31 mm | 21, 25, 31 mm |
| Diameter                       | 017           | 025           | 030           | 040           | 050           |
| Taper                          | 04            | 06            | 07            | 06            | 06            |
| Cross-sectional area           | Rectangular   |               |               |               |               |
| Torque Resistance (g•sm)       | 44,8          | 88,2          | 156,5         | 230,8         | 335,3         |
| Bending Resistance (g•sm)      | 20,9          | 47,8          | 84,7          | 124,9         | 181,4         |
| Color                          | Yellow        | Red           | Blue          | Black         | Yellow        |

Instruments are delivered sterile, packed in blisters (3 instruments in a blister). Instruments are sterilized at manufacturing site by gamma-radiation.

Cutting part of instrument is made of a nickel-titanium alloy which was subjected to special thermal treatment, named M-wire

## 2. Photos

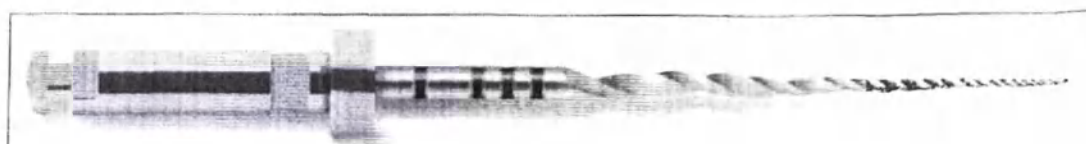
*Protaper Next X1*

*Protaper Next X2*

*Protaper Next X3*

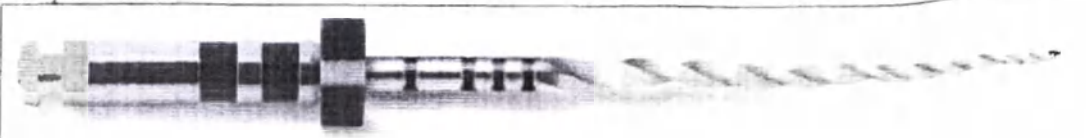*Protaper Next X4*

*Protaper Next X5*

### **3. Indications.**

The PROTAPER NEXT instruments are used in endodontic treatment for shaping & cleaning the root canal system.

These instruments are to be used only in a clinical environment by qualified users.

### **4. Contraindications.**

In common with all mechanically driven root canal instruments, PROTAPER NEXT instruments should not be used in cases of severe and sudden apical curvatures.

### **5. Warnings.**

- This product contains Nickel and should not be used for individuals with known allergic sensitivity to this metal.
- In order to prevent infectious agent transfer it is highly recommended to use a rubber dam system during the endodontic procedure.
- PROTAPER NEXT is provided sterile and reuse can increase the risk of cross contamination or breakage.

### **6. Precautions**

- PROTAPER NEXT files are single use devices. They can become less efficient after multiple uses causing undue stress to the file. This can lead to file separation.
- Exercise caution in the apical area and around significant curvatures.
- For your own safety, wear personal protective equipment (gloves, glasses, mask)
- Instruments should not be fully immersed in a sodium hypochlorite solution (NaOCl). Only the working part of the NiTi instrument, which is in contact with the patient should be immersed in a NaOCl solution concentrate at NOT more than 5%.
- Irrigate the root canal copiously and frequently during the shaping procedure.
- Lubricants such as NaOCl, EDTA, ProLube, Glyde™ are recommended.
- Establish a reproducible glide path using small-sized manual and/or dedicated mechanical glide path files.
- Use in a constant rotation at a speed of 300 rpm with light apical pressure.
- For optimal usage, torque control devices are recommended at 2Ncm (adjustable up to 5.2 Ncm according to practitioner experience)
- Clean flutes frequently and check for signs of distortion or wear.
- The PROTAPER NEXT instruments are recommended to be used mechanically (manually in very severe curvatures) in a clockwise continuous motion.
- The PROTAPER NEXT instruments are recommended to be used with brushing motion, away from external root concavities, to facilitate flute unloading and apical file progression.
- Use the PROTAPER NEXT files to passively follow the canal until the working length is achieved.

### **7. Adverse reactions.**

There are no known.

### **8. Step by step instructions for the PROTAPER NEXT files**

1. Prepare straightline access to canal orifice.
2. Explore the canal using small-sized hand files, determine working length, verify patency and confirm a smooth, reproducible glide path.
3. Always irrigate and if necessary, expand the glide path using small-sized hand files or

dedicated mechanical glide path files.

4. in the presence of NaOCl, brush and follow, along the glide path, with the PROTAPER NEXT X1 (017/04) file, in one or more passes, alternatively with small-sized hand files if necessary, until the working length is reached.

5. Use PROPTAPER NEXT X2 (025/06), exactly as described for PROTAPER NEXT X1 file, until the working length is passively reached.

6. Inspect the apical flutes of the PROTAPER NEXT X2 file; if they are loaded with dentin, then the shape is finished, the correspondingly sized gutta percha master cone or size verifier may be fitted, and the canal is ready for disinfection.

7. Alternatively, gauge the foramen, with a size 025 hand file and, if this file is snug at length, the canal is shaped and ready for disinfection.

8. If the size 025 hand file is loose at length, then continue shaping with the PROTAPER NEXT X3 (30/07) and, when necessary, the PROTAPER NEXT X4 (040/06) or PROTAPER NEXT X5 (050/06), gauging after each instrument with the 030, 040 or 050 hand files, respectively.

During protocol of use, irrigate, recapitulate with a small-sized hand file after each sequential PROTAPER NEXT instrument, then re-irrigate.

#### **9. Storage conditions, shelf-life.**

No specific storage conditions. Storing temperature: -50°C - + 50°C. Sterility is guarded during shelf-life period if packaging is not compromised.

Shelf life – 5 years. Expiration date is stated on a package; data format yyyy/mm

#### **10. Transportation conditions.**

Devices should be transported in covered vehicles in accordance with rules established for each type of transportation by local regulations.

#### **11. Environment's protection.**

Instrument doesn't have any impact on environment during its life cycle.

#### **12. Utilization**

The device should be utilized in accordance with local rules and protocols of a particular clinical establishment.

#### **13. Operation Conditions.**

Temperature: +10 - +35°C

The device is for single use and must be discarded if broken. There is no special technical service applicable.

#### **14. Manufacturer's guarantee.**

Device must be used in accordance with IFU's indications and protocol of use. All questions on device malfunctioning should be addressed to manufacturer's Notified Representative.

#### **15. Notified Representative.**

"Dentsply Sirona" LLC, Russia, 115432, Moscow, Andropov prospect, 18, building 6, office 1 39 40.

Frédéric Mottier  
QA RC Director

Abdallah Rizki  
Head Maillefer & Operations Director


**Authentication no. 14,594.-**

The undersigned, **Philippe TANNER**, a **notary public** for the canton of Vaud (Switzerland), having his chambers at **CH-1350 Orbe**, certifies the authenticity of the signatures placed on page 1 of the present document :

- Mr. Frédéric **MOTTIER**, at 1096 Cully (Municipality of Bourg-en-Lavaux), route de Grandvaux 7B, in his capacity as an executive officer of **MAILLEFER INSTRUMENTS HOLDING Sàrl**, a limited-liability company whose registered office is at 1338 Ballaigues (*IDE\UID CHE-100.026.046*), which company is validly bound by his joint signature with one other person (authentication by comparison of the signature placed on the Swiss identity card number E1846585, issued on May 5, 2014, valid until May 4, 2024);
- Mr. Abdallah **RIZKI**, at 39300 Champagnole (France), rue Pierre Langue 1, in his capacity as an executive officer of said company, which is validly bound by his joint signature with one other person (authentication by comparison of the signature placed on the French identity card number 130839200287, issued on August 9, 2013, valid until August 8, 2023).

The signatories proved their identity by producing the above-mentioned documents.

**CH-1350 Orbe**, the seventeenth of october, two thousand and twenty-two.

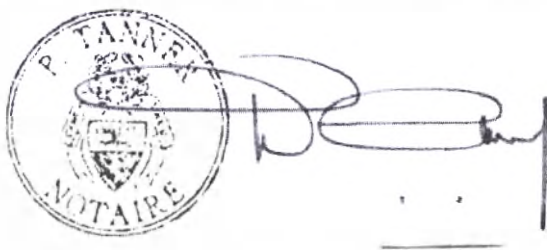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